
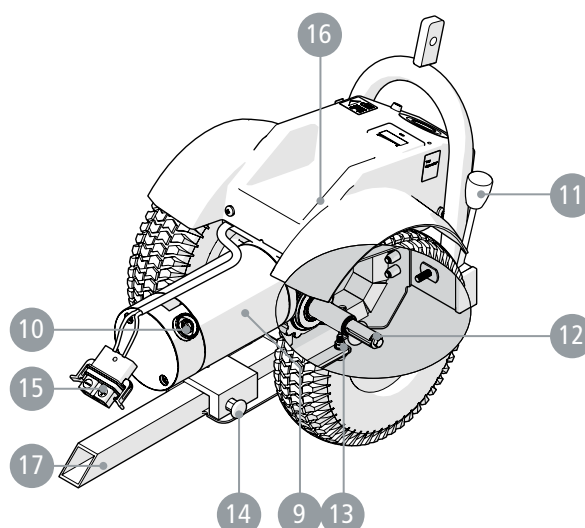






Lille Viking 2015 Low Bow

75253.6 2023-09-25

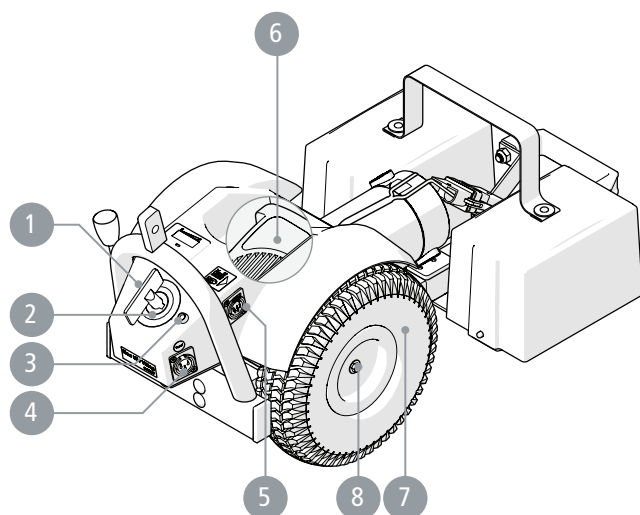

Pos	Benämning	Artikel nr.
1	Nyckel med kedja	UB2198
2	Huvudbrytare	UB2130
3	Lysdiod inkl fäste	82546
4	Kontakt laddare inkl skruv och bricka	82899
5	Kontakt Manöverdosa inkl skruv och bricka	82900
6	Styrenhet Lille Viking 2015	82915
7	Hjul Lille Viking punkteringsfritt kompl	UB6050
8	Hjulbult M6x20	UB9162
9	Motor Lille Viking 2015 inkl växel	82892
10	Kol + kolhållare Lille Viking 2015	82898
11	Utlösarhandtag inkl fästelement	UB200056A
12	Krysskil Lille Viking 2015	82897
13	Snäpplås 12x1 cm bak	UB9244E
14	Snäpplås 10x1 cm fram	UB9244
15	Kontakt Lille Viking 2015	83403
16	Motorkåpa Low LV2015 inkl kablage	83406
17	Chassi alu, LV low bow	82912

Pos	Description	Item no.
1	Main switch key incl chain	UB2198
2	Main switch	UB2130
3	Control lamp incl bracket	82546
4	Charger plug incl screw and washer	82899
5	Contact control unit incl screw and washer	82900
6	Control unit LV 2015	82915
7	Wheel LV incl rim and tyre compl	UB6050
8	Centre bolt M6x20	UB9162
9	Motor Lille Viking 2015 incl gear	82892
10	Brush incl brush holder LV2015 MTM	82898
11	Quick release arm assembly	UB200056A
12	Parallel key Lille Viking 2015	82897
13	Snap lock 12x1 back	UB9244E
14	Snap lock 10x1 front	UB9244
15	Contact LV 2015	83403
16	Motor cover Low LV 2015 incl cabling	83406
17	Chassi alu LV 2015 Low Bow	82912

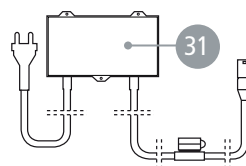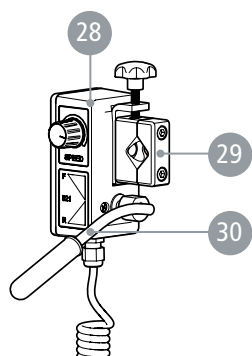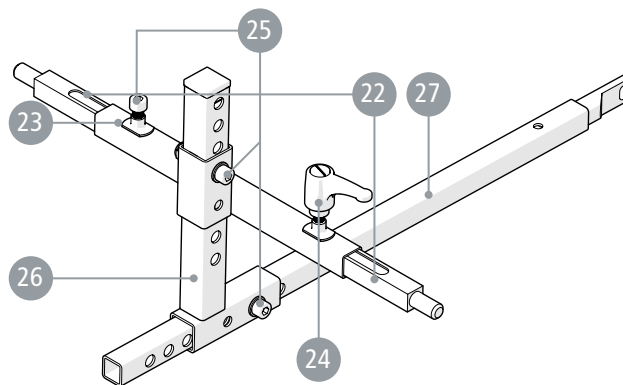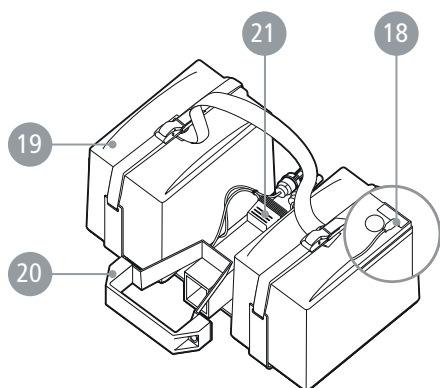

Pos	Benämning	Artikel nr.
19-21	Batteripaket Lille Viking 2015	83407
18	Batteri, Panasonic 12v/12A	83443
19	Batterikåpor inkl rem	82544
20	Handtag	UB9266A
21	Auto-bladsäkring 40A	UB1067A
22	Teleskopstång 200mm m rund tapp o spår	UB200075A
22	Teleskopstång 250mm m rund tapp o spår	UB200077B
23-25	Tvärstag inkl vred/skruv/bricka	83183
24	Vred M8	UB9111
25	Skruv M 8x 16 MC6S/A2	11362
25-26	Vinkelstycke inkl skruv	83184
27	Dragstång 550 mm	83182
28-31	Manöverdosa modell 521E	82641
29	Fäste manöverbox 520E/521E	UB9191B
30	Handtag manöverdosa, körhandtag	82905
	Handtag manöverdosa, körbygel	82906
31	Laddare Pb 3-stift 24V/1.5A	82987

Pos	Description	Item no.
19-21	Battery box Lille Viking 2015	83407
18	Battery 12V/12A	83443
19	Battery cover incl strap handle	82544
20	Bridge handle Lille Viking	UB9266A
21	Main fuse 40A	UB1067A
22	Telescopic bar 200 plain grooved	UB200075A
22	Telescopic bar 250 plain grooved	UB200077B
23-25	Cross bar incl handle/screw/washer	83183
24	Clamping handle M8	UB9111
25	Screw M 8x 16 MC6S/A2	11362
25-26	Pull bar bracket w knob/screw	83184
27	Tie rod 550 Lille Viking	83182
28-31	Control unit model 521E	82641
29	Attachment speed control 520/521	UB9191B
30	Handle speed control for push handle incl screw	82905
	Handle speed control for bow handle incl screw	82906
31	Battery charger Pb 3-pin 24V/1.5A	82987