

# Flamingo high-low

## Exploded views / Part lists

Rev. date (2024-06)

*On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.*



## Contents

|                    |    |
|--------------------|----|
| Seat.....          | 3  |
| Frame.....         | 5  |
| Accessories .....  | 8  |
| Armrest .....      | 11 |
| Footrest.....      | 13 |
| Side support ..... | 15 |




| Position                      | No.      | Description                    | Quantity | Unit of Measure | Ean code      |
|-------------------------------|----------|--------------------------------|----------|-----------------|---------------|
| 36+43<br>45+96<br>97+98<br>99 | 880153   | Angle adj. kit Flamingo back   | PCS      | 1               | 5707292429551 |
| 34                            | 71136    | Thread spindle, Flamingo       | PCS      | 2               | 5707292104410 |
| 35                            | 71133    | T-flange, Flamingo             | PCS      | 8               | 5707292104380 |
| 36                            | 71128    | Thread nut for angle adj.      | PCS      | 2               | 5707292104359 |
| 43                            | 71164    | Knob Ø23, 5 for handle         | PCS      | 2               | 5707292104540 |
| 45                            | 1811112  | Sheet metal screw 4,8*19 phx   | PCS      | 2               | 5707292452184 |
| 91*                           | 71302-1  | Flamingo Seat shell, size 1    | PCS      | 1               | 5707292629401 |
|                               | 71303-1  | Flamingo Seat shell, size 2    | PCS      | 1               | 5707292629418 |
|                               | 71304-1  | Flamingo Seat shell, size 3    | PCS      | 1               | 5707292629425 |
|                               | 71305-1  | Flamingo Seat shell, size 4    | PCS      | 1               | 5707292629432 |
| 92*                           | 71312    | Flamingo back shell, size 1    | PCS      | 1               | 5707292105028 |
|                               | 71313    | Flamingo back shell, size 2    | PCS      | 1               | 5707292105035 |
|                               | 71314    | FLAMINGO BACK SHELL, SIZE 3    | PCS      | 1               | 5707292105042 |
|                               | 71315    | Flamingo back shell, size 4    | PCS      | 1               | 5707292105059 |
| 95                            | 102429   | Seatlock for Flamingo          | PCS      | 2               | 5707292562036 |
| 97                            | 71144    | Hand wheel, Flamingo           | PCS      | 2               | 5707292104427 |
| 99                            | 1940038  | ECO-SYN UH Ø4*14               | PCS      | 8               | 5707292465597 |
| 100                           | 0659200  | Button head screw 6*12 stainl. | PCS      | 2               | 5707292139436 |
| 101                           | 1310536  | Covering button,white 13*5,0mm | PCS      | 16              | 5707292397911 |
| 102                           | 81138-90 | Mounting for neck support      | PCS      | 1               | 5707292227157 |
| 103                           | 0609727  | Stainless insex 6x12 uh        | PCS      | 4               | 5707292138484 |
| 104                           | 112708   | Knob for Seat m6x23            | PCS      | 2               | 5707292604286 |
| 105                           | 3501050  | Star grip Ø40 M8*15mm white    | PCS      | 1               | 5707292146717 |
| 106                           | 0631953  | Button head M5*6, stainless    | PCS      | 4               | 5707292599988 |
| 107                           | 110202   | Snap Lock R82-25 With ø6       | PCS      | 4               | 5707292575913 |



| Position                | No.       | Description                                             | Quantity | Unit of Measure | Ean code      |
|-------------------------|-----------|---------------------------------------------------------|----------|-----------------|---------------|
| 1*                      | 135994-10 | Frame Bottom, size 3+4                                  | PCS      | 1               | 5707292877178 |
|                         | 135990-10 | Frame Bottom, size 1+2                                  | PCS      | 1               | 5707292877161 |
| 2*                      | 138539-10 | Frame top, size 3+4                                     | PCS      | 1               | 5707292893826 |
|                         | 138414-10 | Frame top, size 1+2                                     | PCS      | 1               | 5707292893802 |
| 3*                      | 3006955   | 100MM DOUBLE WHEEL, WHITE                               | PCS      | 4               | 5707292601872 |
|                         | 3006830   | 75MM WHITE STEINCO WHEEL                                | PCS      | 4               | 5707292368560 |
|                         | 3006955   | 100mm double wheel, white                               | PCS      | 4               | 5707292601872 |
| 4                       | 2507006   | Ferrule for pipe, grey Ø32                              | PCS      | 6               | 5707292580917 |
| 5                       | 111869-1  | Release pedal complete f. tilt                          | PCS      | 1               | 5707292839596 |
| 6                       | 114288-1  | Release pedal komplet for lift                          | PCS      | 1               | 5707292839602 |
| 7                       | 100236    | Bush ø12 x M8 x 22,5                                    | PCS      | 2               | 5707292559746 |
| 8*                      | 112413-10 | Seatframe complete size 4                               | PCS      | 1               | 5707292602817 |
|                         | 112412-10 | Seatframe complete size 3                               | PCS      | 1               | 5707292602800 |
|                         | 112411-10 | Seatframe complete size 2                               | PCS      | 1               | 5707292602794 |
|                         | 112410-10 | Seatframe complete size 1                               | PCS      | 1               | 5710000000000 |
| 9                       | 100234    | Bearing ø25x18x28                                       | PCS      | 2               | 5707292560001 |
| 10                      | 100233    | Disc ø27x18,2x2                                         | PCS      | 2               | 5707292560025 |
| 11                      | 82268     | Wire mounting by pedal                                  | PCS      | 2               | 5707292123039 |
| 11.1                    | 0310100   | Pressure spring 0,9*8*23*10,5v                          | PCS      | 1               | 5707292135513 |
| 12*                     | 2206510   | Lockable Gas traction spring                            | PCS      | 1               | 5707292603838 |
|                         | 2206505   | Lockable Gas Traction spring                            | PCS      | 1               | 5707292630087 |
| 13*                     | 2206160-2 | Bloc-o-lift, 174268, 800N                               | PCS      | 1               | 5707292895851 |
|                         | 2206150-2 | Bloc-o-lift, 174029, 600N                               | PCS      | 1               | 5707292895844 |
| 14                      | 0631952   | Button head M5*6, rf tufflok                            | PCS      | 4               | 5707292503725 |
| 15                      | 0661153   | Button head screw 8*10 stainl.                          | PCS      | 4               | 5707292616654 |
| 16                      | 1105080   | Stainless washer 8mm                                    | PCS      | 2               | 5707292140364 |
| 17                      | 1631080   | Stainless lock nut 8mm                                  | PCS      | 2               | 5707292141453 |
| 18                      | 1631120   | Stainless lock nut 12mm                                 | PCS      | 2               | 5707292502049 |
| 19                      | 0609835   | Stainless insex screw 8*35                              | PCS      | 1               | 5707292138606 |
| 20                      | 0609832   | Stainless insex screw 8*30                              | PCS      | 2               | 5707292606037 |
| 22                      | 138241    | House for gasspring                                     | PCS      | 2               | 5707292839923 |
| 23                      | 1761005   | Counter nut 10*0.1                                      | PCS      | 2               | 5707292509925 |
| 24                      | 0380001   | Adj screw U-BA-BZ-E-SM5-VA                              | PCS      | 2               | 5707292829771 |
| 24.1                    | 1310630   | Distance piece, Ø10<br>(size 1 '1 PCS'; Size 2 '2 PCS') | PCS      | 2               | 5707292140753 |
| 25                      | 2200050   | Small release arm                                       | PCS      | 2               | 5707292143075 |
| 26                      | 2501600   | Ferrule d15*1,0 black                                   | PCS      | 1               | 5707292143969 |
| 27                      | 0607999   | Stainless Insex screw 12*50                             | PCS      | 2               | 5707292482013 |
| 28                      | 1310645   | Washer, nature Ø16*8*0,5mm                              | PCS      | 1               | 5707292446015 |
| 29                      | 1810050   | Plate screw 7981c st 2,9*13                             | PCS      | 2               | 5707292404671 |
| 30                      | 113085    | Cover, outside                                          | PCS      | 1               | 5707292604279 |
| 31                      | 113086    | Cover, inside                                           | PCS      | 1               | 5707292604262 |
| 32                      | 113099    | Cover, right                                            | PCS      | 1               | 5707292604255 |
| 33                      | 113098    | Cover, left                                             | PCS      | 1               | 5707292604248 |
| 35                      | 112354    | Bellow for frame/adjust. arm                            | PCS      | 2               | 5707292602039 |
| 36*                     | 113783    | Outher casing for lift, 3-4                             | PCS      | 1               | 5707292604347 |
|                         | 140634    | outher casing f. lift size 1-2                          | PCS      | 1               | 5707290000000 |
| *Choose size/color/type |           |                                                         |          |                 |               |

|                         |           |                                |     |   |               |
|-------------------------|-----------|--------------------------------|-----|---|---------------|
| 37*                     | 109043    | Cable seat tilt size 3-4       | PCS | 1 | 5707292575593 |
|                         | 140635    | Outer casing f.tilt size 1-2   | PCS | 1 | 5707290000000 |
| 38                      | 0371854-1 | Soft inner cable for Stingray  | PCS | 2 | 5707292536945 |
| 39*                     | 116969-4  | Flamingo footrest kompl.size 4 | PCS | 1 | 5707292609588 |
|                         | 116969-3  | Flamingo footsup,compl. s.3    | PCS | 1 | 5707292609571 |
|                         | 116969-2  | Flamingo footsup,compl.s 2     | PCS | 1 | 5707290000000 |
|                         | 116969-1  | Flamingo footsup,complete.s 1  | PCS | 1 | 5710000000000 |
| 40                      | 100229    | Bush                           | PCS | 2 | 5707292560018 |
| 42*                     | 71004     | Back&seat, Flamingo size 4     | PCS | 1 | 5707292103956 |
|                         | 71003     | Back&seat, Flamingo size 3     | PCS | 1 | 5707292103949 |
|                         | 71002     | Back&seat, Flamingo size 2     | PCS | 1 | 5707290000000 |
|                         | 71001     | Flamingo size&back sz 1, compl | PCS | 1 | 5710000000000 |
| 43*                     | 880192    | Splashguard, high, size 2-4    | PCS | 1 | 5707292134547 |
|                         | 880191    | Splashguard, high, size 1      | PCS | 1 | 5710000000000 |
| 44*                     | 880164    | Upholstery for Seat size 4     | PCS | 1 | 5707292134448 |
|                         | 880163    | Upholstery Flamingo Seat sz.3  | PCS | 1 | 5707292134431 |
|                         | 880162    | Upholstery Flamingo Seat sz.2  | PCS | 1 | 5707290000000 |
|                         | 880161    | Upholstery Flamingo Seat sz.1  | PCS | 1 | 5710000000000 |
| 45*                     | 880174    | Upholstery sz.4,back,flamingo  | PCS | 1 | 5707292134486 |
|                         | 880173    | Upholstery Flamingo back sz.3  | PCS | 1 | 5707292134479 |
|                         | 880172    | Upholstery Flamingo back sz.2  | PCS | 1 | 5707290000000 |
|                         | 880171    | Upholstery Flamingo back sz.1  | PCS | 1 | 5710000000000 |
| 46                      | 0659555   | Button head M6*20, A2 w/ ES-Lo | PCS | 2 | 5707292612120 |
| 47                      | 1781080   | Stainless top nut 8mm          | PCS | 1 | 5707292141576 |
| 48                      | 1100500   | Spring washer 8mm, stainless   | PCS | 1 | 5707292140142 |
| 49                      | 0388006   | Screwnipple 5*7,5mm            | PCS | 2 | 5707292136114 |
| 50                      | 0387000   | End nipple for outer casing    | PCS | 1 | 5707292136091 |
| 51                      | 140227    | Lock for springstopper         | SÆT | 1 | 5707292881328 |
| 52                      | 0603533   | Insex M8x20, lav hoved RF      | PCS | 1 | 5707292575296 |
| 53                      | 0506509   | Rivet 4*10, stainless          | PCS | 1 | 5707292472441 |
| 54                      | 1310645   | Washer, nature Ø16*8*0,5 mm    | PCS | 2 | 5707292446015 |
| *Choose size/color/type |           |                                |     |   |               |

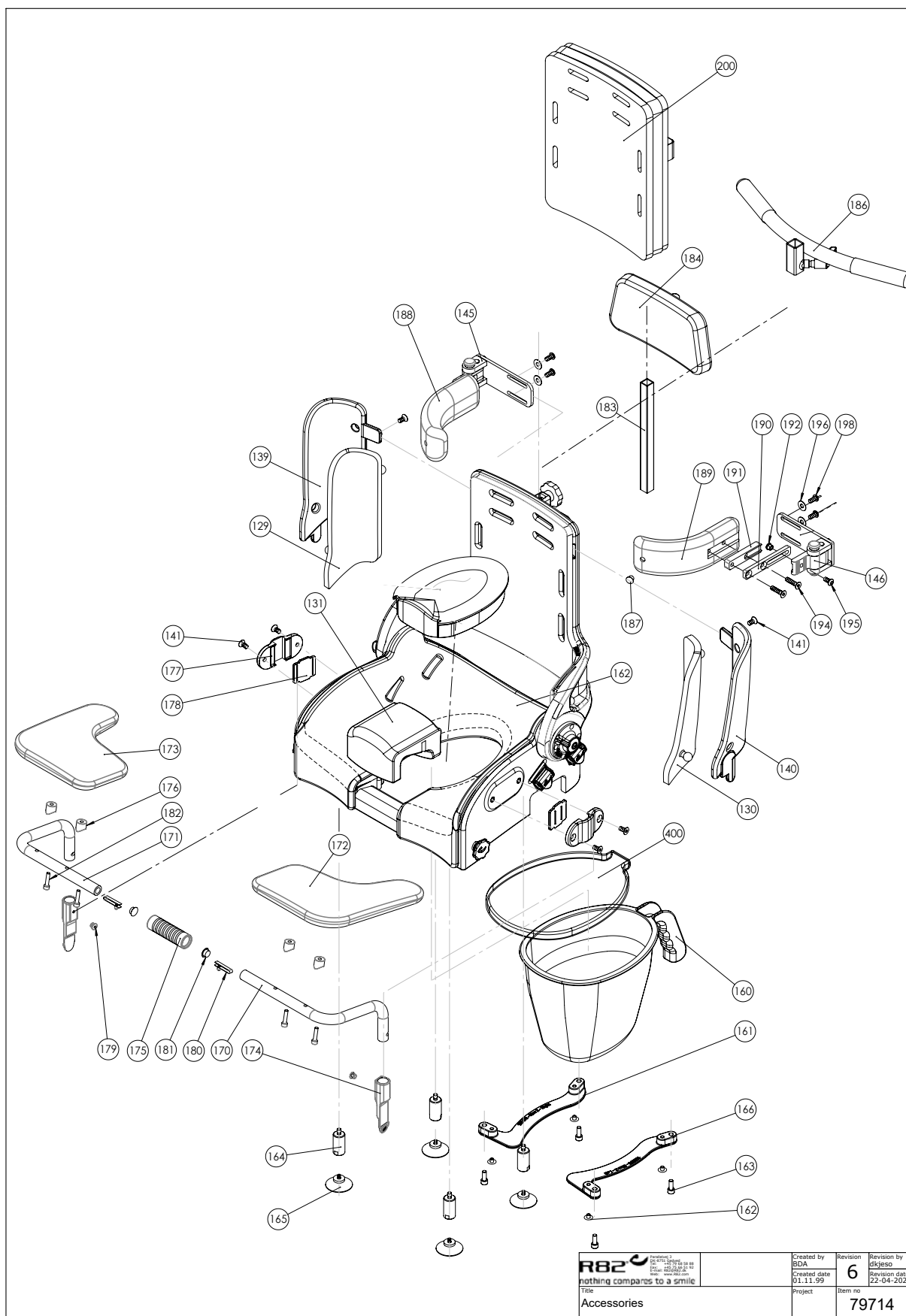




| Position            | No.        | Description                    | Quantity | Unit of Measure | Ean code      |
|---------------------|------------|--------------------------------|----------|-----------------|---------------|
| 129*                | 880181-H   | Upholst. F/right side,flam 1   | 1        | PCS             | 5707292254467 |
|                     | 880181-61H | Side upholstery, sz.1, right   | 1        | PCS             | 5707292615893 |
|                     | 880182-H   | Upholst. F/right side, flam. 2 | 1        | PCS             | 5707292254481 |
|                     | 880182-61H | Side upholstery, sz.2, right   | 1        | PCS             | 5707292615916 |
|                     | 880183-H   | Upholst. F/right side,flam. 3  | 1        | PCS             | 5707292254504 |
|                     | 880183-61H | Side upholstery, sz.3, right   | 1        | PCS             | 5707292615930 |
|                     | 880184-H   | Upholst. F/right side,flam. 4  | 1        | PCS             | 5707292254528 |
|                     | 880184-61H | Side upholstery, sz.4, right   | 1        | PCS             | 5707292615954 |
| 130*                | 880181-V   | Upholst.f/left side flam. Sz.1 | 1        | PCS             | 5707292254474 |
|                     | 880181-61V | Side upholstery, sz.1, left    | 1        | PCS             | 5707292615909 |
|                     | 880182-V   | Upholst. F/left side, flam. 2  | 1        | PCS             | 5707292254498 |
|                     | 880182-61V | Side upholstery, sz.2, left    | 1        | PCS             | 5707292615923 |
|                     | 880183-V   | Upholst. F/left side, flam. 3  | 1        | PCS             | 5707292254511 |
|                     | 880183-61V | Side upholstery, sz.3, left    | 1        | PCS             | 5707292615947 |
|                     | 880184-V   | Upholst f/left side,flam. 4    | 1        | PCS             | 5707292254535 |
|                     | 880184-61V | Side upholstery, sz.4, left    | 1        | PCS             | 5707292615961 |
| 139*                | 71322      | Backside size 1,right-Flamingo | 1        | PCS             | 5707292105066 |
|                     | 71323      | Backside size 2,right-Flamingo | 1        | PCS             | 5707292105073 |
|                     | 71324      | Backside size 3,right-Flamingo | 1        | PCS             | 5707292105080 |
|                     | 71325      | Backside size 4,right-Flamingo | 1        | PCS             | 5707292105097 |
| 140*                | 71332      | Backside size 1, left-Flamingo | 1        | PCS             | 5707292105103 |
|                     | 71333      | Backside size 2, left-Flamingo | 1        | PCS             | 5707292105110 |
|                     | 71334      | Backside size 3, left,flamingo | 1        | PCS             | 5707292105127 |
|                     | 71335      | Backside size 4, left-Flamingo | 1        | PCS             | 5707292105134 |
| 141                 | 0609727    | Stainless insex 6x12 uh        | 2        | PCS             | 5707292138484 |
| 145*                | 5982560-R  | Sidesupport 118*42,5mm, right  | 1        | PCS             | 5707292396082 |
|                     | 5982580-R  | Svingbar sidestøtte højre      | 1        | PCS             | 5707292396105 |
| 146*                | 5982560-L  | Sidesupport 118*42,5mm, left   | 1        | PCS             | 5707292396020 |
|                     | 5982580-L  | Svingbar sidestøtte venstre    | 1        | PCS             | 5707292396044 |
| 160*                | 880128     | Comode pan, Flamingo size 1+2  | 1        | PCS             | 5707292134301 |
|                     | 880129     | Comode pan, Flamingo size 3+4  | 1        | PCS             | 5707292134318 |
| 161*                | 135470     | Pottebøjle str.2, højre        | 1        | PCS             | 5707292893680 |
|                     | 136218     | Pottebøjle str.1, højre        | 1        | PCS             | 5707292893659 |
|                     | 135544     | Pottebøjle str.3, højre        | 1        | PCS             | 5707292893697 |
| 162                 | 1310536    | Covering button,white 13*5,0mm | 4        | PCS             | 5707292397911 |
| 163                 | 0602206    | Insex M6*16 rustfri m/ nylon   | 4        | PCS             | 5707292896629 |
| 164                 | 71134      | Distancepart for suction cup   | 4        | PCS             | 5707292104397 |
| 165                 | 2522015    | Suction cup Ø80mm M6*12mm      | 4        | PCS             | 5707292144461 |
| 166*                | 138575     | Flaminco str. 2 - ABS          | 1        | PCS             | 5707292893673 |
|                     | 136237     | Pottebøjle str.1, venstre      | 1        | PCS             | 5707292893666 |
|                     | 135538     | Pottebøjle str.3, venstre      | 1        | PCS             | 5707292873040 |
| 170*                | 71371      | Supporting tube 1 left         | 1        | PCS             | 5707292105394 |
|                     | 71373      | Support rod for size 2         | 1        | PCS             | 5707292105417 |
|                     | 71367      | Supporting tube 3 left         | 1        | PCS             | 5707292436771 |
|                     | 71369      | Support rod size 4, left       | 1        | PCS             | 5707292436757 |
| 171*                | 71370      | Supporting tube 1 right        | 1        | PCS             | 5707292105387 |
|                     | 71372      | Supporting tube 2 right        | 1        | PCS             | 5707292105400 |
|                     | 71366      | Supporting tube 3 right        | 1        | PCS             | 5707292436764 |
|                     | 71368      | Support rod size 4, right      | 1        | PCS             | 5707292436740 |
| * choose size/color |            |                                |          |                 |               |

| Position            | No.      | Description                             | Quantity | Unit of Measure | Ean code      |
|---------------------|----------|-----------------------------------------|----------|-----------------|---------------|
| 172*                | 6175361  | Armsupport tray small, left             | 1        | PCS             | 5707292189707 |
|                     | 6075361  | Arm rest table small, left PUR          | 1        | PCS             | 5707292613103 |
|                     | 6175361  | Armsupport tray small, left             | 1        | PCS             | 5707292189707 |
|                     | 6175365  | Armsupport tray large, left             | 1        | PCS             | 5707292189721 |
|                     | 6075365  | Arm rest left, big - Hell Blau          | 1        | PCS             | 5707292613066 |
| 173*                | 6175359  | Armsupport tray small,right             | 1        | PCS             | 5707292189691 |
|                     | 6075359  | Arm rest table small,right PUR          | 1        | PCS             | 5707292613110 |
|                     | 6175363  | Armsupport tray large, right            | 1        | PCS             | 5707292189714 |
|                     | 6075363  | Arm rest right, big - Hellblau          | 1        | PCS             | 5707292613073 |
| 174                 | 71130-90 | Bushing, white                          | 2        | PCS             | 5707292223593 |
| 175                 | 71132    | Locking socket,white                    | 1        | PCS             | 5707292104373 |
| 176                 | 104377   | Saddle disc Ø15,5Ø5,3, h10,0            | 4        | PCS             | 5707292568878 |
| 177                 | 72134-90 | Mounting for armrest, white             | 2        | PCS             | 5707292223760 |
| 178                 | 82408-90 | Underlying plate armrest flam           | 2        | PCS             | 5707292231482 |
| 179                 | 0506519  | Stainless blind rivet 4*12              | 2        | PCS             | 5707292137388 |
| 180                 | 6153700  | Knob spring f.activity centre           | 2        | PCS             | 5707292188601 |
| 181                 | 1310552  | Hedge plug k 5 natural                  | 2        | PCS             | 5707292581747 |
| 182                 | 0603522  | Insex ch M5*22 stainless                | 4        | PCS             | 5707292490988 |
| 183                 | 108407   | Tube for back extension                 | 1        | PCS             | 5707292573452 |
| 184*                | 880118   | Back extension, grey PU, sz. 1+2        | N/A      | N/A             | 5707292572936 |
|                     | 880119   | Back extension, grey PU, sz. 3+4        | N/A      | N/A             | 5707292572943 |
| 186                 | 880110   | Push brace for flamingo/swan            | 1        | PCS             | 5707292134219 |
| 187                 | 2502519  | Tube leg, grey, Ø16mm                   | 2        | PCS             | 5707292394415 |
| 188+189*            | 6075407  | Side cushion, large, curved, lightblue  | 2        | PCS             | 5707292613332 |
|                     | 6175407  | Side cushion, large, curved             | 2        | PCS             | 5707292389237 |
|                     | 6175409  | Side cushion, small, curved             | 2        | PCS             | 5707292389251 |
|                     | 6075408  | Side cushion, medium, curved, lightblue | 2        | PCS             | 5707292613349 |
|                     | 6175408  | Side cushion, medium, curved, grey      | 2        | PCS             | 5707292389244 |
|                     | 6075409  | Side cushion, small, curved, light blue | 2        | PCS             | 5707292613356 |
|                     | 6175403  | Side cushion moon shaped                | 2        | PCS             | 5707292381132 |
|                     | 6175402  | Side cushion, square                    | 2        | PCS             | 5707292381125 |
|                     | 6175401  | Side cushion, long, straight            | 2        | PCS             | 5707292381118 |
| 190*                | 82569    | Bar,straight,stainless, 9,5cm           | 2        | PCS             | 5707292124869 |
|                     | 82648    | Bar, straight,stainless,16cm            | 2        | PCS             | 5707292389275 |
|                     | 81968    | Bar, curved, 25 mm, l:12 cm             | 2        | PCS             | 5707292120366 |
| 191                 | 82570    | Plastic spacer for Flamingo             | 2        | PCS             | 5707292124876 |
| 192                 | 82558    | Nut 6x16                                | 2        | PCS             | 5707292124753 |
| 194*                | 0609730  | Stainless insex 6*20 uh.                | 4        | PCS             | 5707292138507 |
|                     | 0609727  | Stainless insex 6x12 uh                 | 4        | PCS             | 5707292138484 |
| 195                 | 0659200  | Buttonhead screw M6*12 stainl.          | 2        | PCS             | 5707292139436 |
| 196+198             | 5001009  | Screws etc. For side supports           | 1        | PCS             | 5707292154330 |
| 200                 | 80172-44 | Seat depth reduction kit, s. 2          | N/A      | N/A             | 5707292992895 |
|                     | 80173-44 | Seat depth reduction kit, s. 3          | N/A      | N/A             | 5707292992901 |
|                     | 80174-44 | Seat depth reduction kit, s. 4          | N/A      | N/A             | 5707292992918 |
| 400*                | 137341   | Lay for pot 1                           | 1        | PCS             | 5707292837240 |
|                     | 137345   | Lay for pot 2                           | 1        | PCS             | 5707292837226 |
| * choose size/color |          |                                         |          |                 |               |

|                                                                   |                                                              |                                                          |                                                               |
|-------------------------------------------------------------------|--------------------------------------------------------------|----------------------------------------------------------|---------------------------------------------------------------|
| <b>R82</b><br><small>nothing compares to a smile</small><br>Title | Checked by<br>Admin<br>Created date<br>31-03-2011<br>Project | Revision<br>C<br>Revision date<br>28-01-2016<br>Item no. | Revision by<br>PER<br>Revision date<br>28-01-2016<br>Item no. |
|                                                                   |                                                              |                                                          |                                                               |

Armrest for Flamingo

C:\PROMOTION\spareparts\flamingo\104369

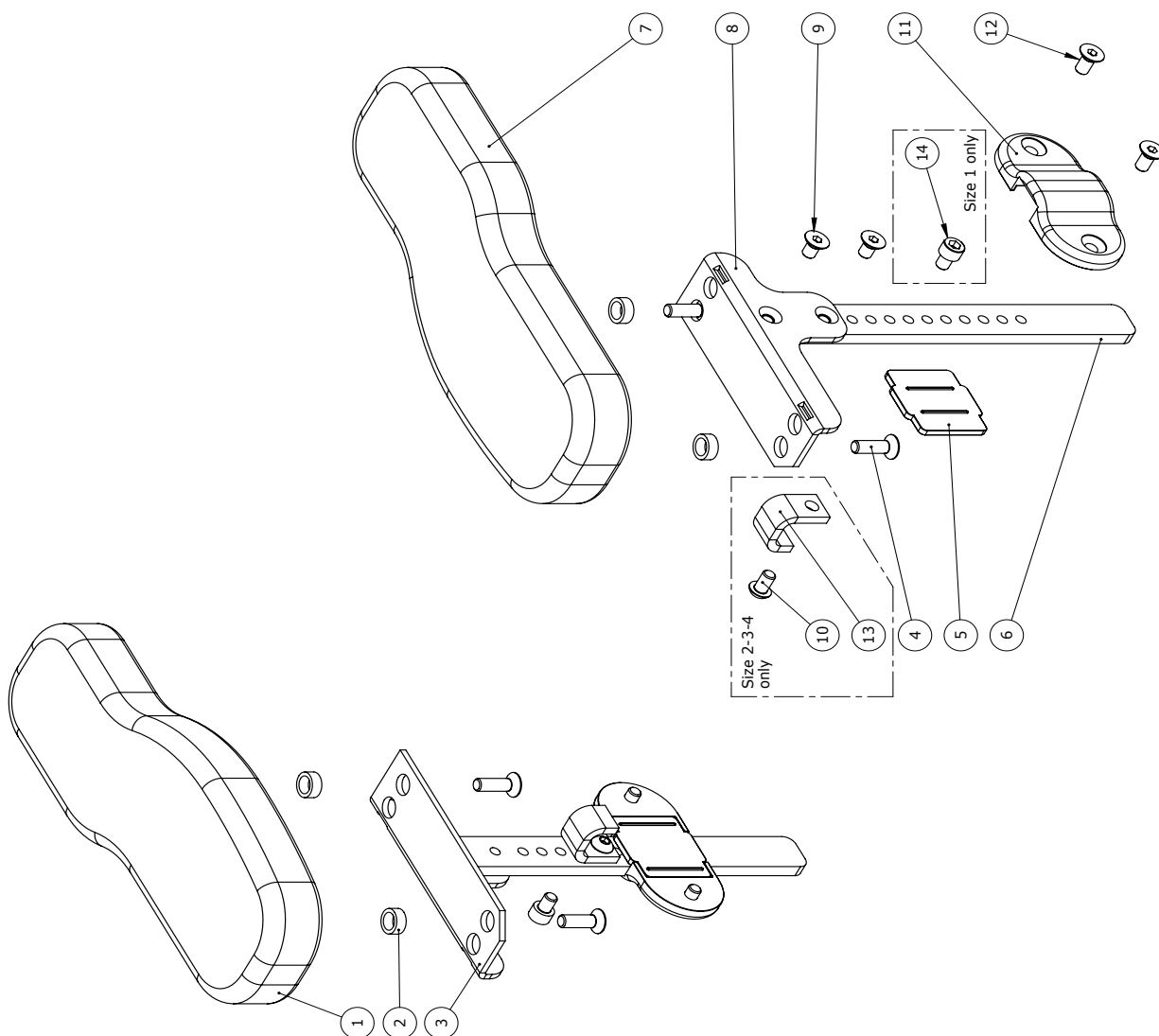


| Position | Art. No. | Description                     | Unit of Measure | Quantity | Ean code      |
|----------|----------|---------------------------------|-----------------|----------|---------------|
| 1        | 6175150  | Armrest cushion, size 1, right  | PCS             | 1        | 5707292189226 |
| 2        | 1310539  | Distance piece 12/8,4*5mm       | PCS             | 4        | 5707292423474 |
| 3        | 104360   | Armrest bracket, right          | PCS             | 1        | 5707292570703 |
| 4        | 0609730  | Stainless insex 6*20 uh.        | PCS             | 4        | 5707292138507 |
| 6        | 117054   | Amrest bar                      | PCS             | 2        | 5707292611383 |
| 7        | 6175152  | Armrest cushion, size 1, left   | PCS             | 1        | 5707292189233 |
| 8        | 104359   | Armrest bracket, left           | PCS             | 1        | 5707292570697 |
| 9        | 0609760  | Rustfri insex 6*10 uh. tuflok   | PCS             | 4        | 5707292570796 |
| 10       | 0659005  | Insex button head M6*10, stainl | PCS             | 2        | 5707292423740 |
| 13       | 117055   | Brace                           | PCS             | 2        | 5707292611390 |
| 14       | 0602479  | Insex M6*8, stainless           | PCS             | 2        | 5707292465412 |
| 5+11+12  | 5001013  | Plastic bag enclosing small     | SET             | 1        | 5707292154378 |



| Position                | No.     | Description                     | Quantity | Unit of Measure | Ean code      |
|-------------------------|---------|---------------------------------|----------|-----------------|---------------|
| 30                      | 1310717 | NYLON WASHER, NAT O17-8,4*2MM   | 2        | PCS             | 5707292140821 |
| 40                      | 3502008 | KNOB HANDLE Ø20 M6              | 2        | PCS             | 5707292146830 |
| 60                      | 0609732 | INSEX 6*22, STAINLESS           | 2        | PCS             | 5707292409669 |
| 62*                     | 71207   | FOOTPLATE LEFT, FLAM. SZ 1-2    | 1        | PCS             | 5707292104625 |
|                         | 71209   | FOOTPLATE LEFT, FLAM. SZ 3-4    | 1        | PCS             | 5707292104649 |
| 63*                     | 71208   | FOOTPLATE RIGHT, FLAM. SZ 1-2   | 1        | PCS             | 5707292104632 |
|                         | 71210   | FOOTPLATE RIGHT, FLAM. SZ 3-4   | 1        | PCS             | 5707292104656 |
| 64*                     | 71282   | FOOTPLATE INLAY FLAM. SIZE 1    | 1        | PCS             | 5707292104816 |
|                         | 71283   | FOOTPLATE INLAY FLAM. SIZE 2    | 1        | PCS             | 5707292104823 |
|                         | 71284   | FOOTPLATE INLAY FLAM. SIZE 3    | 1        | PCS             | 5707292104830 |
|                         | 71285   | FOOTPLATE INLAY FLAM. SIZE 4    | 1        | PCS             | 5707292104847 |
| 65                      | 1631060 | STAINLESS LOCK NUT 6MM          | 2        | PCS             | 5707292141446 |
| 66                      | 1109000 | STARLOCK SKIVE M12, U/KAP, RF.  | 2        | PCS             | 5707292140432 |
| 67                      | 71119   | FICTIONBUSH FOR FLAMINGO        | 2        | PCS             | 5707292104267 |
| 69                      | 71123   | LOCK RATCHET                    | 2        | PCS             | 5707292104304 |
| 70                      | 0310147 | STAINLESS SPRING 1*9,3*20MM*7,5 | 2        | PCS             | 5707292135636 |
| 73                      | 0806030 | STAINLESS STOP SCREW 6*30MM     | 2        | PCS             | 5707292140029 |
| 74*                     | 71242   | TELESCOPIC PIPE SZ. 1           | 2        | PCS             | 5707292104687 |
|                         | 71243   | TELESCOPIC PIPE SZ. 2-4         | 2        | PCS             | 5707292104694 |
| 75                      | 3503210 | FINGER SCREW, WHITE M6X6        | 2        | PCS             | 5707292147004 |
| 76                      | 71118   | BUSHING Ø22-18 FLAM.            | 2        | PCS             | 5707292104250 |
| 77*                     | 71272   | TELESCOPE EXTENSION LEFT, SZ 1  | 1        | PCS             | 5707292104755 |
|                         | 71273   | TELESCOPE ARM LEFT 2            | 1        | PCS             | 5707292104762 |
|                         | 71274   | TELESCOPE EXT. LEFT, SZ 3+4     | 1        | PCS             | 5707292104779 |
| 78                      | 2502005 | FERRULE D18*1,0-2,0 INT. WHITE  | 2        | PCS             | 5707292144010 |
| 79                      | 1750600 | LOCK NUT, STAINLESS M6          | 2        | PCS             | 5707292141477 |
| 80*                     | 71277   | TELESCOPE EXTENSION RIGHT       | 1        | PCS             | 5707292104786 |
|                         | 71278   | TELESCOPE EXTENSION RIGHT, SZ 2 | 1        | PCS             | 5707292104793 |
|                         | 71279   | TELESCOPE EXTENSION RIGHT, SZ 3 | 1        | PCS             | 5707292104809 |
| *Choose size/color/type |         |                                 |          |                 |               |

| Pos no. | Qty. | Item no.  | Description.                   |
|---------|------|-----------|--------------------------------|
| 145     | 1    | 5982560-R | Svingbar sidestøtte højre      |
| 146     | 1    | 5982560-L | Svingbar sidestøtte venstre    |
| 187     | 2    | 2502519   | DÆKHÆTTE Spec                  |
| 189     | 2    | 6175408   | Sidepelot mellem buet grå      |
| 190     | 2    | 155641    | Bar, curved, 25 mm, l:10,5 cm  |
| 191     | 2    | 82570     | Mellemlæg svb.sidest.flam.hvid |
| 192     | 2    | 82558     | M6 møtrik                      |
| 194     | 4    | 0609730   | INSEX UNDERS. M6x20            |
| 195     | 2    | 0659200   | BUTTON HEAD M6x12              |
| 196     | 4    | 1105060   | Rustfri skive 6,4*12*1,6 (a4)  |
| 198     | 4    | 0659200   | Button head M6*12, rustfri     |

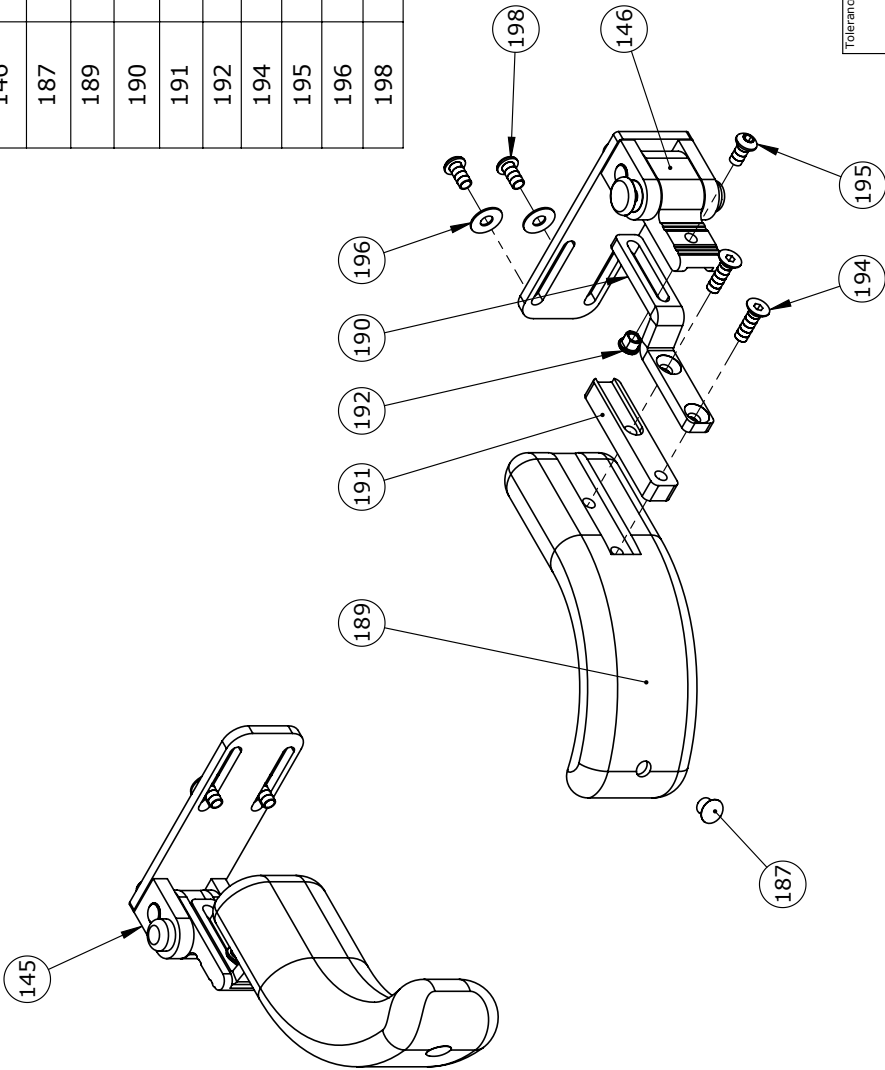

|           |                                  |                                                                                                                                                                                                                                                                                                                                                                        |                    |                            |                    |                             |
|-----------|----------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------|----------------------------|--------------------|-----------------------------|
| Tolerance | Project                          | <div><b>R82</b><br/>nothing compares to a smile</div> <div><small>Attention: 3 Dept.<br/>R&amp;D<br/>Tel: +352 46 46 87 80<br/>Fax: +352 46 46 87 82<br/>E-mail: <a href="mailto:info@r82.com">info@r82.com</a><br/>Web: <a href="http://www.r82.com">www.r82.com</a></small></div> |                    |                            |                    |                             |
| Surface   | Tool                             | Created by<br>PER                                                                                                                                                                                                                                                                                                                                                      | Revision<br>4      | Created date<br>11-02-2021 | Revision by<br>PER | Revision date<br>08-04-2021 |
| Material  |                                  | Scale<br>1:3                                                                                                                                                                                                                                                                                                                                                           | Weight<br>425.299g |                            |                    |                             |
| Title     | Side Support Swing away Flamingo |                                                                                                                                                                                                                                                                                                                                                                        | Format<br>A4       | Item no<br>155650          |                    |                             |

C:\EPDM\TEGN\Explode\Flamingo\155650