



Coaster Replacement Parts

Convaid
by Etac

RP2601-ENG-REV.00 2026-08-01



.....

Table of Contents

Coaster Rear Wheels
997504 (Coaster 12, 14, 16, 18) 1

Coaster Front Wheels
997505 (Coaster 12, 14, 16, 18) 2

Coaster Seat Back Assembly
997506 Coaster Seat Back Assembly (Coaster 12)..... 3
997507 Coaster Seat Back Assembly (Coaster 14)4
997508 Coaster Seat Back Assembly (Coaster 16)5
997509 Coaster Seat Back Assembly (Coaster 18)6

Upholstery Assembly
997510 Coaster Upholstery Assembly (Coaster 12)7
997511 Coaster Upholstery Assembly (Coaster 14)8
997512 Coaster Upholstery Assembly (Coaster 16)9
997513 Coaster Upholstery Assembly (Coaster 18)10

Wheel Lock
997514 Coaster Wheel Lock (Coaster 12) 11
997515 Coaster Wheel Lock (Coaster 14) 12
997516 Coaster Wheel Lock (Coaster 16) 13
997517 Coaster Wheel Lock (Coaster 18) 14

Quick Release Axle
997518 (Coaster 12, 14,16, 18) 15



Coaster Front Wheels
997519 Coaster Front X Brace (Coaster 12) 16
997520 Coaster Front X Brace (Coaster 14) 17
997521 Coaster Front X Brace (Coaster 16) 18
997522 Coaster Front X Brace (Coaster 18) 19

Coaster Push Handle
997523 Coaster Push Handle (Coaster 12) 20
997524 Coaster Push Handle (Coaster 14) 21
997525 Coaster Push Handle (Coaster 16) 22
997526 Coaster Push Handle (Coaster 18) 23

Coaster Rear Wheels

Coaster 12,14,16,18
997504
Kit Includes

Item	Parts#	Description	Qty
1	510913	Rear Wheel 290MM X 60MM - Black	1

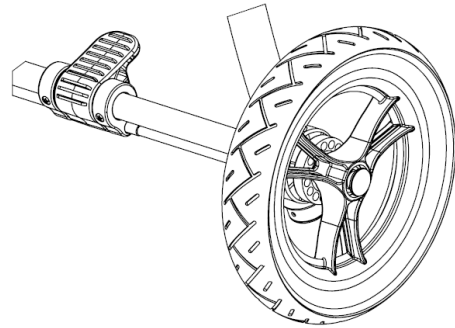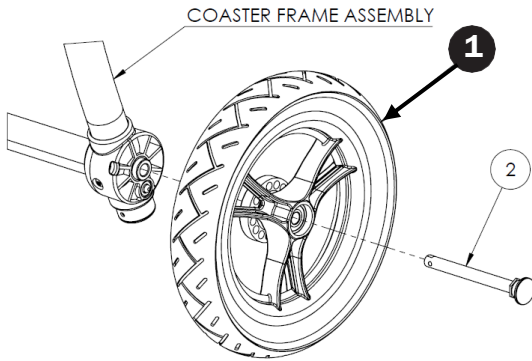

INSTRUCTIONS

1. Remove the existing wheel **item 1** by pushing inward the quick release axle **item 2** then pull out the wheel **item 1**.
2. After cleaning the existing quick release and its receiver in the frame assembly, install the quick release onto the new wheel then in the frame assembly by pushing the quick release and the wheel all the way. Check that the assembly is complete by pulling on the wheel.



Coaster Front Wheels

Coaster 12,14,16,18


997505

Kit Includes

Item	Parts#	Description	Qty
1	101087	Front Wheel W/3-Spokes 182.5MM X 51MM	1
2	100765	Bolt M8-1.25 MM Socket Cap Screw	1
3	100070	Washer 5/16" X 9/16"	6
4	100766	Locknut M8-1.25 MM Nylon Insert	1

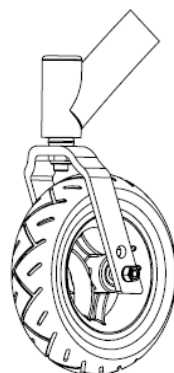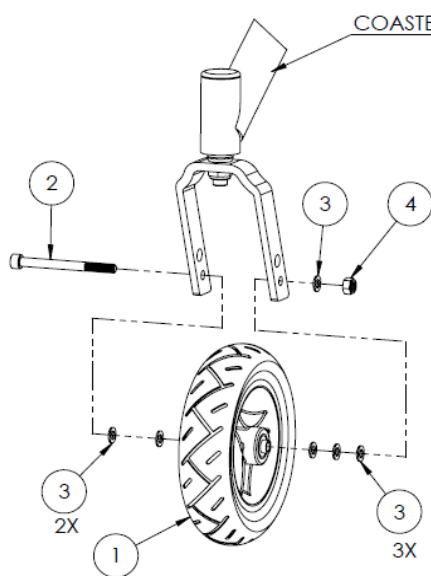

FRONT WHEEL ASSEMBLED
RIGHT HAND

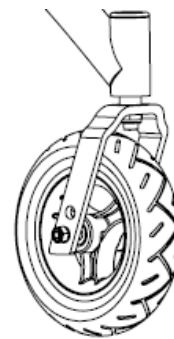

FRONT WHEEL ASSEMBLED
LEFT HAND

INSTRUCTIONS

1. Remove the existing wheel **item 1** by unscrewing bolt **item 2** & nut **item 4**, remove all six washers **item 3**


2. Install the new wheel **item 1** using the new hardware as shown.

NOTE: The exploded view shown is for the right-hand front wheel, for the left-hand front wheel assembly, the hardware should be assembled in the opposite direction.

Coaster Seatback Assembly

Coaster 12
997506
Kit Includes

Item	Parts#	Description	Qty
1	510892	Seatback Assembly 12	1
2	101081	Bolt 1/4"-20 X 3-1/2" Socket Button Head	2
3	100025	Nylon Washer 1/4" X 1/2"	2
4	100089	Washer 1/4" AN-960-416	2
5	100065	Coved Washer 3/4"	2
6	100478	Nylon Washer 1/4" X 3/4" X 1/8"	2
7	100435	Spacer 1/4" X 1-1/16"	2
8	100263	Spacer 1/4" X 1/2" Heavy	2
9	101095	Binding Barrel 1/4"-20 X .49"	2
10	100063	Sheet Metal Screw #10 X 1/2"	3
11	100652	Sheet Metal Screw #10 X 3/4"	1
12	507252	CB Closure Strap	1

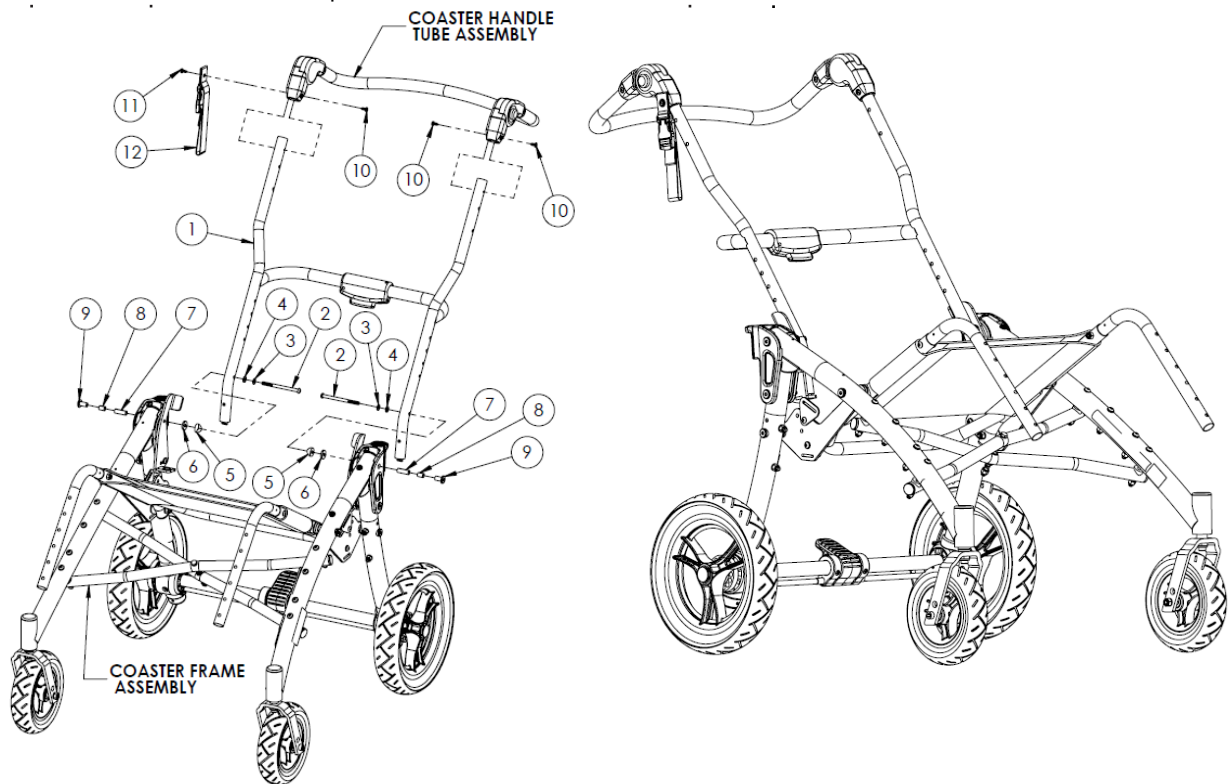



INSTRUCTIONS

1. Disassemble the handle tube and closure strap **item 12** by removing the 4 screws **items 10 & 11**.
2. Disassemble the backrest **item 1** by removing hardware **items 2, 3, 4, 5, 6, 7, 8, 9**
3. Use the new hardware to assemble the new backrest **item 1** as shown.
4. Re-assemble the existing handle tube and the new closure strap using the new hardware (4 screws) as shown.

Coaster Seatback Assembly

Coaster 14
997507
Kit Includes

Item	Parts#	Description	Qty
1	510891	Seatback Assembly 14	1
2	101081	Bolt 1/4"-20 X 3-1/2" Socket Button Head	2
3	100025	Nylon Washer 1/4" X 1/2"	2
4	100089	Washer 1/4" AN-960-416	2
5	100065	Coved Washer 3/4"	2
6	100478	Nylon Washer 1/4" X 3/4" X 1/8"	2
7	100435	Spacer 1/4" X 1-1/16"	2
8	100263	Spacer 1/4" X 1/2" Heavy	2
9	101095	Binding Barrel 1/4"-20 X .49"	2
10	100063	Sheet Metal Screw #10 X 1/2"	3
11	100652	Sheet Metal Screw #10 X 3/4"	1
12	507252	CB Closure Strap	1

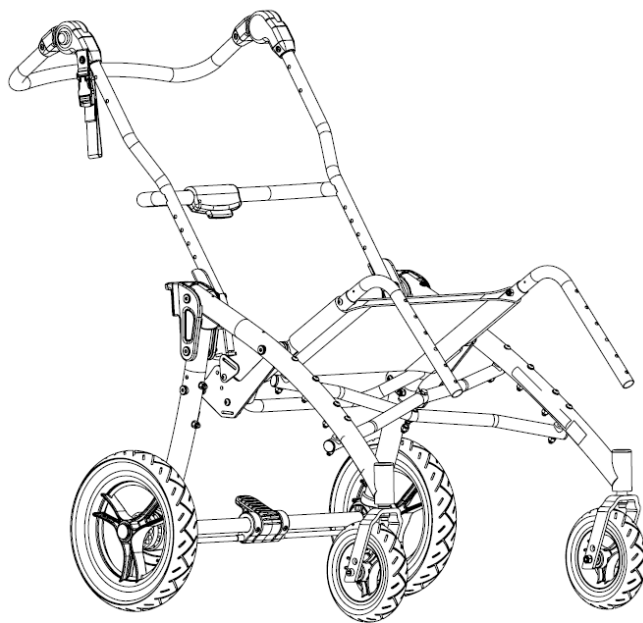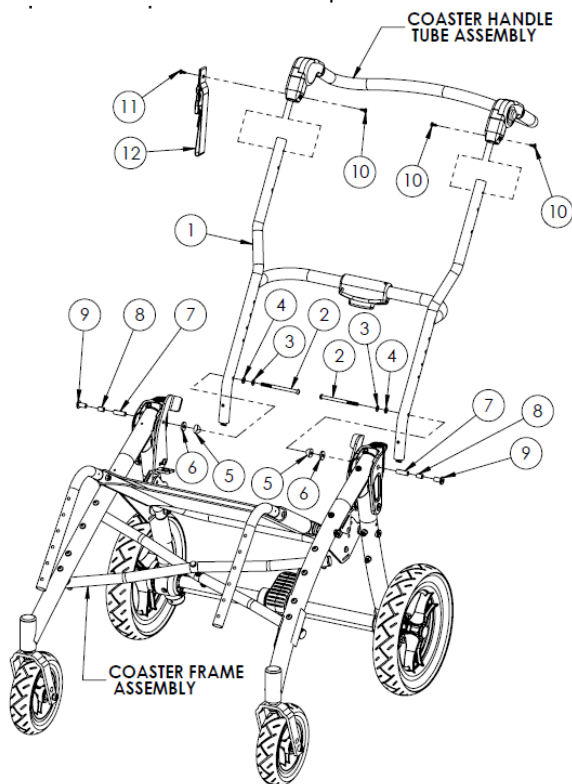



INSTRUCTIONS

1. Disassemble the handle tube and closure strap **item 12** by removing the 4 screws **items 10 & 11**.
2. Disassemble the backrest **item 1** by removing hardware **items 2, 3, 4, 5, 6, 7, 8, 9**.
3. Use the new hardware to assemble the new backrest **item 1** as shown.
4. Re-assemble the existing handle tube and the new closure strap using the new hardware (4 screws) as shown.

Coaster Seatback Assembly

Coaster 16
997508
Kit Includes

Item	Parts#	Description	Qty
1	510890	Seatback Assembly 16	1
2	101081	Bolt 1/4"-20 X 3-1/2" Socket Button Head	2
3	100025	Nylon Washer 1/4" X 1/2"	2
4	100089	Washer 1/4" AN-960-416	2
5	100065	Coved Washer 3/4"	2
6	100478	Nylon Washer 1/4" X 3/4" X 1/8"	2
7	100435	Spacer 1/4" X 1-1/16"	2
8	100263	Spacer 1/4" X 1/2" Heavy	2
9	101095	Binding Barrel 1/4"-20 X .49"	2
10	100063	Sheet Metal Screw #10 X 1/2"	3
11	100652	Sheet Metal Screw #10 X 3/4"	1
12	507252	CB Closure Strap	1

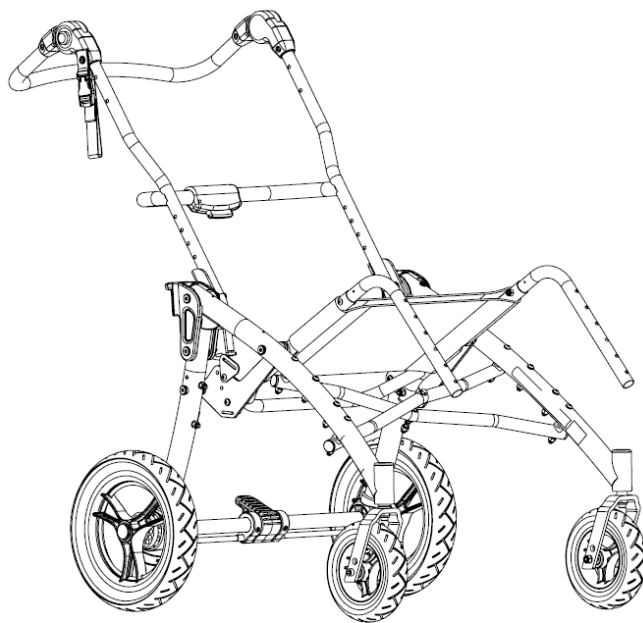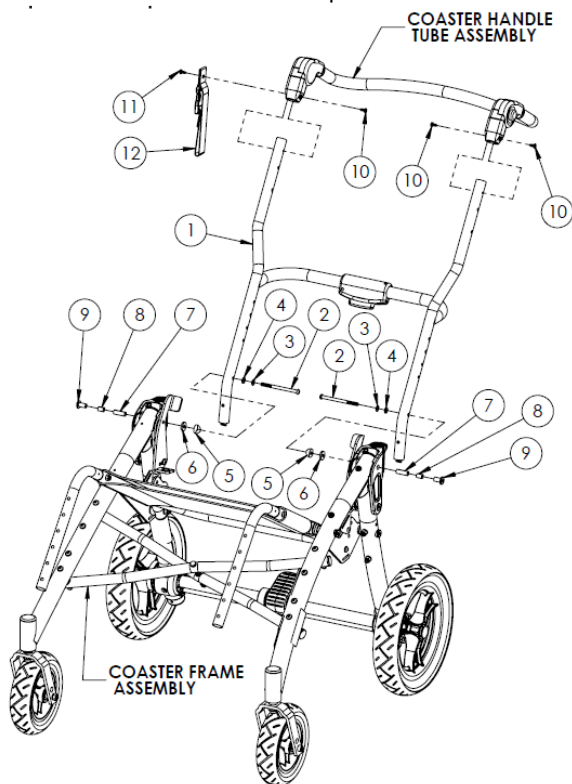


INSTRUCTIONS

1. Disassemble the handle tube and closure strap **item 12** by removing the 4 screws **items 10 & 11**.
2. Disassemble the backrest **item 1** by removing hardware **items 2, 3, 4, 5, 6, 7, 8, 9**
3. Use the new hardware to assemble the new backrest **item 1** as shown.
4. Re-assemble the existing handle tube and the new closure strap using the new hardware (4 screws) as shown.

Coaster Seatback Assembly

Coaster 18
997509
Kit Includes

Item	Parts#	Description	Qty
1	510889	Seatback Assembly 18	1
2	101081	Bolt 1/4"-20 X 3-1/2" Socket Button Head	2
3	100025	Nylon Washer 1/4" X 1/2"	2
4	100089	Washer 1/4" AN-960-416	2
5	100065	Coved Washer 3/4"	2
6	100478	Nylon Washer 1/4" X 3/4" X 1/8"	2
7	100435	Spacer 1/4" X 1-1/16"	2
8	100263	Spacer 1/4" X 1/2" Heavy	2
9	101095	Binding Barrel 1/4"-20 X .49"	2
10	100063	Sheet Metal Screw #10 X 1/2"	3
11	100652	Sheet Metal Screw #10 X 3/4"	1
12	507252	CB Closure Strap	1

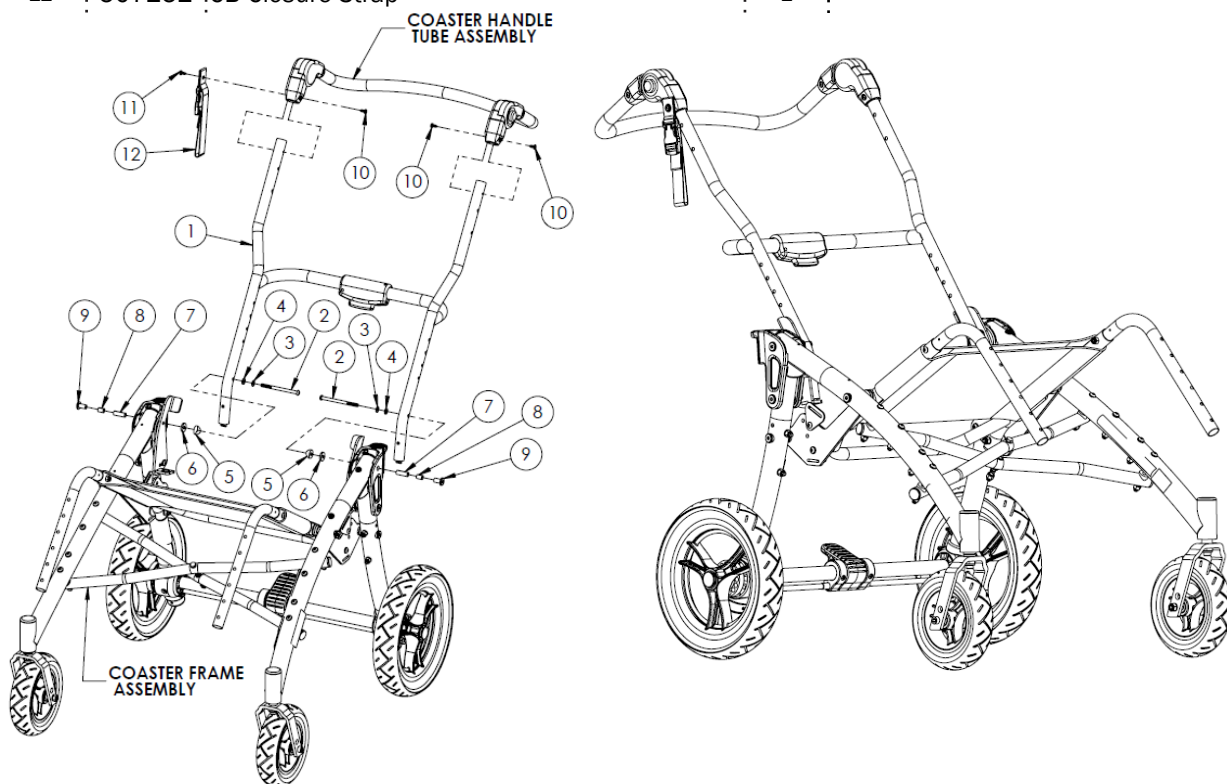


INSTRUCTIONS

1. Disassemble the handle tube and closure strap **item 12** by removing the 4 screws **items 10 & 11**.
2. Disassemble the backrest **item 1** by removing hardware **items 2, 3, 4, 5, 6, 7, 8, 9**.
3. Use the new hardware to assemble the new backrest **item 1** as shown.
4. Re-assemble the existing handle tube and the new closure strap using the new hardware (4 screws) as shown.

Coaster Upholstery Assembly

Coaster 12
997510
Kit Includes

Item	Parts#	Description	Qty
1	511243	CTR-12 Standard Seat Assembly +3"	1
2	100063	Sheet Metal Screw # 10 X 1/2"	2

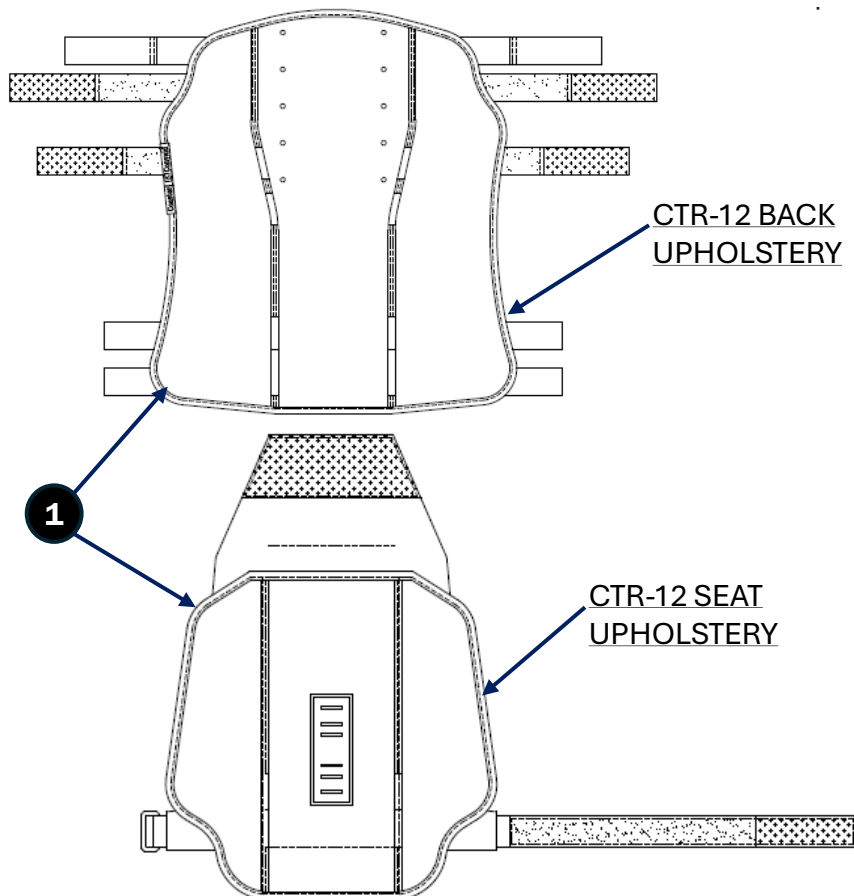


INSTRUCTIONS

1. Remove the existing Back Upholstery of **item 1**
2. Remove the two screws **item 2** then remove the existing Seat Upholstery of **item 1**
3. Assemble the new Seat Upholstery of **item 1** using the two new screws **item 2**
4. Assemble the new Back Upholstery of **item 1** the same way as the old configuration as shown.

Coaster Upholstery Assembly

Coaster 14
997511
Kit Includes

Item	Parts#	Description	Qty
1	511244	CTR-14 Standard Seat Assembly +3"	1
2	100063	Sheet Metal Screw # 10 X 1/2"	2

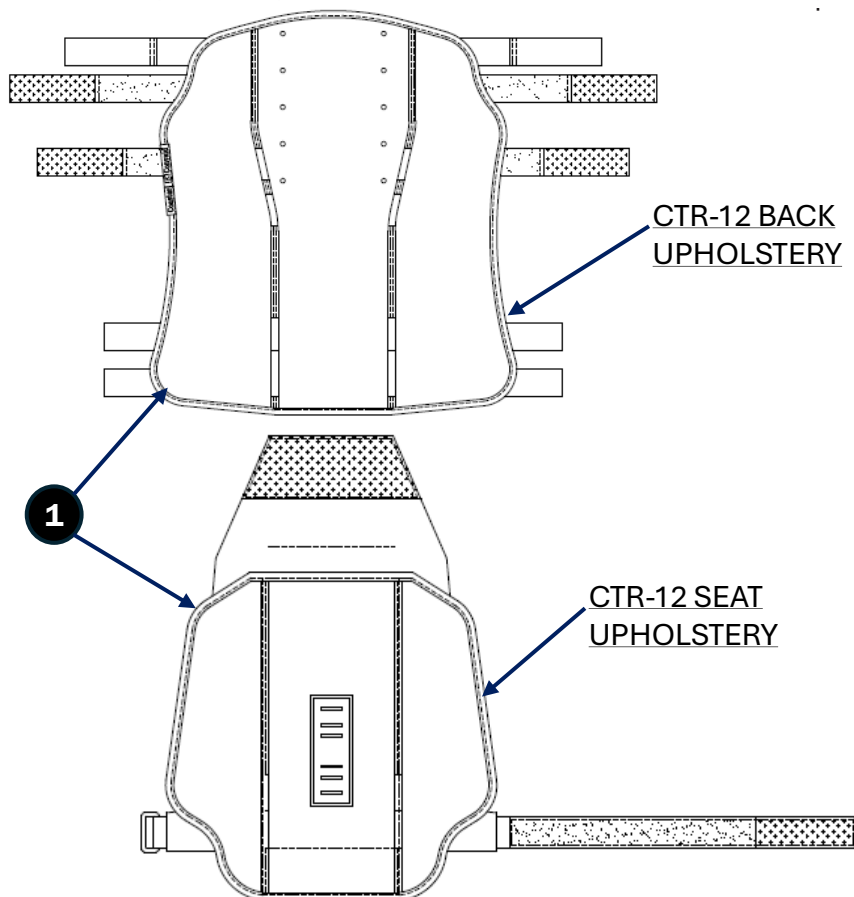


INSTRUCTIONS

1. Remove the existing Back Upholstery of **item 1**
2. Remove the two screws **item 2** then remove the existing Seat Upholstery of **item 1**
3. Assemble the new Seat Upholstery of **item 1** using the two new screws **item 2**
4. Assemble the new Back Upholstery of **item 1** the same way as the old configuration as shown.

Coaster Upholstery Assembly

Coaster 16
997512
Kit Includes

Item	Parts#	Description	Qty
1	511245	CTR-16 Standard Seat Assembly +3"	1
2	100063	Sheet Metal Screw # 10 X 1/2"	2

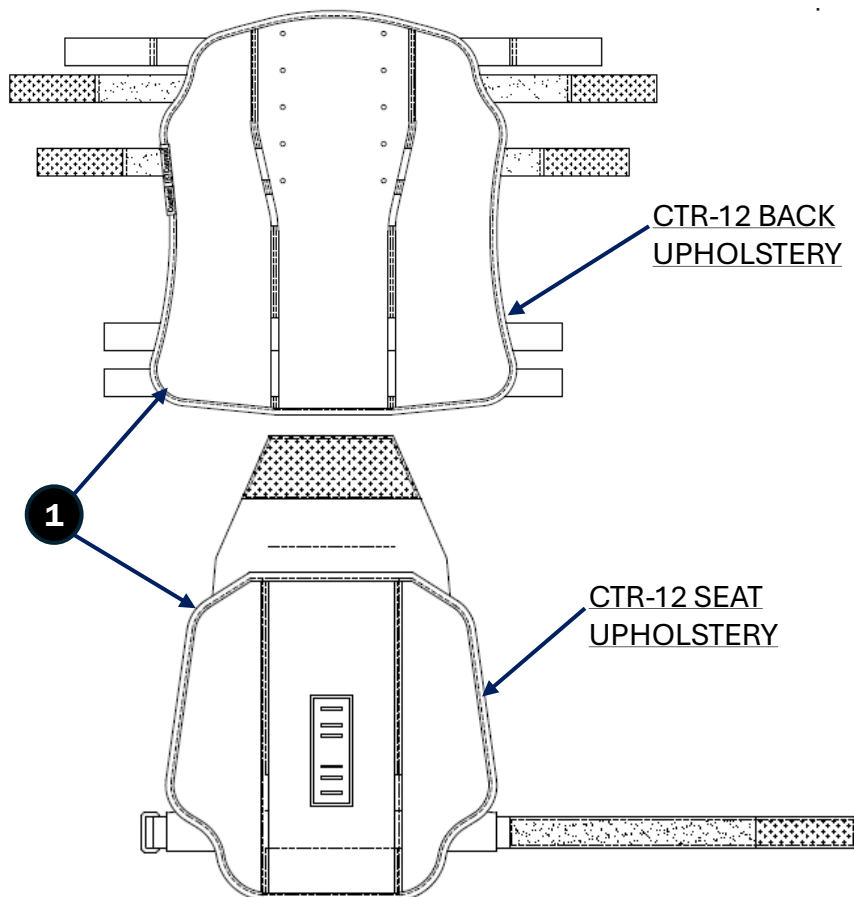


INSTRUCTIONS

1. Remove the existing Back Upholstery of **item 1**
2. Remove the two screws **item 2** then remove the existing Seat Upholstery of **item 1**
3. Assemble the new Seat Upholstery of **item 1** using the two new screws **item 2**
4. Assemble the new Back Upholstery of **item 1** the same way as the old configuration as shown.

Coaster Upholstery Assembly

Coaster 18
997513
Kit Includes

Item	Parts#	Description	Qty
1	511246	CTR-18 Standard Seat Assembly +3"	1
2	100063	Sheet Metal Screw # 10 X 1/2"	2

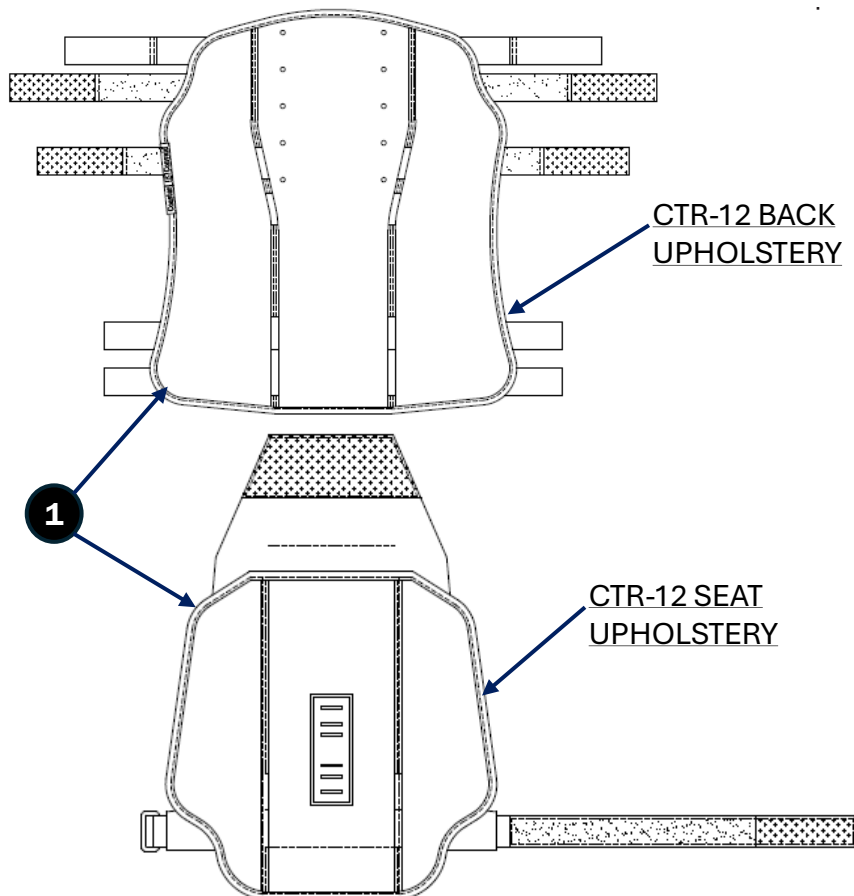

INSTRUCTIONS

- 1 . Remove the existing Back Upholstery of **item 1**
- 2 . Remove the two screws **item 2** then remove the existing Seat Upholstery of **item 1**
- 3 . Assemble the new Seat Upholstery of **item 1** using the two new screws **item 2**
- 4 . Assemble the new Back Upholstery of **item 1** the same way as the old configuration as shown.


Coaster Wheel Lock

Coaster 12

997514

Kit Includes

Item	Parts#	Description	Qty
1	510622	Break Pin Sleeve	2
2	100846	Comp. Spring 0.344 ID X 0.420 OD X 1.50 LG	2
3	510655	Brake Pin Outer	2
4	510754	Brake Tube 12	2
5	510653	Middle Brake Pin	2
6	100844	Comp. Spring 0.118 ID X 0.188 OD X 0.750 LG	2
7	508415	Trekker Brake Pin Inner	2

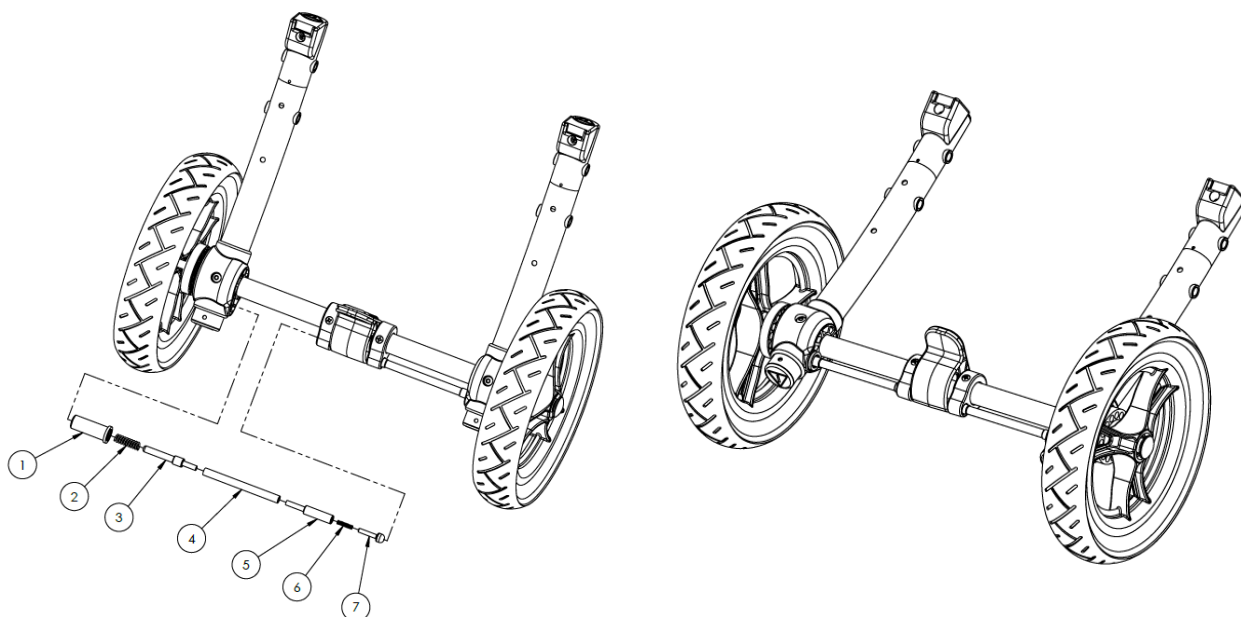

INSTRUCTIONS

1. Remove the existing wheel lock parts **items 1, 2, 3, 4, 5, 6, 7** from one side, by pushing **item 3** to the left and **item 4** to the right.
2. Pull out **items 2, 4** then other items. Do the same for the other side.
3. After cleaning the wheel lock mechanism, install the new wheel lock parts in the opposite sequence of disassembly.
4. Check that the wheel lock is working properly.


Coaster Wheel Lock

Coaster 14

997515

Kit Includes

Item	Parts#	Description	Qty
1	510622	Break Pin Sleeve	2
2	100846	Comp. Spring 0.344 ID X 0.420 OD X 1.50 LG	2
3	510655	Brake Pin Outer	2
4	510768	Brake Tube 14	2
5	510653	Middle Brake Pin	2
6	100844	Comp. Spring 0.118 ID X 0.188 OD X 0.750 LG	2
7	508415	Trekker Brake Pin Inner	2

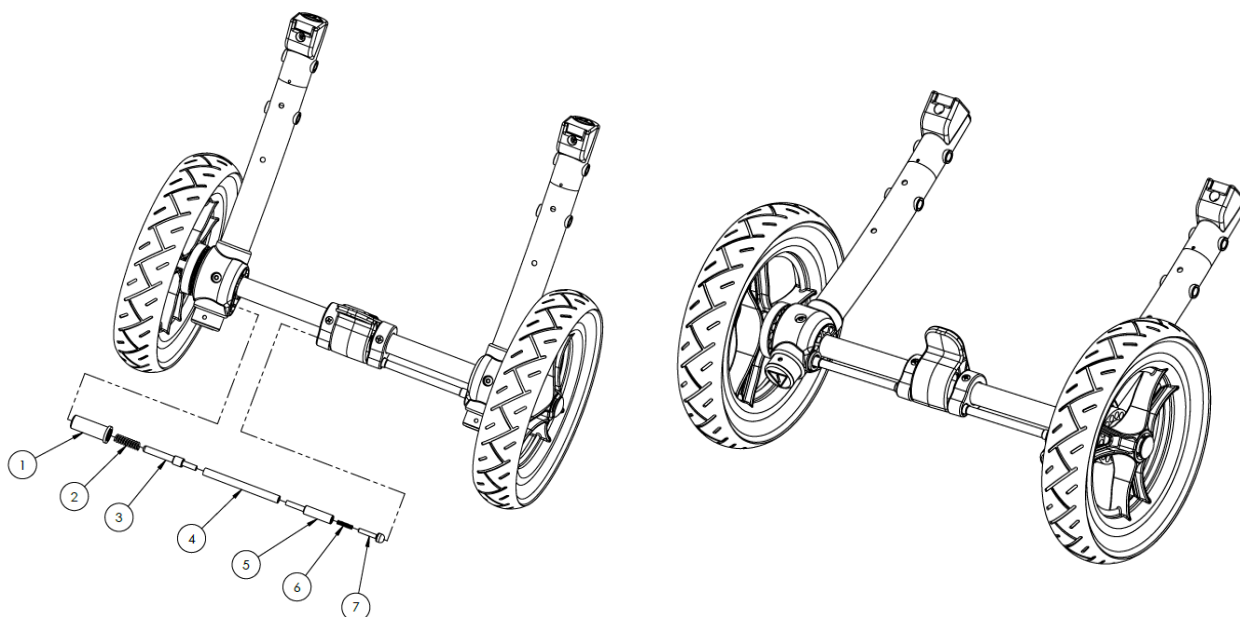

INSTRUCTIONS

1. Remove the existing wheel lock parts **items 1, 2, 3, 4, 5, 6, 7** from one side, by pushing **item 3** to the left and **item 4** to the right.
2. Pull out **items 2, 4** then other items. Do the same for the other side.
3. After cleaning the wheel lock mechanism, install the new wheel lock parts in the opposite sequence of disassembly.
4. Check that the wheel lock is working properly.


Coaster Wheel Lock

Coaster 16

997516

Kit Includes

Item	Parts#	Description	Qty
1	510622	Break Pin Sleeve	2
2	100846	Comp. Spring 0.344 ID X 0.420 OD X 1.50 LG	2
3	510655	Brake Pin Outer	2
4	510782	Brake Tube 16	2
5	510653	Middle Brake Pin	2
6	100844	Comp. Spring 0.118 ID X 0.188 OD X 0.750 LG	2
7	508415	Trekker Brake Pin Inner	2

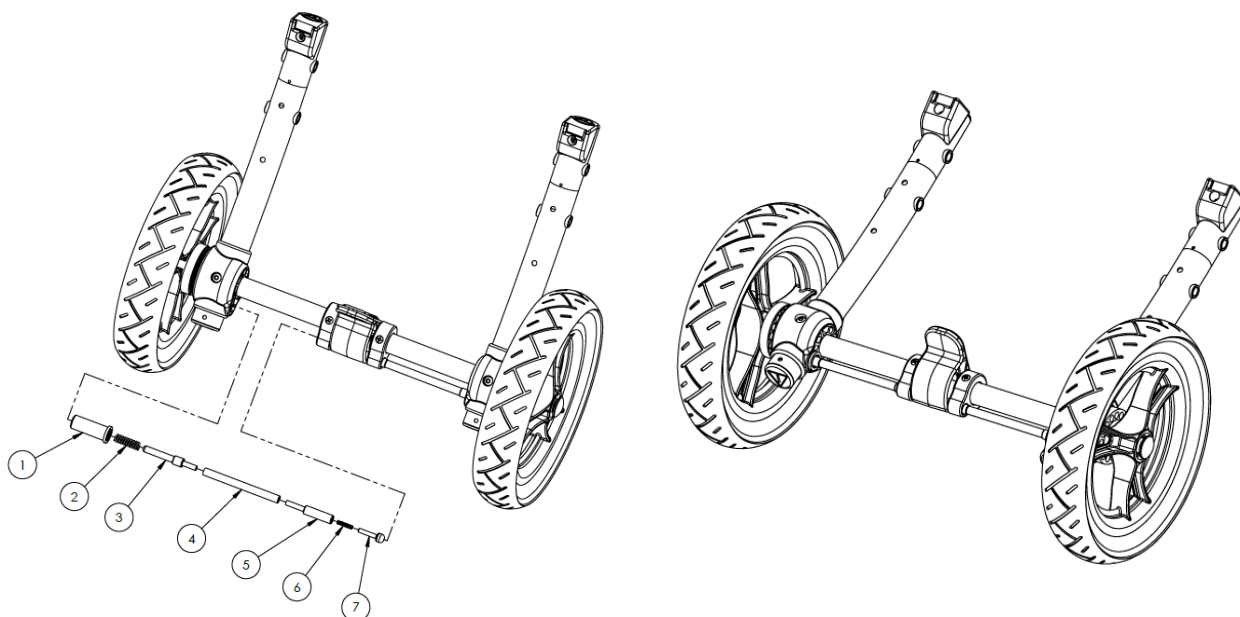

INSTRUCTIONS

1. Remove the existing wheel lock parts **items 1, 2, 3, 4, 5, 6, 7** from one side, by pushing **item 3** to the left and **item 4** to the right.
2. Pull out **items 2, 4** then other items. Do the same for the other side.
3. After cleaning the wheel lock mechanism, install the new wheel lock parts in the opposite sequence of disassembly.
4. Check that the wheel lock is working properly.


Coaster Wheel Lock

Coaster 18

997517

Kit Includes

Item	Parts#	Description	Qty
1	510622	Break Pin Sleeve	2
2	100846	Comp. Spring 0.344 ID X 0.420 OD X 1.50 LG	2
3	510655	Brake Pin Outer	2
4	510654	Brake Tube 18	2
5	510653	Middle Brake Pin	2
6	100844	Comp. Spring 0.118 ID X 0.188 OD X 0.750 LG	2
7	508415	Trekker Brake Pin Inner	2

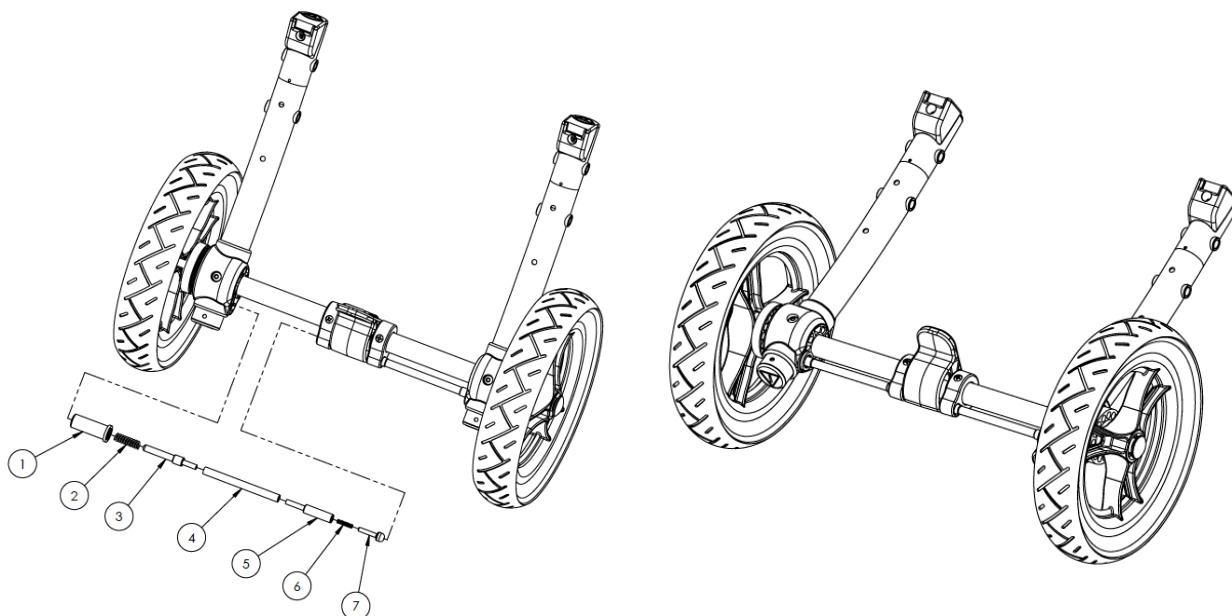


INSTRUCTIONS

1. Remove the existing wheel lock parts **items 1, 2, 3, 4, 5, 6, 7** from one side, by pushing **item 3** to the left and **item 4** to the right.
2. Pull out **items 2, 4** then other items. Do the same for the other side.
3. After cleaning the wheel lock mechanism, install the new wheel lock parts in the opposite sequence of disassembly.
4. Check that the wheel lock is working properly.

Coaster Quick Release Axle

Coaster 12,14,16,18
997518
Kit Includes

Item	Parts#	Description	Qty
1	101048	Flyer Quick Release Axle	1

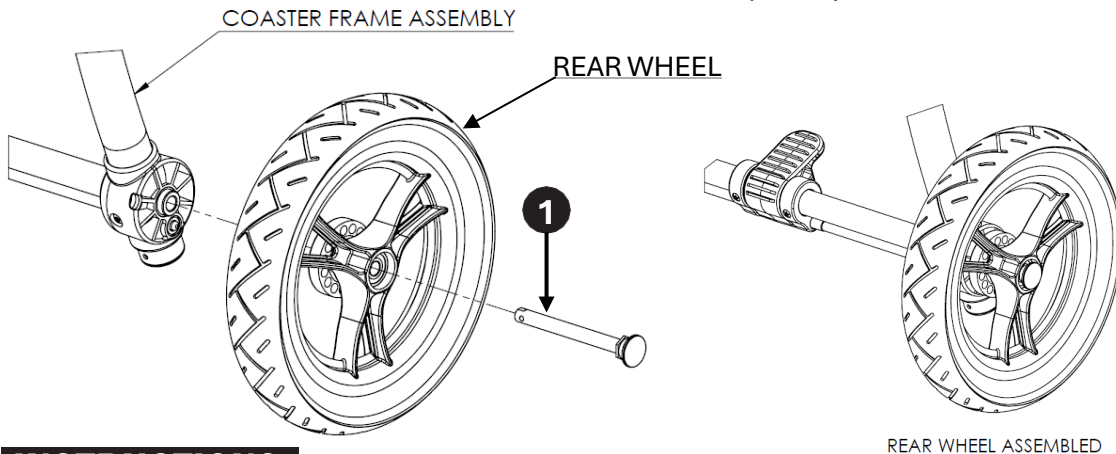

INSTRUCTIONS

1. Remove the **rear wheel** by pushing inward the existing **quick release axle** then pull out the rear wheel and the quick release **item 1**.
2. Remove the existing **quick release** from the **rear wheel**.
3. Install the new quick release **item 1** onto the wheel then in the frame assembly by pushing the quick release and the rear wheel all the way. Check that the assembly is complete by pulling on the wheel.


Coaster Front X Brace

Coaster 12

997519

Kit Includes

Item	Parts#	Description	Qty
1	511341	CTR 12 Front X Assembly	1
2	100755	Bolt 1/4"-28 X 3" Socket Button Head	4
3	100025	Nylon Washer 1/4" X 1/2"	4
4	100065	Coved Washer 3/4"	4
5	100089	Washer 1/4" AN-960-416	4
6	100172	Locknut 1/4"-28 Capped Nylon	4

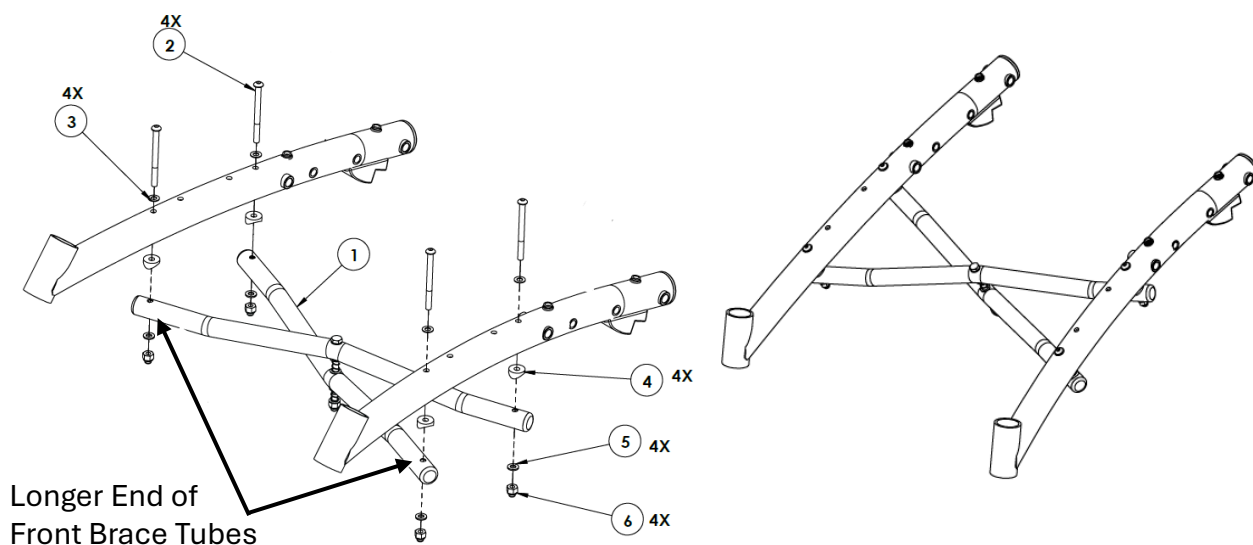


INSTRUCTIONS

1. Disassemble the existing Front X **item 1** by removing all hardware **items 2, 3, 4, 5, 6**.
2. Assemble new Front X **item 1** using the new hardware **items 2, 3, 4, 5, 6**. Make sure that longer ends of Front X Brace Tubes fit on bottom hole of front legs as shown.

Coaster Front X Brace

Coaster 14
997520
Kit Includes

Item	Parts#	Description	Qty
1	511342	CTR 14 Front X Assembly	1
2	100755	Bolt 1/4"-28 X 3" Socket Button Head	4
3	100025	Nylon Washer 1/4" X 1/2"	4
4	100065	Coved Washer 3/4"	4
5	100089	Washer 1/4" AN-960-416	4
6	100172	Locknut 1/4"-28 Capped Nylon	4

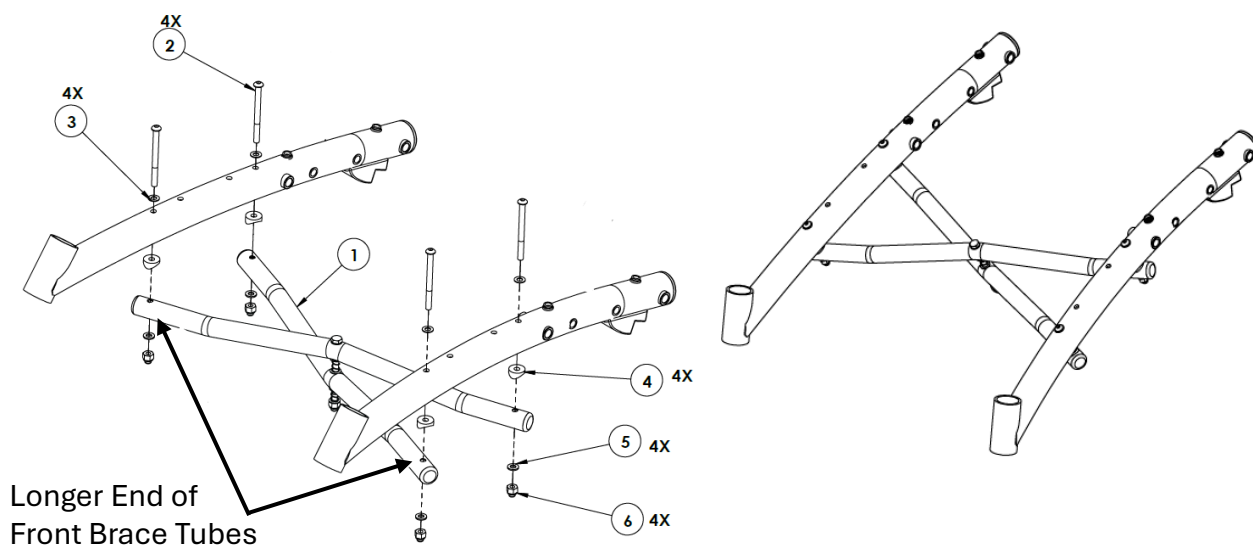

INSTRUCTIONS

1. Disassemble the existing Front X **item 1** by removing all hardware **items 2, 3, 4, 5, 6**.
2. Assemble new Front X **item 1** using the new hardware **items 2, 3, 4, 5, 6**. Make sure that longer end of Front X Brace Tubes fit on bottom hole of front legs as shown.


Coaster Front X Brace

Coaster 16

997521

Kit Includes

Item	Parts#	Description	Qty
1	511343	CTR 16 Front X Assembly	1
2	100755	Bolt 1/4"-28 X 3" Socket Button Head	4
3	100025	Nylon Washer 1/4" X 1/2"	4
4	100065	Coved Washer 3/4"	4
5	100089	Washer 1/4" AN-960-416	4
6	100172	Locknut 1/4"-28 Capped Nylon	4

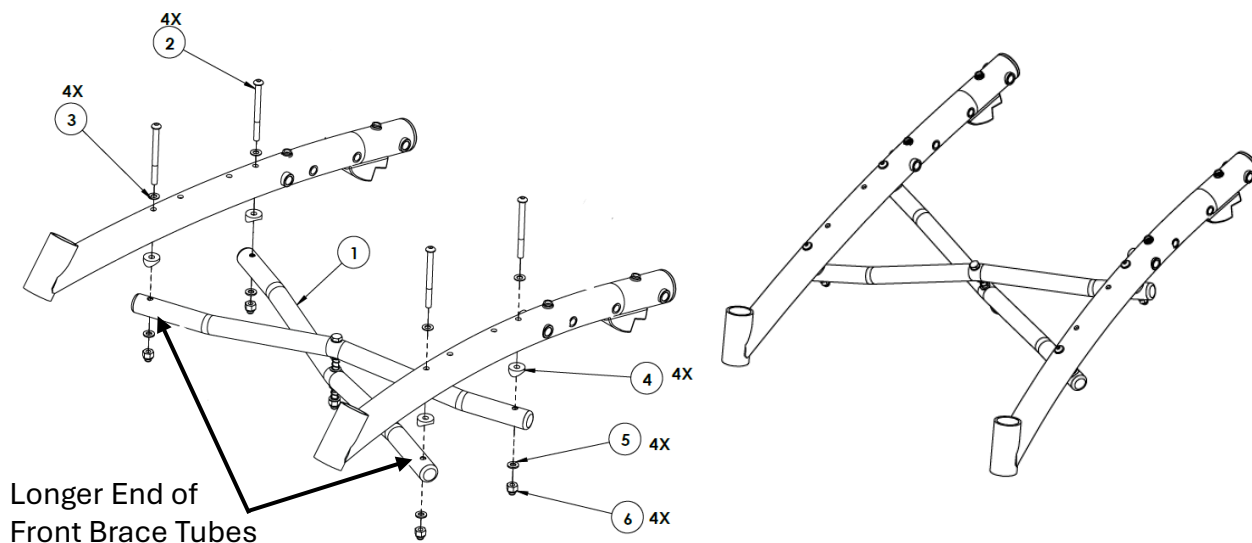


INSTRUCTIONS

1. Disassemble the existing Front X **item 1** by removing all hardware **items 2, 3, 4, 5, 6**.
2. Assemble new Front X **item 1** using the new hardware **items 2, 3, 4, 5, 6**. Make sure that longer ends of Front X Brace Tubes fit on bottom hole of front legs as shown.

Coaster Front X Brace

Coaster 18
997522
Kit Includes

Item	Parts#	Description	Qty
1	511344	CTR 18 Front X Assembly	1
2	100755	Bolt 1/4"-28 X 3" Socket Button Head	4
3	100025	Nylon Washer 1/4" X 1/2"	4
4	100065	Coved Washer 3/4"	4
5	100089	Washer 1/4" AN-960-416	4
6	100172	Locknut 1/4"-28 Capped Nylon	4

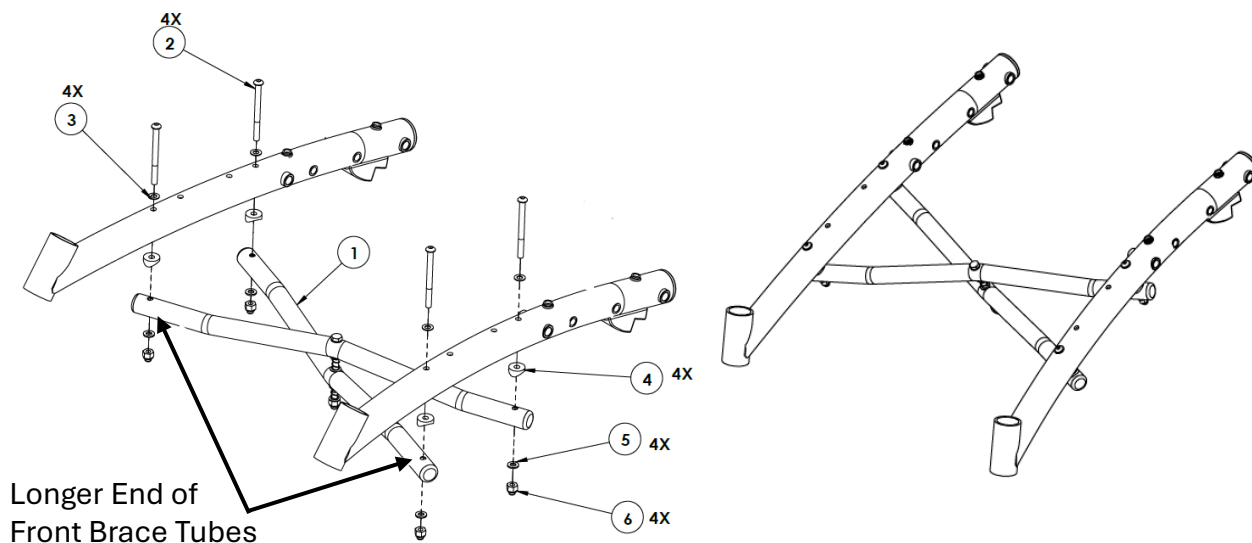


INSTRUCTIONS

1. Disassemble the existing Front X **item 1** by removing all hardware **items 2, 3, 4, 5, 6**.
2. Assemble new Front X **item 1** using the new hardware **items 2, 3, 4, 5, 6**. Make sure that longer ends of Front X Brace Tubes fit on bottom hole of front legs as shown.

Coaster Push Handle

Coaster 12
997523
Kit Includes

Item	Parts#	Description	Qty
1	510930	Handle Tube Assembly 12	1
2	100063	Sheet Metal Screw #10 X 1/2"	3
3	100652	Sheet Metal Screw #10 X 3/4"	1
4	507252	CB Closure Stap	1

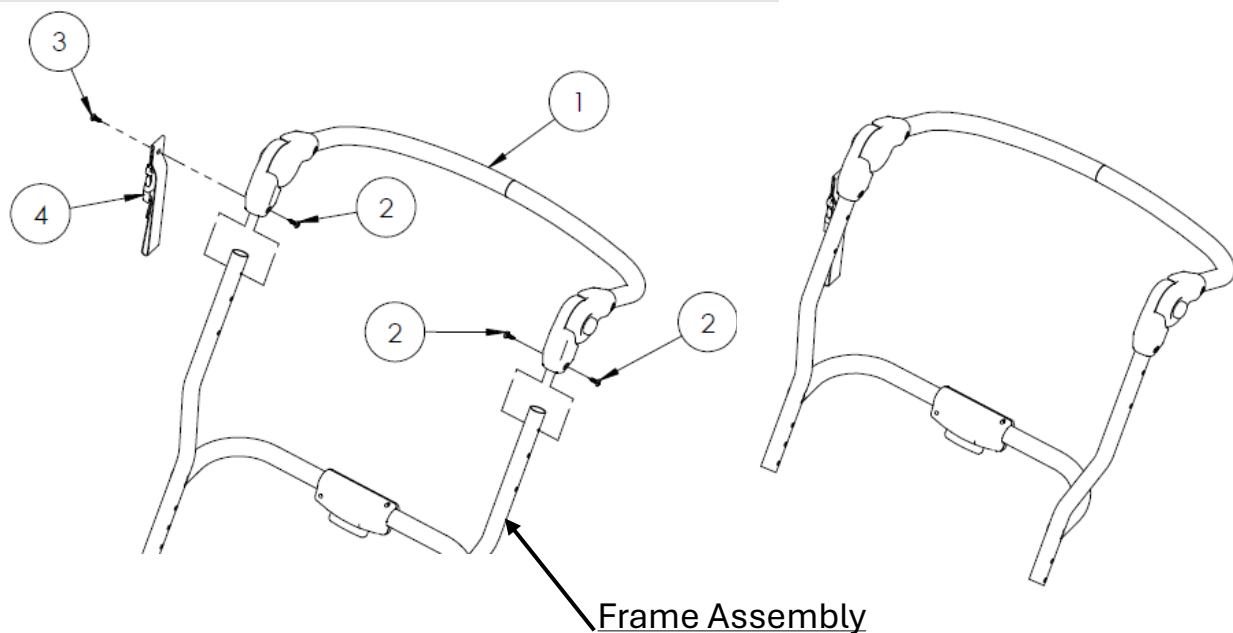


INSTRUCTIONS

1. Remove the existing Push Handle and Closure Strap **items 1, 4** by removing screws **items 2, 3**
2. Assemble the new Push Handle and the new Closure Strap **item 1, 4** using the new hardware **items 2, 3** as shown.

Coaster Push Handle

Coaster 14
997524
Kit Includes

Item	Parts#	Description	Qty
1	510931	Handle Tube Assembly 14	1
2	100063	Sheet Metal Screw #10 X 1/2"	3
3	100652	Sheet Metal Screw #10 X 3/4"	1
4	507252	CB Closure Stap	1

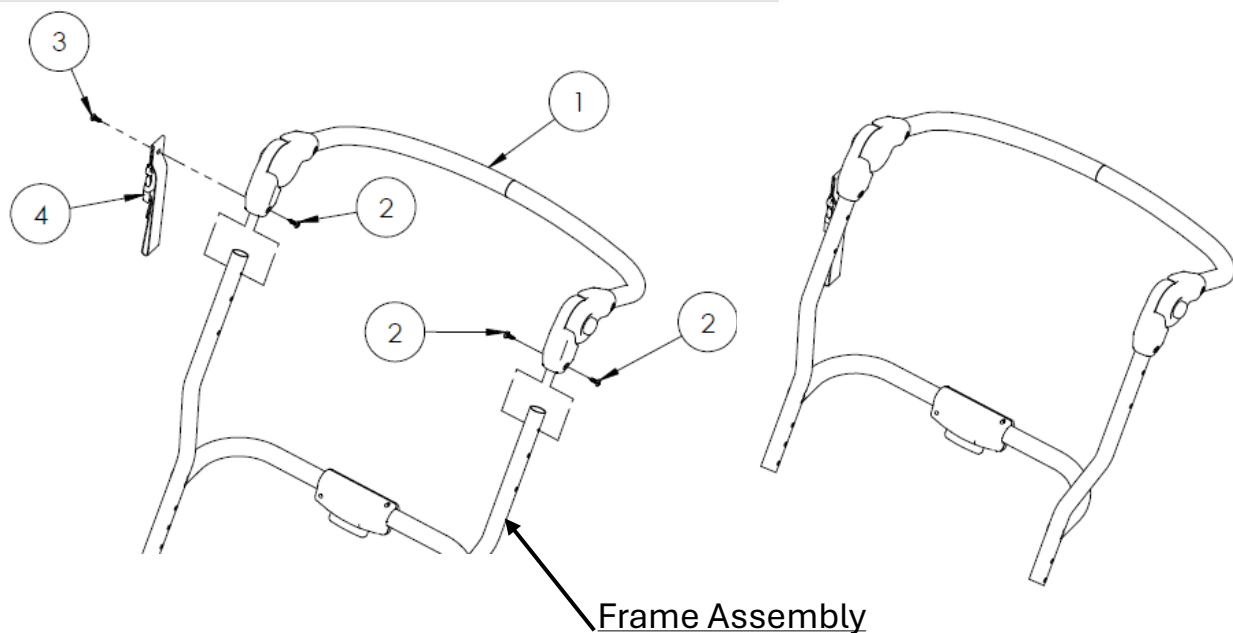


INSTRUCTIONS

1. Remove the existing Push Handle and Closure Strap **items 1, 4** by removing screws **items 2, 3**
2. Assemble the new Push Handle and the new Closure Strap **item 1, 4** using the new hardware **items 2, 3** as shown.

Coaster Push Handle

Coaster 16
997525
Kit Includes

Item	Parts#	Description	Qty
1	510932	Handle Tube Assembly 16	1
2	100063	Sheet Metal Screw #10 X 1/2"	3
3	100652	Sheet Metal Screw #10 X 3/4"	1
4	507252	CB Closure Stap	1

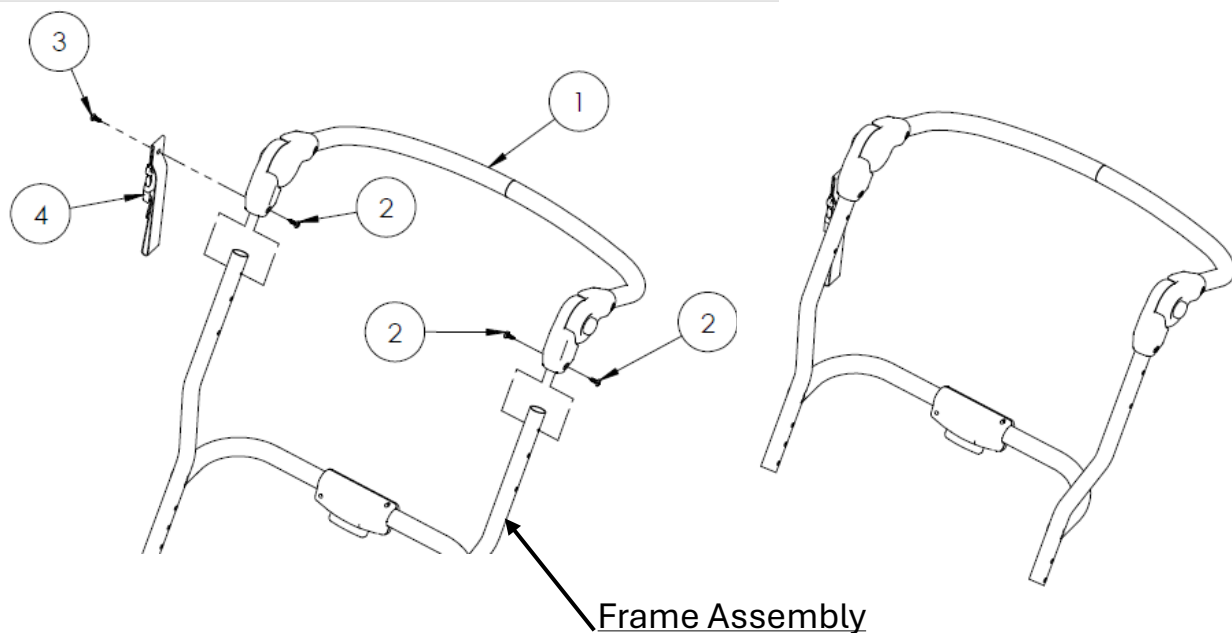


INSTRUCTIONS

1. Remove the existing Push Handle and Closure Strap **items 1, 4** by removing screws **items 2, 3**
2. Assemble the new Push Handle and the new Closure Strap **item 1, 4** using the new hardware **items 2, 3** as shown.

Coaster Push Handle

Coaster 18
997526
Kit Includes

Item	Parts#	Description	Qty
1	510933	Handle Tube Assembly 18	1
2	100063	Sheet Metal Screw #10 X 1/2"	3
3	100652	Sheet Metal Screw #10 X 3/4"	1
4	507252	CB Closure Stap	1

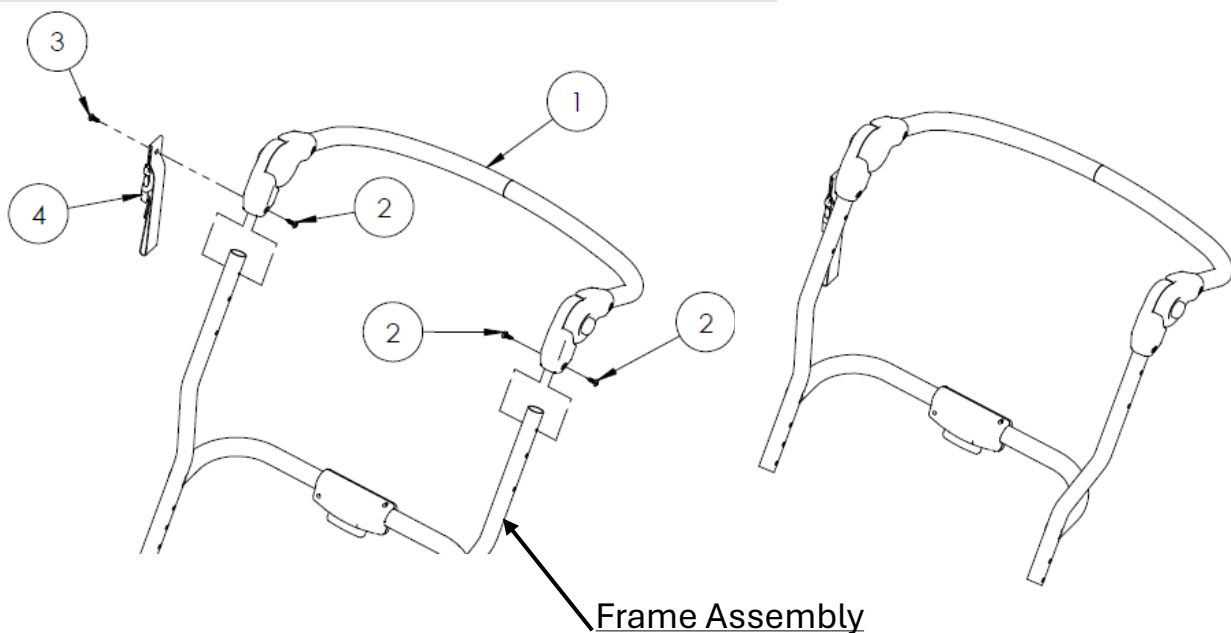

INSTRUCTIONS

1. Remove the existing Push Handle and Closure Strap **items 1, 4** by removing screws **items 2, 3**
2. Assemble the new Push Handle and the new Closure Strap **item 1, 4** using the new hardware **items 2, 3** as shown.



Convaïd Products, LLC

2830 California Street
Torrance, CA 90503, USA



+1 (844) US-Mobility (876-6245)



convaidsales.us@etac.com



www.etac.us.com



International.convaid.us@etac.com



www.etac.com

 **Convaïd**
by Etac