


Addax

Exploded views / Part lists

Rev. date (2025.11)

On the following pages you will find the parts used to assemble this product.
Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.

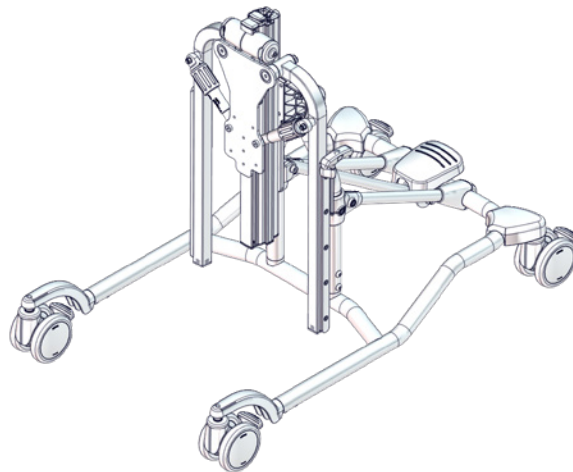


Contents

Addax size 1, 2	4
Item numbers for Addax size 1, 2	5
Addax frame size 1	6
Item numbers for Addax frame size 1.....	7
Addax frame size 2	8
Item numbers for Addax frame size 2.....	9
Profile and column bracket size 1, 2	10
Item numbers for Profile and column bracket size 1, 2	11
Abduction assembly size 1, 2.....	12
Item numbers for Abduction assembly size 1, 2.....	13
Addax accessories	14
Item numbers for Addax accessories.....	15
Cover - Knee support S (913170-60, 913170-90)	16
Item numbers for Cover - Knee support S (913170-60, 913170-90).....	16
Cover - Pelvic/chest/ext pad S (913140-60, 913140-90).....	16
Item numbers for Cover - Pelvic/chest/ext pad S (913140-60, 913140-90)	16
Cover - Chest sup. for tray S (913155-60, 913155-90)	16
Cover - Chest sup. for tray S (913155-60, 913155-90)	16
Cover - Sup. belt, fixed/SW (913101-60, 913101-90, 913102-60, 913102-90)	17
Item numbers for Cover - Sup. belt, fixed/SW (913101-60, 913101-90, 913102-60, 913102-90)	17
Cover - Sup. belt, ratchet (913111-60, 913111-90, 913112-60, 913112-90).....	17
Item numbers for Cover - Sup. belt, ratchet (913111-60, 913111-90, 913112-60, 913112-90).....	17
Cover - Sacrum support S (913160-60, 913160-90).....	18
Item numbers for Cover - Sacrum support S (913160-60, 913160-90).....	18
Cover - Sacrum support rod (913159).....	18
Item numbers for Cover - Pelvic/chest/ext pad S (913140-60, 913140-90)	18
Cover - Ext. pad, mount. opt S (913143-60, 913143-90)	18
Cover - Chest sup. for tray S (913155-60, 913155-90)	18
Cover - Upper arm sup. S Right (913131-60R, 913131-90R).....	19
Item numbers for Cover - Upper arm sup. S Right (913131-60R, 913131-90R).....	19
Cover - Upper arm sup. S Left (913131-60L, 913131-90L).....	19
Item numbers for Cover - Upper arm sup. S Left (913131-60L, 913131-90L)	19

Cover - Upper arm sup. M Right (913132-60R, 913132-90R)	20
Item numbers for Cover - Upper arm sup. M Right (913132-60R, 913132-90R)	20
Cover - Upper arm sup. M Left (913132-60L, 913132-90L)	20
Item numbers for Cover - Upper arm sup. M Left (913132-60L, 913132-90L)	20
Castors, 100 mm (913160, 913161)	21
Item numbers for Castors, 100 mm (913160, 913161)	21
Gas spring, Addax size 1 (913162)	21
Item numbers for Gas spring, Addax size 1 (913162)	21
Gas spring, Addax size 2 (913163)	21
Item numbers for Gas spring, Addax size 2 (913163)	21
Foot pedal for tilt (174692)	22
Item numbers for Foot pedal for tilt (174692)	22
Spindle for angle adjustment (abduction) (173151)	23
Item numbers for Spindle for angle adjustment (abduction) (173151)	23


Addax size 1, 2

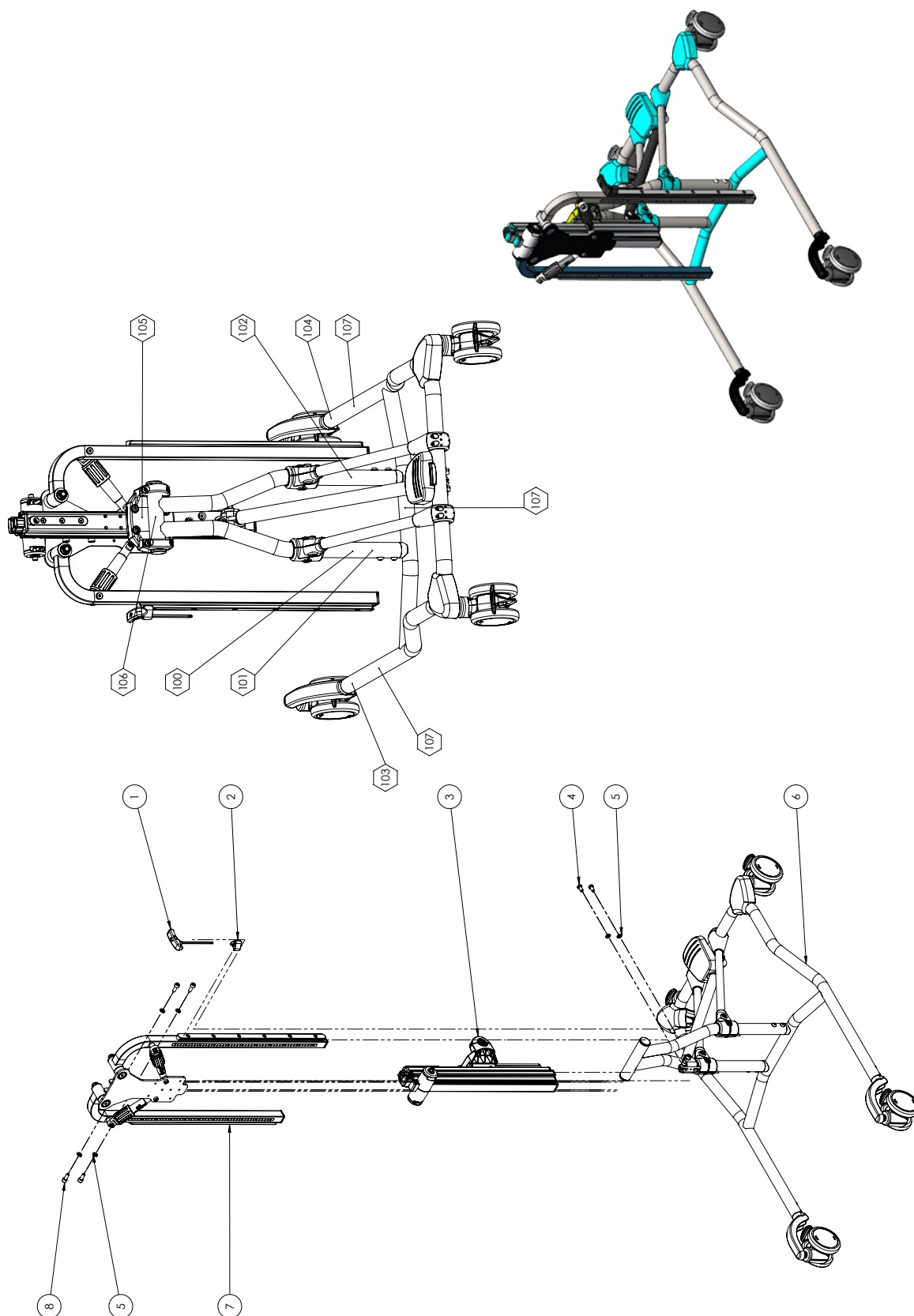

Item numbers for Addax size 1, 2

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	75291	Hexagon key nv=5	1	PCS	5707292437655
2	124384	Tool Clip	1	PCS	5707292779922
3*	169629	Profile & Column bracket sz 1	1	PCS	5715393038985
	175180	Profile & Column bracket sz 2	1	PCS	5715393038992
4	603528	Insex low head M8x14	2	PCS	5707292564474
5	1103200	Fazet washer 8x16x1,5mm, FZB	6	PCS	5707292140296
6*	171814	Frame - Size 1	1	PCS	5715393020263
	175165	Frame - Size 2	1	PCS	5715393020256
7*	171727	Abductions module - Size 1	1	PCS	5715393031238
	169373	Abductions module - Size 2	1	PCS	5715393031382
8	602800	Insex screw 8x20	4	PCS	5707292138194
100+101	493400	Label termo,3M 7872,50x24	2	PCS	5707292136855
102	494661	ISO M002 label	1	PCS	5707292989321
103	494614	R82 by Etac logo - Silver	1	PCS	5707292921611
104	494669	Addax- silver	1	PCS	5707292989406
105	0494670	Addax product label	1	PCS	5707292989413
* Choose size / color					


Addax frame size 1

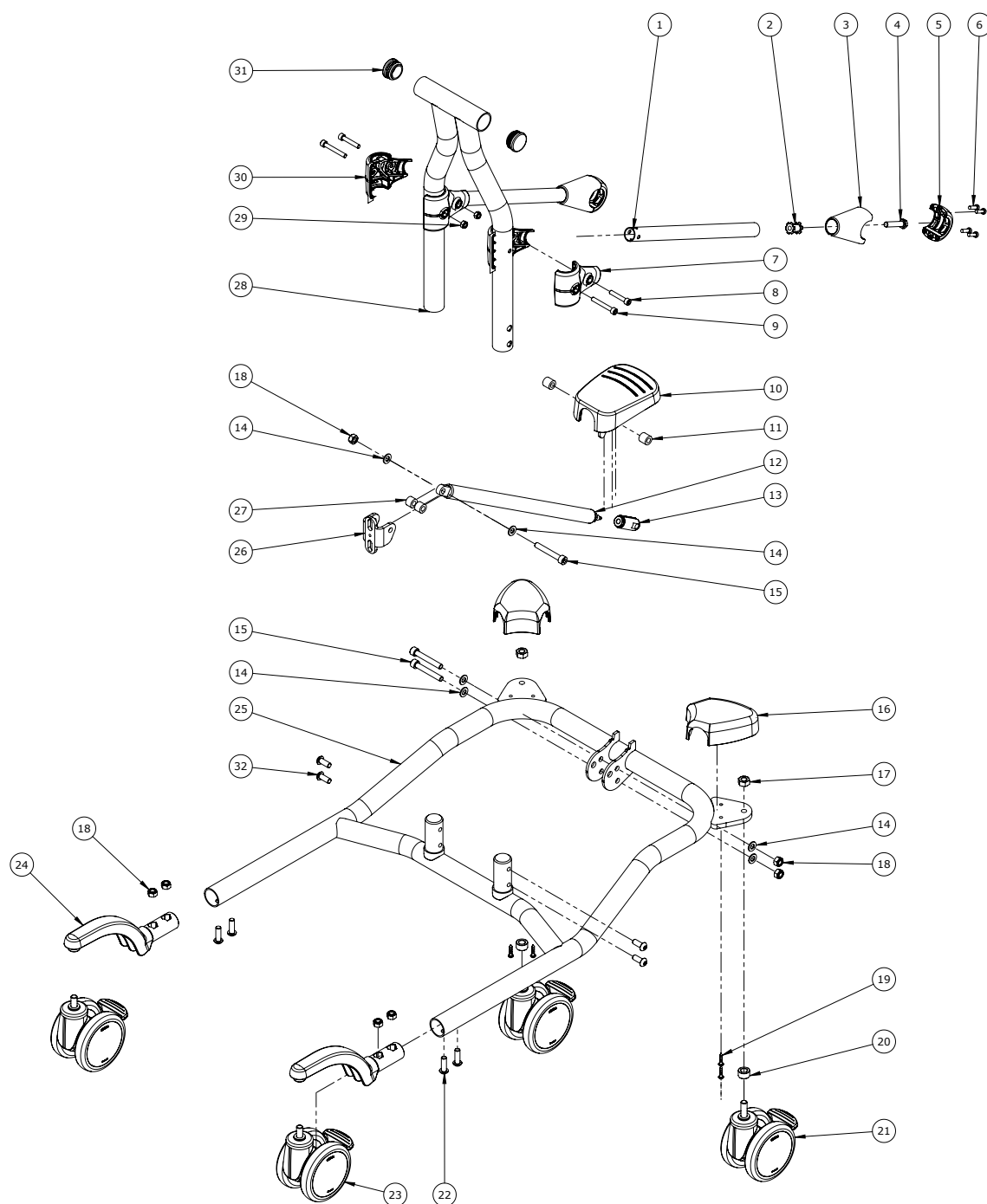

Item numbers for Addax frame size 1

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	175408	Reinforcement pipe - Size 1	2	PCS	5715393029594
2	6108119	M8 thread inserts ø18,9-20,2	2	PCS	5715393039197
3	174570	Pipe reinforcement	2	PCS	5715393029624
4	602492	Sekskantskrue M8x40 22s, A2	2	PCS	5707292412942
5	174572	Lid for pipe reinforcement	2	PCS	5715393029617
6	1940007	Eco syn 4x16 torx	8	PCS	5707292520593
7	175733	Lid pipe reinforcement	2	PCS	5715393029631
8	602435	Insex screw 6x35	2	PCS	5707292138132
9	602445	Insex M6x45 FZB	2	PCS	5707292138156
10	174692	Foot pedal, for Actuator	1	PCS	5715393031221
11	1350024	Distance piece 14x15x8,2mm	2	PCS	5707292140920
12	2205040	Bloc-o-lift 212768, 400N	1	PCS	5715393040179
13	172173	Release head M8x1	1	PCS	5715393040483
14	1103200	Fazet washer 8x16x1,5mm, FZB	6	PCS	5707292140296
15	615011	Insex M8x55 fzb shortened	3	PCS	5707292928023
16	169554	Wheel cover small	2	PCS	5715393029419
17	1607008	Nut M 10, A2 m/el-zink	2	PCS	5707292619662
18	1607000	Lock nut 8mm, large pack	7	PCS	5707292141378
19	1940009	Eco syn 4x20 torx	4	PCS	5707292550897
20	178251	Distance bushing Ø18x10,4	2	PCS	5715393040230
21	3006938	100mm double wheel with brake	2	PCS	5707292574930
22	661125	Button head 8x25 FZB	4	PCS	5707292499769
23	3006939	100mm double wheel with brake	2	PCS	5715393031399
24	173238	Wheel bracket	2	PCS	5715393029426
25	171809	Bottom frame welded - Size 1	1	PCS	5715393029433
26	166275	Bracket for angle adjustment	1	PCS	5715393029808
27	1350021	Distance piece 11x15x8,2mm	2	PCS	5707292140890
28	171803	Top frame - Size 1	1	PCS	5715393029525
29	1606500	Lock nut 6 mm	4	PCS	5707292141361
30	175731	Lid pipe reinforcement	2	PCS	5715393029648
31	3501860	Dupsko DPD32	2	PCS	5707292840097
32	662002	Button head M8x20, A2 w/glue	4	PCS	5707292600554


Addax frame size 2

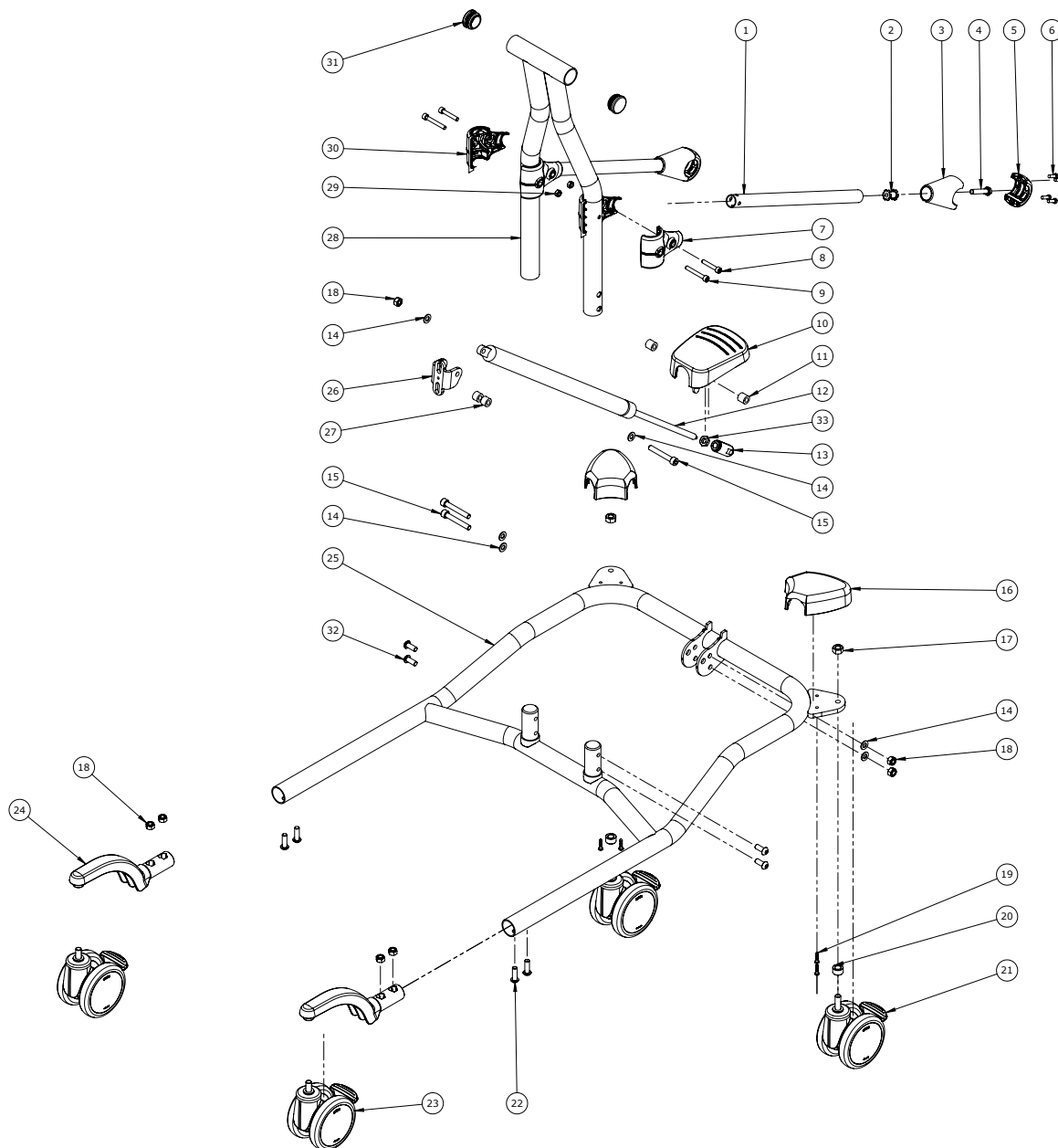

Item numbers for Addax frame size 2

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	174839	Reinforcement pipe - Size 2	2	PCS	5715393029600
2	6108119	M8 thread inserts ø18,9-20,2	2	PCS	5715393039197
3	174570	Pipe reinforcement	2	PCS	5715393029624
4	602492	Sekskantskrue M8x40 22s, A2	2	PCS	5707292412942
5	174572	Lid for pipe reinforcement	2	PCS	5715393029617
6	610615	Screw ø5,0 x16 Delta PT FZB	8	PCS	5715393044290
7	175733	Lid pipe reinforcement	2	PCS	5715393029631
8	602435	Insex screw 6x35	2	PCS	5707292138132
9	602445	Insex M6x45 FZB	2	PCS	5707292138156
10	174692	Foot pedal, for Actuator	1	PCS	5715393031221
11	1350024	Distance piece 14x15x8,2mm	2	PCS	5707292140920
12	2205050	Bloc-o-lift 5415HI, 500N	1	PCS	5715393040186
13	6118822	Horizontal release	1	PCS	5707292186744
14	1103200	Fazet washer 8x16x1,5mm, FZB	6	PCS	5707292140296
15	615011	Insex M8x55 fzb shortened	3	PCS	5707292928023
16	169554	Wheel cover small	2	PCS	5715393029419
17	1607008	Nut M 10, A2 m/el-zink	2	PCS	5707292619662
18	1607000	Lock nut 8mm, large pack	7	PCS	5707292141378
19	1940009	Eco syn 4x20 torx	4	PCS	5707292550897
20	178251	Distance bushing Ø18x10,4	2	PCS	5715393040230
21	3006938	100mm double wheel with brake	2	PCS	5707292574930
22	661125	Button head 8x25 FZB	4	PCS	5707292499769
23	3006939	100mm double wheel with brake	2	PCS	5715393031399
24	173238	Wheel bracket	2	PCS	5715393029426
25	169358	Bottom frame welded - Size 2	1	PCS	5715393029440
26	166275	Bracket for angle adjustment	1	PCS	5715393029808
27	1350021	Distance piece 11x15x8,2mm	2	PCS	5707292140890
28	169360	Top frame - Size 2	1	PCS	5715393029532
29	1606500	Lock nut 6 mm	4	PCS	5707292141361
30	175731	Lid pipe reinforcement	2	PCS	5715393029648
31	3501860	Dupsko DPD32	2	PCS	5707292840097
32	662002	Button head M8x20, A2 w/glue	4	PCS	5707292600554
33	1761000	Counter nut 10x1,0 fine thread	1	PCS	5707292141538


Profile and column bracket size 1, 2

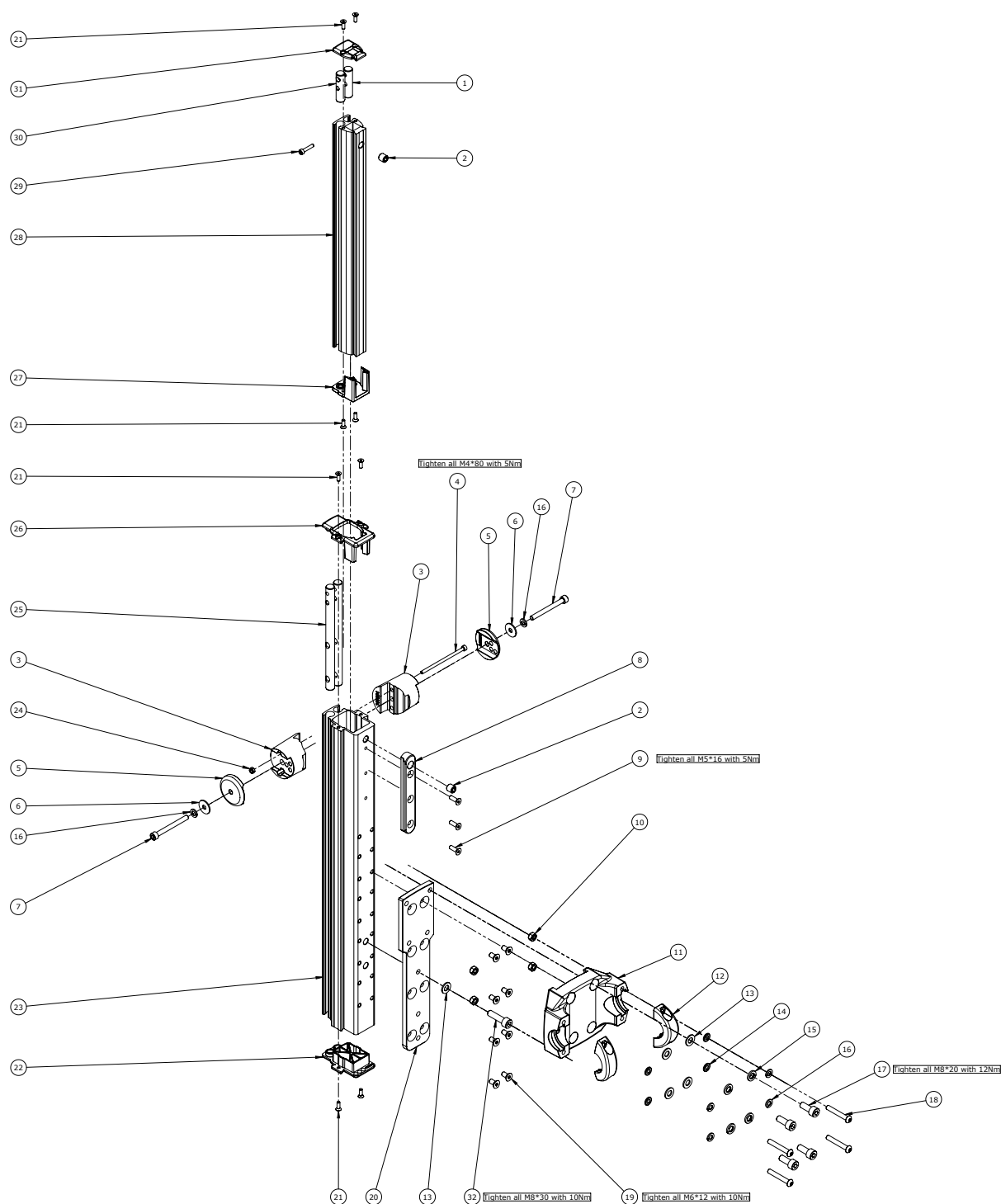

Item numbers for Profile and column bracket size 1, 2

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	153760	Chest module connecting shaft	1	PCS	5715393030293
2	701011	Stopscrew M10x10 flat rf	2	PCS	5707292533586
3	169984	Clamp house	2	PCS	5715393030057
4	602447	Insex M4x80 FZB	1	PCS	5715393023271
5	169779	Clamp house lid	2	PCS	5715393030064
6	1102001	Washer 6x20x1,5 mm, RF	2	PCS	5707292565754
7	604471	Socket head cap screw M6*x60	2	PCS	5707292950376
8	172107	Dovetail Add-on	1	PCS	5715393029761
9	615016	Insex screw 5x16	3	PCS	5707292138859
10	1606500	Lock nut 6 mm	4	PCS	5707292141361
11	173261	Column bracket	1	PCS	5715393029785
12	176149	Column bracket lid	2	PCS	5715393029792
13	1103200	Fazet washer 8x16x1,5mm, FZB	4	PCS	5707292140296
14	1103000	Fazet washer 6mm	4	PCS	5707292140289
15	1101000	Center spring washer 8mm	4	PCS	5707292140173
16	1100800	Center spring washer 6mm	4	PCS	5707292140166
17	602800	Insex screw 8x20	4	PCS	5707292138194
18	641627	Button head M6x40 FZB	4	PCS	5715393022434
19	621200	Insex screw 6x12mm	8	PCS	5707292138958
20	173188	Column bracket plate	1	PCS	5715393029778
21	614012	Insex base M4x12, FZB	8	PCS	5707292138811
22	152486	Bottom cover for outer column	1	PCS	5715393029730
23	169627	Main Column	1	PCS	5715393029709
24	1606275	Lock nut 4mm, large pack	1	PCS	5707292141347
25	158941	Abduc. module connecting shaft	2	PCS	5715393029884
26	152467	Top cover for outer column	1	PCS	5715393029723
27	175414	Bottom cover for inner column	1	PCS	5715393029754
28	152464	Inner Column	1	PCS	5715393029716
29	609721	Insex 4x20 FZB	1	PCS	5707292439222
30	178517	Chest module connecting shaft	1	PCS	5715393037124
31	152470	Top cover for inner column	1	PCS	5715393029747
32	602600	Insex 8x30	1	PCS	5707292138170


Abduction assembly size 1, 2

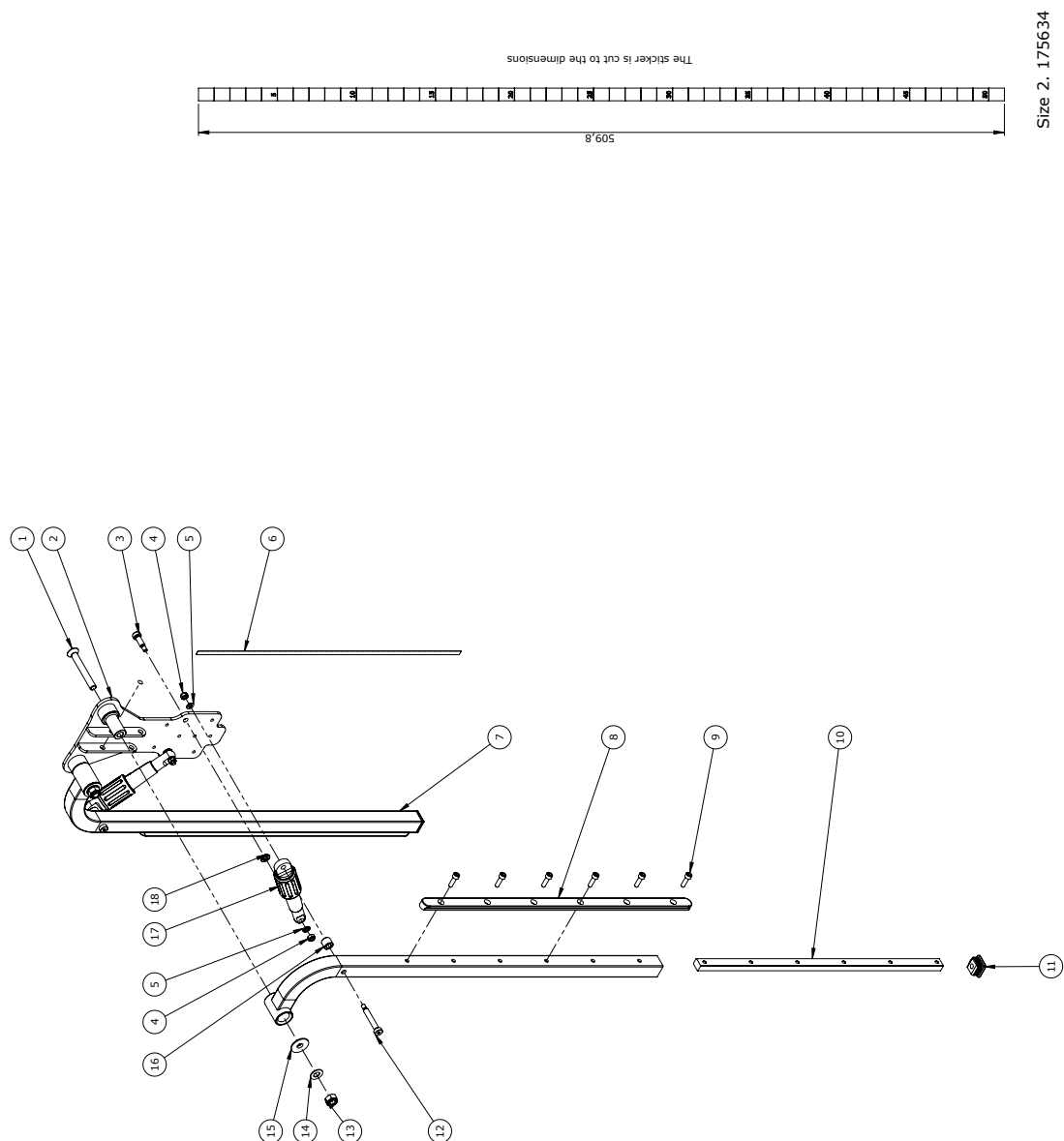

Item numbers for Abduction assembly size 1, 2

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	631808	Counter screw M10x80, FZB	2	PCS	5715393027491
2	158939	Abductions bracket	1	PCS	5715393029815
3	683815	Tight-fitting bolt, 8x20 (M6)	2	PCS	5707292488541
4	1606500	Lock nut 6 mm	4	PCS	5707292141361
5	1103000	Fazet washer 6mm	4	PCS	5707292140289
6	102510	Adjustment indicator, size 2	2	PCS	5707292562548
7	168747	Inner leg welded - Size 2	2	PCS	5715393029907
8	174782	Leg attachment for clamps sz 2	2	PCS	5715393029983
9	602300	Insex screw 6x20	12	PCS	5707292138101
10	175890	Bolting bracket	2	PCS	5715393030002
11	2510250	End cap square 25 mm, black	2	PCS	5715393041190
12	683825	Tight-fitting bolt, 8x45 (M6)	2	PCS	5715393023288
13	1607500	Lock nut 10 mm	2	PCS	5707292141385
14	1102500	Planning washer 10 mm	2	PCS	5707292140234
15	168710	Conical bushing	2	PCS	5715393029969
16	168724	Distance bushing	2	PCS	5715393029945
17	173151	Spindle for angle adj.	2	PCS	5715393029853
18	168725	Distance bushing	2	PCS	5715393029952


Addax accessories

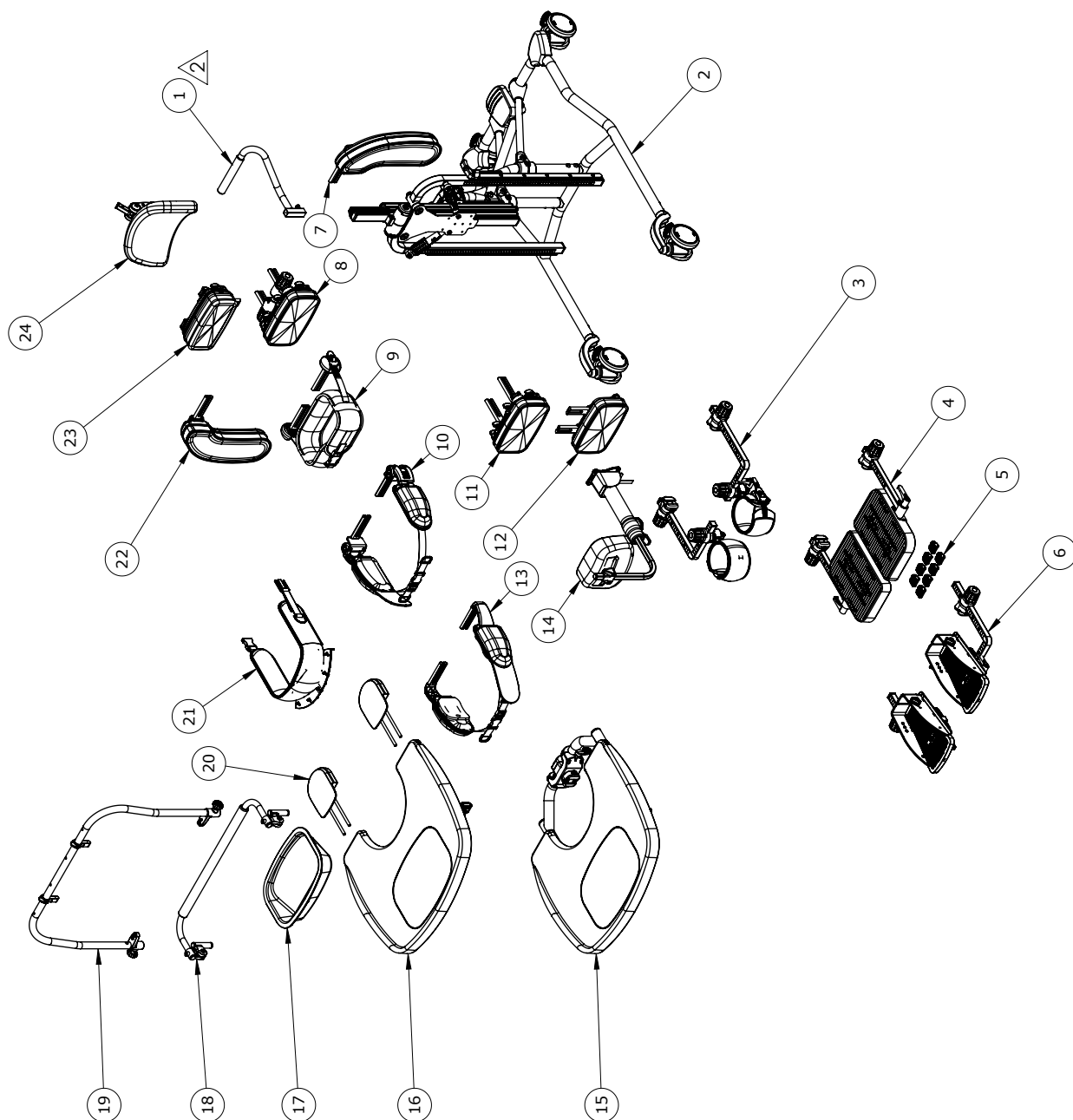

Item numbers for Addax accessories

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	912165	Push handle	1	PCS	5715393059799
2*	912001000	Addax size 1	1	PCS	5715393022809
	912002000	Addax size 2	1	PCS	5715393022823
3*	912170-60	Knee support 3D S, grey	1	SET	5715393022885
	912170-90	Knee support 3D S, black wipeable	1	SET	5715393022892
4	912181	Foot supports, parted S	1	SET	5715393022847
5	912164	Fix-locks, black	1	SET	5715393039685
6*	912182	Sandals, 3D adjustable, size 2	1	SET	5715393022854
	912183	Sandals, 3D adjustable, size 3	1	SET	5715393022861
7+22*	912131-60	Upper arm supports S, grey	1	SET	5715393023134
	912131-90	Upper arm supports S, black wipeable	1	SET	5715393023158
	912132-60	Upper arm supports M, grey	1	SET	5715393023141
	912132-90	Upper arm supports M, black wipeable	1	SET	5715393023165
8*	912142-60	Chest pad S, grey	1	PCS	5715393023073
	912142-90	Chest pad S, black wipeable	1	PCS	5715393023080
9*	912111-60	Support belt, ratchet S, grey	1	PCS	5715393022984
	912111-90	Support belt, ratchet S, black wipeable	1	PCS	5715393023004
	912112-60	Support belt, ratchet M, grey	1	PCS	5715393022991
	912112-90	Support belt, ratchet M, black wipeable	1	PCS	5715393023011
10*	912121-60	Support belt, swing away S, grey	1	PCS	5715393023028
	912121-90	Support belt, swing away S, black wipeable	1	PCS	5715393023042
	912122-60	Support belt, swing away M, grey	1	PCS	5715393023035
	912122-90	Support belt, swing away M, black wipeable	1	PCS	5715393023035
11*	912140-60	Pelvic pad S, grey	1	PCS	5715393022908
	912140-90	Pelvic pad S, black wipeable	1	PCS	5715393022915
12*	912141-60	Extension pad S, grey	1	PCS	5715393022922
	912141-90	Extension pad S, black wipeable	1	PCS	5715393022939
13*	912101-60	Support belt, fixed S, grey	1	PCS	5715393022946
	912101-90	Support belt, fixed S, black wipeable	1	PCS	5715393022960
	912102-60	Support belt, fixed M, grey	1	PCS	5715393022953
	912102-90	Support belt, fixed M, black wipeable	1	PCS	5715393022977
14*	912160-60	Sacrum support S incl rod + grey sup. Cov	1	PCS	5715393023066
	912160-90	Sacrum support S incl rod + black wip. sup. Cov	1	PCS	5715393039319
15	912151	Tray, supine S	1	PCS	5715393023189
16	912150	Tray, prone S	1	PCS	5715393023172
17	912154	Bowl, for tray	1	PCS	5715393023202
18	8910750	Grip handle for tray	1	PCS	5707292486622
19	8910740	Toy bar for tray	1	PCS	5707292457899
20	912153	Elbow support	1	SÆT	5715393023196
21*	912155-60	Chest sup. for tray, prone S, grey	1	PCS	5715393023097
	912155-90	Chest sup. for tray, prone S, black wipeable	1	PCS	5715393023103
23*	912143-60	Extension pad, mount. option S, grey	1	PCS	5715393023110
	912143-90	Extension pad, mount. option S, black wipeable	1	PCS	5715393023127
24	N/A	see "Head supports" spare parts list	N/A	N/A	N/A
* Choose size / color					


Cover - Knee support S (913170-60, 913170-90)

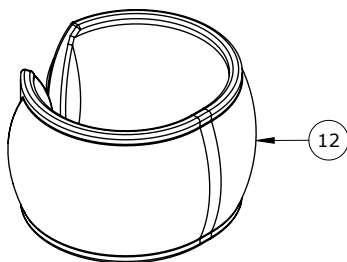

Item numbers for Cover - Knee support S (913170-60, 913170-90)

Position	No.	Description	Quantity	Unit of Measure	Ean code
12*	176727-60	Sub. Assy. Knee Support, 3D, grey	1	PCS	5715393041541
	176727-90	Sub. Assy. Knee Support, 3D, black wipeable	1	PCS	5715393041558
* Choose size / color					


Cover - Pelvic/chest/ext pad S (913140-60, 913140-90)

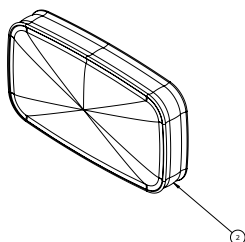

Item numbers for Cover - Pelvic/chest/ext pad S (913140-60, 913140-90)

Position	No.	Description	Quantity	Unit of Measure	Ean code
2*	176675-60	Cushion Chest/pelvic/ext pad S, grey	1	PCS	5715393041527
	176675-90	Cushion Chest/pelvic/ext pad S, black wipeable	1	PCS	5715393041534
* Choose size / color					


Cover - Chest sup. for tray S (913155-60, 913155-90)

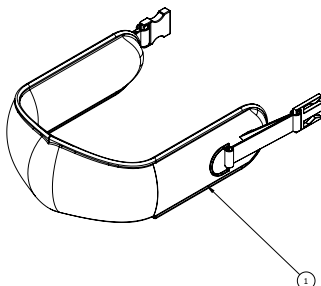

Cover - Chest sup. for tray S (913155-60, 913155-90)

Position	No.	Description	Quantity	Unit of Measure	Ean code
1*	178952-60	Cov. Chest sup. Tray prone S, grey	1	PCS	5715393041671
	178952-90	Cov. Chest sup. Tray prone S, black wipeable	1	PCS	5715393041688
* Choose size / color					

Cover - Sup. belt, fixed/SW (913101-60, 913101-90, 913102-60, 913102-90)

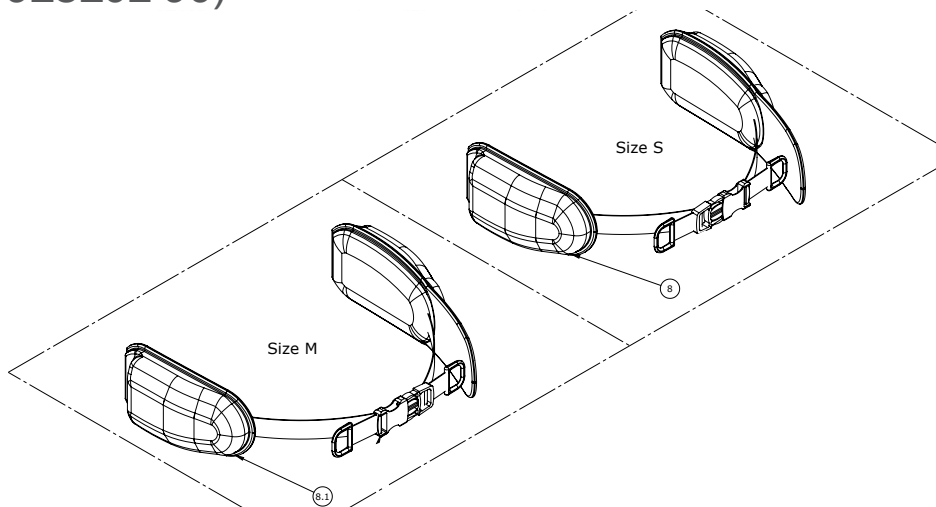


Item numbers for Cover - Sup. belt, fixed/SW (913101-60, 913101-90, 913102-60, 913102-90)

Position	No.	Description	Quantity	Unit of Measure	Ean code
8*	176582-60	Cover, belt swingaway/fixed S, grey	1	PCS	5715393032631
	176582-90	Cover, belt swingaway/fixed S, black wipeable	1	PCS	5715393032648
8.1*	176583-60	Cover, belt swingaway/fixed M, grey	1	PCS	5715393032617
	176583-90	Cover, belt swingaway/fixed M, black wipeable	1	PCS	5715393032624

* Choose size / color

Cover - Sup. belt, ratchet (913111-60, 913111-90, 913112-60, 913112-90)

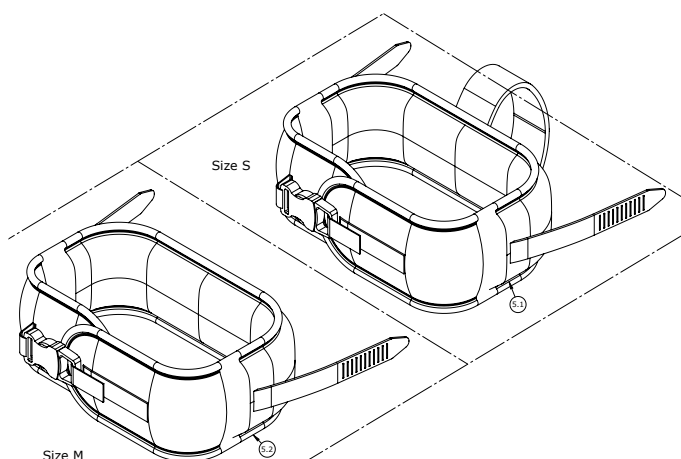


Item numbers for Cover - Sup. belt, ratchet (913111-60, 913111-90, 913112-60, 913112-90)

Position	No.	Description	Quantity	Unit of Measure	Ean code
5.1*	175349-60	Cover support belt, rachet S, grey	1	PCS	5715393032594
	175349-90	Cover support belt, rachet S, black wipeable	1	PCS	5715393032600
5.2*	175350-60	Cover support belt, rachet M, grey	1	PCS	5715393032570
	175350-90	Cover support belt, rachet M, black wipeable	1	PCS	5715393032587

* Choose size / color


Cover - Sacrum support S (913160-60, 913160-90)

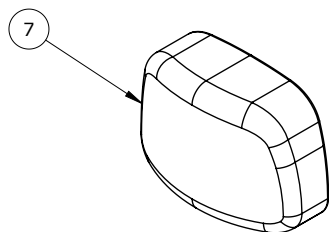

Item numbers for Cover - Sacrum support S (913160-60, 913160-90)

Position	No.	Description	Quantity	Unit of Measure	Ean code
7*	175346-60	Cover for Sacrum Support, grey	1	PCS	5715393032549
	175346-90	Cover for Sacrum Support, black wipeable	1	PCS	5715393032556
* Choose size / color					


Cover - Sacrum support rod (913159)

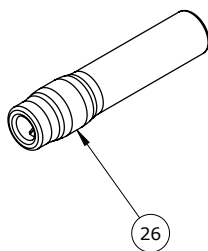

Item numbers for Cover - Pelvic/chest/ext pad S (913140-60, 913140-90)

Position	No.	Description	Quantity	Unit of Measure	Ean code
26	176399-90	Cover sacrum support rod	1	PCS	5715393032563


Cover - Ext. pad, mount. opt S (913143-60, 913143-90)

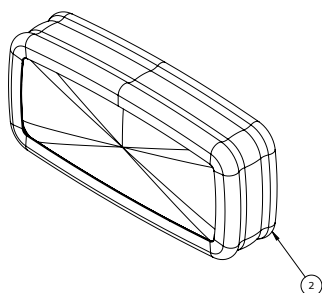

Cover - Chest sup. for tray S (913155-60, 913155-90)

Position	No.	Description	Quantity	Unit of Measure	Ean code
2*	176674-60	Cush. ext. pad w. mount opt. S, grey	1	PCS	5715393041572
	176674-90	Cush. ext. pad w. mount opt. S, black wipeable	1	PCS	5715393041589
* Choose size / color					


Cover - Upper arm sup. S Right (913131-60R, 913131-90R)

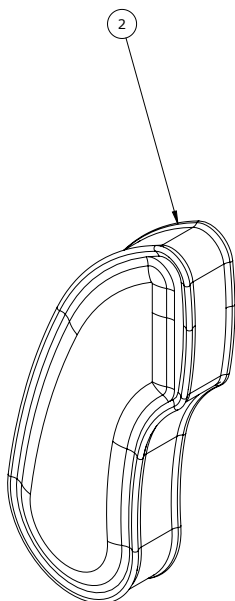

Item numbers for Cover - Upper arm sup. S Right (913131-60R, 913131-90R)

Position	No.	Description	Quantity	Unit of Measure	Ean code
2*	175342-60	Cover, Upper arm support S, R, grey	PCS	1	5715393032686
	175342-90	Cover, Upper arm support S, R, black wipeable	PCS	1	5715393032693
* Choose size / color					


Cover - Upper arm sup. S Left (913131-60L, 913131-90L)

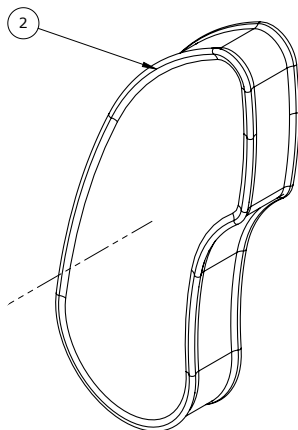

Item numbers for Cover - Upper arm sup. S Left (913131-60L, 913131-90L)

Position	No.	Description	Quantity	Unit of Measure	Ean code
2*	175343-60	Cover, Upper arm support S, L, grey	1	PCS	5715393032921
	175343-90	Cover, Upper arm support S, L, black wipeable	1	PCS	5715393032679
* Choose size / color					


Cover - Upper arm sup. M Right (913132-60R, 913132-90R)

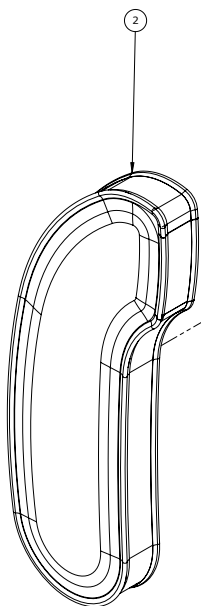

Item numbers for Cover - Upper arm sup. M Right (913132-60R, 913132-90R)

Position	No.	Description	Quantity	Unit of Measure	Ean code
2*	175344-60	Cover, Upper arm support M, R, grey	1	PCS	5715393032655
	175344-90	Cover, Upper arm support M, R, black wipeable	1	PCS	5715393032662
* Choose size / color					



Cover - Upper arm sup. M Left (913132-60L, 913132-90L)

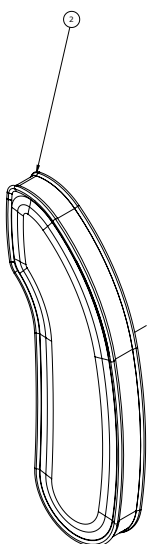

Item numbers for Cover - Upper arm sup. M Left (913132-60L, 913132-90L)

Position	No.	Description	Quantity	Unit of Measure	Ean code
2*	175345-60	Cover, Upper arm support M, L, grey	1	PCS	5715393037087
	175345-90	Cover, Upper arm support M, L, black wipeable	1	PCS	5715393037070
* Choose size / color					


Castors, 100 mm (913160, 913161)

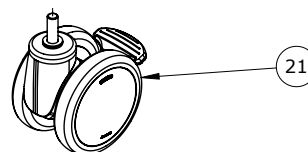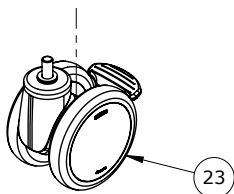

Item numbers for Castors, 100 mm (913160, 913161)

Position	No.	Description	Quantity	Unit of Measure	Ean code
21	3006938	100mm double wheel with brake	2	PCS	5707292574930
23	3006939	100mm double wheel with brake	2	PCS	5715393031399


Gas spring, Addax size 1 (913162)

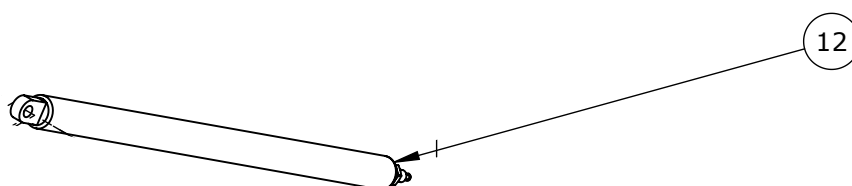

Item numbers for Gas spring, Addax size 1 (913162)

Position	No.	Description	Quantity	Unit of Measure	Ean code
12	2205040	Bloc-o-lift 212768, 400N	1	PCS	5715393040179


Gas spring, Addax size 2 (913163)

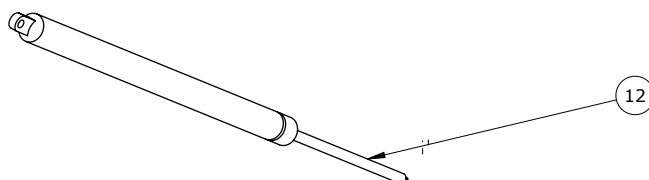

Item numbers for Gas spring, Addax size 2 (913163)

Position	No.	Description	Quantity	Unit of Measure	Ean code
12	2205050	Bloc-o-lift 5415HI, 500N	1	PCS	5715393040186


Foot pedal for tilt (174692)

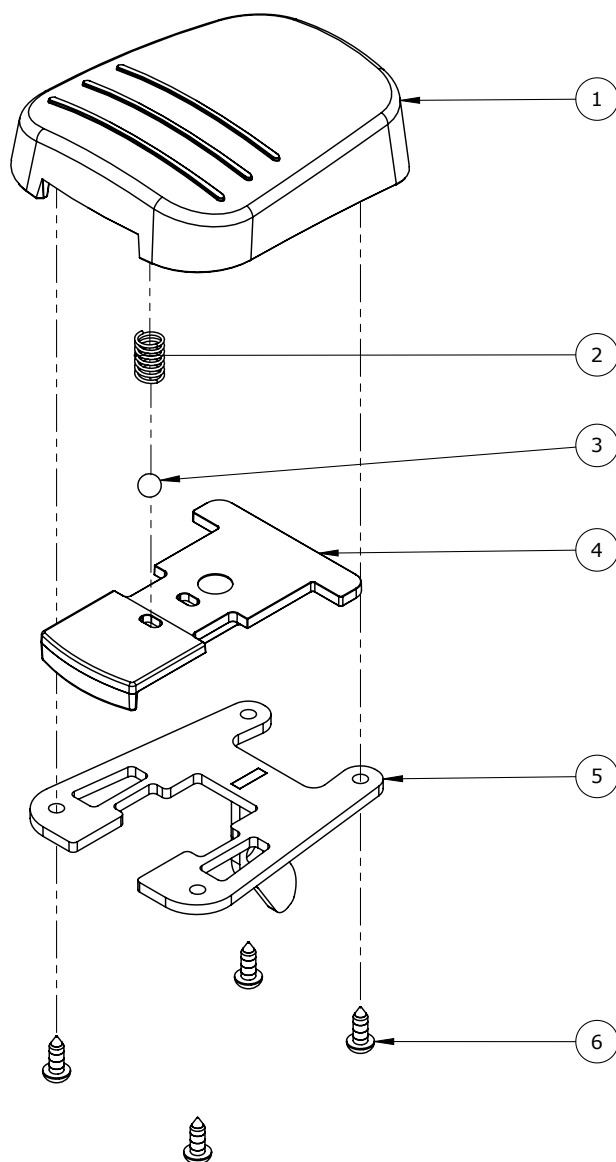

Item numbers for Foot pedal for tilt (174692)

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	174990	Actuator Pedal	1	PCS	5715393029686
2	310125	Pressure spring 0,9x8u/Ø12+0,5	1	PCS	5707292135575
3	1940052	Steel ball Ø8mm class G40	1	PCS	5715393039296
4	174693	Actuator Slider	1	PCS	5715393029693
5	175205	Actuator Bracket	1	PCS	5715393029655
6	1940027	Ecosyn phx 5x14 Torx FZB	4	PCS	5707292577818

Spindle for angle adjustment (abduction) (173151)

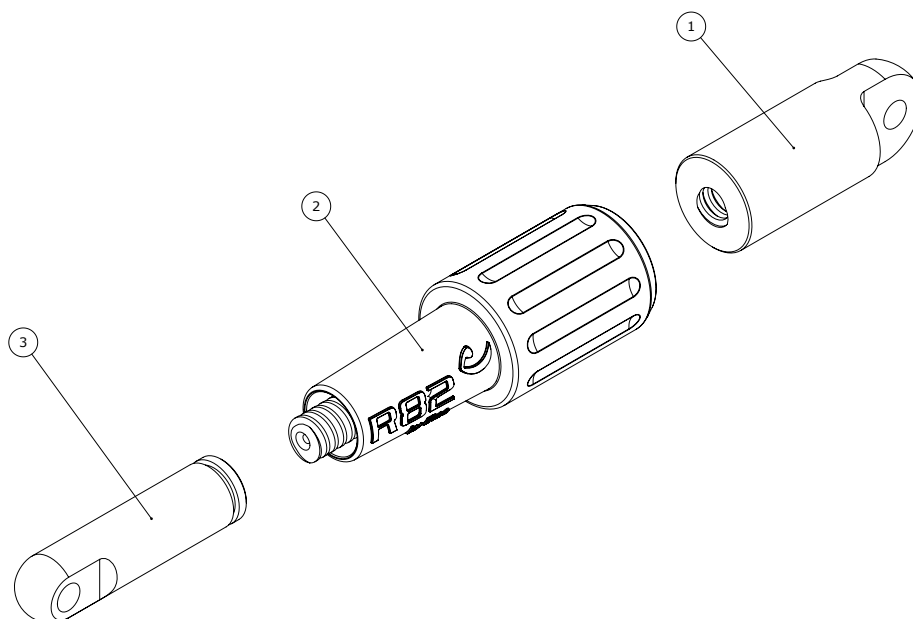

Item numbers for Spindle for angle adjustment (abduction) (173151)

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	173149	Threaded bush	1	PCS	5715393029877
2	173150	Spindle house	1	PCS	5715393029860
3	101197	Lower threaded bush, spindle	1	PCS	5707292569271