

bronco


Exploded views / Part lists

Rev. date (June 2006)




Contents

Bronco size 0	3
Accessories for Bronco size 0	6
Bronco size 1+2	8
Accessories for Bronco size 1+2.....	11

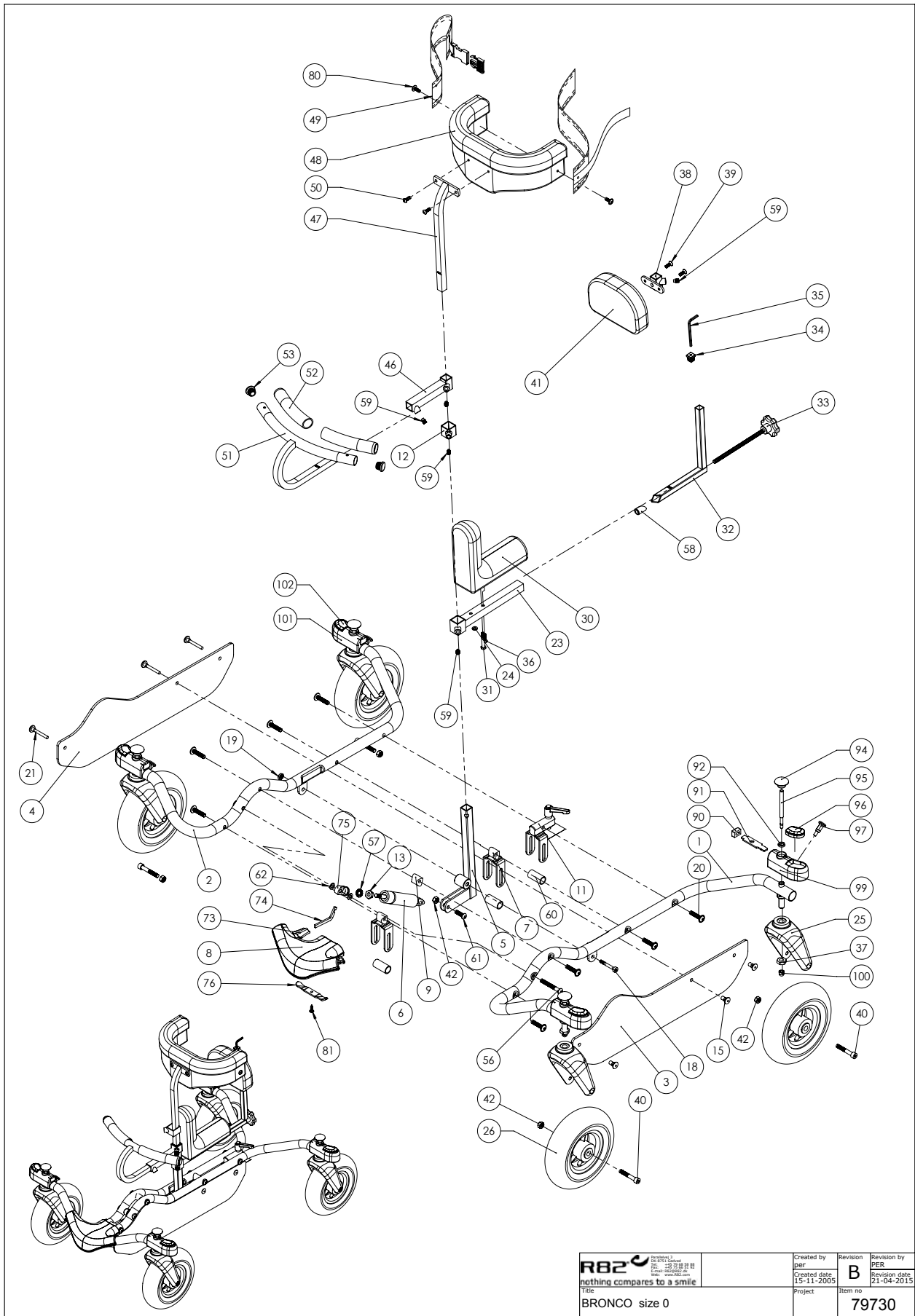

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	73002	FRAME PIPE LEFT, SIZE 0	1	PCS	5707292108364
2	73001	FRAME PIPE RIGHT SIZE 0	1	PCS	5707292108357
3	73004	LEG SEPARATOR LEFT SIZE 0	1	PCS	5707292108388
4	73003	LEG SEPARATOR RIGHT SIZE 0	1	PCS	5707292108371
5	73006	TILT ROD SIZE 0 BRONCO	1	PCS	5707292108401
6	2205250	BLOC-O-LIFT 2731HT 500 FAST	1	PCS	5707292143358
7	73032	SEPARATING BRACKET BRONCO	2	PCS	5707292108647
8	73000	BUMPER T/BRONCO	1	PCS	5707292108340
9	73030	SPACER FOR BRONCO SIZE 1	1	PCS	5707292108623
11	73008	PUSH BRACKET, BRONCO	1	PCS	5707292108425
12	81164-66	HOLDER FOR FOOT SUPPORT, IMPALA	1	PCS	5707292227218
13	1761000	COUNTER NUT 10*1,0 FINE THREAD	1	PCS	5707292141538
15	72208	COVER FOR SCREW, BLACK	3	PCS	5707292106599
18	0603545	INSEX LAV HOVED M8*45, RUSTFRI	1	PCS	5707292138248
19	1750800	LOCK NUT, STAINLESS M8	1	PCS	5707292141484
20	0663032	B.H. M/FLANGE 8*30 ZN/FE&SORT	8	PCS	5707292361288
21	72211	SLOTTED SCREW BLACK 48MM*M6	3	PCS	5707292106629
23	82210	ROD FOR SEAT PONY 0, NEW	1	PCS	5707292122568
24	1103000	FAZET WASHER 6MM	1	PCS	5707292140289
25	73051	WHEEL FORK, BRONCO O200	4	PCS	5707292381941
26	3001500	200*50 BLACK/PMEUM W.O.BR	4	PCS	5707292145628
30	6175200	SEAT PONY 0, PUR	1	PCS	5707292189363
31	0645635	BUTTON HEAD SCREW 6*35 BLACK	1	PCS	5707292139337
32	82013	ROD FOR BACKSUPPORT PONY 0, NEW	1	PCS	5707292120809
33	3502980	THREADED ROD 50MM, M8*175MM	1	PCS	5707292146977
34	2503960	FERRULE 15*15 INTERN.	1	PCS	5707292144096
35	0589400	INSEX KEYS N4	1	PCS	5707292137852
36	0622800	INSEX SCREW COUNTERS. 6*20	1	PCS	5707292138972
37	1761200	VERSUS NUT 12*1, FINE THREADED	4	PCS	5707292141545
38	82163	"BUTTOM-MOUNTING" GAZELLE	1	PCS	5707292122124
39	0621600	INSEX SCREW COUNTERS. 6*16	2	PCS	5707292138965
40	0603750	INSEX M8*75, FZB	4	PCS	5707292138309
41	6175211	BACK SUPPORT PONY 0,1&2,PUR	1	PCS	5707292189394
42	1607000	LOCK NUT 8MM, LARGE PACK	5	PCS	5707292141378
46	81855-42	EXT.PIPE FOR STEER, PONY 0	1	PCS	5707292229434
47	81874	ROD FOR CHESTSUPPORT, PONY 0	1	PCS	5707292119445
48	6175190	CHESTSUPPORT F/PONY 0,PUR	1	PCS	5707292189325
49	5540250	GIRTH FOR PONY 0	1	SET	5707292178312
50	0641600	BUTTON HEAD SCREW 6*16	2	PCS	5707292139269
51	81846	HAND BRACE, PONY 1	1	PCS	5707292119186
52	2500650	PLASTIC GRIPS BLACK 200MM.LONG	2	PCS	5707292143839
53	2503010	FERRULE D22*1,8-3,5, BLACK	2	PCS	5707292357632
56	0663060	BUTTON HEAD M8*60, ZN/FE&SORT	1	PCS	5707292139641
57	1141000	SAFETY WASHER 10MM	1	PCS	5707292140555
58	6148180	LOCK FOR BACKSUPPORT ROD	1	PCS	5707292188083
59	0808022	STAINLESS STOPSCREW M8*16	1	PCS	5707292140081
59	0808020	STAINLESS STOPSCREW, M8*12	2	PCS	5707292140074
60	73034	METAL SPACER FOR BRONCO	3	PCS	5707292108661
61	0662500	BUTTON HEAD SCREW 8*25 STAINL.	1	PCS	5707292139580

62	1103200	FAZET WASHER 8MM	2	PCS	5707292140296
73	2200030	BUTTON FOR RELEASE ARM	1	PCS	5707292143051
74	2200009	ARM FOR RELEASE BEND	1	PCS	5707292142955
75	6118820	HOUSE FOR GASSPRING, BLACK	1	PCS	5707292186720
76	73033	TIGHTENING MOUNTING FOR BUMPER	1	PCS	5707292108654
80	0659501	BUTTON HEAD M/FLANGE M6*16 FZB	2	PCS	5707292356000
81	1811110	SCREW 4.8*16 PH STAINLESS	1	PCS	5707292141705
90	73045	PLUG FOR BRAKE, BRONCO	4	PCS	5707292372994
91	73047	STRIKER PLATE	4	PCS	5707292381965
92	73048	NUT FOR WHEEL BOX 73027	4	PCS	5707292381927
93	0310100	PRESSURESPRING 25 MM F.SPINDEL	4	PCS	5707292135513
94	73044	BRAKE PLATE FOR BRONCO	4	PCS	5707292372987
95	73046	BRAKE LEVER FOR BRONCO	4	PCS	5707292381910
96	73042	RELEASE PLATE FOR BRONCO, LEFT	2	PCS	5707292372963
97	0380186	HOUSE FOR LOCK, BRONCO	4	PCS	5707292381996
98	0310122	COMPRESSION SPRING 0,9*8U	4	PCS	5707292135544
99	73041	BUMPER F/ LEFT BRAKE,BRONCO	2	PCS	5707292372956
100	1606500	LOCK NUT 6 MM	4	PCS	5707292141361
101	73040	BUMPER F/ RIGHT BRAKE,BRONCO	2	PCS	5707292372949
102	73043	RELEASE PLATE FOR BRONCO,RIGHT	2	PCS	5707292372970

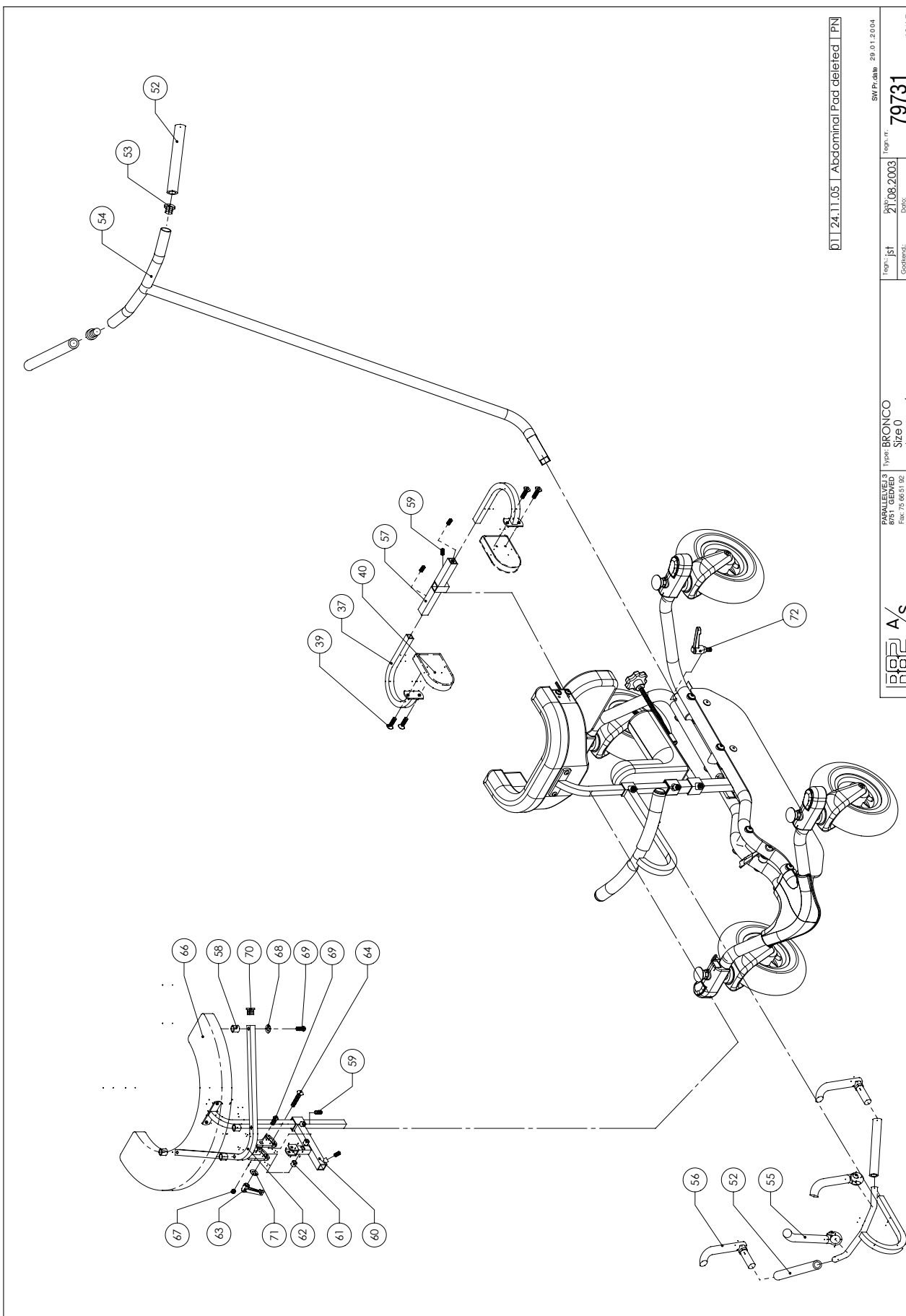

01 | 24.11.05 | Abdominal Pad deleted | PN

SW Pr.date 29.01.2004


79731

Legn.: jst	21.08.2003	Page n.:
Godkendt:	Date:	

PARALLELVEJ 3	Typ: BRONCO
8751 GEDVED	Size 0
Tel: 79 68 58 88	Accessories

R82 A/S

Position	No.	Description	Quantity	Unit of Measure	Ean code
37	81861	PELOT BRACE PONY 0+1+2+3	2	PCS	5707292119322
39	0641600	BUTTON HEAD SCREW 6*16	4	PCS	5707292139269
40	6175230	PUR SIDESTØTTE STR.0, PONY	2	PCS	5707292189417
52	2500650	PLASTIC GRIPS BLACK 200MM.LONG	2	PCS	5707292143839
53	2503010	FERRULE D22*1,8-3,5, BLACK	2	PCS	5707292357632
54	73010	SKUBBEHÅNDTAG,BRONCO	1	PCS	5707292108449
55	3503700	ALLOY BAR END, SKI TYPE	1	SET	5707292147073
56	3503600	ALLOY BAR END,INSERT TYPE	1	SET	5707292147066
57	81852-42	PELOT HOLDER PONY STR.0	1	PCS	5707292229410
58	81610	DISTANCE BUSH	4	PCS	5707292117014
59	0808020	STAINLESS STOPSCREW,M8*12	3	PCS	5707292140074
60	81839-42	YDERRØR F/STYR ARMPL.STR.0	1	PCS	5707292229366
61	81798	AFSTANDSSTYKKE,BORD PONY	2	PCS	5707292118745
62	81834	BØJLE F/ARMSTØTTE STR.0+1	1	PCS	5707292119070
63	3502300	TIGHTENING HANDLE 65MM M8 BUSH	1	PCS	5707292146885
64	1905700	BRÆDDEBOLT M8*65 FZB M/MØTRIK	1	PCS	5707292142351
66	6175235	PUR ARMSTØTTE STR PONY 0&1	1	PCS	5707292189431
67	1606500	LOCK NUT 6 MM	1	PCS	5707292141361
69	0645630	BUTTON HEAD SCREW 6*30 BLACK	5	PCS	5707292139320
70	2504000	FERRULE E15*1,0-2,0 INT.BLACK	2	PCS	5707292144102
71	1102000	PLANING WASHER 8MM	1	PCS	5707292140227
72	3502635	TIGHTENING HANDLE 6MM M8*35MM	1	PCS	5707292146939

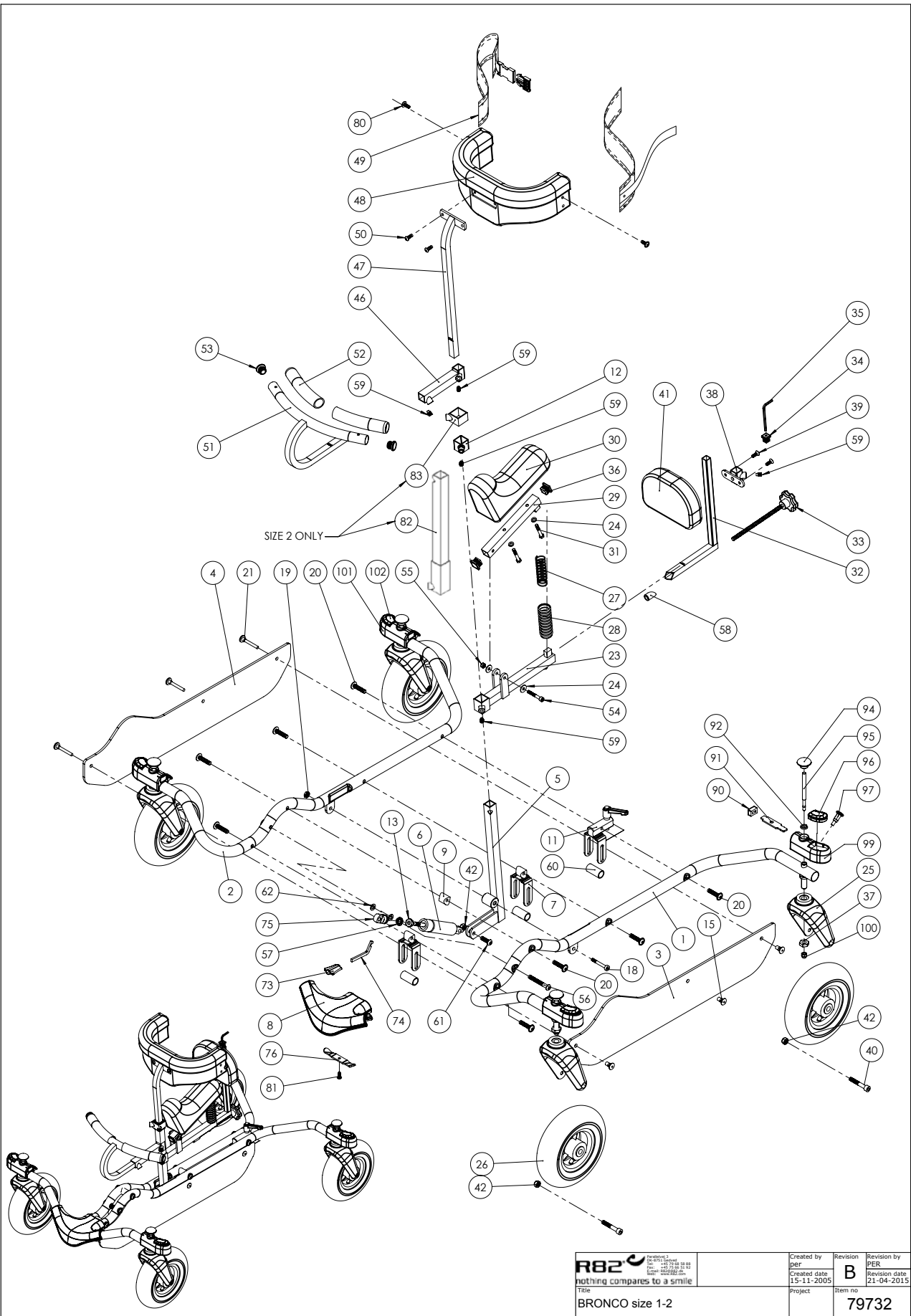


R82 nothing compares to a smile R82 A/S Parallelvej 3 8751 Gedved Denmark Tel: +45 79 68 58 88 Fax: +45 75 66 51 92 E-mail: r82@r82.dk Web: www.r82.com	Created by per	Revision B	Revision by PER
	Created date 15-11-2005	Revision date 21-04-2015	
Title BRONCO size 1-2		Project	Item no 79732

Position	No.	Description	Quantity	Unit of Measure	Ean code
1*	73012	STELRØR VENSTRE STR.1,BRONCO	1	PCS	5707292108463
	73022	STELRØR VENSTRE STR.2,BRONCO	1	PCS	5707292108555
2*	73011	STELRØR HØJRE STR.1,BRONCO	1	PCS	5707292108456
	73021	STELRØR HØJRE STR.2,BRONCO	1	PCS	5707292108548
3*	73014	BENSKILLER VENSTRE STR.1,BRONC	1	PCS	5707292108487
	73024	LEG-SEPARATOR LEFT SIDE SIZE 2	1	PCS	5707292108579
4*	73013	BENSKILLER HØJRE STR.1,BRONCO	1	PCS	5707292108470
	73023	LEG-SEPARATOR RIGHTSIDE SIZE 2	1	PCS	5707292108562
5*	73016	KIPPIND STR.1 BRONCO	1	PCS	5707292108500
	73026	KIPPIND STR.2 BRONCO	1	PCS	5707292108593
6	2205250	BLOC-O-LIFT 2731HT 500 FAST	1	PCS	5707292143358
7	73032	SEPARATING BRACKET BRONCO	2	PCS	5707292108647
8	73000	BUMPER T/BRONCO	1	PCS	5707292108340
9	73030	SPACER FOR BRONCO SIZE 1	1	PCS	5707292108623
11	73008	PUSH BRACKET, BRONCO	1	PCS	5707292108425
12	81164-66	HOLDER FOR FOOTSUPPORT,IMPALA	1	PCS	5707292227218
13	1761000	COUNTER NUT 10*1,0 FINE THREAD	1	PCS	5707292141538
15	72208	COVER FOR SCREW, BLACK	3	PCS	5707292106599
18	0603545	INSEX LAV HOVED M8*45, RUSTFRI	1	PCS	5707292138248
19	1750800	LOCK NUT, STAINLESS M8	1	PCS	5707292141484
20	0663032	B.H. M/FLANGE 8*30 ZN/FE&SORT	8	PCS	5707292361288
21	72211	SLOTTED SCREW BLACK 48MM*M6	3	PCS	5707292106629
23*	82475	HOLDER FOR BACKSUPPORT SIZE 2	1	PCS	5707292353429
	82473	ROD FOR BACK SUPPORT SIZE 1	1	PCS	5707292353382
24	1103000	FAZET WASHER 6MM	2	PCS	5707292140289
25	73051	WHEEL FORK, BRONCO O200	4	PCS	5707292381941
26	3001500	200*50 BLACK/PMEUM W.O.BR	4	PCS	5707292145628
27*	0302000	SPRING 2,25*62 MM	1	PCS	5707292135445
	0310120	COMPRESSION SPRING FOR PONY 1	1	PCS	5707292135520
28	2500700	CUFF, PLASTIC 100MM, FOR SPRING	1	PCS	5707292143853
29*	82476	ROD FOR SEAT SIZE 2	1	PCS	5707292353405
	82476-44	BAR FOR BACKSUPPORT 1	1	PCS	5707292353412
30	6175201	SEAT PONY 1,PUR	1	PCS	5707292189370
31	0645635	BUTTON HEAD SCREW 6*35 BLACK	2	PCS	5707292139337
32	81996	ROD FOR BACKSUP.PONY 1+2	1	PCS	5707292120632
33	3502980	THREADED ROD 50MM,M8*175MM	1	PCS	5707292146977
34	2503960	FERRULE 15*15 INTERN.	1	PCS	5707292144096
35	0589400	INSEX KEYS N4	1	PCS	5707292137852
36	2504500	FERRULE E20*1,0-2,0 INT.BLACK	2	PCS	5707292144126
37	1761200	VERSUS NUT 12*1, FINE THREADED	4	PCS	5707292141545
38	82163	“BUTTOM-MOUNTING” GAZELLE	1	PCS	5707292122124
39	0621600	INSEX SCREW COUNTERS. 6*16	2	PCS	5707292138965
40	0603750	INSEX M8*75, FZB	4	PCS	5707292138309
41	6175211	BACK SUPPORT PONY 0,1&2,PUR	1	PCS	5707292189394
42	1607000	LOCK NUT 8MM, LARGE PACK	5	PCS	5707292141378
46*	81855-44	EXT.PIPE FOR STEER, PONY 1	1	PCS	5707292229441
	81825	EXT.PIPE F/STEER-HOLD, PONY 2	1	PCS	5707292118998
47*	81847	ROD FOR CHEST-SUPPORT, PONY	1	PCS	5707292119193
	81827	CHEST-SUPPORT ROD, PONY 2	1	PCS	5707292119018

48*	6175191	PUR CHESTSUPPORT SZ.1, PONY	1	PCS	5707292189332
	6175192	PUR CHESTSUPPORT SZ. 2	1	PCS	5707292189349
49*	5540300	GIRTH FOR PONY 1	1	SET	5707292178329
	5540350	GIRTH FOR PONY 2	1	SET	5707292178336
50	0641600	BUTTON HEAD SCREW 6*16	2	PCS	5707292139269
51*	81846	HAND BRACE, PONY 1	1	PCS	5707292119186
	81826	HAND BRACE, PONY 2	1	PCS	5707292119001
52	2500650	PLASTIC GRIPS BLACK 200MM.LONG	2	PCS	5707292143839
53	2503010	FERRULE D22*1,8-3,5, BLACK	2	PCS	5707292357632
54	1901800	STEEL SET SCREW 6*40MM	1	PCS	5707292141972
55	1606500	LOCK NUT 6 MM	1	PCS	5707292141361
56	0663060	BUTTON HEAD M8*60, ZN/FE&SORT	1	PCS	5707292139641
57	1141000	SAFETY WASHER 10MM	1	PCS	5707292140555
58	6148180	LOCK FOR BACKSUPPORT ROD	1	PCS	5707292188083
59*	0808020	STAINLESS STOPSCREW,M8*12	2	PCS	5707292140074
	0808022	STAINLESS STOPSCREW M8*16	1	PCS	5707292140081
60	73034	METAL SPACER FOR BRONCO	3	PCS	5707292108661
61	0662500	BUTTON HEAD SCREW 8*25 STAINL.	1	PCS	5707292139580
62	1103200	FAZET WASHER 8MM	2	PCS	5707292140296
73	2200030	BUTTON FOR RELEASE ARM	1	PCS	5707292143051
74	2200009	ARM FOR RELEASE BEND	1	PCS	5707292142955
75	6118820	HOUSE FOR GASSPRING, BLACK	1	PCS	5707292186720
76	73033	TIGHTENING MOUNTING FOR BUMPER	1	PCS	5707292108654
80	0659501	BUTTON HEAD M/FLANGE M6*16 FZB	4	PCS	5707292356000
81	1811110	SCREW 4.8*16 PH STAINLESS	1	PCS	5707292141705
82	73020	HOLDER	1	PCS	5707292108531
83	73037	EXTENSION ROD, WELDED	1	PCS	5707292108692
90	73045	PLUG FOR BRAKE, BRONCO	4	PCS	5707292372994
91	73047	STRIKER PLATE	4	PCS	5707292381965
92	73048	NUT FOR WHEEL BOX 73027	4	PCS	5707292381927
93	0310100	PRESSURESPRING 25 MM F.SPINDEL	4	PCS	5707292135513
94	73044	BRAKE PLATE FOR BRONCO	4	PCS	5707292372987
95	73046	BRAKE LEVER FOR BRONCO	4	PCS	5707292381910
96	73042	RELEASE PLATE FOR BRONCO, LEFT	2	PCS	5707292372963
97	0380186	HOUSE FOR LOCK, BRONCO	4	PCS	5707292381996
98	0310122	COMPRESSION SPRING 0,9*8U	4	PCS	5707292135544
99	73041	BUMPER F/ LEFT BRAKE,BRONCO	2	PCS	5707292372956
100	1606500	LOCK NUT 6 MM	4	PCS	5707292141361
101	73040	BUMPER F/ RIGHT BRAKE,BRONCO	2	PCS	5707292372949
102	73043	RELEASE PLATE FOR BRONCO,RIGHT	2	PCS	5707292372970

*		CHOOSE SIZE
---	--	-------------

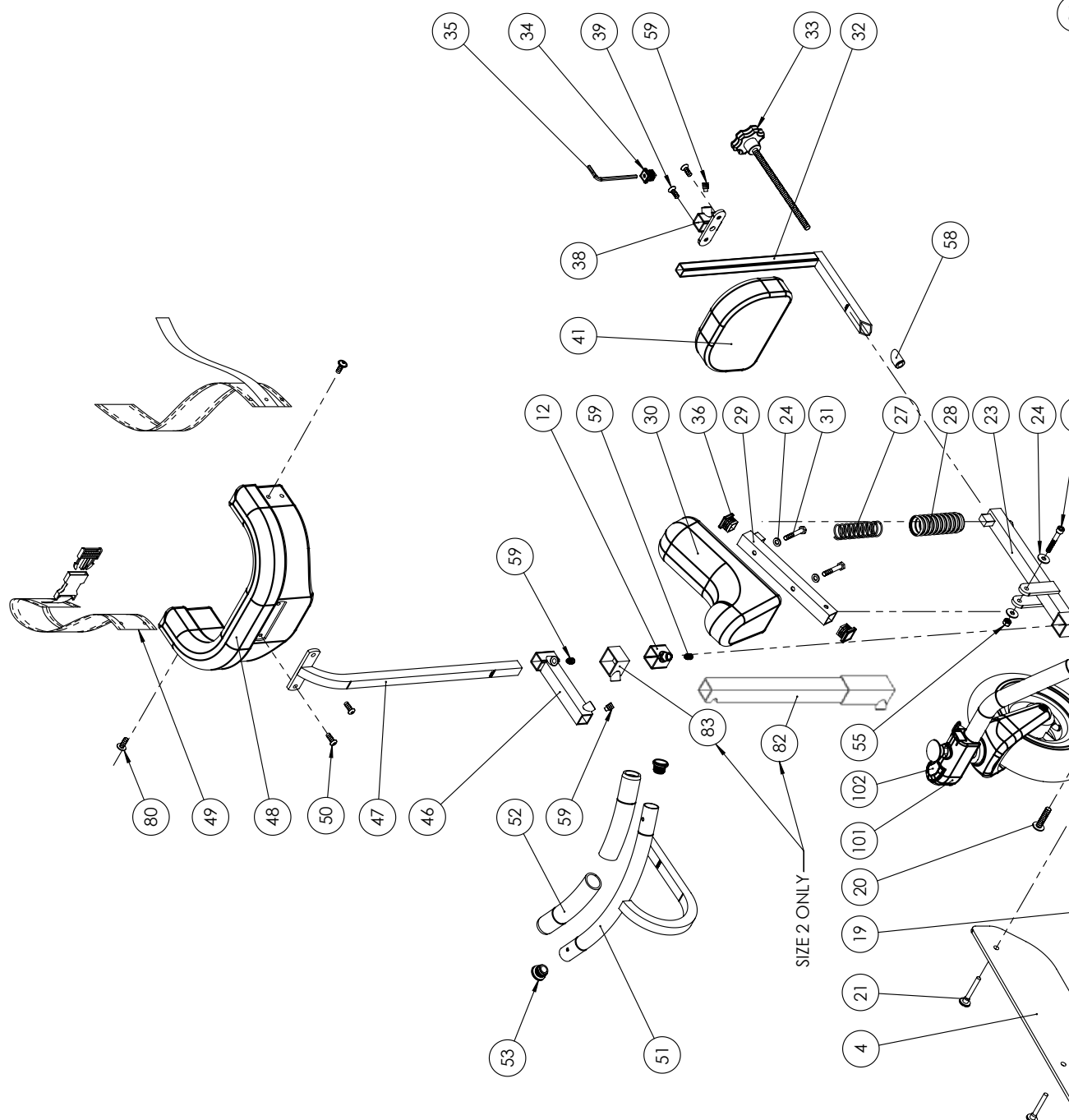

Position	No.	Description	Quantity	Unit of Measure	Ean code
37	81861	PELOT BRACE PONY 0+1+2+3	2	PCS	5707292119322
39	0641600	BUTTON HEAD SCREW 6*16	4	PCS	5707292139269
40	6175231	CUSH. SIDESUPP. SIZE 1+2	2	PCS	5707292189424
42*	81833	MOUNTING F/ABDOMENSUPPORT,PONY	1	PCS	5707292119063
	81832	HOLDER F/ABDOMENSUPPORT,PONY 2	1	PCS	5707292119056
43	2501600	FERRULE D15*1,0 BLACK	2	PCS	5707292143969
44	6175221	ABD. CUSHION SIZE 1+2	1	PCS	5707292189400
45*	81838-44	HOLDER FOR ABDOMINAL CUSHION	1	PCS	5707292229359
	81833	MOUNTING F/ABDOMENSUPPORT,PONY	1	PCS	5707292119063
52	2500650	PLASTIC GRIPS BLACK 200MM.LONG	2	PCS	5707292143839
53	2503010	FERRULE D22*1,8-3,5, BLACK	2	PCS	5707292357632
54	73010	PUSHHANDLE BRONCO	1	PCS	5707292108449
55	3503700	ALLOY BAR END, SKI TYPE	1	SET	5707292147073
56	3503600	ALLOY BAR END,INSERT TYPE	1	SET	5707292147066
57*	81852-44	HOLDER FOR PELOT, PONY 1	1	PCS	5707292229427
	81822	PELOT HOLDER, PONY 2	1	PCS	5707292118967
58	81610	DISTANCE BUSH	4	PCS	5707292117014
59	0808020	STAINLESS STOPSCREW,M8*12	3	PCS	5707292140074
60*	81839-44	TUPE FOR HANDBRACE PONY 1	1	PCS	5707292229373
	81840	EXTERNAL PIPE FOR HANDLE SIZE2	1	PCS	5707292119124
61	81798	DISTANCE PIECE FOR ARMREST	2	PCS	5707292118745
62*	81834	BRACE FOR ARMREST SIZE 0+1	1	PCS	5707292119070
	81835	BRACE FOR ARMREST SIZE 2	1	PCS	5707292119087
63	3502300	TIGHTENING HANDLE 65MM M8 BUSH	1	PCS	5707292146885
64	1905700	BOLT M8*65 WITH NUT	1	PCS	5707292142351
66*	6175235	CUSHION ARMREST SIZE 0+1	1	PCS	5707292189431
	6175236	CUSHION ARMREST SIZE 2	1	PCS	5707292189448
67	1606500	LOCK NUT 6 MM	1	PCS	5707292141361
69	0645630	BUTTON HEAD SCREW 6*30 BLACK	5	PCS	5707292139320
70	2504000	FERRULE E15*1,0-2,0 INT.BLACK	2	PCS	5707292144102
71	1102000	PLANING WASHER 8MM	1	PCS	5707292140227
72	3502635	TIGHTENING HANDLE 6MM M8*35MM	1	PCS	5707292146939

*		CHOOSE SIZE
---	--	-------------