

combi frame

Exploded views / Part lists

Rev. date (05.2009)




Contents

Tilt, wheels and brakes	3
Frame.....	6
Seat.....	9
Foot supports, backs, brakes, adaptors and mudguards.....	13

Position	No.	Description	Quantity	Unit of Measure	Ean code
65	1631060	STAINLESS LOCK NUT 6MM	2	PCS	5707292141446
93	1607001	SELF LOCKING NUT 8MM	2	PCS	5707292401281
207	1105080	STAINLESS WASHER 8MM	4	PCS	5707292140364
301	74731	COVER INNER, BRAKE,	1	PCS	5707292412409
302	74730	COVER OUTER, BRAKE, COUGAR	1	PCS	5707292412416
303	6109200	M6 THREAD BUSH, BRAKING SYSTEM	2	PCS	5707292185952
304	74013	BRAKING ARM	1	PCS	5707292378033
305	74644	BRAKE BRACKET	1	PCS	5707292407955
306	0641100	B.H. W/FLANGE M6*10	2	PCS	5707292139245
307	0503175	BLACK CASE, Ø6,35*17,5MM	2	PCS	5707292137258
308	74764	BRAKING STICK	1	PCS	5707292435958
309	72338	MOUNTING 12X4X34,5	2	PCS	5707292107794
310	74138	SPRING FOR BRAKE, RIGHT	1	PCS	5707292377500
311	74139	SPRING FOR BRAKE, LEFT	1	PCS	5707292377517
312	0659500	BUTTON HEAD SCREW	3	PCS	5707292139450
313	0659550	BUTTON HEAD M6*20, STAINLESS	2	PCS	5707292139467
320*	2965750	FRONT FORK WITH BOLT, 45MM	2	PCS	5707292415219
	2965760	FRONT FORK WITH BOLT, 60MM	2	PCS	5707292415226
321*	0602493	INSEX M8*65, STAINLESS	2	PCS	5707292423757
	0602495	INSEX M8*80, RUSTFRI	2	PCS	5707292423986
322*	3007865	150*30 WHEEL, AIR	2	PCS	5707292378477
	3001500	200*50 BLACK/PMEUM W.O.BR	2	PCS	5707292145628
	3007870	150*30 WHEEL, FLEXEL	2	PCS	5707292378484
	3001600	200*50 BLACK/FLEXEL WHEEL	2	PCS	5707292425058
323	0310105	PRESSURESPRING 1,5*16,2	1	PCS	5707292465191
324	0683950	TIGHT-WASHER 16*22*0,5	2	PCS	5707292139733
330	2985025	BUSH WITH WASHER AND NUT	2	PCS	5707292416414
331	2985068	COMPL. STAINLESS Q-RELEASE	2	PCS	5707292416407
332*	3302400	20" WHEEL, RADIAL, AIR	2	PCS	5707292414779
	3302410	22" WHEEL, RADIAL, AIR	2	PCS	5707292414793
	3302420	24" WHEEL, RADIAL, AIR	2	PCS	5707292414809
	3302200	20" WHEEL, RADIAL, FLEXEL	2	PCS	5707292414717
	3302210	22" WHEEL, RADIAL, FLEXEL	2	PCS	5707292414724
	3302220	24" WHEEL, RADIAL, FLEXEL	2	PCS	5707292414731
333*	3302500	20" WHEEL WITH DRUMBR, AIR	2	PCS	5707292414816
	3302510	22" WHEEL WITH DRUMBR, AIR	2	PCS	5707292414823
	3302520	24" WHEEL WITH DRUMBR, AIR	2	PCS	5707292414830
	3302300	20" WHEEL WITH DRUMBR., FLEXEL	2	PCS	5707292414748
	3302310	22" WHEEL WITH DRUMBR., FLEXEL	2	PCS	5707292414755
	3302320	24" WHEEL WITH DRUMBR., FLEXEL	2	PCS	5707292414762
344	2983051	ANCHOR PLATE, LEFT, COUGAR	1	PCS	5707292483867
345	2983052	ANCHOR PLATE, RIGHT, COUGAR	1	PCS	5707292483874
346	0602425	INSEX M6*25, STAINLESS	2	PCS	5707292404596
347	0380100	CLIP FOR CABLE	2	PCS	5707292136015
348	1101950	Washer ø6,1-16*1 MM, FZB	2	PCS	5707292140210
351	3503535	BRAKE PARKING LEVER WO/CLAMP	2	PCS	5707292413536
353	5002751	CABLE FOR FRAKE, SIZE 3+4	1	PCS	5707292413642
355	5002750	CABLE FOR BRAKE, SIZE 1+2	1	PCS	5707292413635
356	0371853	STRAINED WIRES 1,5MM 19T.	3	PCS	5707292135889

357	5002756	CABLE FOR RECLINE, OUTER 1-4	1	PCS	5707292413673
360	0388006	SCREWNIPPLE 5*7,5MM	3	PCS	5707292136114
370	3002500	12½*2 1/4 BLACK WHEEL,BR/LEFT	1	PCS	5707292145642
371	3003000	12½*2 1/4" BLACK WHEEL,BR/RIGH	1	PCS	5707292145659
372	1902400	STEEL SET SCREW 12*110MM	2	PCS	5707292142061
373	0602425	INSEX M6*25, STAINLESS	2	PCS	5707292404596
374	5002751	CABLE FOR FRAKE, SIZE 3+4	2	PCS	5707292413642
375	0380000	CABLE TIGHTER M5*23	2	PCS	5707292135933
376	1604500	STEEL NUT 12MM, LARGE PACK	2	PCS	5707292141309
377	1608000	LOCK NUT 12MM, LARGE PACK	2	PCS	5707292141392
379	0380060	CABLING, FLEXIBLE	3	PCS	5707292425195
381	2704300	BLACK HUB CAP FOR ROLKO WHEEL	2	PCS	5707292144515
350+363	3503532	GAS PUMP LEVER WITHOUT CLAMP	1	PCS	5707292427755
350+364	3503531	GAS PUMP LEVER WITH CLAMP	1	PCS	5707292413529

*		CHOOSE SIZE			
---	--	-------------	--	--	--

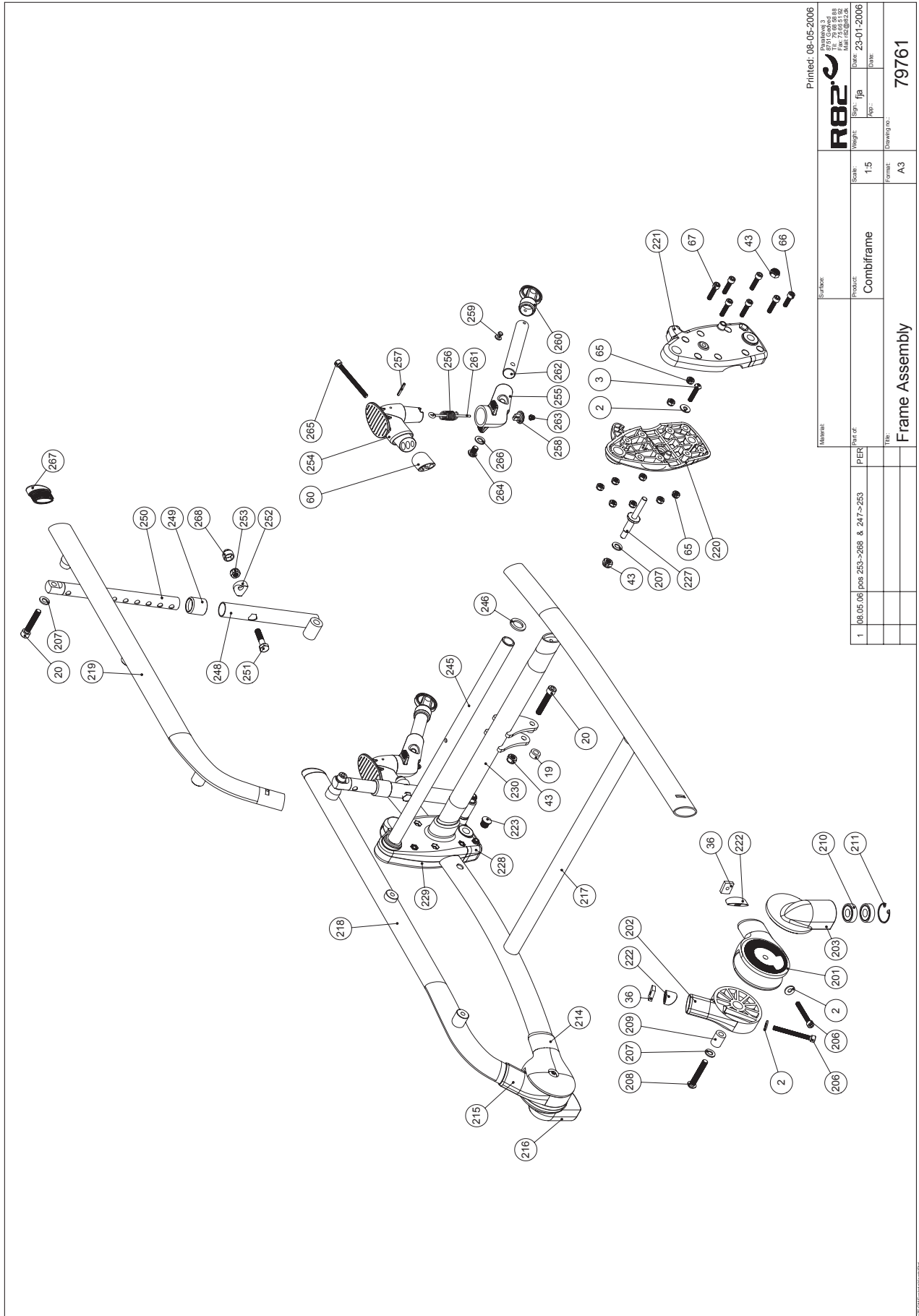


Printed: 08-05-2006

R82		Scale: 1:5	Product: Combiframe
Weight:	Sign: jfb	App:	Date: 23-01-2006
Drawing no.: 79761		Formet: A3	
Title: Frame Assembly		Part of: PEF	
1 08.05.06 pos 263-268 & 247-253			

Position	No.	Description	Quantity	Unit of Measure	Ean code
2	1106025	WASHER M6 O16/O6,4X1,6	6	PCS	5707292404664
3	0659580	BUTTON HEAD SCREW 6*30 MM	2	PCS	5707292139481
19	1350010	DISTANCEPART BLACK 8*15*8,2MM	2	PCS	5707292140883
20	0663046	INSEX M8*45, FULL THREAD	4	PCS	5707292432902
36	74090	MOUNTING FOR FRONTFORK	6	PCS	5707292374066
43	1631080	STAINLESS LOCK NUT 8MM	5	PCS	5707292141453
60	74662	LOCK, MOVING PART	2	PCS	5707292401380
65	1631060	STAINLESS LOCK NUT 6MM	18	PCS	5707292141446
66	0609827	STAINLESS INSEX M6*20	2	PCS	5707292375049
67	0602425	INSEX M6*25, STAINLESS	12	PCS	5707292404596
201	74703	FOR WHEEL LOWER, LEFT	1	PCS	5707292411631
202	74705	HOLDER FRONT WHEEL UPPER,LEFT	1	PCS	5707292411648
203	74701	FOR WHEEL QR, LEFT	1	PCS	5707292411655
206	0602460	INSEX M6*60, STAINLESS	4	PCS	5707292400727
207	1105080	STAINLESS WASHER 8MM	6	PCS	5707292140364
208	0663050	BUTTON HEAD M8*50, RUSTFRI	2	PCS	5707292400697
209	74654	BUSH Ø16/Ø8, 1*21, COUGAR	2	PCS	5707292411662
210	2985210	BALL BEARING	2	PCS	5707292379412
211	1100220	LOCKING RING F 28MM BORE	1	PCS	5707292392671
214	74702	FOR WHEEL LOWER, RIGHT	1	PCS	5707292411686
215	74704	HOLDER FRONT WHEEL UPPER,RIGHT	1	PCS	5707292411693
216	74700	FOR WHEEL QR, RIGHT	1	PCS	5707292411709
217*	74690-54	LOWER FRAME WELDED, SIZE 1	1	PCS	5707292413031
	74691-54	LOWER FRAME WELDED, SIZE 2	1	PCS	5707292413000
	74692-54	LOWER FRAME WELDED, SIZE 3	1	PCS	5707292413048
	74693-54	LOWER FRAME, WELDED, SIZE 4	1	PCS	5707292413055
218	73701-54	UPPER FRAME, RIGHT, SILVER	1	PCS	5707292431295
219	73702-54	UPPER FRAME, LEFT, SILVER	1	PCS	5707292431325
220	74708	WHEEL SUSPENSION INNER LEFT	1	PCS	5707292411723
221	74710	WHEEL SUSPENSION OUTHER LEFT	1	PCS	5707292411730
222	74683	LOCK, MOVING PART, SHORT	4	PCS	5707292425904
223	2501500	FERRULE D12*0,5-1,5 INT.BLACK	2	PCS	5707292143952
227	74667	PINBOLT FOR ACTUATOR, COUGAR	2	PCS	5707292411747
228	74709	WHEEL SUSPENSION INNER RIGHT	1	PCS	5707292411754
229	74711	WHEEL SUSPENSION OUTHER RIGHT	1	PCS	5707292411761
230*	73726-54	CROSS PIPE SIZE 1, SILVER	1	PCS	5707292431356
	73727-54	CROSS PIPE SIZE 2, SILVER	1	PCS	5707292431387
	73728-54	CROSS PIPE SIZE 3, SILVER	1	PCS	5707292431417
	73729-54	CROSS PIPE SIZE 4, SILVER	1	PCS	5707292431448
245*	74727	Ø20 CROSS TUBE, SIZE 1,	1	PCS	5707292412300
	74609	Ø20 CROSS TUBE, SIZE 2	1	PCS	5707292412195
	74616	Ø20 CROSS TUBE, SIZE 3	1	PCS	5707292412218
	74623	Ø20 CROSS TUBE, SIZE 4	1	PCS	5707292412232
246	1102600	WASHER 16*22*2MM	2	PCS	5707292400680
248	74839	ELEVATION TUBE	1	PCS	5707292411983
249	72215	FERRULE FOR PANTHER	1	PCS	5707292106667
250	74837	INNER TUBE FOR ELEVATION	1	PCS	5707292411990
251	0602489	HEXAGON SCREW M8*35	1	PCS	5707292412447
252	74641	SADDLE-WASHER, ELEVATION TUBE	1	PCS	5707292405654

253	1631080	STAINLESS LOCK NUT 8MM	1	PCS	5707292141453
254	74770	UPPER PART, ANTITIP	2	PCS	5707292411617
255	74771	LOWER PART ANTITIP	2	PCS	5707292411624
256	0310520	TENSION SPRING 1,5X12X55,5IN	2	PCS	5707292406361
257	0532210	SPAN PIN 1481 3*22 STAINLESS	2	PCS	5707292423313
258	6109361	SPRING HOLDER F/ANTI TIP	2	PCS	5707292415479
259	0506519	STAINLESS BLIND RIVET 4*12	1	PCS	5707292137388
260	72298	WHEELHOLDER FOR ANTITIP DEVICE	1	PCS	5707292399564
261	74642	LOCKING EYE BOLT, ANTTIP,COUGA	2	PCS	5707292415752
262*	74677	EXTENSION BAR F. ANTITIP, LONG	1	PCS	5707292415844
	74676	EXTENSION BAR F. ANTITIP, SHORT	1	PCS	5707292415837
263	1781030	STAINLESS TOP NUT M3	2	PCS	5707292415769
264	0662000	BUTTON HEAD SCREW 8*20 STAINL.	2	PCS	5707292139573
265	0602196	INSEX M6*75, STAINLESS	2	PCS	5707292423269
266	1202010	NORD-LOCK SKIVE Ø 8MM.	1	SET	5707292419712
267	74767	PLUG FOR ELLIPSE 20 * 40	2	PCS	5707292412669
268	2501406	NUT PROTECTION, BLACK, F/M8	1	PCS	5707292375919

*		CHOOSE SIZE
---	--	-------------

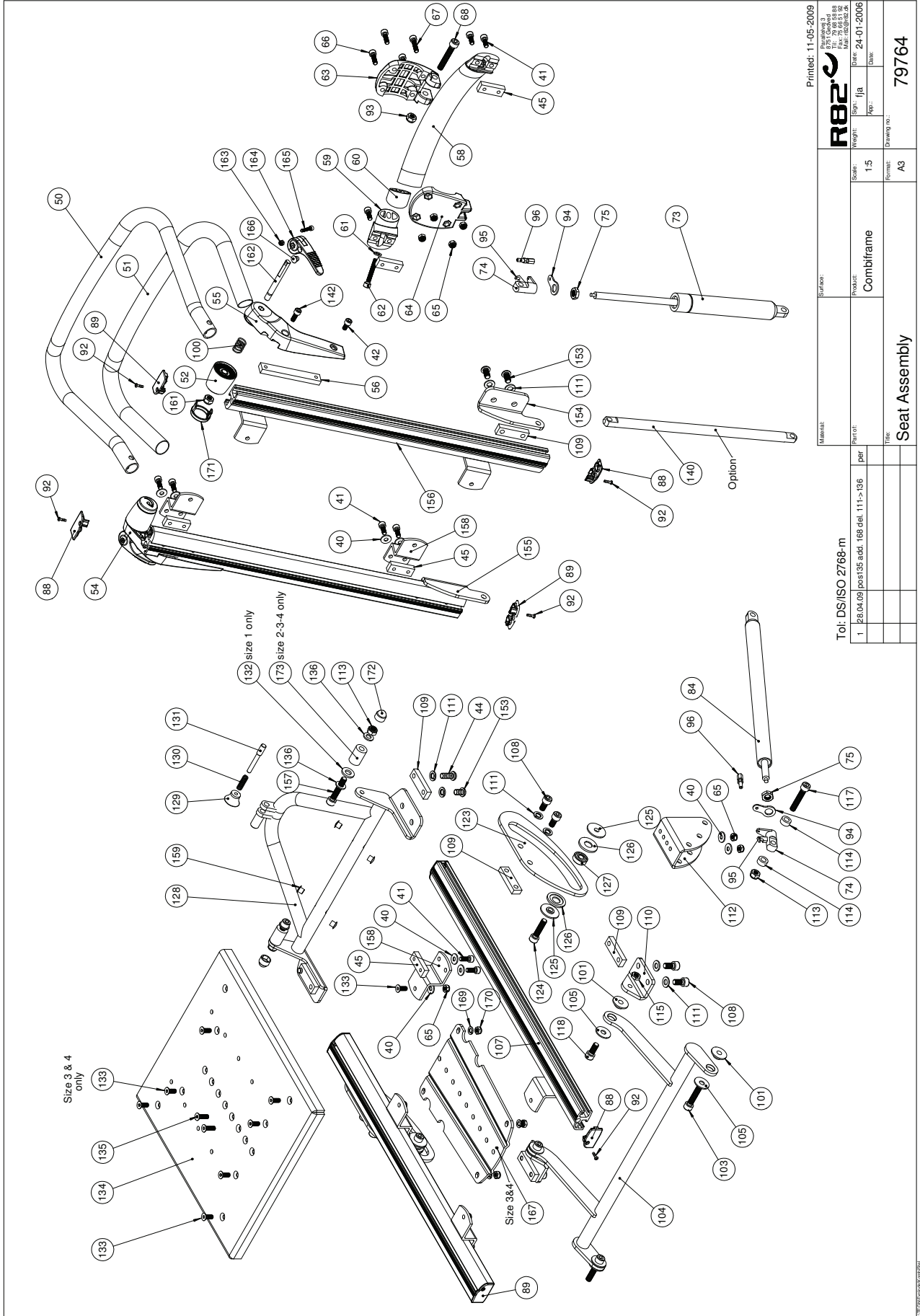

Printed: 11-05-2009

Material:	Supplier:	Part of:	Part of:
		1 88.04.09 pos: 135, 066, 168, 068, 111-136	part
Weight:	Sign:	Drawn:	Date:
1,5	fjb	24-01-2006	
Form:	Form:	Form:	Form:
A3	A3	A3	A3
Seat Assembly		79764	


Tol. DS/ISO 2768-m

Position	No.	Description	Quantity	Unit of Measure	Ean code
40	1100490	SPRING WASHER M6 STAINLESS	22	PCS	5707292411297
41	0602205	STAINLESS INSEX M6*16	20	PCS	5707292404732
42	0602195	INSEX M6*12 STAINLESS	2	PCS	5707292404701
44	0662000	BUTTON HEAD SCREW 8*20 STAINLESS	2	PCS	5707292139573
45	80609	SLIDEBAR 40*15*8, M6*2	10	PCS	5707292405418
50*	74833	HAND-BRACE SIZE 1, COUGAR	1	PCS	5707292411501
	74824	HAND-BRACE SIZE 2	1	PCS	5707292411471
	74825	HAND-BRACE SIZE 3	1	PCS	5707292410771
	74828	HAND-BRACE SIZE 4	1	PCS	5707292411495
51	2500520	TPE COVER O20MM	2	PCS	5707292401649
52	74762	TOOTHED SUPPORT, HANDEL	2	PCS	5707292410788
54	74802	TOOTED HOLDER, RIGHT	1	PCS	5707292412348
55	74803	TOOTED HOLDER, LEFT	1	PCS	5707292449849
56	80613	SLIDEBAR 100*15*8, M6*2	2	PCS	5707292410818
58*	73757	CROSS TUBE, SIZE 0	1	PCS	5707292430786
	74679	CROSS TUBE,BACK, SIZE 1 260mm	1	PCS	5707292415486
	74605	CROSS TUBE BACK, SIZE 2 310 mm	1	PCS	5707292403070
	74612	CROSS TUBE,BACK SIZE 3	1	PCS	5707292403124
59	74531	PLUG FOR CROSSTUBE	2	PCS	5707292410825
60	74662	LOCK, MOVING PART	2	PCS	5707292401380
61	1105060	STAINLESS WASHER 6MM	2	PCS	5707292140357
62	0602450	INSEX M6*40 STAINLESS	2	PCS	5707292400673
63	74738	HOLDER FRONT, BACK, COUGAR	1	PCS	5707292410832
64	74739	COVER BASK FOR HOLDER, COUGAR	1	PCS	5707292410849
65	1631060	STAINLESS LOCK NUT 6MM	10	PCS	5707292141446
66	0659550	BUTTON HEAD M6*20, STAINLESS	2	PCS	5707292139467
67	0659570	BUTTON HEAD SCREW 6*25 STAINL.	2	PCS	5707292139474
68	1489771	INSEX M8*50 STAINLESS	1	PCS	5707292412850
73	2206010	GAS SPRING	1	PCS	5707292436474
74	2200010	RELEASE HOUSE	1	PCS	5707292142962
75	1761000	COUNTER NUT 10*1,0 FINE THREAD	1	PCS	5707292141538
84	2200400	GAS SPRING, NO PRESSURE	1	PCS	5707292432629
88	74765	PLUG FOR SIDE PART, LEFT	6	PCS	5707292410931
89	74766	PLUG FOR SIDE PART, RIGHT	6	PCS	5707292410948
92	1810050	PLATE SCREW 7981C ST 2,9*13	12	PCS	5707292404671
93	1607002	LOCK NUT 8MM, A2	1	PCS	5707292410955
94	82268	WIRE MOUNTING BY PEDAL	1	PCS	5707292123039
95	2200050	SMALL RELEASE ARM	1	PCS	5707292143075
96	0380000	CABLE TIGHTER M5*23	1	PCS	5707292135933
100	0310152	STAINLESS SPRING. 1,2*12*30M	2	PCS	5707292388872
101	73733	BUSH	4	PCS	5707292431011
103	0602485	INSEX M8*25,LOW HEAD,STAINLESS	2	PCS	5707292432216
104*	73771-54	TILT ARM SIZE 1, SILVER	1	PCS	5707292432384
	73772-54	TILT ARM SIZE 2, SILVER	1	PCS	5707292432377
	73773-54	TILT ARM SIZE 3, SILVER	1	PCS	5707292432360
	73774-54	TILT ARM SIZE 4, SILVER	1	PCS	5707292432353

105	1108000	WASHER 8MM	4	PCS	5707292140388
107*	73736	SEAT PROFILE SIZE 1	2	PCS	5707292430939
	73737	SEAT PROFILE SIZE 2	2	PCS	5707292430922
	73738	SEAT PROFILE SIZE 3	2	PCS	5707292430908
	73739	SEAT PROFILE SIZE 4	2	PCS	5707292430885
108	0609816	STAINLESS INSEX SCREW 8*16	4	PCS	5707292138569
109	80617	SLIDEBAR 50*15*8,M8*2	6	PCS	5707292405401
110	73745	BRACKET FOR SEAT	2	PCS	5707292430847
111	1101000	CENTER SPRING WASHER 8MM	12	PCS	5707292140173
112	73732	BRACKET F. SEAT	1	PCS	5707292430946
113	1631080	STAINLESS LOCK NUT 8MM	3	PCS	5707292141453
114	1310719	DISTANCE PIECE, 18*8,2*7,5	2	PCS	5707292140845
115	1750800	LOCK NUT, STAINLESS M8	2	PCS	5707292141484
117	0663046	INSEX M8*45, FULL THREAD	1	PCS	5707292432902
118	0609820	STAINLESS INSEX SCREW 8*20	2	PCS	5707292138576
123	73852	SLIDE FITTING	2	PCS	5707292474148
124	0609835	STAINLESS INSEX SCREW 8*35	2	PCS	5707292138606
125	73751	DISK	4	PCS	5707292430823
126	73752	DISK	4	PCS	5707292430816
127	2985211	BALL BEARING 608-2RSH	2	PCS	5707292432124
128*	73781	SEAT TIP SIZE 1	1	PCS	5707292430601
	73782	SEAT TIP SIZE 2	1	PCS	5707292430595
	73783	SEAT TIP SIZE 3	1	PCS	5707292430588
	73784	SEAT TIP SIZE 4	1	PCS	5707292430571
129	3706256	BUTTON Ø25, M6 THREAD	1	PCS	5707292147219
130	0310100	PRESSURESPRING 25 MM F.SPINDEL	1	PCS	5707292135513
131	6109348	PIN FOR GAS SPRING BACK	1	PCS	5707292186072
132	1310610	NYLON WASHER DIN 125A, M8	2	PCS	5707292374684
133	0609728	INSEX, STAINLESS 6*16 UH	6	PCS	5707292138491
134*	73761	SEATPLATE SIZE 1	1	PCS	5707292430762
	73762	SEATPLATE SIZE 2	1	PCS	5707292430755
	73763	SEATPLATE SIZE 3	1	PCS	5707292430656
	73764	SEATPLATE SIZE 4	1	PCS	5707292430649
135	0609728	INSEX, STAINLESS 6*16 UH	2	PCS	5707292138491
136	1105080	STAINLESS WASHER 8MM	2	PCS	5707292140364
140	74664	RECLAIN AD ON	1	PCS	5707292419767
142	0602207	INSEX M6*18, STAINLESS	2	PCS	5707292430380
153	0661500	BUTTON HEAD SCREW 8*16 STAINL.	4	PCS	5707292139559
154	73755	BACKBRACKET, LEFT	1	PCS	5707292430809
155	73756	BACKBRACKET, RIGHT	1	PCS	5707292430793
156*	73791	BACK PROFILE SIZE 1	2	PCS	5707292430533
	73792	BACK PROFILE SIZE 2	2	PCS	5707292430540
	73793	BACK PROFILE SIZE 3	2	PCS	5707292430557
	73794	BACK PROFILE SIZE 4	2	PCS	5707292430564
157	0602485	INSEX M8*25, LOW HEAD, STAINLESS	2	PCS	5707292432216
158	73724	BRACKET FOR SEATPLATE	8	PCS	5707292431004
159	1310529	COVERING STUD, BLACK 6,5MM	4	PCS	5707292445445
161	1631080	STAINLESS LOCK NUT 8MM	2	PCS	5707292141453
162	74768	LOCK SHAFT, HANDBRACE	2	PCS	5707292438669
163	1606275	LOCK NUT 4MM, LARGE PACK	2	PCS	5707292141347

164	74786	HANDLE FOR LOCK	2	PCS	5707292419866
165	0609721	INSEX 4*20 FZB	2	PCS	5707292439222
166	2501408	BLACK BLIND FERRULE	2	PCS	5707292143921
167	73786	PLATE FOR SIZE 3+4	1	PCS	5707292438911
169	1100490	SPRING WASHER M6 STAINLESS	4	PCS	5707292411297
170	1631060	STAINLESS LOCK NUT 6MM	4	PCS	5707292141446
171	74769	LID FOR TOOTHED WHEEL	2	PCS	5707292468437
172	2501406	NUT PROTECTION, BLACK F/M8	2	PCS	5707292375919
173	1350026	DISTANCE PIECE 8.2*18*27MM	2	PCS	5707292501363

*		CHOOSE SIZE			
---	--	-------------	--	--	--

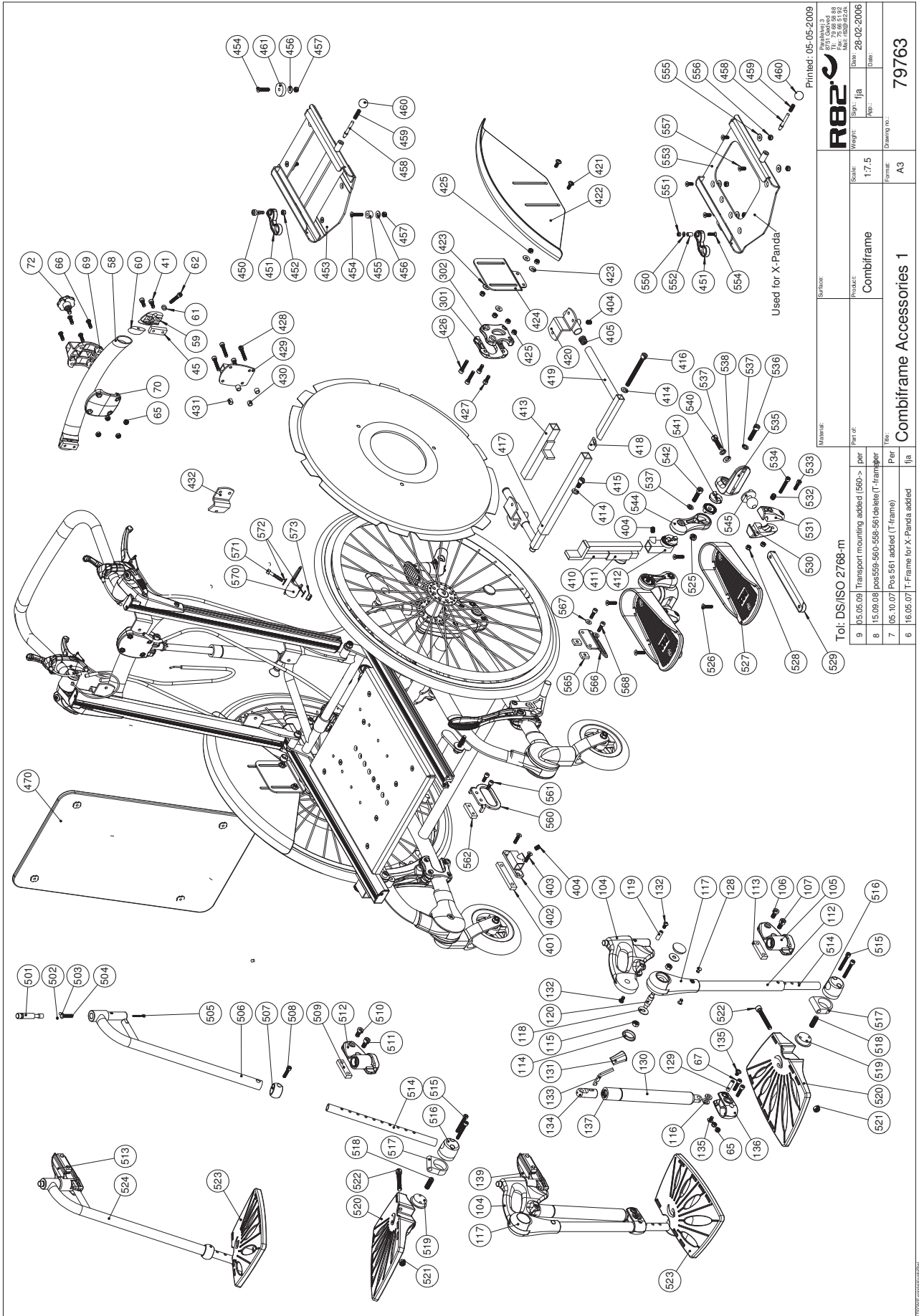

Printed: 05-05-2009

Part No. 79763

Scale: 1:7.5

Product: CombiFrame

Weight: 17.5

Form: A3

Material: Steel

Manufacturer: R82

Address: R82 A/S, Parallelvej 3, 8751 Gedved, Denmark

Phone: +45 79 68 58 88

Fax: +45 75 66 51 92

E-mail: r82@r82.dk

Web: www.r82.com

Part of: CombiFrame

Version: 1.7.5

Scale: 1:7.5

Product: CombiFrame

Weight: 17.5

Form: A3

Material: Steel

Manufacturer: R82

Address: R82 A/S, Parallelvej 3, 8751 Gedved, Denmark

Phone: +45 79 68 58 88

Fax: +45 75 66 51 92

E-mail: r82@r82.dk

Web: www.r82.com

Part of: CombiFrame

Version: 1.7.5

Scale: 1:7.5

Product: CombiFrame

Weight: 17.5

Form: A3

Material: Steel

Manufacturer: R82

Address: R82 A/S, Parallelvej 3, 8751 Gedved, Denmark

Phone: +45 79 68 58 88

Fax: +45 75 66 51 92

E-mail: r82@r82.dk

Web: www.r82.com

Position	No.	Description	Quantity	Unit of Measure	Ean code
41	0602205	STAINLESS INSEX M6*16	4	PCS	5707292404732
45	80609	SLIDEBAR 40*15*8, M6*2	2	PCS	5707292405418
58*	74679	CROSS TUBE,BACK, SIZE 1 260mm	1	PCS	5707292415486
	74605	CROSS TUBE BACK, SIZE 2 310 mm	1	PCS	5707292403070
	74612	CROSS TUBE,BACK SIZE 3	1	PCS	5707292403124
59	74531	Plug For Cross Tube, Cougar	2	PCS	5707292410825
60	74662	LOCK, MOVING PART	2	PCS	5707292401380
61	1105060	STAINLESS WASHER 6MM	2	PCS	5707292140357
62	0602451	INSEX M6*45, STAINLESS	2	PCS	5707292412461
65	1631060	STAINLESS LOCK NUT 6MM	4	PCS	5707292141446
66	0659550	BUTTON HEAD M6*20, STAINLESS	4	PCS	5707292139467
67	0602425	INSEX M6*25, STAINLESS	4	PCS	5707292404596
69	74745	HOLDER FOR HEADREST	1	PCS	5707292410856
70	74746	COVER FOR HOLDER, HEADREST	1	PCS	5707292410863
72	3503001	R82 HANDLE M8*15MM SCREW	1	PCS	5707292413925
104	74811	REMOVABLE HANDLE LEFT, COUGAR	2	PCS	5707292411914
105	74813	STATIONARY BRACKET LEFT	1	PCS	5707292411921
106	0603540	INSEX LOW HEAD M8x40 STAINLESS	2	PCS	5707292412485
107	0609816	STAINLESS INSEX SCREW 8*16	2	PCS	5707292138569
112	74852	TUBE UPPER PART	2	PCS	5707292418586
113	80617	SLIDEBAR 50*15*8,M8*2	2	PCS	5707292405401
114	2985000	PRESSUREPLUG BLACK SOFT RUBBER	4	PCS	5707292145314
115	1607002	LOCK NUT 8MM, A2	4	PCS	5707292410955
116	1350005	PVC BUSH BLACK 15*8,2*4MM	4	PCS	5707292425133
117	74749	ROTARY BUSH	2	PCS	5707292415974
118	1108000	WASHER 8MM	4	PCS	5707292140388
119	74678	BUSH ø8-M6, FOOTREST	2	PCS	5707292418562
120	74653	BUSH Ø12-M8, footrest	2	PCS	5707292418555
128	0506513	STAINLESS BLIND RIVET Ø4, 8*12	4	PCS	5707292137326
129	74675	BUSH Ø8x28-M6, COUGAR	2	PCS	5707292412065
130	P-2206010	GASSPRING F/COUGAR REBORED	2	PCS	5707292446121
131	2200030	BUTTON FOR RELEASE ARM	2	PCS	5707292143051
132	0659200	BUTTON HEAD SCREW 6*12 STAINL.	4	PCS	5707292139436
133	2200022	RELEASE LEVER, COUGAR	2	PCS	5707292412126
134	6118823	HOUSE FOR RELEASE, COUGAR	2	PCS	5707292412133
135	0659010	BUTTON HEAD WITH FLANGE M6*10	4	PCS	5707292356819
136	74759	EXTENSION COUPLING, FOOTREST	2	PCS	5707292412140
137	1761000	COUNTER NUT 10*1,0 FINE THREAD	2	PCS	5707292141538
139	74812	STATIONARY BRACKET, RIGHT	1	PCS	5707292415813
301	74731	COVER INNER, BRAKE,	2	PCS	5707292412409
302	74730	COVER OUTER, BRAKE, COUGAR	2	PCS	5707292412416
401	80618	SLIDEBAR 80*15*8, M8*2	2	PCS	5707292434258
402	73741	FOOTREST MOUNTING	2	PCS	5707292433480
403	0602486	INSEX M8*16, RF UNDERSÆNKET	4	PCS	5707292434272
404	0808012	STAINLESS STOP SCREW 8*12	2	PCS	5707292140067
405	2501600	FERRULE D15*1,0 BLACK	2	PCS	5707292143969
410*	81495	DOUBLE BAR, SIZE 0	1	PCS	5707292115898
	81169	DOUBLE FOOTSUPPORT-ROD SZ. 1	1	PCS	5707292112897
	81170	DOUBLE FOOTSUPPORT-ROD SZ. 2	1	PCS	5707292112903

	81494	DOUBLE FOOTSUPPORT ROD SIZE 3	1	PCS	5707292115881
	81496	DOUBLE FOOTREST ROD 50CM	1	PCS	5707292115904
411	73734	MOUNTING FOR FOOTREST,RIGHT	1	PCS	5707292433466
412	73735	MOUNTING FOR FOOTREST,LEFT	1	PCS	5707292433473
413	72373	BAR FOR SINGLE FOOTREST,245MM	1	PCS	5707292108029
414	1103200	FAZET WASHER 8MM	3	PCS	5707292140296
415	0602900	INSEX SCREW 8*16	1	PCS	5707292138200
416	0603980	INSEX M8*115, FZB	2	PCS	5707292436580
417	73643	FRONTBAR SIZE 1+2, COMBIFRAME	1	PCS	5707292436122
418	73453	BUSHING FOR CROCODILE	2	PCS	5707292352255
419	73641	BAR FOR FOOT REST, ANGLE	2	PCS	5707292436153
420	73754	MOUNTING	2	PCS	5707292433534
421	0659501	BUTTON HEAD W/FLANGE M6*16 FZB	4	PCS	5707292356000
422	74094	MUDGUARD LEFT/RIGHT	2	PCS	5707292432148
423	1105060	STAINLESS WASHER 6MM	8	PCS	5707292140357
424	73731	PLATE FOR MUD GUARD	2	PCS	5707292430953
425	1631060	STAINLESS LOCK NUT 6MM	12	PCS	5707292141446
426	0602430	INSEX M6*30 RUSTFRI	4	PCS	5707292407276
427	0602205	STAINLESS INSEX M6*16	4	PCS	5707292404732
428	0602435	INSEX SCREW 6*35	4	PCS	5707292138132
429	74880	MOUNTING FOR HEADSUPPORT	1	PCS	5707292432230
430	81693	DISTANCE PART, SEAT/ADAPTOR	2	PCS	5707292117847
431	81219	BUSH FOR HEADRESTMOUNTING	2	PCS	5707292434265
432	980110	FITTINGS CURVED FOR WOOD BACK	1	PCS	5707292433541
450	0683810	PASBOLT M/INDV.6KT. 8*12 (M6)	1	PCS	5707292361929
451	82880	HANDLE FOR ADAPTOR	1	PCS	5707292361318
452	1606500	LOCK NUT 6 MM	1	PCS	5707292141361
453	81763	T-FRAME FOR AUTOPLATE	1	PCS	5707292118486
454	0609722	INSEX 6*30 UH, STAINLESS	4	PCS	5707292439215
455	73719	WASHER FOR T-FRAME	2	PCS	5707292436818
456	1105060	STAINLESS WASHER 6MM	4	PCS	5707292140357
457	1631060	STAINLESS LOCK NUT 6MM	4	PCS	5707292141446
458	6113000	SPINDLE, T-PART/FRAME	1	PCS	5707292186362
459	0310100	PRESSURESPRING 0,9*8*23*10,5V	1	PCS	5707292135513
460	3706256	KNOB 25, M6 THREAD	1	PCS	5707292147219
461	73720	WASHER FOR T-FRAME	2	PCS	5707292436696
470*	980300-1	WOODEN BACK SIZE 1 WITH SCREWS	1	PCS	5707292431165
	980300-2	WOODEN BACK SIZE 2 WITH SCREWS	1	PCS	5707292431172
	980300-3	WOODEN BACK SIZE 3 WITH SCREWS	1	PCS	5707292431189
	980300-4	WOODEN BACK SIZE 4 WITH SCREWS	1	PCS	5707292431196
501	74816	PRESS BUTTON FOR FOOT REST	1	PCS	5707292425843
502	0250020	Steel ball RB5	2	PCS	5707292415196
503	0101500	O-RING Ø9*1,5 EP7010, EPDM	1	PCS	5707292415516
504	0310100	PRESSURESPRING 0,9*8*23*10,5V	1	PCS	5707292135513
505	0602457	INSEX M3*30, STAINLESS	1	PCS	5707292412478
506	74877	UPPER PART FOR FOOT REST,LEFT	1	PCS	5707292425881
507	74814	BUSH FOR FOOTREST	2	PCS	5707292426239
508	0602425	INSEX M6*25, STAINLESS	2	PCS	5707292404596
509	80617	SLIDEBAR 50*15*8,M8*2	2	PCS	5707292405401
510	0603540	INSEX LOW HEAD M8x40 STAINLESS	2	PCS	5707292412485

511	0609816	STAINLESS INSEX SCREW 8*16	2	PCS	5707292138569
512	5097001	STATIONARY BRACKET, LEFT	1	PCS	5707292412744
513	5097002	STATIONARY BRACKET, RIGHT	1	PCS	5707292412751
514	74851	TUBE HEIGHT ADJUSTABLE	1	PCS	5707292418579
515	0602470	INSEX M6*70, STAINLESS	2	PCS	5707292407870
516	74672	STAND FOR TUBE, COUGAR	1	PCS	5707292411938
517	74673	HOUSE, COUGAR FOOTREST	1	PCS	5707292412010
518	1237160	STAINLESS SCREW M10*30,POINTED	1	PCS	5707292415011
519	74674	FLANGE FOR FOOTREST, COUGAR	1	PCS	5707292412034
520*	74757	BASEPLATE, LEFT	1	PCS	5707292412041
	74747	FOOT PLATE, LEFT SIZE 2	1	PCS	5707292434074
521	1631080	STAINLESS LOCK NUT 8MM	1	PCS	5707292141453
522	0602491	INSEX M8*60, STAINLESS	1	PCS	5707292412058
523*	74756	BASEPLATE, RIGHT, SIZE 1	1	PCS	5707292412157
	74748	FOOT PLATE, RIGHT, SIZE 2	1	PCS	5707292434081
524	74878	UPPER PART FOR FOOT REST,RIGHT	1	PCS	5707292425867
525	1607000	LOCK NUT 8MM, LARGE PACK	2	PCS	5707292141378
526	0612500	INSEX SCREW COUNTERS. 6*25	4	PCS	5707292138712
527*	82156-60	FOOTPLATE SIZE 1,BLACK,GAZELLE	2	PCS	5707292230492
	82154-60	FOOTPLATE SIZE 2,BLACK,GAZELLE	2	PCS	5707292230478
	82158-60	FOOTPLATE SIZE 3 BLACK,GAZELLE	2	PCS	5707292230508
528	1606500	LOCK NUT 6 MM	4	PCS	5707292141361
529*	73648	STICK SIZE 1, FOOTREST	2	PCS	5707292445797
	73649	STICK SIZE 2, FOOTREST	2	PCS	5707292445803
530	72724	BALL JOINING RIGHT	2	PCS	5707292421845
531	72723	BALL JOINING LEFT	2	PCS	5707292421814
532	0701010	SET SCREW M10*10 W/FLAT END	2	PCS	5707292434098
533	0602300	INSEX SCREW 6*20	2	PCS	5707292138101
534	0602440	INSEX M6*40 FZB	2	PCS	5707292138149
535	72713	BRACKED	2	PCS	5707292420763
536	0603000	INSEX SCREW 8*35	2	PCS	5707292138217
537	1101000	CENTER SPRING WASHER 8MM	6	PCS	5707292140173
538	1102010	FACE PLATE 8,4*18*2MM, FZB	2	PCS	5707292429766
540	0602700	INSEX SCREW 8*25	2	PCS	5707292138187
541	74062	TOOTHED DISK 30*14	6	PCS	5707292377951
542	0602600	INSEX 8*30	2	PCS	5707292138170
544	72712	KNEE JOINT	2	PCS	5707292420756
545	72721	BALLHEAD	2	PCS	5707292428875
550	1102900	FAZET WASHER 5 MM	1	PCS	5707292140272
551	1606300	LOCK NUT 5 MM	1	PCS	5707292141354
552	75641	BUSH 5-8*11,8	1	PCS	5707292466501
553	75609-01	T-FRAME,X:PANDA PÅ COMBI FRAME	1	PCS	5707292466440
554	0615025	INSEX SCREW 5*25	1	PCS	5707292138873
555	1105060	STAINLESS WASHER 6MM	1	PCS	5707292140357
556	1605000	STAINLESS NUT M6	1	PCS	5707292141316
557	0609732	INSEX 6*22, STAINLESS	1	PCS	5707292409669
560	73746	TRANSPORT FITTING	2	PCS	5707292445063
561	0602205	STAINLESS INSEX M6*16	4	PCS	5707292404732
562	80609	SLIDEBAR 40*15*8,M6*2	2	PCS	5707292405418
565	74090	MOUNTING FOR FRONTFORK	4	PCS	5707292374066

566	73718	CLAMPING BRACKET	2	PCS	5707292470188
567	1100800	CENTER SPRING WASHER 6MM	4	PCS	5707292140166
568	0602300	INSEX SCREW 6*20	4	PCS	5707292138101
570	73750	BELT FITTING	1	PCS	5707292439857
571	0683500	INSEX SCREW 6*35 STAINLESS	1	PCS	5707292139702
572	1100490	SPRING WASHER M6 STAINLESS	2	PCS	5707292411297
573	1631060	STAINLESS LOCK NUT 6MM	1	PCS	5707292141446

*		CHOOSE SIZE			
---	--	-------------	--	--	--