

high-low:x

Exploded views / Part lists


03-2021

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.




Contents

Size 1, indoor	3
Size 1, outside	5
Size 2, indoor	8
Size 2, inside (hydraulic).....	11
Size 2, outside	13
Size 2, outside (hydraulic).....	17
Size 3, indoor.....	20
Foot support.....	23

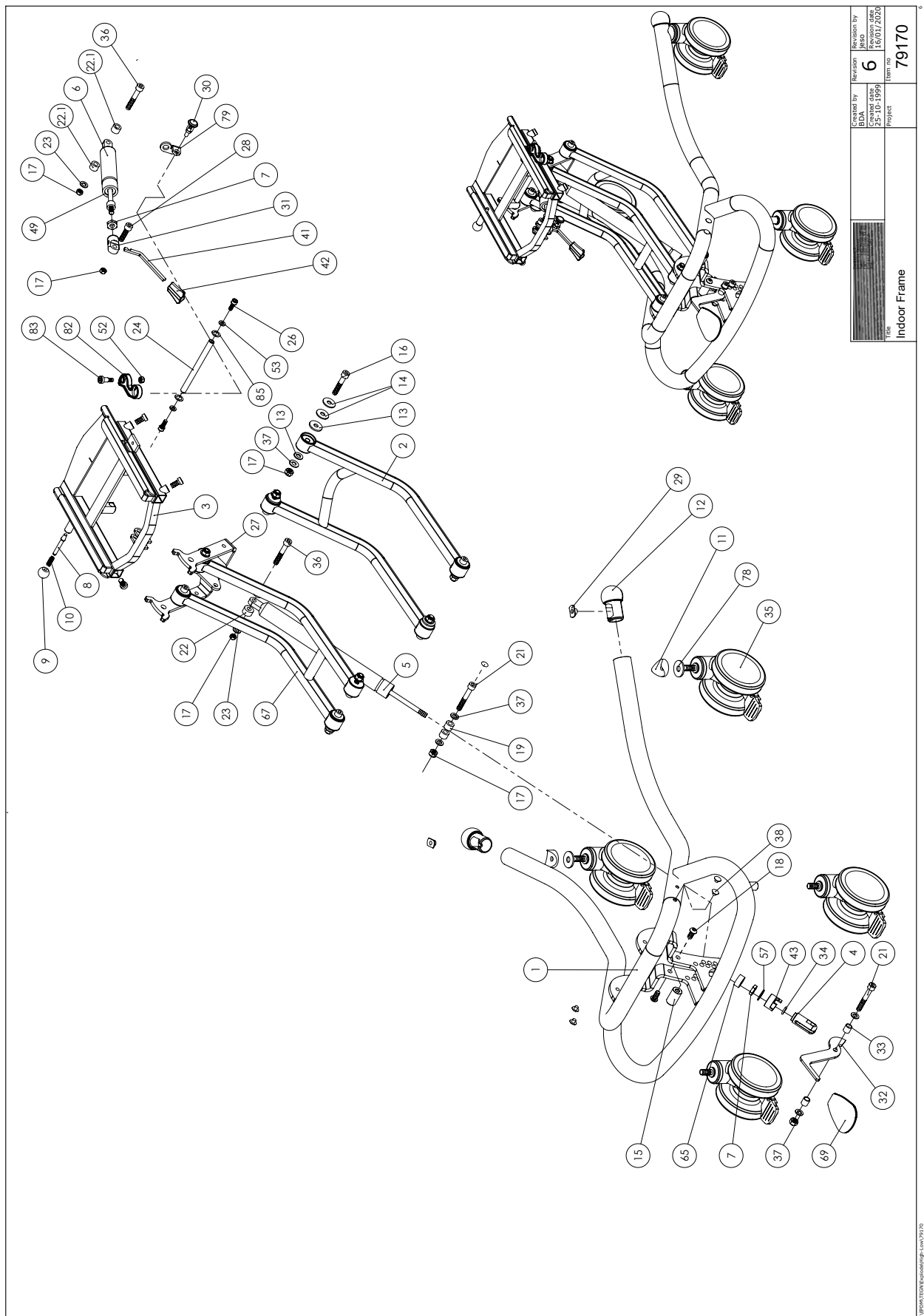


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	82820-60	Frame, black mother-of-pearl	1	PCS	5707292459718
2	82903-60	Carry arm, welded, high-low 2	1	PCS	5707292459404
4	6118822	Horizontal release	1	PCS	5707292186744
11	82894	Saddle washer	2	PCS	5707292387578
12+29	5082829	Mounting of knob and weld nut	2	PCS	5707292569790
13	1310705	Nylon washer, white, Ø25*8,4*2,5	16	PCS	5707292140791
14	1108000	Washer 8mm	16	PCS	5707292140388
15	82808	Assembly bush	1	PCS	5707292125224
16	603505	Insex M8*45, zink flake, 12.9	8	PCS	5707292482846
17	1607000	Lock nut 8mm, large pack	13	PCS	5707292141378
18	661500	Button head screw 8*16 stainl.	2	PCS	5707292139559
19	1350000	Distancepart black 6*15*8,2 mm	2	PCS	5707292140876
21	603600	Insex screw 8*55	2	PCS	5707292138262
22.1	1350021	Distance piece 11*15*8,2mm	2	PCS	5707292140890
23	1103200	Fazet washer 8mm	2	PCS	5707292140296
24	82806	Axle	1	PCS	5707292125200
26	602200	Insex screw 6*16	2	PCS	5707292138095
27	76800-60	Tilt brace mother of pearl	1	PCS	5707292497116
28	615010	Insex M8*30 fzb, shortened	1	PCS	5707292367990
31	155775	Release for gas spring	1	PCS	5707292928733
32	82851	Arm for foot pedal	1	PCS	5707292125507
33	82859	Bush for foot pedal	2	PCS	5707292125552
34	108000	O-rings o13*2,5 quality xp 70	1	PCS	5707292135124
35	3006938	100mm double wheel with brake	4	PCS	5707292574930
36	603555	Insex M8*50, 12.9, zinkflage	2	PCS	5707292498892
37	1102000	Planing washer 8mm	14	PCS	5707292140227
38	1310508	DR. fastener - afd. prop	4	PCS	5707292829795
43	82866	Lock for release head	1	PCS	5707292125606
53	1106000	Washer 6mm	2	PCS	5707292140371
57	1102500	Planning washer 10 mm	1	PCS	5707292140234
67	82888-60	Carryam, black mother of pearl	1	PCS	5707292459695
69	3470335	Release button 4*8mm	1	PCS	5707292146632
78	1108050	Washer M10, din9021	2	PCS	5707292140401
85	1310640	Washer, nature Ø16*10*0,5mm	2	PCS	5707292140760
130	1101000	Center spring washer 8mm	1	PCS	5707292140173
12+29	5082829	Mounting of knob and weld nut	2	PCS	5710000000000
8+9 10+52 82+83 94+95	5075600-60	T-frame mounted, black	1	PCS	5707292460127
6+7 30+31 79+41 42	5040051	Complete bloc-o-lift f.highlow	1	PCS	5707292397676
5+7 65	5040050	Gas spring for high low	1	PCS	5707292397669
6+7 31+85	89550-TILT-NO	Kit til mont. af tiltudløser,	1	PCS	5707292825513
8+9 10+52 82+83 94+95	5075610-60	T-frame mounted, black	1	PCS	5707292523327

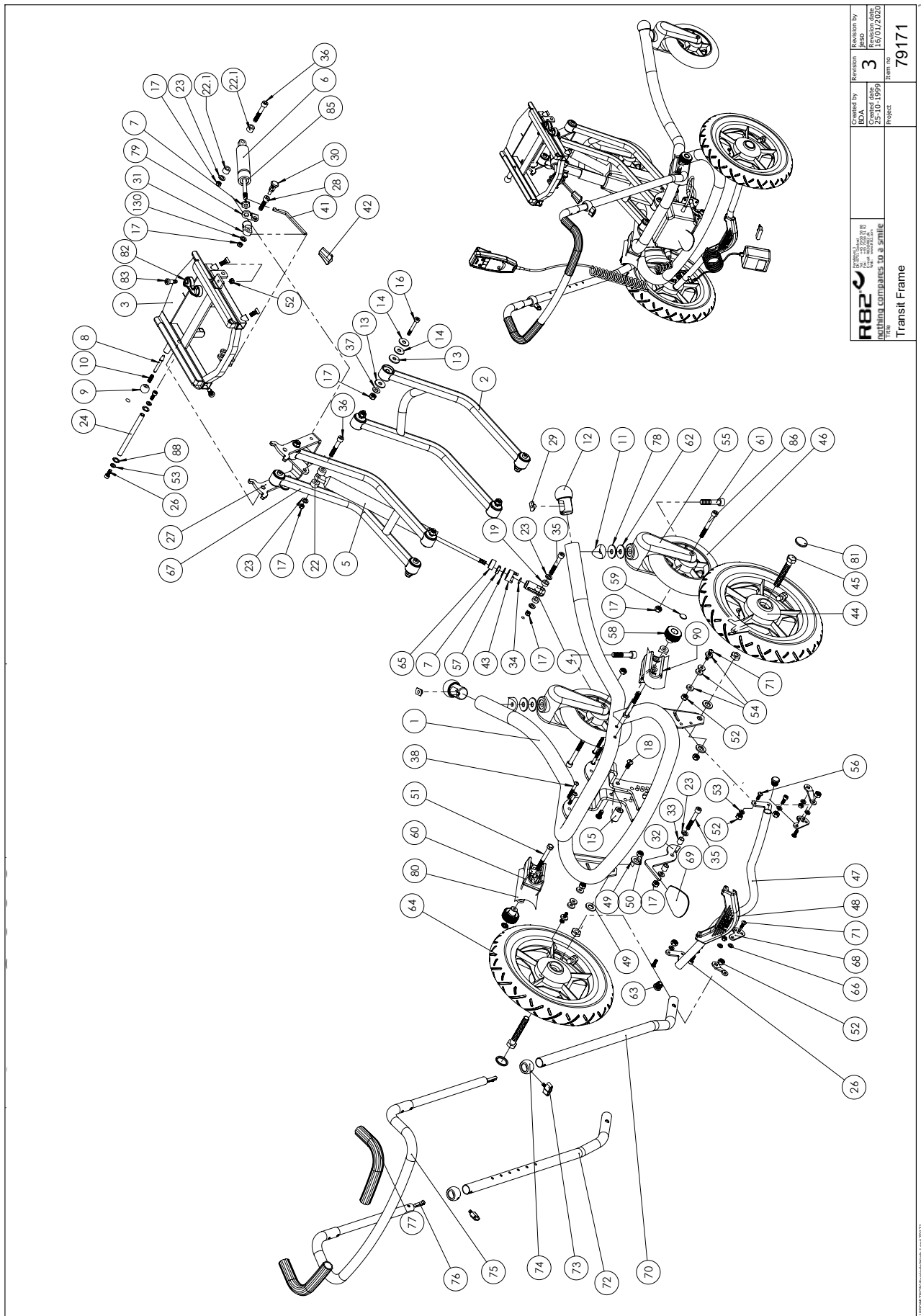

R82 [®] nothing compares to a smile Title	Created by BDA 25-10-1999	Revision 3	Revision by JSD 16/01/2020
Transit Frame	Project	Item no	79171

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	82656-60	frame	1	PCS	5707292460295
2	82903-60	Carry arm, welded, high-low 2	1	PCS	5707292459404
4	6118822	Horizontal release	1	PCS	5707292186744
11	82894	Saddle washer	2	PCS	5707292387578
12+29	5082829	Mounting of knob and weld nut	2	PCS	5707292569790
13	1310705	Nylon washer, white, Ø25*8,4*2,5	16	PCS	5707292140791
14	1108000	Washer 8mm	16	PCS	5707292140388
15	82808	Assembly bush	1	PCS	5707292125224
16	0603505	Insex M8*45, zink flake, 12.9	8	PCS	5707292482846
17	1607000	Lock nut 8mm, large pack	15	PCS	5707292141378
18	0661500	Button head screw 8*16 stainl.	2	PCS	5707292139559
19	1350000	Distancepart black 6*15*8,2 mm	2	PCS	5707292140876
21	1103000	Fazet washer 6mm	6	PCS	5710000000000
22	1350100	Distance part white 8,2*18*10	4	PCS	5707292140937
23	1103200	Fazet washer 8mm	2	PCS	5707292140296
24	82806	Axle	1	PCS	5707292125200
26	0602200	Insex screw 6*16	4	PCS	5707292138095
27	76800-60	Tilt brace mother of pearl	1	PCS	5707292497116
28	0615010	Insex M8*30 fzb, shortened	1	PCS	5707292367990
31	155775	Release for gas spring	1	PCS	5707292928733
32	82851	Arm for foot pedal	1	PCS	5707292125507
33	82859	Bush for foot pedal	2	PCS	5707292125552
34	0108000	O-rings o13*2,5 quality xp 70	1	PCS	5707292135124
35	0603600	Insex screw 8*55	2	PCS	5707292138262
36	0603555	Insex M8*50, 12.9, zinkflake	2	PCS	5707292498892
37	1102000	Planing washer 8mm	12	PCS	5707292140227
38	0645645	Button head M6*45 zn/black	4	PCS	5707292139351
43	82866	Lock for release head	1	PCS	5707292125606
44	3003510	12½*2 1/4" br/left, high-low	1	PCS	5707292397492
44	3002500	12½*2 1/4 black wheel, br/right	1	PCS	5710000000000
44	3003007	12½*2 1/4 wheel, solid, right	1	PCS	5710000000000
45	1902380	Steel set screw 12*80mm	2	PCS	5707292142030
46	3007910	180*45 black/pneum. Wheel	2	PCS	5707292146045
46	3007875	175*40 wheel, black, solid	2	PCS	5707290000000
47	82840	Brake brace	1	PCS	5707292125415
48	82830	Brake pedal, indoor frame	1	PCS	5707292125347
49	1101600	Center spring washer 12mm	4	PCS	5707292140197
50	1604500	Steel nut 12mm, large pack	2	PCS	5707292141309
51	1902320	Steel set screw 8*70mm	2	PCS	5707292142023
52	1606500	Lock nut 6 mm	8	PCS	5707292141361
53	1106000	Washer 6mm	2	PCS	5707292140371
54	1125618	Screenwasher fz6*18*3mm	8	PCS	5707292140494
55	2965155	Frontfork 200mm, black	2	PCS	5707292144935
56	0641620	Button head screw 6*20	4	PCS	5707292139276
57	1102500	Planning washer 10 mm	1	PCS	5707292140234
58	3470400	Hand grip without cover	2	PCS	5707292146663
59	1109500	Starlock washer with cape 6	2	PCS	5707292140449
60	82899	Hand brace holder, left	1	PCS	5707292401397
61	0606000	Insex screw 10*50	2	PCS	5707292138408


62	1103400	Fazet washer 10 mm	2	PCS	5707292140319
63	2502002	Ferrule d18*0,8-2,5 black	2	PCS	5707292433947
64	3004010	12½*2 1/4" br/right, high-low	1	PCS	5707292397508
64	3003000	12½*2 1/4" black wheel,br/left	1	PCS	5710000000000
64	3003005	12½*2 1/4 wheel, solid, left	1	PCS	5710000000000
66	1310628	Distance pcs,6,4*10*2,8mm	2	PCS	5707292140746
67	82888-60	Carryam, black mother of pearl	1	PCS	5707292459695
68	82838	Linkarm	2	PCS	5707292125392
69	3470335	Release button 4*8mm	1	PCS	5707292146632
70	82856-60	Rod for handbrace, right	1	PCS	5707292459862
71	0641625	Button head screw 6*25 mm	2	PCS	5707292139283
72	82855-60	Rod for handbrace, left	1	PCS	5707292459831
73	3652501	R82 handle M6*16mm screw	2	PCS	5707292413963
74	71137	Bush ball for handbrace	2	PCS	5707292361813
75	82858-60	Handle	1	PCS	5707292459879
76	6153500	Knob spring 0,5*6,5 mm.	2	PCS	5707292188571
77	2500563	Grip f/Ø22 handbrace 400mm	2	PCS	5707292612311
78	1108050	Washer M10, din9021	2	PCS	5707292140401
80	1357500	Distance pcs 20*8,4*4mm	2	PCS	5707292141026
81	2704300	Black hub cap for rolko wheel	2	PCS	5707292144515
85	1310640	Washer, nature Ø16*10*0,5mm	2	PCS	5710000000000
86	0603700	Insex screw 8*70	2	PCS	5707292138293
88	1310640	Washer, nature Ø16*10*0,5mm	2	PCS	5707292140760
90	82828	Hand brace holder, high-low	1	PCS	5707292125330
91	1101600	Center spring washer 12mm	4	PCS	5710000000000
8+9 10+52 82+83 94+95	5076855-60	T-frame, transport, assembled	1	PCS	5707292528186
6+7 30+31 79+41 42	5040051	Complete bloc-o-lift f.highlow	1	PCS	5707292397676
5+7 65	5040050	Gas spring for high low	1	PCS	5707292397669
6+7 31+85	89550-TILT- NOU	Kit til mont. tiltudløser,ude	1	PCS	5707292830265

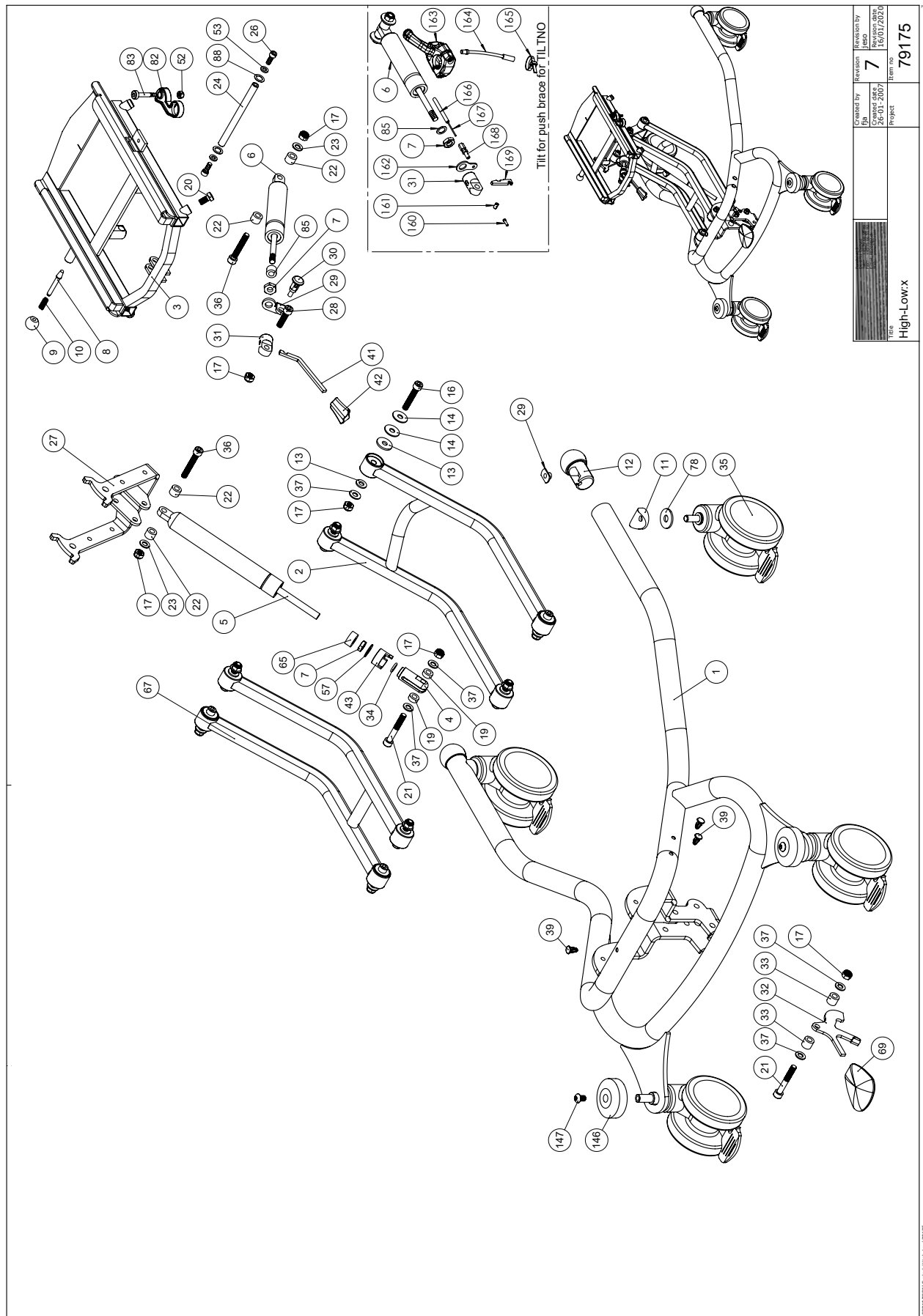


Created by J. J. J.	Revision 7	Created date 26-01-2007	Revision date 16-01-2020
Project	Item no. 79175		

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	75666-60	Frame, High/low 2,	1	PCS	5707292472885
2	82903-60	Carry arm, welded, high-low 2	1	PCS	5707292459404
4	6118822	Horizontal release	1	PCS	5707292186744
11	82894	Saddle washer	2	PCS	5707292387578
12+29	5082829	Mounting of knob and weld nut	2	PCS	5707292569790
13	1310705	Nylon washer,white,Ø25*8,4*2,5	16	PCS	5707292140791
14	1108000	Washer 8mm	16	PCS	5707292140388
16	0603505	Insex M8*45, zink flake, 12.9	8	PCS	5707292482846
17	1607000	Lock nut 8mm, large pack	13	PCS	5707292141378
19	1350000	Distancepart black 6*15*8,2 mm	2	PCS	5707292140876
21	0603600	Insex screw 8*55	2	PCS	5707292138262
22	1350021	Distance piece 11*15*8,2mm	2	PCS	5707292140890
23	1103200	Fazet washer 8mm	2	PCS	5707292140296
24	82806	Axle	1	PCS	5707292125200
26	0602200	Insex screw 6*16	2	PCS	5707292138095
27	76800-60	Tilt brace mother of pearl	1	PCS	5707292497116
28	0615010	Insex M8*30 fzb, shortened	1	PCS	5707292367990
31	155775	Release for gas spring	1	PCS	5707292928733
32	82914	Pedal, Wombat	1	PCS	5707292125712
33	1350023	Distance piece, black	2	PCS	5707292140913
34	0108000	O-rings o13*2,5 quality xp 70	1	PCS	5707292135124
35	3006938	100mm double wheel with brake	4	PCS	5707292574930
36	0603555	Insex M8*50, 12.9, zinkflage	2	PCS	5707292498892
37	1102000	Planing washer 8mm	14	PCS	5707292140227
39	1310508	DR. fastener - afd. prop	5	PCS	5707292829795
43	82866	Lock for release head	1	PCS	5707292125606
53	1106000	Washer 6mm	2	PCS	5707292140371
57	1102500	Planning washer 10 mm	1	PCS	5707292140234
67	82888-60	Carryam, black mother of pearl	1	PCS	5707292459695
69	3470335	Release button 4*8mm	1	PCS	5707292146632
78	1108050	Washer M10, din9021	2	PCS	5707292140401
85	1310640	Washer, nature Ø16*10*0,5mm	2	PCS	5707292140760
88	1310640	Washer, nature Ø16*10*0,5mm	2	PCS	5707292140760
98	1102800	Fazet washer 4mm	4	PCS	5707292140265
99+100	2222161	Charger with plugs	1	PCS	5707292574046
101	2220600	Handset w/r82 print & charging	1	PCS	5707292492241
102	1902590	Crosshead screw M3*10, FZB	4	PCS	5707292408389
103	2222110	Batteryboks ba18027	1	PCS	5707292492258
104	1607000	Lock nut 8mm, large pack	2	PCS	5707292141378
105	1102000	Planing washer 8mm	4	PCS	5707292140227
106	1606250	Self-locking nut 3mm	4	PCS	5707292408396
107	2505000	Ferrule e30*301,0-2,0 int.blac	1	PCS	5707292144140
108	1902760	Cross-headed screw 5*16 mm	4	PCS	5707292142146
109	77576	Battery console highlow	1	PCS	5707292490759
110	1606300	Lock nut 5 mm	4	PCS	5707292141354
111	603600	Insex screw 8*55	2	PCS	5707292138262
115	2222135	Actuator la28 f/highlow 2	1	PCS	5707292492289
116	82848	Bush	2	PCS	5707292125484
117	603555	Insex M8*50, 12.9, zinkflage	1	PCS	5707292498892

Position	No.	Description	Quantity	Unit of Measure	Ean code
118	1102900	Fazet washer 5 mm	4	PCS	5707292140272
119	P-6108200	Plast tube pa11 Ø10/Ø8mmx30mm	1	PCS	5707292585424
130	1101000	Center spring washer 8mm	1	PCS	5707292140173
146	2522100	Black bumpwheel for Pony 60mm	2	PCS	5707292144478
146	2522100	Black bumpwheel for Pony 60mm	2	PCS	5707292144478
147	0661300	button head with flange 8*12	2	PCS	5707292408068
147	661300	button head with flange 8*12	2	PCS	5707292408068
3+8 9+10 20+52 82+83	5075630-60	T-frame, black	1	PCS	5707292459428
6+7 29+30 31+41 42+85	5040051	Complete bloc-o-lift f.highlow	1	PCS	5707292397676
5+7 65	5040055	Bloc-o-lift1400n kit-high/low2	1	PCS	5707292460110
6+7 31+85 160-169	89550-TILT-NO	Kit til mont. af tiltudløser,	1	PCS	5707292825513

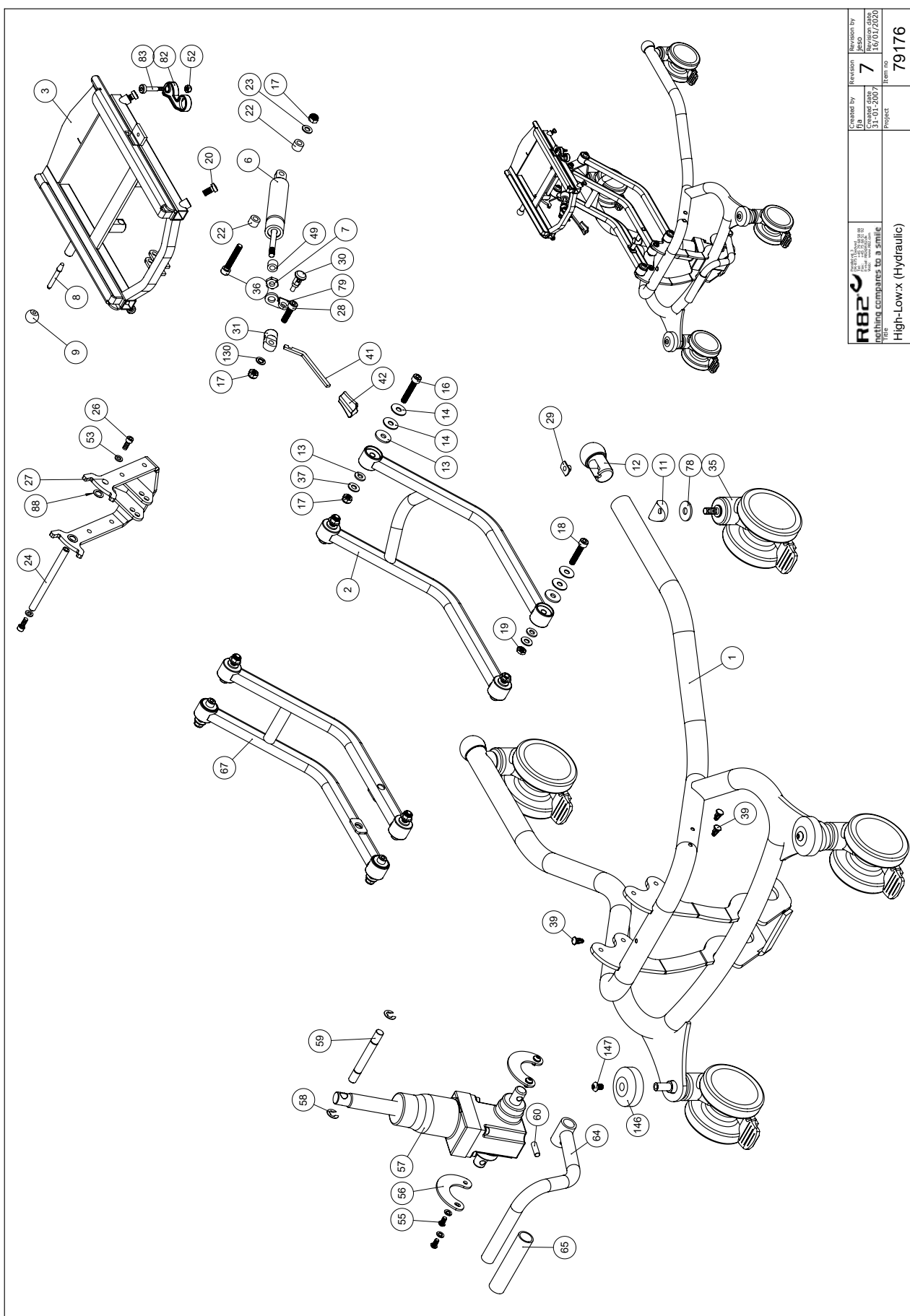

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	75665-60	frame size 2 high/low	1	PCS	5707292472861
2	82903-60	Carry arm, welded, high-low 2	1	PCS	5707292459404
11	82894	Saddle washer	2	PCS	5707292387578
12+29	5082829	Mounting of knob and weld nut	2	PCS	5707292569790
13	1310705	Nylon washer, white, Ø25*8,4*2,5	16	PCS	5707292140791
14	1108000	Washer 8mm	16	PCS	5707292140388
16	0603505	Insex M8*45, zink flake, 12.9	6	PCS	5707292482846
17	1607000	Lock nut 8mm, large pack	8	PCS	5707292141378
18	0603455	Insex M8*40, fzb, st 12.9	2	PCS	5707292482853
19	1750800	Lock nut, stainless M8	2	PCS	5707292141484
22	1350021	Distance piece 11*15*8,2mm	2	PCS	5707292140890
23	1103200	Fazet washer 8mm	1	PCS	5707292140296
24	82806	Axle	1	PCS	5707292125200
26	0602200	Insex screw 6*16	2	PCS	5707292138095
27	76800-60	Tilt brace mother of pearl	1	PCS	5707292497116
28	0615010	Insex M8*30 fzb, shortened	1	PCS	5707292367990
31	155775	Release for gas spring	1	PCS	5707292928733
35	3006938	100mm double wheel with brake	4	PCS	5707292574930
36	0603555	Insex M8*50, 12.9, zinkflage	1	PCS	5707292498892
37	1102000	Planing washer 8mm	8	PCS	5707292140227
39	1310508	DR. fastener - afd. prop	5	PCS	5707292829795
53	1106000	Washer 6mm	2	PCS	5707292140371
55	0641200	Button head screw 6*12	4	PCS	5707292139252
56	75615	Bracket for hydraulic	2	PCS	5707292460202
57	2208000	Compact mk5 hydraulic pump	1	PCS	5707292461148
58	1145095	lock ring for 10mm, din 6799, rf	2	PCS	5707292473332
59	75620	Stift for liftarm	1	PCS	5707292460226
60	0584490	Stift din 1481 8*30 rf	1	PCS	5707292473356
64	75668-60	Arm for hydraulic, black	1	PCS	5707292479631
65	2500516	Grip 87.00 mm length	1	PCS	5707292356727
67	75616-60	carry arm for high/low,	1	PCS	5707292460158
78	1108050	Washer M10, din9021	2	PCS	5707292140401
88	1310640	Washer, nature Ø16*10*0,5mm	2	PCS	5707292140760
130	1101000	Center spring washer 8mm	1	PCS	5707292140173
146	2522100	Black bumpwheel for Pony 60mm	2	PCS	5707292144478
147	0661300	button head with flange 8*12	2	PCS	5707292408068
148	2503010	Ferrule d22*1,8-3,5, black	1	PCS	5707292357632
6+7 30+31 29/79 41+42 49/85	5040051	Complete bloc-o-lift f.highlow	1	PCS	5707292397676
3+8 9+10 20+52 53+82 83	5075630-60	T-frame, black	1	PCS	5707292459428
6+7 30+31 41+42 49+79	5040051	Complete bloc-o-lift f.highlow	1	PCS	5707292397676

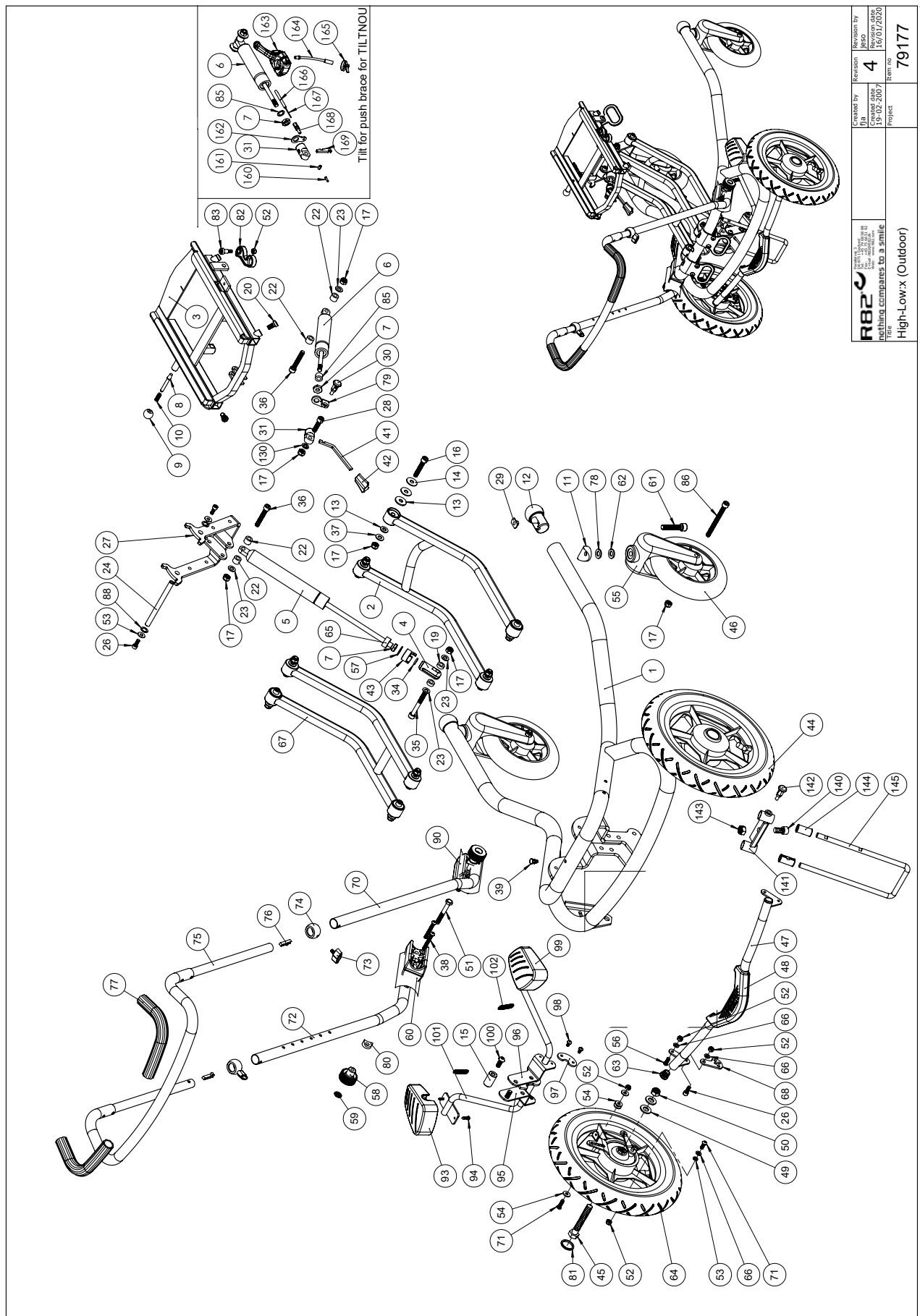


R82 ETC A/S 8250 Røndevej 3 DK-2600 Lyngby Denmark Tel: +45 79 68 58 88 Fax: +45 75 66 51 92 E-mail: r82@r82.dk Web: www.etac.com	Created by	Revision	Revision by
	fla	4	fla
	19-03-2007		19-03-2007
Item	Project	Item no	79177
High-Low:x (Outdoor)			

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	75640-60	High-low 2 outdoor frame	1	PCS	5707292460011
2	82903-60	Carry arm, welded, high-low 2	1	PCS	5707292459404
4	6118822	Horizontal release	1	PCS	5707292186744
11	82894	Saddle washer	2	PCS	5707292387578
12+29	5082829	Mounting of knob and weld nut	2	PCS	5707292569790
13	1310705	Nylon washer, white, Ø25*8,4*2,5	16	PCS	5707292140791
14	1108000	Washer 8mm	16	PCS	5707292140388
15	82808	Assembly bush	1	PCS	5707292125224
16	0603505	Insex M8*45, zink flake, 12.9	8	PCS	5707292482846
17	1607000	Lock nut 8mm, large pack	14	PCS	5707292141378
19	1350000	Distancepart black 6*15*8,2 mm	2	PCS	5707292140876
22	1350021	Distance piece 11*15*8,2mm	4	PCS	5707292140890
23	1103200	Fazet washer 8mm	2	PCS	5707292140296
24	82806	Axle	1	PCS	5707292125200
26	0602200	Insex screw 6*16	4	PCS	5707292138095
27	76800-60	Tilt brace mother of pearl	1	PCS	5707292497116
28	0615010	Insex M8*30 fzb, shortened	1	PCS	5707292367990
31	155775	Release for gas spring	1	PCS	5707292928733
34	0108000	O-rings o13*2,5 quality xp 70	1	PCS	5707292135124
35	0603660	Insex screw 8*60	1	PCS	5707292138279
36	0603555	Insex M8*50, 12.9, zinkflage	2	PCS	5707292498892
37	1102000	Planing washer 8mm	14	PCS	5707292140227
38	0645645	Button head M6*45 zn/black	4	PCS	5707292139351
39	1310508	DR. fastener - afd. prop	1	PCS	5707292829795
43	82866	Lock for release head	1	PCS	5707292125606
44	3003510	12½*2 1/4" br/left, high-low	1	PCS	5707292397492
46	3007910	180*45 black/pneum. Wheel	2	PCS	5707292146045
46	3007875	175*40 wheel, black, solid	2	PCS	5,70729E+12
47	82840	Brake brace	1	PCS	5707292125415
48	82830	Brake pedal, indoor frame	1	PCS	5707292125347
49	1101600	Center spring washer 12mm	4	PCS	5707292140197
50	1604500	Steel nut 12mm, large pack	2	PCS	5707292141309
51	1902320	Steel set screw 8*70mm	2	PCS	5707292142023
52	1606500	Lock nut 6 mm	8	PCS	5707292141361
53	1106000	Washer 6mm	2	PCS	5707292140371
54	1125618	Screenwasher fz6*18*3mm	8	PCS	5707292140494
55	2965155	Frontfork 200mm, black	2	PCS	5707292144935
56	0641620	Button head screw 6*20	4	PCS	5707292139276
57	1102500	Planning washer 10 mm	1	PCS	5707292140234
58	3470400	Hand grip without cover	2	PCS	5707292146663
59	1109500	Starlock washer with cape 6	2	PCS	5707292140449
60	82899	Hand brace holder, left	1	PCS	5707292401397
61	0606000	Insex screw 10*50	2	PCS	5707292138408
62	1103400	Fazet washer 10 mm	2	PCS	5707292140319
63	2502002	Ferrule d18*0,8-2,5 black	2	PCS	5707292433947
64	3004010	12½*2 1/4" br/right, high-low	1	PCS	5707292397508
66	1310628	Distance pcs, 6,4*10*2,8mm	2	PCS	5707292140746
67	82888-60	Carryam, black mother of pearl	1	PCS	5707292459695
68	82838	Linkarm	2	PCS	5707292125392

Position	No.	Description	Quantity	Unit of Measure	Ean code
70	82856-60	Rod for handbrace, right	1	PCS	5707292459862
71	0641625	Button head screw 6*25 mm	2	PCS	5707292139283
72	82855-60	Rod for handbrace, left	1	PCS	5707292459831
73	3652501	R82 handle M6*16mm screw	2	PCS	5707292413963
74	71137	Bush ball for handbrace	2	PCS	5707292361813
75	82858-60	Handle	1	PCS	5707292459879
76	6153500	Knob spring 0,5*6,5 mm.	2	PCS	5707292188571
77	2500563	Grip f/Ø22 handbrace 400mm	2	PCS	5707292612311
78	1108050	Washer M10, din9021	2	PCS	5707292140401
80	1357500	Distance pcs 20*8,4*4mm	2	PCS	5707292141026
81	2704300	Black hub cap for rolko wheel	2	PCS	5707292144515
86	0603700	Insex screw 8*70	2	PCS	5707292138293
88	1310640	Washer, nature Ø16*10*0,5mm	2	PCS	5707292140760
90	82828	Hand brace holder, high-low	1	PCS	5707292125330
93	75626	Left pedal high/low frame	1	PCS	5707292459053
94	1940037	Eco-syn z M4*16 FZB	4	PCS	5707292458650
95	75642	Left holder for release lever	1	PCS	5707292468499
96	75643	Right holder for release lever	1	PCS	5707292468505
97	75637	Holder underpart,f. Udløserarm	2	PCS	5707292459978
98	0640911	Button head M5x10 FZB tuflok	4	PCS	5707292428479
99	75625	Right pedal f. High/low frame	1	PCS	5707292459046
100	0662000	Button head screw 8*20 stainl.	2	PCS	5707292139573
101	75645	Release arm f. Hight/low	1	PCS	5707292459084
101	2220600	Handset w/r82 print & charging	1	PCS	5707292492241
102	0310225	Tension spring	2	PCS	5707292472397
102	1902590	Crosshead screw M3*10, FZB	4	PCS	5707292408389
103	2222110	Batteryboks ba18027	1	PCS	5707292492258
104	1607000	Lock nut 8mm, large pack	2	PCS	5707292141378
105	1103200	Fazet washer 8mm	4	PCS	5707292140296
106	1606250	Self-locking nut 3mm	4	PCS	5707292408396
107	2505000	Ferrule e30*301,0-2,0 int.blac	1	PCS	5707292144140
108	1902760	Cross-headed screw 5*16 mm	4	PCS	5707292142146
109	77576	Battery console highlow	1	PCS	5707292490759
110	1606300	Lock nut 5 mm	4	PCS	5707292141354
111	0603600	Insex screw 8*55	2	PCS	5707292138262
115	2222135	Actuator la28 f/highlow 2	1	PCS	5707292492289
116	82848	Bush	2	PCS	5707292125484
117	0603555	Insex M8*50, 12.9, zinkflage	1	PCS	5707292498892
118	1102900	Fazet washer 5 mm	4	PCS	5707292140272
119	P-6108200	Plast tube pa11 Ø10/Ø8mmx30mm	1	PCS	5707292585424
130	1101000	Center spring washer 8mm	1	PCS	5707292140173
140	0604000	Insex screw 10*20	1	PCS	5707292138361
141	80343	Guide antitip high-L-outd.sz 2	1	PCS	5707292469588
142	0380182	Locking house, complete M10*1	1	PCS	5707292357953
143	1607500	Lock nut 10 mm	1	PCS	5707292141385
144	82375	Pvc-bushing for pelot	2	PCS	5707292123763
145	82483	Brace	1	PCS	5707292470102
99+100	2222161	Charger with plugs	1	PCS	5707292574046


Position	No.	Description	Quantity	Unit of Measure	Ean code
3+8 9+10 20+52 82+83	5076857-60	T-frame, transport, assembled	1	PCS	5707292528209
6+7 30+31 41+42 79+85	5040051	Complete bloc-o-lift f.highlow	1	PCS	5707292397676
5+7 65	5040055	Bloc-o-lift1400n kit-high/low2	1	PCS	5707292460110
6+7 31+85 160-169	89550-TILT- NOU	Kit til mont. tiltudløser,ude	1	PCS	5707292830265

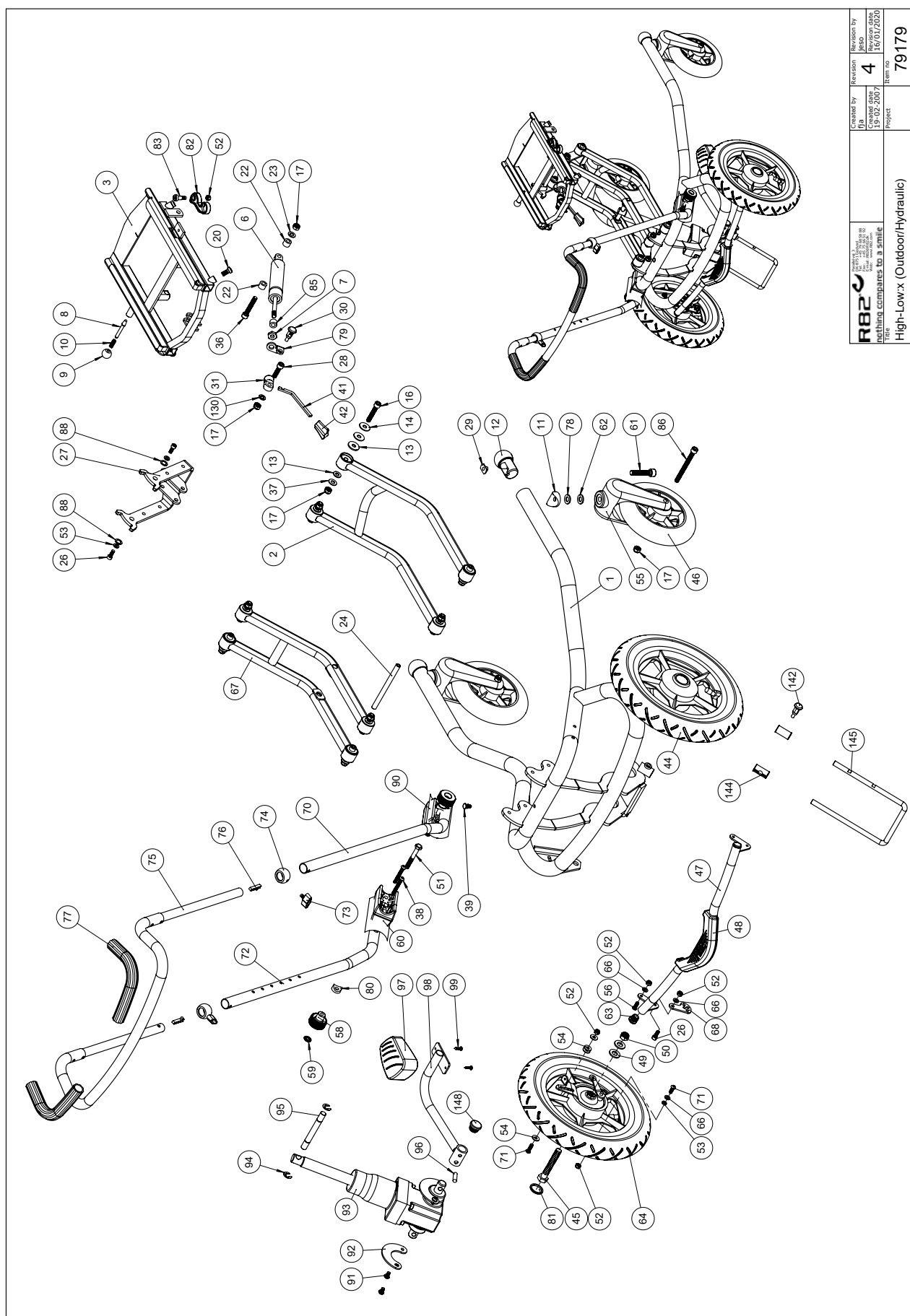

R82 ETAC A/S 8250 Rønde DK-8260 Ålbæk Denmark Tel: +45 79 68 58 88 Fax: +45 79 66 51 92 E-mail: r82@r82.dk Web: www.etac.com	Created by	Revision	Revision by
	fla	4	jebo
	19-03-2007	19-03-2007	19/07/2020
Project		Item no	
High-Low:x (Outdoor/Hydraulic)		79179	

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	75662-60	High/low 2, outdoor, hydraulic	1	PCS	5707292466747
2	82903-60	Carry arm, welded, high-low 2	1	PCS	5707292459404
11	82894	Saddle washer	2	PCS	5707292387578
12+29	5082829	Mounting of knob and weld nut	2	PCS	5707292569790
13	1310705	Nylon washer, white, Ø25*8,4*2,5	16	PCS	5707292140791
14	1108000	Washer 8mm	16	PCS	5707292140388
16	0603505	Insex M8*45, zink flake, 12.9	8	PCS	5707292482846
17	1607000	Lock nut 8mm, large pack	12	PCS	5707292141378
22	1350021	Distance piece 11*15*8,2mm	2	PCS	5707292140890
23	1103200	Fazet washer 8mm	1	PCS	5707292140296
24	82806	Axle	1	PCS	5707292125200
26	0602200	Insex screw 6*16	4	PCS	5707292138095
27	76800-60	Tilt brace mother of pearl	1	PCS	5707292497116
28	0615010	Insex M8*30 fzb, shortened	1	PCS	5707292367990
36	0603555	Insex M8*50, 12.9, zinkflage	1	PCS	5707292498892
37	1102000	Planing washer 8mm	10	PCS	5707292140227
38	0645645	Button head M6*45 zn/black	4	PCS	5707292139351
39	1310508	DR. fastener - afd. prop	1	PCS	5707292829795
44	3003510	12½*2 1/4" br/left, high-low	1	PCS	5707292397492
46	3007910	180*45 black/pneum. Wheel	2	PCS	5707292146045
47	82840	Brake brace	1	PCS	5707292125415
48	82830	Brake pedal, indoor frame	1	PCS	5707292125347
49	1101600	Center spring washer 12mm	4	PCS	5707292140197
50	1604500	Steel nut 12mm, large pack	2	PCS	5707292141309
51	1902320	Steel set screw 8*70mm	2	PCS	5707292142023
52	1606500	Lock nut 6 mm	8	PCS	5707292141361
53	1106000	Washer 6mm	2	PCS	5707292140371
54	1125618	Screenwasher fz6*18*3mm	8	PCS	5707292140494
55	2965155	Frontfork 200mm, black	2	PCS	5707292144935
56	0641620	Button head screw 6*20	4	PCS	5707292139276
58	3470400	Hand grip without cover	2	PCS	5707292146663
59	1109500	Starlock washer with cape 6	2	PCS	5707292140449
60	82899	Hand brace holder, left	1	PCS	5707292401397
61	0606000	Insex screw 10*50	2	PCS	5707292138408
62	1103400	Fazet washer 10 mm	2	PCS	5707292140319
63	2502002	Ferrule d18*0,8-2,5 black	2	PCS	5707292433947
64	3004010	12½*2 1/4" br/right, high-low	1	PCS	5707292397508
66	1310628	Distance pcs, 6,4*10*2,8mm	2	PCS	5707292140746
67	75652-60	Carryarm, hydraulic	1	PCS	5707292470614
68	82838	Linkarm	2	PCS	5707292125392
70	82856-60	Rod for handbrace, right	1	PCS	5707292459862
71	0641625	Button head screw 6*25 mm	2	PCS	5707292139283
72	82855-60	Rod for handbrace, left	1	PCS	5707292459831
73	3652501	R82 handle M6*16mm screw	2	PCS	5707292413963
74	71137	Bush ball for handbrace	2	PCS	5707292361813
75	82858-60	Handle	1	PCS	5707292459879
76	6153500	Knob spring 0,5*6,5 mm.	2	PCS	5707292188571
77	2500563	Grip f/Ø22 handbrace 400mm	2	PCS	5707292612311
78	1108050	Washer M10, din9021	2	PCS	5707292140401


Position	No.	Description	Quantity	Unit of Measure	Ean code
80	1357500	Distance pcs 20*8,4*4mm	2	PCS	5707292141026
81	2704300	Black hub cap for rolko wheel	2	PCS	5707292144515
86	0603700	Insex screw 8*70	2	PCS	5707292138293
88	1310640	Washer, nature Ø16*10*0,5mm	2	PCS	5707292140760
90	82828	Hand brace holder, high-low	1	PCS	5707292125330
91	0641200	Button head screw 6*12	4	PCS	5707292139252
92	75615	Bracket for hydraulic	2	PCS	5707292460202
93	2208000	Compact mk5 hydraulic pump	1	PCS	5707292461148
94	1145095	lock ring for 10mm, din 6799, rf	2	PCS	5707292473332
95	75620	Stift for liftarm	1	PCS	5707292460226
96	0584490	Stift din 1481 8*30 rf	1	PCS	5707292473356
97	75625	Right pedal f. High/low frame	1	PCS	5707292459046
98	75653-60	Arm t. Hydraulik, sort perlemor	1	PCS	5707292470621
99	1940037	Eco-syn z M4*16 FZB	2	PCS	5707292458650
3+8 9+10 20+52 82+83	5076857-60	T-frame, transport, assembled	1	PCS	5707292528209
6+7 30+31 41+42 79+85	5040051	Complete bloc-o-lift f. highlow	1	PCS	5707292397676

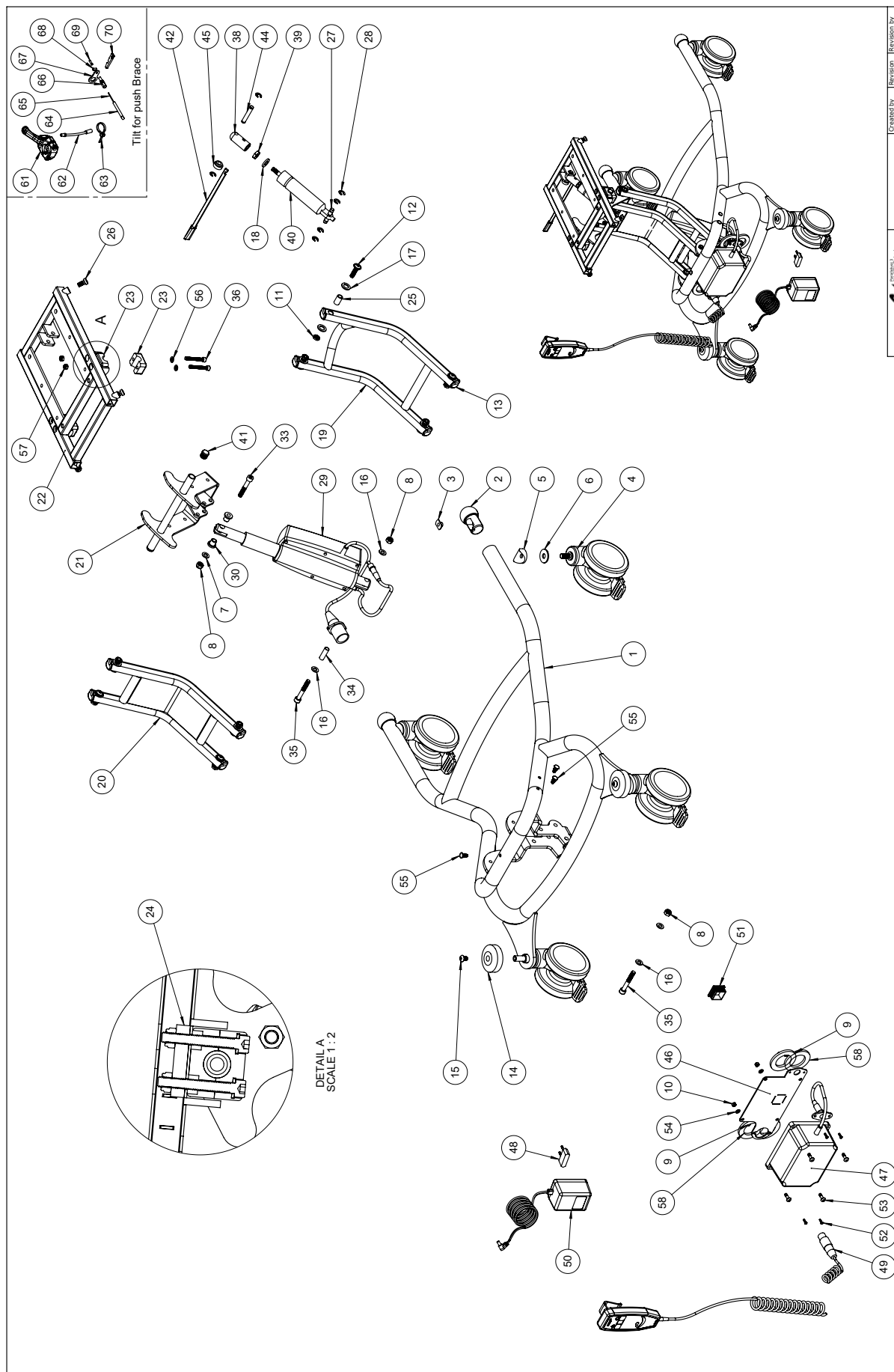

 R82 Indoor Frame, Electr. Actuator nothing compares to a smile www.hormann.com	Created by	Revision	Revision by
	JLB	5	JLB
	Created date 26-01-2007		Revision date 02/04/2020
Title	Project	Item no	
Indoor Frame, Electr. Actuator		79174	

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	75670-60	Frame high/low x 3	1	PCS	5707292489715
2+3	5082829	Mounting of knob and weld nut	2	PCS	5707292569790
4	3006938	100mm double wheel with brake	4	PCS	5707292574930
5	82894	Saddle washer	2	PCS	5707292387578
6	1108050	Washer M10, din9021	2	PCS	5707292140401
7	1103200	Fazet washer 8mm	1	PCS	5707292140296
8	1607000	Lock nut 8mm, large pack	3	PCS	5707292141378
9	1606250	Self-locking nut 3mm	4	PCS	5707292408396
10	1606300	Lock nut 5 mm	4	PCS	5707292141354
11	1750800	Lock nut, stainless M8	8	PCS	5707292141484
12	0663031	Button head 8*30, Wombat	8	PCS	5707292139610
13	82946	End piece 15*30, Wombat	8	PCS	5707292126016
14	2522100	Black bumpwheel for Pony 60mm	2	PCS	5707292144478
15	0661300	button head with flange 8*12	2	PCS	5707292408068
16	1102000	Planing washer 8mm	4	PCS	5707292140227
17	1310710	Nylon washer	16	PCS	5707292140807
18	1310712	Nylon washer, nature	1	PCS	5707292356345
19	82790-60	Carryarm black mother of pearl	1	PCS	5707292434609
20	77564-60	Carryarm, highlow 2, painted	1	PCS	5707292489838
21	77562-60	Tipbracket, seatunit	1	PCS	5707292489791
22	77572-60	Seat frame highlow 2	1	PCS	5707292490827
23	2521840	Pipe holder f/16mmsquare,black	4	PCS	5707292399236
24	77571	Fastening bracket	2	PCS	5707292489982
25	82933	Bearing bushing, Wombat	8	PCS	5707292125880
26	0603530	Insex low head M8x16 stainless	4	PCS	5707292412492
27	80924	Ø8*40 spec locking pin,wombat	1	PCS	5707292111401
28	1145070	Lock washer 7mm	6	PCS	5707292140586
29	2222138	Actuator LA32 til Panda 5	1	PCS	5707292589354
30	82848	Bush	2	PCS	5707292125484
33	0603555	Insex M8*50, 12.9, zinkflage	1	PCS	5707292498892
34	P-6108200	Plast tube pa11 Ø10/Ø8mmx30mm	1	PCS	5707292585424
35	0603610	Insex M8*55, 12.9 flake zinc	2	PCS	5707292517630
36	0602445	Insex M6*45 FZB	4	PCS	5707292138156
38	6118821	Horizontal release, Wombat	1	PCS	5707292186737
39	1761000	Counter nut 10*1,0 fine thread	1	PCS	5707292141538
40	2201500-1	Bloc-o-lift 549496mp 500n fast	1	PCS	5707292839879
41	2501600	Ferrule d15*1,0 black	2	PCS	5707292143969
42	2200021	Arm for heightadjustment	1	PCS	5707292143044
44	80921	locking pin, Wombat	1	PCS	5707292111371
45	77574	Sleeve f. Strike	1	PCS	5707292490773
46	77576	Battery console highlow	1	PCS	5707292490759
47	2222110	Batteryboks ba18027	1	PCS	5707292492258
49	2220600	Handset w/r82 print & charging	1	PCS	5707292492241
48+50	2222161	Charger with plugs	1	PCS	5707292574046
51	2505000	Ferrule e30*301,0-2,0 int.blac	1	PCS	5707292144140
52	1902590	Crosshead screw M3*10, FZB	4	PCS	5707292408389
53	1902760	Cross-headed screw 5*16 mm	4	PCS	5707292142146
54	1102900	Fazet washer 5 mm	4	PCS	5707292140272
55	1310508	DR. fastener - afd. prop	5	PCS	5707292829795

Position	No.	Description	Quantity	Unit of Measure	Ean code
56	1105060	Stainless washer 6mm	4	PCS	5707292140357
57	1606500	Lock nut 6 mm	4	PCS	5707292141361
58	1102800	Fazet washer 4mm	4	PCS	5707292140265
61	3503531	Gas pump lever with clamp	1	PCS	5707292413529
62	0380060	Flexible v-brake pipe.	1	PCS	5707292425195
63	0235075	Cable tape, black	1	PCS	5707292135308
64	5002750	Cable for brake, size 1+2	1	PCS	5707292413635
65	0371853	Strained wires 1,5mm 190 cm.	1	PCS	5707292135889
66	0380000	Cable tighter M5*23	1	PCS	5707292135933
67	82268	Wire mounting by pedal	1	PCS	5707292123039
68	0388006	Screwnipple 5*7,5mm	1	PCS	5707292136114
69	0387000	End nipple for outer casing	1	PCS	5707292136091
70	2200050	Small release arm	1	PCS	5707292143075

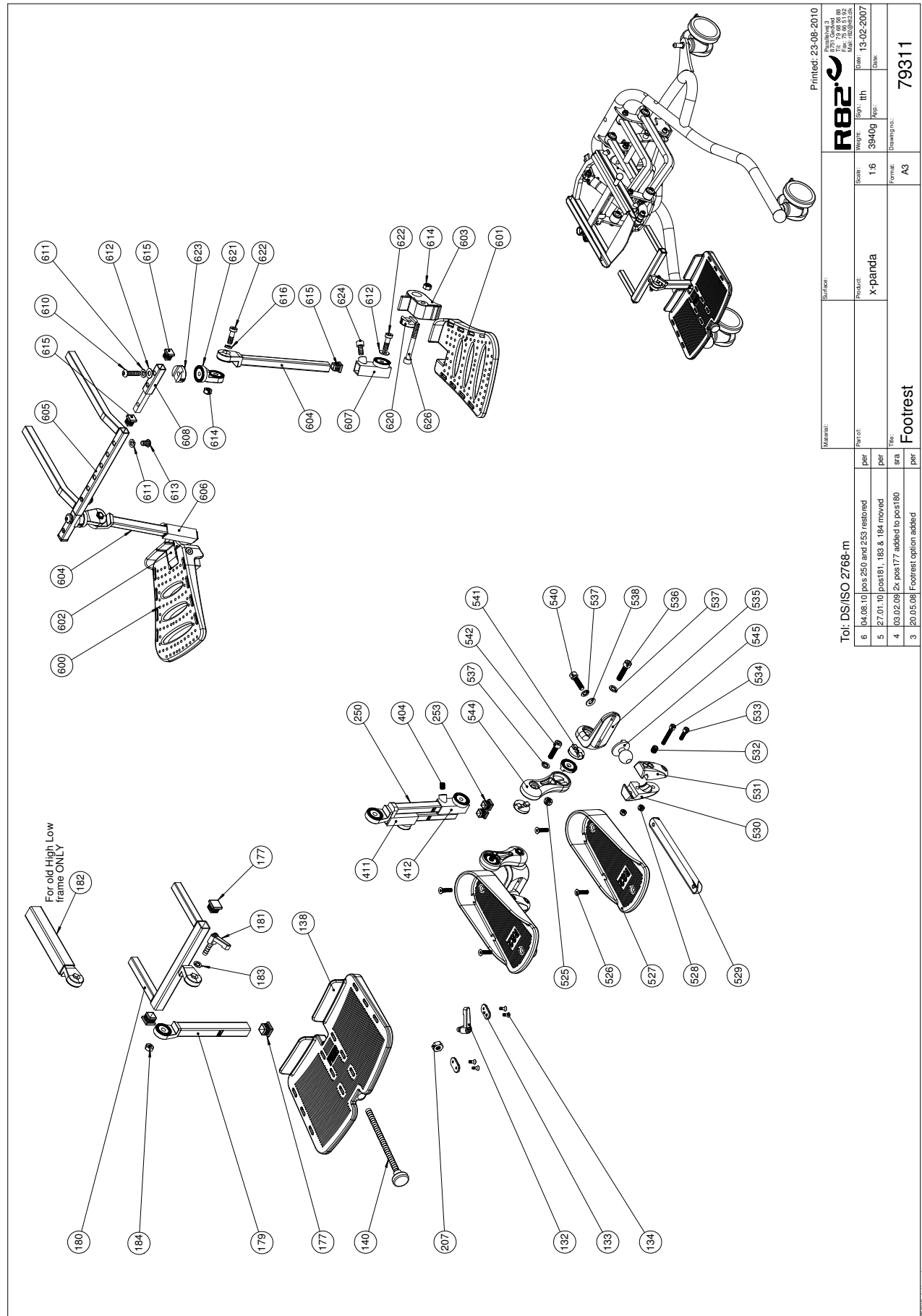

Material:		Surface:		Product:		Scale:		Project:		Date:	
				x-panda		1:6		39409		13-02-2007	
				Footrest		A3		79311			

Part of:		Part of:		Part of:		Part of:		Part of:		Part of:	
per		per		per		per		per		per	
ser		ser		ser		ser		ser		ser	
per		per		per		per		per		per	

Total: DS/ISO 2768-m		Total: DS/ISO 2768-m		Total: DS/ISO 2768-m		Total: DS/ISO 2768-m		Total: DS/ISO 2768-m		Total: DS/ISO 2768-m	
6		5		4		3		2		1	
[04.08.10] pos 250 and 253 restored		[27.01.10] pos 181, 183 & 184 moved		[03.02.09] 2x pos 177 added to pos 180		[20.05.08] Footrest option added					

Position	No.	Description	Quantity	Unit of Measure	Ean code
132	3502516	TIGHTENING HANDLE 45MM M8*16	1	PCS	5707292146915
133	82950	HOLLOW PLATE 16*32, WOMBAT	2	PCS	5707292126030
134	1940017	SHEET METAL SCREW UHX4,2*13FZB	4	PCS	5707292142597
138	82944	FOOTPLATE SIZE 1, WOMBAT	1	PCS	5707292125996
138	82940	FODPLADE STR.2, WOMBAT	1	PCS	5707292125958
140	3502780	HANDLE 28MM,M10*160MM THREAD	1	PCS	5707292363145
140	3502790	HAND GRIP 28MM, M10*196MM	1	PCS	5707292363152
177	2504500	FERRULE E20*1,0-2,0 INT.BLACK	5	PCS	5707292144126
179	75260	Rod size 0 for footrest	1	PCS	5707292458346
179	75262	Rod size 2 for footrest	1	PCS	5707292458353
179	75263	Rod size 3 for footrest	1	PCS	5707292458360
180	75270	HOLDER SIZE 1 FOR FOOTREST	1	PCS	5707292455598
180	75272	HOLDER SIZE 2 FOR FOOTREST	1	PCS	5707292458384
181	3504025	TIGHTENING HANDLE 45MM M8*25	1	PCS	5707292147103
182*	75266	HOLDER SIZE 1 FOR FOOTREST	1	PCS	5707292455611
	75267	HOLDER SIZE 2 FOR FOOTREST	1	PCS	5707292455604
183	1101000	CENTER SPRING WASHER 8MM	1	PCS	5707292140173
184	1603500	STEEL NUT 8MM, LARGE PACK	1	PCS	5707292141286
207	1607500	Lock nut 10 mm	1	PCS	5707292141385
250*	75285	Rod dobblet size 0	1	PCS	5707292458445
	75287	HOLDER SIZE 2,DOUBBLE FOOTREST	1	PCS	5707292458469
	75288	HOLDER SIZE 3,DOUBBLE FOOTREST	1	PCS	5707292458476
253	2504000	FERRULE E15*1,0-2,0 INT.BLACK	2	PCS	5707292144102
404	0701000	Stop screw with crater 8*10	2	PCS	5707292139870
411	73734	Mounting for footrest,right	1	PCS	5707292433466
412	73735	Mounting for footrest,left	1	PCS	5707292433473
525	1607000	Lock nut 8mm, large pack	2	PCS	5707292141378
526	0612500	Insex screw counters. 6*25	4	PCS	5707292138712
527	82156-60	Footplate size 1,black,gazelle	2	PCS	5707292230492
527	82154-60	Footplate size 2,black,gazelle	2	PCS	5707292230478
527	82158-60	Footplate size 3 black,gazelle	2	PCS	5707292230508
528	1606500	Lock nut 6 mm	4	PCS	5707292141361
529	73648	Stick size 1, footrest	2	PCS	5707292445797
529	73649	Stick size 2, footrest	2	PCS	5707292445803
530	72724	Ball joining right	2	PCS	5707292421845
531	72723	Ball joining left	2	PCS	5707292421814
532	0701010	Set screw M10*10 w/flat end	2	PCS	5707292434098
533	0602300	Insex screw 6*20	2	PCS	5707292138101
534	0602440	Insex M6*40 FZB	2	PCS	5707292138149
535	72713	Bracked	2	PCS	5707292420763
536	0603000	Insex screw 8*35	2	PCS	5707292138217
537	1101000	Center spring washer 8mm	6	PCS	5707292140173
538	1102010	Face plate 8,4*18*2mm, FZB	2	PCS	5707292429766
540	0602700	Insex screw 8*25	2	PCS	5707292138187
541	74062	Toothed disk 30*14	6	PCS	5707292377951
542	0602850	Insex M8*22, FZB	2	PCS	5707292435408
544	72712	Knee joint	2	PCS	5707292420756
545	72721	Ballhead	2	PCS	5707292428875
600	74052	Foot plate 1, right	1	PCS	5707292373748

601	74053	Foot plate 1, left	1	PCS	5707292373755
602	74044	Mounting for foot plate, right	1	PCS	5707292373687
603	74045	Mounting for foot plate, left	1	PCS	5707292373694
604	75317	Rod for footrest size 0, 21 cm	2	PCS	5707292477774
604	75319	Rod for footrest size 2, 34 cm	2	PCS	5707292477804
604	75325	Rod for footrest size 3, 38 cm	2	PCS	5707292477811
605	75327	Holder for footrest size 1	1	PCS	5707292477545
605	75312	Holder for footrest size 2	1	PCS	5707292477538
606	73734	Mounting for footrest, right	1	PCS	5707292433466
606	73735	Mounting for footrest, left	1	PCS	5707292433473
607	73735	Mounting for footrest, left	1	PCS	5707292433473
607	73734	Mounting for footrest, right	1	PCS	5707292433466
610	0663000	Button head screw 8*30 stainl.	2	PCS	5707292139597
611	1100500	Spring washer 8mm, stainless	2	PCS	5707292140142
612	1102000	Planing washer 8mm	4	PCS	5707292140227
614	1607000	Lock nut 8mm, large pack	4	PCS	5707292141378
615	2504000	Ferrule e15*1,0-2,0 int.black	4	PCS	5707292144102
616	1100495	Spring washer M8	2	PCS	5707292458674
620	74061	Tooth washer Ø30	2	PCS	5707292373816
621	75315	Rotary joint	2	PCS	5707292477927
622	0603535	Insex low head M8*25	4	PCS	5707292575319
623	75329	Tooth disk Ø30x10	2	PCS	5707292482389
624	0603530	Insex low head M8x16 stainless	2	PCS	5707292412492
626	0603700	Insex screw 8*70	2	PCS	5707292138293