

# high-low

## Exploded views / Part lists


Rev. date (03-2021)

*On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.*



## Contents


|                                  |    |
|----------------------------------|----|
| High-low indoor frame .....      | 3  |
| High-low outdoor frame.....      | 6  |
| High-low indoor with power ..... | 10 |

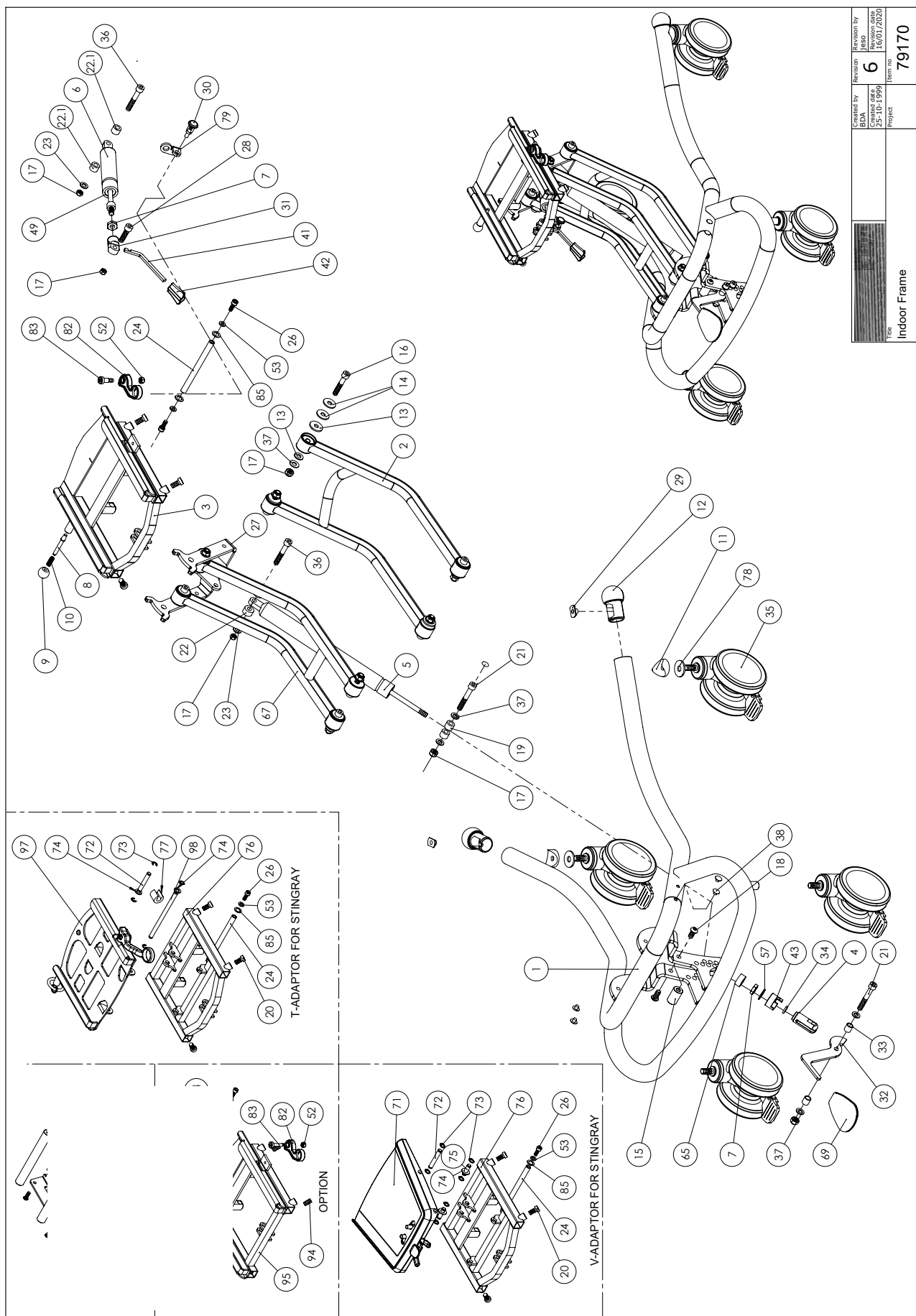

| Position | No.      | Description                        | Quantity | Unit of Measure | Ean code      |
|----------|----------|------------------------------------|----------|-----------------|---------------|
| 1*       | 82820-60 | FRAME, MOTHER OF PEARL BLACK       | 1        | PCS             |               |
|          | 82820-01 | FRAME, BLACK                       | 1        | PCS             | 5707292231659 |
|          | 82820-20 | FRAME, BLUE                        | 1        | PCS             | 5707292231666 |
|          | 82820-21 | FRAME, GREEN                       | 1        | PCS             | 5707292231673 |
| 2*       | 82822-60 | ARM FOR SUPPORT HIGHLOW, MOP.BLACK | 1        | PCS             |               |
|          | 82822-01 | ARM FOR SUPPORT HIGHLOW, BLACK     | 2        | PCS             | 5707292231697 |
|          | 82822-20 | ARM FOR SUPPORT HIGHLOW, BLUE      | 2        | PCS             | 5707292231703 |
|          | 82822-21 | ARM FOR SUPPORT HIGHLOW, GREEN     | 2        | PCS             | 5707292231710 |
| 3*       | 82825-60 | T-FRAME, MOTHER OF PEARL BLACK     | 1        | PCS             |               |
|          | 82825-01 | T-FRAME, BLACK                     | 1        | PCS             | 5707292231772 |
|          | 82825-20 | T-FRAME, BLUE                      | 1        | PCS             | 5707292231789 |
|          | 82825-21 | T-FRAME, GREEN                     | 1        | PCS             | 5707292231796 |
| 4        | 6118822  | HORIZONTAL RELEASE                 | 1        | PCS             | 5707292186744 |
| 5        | 5040050  | GASSPRING FOR HIGH LOW             | 1        | PCS             | 5707292397669 |
| 6        | 2201500  | BLOC-O-LIFT 9127MP 500 N           | 1        | PCS             | 5707292143181 |
| 7        | 1761000  | COUNTER NUT 10*1,0 FINE THREAD     | 1        | PCS             | 5707292141538 |
| 8        | 6113000  | SPINDLE, T-PART/FRAME              | 1        | PCS             | 5707292186362 |
| 9        | 3502008  | KNOB HANDLE Ø20 M6                 | 1        | PCS             | 5707292146830 |
| 10       | 0310100  | PRESSURESPRING 25 MM F.SPINDEL     | 1        | PCS             | 5707292135513 |
| 11       | 82894    | WASHER                             | 2        | PCS             | 5707292387578 |
| 12       | 82829-30 | BALL Ø40, GREEN                    | 2        | PCS             | 5707292231857 |
| 13       | 1310705  | NYLON WASHER, WHITE, Ø25*8,4*2,5   | 16       | PCS             | 5707292140791 |
| 14       | 1108000  | WASHER 8MM                         | 16       | PCS             | 5707292140388 |
| 15       | 82808    | COLLECTING BUSH                    | 1        | PCS             | 5707292125224 |
| 16       | 0603500  | INSEX SCREW 8*45                   | 8        | PCS             | 5707292138231 |
| 17       | 1607000  | LOCK NUT 8MM, LARGE PACK           | 13       | PCS             | 5707292141378 |
| 18       | 0661500  | BUTTON HEAD SCREW 8*16 STAINL.     | 2        | PCS             | 5707292139559 |
| 19       | 1350000  | DISTANCEPART BLACK 6*15*8,2 MM     | 2        | PCS             | 5707292140876 |
| 20       | 3502516  | TIGHTENING HANDLE 45MM M8*16       | 1        | PCS             | 5707292146915 |
| 21       | 0603600  | INSEX SCREW 8*55                   | 3        | PCS             | 5707292138262 |
| 22       | 1350100  | DISTANCE PART BLACK 8,2*18*10      | 4        | PCS             | 5707292140937 |
| 23       | 1103200  | FAZET WASHER 8MM                   | 4        | PCS             | 5707292140296 |
| 24       | 82806    | AXLE                               | 1        | PCS             | 5707292125200 |
| 26       | 0602200  | INSEX SCREW 6*16                   | 2        | PCS             | 5707292138095 |
| 27       | 76800-60 | BRACE FOR TILT, BLACK              | 1        | PCS             | 5707292497116 |
| 28       | 0615010  | INSEX M8x30 FZB SHORTENED          | 1        | PCS             | 5707292367990 |
| 29       | 1600525  | STAINL. WELDING NUT W/FLANGE       | 2        | PCS             | 5707292141149 |
| 30       | 0380184  | SPEC. LOCKING HOUSE M10*1, 28MM    | 1        | PCS             | 5707292366030 |
| 31       | 155775   | RELEASE FOR GAS SPRING             | 1        | PCS             | 5707292928733 |
| 32       | 82851    | ARM FOR FOOT PEDAL                 | 1        | PCS             | 5707292125507 |
| 33       | 82859    | BUSH FOR FOOT PEDAL                | 2        | PCS             | 5707292125552 |
| 34       | 0108000  | O-RINGS O13*2,5 QUALITY XP 70      | 1        | PCS             | 5707292135124 |
| 35       | 3006937  | 100MM DOUBLE WHEEL WITH BRAKE      | 4        | PCS             | 5707292548030 |
| 36       | 0603550  | INSEX SCREW 8*50                   | 2        | PCS             | 5707292138255 |
| 37       | 1102000  | PLANING WASHER 8MM                 | 16       | PCS             | 5707292140227 |
| 38       | 1310540  | COVERING PUSHBUTTON, BLACK         | 4        | PCS             | 5707292140692 |
| 40       | 2504000  | FERRULE E15*1,0-2,0 INT.BLACK      | 4        | PCS             | 5707292144102 |
| 41       | 2200013  | ARM F. RELEASE, F.INDOOR FRAME     | 1        | PCS             | 5707292142993 |
| 42       | 2200030  | BUTTON FOR RELEASE ARM             | 1        | PCS             | 5707292143051 |

|     |          |                                  |   |     |               |
|-----|----------|----------------------------------|---|-----|---------------|
| 43  | 82866    | LOCK FOR RELEASE HEAD            | 1 | PCS | 5707292125606 |
| 49  | 1310712  | NYLON WASHER, NATURE             | 1 | PCS | 5707292356345 |
| 52  | 1606500  | LOCK NUT 6 MM                    | 1 | PCS | 5707292141361 |
| 53  | 1103000  | FAZET WASHER 6MM                 | 2 | PCS | 5707292140289 |
| 57  | 1102500  | PLANNING WASHER 10 MM            | 1 | PCS | 5707292140234 |
| 65  | 1310630  | DISTANCE PIECE, Ø10,3*22*10MM    | 1 | PCS | 5707292140753 |
| 67* | 82888-60 | CARRYING ARM HIGH/LOW, MOP.BLACK | 1 | PCS |               |
|     | 82888-01 | CARRYING ARM HIGH/LOW, BLACK     | 1 | PCS | 5707292369369 |
|     | 82888-20 | CARRYING ARM HIGH/LOW, BLUE      | 1 | PCS | 5707292369376 |
|     | 82888-21 | CARRYING ARM HIGH/LOW, GREEN     | 1 | PCS | 5707292369383 |
| 69  | 3470335  | RELEASE BUTTON Ø8, 5*8MM         | 1 | PCS | 5707292146632 |
| 78  | 1108050  | WASHER M10, DIN9021              | 2 | PCS | 5707292140401 |
| 79  | 82481    | SAFETY MOUNTING FOR GASSPRING    | 1 | PCS | 5707292124500 |
| 82  | 82880    | HANDLE F/ADAPTOR T/HIGH-LOW      | 1 | PCS | 5707292361318 |
| 83  | 1106000  | WASHER 6MM                       | 2 | PCS | 5707292140371 |
| 85  | 1310640  | WASHER, NATURE Ø16*10*0,5MM      | 2 | PCS | 5707292140760 |
| 88  | 1310640  | WASHER, NATURE Ø16*10*0,5MM      | 4 | PCS | 5707292140760 |
| 91  | 0683803  | PASBOLT M/INDV.6KT. 10*25 (M8)   | 2 | PCS | 5707292460066 |
| 92  | 1607000  | LOCK NUT 8MM, LARGE PACK         | 2 | PCS | 5707292141378 |
| 93  | 1102000  | PLANING WASHER 8MM               | 2 | PCS | 5707292140227 |
| 94  | 0603540  | INSEX LOW HEAD M8X40 STAINLESS   | 4 | PCS | 5707292412485 |
| 95  | 75600-60 | T-FRAME, X:PANDA 1+2             | 1 | PCS | 5707292459541 |
| 130 | 1101000  | CENTER SPRING WASHER 8MM         | 1 | PCS | 5707292140173 |

|   |  |               |
|---|--|---------------|
| * |  | CHOOSE COLOUR |
|---|--|---------------|

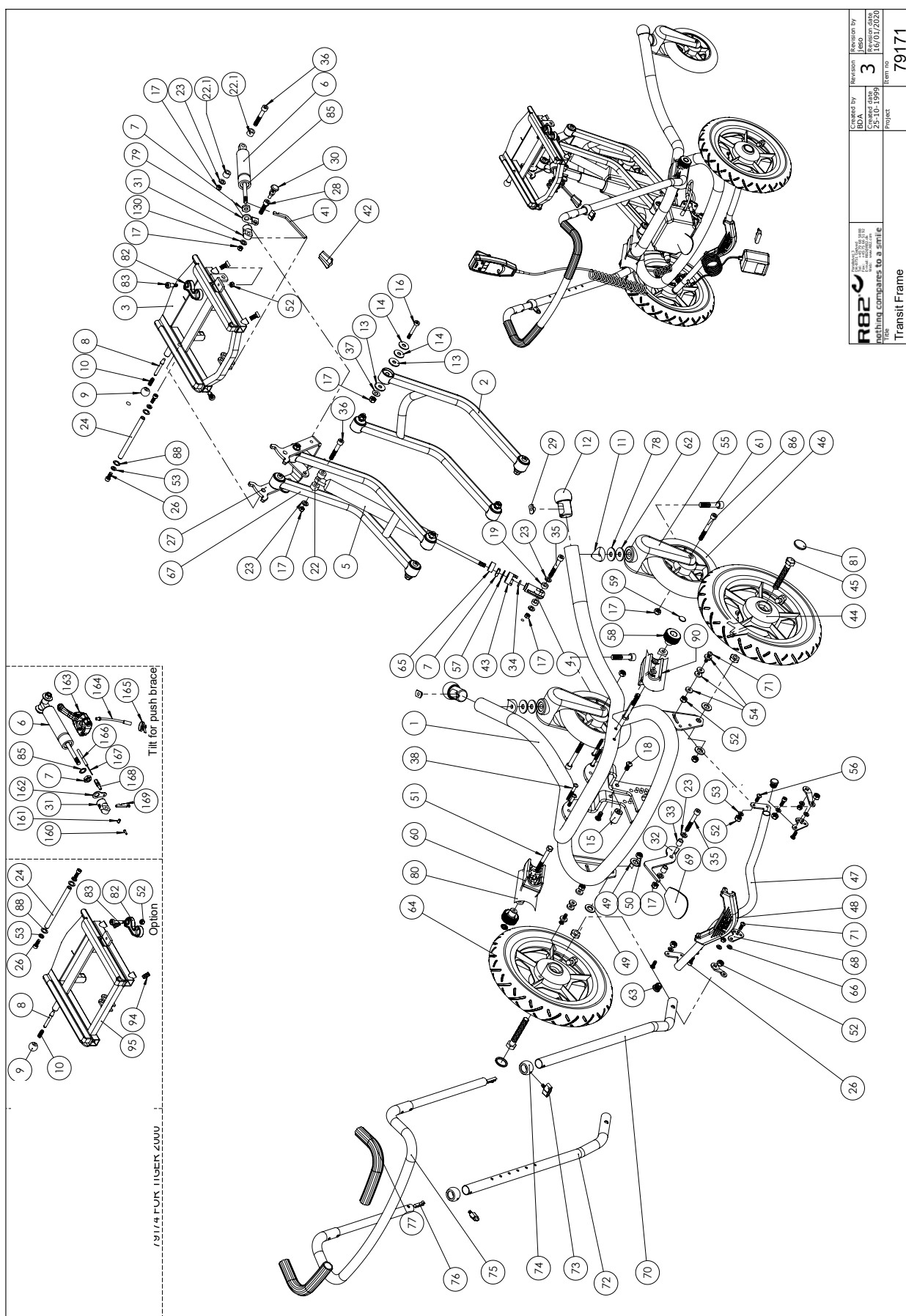

|                                                                                                                       |                                                                              |                   |               |                     |                             |
|-----------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------|-------------------|---------------|---------------------|-----------------------------|
| <br>Nothing<br>comparing to a smic | Approved by<br>Date<br>25-10-1999<br>For<br>25-10-1999<br>From<br>25-10-1999 | Created by<br>BDA | Revision<br>3 | Revision by<br>Jaso | Revision date<br>16/01/2020 |
|                                                                                                                       | Project                                                                      |                   |               |                     |                             |
|                                                                                                                       | Item no                                                                      |                   |               |                     |                             |
| Transit Frame                                                                                                         |                                                                              |                   |               |                     |                             |
| 79171                                                                                                                 |                                                                              |                   |               |                     |                             |


| Position | No.      | Description                         | Quantity | Unit of Measure | Ean code      |
|----------|----------|-------------------------------------|----------|-----------------|---------------|
| 1*       | 82656-60 | TRANSITFRAME, MOTHER OF PEARL BLACK | 1        | PCS             |               |
|          | 82656-01 | TRANSITFRAME, BLACK                 | 1        | PCS             | 5707292397690 |
|          | 82656-20 | TRANSITFRAME, BLUE                  | 1        | PCS             | 5707292397706 |
|          | 82656-21 | TRANSITFRAME, GREEN                 | 1        | PCS             | 5707292397713 |
| 2*       | 82822-60 | ARM FOR SUPPORT HIGHLOW, MOP.BLACK  | 1        | PCS             |               |
|          | 82822-01 | ARM FOR SUPPORT HIGHLOW, BLACK      | 2        | PCS             | 5707292231697 |
|          | 82822-20 | ARM FOR SUPPORT HIGHLOW, BLUE       | 2        | PCS             | 5707292231703 |
|          | 82822-21 | ARM FOR SUPPORT HIGHLOW, GREEN      | 2        | PCS             | 5707292231710 |
| 4        | 6118822  | HORIZONTAL RELEASE                  | 1        | PCS             | 5707292186744 |
| 5        | 5040050  | GASSPRING FOR HIGH LOW              | 1        | PCS             | 5707292397669 |
| 7        | 1761000  | COUNTER NUT 10*1,0 FINE THREAD      | 1        | PCS             | 5707292141538 |
| 8        | 6113000  | SPINDLE, T-PART/FRAME               | 1        | PCS             | 5707292186362 |
| 9        | 3502008  | KNOB HANDLE Ø20 M6                  | 1        | PCS             | 5707292146830 |
| 10       | 0310100  | PRESSURESPRING 0,9*8,23*10,5V       | 1        | PCS             | 5707292135513 |
| 11       | 82894    | WASHER                              | 2        | PCS             | 5707292387578 |
| 12       | 82829-30 | BALL Ø40, GREEN                     | 2        | PCS             | 5707292231857 |
| 13       | 1310705  | NYLON WASHER, WHITE, Ø25*8,4*2,5    | 16       | PCS             | 5707292140791 |
| 14       | 1108000  | WASHER 8MM                          | 16       | PCS             | 5707292140388 |
| 15       | 82808    | COLLECTING BUSH                     | 1        | PCS             | 5707292125224 |
| 16       | 0603500  | INSEX SCREW 8*45                    | 8        | PCS             | 5707292138231 |
| 17       | 1607000  | LOCK NUT 8MM, LARGE PACK            | 16       | PCS             | 5707292141378 |
| 18       | 0661500  | BUTTON HEAD SCREW 8*16 STAINL.      | 2        | PCS             | 5707292139559 |
| 19       | 1350000  | DISTANCEPART BLACK 6*15*8,2 MM      | 2        | PCS             | 5707292140876 |
| 22       | 1350100  | DISTANCE PART BLACK 8,2*18*10       | 2        | PCS             | 5707292140937 |
| 23       | 1103200  | FAZET WASHER 8MM                    | 4        | PCS             | 5707292140296 |
| 26       | 0602200  | INSEX SCREW 6*16                    | 4        | PCS             | 5707292138095 |
| 27       | 76800-60 | BRACE FOR TILT, BLACK               | 1        | PCS             | 5707292497116 |
| 29       | 1600525  | STAINL. WELDING NUT W/FLANGE        | 2        | PCS             | 5707292141149 |
| 31       | 155775   | RELEASE FOR GAS SPRING              | 1        | PCS             | 5707292928733 |
| 32       | 82851    | ARM FOR FOOT PEDAL                  | 1        | PCS             | 5707292125507 |
| 33       | 82859    | BUSH FOR PEDAL                      | 2        | PCS             | 5707292125552 |
| 34       | 0108000  | O-RINGS O13*2,5 QUALITY XP 70       | 1        | PCS             | 5707292135124 |
| 35       | 0603600  | INSEX SCREW 8*55                    | 1        | PCS             | 5707292138262 |
| 36       | 0603550  | INSEX SCREW 8*50                    | 2        | PCS             | 5707292138255 |
| 37       | 1102000  | PLANING WASHER 8MM                  | 14       | PCS             | 5707292140227 |
| 38       | 0645645  | BUTTON HEAD M6*45 ZN/BLACK          | 4        | PCS             | 5707292139351 |
| 43       | 82866    | LOCK FOR RELEASE HEAD               | 1        | PCS             | 5707292125606 |
| 40       | 2504000  | FERRULE E15*1,0+2,0 INT.BLACK       | 4        | PCS             | 5707292144102 |
| 44*      | 3003510  | 12½*2 1/4" BRAKE/RIGHT, HIGH-LOW    | 1        | PCS             | 5707292397492 |
|          | 3004010  | 12½*2 1/4" BRAKE/LEFT, HIGH-LOW     | 1        | PCS             | 5707292397508 |
|          | 3003510  | 12½*2 1/4" BRAKE/RIGHT, HIGH-LOW    | 1        | PCS             | 5707292397492 |
|          | 3002500  | 12½*2 1/4 BRAKE/LEFT, HIGH LOW      | 1        | PCS             | 5707292145642 |
| 45       | 1902380  | STEEL SET SCREW 12*80MM             | 2        | PCS             | 5707292142030 |
| 46       | 3007910  | 175*40 BLACK/PNEUM. WHEEL           | 2        | PCS             | 5707292146045 |
| 47       | 82840    | BRAKE BRACE                         | 1        | PCS             | 5707292125415 |
| 48       | 82830    | BRAKEPEDAL PANDA INDOORFRAME        | 1        | PCS             | 5707292125347 |
| 49       | 1310712  | NYLON WASHER, NATURE                | 1        | PCS             | 5707292356345 |
| 50       | 1604500  | STEEL NUT 12MM, LARGE PACK          | 2        | PCS             | 5707292141309 |
| 51       | 1902320  | STEEL SET SCREW M8*70, FZB          | 2        | PCS             | 5707292142023 |



|     |          |                                      |   |     |               |
|-----|----------|--------------------------------------|---|-----|---------------|
| 52  | 1606500  | LOCK NUT 6 MM                        | 2 | PCS | 5707292141361 |
| 53  | 1106000  | WASHER 6MM                           | 2 | PCS | 5707292140371 |
| 54  | 1125618  | SCREENWASHER FZ6*18*3MM              | 8 | PCS | 5707292140494 |
| 55  | 2965155  | FRONTFORK 200MM, BLACK               | 2 | PCS | 5707292144935 |
| 56  | 0641620  | BUTTON HEAD SCREW 6*20               | 4 | PCS | 5707292139276 |
| 57  | 1102500  | PLANNING WASHER 10 MM                | 1 | PCS | 5707292140234 |
| 58  | 3470400  | HAND GRIP WITHOUT COVER              | 2 | PCS | 5707292146663 |
| 59  | 1109500  | STARLOCK SKIVE M/KAPPE Ø6            | 2 | PCS | 5707292140449 |
| 60  | 82828    | HAND BRACE HOLDER, HIGH-LOW          | 1 | PCS | 5707292125330 |
| 61* | 0606000  | INSEX SCREW 10*50                    | 2 | PCS | 5707292138408 |
|     | 1101600  | CENTER SPRING WASHER 12MM            | 4 | PCS | 5707292140197 |
| 62  | 1103400  | FAZET WASHER 10 MM                   | 2 | PCS | 5707292140319 |
| 63  | 2502002  | FERRULE D18*0,8-2,5 BLACK            | 2 | PCS | 5707292433947 |
| 64* | 3004010  | 12½*2 1/4" BRAKE/LEFT, HIGH-LOW      | 1 | PCS | 5707292397508 |
|     | 3003510  | 12½*2 1/4" BRAKE/RIGHT, HIGH-LOW     | 1 | PCS | 5707292397492 |
|     | 3004010  | 12½*2 1/4" BRAKE/LEFT, HIGH-LOW      | 1 | PCS | 5707292397508 |
|     | 3003000  | 12½*2 1/4" BRAKE/RIGHT, HIGH-LOW     | 1 | PCS | 5707292145659 |
| 65  | 1310630  | DISTANCE PIECE, Ø10,3*22*10MM        | 1 | PCS | 5707292140753 |
| 66  | 1310628  | DISTANCE PCS NATUREL Ø6,4*10*2,8MM   | 2 | PCS | 5707292140746 |
| 67* | 82888-60 | CARRYING ARM HIGH/LOW, MOP.BLACK     | 1 | PCS |               |
|     | 82888-01 | CARRYING ARM HIGH/LOW, BLACK         | 1 | PCS | 5707292369369 |
|     | 82888-20 | CARRYING ARM HIGH/LOW, BLUE          | 1 | PCS | 5707292369376 |
|     | 82888-21 | CARRYING ARM HIGH/LOW, GREEN         | 1 | PCS | 5707292369383 |
| 68  | 82838    | LINKARM                              | 2 | PCS | 5707292125392 |
| 69  | 3470335  | RELEASE BUTTON Ø8, 5*8MM             | 1 | PCS | 5707292146632 |
| 70* | 82856-60 | ROD FOR HAND BRACE, RIGHT, MOP.BLACK | 1 | PCS |               |
|     | 82856-01 | ROD FOR HAND BRACE, RIGHT, BLACK     | 1 | PCS | 5707292232007 |
|     | 82856-20 | ROD FOR HAND BRACE, RIGHT, BLUE      | 1 | PCS | 5707292232014 |
|     | 82856-21 | ROD FOR HAND BRACE, RIGHT, GREEN     | 1 | PCS | 5707292232021 |
| 71  | 0641625  | BUTTON HEAD SCREW 6*25 MM            | 2 | PCS | 5707292139283 |
| 72* | 82855-60 | ROD FOR HAND BRACE, LEFT, MOP.BLACK  | 1 | PCS |               |
|     | 82855-01 | ROD FOR HAND BRACE, LEFT, BLACK      | 1 | PCS | 5707292231970 |
|     | 82855-20 | ROD FOR HAND BRACE, LEFT, BLUE       | 1 | PCS | 5707292231987 |
|     | 82855-21 | ROD FOR HAND BRACE, LEFT, GREEN      | 1 | PCS | 5707292231994 |
| 73  | 3652500  | WING SCREW 28*11 M6*16MM BLACK       | 2 | PCS | 5707292147172 |
| 74* | 71137    | BUSH BALL FOR HANDBRACE              | 2 | PCS | 5707292361813 |
|     | 71118    | BUSHING Ø22-18 FLAM.                 | 2 | PCS | 5707292104250 |
| 75* | 82858-60 | HANDLE, MOTHER OF PEARL BLACK        | 1 | PCS |               |
|     | 82858-01 | HANDLE, BLACK                        | 1 | PCS | 5707292232038 |
|     | 82858-20 | HANDLE, BLUE                         | 1 | PCS | 5707292232045 |
|     | 82858-21 | HANDLE, GREEN                        | 1 | PCS | 5707292232052 |
| 76  | 6153500  | KNOB SPRING 0,5*6,5 MM.              | 2 | PCS | 5707292188571 |
| 77  | 2500560  | GRIP F/Ø22 HANDBRACE 800MM           | 1 | PCS | 5707292388537 |
| 78  | 1108050  | WASHER M10, DIN9021                  | 2 | PCS | 5707292140401 |
| 80  | 1357500  | DISTANCE PCS BLACK 20*8,4*4MM        | 2 | PCS | 5707292141026 |
| 81  | 2704300  | BLACK HUB CAP FOR ROLKO WHEEL        | 2 | PCS | 5707292144515 |
| 82  | 82880    | HANDLE FOR ADAPTOR                   | 1 | PCS | 5707292361318 |
| 83  | 0683810  | PASBOLT M/INDV.6KT. 8*12 (M6)        | 1 | PCS | 5707292361929 |



|     |          |                                        |   |     |               |
|-----|----------|----------------------------------------|---|-----|---------------|
| 84  | 0659500  | BUTTON HEAD SCREW                      | 4 | PCS | 5707292139450 |
| 85  | 1310640  | WASHER, NATURE Ø16*10*0,5MM            | 2 | PCS | 5707292140760 |
| 86  | 0603700  | INNSEX SCREW 8*70                      | 2 | PCS | 5707292138293 |
| 88  | 70800    | SEAT CONTROL TIGER 2000/SERVAL         | 2 | PCS | 5707292102782 |
| 89* | 82827-60 | ADAÅTPR FPR TOGER 2000 SEAT, MOP.BLACK | 1 | PCS |               |
|     | 82827-01 | ADAPTOR FOR TIGER 2000 SEAT, BLACK     | 1 | PCS | 5707292231819 |
|     | 82827-20 | ADAPTOR FOR TIGER 2000 SEAT, BLUE      | 1 | PCS | 5707292231826 |
|     | 82827-21 | ADAPTOR FOR TIGER 2000 SEAT, GREEN     | 1 | PCS | 5707292231833 |
| 90  | 82899    | HANDBRACE HOLDER, LEFT                 | 1 | PCS | 5707292401397 |
| 91  | 0683803  | PASBOLT M/INDV.6KT. 10*25 (M8)         | 2 | PCS | 5707292460066 |
| 92  | 1607000  | LOCK NUT 8MM, LARGE PACK               | 2 | PCS | 5707292141378 |
| 93  | 1102000  | PLANING WASHER 8MM                     | 2 | PCS | 5707292140227 |
| 94  | 0603540  | INSEX LOW HEAD M8X40 STAINLESS         | 2 | PCS | 5707292412485 |
| 95  | 75600-60 | T-FRAME, X:PANDA 1+2                   | 1 | PCS | 5707292459541 |
| 130 | 1101000  | CENTER SPRING WASHER 8MM               | 1 | PCS | 5707292140173 |
| *   |          | CHOOSE COLOUR                          |   |     |               |

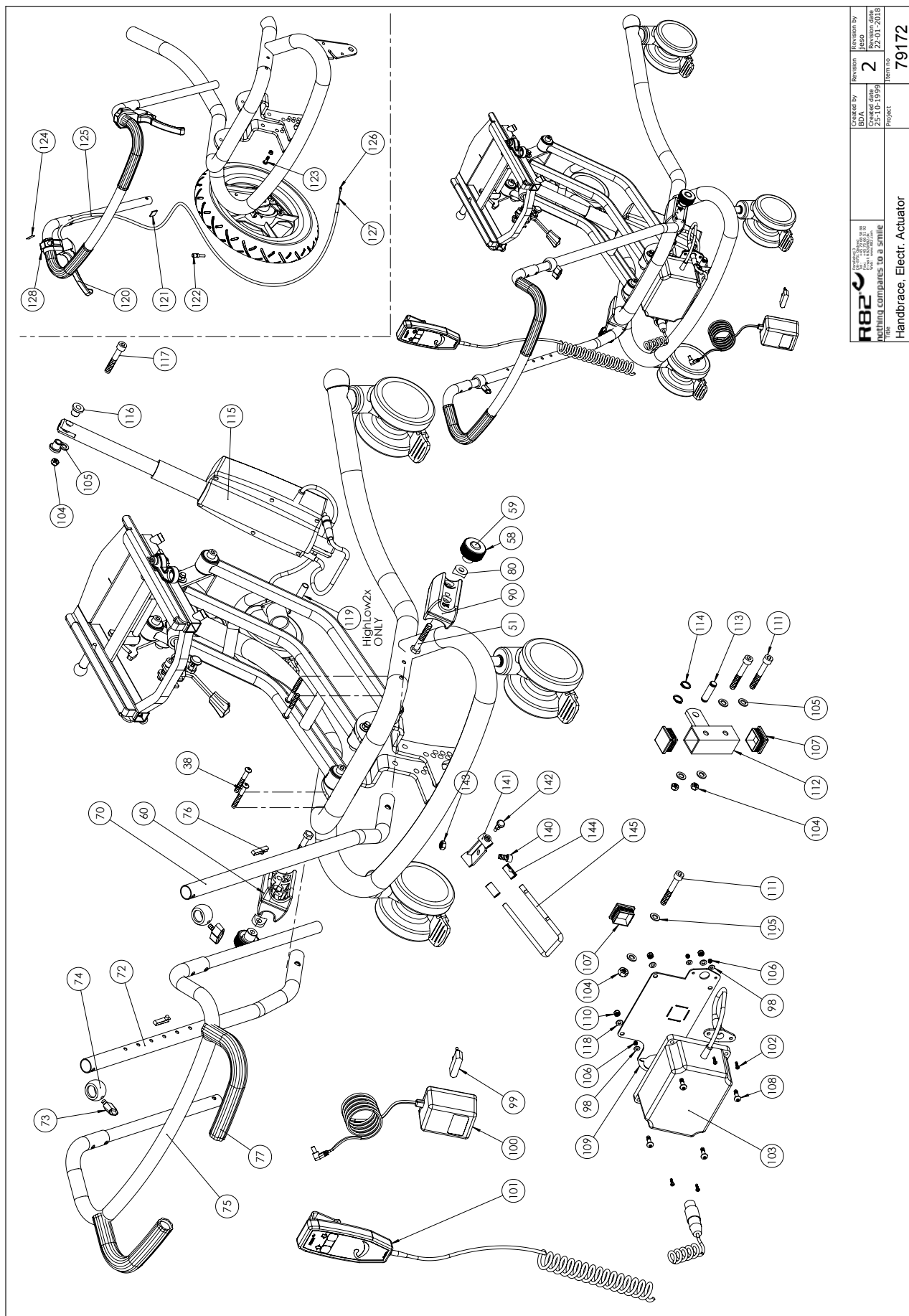

|                                                          |                   |                  |                    |
|----------------------------------------------------------|-------------------|------------------|--------------------|
| <b>R82</b><br><small>Nothing compares to a smile</small> | Created by<br>BDA | Revision<br>2    | Revision by<br>BDA |
|                                                          | 25-10-1999        | 25-10-1999       | 22-01-2018         |
| Project<br>Handbrace, Electr. Actuator                   |                   | Item no<br>79172 |                    |

| Position | No.      | Description                          | Quantity | Unit of Measure | Ean code      |
|----------|----------|--------------------------------------|----------|-----------------|---------------|
| 8        | 6113000  | SPINDLE, T-PART/FRAME                | 1        | PCS             | 5707292186362 |
| 9        | 3502008  | KNOB HANDLE O20 M6                   | 1        | PCS             | 5707292146830 |
| 10       | 0310100  | PRESSURESPRING 0,9*8*23*10,5V        | 1        | PCS             | 5707292135513 |
| 38       | 1310540  | COVERING PUSHBUTTON, BLACK           | 4        | PCS             | 5707292140692 |
| 40       | 2504000  | FERRULE E15*1,0-2,0 INT.BLACK        | 2        | PCS             | 5707292144102 |
| 51       | 1902320  | STEEL SET SCREW 8*70MM               | 2        | PCS             | 5707292142023 |
| 52       | 1606500  | LOCK NUT 6 MM                        | 1        | PCS             | 5707292141361 |
| 58       | 3470400  | HAND GRIP WITHOUT COVER              | 2        | PCS             | 5707292146663 |
| 59       | 1109500  | STARLOCK WASHER WITH CAPE 6          | 2        | PCS             | 5707292140449 |
| 60       | 82899    | HAND BRACE HOLDER, LEFT              | 1        | PCS             | 5707292401397 |
| 70*      | 82855-60 | ROD FOR HAND BRACE, LEFT, MOP.BLACK  | 1        | PCS             |               |
|          | 82855-01 | ROD FOR HAND BRACE, LEFT, BLACK      | 1        | PCS             | 5707292231970 |
|          | 82855-20 | ROD FOR HAND BRACE, LEFT, BLUE       | 1        | PCS             | 5707292231987 |
|          | 82855-21 | ROD FOR HAND BRACE, LEFT, GREEN      | 1        | PCS             | 5707292231994 |
| 72*      | 82856-60 | ROD FOR HAND BRACE, RIGHT, MOP.BLACK | 1        | PCS             |               |
|          | 82856-01 | ROD FOR HAND BRACE, RIGHT, BLACK     | 1        | PCS             | 5707292232007 |
|          | 82856-20 | ROD FOR HAND BRACE, RIGHT, BLUE      | 1        | PCS             | 5707292232014 |
|          | 82856-21 | ROD FOR HAND BRACE, RIGHT, GREEN     | 1        | PCS             | 5707292232021 |
| 73       | 3652501  | R82 HANDLE M6*16MM SCREW             | 2        | PCS             | 5707292413963 |
| 74       | 71137    | BUSH BALL FOR HANDBRACE              | 2        | PCS             | 5707292361813 |
| 75*      | 82858-60 | HANDLE, MOTHER OF PEARL BLACK        | 1        | PCS             |               |
|          | 82858-01 | HANDLE, BLACK                        | 1        | PCS             | 5707292232038 |
|          | 82858-20 | HANDLE, BLUE                         | 1        | PCS             | 5707292232045 |
|          | 82858-21 | HANDLE, GREEN                        | 1        | PCS             | 5707292232052 |
| 76       | 6153500  | KNOB SPRING 0,5*6,5 MM.              | 2        | PCS             | 5707292188571 |
| 77       | 2500560  | GRIP F/ø22 HANDBRACE 800MM           | 1        | PCS             | 5707292388537 |
| 80       | 1357500  | DISTANCE PCS 20*8,4*4MM              | 2        | PCS             | 5707292141026 |
| 82       | 82880    | HANDLE FOR ADAPTOR                   | 1        | PCS             | 5707292361318 |
| 83       | 0683810  | PASBOLT M/INDV.6KT. 8*12 (M6)        | 1        | PCS             | 5707292361929 |
| 88       | 1310640  | WASHER, NATURE O16*10*0,5MM          | 4        | PCS             | 5707292140760 |
| 90       | 82828    | HAND BRACE HOLDER, HIGH-LOW          | 1        | PCS             | 5707292125330 |
| 91       | 0683803  | PASBOLT M/INDV.6KA.10*25 (M8)        | 2        | PCS             | 5707292460066 |
| 92       | 1607000  | LOCK NUT 8MM, LARGE PACK             | 2        | PCS             | 5707292141378 |
| 93       | 1102000  | PLANNING WASHER 8MM                  | 2        | PCS             | 5707292140227 |
| 94       | 0603540  | INSEX LOW HEAD M8X40 STAINLESS       | 4        | PCS             | 5707292412485 |
| 95       | 75600-60 | T-FRAME, X:PANDA 1+2                 | 1        | PCS             | 5707292459541 |
| 99       | 2222165  | PLUG F/DK                            | 1        | PCS             | 5707292492388 |
| 99       | 2222166  | PLUG F/USA+CANADA+TAIWAN             | 1        | PCS             | 5707292492395 |
| 99       | 2222167  | PLUG FOR UK                          | 1        | PCS             | 5707292492401 |
| 99       | 2222169  | PLUG F/NEW ZEALAND + AUS             | 1        | PCS             | 5707292492425 |
| 100      | 2222160  | CHARGER WO/PLUG                      | 1        | PCS             | 5707292492296 |
| 101      | 2220600  | HANDSET W/R82 PRINT & CHARGING       | 1        | PCS             | 5707292492241 |
| 102      | 1902590  | CROSSHEAD SCREW M3*10, FZB           | 4        | PCS             | 5707292408389 |
| 103      | 2222110  | BATTERIBOKS BA18027                  | 1        | PCS             | 5707292492258 |
| 104      | 1607000  | LOCK NUT 8MM, LARGE PACK             | 3        | PCS             | 5707292141378 |
| 105      | 1102000  | PLANING WASHER 8MM                   | 6        | PCS             | 5707292140227 |
| 106      | 1606250  | SELF-LOCKING NUT 3MM                 | 4        | PCS             | 5707292408396 |
| 107      | 2505000  | FERRULE E30*301,0-2,0 INT.BLAC       | 3        | PCS             | 5707292144140 |
| 108      | 1902760  | CROSS-HEADED SCREW 5*16 MM           | 4        | PCS             | 5707292142146 |

|     |         |                                |   |     |               |
|-----|---------|--------------------------------|---|-----|---------------|
| 109 | 77576   | BRACKET FOR BATTERY            | 1 | PCS | 5707292125477 |
| 110 | 1606300 | LOCK NUT 5 MM                  | 4 | PCS | 5707292141354 |
| 111 | 0603600 | INSEX SCREW 8*55               | 3 | PCS | 5707292138262 |
| 112 | 82844   | AC MOUNTING FOR INDOOR FRAME   | 1 | PCS | 5707292125446 |
| 113 | 82849   | RATCHET Ø10                    | 1 | PCS | 5707292125491 |
| 114 | 1100100 | STAINLESS LOCKRING Ø10 AXLE    | 2 | PCS | 5707292140111 |
| 115 | 2222130 | ACTUATOR LA28 F/HIGHLOW        | 1 | PCS | 5707292492265 |
| 116 | 82848   | BUSH                           | 2 | PCS | 5707292125484 |
| 117 | 0603550 | INSEX SCREW 8*50               | 1 | PCS | 5707292138255 |
| 120 | 3503535 | BRAKE PARKING LEVER WO/CLAMP   | 2 | PCS | 5707292413536 |
| 121 | 2521854 | KABELKLIPS 4MM, NATUR          | 2 | PCS | 5707292144416 |
| 122 | 0380010 | KABELSTRAMMER M5*23 M.RUND MØT | 2 | PCS | 5707292135971 |
| 123 | 0380040 | CABLE BENCH SCREW M5           | 2 | PCS | 5707292135988 |
| 124 | 0494000 | LABEL "BRAKE" MET. FILM        | 2 | PCS | 5707292137159 |
| 125 | 5001200 | CABLE FOR TB-PANDA             | 2 | PCS | 5707292154446 |
| 126 | 0387000 | END NIPPLE FOR OUTER CASING    | 2 | PCS | 5707292136091 |
| 127 | 0371853 | STRAINED WIRES 1,5MM 19T.      | 2 | PCS | 5707292135889 |
| 128 | 3503536 | CLAMP W/SCREWS                 | 2 | PCS | 5707292427762 |
| 140 | 0604000 | INSEX SCREW 10*20              | 1 | PCS | 5707292138361 |
| 141 | 82488   | STEER FOR ANTI TIP DEVICE      | 1 | PCS | 5707292357847 |
| 142 | 0380182 | LOCKING HOUSE, COMPLETE M10*1  | 1 | PCS | 5707292357953 |
| 143 | 1607500 | LOCK NUT 10 MM                 | 1 | PCS | 5707292141385 |
| 144 | 82375   | PVC-BUSHING FOR PELOT          | 2 | PCS | 5707292123763 |
| 145 | 82484   | BRACE FOR ANTI TIP DEVICE      | 1 | PCS | 5707292357892 |

|    |  |                |
|----|--|----------------|
| *  |  | CHOOSE COLOUR  |
| ** |  | CHOOSE COUNTRY |