

Kudu[®]

Accessories

Exploded views / Part lists

Rev. date (2025.04)

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.



Accessories



Head support with cover
950000-xx
For correct position and support

See the Accessories section at the back of the catalogue, for further information on options and measurements



Back extension cushion
957432-xx
Supports the upper back

Size	Measurements
1	+5 cm or +10 cm
2	+5 cm or +10 cm
3	+5 cm or +10 cm
4	+5 cm or +10 cm



Fixed side supports
954011-xx

Type	Measurements
9	W:9 X H: 10 cm
5	W:16 X H: 15 cm
6	W:12 X H: 14 cm



Swing-away side supports
837030-xx

See the Accessories section at the back of the catalogue, for further information on options and measurements



Arm rests, incl. side plates
957122-xx
Height adjustable and provide arm support

One size



Tray
9576350

One size

See the Accessories section at the back of the catalogue, for further information on options and measurements



Swing-away knee supports, set
958000-xx
Provide control and positioning

Type	Size
1+2	Small
3+4	Medium



Rucksack
944800-xx
Extra space for storage

Type	Size
1+2	Small
3+4	Medium



Calf support
85131-xx
For parted foot rests
For extra support and comfort

One size



Calf support
85131
Centrally mounted foot plate
For extra support and comfort

Size	Measurements
1	H:15 x W: 30 cm



Heel band for parted foot rests
85132-xx

One size



Mud guards
9520xx
Available for 20", 22" and 24"

One size


nothing compares to a smile

R82

Contents

Contents	2
Tray shape 8, with edge. Kit 9576351	3
Item numbers for Tray shape 8, with edge. Kit 9576351	3
Hip supports kit: size 1+2 95713-10, size 3+4 95713-20	4
Item numbers for Hip supports kit: size 1+2 95713-10, kit size 3+4 95713-20	4
Arm rests kit: 95712-20, 95712-40.....	5
Item numbers for Arm rests kit: 95712-20, 95712-40.....	5
Fixed side supports kit: 95401-10, 95401-20, 95401-30	7
Item numbers for Fixed side supports kit: 95401-10, 95401-20, 95401-30	7
Swing-away knee supports kit: small 95860-10, medium 95860-20	8
Item numbers for Swing-away knee supports kit: small 95860-10, medium 95860-20	8
Mudguards kit:P-952020, P-952022, P-952022-7, P-952024, P952024-7	9
Item numbers for Mudguards kit:P-952020, P-952022, P-952022-7, P-952024, P952024-7	9
Vent tray kit 95281-10.....	10
Item numbers for Vent tray kit 95281-10.....	10
Transport fittings, rear kit 95900-16.....	11
Item numbers for Transport fittings, rear kit 95900-16.....	11
Strap back-long_410mm. Size 3 950033-KIT, size 4 950043-KIT	12
Item numbers for Strap back-long_410mm. Size 3 950033-KIT, size 4 950043-KIT	13
Strap back- 289 mm - short. Size 3 950031-KIT, size 4 950041-KIT.....	14
Item numbers for Strap back- 289 mm - short. Size 3 950031-KIT, size 4 950041-KIT.....	15
Turnable anti tip device, kit set 950958-1.....	16
Item numbers for Turnable anti tip device, kit set 950958-1.....	16
Back plate kit 950012-PLATES, 950013-PLATES 950014-PLATES.....	17
Item numbers for Back plate kit 950012-PLATES, 950013-PLATES 950014-PLATES.....	17
Closed wheel covers for 16" 952016-C.....	18
Item numbers for Closed wheel covers for 16" 952016-C.....	18
Mud guards for 16" wheels 952016-1.....	19
Item numbers for Mud guards for 16" wheels 952016-1	19


Tray shape 8, with edge. Kit 9576351

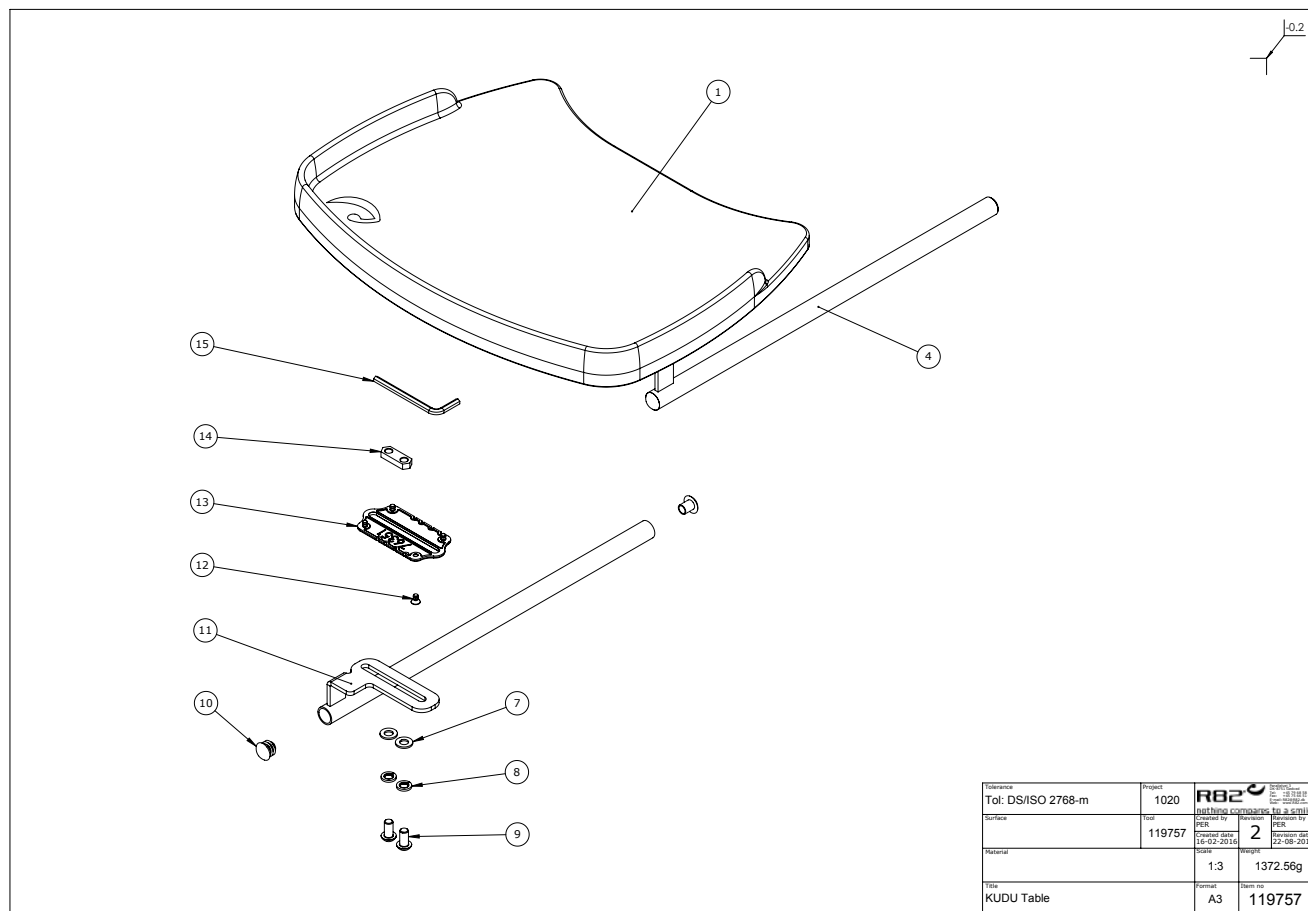

Item numbers for Tray shape 8, with edge. Kit 9576351

Position	Art nr.	Description	Quantity	Unit of measure	Ean code
1	76350	Table for Stingray, Grey	1	PCS	5707292541727
4	111547	Bracket for table, Right	1	PCS	5707292586940
7	1105080	Stainless washer 8mm	4	PCS	5707292140364
8	1100500	Spring washer 8mm, stainless	4	PCS	5707292140142
9	0661500	Button head screw 8x16 stainl.	4	PCS	5707292139559
10	2502002	Ferrule d18x0,8-2,5 black	4	PCS	5707292433947
11	111550	Bracket for table, Left	1	PCS	5707292586964
12	1940011	Plastite 60, 4x8	8	PCS	5707292545978
13	76351	Mounting plate	2	PCS	5707292541741
14	77557	Slide bar (M8x2)	2	PCS	5707292544995
15	0589500	Insex keys n5	1	PCS	5707292137869

Hip supports kit: size 1+2 95713-10, size 3+4 95713-20

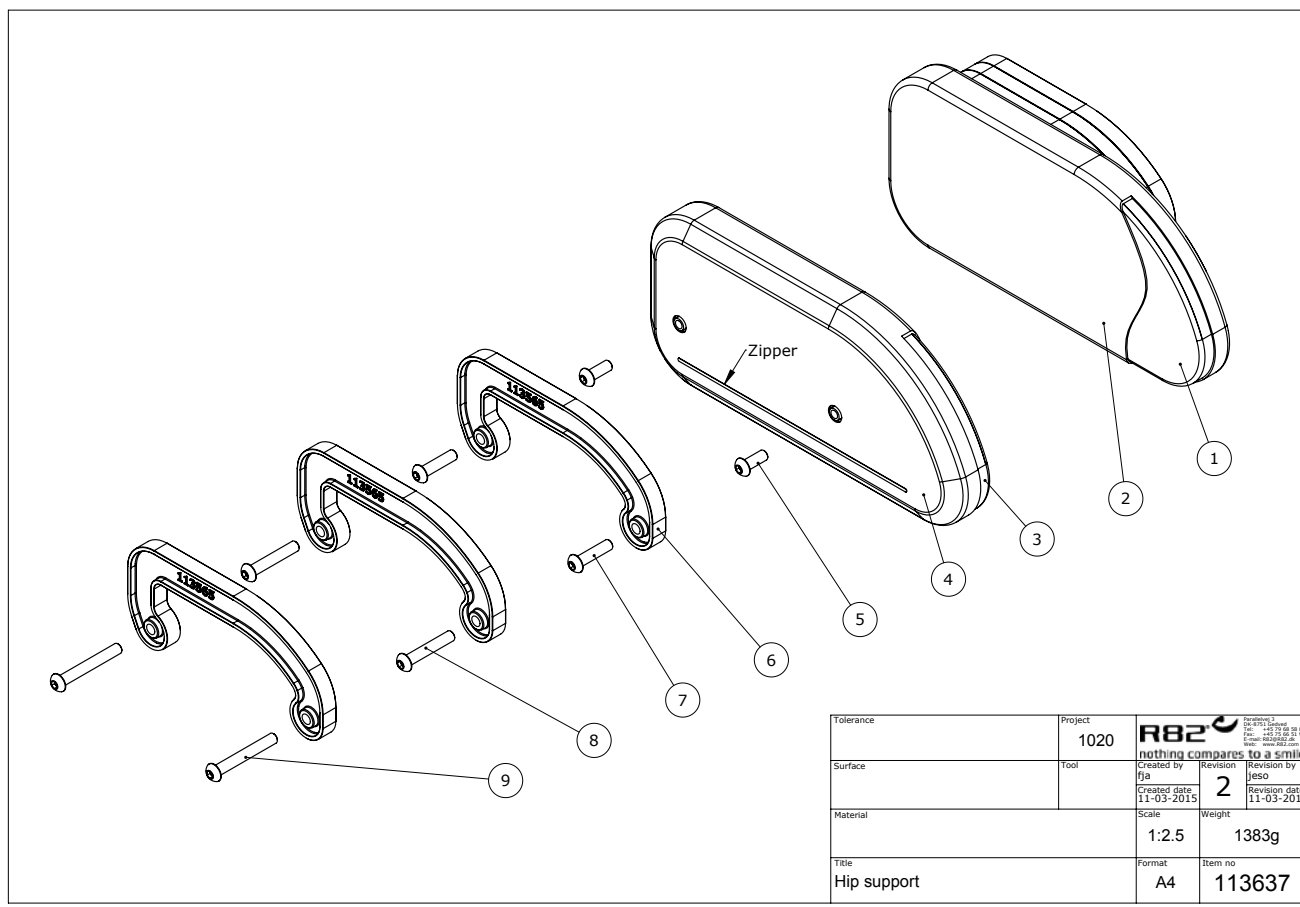


Item numbers for Hip supports kit: size 1+2 95713-10, kit size 3+4 95713-20

Position	Art nr.	Description	Quantity	Unit of measure	Ean code
1*	113627	Hipsupport w. Padding size 2	1	PCS	5707292600943
	113617	Hipsupport w. Padding size 1	1	PCS	5707292600851
2*	95713-20160L	Cover for hip support, left	1	PCS	5707292600974
	95713-10160L	Cover for hip support, left	1	PCS	5707292600967
3*	113628	Hipsupport w. Padding size 2	1	PCS	5707292600998
	113618	Hipsupport w. Padding size 1	1	PCS	5707292600936
4*	95713-20160R	Cover for hip support, right	1	PCS	5707292600721
	95713-10160R	Cover for hip support, right	1	PCS	5707292600714
5	0659500	Button head screw M6x16	4	PCS	5707292139450
6	113565	Distance piece	6	PCS	5707292600776
7	0659570	Button head screw 6x25 stainl.	4	PCS	5707292139474
8	0659585	Button head M6x35, stainless	4	PCS	5707292139498
9	0659600	Button head M6x45, stainless	4	PCS	5707292356826

*Choose size, color or type

Arm rests kit: 95712-20, 95712-40

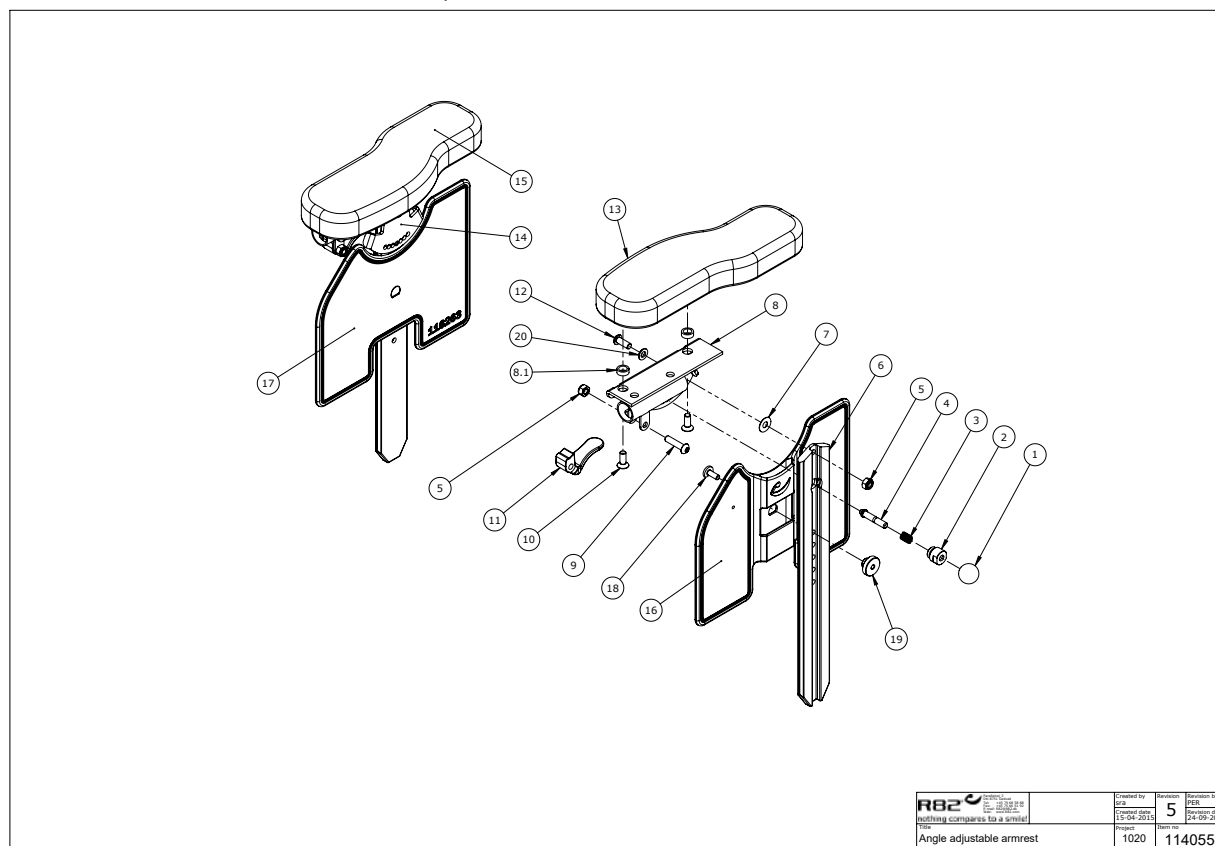


Item numbers for Arm rests kit: 95712-20, 95712-40

Position	Art nr.	Description	Quantity	Unit of measure	Ean code
1	3502008	Knob handle Ø20 M6	2	PCS	5707292146830
2	6118900	Springhouse, panda+impala	2	PCS	5707292186768
3	0310125	Pressure spring 0,9x8u/Ø12+0,5	2	PCS	5707292135575
4	6118810	Pin f/springhouse panda+impala	2	PCS	5707292186706
5	1606500	Lock nut 6 mm	4	PCS	5707292141361
6	114060	Armrest profile piece	2	PCS	5707292612182
7	1310709	Skive Ø15,9x6,5x0,5mm	2	PCS	5707292490520
8	114072	Angle adjustable armrest	1	PCS	5707292617606
8.1	1310539	Distance piece 12/8,4x5mm	4	PCS	5707292423474
9	0659570	Button head screw 6x25 stainl.	2	PCS	5707292139474
10	0621600	Insex screw counters. 6x16	4	PCS	5707292138965
11	113713	Armrest handle	2	PCS	5707292607867
12	0641200	Button head screw 6x12	2	PCS	5707292139252
13*	6175152	Armrest cushion, size 1, left	1	PCS	5707292189233
	6175160	Armrest pad,s.4,left,black	1	PCS	5707292189271
14	114071	Angle adjustable armrest	1	PCS	5707292617590
15*	6175150	Armrest cushion, size 1, right	1	PCS	5707292189226
	6175158	Armrest pad s.4,right,black	1	PCS	5707292189264
16*	116262	Adj. armrestsupport left Sz. 1–2	1	PCS	5707292618139
	116259	Adj. Armsupport left Sz. 3–4	1	PCS	5707292618146
17*	116263	Adj. armrest support right Sz. 1–2	1	PCS	5707292618115
	116260	Adj. armrest support right Sz. 3–4	1	PCS	5707292618122

*Choose size, color or type

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
18	111101	Armrest bolt	2	PCS	5707292586056
19	111103	Armrest nut, Kudu	2	PCS	5707292586070
*Choose size, color or type					

Fixed side supports kit: 95401-10, 95401-20, 95401-30

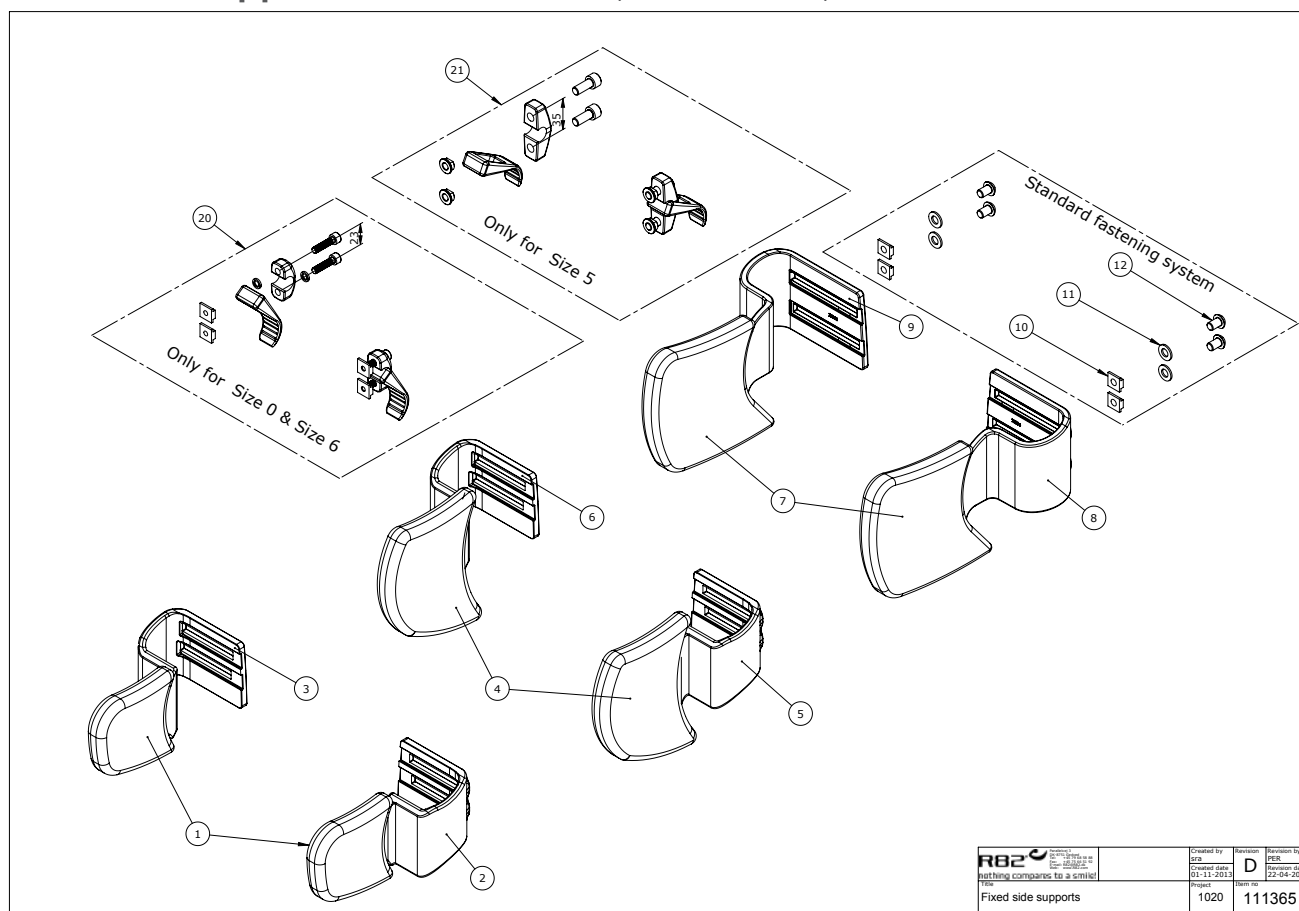


Item numbers for Fixed side supports kit: 95401-10, 95401-20, 95401-30

Position	Art nr.	Description	Quantity	Unit of measure	Ean code
1	5027197	Cover for back sides, size 0, Right+Left	2,00	Pieces	5707292587794
2	5700563	Back sides, size 0, Left	1,00	Pieces	5707292587770
3	5700564	Back sides, size 0, Right	1,00	Pieces	5707292587787
4	5027199	Cover for back sides, size 6, Right+Left	2,00	Pieces	5707292544193
5	5700566	Back sides, size 6, Right	1,00	Pieces	5707292538413
6	5700565	Back sides, size 6, Left	1,00	Pieces	5707292538406
7	5027204	Cover for back sides, size 5, Right+Left	2,00	Pieces	5707292514028
8	5700528	Back sides, size 5, Left	1,00	Pieces	5707292514004
9	5700529	Back sides, size 5, Right	1,00	Pieces	5707292514011
10	75203-1	M8 slide with threads	4,00	Pieces	5707292517593
11	1105080	Stainless washer 8mm	4,00	Pieces	5707292140364
12	0661300	Button head m/flange 8x12	4,00	Pieces	5707292408068

*Choose size, color or type

Swing-away knee supports kit: small 95860-10, medium 95860-20

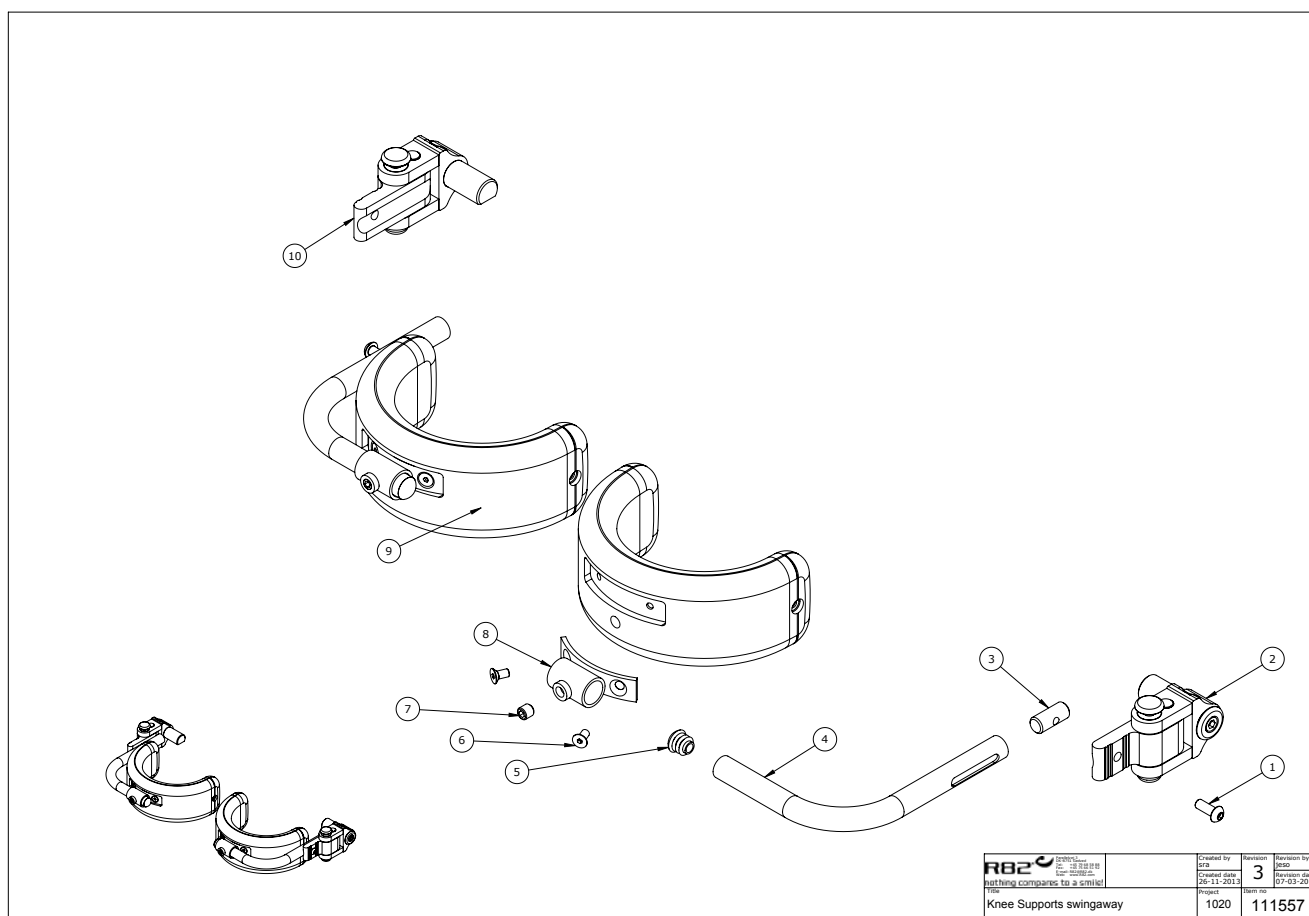


Item numbers for Swing-away knee supports kit: small 95860-10, medium 95860-20

Position	Art nr.	Description	Quantity	Unit of measure	Ean code
1	0659500	Button head screw	2	PCS	5707292139450
2	111558	Svingbar knæstøtte venstre	1	PCS	5707292588845
3	82935	Thread pal	2	PCS	5707292125903
4	82374	Curved knee brace Wombat	2	PCS	5707292123756
5	2501600	Ferrule d15x1,0 black	2	PCS	5707292143969
6	0609719	Stainless insex 5x10 uh	2	PCS	5707292446046
7	0808008	Stainless stop screw 8x8	2	PCS	5707292140036
8	82372	Holder for knee pelot	2	PCS	5707292123732
9*	6175161	Kneesupport size 1 Panda	2	PCS	5707292189288
	6175162	Kneesupport size 2 Panda	2	PCS	5707292189295
10	111559	Svingbar knæstøtte Højre	1	PCS	5707292588869

*Choose size, color or type

Mudguards kit:P-952020, P-952022, P-952022-7, P-952024, P952024-7

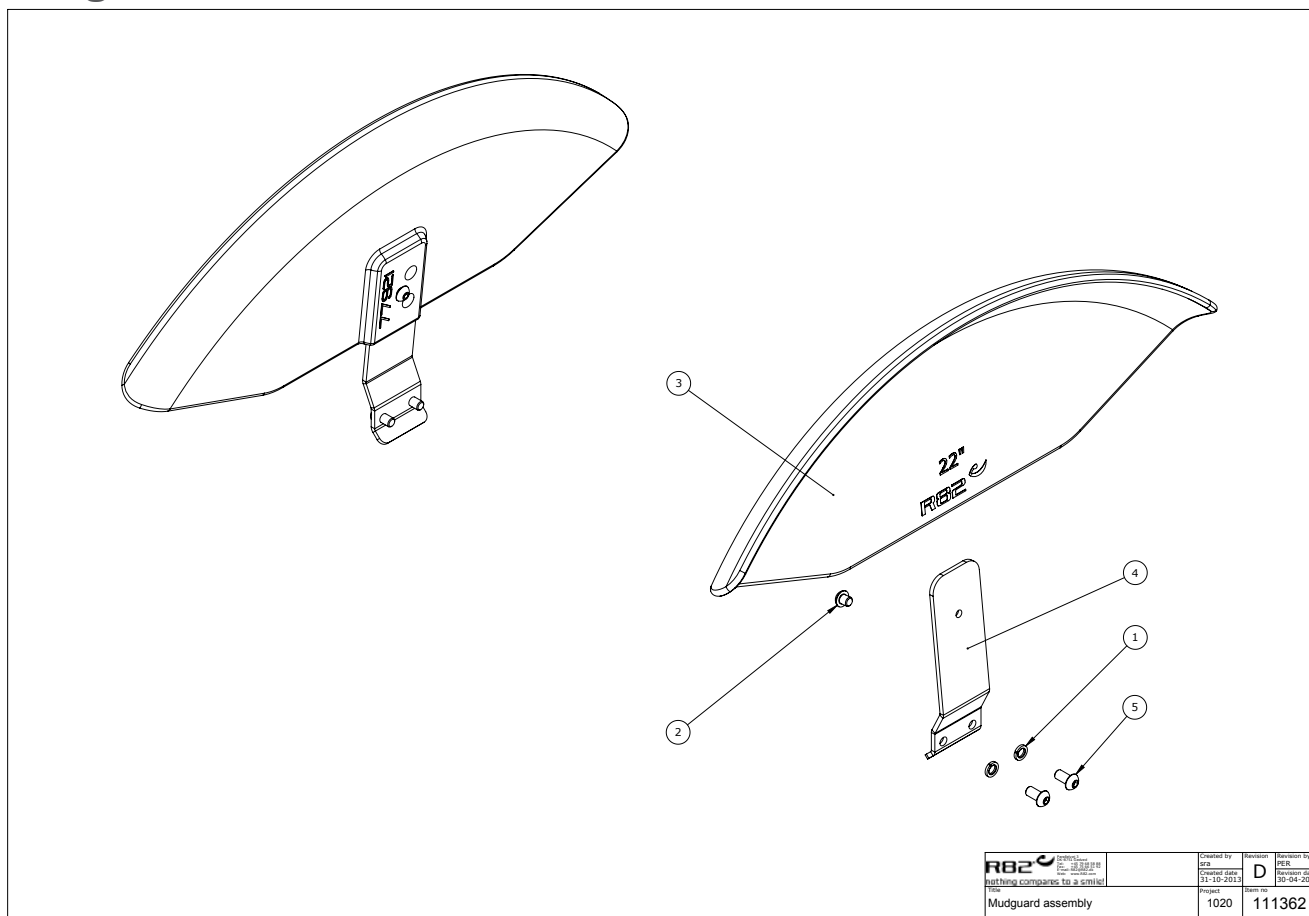


Item numbers for Mudguards kit:P-952020, P-952022, P-952022-7, P-952024, P952024-7

Position	Art nr.	Description	Quantity	Unit of measure	Ean code
1	1100900	Spring washer M6 fzb, 6.1/9.9	4	PCS	5707292459336
2	0631961	Buttonhead M6x6,zn/fe&black	2	PCS	5707292577764
3*	77986	Fender 24"	2	PCS	5707292571700
	77921	Fender 22"	2	PCS	5707292571694
	77895	20" Fender	2	PCS	5707292571687
4*	111266	Fender mount 24"	2	PCS	5707292585950
	108137	Fender mount 20"	2	PCS	5707292585936
	111265	Fender mount 22"	2	PCS	5707292585943
	108137	Fender mount 20"	2	PCS	5707292585936
5	0663120	Button Head M6x16 stainless st	4	PCS	5707292581532

*Choose size, color or type


Vent tray kit 95281-10

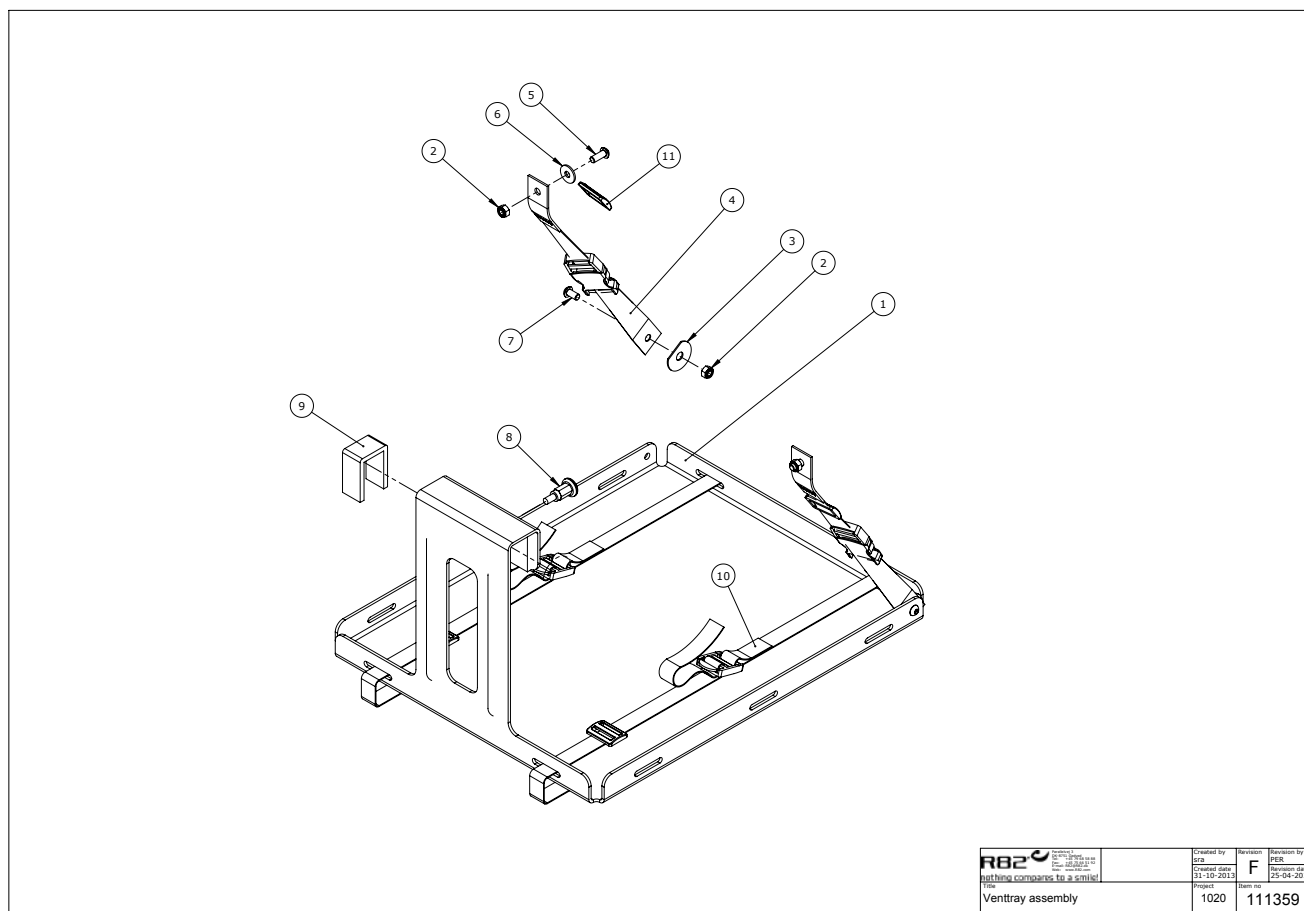

Item numbers for Vent tray kit 95281-10

Position	Art nr.	Description	Quantity	Unit of measure	Ean code
1	109123	Vent tray	1	PCS	5707292584038
2	1631060	Stainless lock nut 6mm	4	PCS	5707292141446
3	1125000	Screenwasher FZB black 5/16x30	2	PCS	5707292140470
4	5540541	Strap for vent tray	2	PCS	5707292552686
5	0659500	Button head screw M6x16	2	PCS	5707292139450
6	1130600	Stainless washer M6 din9021	2	PCS	5707292140517
7	0659200	Buttonhead screw M6x12 stainl.	2	PCS	5707292139436
8	0380187	Spec. House for lock, Kudu	1	PCS	5707292590831
9	I-109065	Shim Pad f. ventray	0,2	MTR	5707292577542
10	5540545	Strap for oxygen bottle	2	PCS	5707292480798
11	0467000	Belt clamp 25 mm black	2	PCS	5707292136558

Transport fittings, rear kit 95900-16

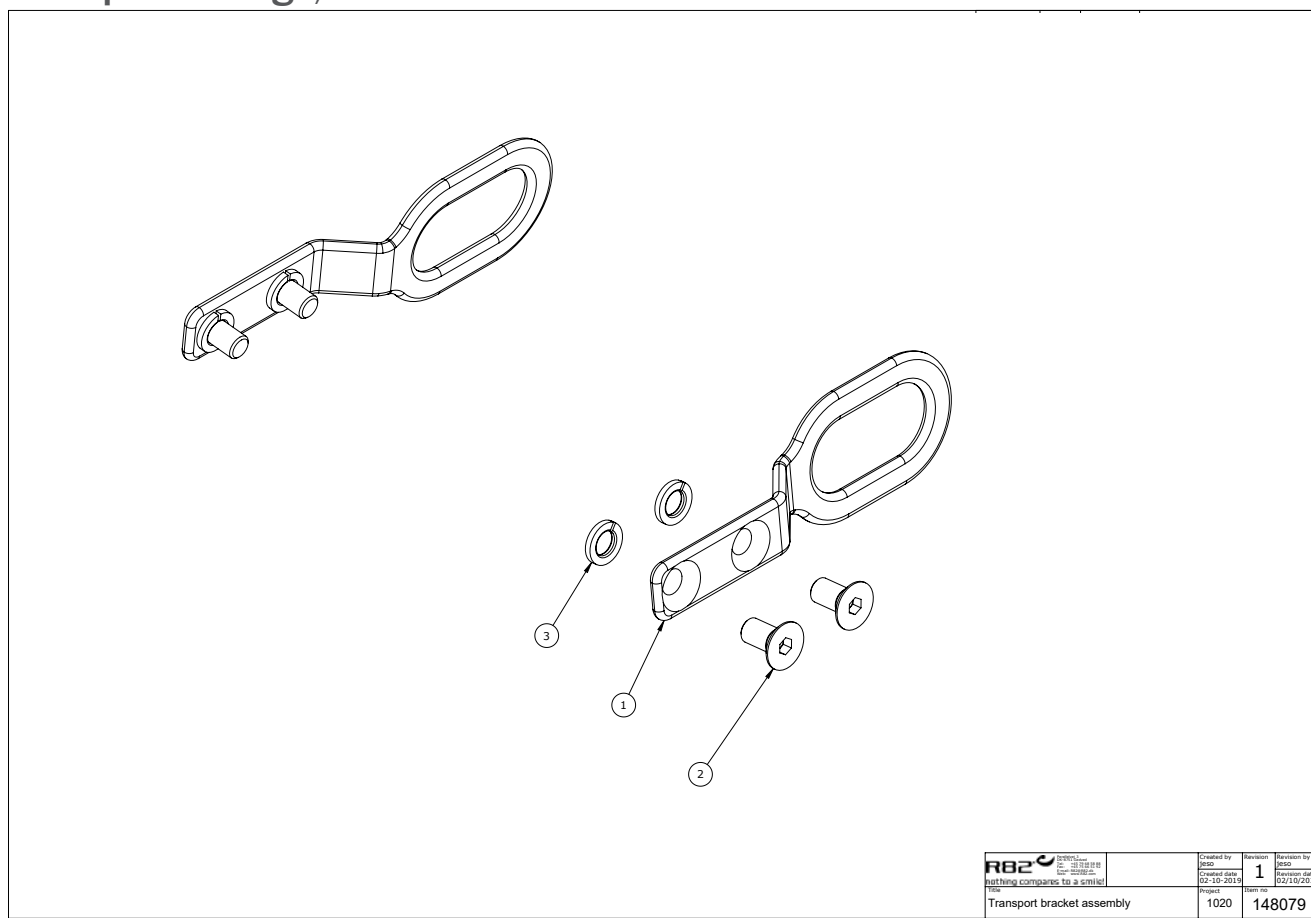


Item numbers for Transport fittings, rear kit 95900-16

Position	Art nr.	Description	Quantity	Unit of measure	Ean code
1	139042	Clamping bracket	2	PCS	5707292845177
2	0602486	Insex M8x16	4	PCS	5707292434272
3	1100500	Spring washer 8mm, stainless	4	PCS	5707292140142

Strap back-long_410mm. Size 3 950033-KIT, size 4 950043-KIT

C:\pmp\pmp\pmp\1020\1020 - 603\COO 1836 - Kudu R820 back\1836

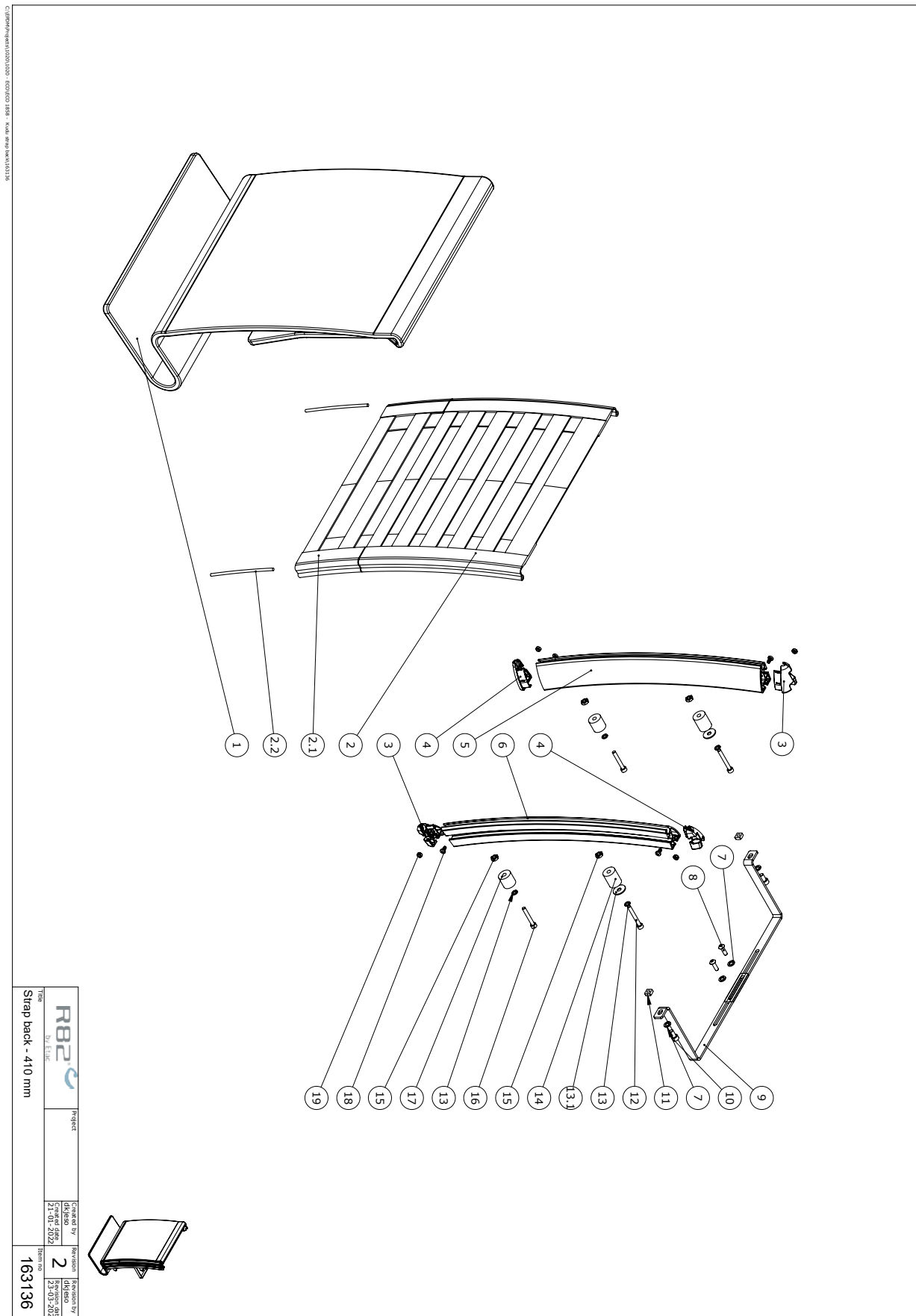

Item numbers for Strap back-long_410mm. Size 3 950033-KIT, size 4 950043-KIT


Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1*	50-63204-68	Cover 350 mm	1	PCS	5707292947376
	50-63205-68	Cover size 4 8375mm)	1	PCS	5707292948847
2	50-62953-60	Strap (loop) back	1	PCS	5707292947369
3	50-51131	Plastic end plug, right	2	PCS	5707292947345
4	50-51130	Plastic end plug left	2	PCS	5707292947338
5	50-83315-2	Left, back profile Prio 410mm	1	PCS	5707292947390
6	50-83315-1	Right, Back profile Prio 410mm	1	PCS	5707292947383
7	1100800	Center spring washer 6mm	4	PCS	5707292140166
8	0658016	Button head M5x16 stainless	2	PCS	5707292139399
9	161462	Bracket for cable extender	2	PCS	5707292944504
10	0602190	Insex screw 6x12	2	PCS	5707292138088
11	50-12058	4-edged nut	2	PCS	5707292947321
12	0602442	Insex M5x40 FZB	2	PCS	5707292947536
13	1100485	Spring washer M5, stainless	4	PCS	5707292478603
14	70024	Distance bush Ø20x25	2	PCS	5707292101327
15	1600510	Welding nut -Hexagonal	4	PCS	5707292947475
16	0602436	Insex M5x35 FZB	2	PCS	5707292947512
17	70023	Distance bush Ø20x20	2	PCS	5707292101310
18	50-11441	Bolt, for end plug	4	PCS	5707292947352
19	1606275	Lock nut 4mm, large pack	4	PCS	5707292141347
13.1	1102001	Washer 6x20x1,5 mm, RF	2	PCS	5707292565754
2.1	50-63351-60	Upholstery back 3A	1	PCS	5707292947406
2.2	50-28715	plastic for strap back	2	PCS	5707292952080
*Choose size, color or type					

R82 by ELC	Project	Created by Checked date 12-01-2022	Revision 2 25-03-2022
Type Strap back - 289 mm		Item no. 162908	

Item numbers for Strap back- 289 mm - short. Size 3 950031-KIT, size 4 950041-KIT

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1*	50-63204-68	Cover 350 mm	1	PCS	5707292947376
	50-63205-68	Cover size 4 8375mm)	1	PCS	5707292948847
2	50-62953-60	Strap (loop) back	1	PCS	5707292947369
3	50-51131	Plastic end plug, right	2	PCS	5707292947345
4	50-51130	Plastic end plug left	2	PCS	5707292947338
5	50-83013-2	Right, back profile	1	PCS	5707292947314
6	50-83013-1	left, back profile	1	PCS	5707292947260
7	1100800	Center spring washer 6mm	4	PCS	5707292140166
8	0658016	Button head M5x16 stainless	2	PCS	5707292139399
9	161462	Bracket for cable extender	2	PCS	5707292944504
10	0602190	Insex screw 6x12	2	PCS	5707292138088
11	50-12058	4-edged nut	2	PCS	5707292947321
12	0602442	Insex M5x40 FZB	2	PCS	5707292947536
13	1100485	Spring washer M5, stainless	4	PCS	5707292478603
14	70024	Distance bush Ø20x25	2	PCS	5707292101327
15	1600510	Welding nut -Hexagonal	4	PCS	5707292947475
16	0602436	Insex M5x35 FZB	2	PCS	5707292947512
17	70023	Distance bush Ø20x20	2	PCS	5707292101310
18	50-11441	Bolt, for end plug	4	PCS	5707292947352
19	1606275	Lock nut 4mm, large pack	4	PCS	5707292141347
13.1	1102001	Washer 6x20x1,5 mm, RF	2	PCS	5707292565754
*Choose size, color or type					


Turnable anti tip device, kit set 950958-1

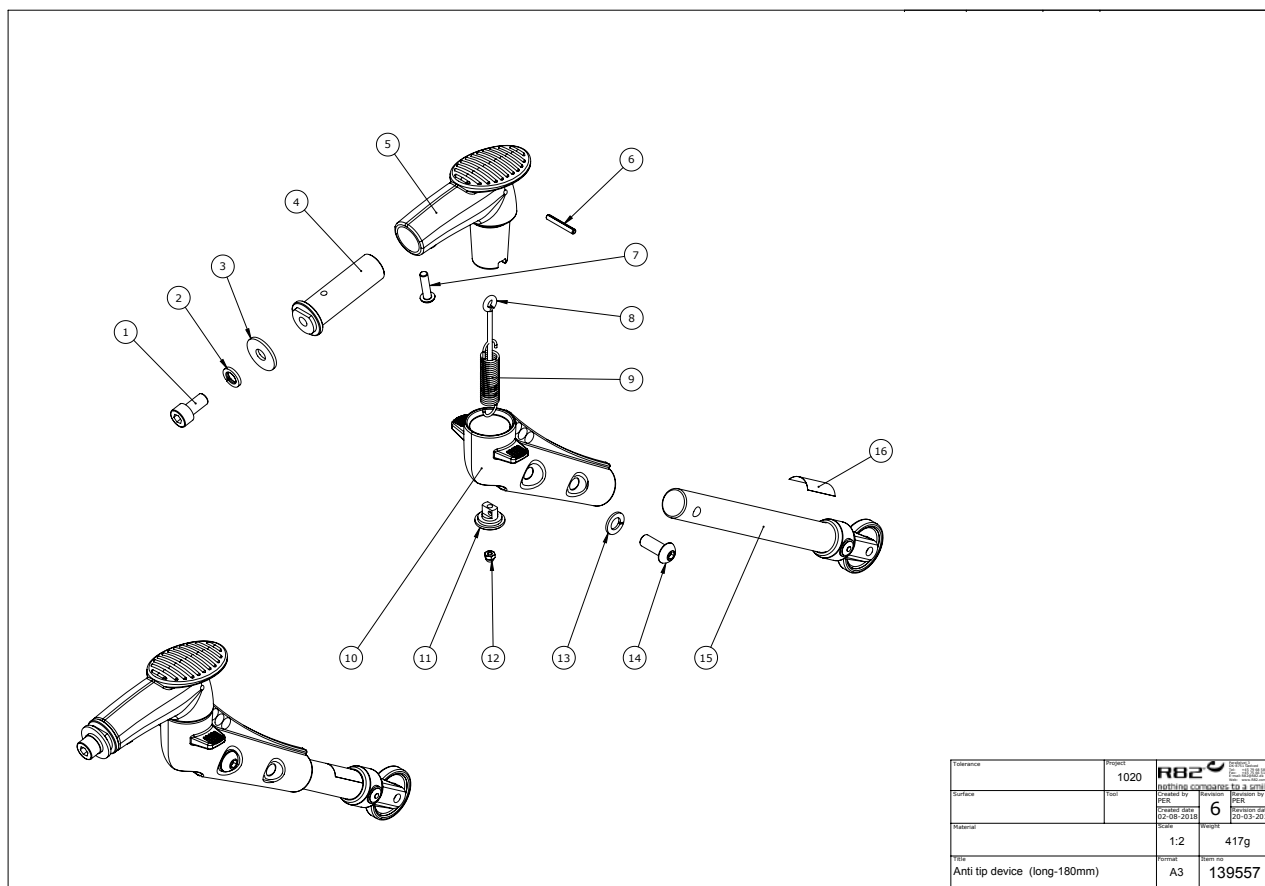

Item numbers for Turnable anti tip device, kit set 950958-1

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	0602900	Insex screw 8x16	2	PCS	5707292138200
2	1100500	Spring washer 8mm, stainless	2	PCS	5707292140142
3	1108025	Washer M8,stainless a4,din9021	2	PCS	5707292140395
4	139558	Extended Antitip Mounting stud	2	PCS	5707292867490
5	138453	Top Joint	2	PCS	5707292867254
6	0532210	Span pin 1481 3x22 stainless	2	PCS	5707292423313
7	0658020	Button head M5 x 20,stainless	2	PCS	5707292139405
8	74642	Locking eye bolt, anttip,couga	2	PCS	5707292415752
9	0310520	Tension spring 1,5x12x55,5in	2	PCS	5707292406361
10	138280	Lower part anti tilt Kudu	2	PCS	5707292867018
11	6109361	Spring holder f/anti tip	2	PCS	5707292415479
12	1781030	Stainless top nut M3	2	PCS	5707292415769
13	1101075	Spring washer M8, wavy	2	PCS	5707292580962
14	0609816	Stainless insex screw 8x15	2	PCS	5707292138569
15	5097003	Extentionbar with wheel,long	2	PCS	5707292416209
16	0492052	Anti Tip Advice	2	PCS	5707292806130


Back plate kit 950012-PLATES, 950013-PLATES 950014-PLATES

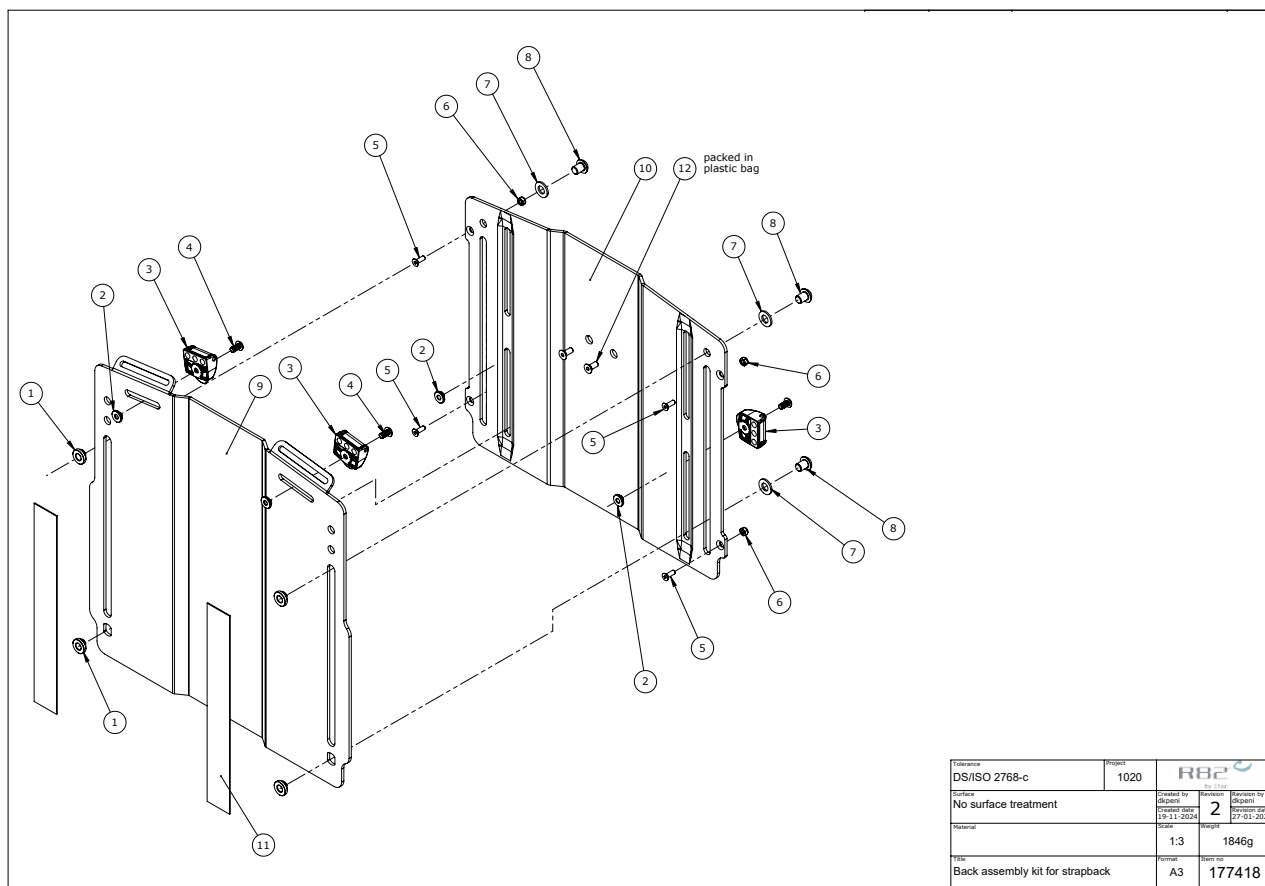

Item numbers for Back plate kit 950012-PLATES, 950013-PLATES 950014-PLATES

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	6118920	M8 Nut	4	PCS	5707292433237
2	78150	Groove nut M5	4	PCS	5707292576118
3	75306	Snap lock R82 25	4	PCS	5707292437624
4	631952	Button head M5*6, rf tuflok	4	PCS	5707292503725
5	614012	Insex base M4*12, FZB	4	PCS	5707292138811
6	1606275	Lock nut 4mm, large pack	4	PCS	5707292141347
7	1105080	Stainless washer 8mm	4	PCS	5707292140364
8	661150	Button head screw 8*10 stainl.	4	PCS	5707292579041
9	137463	Back plate front size 2	1	PCS	5707292870179
9	137464	Back plate front size 3	1	PCS	5707292870186
9	137465	Back plate front size 4	1	PCS	5707292870193
10	137468	Back plate rear size 2	1	PCS	5707292870216
10	137470	Back plate rear size 3	1	PCS	5707292870223
10	137471	Back plate rear size 4	1	PCS	5707292870230
11	4913040	30mm adhesive hook, black	0,55	MTR	5707292786517
12	631862	Insex uh. 5x12, FZB	2	PCS	5707292352293


Closed wheel covers for 16" 952016-C

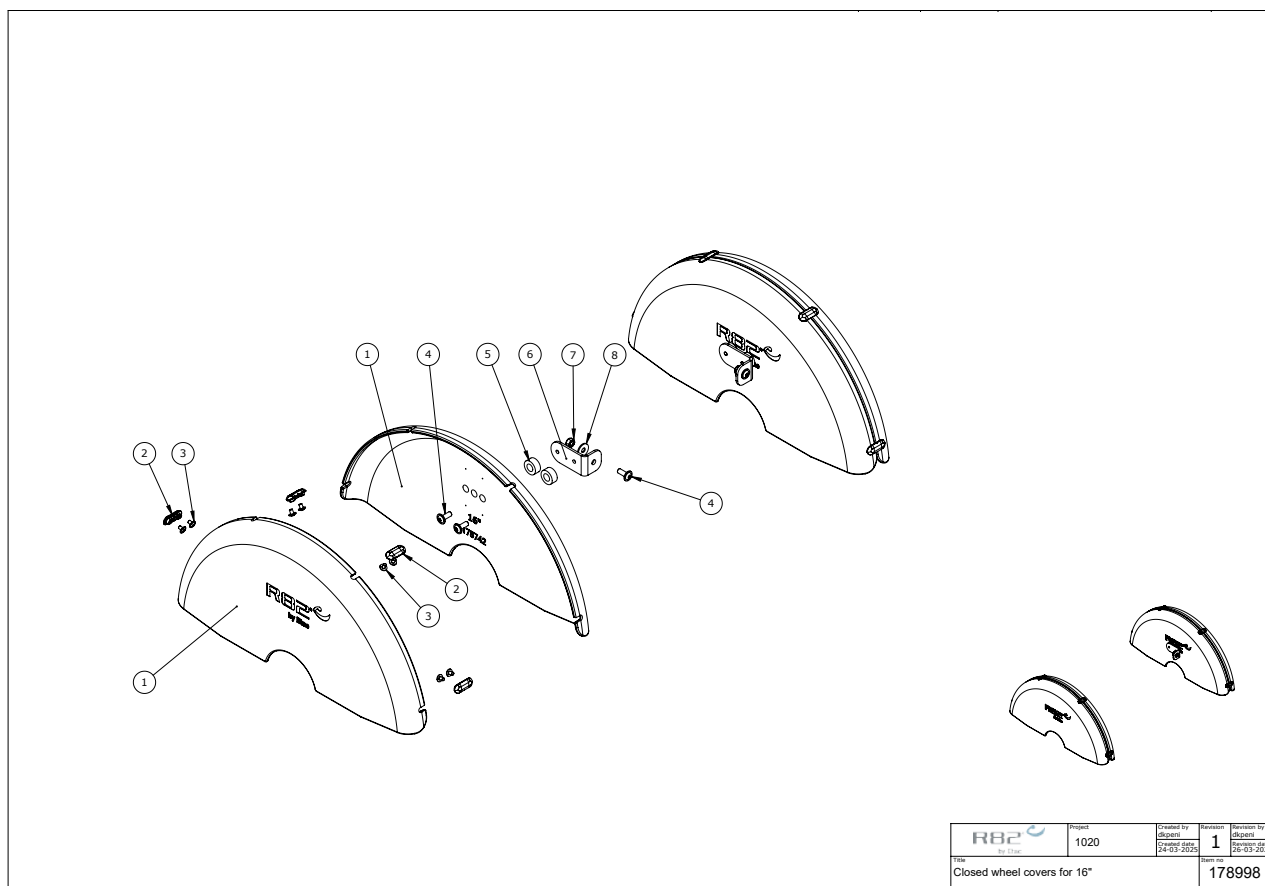

Item numbers for Closed wheel covers for 16" 952016-C

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	175742	16" Mudguard inner	4	PCS	5715393039241
2	173771	Double blind plug	8	PCS	5715393039272
3	0602184	Flange head screw M4x6	16	PCS	5715393039401
4	0659501	Button head w/flange M6*16 FZB	6	PCS	5707292356000
5	1350007	Pvc bush black 20*10,5*9,0mm	4	PCS	5707292545527
6	177209	Bracket for mud guard 16"	2	PCS	5715393020621
7	1606500	Lock nut 6 mm	2	PCS	5707292141361
8	1106000	Washer 6mm	2	PCS	5707292140371

Mud guards for 16" wheels 952016-1

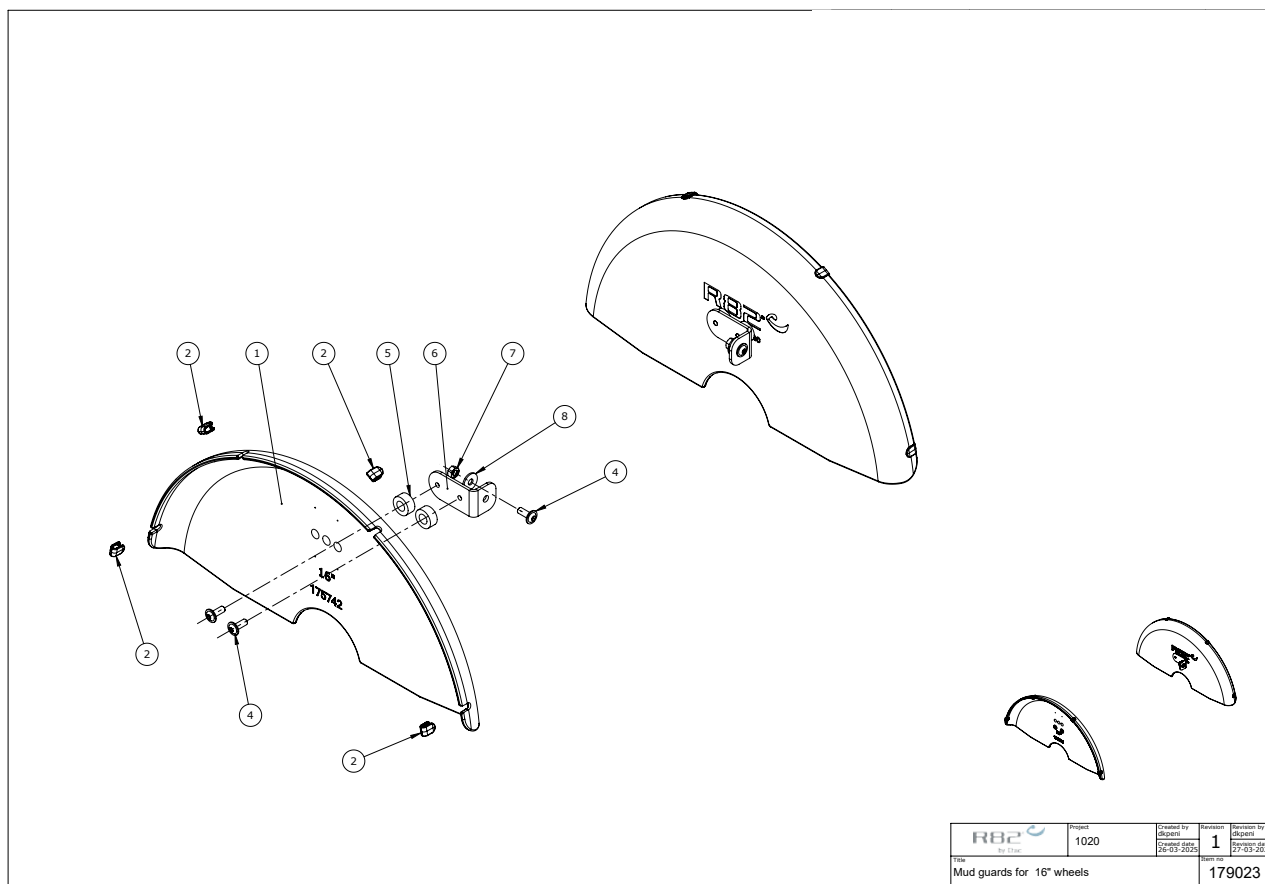

Item numbers for Mud guards for 16" wheels 952016-1

Position	Art nr.	Description	Quantity	Unit of measure	Ean kode
1	175742	16" Mudguard inner	2	PCS	5715393039241
2	173768	Clips for single mudguard	8	PCS	5715393039265
4	659501	Button head w/flange M6*16 FZB	6	PCS	5707292356000
5	1350007	Pvc bush black 20*10,5*9,0mm	4	PCS	5707292545527
6	177209	Bracket for mud guard 16"	2	PCS	5715393020621
7	1606500	Lock nut 6 mm	2	PCS	5707292141361
8	1106000	Washer 6mm	2	PCS	5707292140371