


meerkat

Exploded views / Part lists

Rev. date (05.2024)


On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.

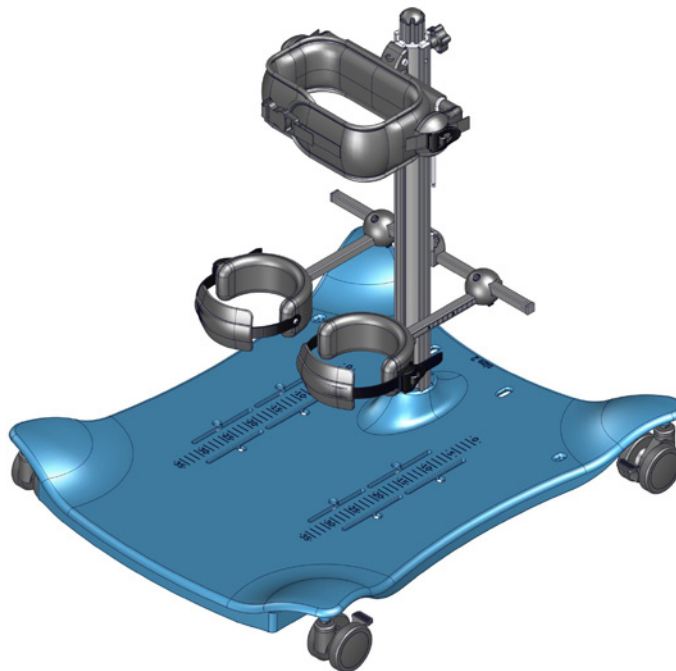

Contents

Base Assembly	3
Item numbers for Base Assembly	3
Column Assembly	4
Item numbers for Column Assembly	5
Chest Support	6
Item numbers for Chest Support	6
Hip Support	7
Item numbers for Hip Support	7
Knee Supports size 1	8
Item numbers for Knee Supports size 1	8
Knee Supports size 2-3	9
Item numbers for Knee Supports size 2-3	9
Sandals	10
Item numbers for Sandals	10
Rocker	11
Item numbers for Rocker	11
Extension Kit	12
Item numbers for Extension Kit	12
Tray plastic grey	13
Item numbers for Tray plastic grey	13
Tray, plastic with bowl and cover	14
Item numbers for Tray, plastic with bowl and cover	15


Base Assembly

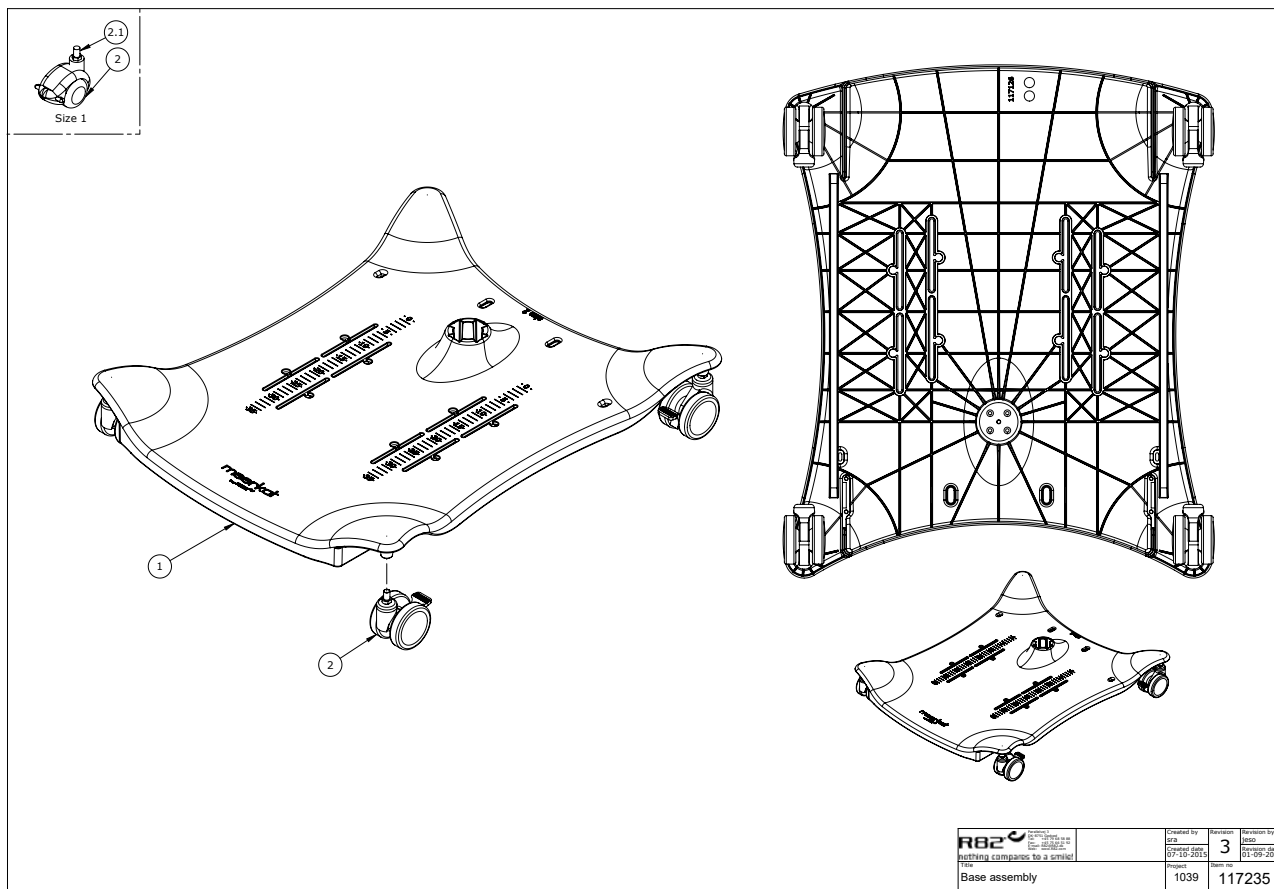

Item numbers for Base Assembly

Position	No.	Description	Quantity	Unit of Measure	Ean code
2	3006820	75mm wheel w/brake, black	4	PCS	5707292371393
2	3035000	Wheel - plane - colour grey	4	PCS	5707292790019
2	3035010	Wheel Cow - white	4	PCS	5707292790026

Column Assembly

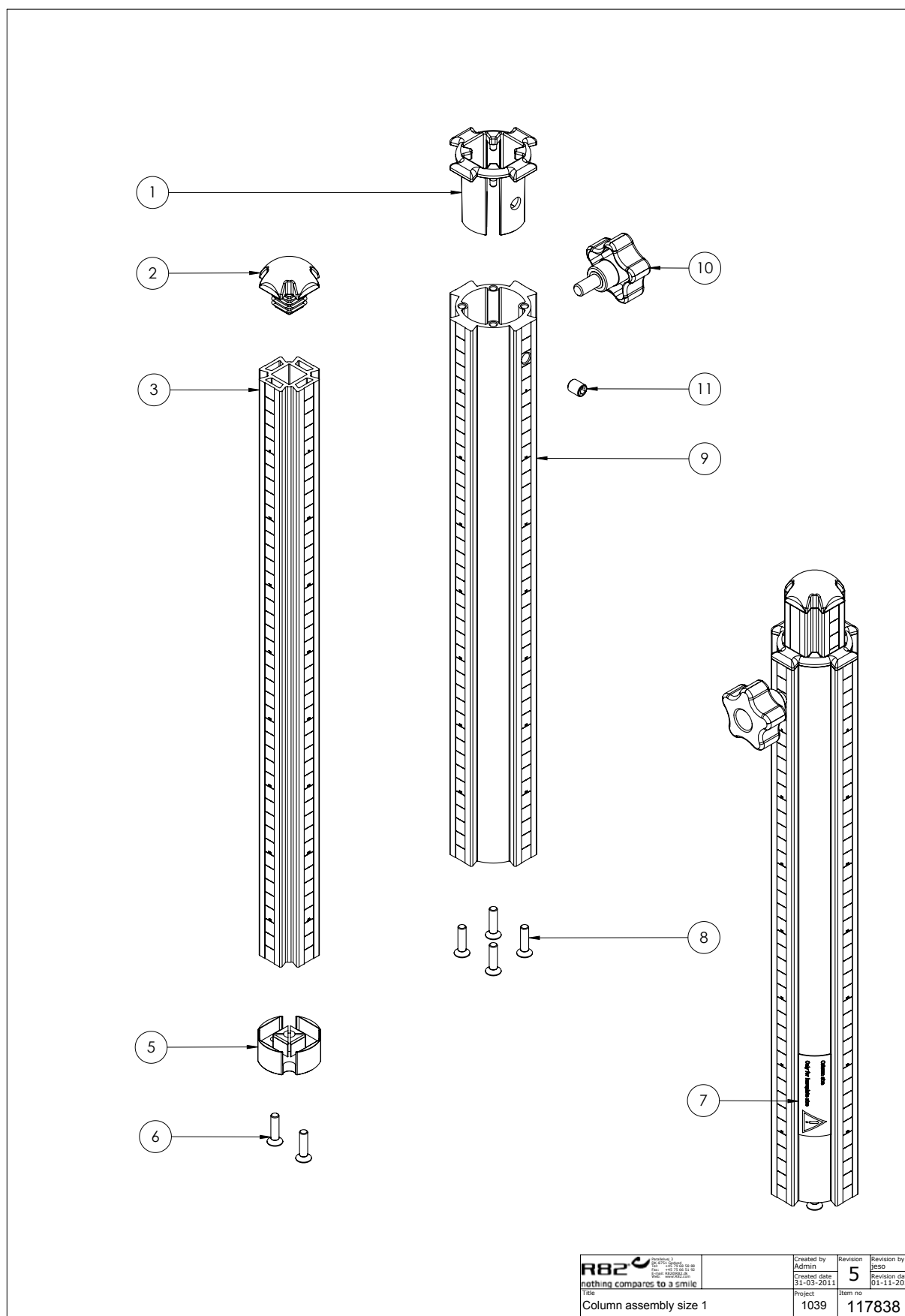


R82 nothing compares to a smile	Created by Admin	Revision 5	Revision by jeso
	Created date 31-03-2011	Item no. 1039	Revision date 01-11-2018
Title Column assembly size 1	Project 1039	Item no. 117838	

C:\PDM\Projects\1039\Columns\117838

Item numbers for Column Assembly

Position	No.	Description	Quantity	Unit of Measure	Ean code
7	0494602	Etiket Column size 2 Meerkat	1	STK	5707292793706
7	0494603	Etiket Column size 3 Meerkat	1	STK	5707292793713
7	0494601	Etiket Column size 1 Meerkat	1	STK	5707292793690


Chest Support

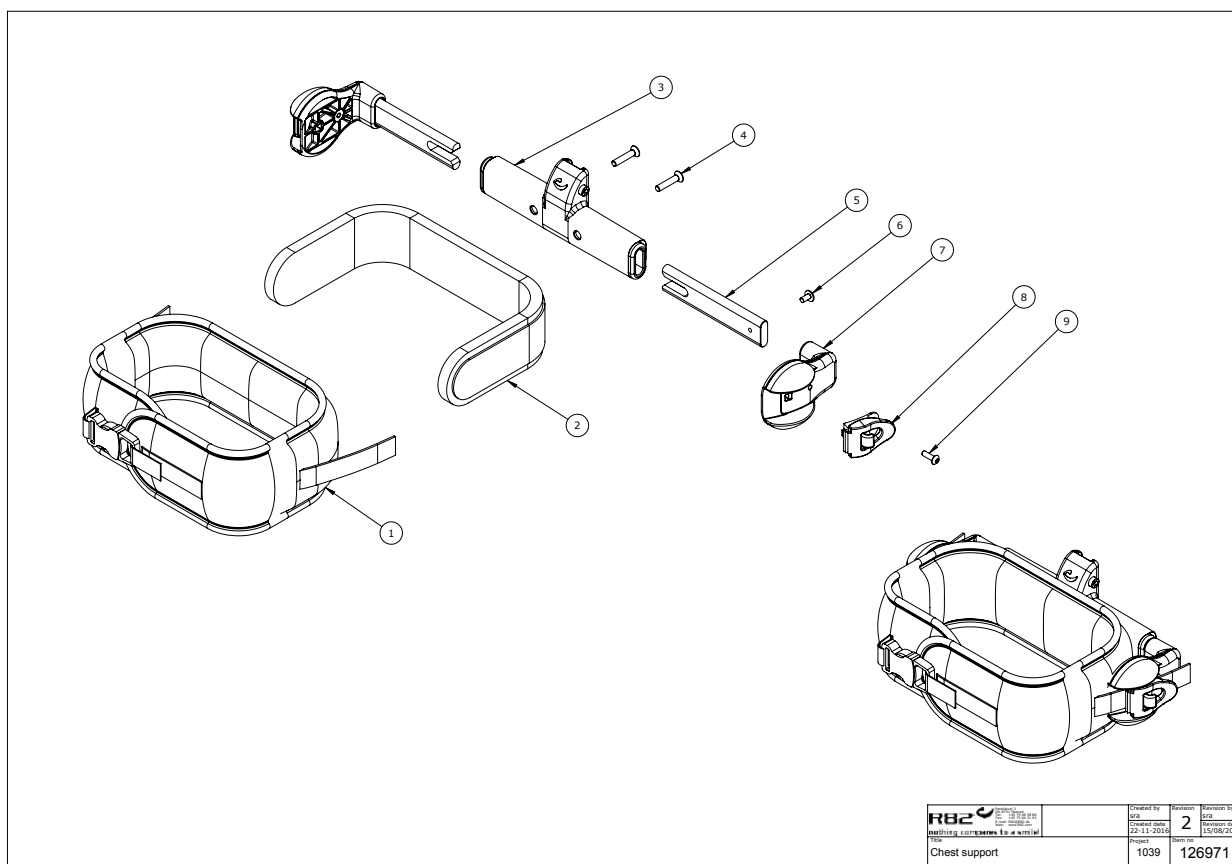

Item numbers for Chest Support

Position	No.	Description	Quantity	Unit of Measure	Ean code
1+2	869480-1	Belt, chest support sz. small	1	PCS	5707292796530
1+2	869481-1	Belt, chest support sz.medium	1	PCS	5707292797230
1+2	869482-1	Belt, chest support sz.large	1	PCS	5707292797247
6	0640913	Button head/flange M5*10 FZB	2	PCS	5707292558985
7	121395	Hip strap mounting bracket	2	PCS	5707292780140
8	0454005	Buckle 24mm, black	2	PCS	5707292786425
9	1102780	Phillips screw	2	PCS	5707292800183


Hip Support

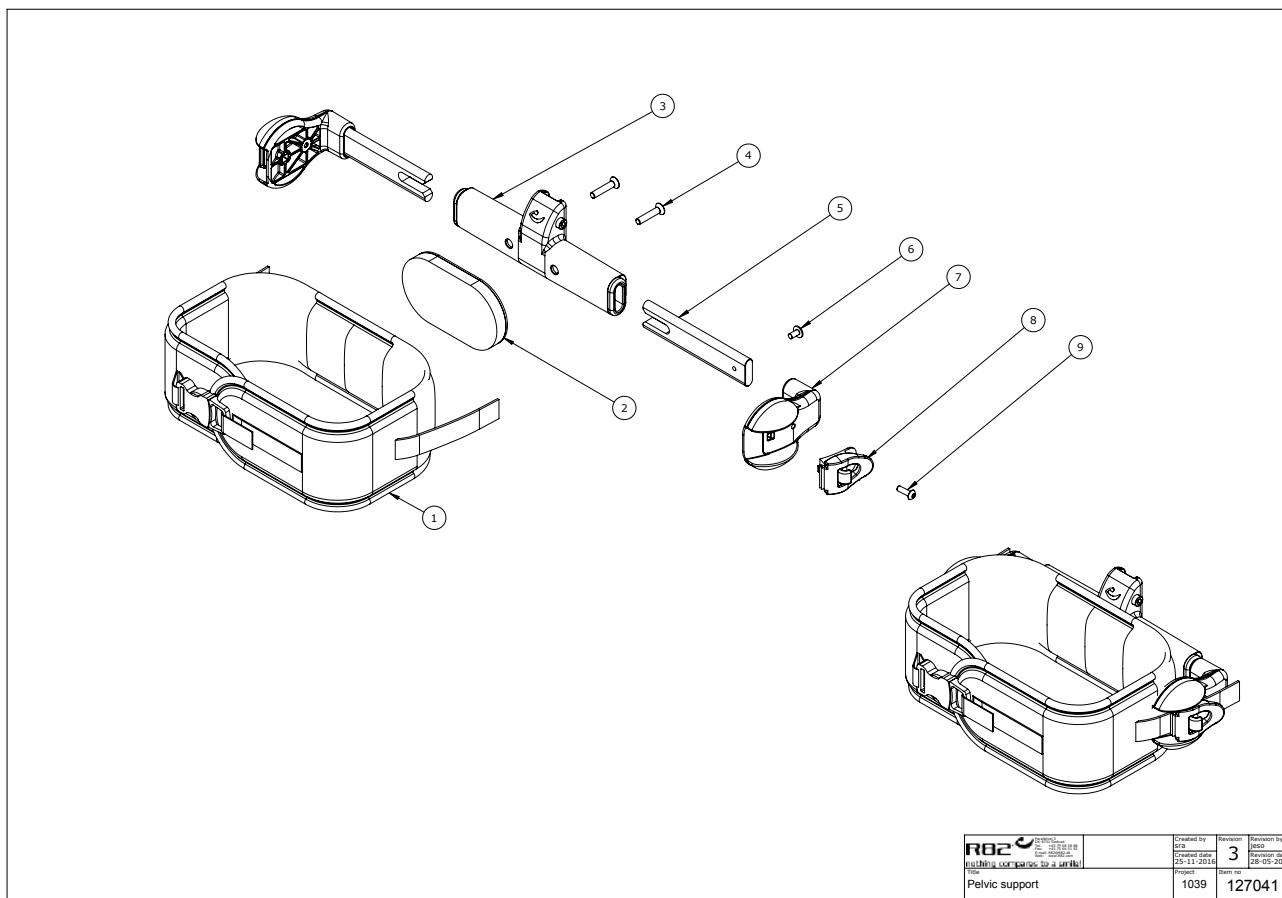

Item numbers for Hip Support

Position	No.	Description	Quantity	Unit of Measure	Ean code
1+2	869240-1	Belt, hip support sz. small	1	PCS	5707292797254
1+2	869241-1	Belt, hipsupport sz. medium	1	PCS	5707292797261
1+2	869242-1	Belt, hipsupport sz. large	1	PCS	5707292797278
6	0640913	Button head/flange M5*10 FZB	2	PCS	5707292558985
7	121395	Hip strap mounting bracket	2	PCS	5707292780140
8	0454005	Buckle 24mm, black	2	PCS	5707292786425
9	1102780	Phillips screw	2	PCS	5707292800183


Knee Supports size 1

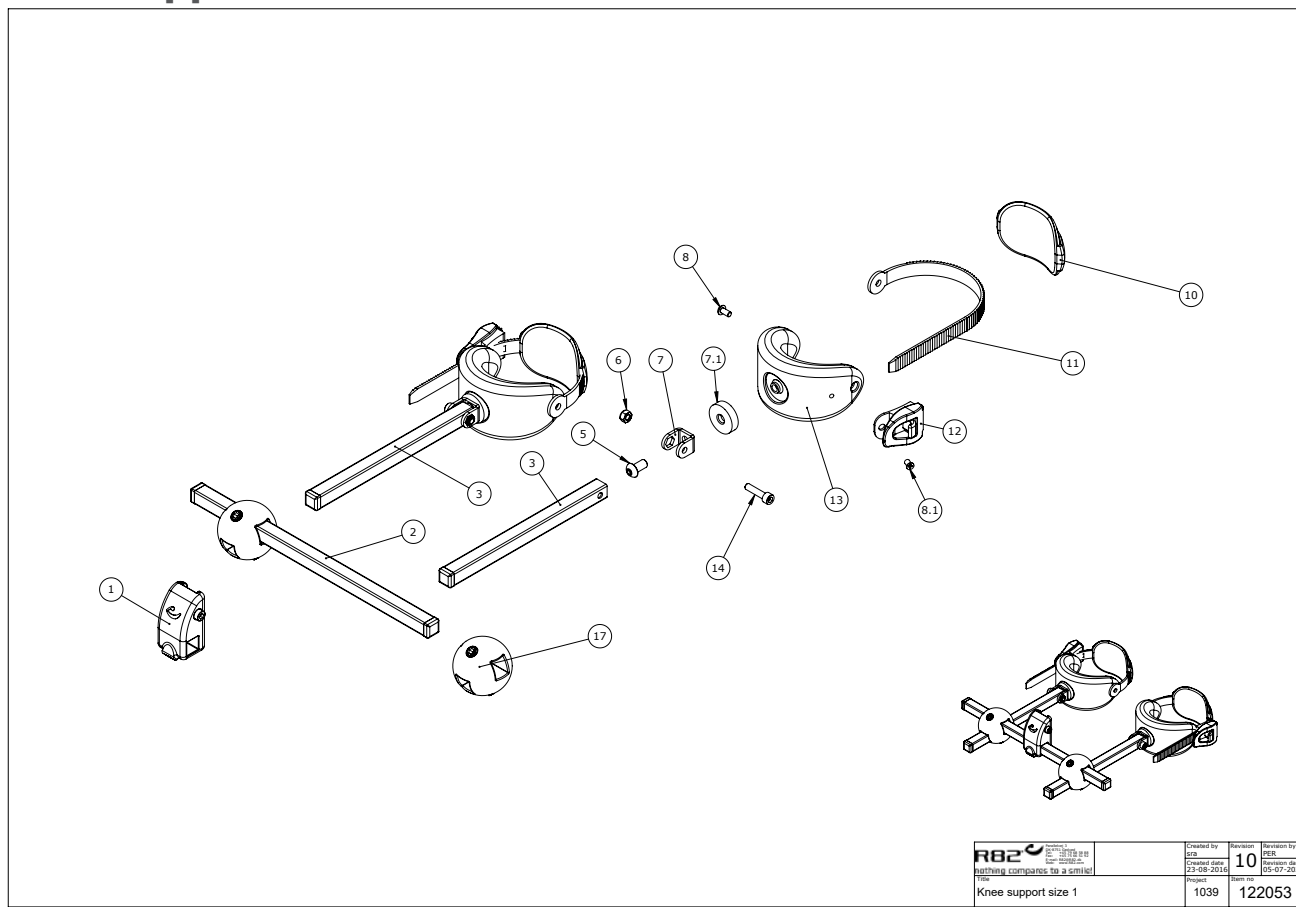

Item numbers for Knee Supports size 1

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	117832	Column mount complete	1	PCS	5707292779885
8	0641000	Button head screw 6*10	2	PCS	5707292139238
8.1	1102781	UH screw	2	PCS	5707292836465
10	127068	Knee pad size 1	2	PCS	5707292783431
11	0454010	Strap/ratchet 19*350mm, black	2	PCS	5707292786456
12	0454000-1	Buckle 19mm, black bukket	2	PCS	5707292915436
13	135310	Knee support cup size S	2	PCS	5707292829290


Knee Supports size 2-3

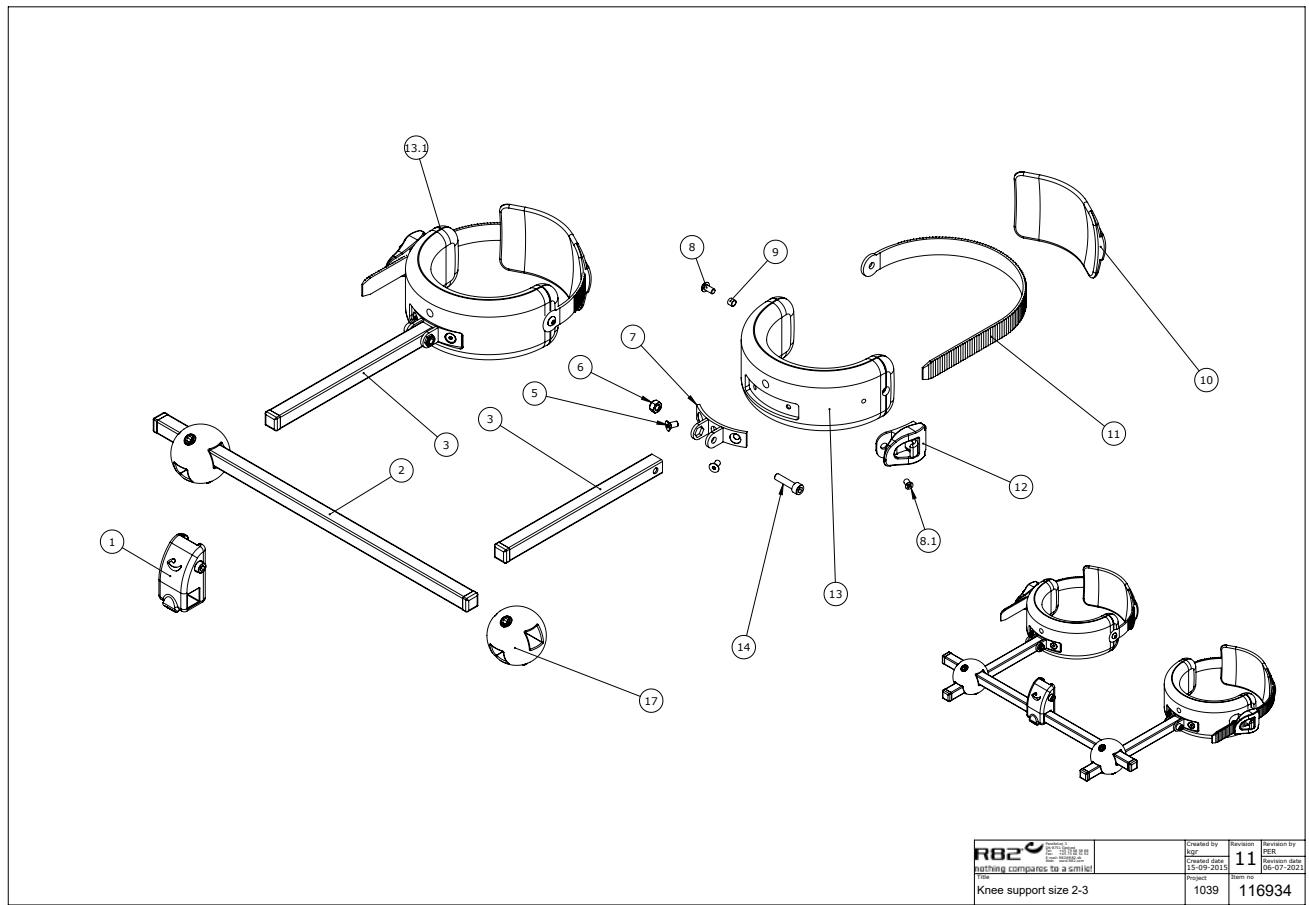

Item numbers for Knee Supports size 2-3

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	117832	Column mount complete	1	PCS	5707292779885
8	0658010	Button head M5 x 10, stainless	2	PCS	5707292139375
8.1	1102781	UH screw	2	PCS	5707292836465
9	121133	Distance bush	2	PCS	5707292780096
10	127069	Knee pad size 2	2	PCS	5707292783448
10	127070	Knee pad size 3	2	PCS	5707292783455
11	0454010	Strap/ratchet 19*350mm, black	2	PCS	5707292786456
12	0454000-1	Buckle 19mm, black bucket	2	PCS	5707292915436
13	135311	Knee support cup size M right	1	PCS	5707292829306
13	135313	Knee support cup size L right	1	PCS	5707292829337
13.1	135312	Knee support cup size M left	1	PCS	5707292829313
13.1	135314	Knee support cup size L left	1	PCS	5707292829320


Sandals

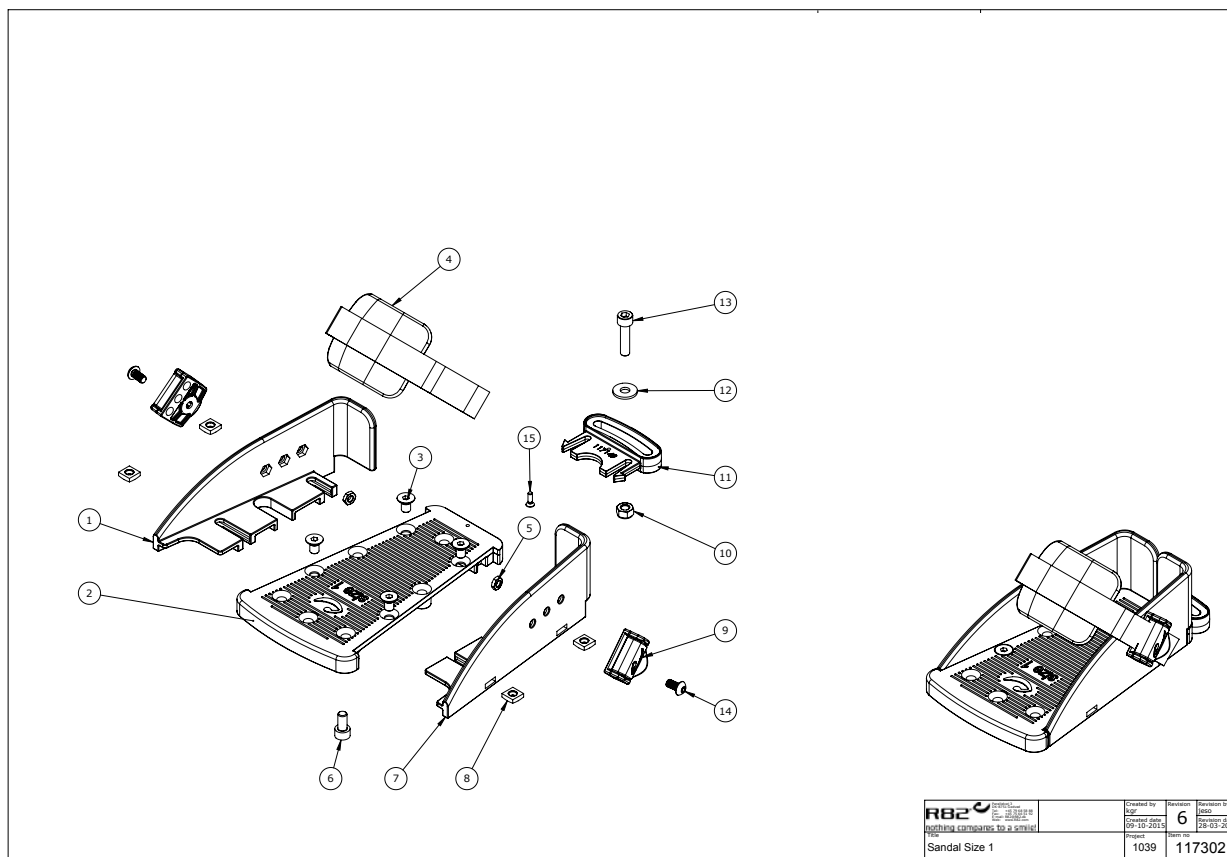

Item numbers for Sandals

Position	No.	Description	Quantity	Unit of Measure	Ean code
4	127085	Foot strap size 1+2	2	PCS	5707292783585
4	127086	Strap size 3	2	PCS	5707292783424


Rocker

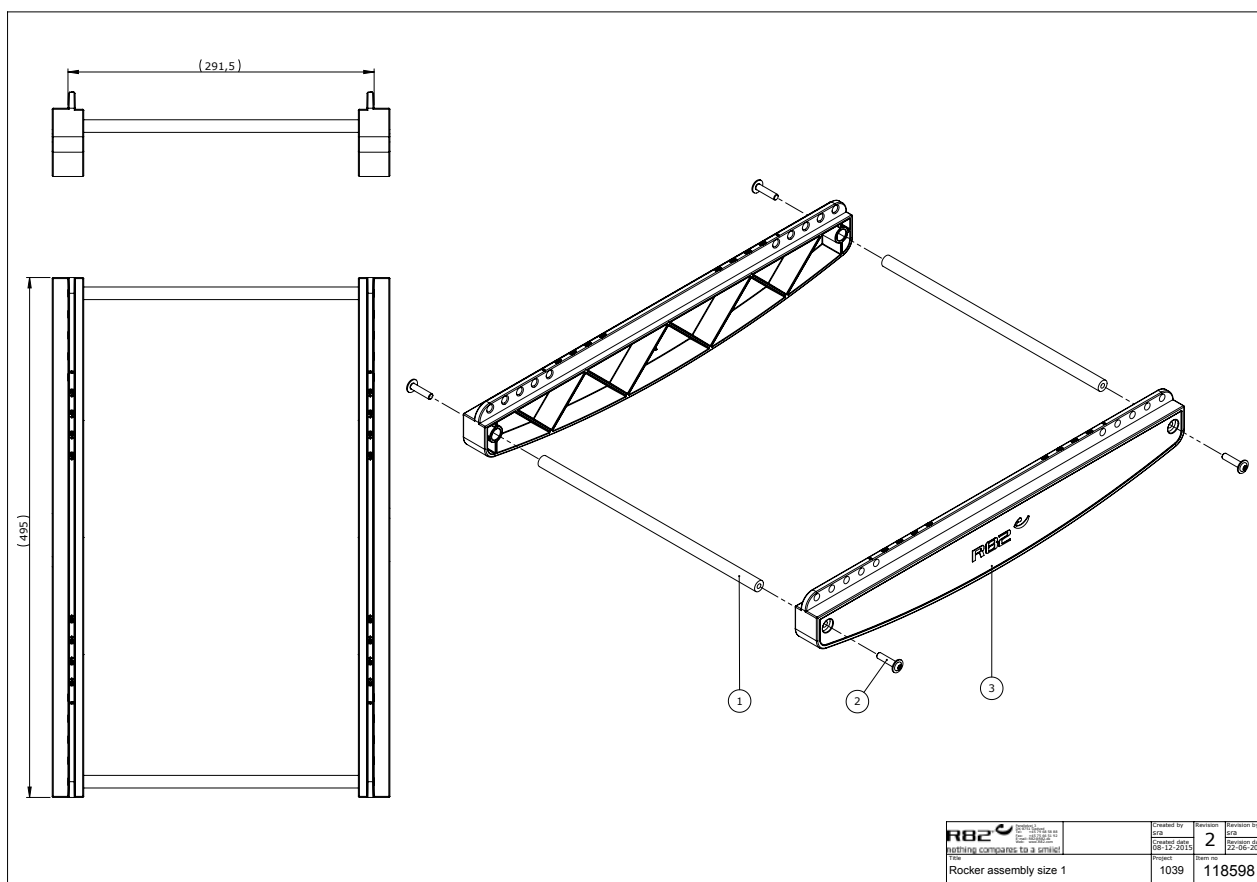

Item numbers for Rocker

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	117049	Rocker frame cross bar size 2	2	PCS	5707292780003
1	118599	Rocker frame cross bar size 1	2	PCS	5707292779991
2	0641621	Button head w/flange M6*20 FZB	4	PCS	5707292360960
3	117048	Rocker size 2	2	PCS	5707292779984
3	118717	Rocker Size 1	2	PCS	5707292779977


Extension Kit

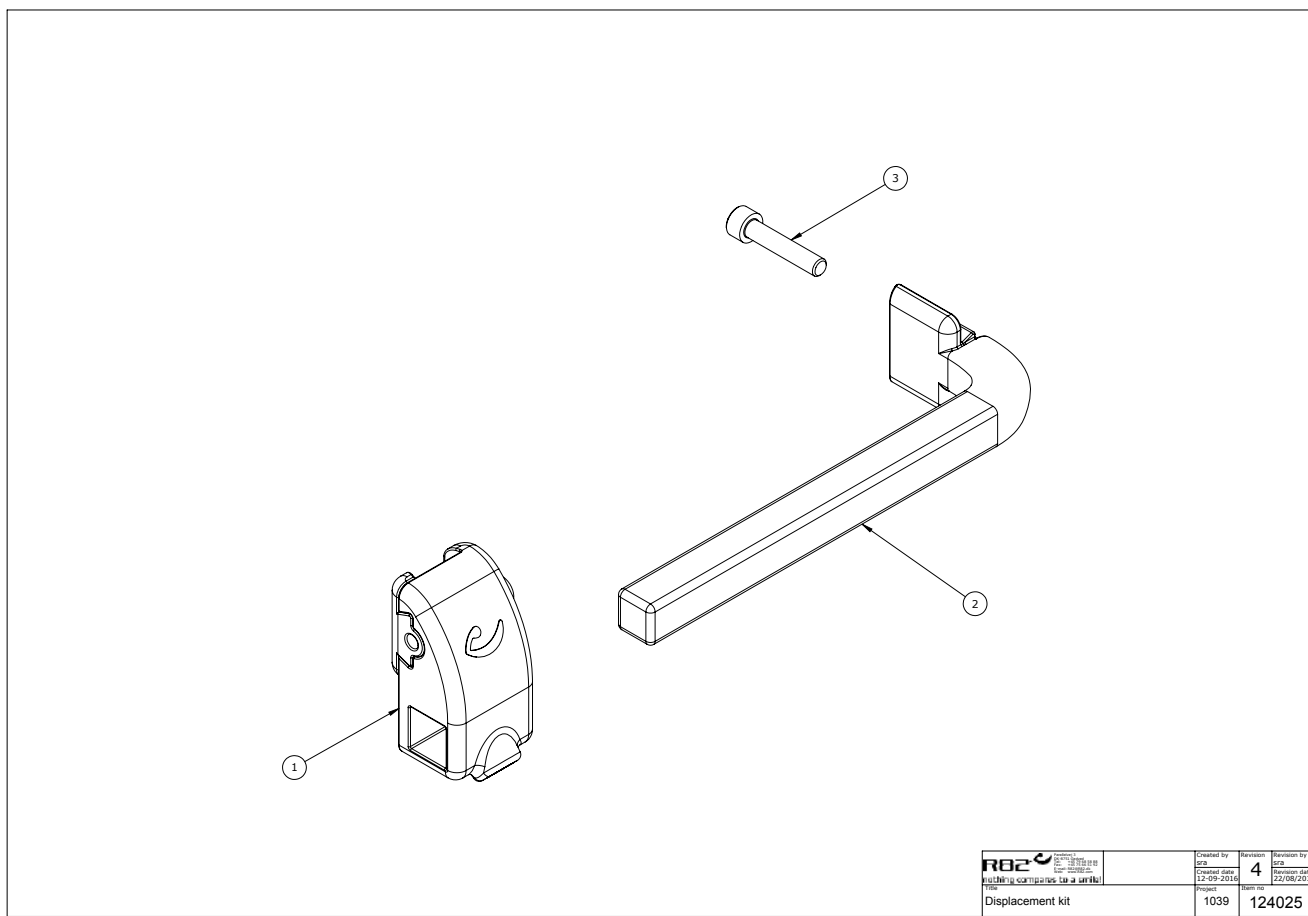

Item numbers for Extension Kit

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	117832	Column mount complete	1	PCS	5707292779885


Tray plastic grey

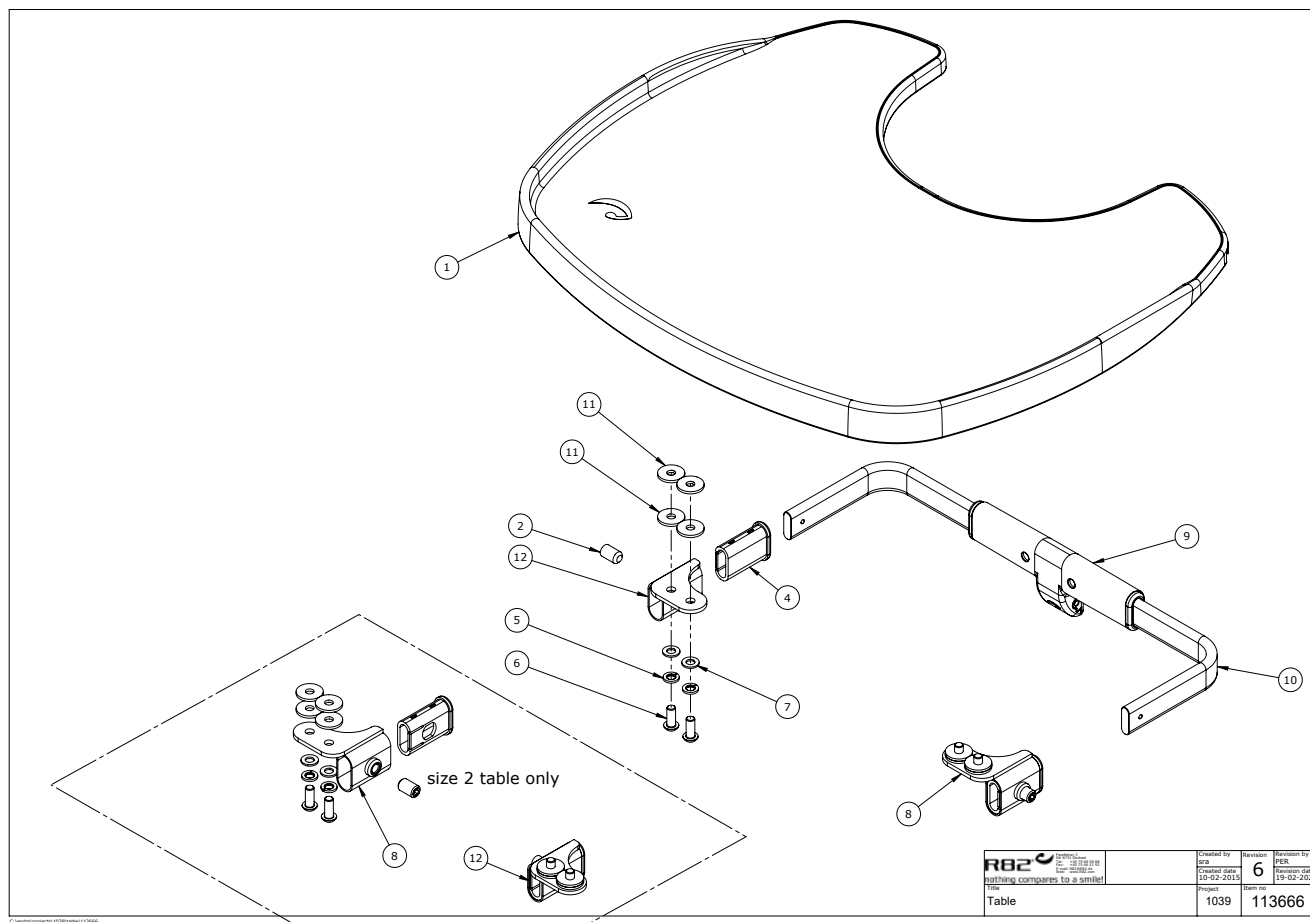

Item numbers for Tray plastic grey

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	75362	Table size 2	1	PCS	5707292456298
1	75361	Table size 1	1	PCS	5707292455994
2	0701010	Set screw M10*10 w/flat end	2	PCS	5707292434098
4	112630	Bush for Hip support	2	PCS	5707292593405
5	1101000	Center spring washer 8mm	4	PCS	5707292140173
6	0662000	Button head screw 8*20 stainl.	4	PCS	5707292139573
7	1103200	Fazet washer 8mm	4	PCS	5707292140296
8	117898	Table Bracket right	1	PCS	5707292780256
9	117819	Column clamp size 2-3	1	PCS	5707292780133
10	117901-1	Arm for table bracket	2	PCS	5707292874719
11	1310705	Nylon washer,white,Ø25*8,4*2,5	8	PCS	5707292140791
12	117897	Table bracket left	1	PCS	5707292780249

Tray, plastic with bowl and cover

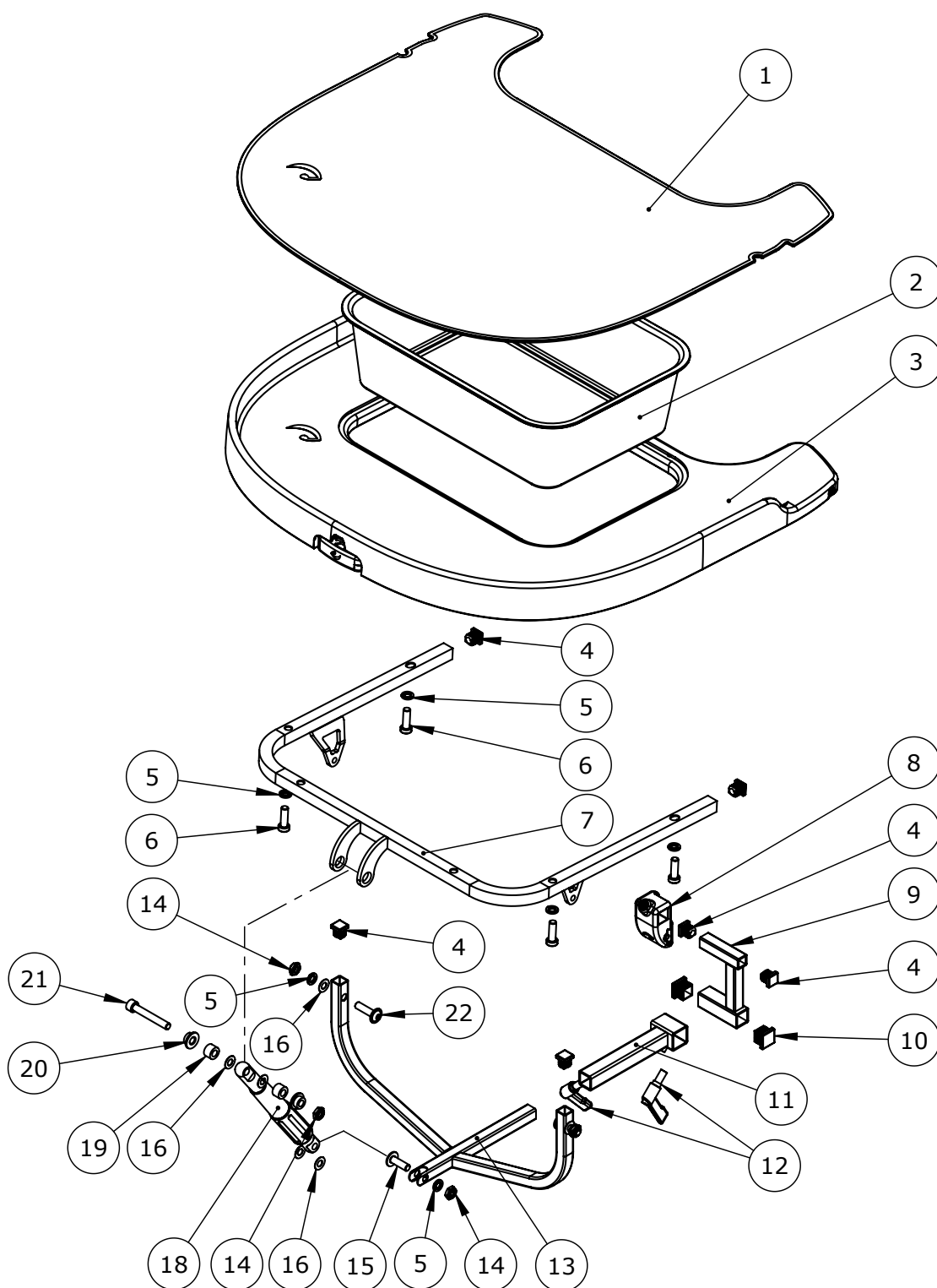

Item numbers for Tray, plastic with bowl and cover

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	75401	Table cover	1	PCS	5707292505934
2	75402	Tub for table	1	PCS	5707292505965
3	5075400	Table, plast	1	PCS	5707292518675
4	2504000	Ferrule e15*1,0-2,0 int.black	6	PCS	5707292144102
5	1100500	Spring washer 8mm, stainless	7	PCS	5707292140142
6	0603535	Insex low head M8*25	4	PCS	5707292575319
7	75405	Center support bracket	1	PCS	5707292507235
8	117832	Column mount complete	1	PCS	5707292779885
9	173140	Mounting for table welded	1	PCS	5707292995391
10	2504500	Ferrule e20*1,0-2,0 int.black	2	PCS	5707292144126
11	167052	Tray support bracket	1	PCS	5707292969279
12	3502516	Tightening handle 45mm M8*16	2	PCS	5707292146915
13	166801	Tray support frame	1	PCS	5707292969262
14	1607002	Lock nut 8mm, A2	4	PCS	5707292410955
15	0662501	Button head w/flange 8*25	1	PCS	5707292403759
16	1310645	Washer, nature Ø16*8*0,5mm	6	PCS	5707292446015
18	107634	Spindle for angle adj.	1	PCS	5707292572004
19	1350012	Afs.styk. Ø15 x Ø8,2 x 10 mm	2	PCS	5707292904393
20	72878	Bush for holder	2	PCS	5707292418678
21	0603600	Insex screw 8*55	1	PCS	5707292138262
22	0663031	Button head 8*30, Wombat	2	PCS	5707292139610