

serval


Exploded views / Part lists

Rev. date (05.2010)



Contents

Seat and back.....	3
Frame.....	6
Accessories	9

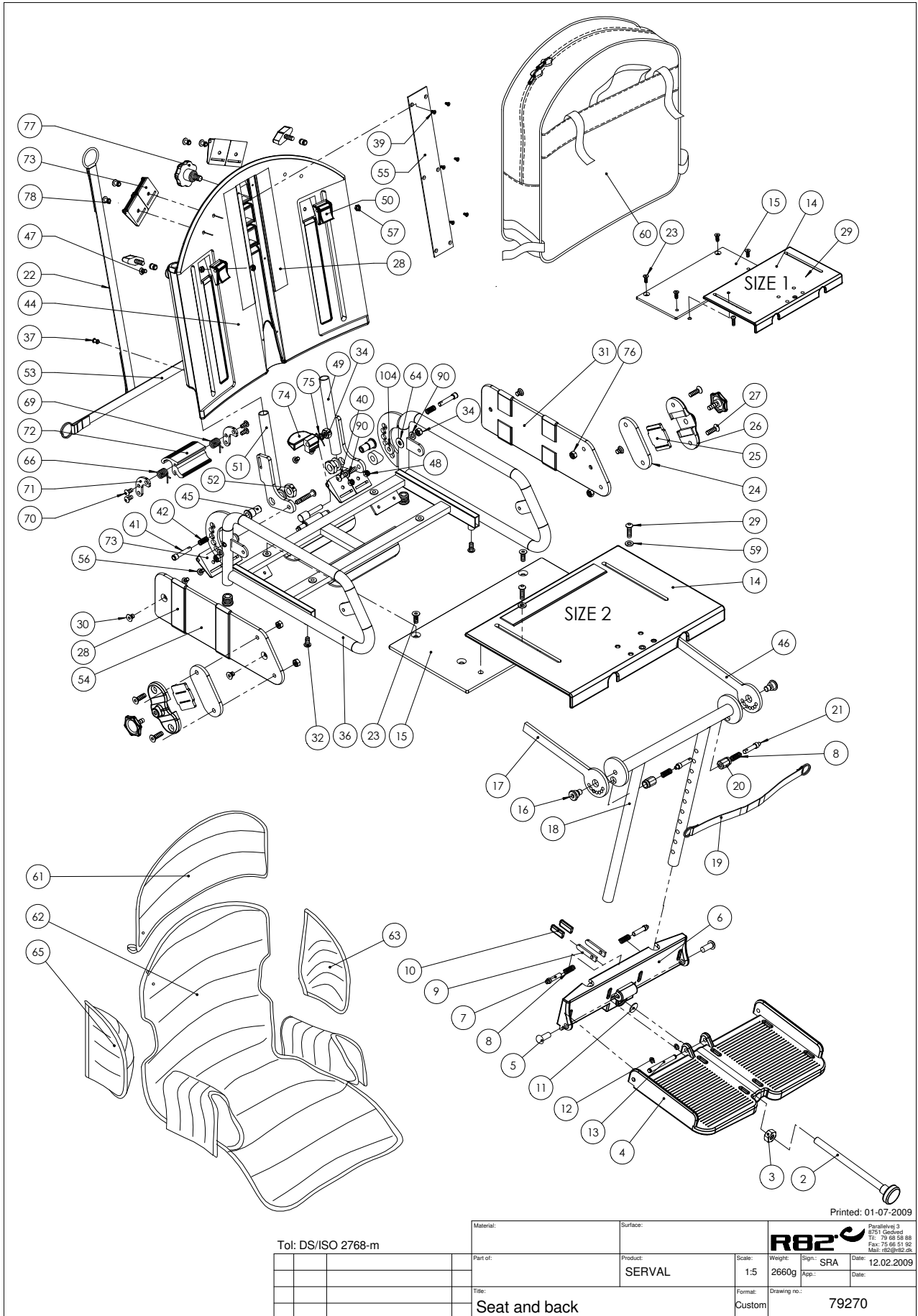

Printed: 01-07-2009


Tol: DS/ISO 2768-m

Material:	Surface:	Parallelsvej 3 8751 Gedved Tlf: 75 66 58 88 Fax: 75 66 51 92 Mail: r82@r82.dk			
Part of:	Product:	Scale:	Weight:	Sign:	Date:
	SERVAL	1:5	2660g	SRA	12.02.2009
Title:	Format:	Drawing no.:			
Seat and back	Custom	79270			

Position	No.	Description	Quantity	Unit of Measure	Ean code
2*	3502760	Handle 28mm, M10*141mm thread	PIECES	1	5707292363121
	3502770	Handle 28mm, M10*155mm thread	PIECES	1	5707292363138
3	1607500	Lock nut 10 mm	PIECES	1	5707292141385
4*	70108	Footplate size 1	PIECES	1	5707292101419
	70117	Footplate size 2	PIECES	1	5707292101501
5	0506520	Rivet 8*20 Alu	PIECES	2	5707292137395
6*	70107	Heel support, size 1	PIECES	1	5707292101402
	70116	Heel support, size 2	PIECES	1	5707292101495
7	6108174	Lock pin	PIECES	2	5707292185563
8	0310122	Compression spring 0,9*8U	PIECES	4	5707292135544
9	6108176	Mounting f.release handle	PIECES	2	5707292185570
10	3501850	Black PVC cap 5,5*24MM	PIECES	2	5707292146779
11	1358000	Distance part 16*1,25	PIECES	1	5707292141033
12	1145040	Lock washer 4MM, stainless	PIECES	2	5707292140562
13	6108178	Pin Ø6*58	PIECES	1	5707292185587
14*	70852	Adjust.seat, size 1	PIECES	1	5707292103239
	70850	Adjust.seat, size 2	PIECES	1	5707292103215
15*	70851	Solid seat size 1	PIECES	1	5707292103222
	70849	Solid seat size 2	PIECES	1	5707292103208
16	6109002	Screw M8	PIECES	2	5707292466754
17*	70184	Mounting f.footsupport, right	PIECES	1	5707292101747
	70877	Fitting for footrest size 2	PIECES	1	5707292103482
18*	70853	Seat extension size 1	PIECES	1	5707292103246
	70854	Seat extension size 2	PIECES	1	5707292103253
19*	5550070	Release strap for footsupport	PIECES	1	5707292178381
	5550060	Release strap for footsupport	PIECES	1	5707292178374
20	6108222	Springhouse	PIECES	2	5707292185747
21	6108172	Lock pin	PIECES	2	5707292185556
22*	5550010	Release band for back size 1	PIECES	1	5707292358936
	5550020	Release band for back size 2	PIECES	1	5707292358950
23	0611600	Insex screw counters 6*12	PIECES	4	5707292138705
24	80572	Mounting 40*3*85	PIECES	2	5707292111227
25	82408	Underlying plate, PVC armrest	PIECES	2	5707292124043
26	72134	Mounting for armrest	PIECES	2	5707292105899
27	0611000	Insex screw counters 6*20	PIECES	4	5707292138682
28*	4913035	30mm black PS14 with glue hook	METER	0,94	5707292469755
	4913035	30mm black PS14 with glue hook	METER	0,4	5707292469755
29*	0645625	Button head screw 6*25	PIECES	2	5707292139313
	0645600	Button head screw 6*16 BLACK	PIECES	2	5707292139290
30	0610800	Insex unders.M6*8,ZN/FE&black	PIECES	4	5707292138675
31*	70846	Side plate, size 1	PIECES	1	5707292103178
	70845	Side plate, size 2	PIECES	1	5707292103161
32	0645600	Button head screw 6*16 black	PIECES	2	5707292139290
33	0506506	Rivet TAS D 68 BS 5,0x10,9	PIECES	4	5707292395795
34	1606500	Lock nut 6 MM	PIECES	3	5707292141361
36*	70834-54	Seat frame, welded, size 1	PIECES	1	5707292381255
	70835-54	Seat frame, welded, size 2	PIECES	1	5707292381491
37	0506519	Stainless blind rivet 4*12	PIECES	2	5707292137388
39	1940010	Plastite 60, 2,9*6,5	PIECES	6	5707292142573

40	0645635	Button head screw 6*35 black	PIECES	2	5707292139337
41	70848	Pin for back reclining	PIECES	2	5707292103192
42	0310100	Pressure spring 0,9*8*23*10,5V	PIECES	2	5707292135513
44*	70091	Back size 1	PIECES	1	5707292358929
	70092	Back size 2	PIECES	1	5707292358912
45	70847	Bush for back	PIECES	2	5707292103185
46	70899	Mounting for left foot support	PIECES	1	5707292103673
47	0631860	Insex 5*10, FZB	PIECES	2	5707292139191
48	1606300	Lock nut 5 MM	PIECES	4	5707292141354
49	70832-54	Left back tilt, welded	PIECES	1	5707292381248
50	0442060	Fixlock 20MM, black w/hole	PIECES	2	5707292136374
51	70831-54	Right back tilt, welded	PIECES	1	5707292381231
52	1761200	Versus nut 12*1, fine threaded	PIECES	2	5707292141545
53*	5550050	Girtband f.foot support	PIECES	1	5707292178367
	5550095	Girtband f.foot support, size 2	PIECES	1	5707292178404
54*	70843	Side plade, size 1	PIECES	1	5707292103147
	70844	Side plade, size 2	PIECES	1	5707292103154
55*	70030	Cover plate, size 1	PIECES	1	5707292358134
	70031	Cover plate, size 2	PIECES	1	5707292358141
56	0658010	Button head M5 X 10, stainless	PIECES	4	5707292139375
57	1760005	Compensating nut 8MM	PIECES	2	5707292141491
59	1130600	Stainless washer 6MM	PIECES	2	5707292140517
60*	0581015	R82 Bag, black, size 1	PIECES	1	5707292137692
	0581020	R82 Bag, black, size 2	PIECES	1	5707292137708
61*	87451-115	Cover for backrest ext., size 1, blue	PIECES	1	5707292385925
	87451-215	Cover for backrest ext., size 2, blue	PIECES	1	5707292385932
62*	87473-115	Cover for Serval size 1, blue	PIECES	1	5707292385901
	87473-215	Cover for Serval size 2, blue	PIECES	1	5707292385918
63*	87456-115	Cover for seat side size 1, blue	PIECES	1	5707292386021
	87456-215	Cover for seat side size 2, blue	PIECES	1	5707292386052
64	1101950	Washer ø6,1-16*1 MM, FZB	PIECES	2	5707292140210
65*	87456-115	Cover for seat side size 1, blue	PIECES	1	5707292386021
	87456-215	Cover for seat side size 2, blue	PIECES	1	5707292386052
66	0310508	Torsion bar stainless, right	PIECES	1	5707292135759
69	0310510	Torsion bar stainless, left	PIECES	1	5707292135766
70	0658020	Button head M5 x 20, stainless	PIECES	4	5707292139405
71	70808	Bracket for snap lock	PIECES	2	5707292102850
72	70842	Handle for lock	PIECES	1	5707292103130
73	75306	Snap lock R82 25	PIECES	4	5707292437624
74	70930	Lock handle	PIECES	1	5707292376893
75	0310516	Twist spring for lock handle	PIECES	1	5707292375896
76	1605000	Stainless nut M6	PIECES	4	5707292141316
77	3503001	R82 handle M8*15MM screw	PIECES	1	5707292413925
78*	0506513	Stainless blind rivet Ø4, 8*12	PIECES	4	5707292137326
	0506506	Rivet TAS D 68 BS 5,0x10,9	PIECES	4	5707292395795
90	1103000	Fazet washer 6MM	PIECES	4	5707292140289
104	1125618	Screen washer FZ6*18*3MM	PIECES	2	5707292140494

*		Choose size
---	--	-------------

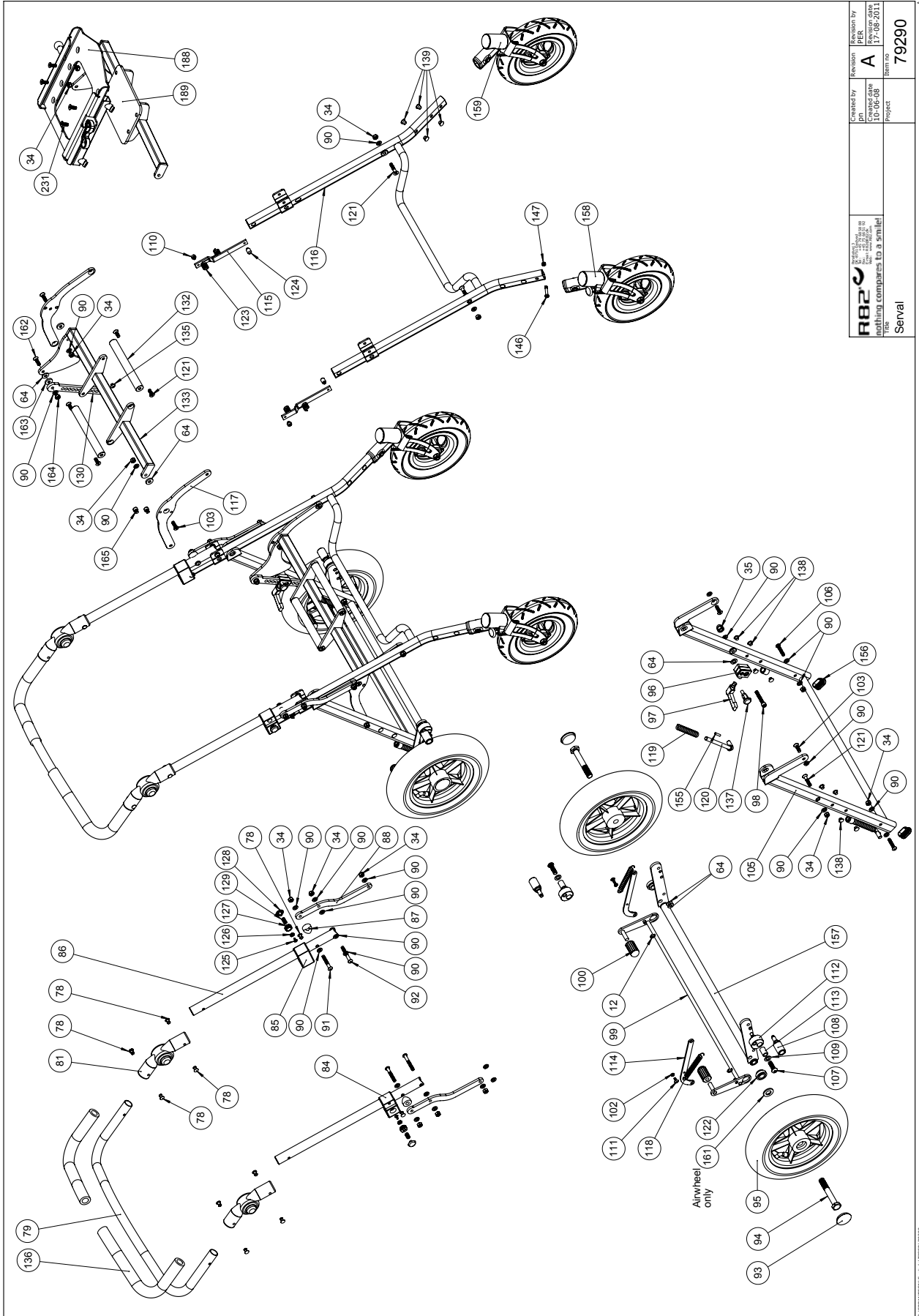



<p>R82 Serval Sveinbjørn A/S Sveinbjørnvej 3 DK-8270 Lyngby</p>	Created by dn Approved date 15/08/2018 Project Serval	Revision PER A Revision by PER Date 15/08/2018 Item no. 79290
	Title Serval	

Position	No.	Description	Quantity	Unit of Measure	Ean code
12	1145040	Lock washer 4MM, stainless	PIECES	2	5707292140562
34	1606500	Lock nut 6 MM	PIECES	14	5707292141361
35	1780500	Top nut M6, FZB, lock	PIECES	1	5707292475688
64	1101950	Washer ø6,1-16*1 MM, FZB	PIECES	8	5707292140210
78	0506508	Rivet 5*14, stainless	PIECES	10	5707292359476
79*	70302	Hand bar, size 1	PIECES	1	5707292483348
	70209	Hand bar, size 2	PIECES	1	5707292483331
81	0553090	Locking joint Ø20MM	PIECES	2	5707292482235
84	70905	Lock fitting, right	PIECES	1	5707292103710
85	70904	Lock fitting, left	PIECES	1	5707292103703
86	70206-54	Tube for handbrace, Painted	PIECES	2	5707292483232
87	1357000	Distance piece black 6*20*6,4MM	PIECES	2	5707292141019
88	70751-54	Z-mounting	PIECES	2	5707292381330
90	1103000	Fazet washer 6MM	PIECES	29	5707292140289
91	0645640	Button head M6*40 ZN/black	PIECES	2	5707292139344
92	0645645	Button head M6*45 ZN/black	PIECES	2	5707292139351
93	2704300	Black hub cap for wheel	PIECES	2	5707292144515
94	1902380	Steel set screw 12*80MM	PIECES	2	5707292142030
95*	3009502	250*50 Air wheel	PIECES	2	5707292483379
	3009503	250*50 Solid wheel	PIECES	2	5707292483386
96	70938-54	Mounting for locking seat tilt	PIECES	1	5707292396570
97	3502115	Tighning handle M6X15MM	PIECES	1	5707292146854
98	0602437	Insex M6*37 12.9 zinkflage	PIECES	1	5707292476203
99*	70890	Bar for foot brace size 1	PIECES	1	5707292103581
	70891	Bar for foot brake size 2	PIECES	1	5707292103598
100	2522005	Coating for foot brake, right	PIECES	2	5707292144447
102	1602800	Steel nut 5MM, FZB	PIECES	1	5707292141262
103	0611000	Insex screw counters. 6*20	PIECES	4	5707292138682
105*	70919-54	Back frame, welded, size 1	PIECES	1	5707292381279
	70927-54	Back frame, welded, size 2	PIECES	1	5707292381460
106	0645635	Button head screw 6*35 black	PIECES	2	5707292139337
107	0662500	Button head screw 8*25 stainless	PIECES	2	5707292139580
108	70892	Brake block	PIECES	2	5707292103604
109	1105080	Stainless washer 8MM	PIECES	2	5707292140364
110	70914	Spring knop	PIECES	2	5707292103802
111	0658010	Button head M5 X 10, stainless	PIECES	1	5707292139375
112	70895	Brake excentric	PIECES	2	5707292103635
113	70893	Distance bush for excentric	PIECES	2	5707292103611
114	70885	Pull arm	PIECES	1	5707292103550
115	70912	Release arm	PIECES	2	5707292103789
116*	70213-54	Frame, coated, size 1	PIECES	1	5707292483300
	70214-54	Frame, coated, size 2	PIECES	1	5707292483324
117	70782-54	U-fitting	PIECES	2	5707292527318
118	0310220	Tension spring O1,2*10*50MM	PIECES	2	5707292135735
119	0310161	Tension spring O2,5*13*80MM*20	PIECES	1	5707292135698
120	70829	Arm for spring, welded	PIECES	1	5707292103048
121	0612600	Insex screw counters M6*30	PIECES	3	5707292138729
122	70894	Distance bush for wheel	PIECES	2	5707292103628
123	0310515	Double spring f.lock	PIECES	2	5707292135803

124	70915	Spring knop	PIECES	2	5707292103819
125	0683250	Insex M3*8	PIECES	1	5707292139672
126	6108340	Washer for release button	PIECES	1	5707292185877
127	82557	Release house	PIECES	1	5707292124746
128	82559	Release button	PIECES	1	5707292124760
129	0308710	RF. spring 0.5*8*10MM*4V	PIECES	1	5707292135483
130	70937	Rack for seat recline	PIECES	1	5707292396525
132	70800	Seat control	PIECES	2	5707292102782
133	70780-54	Seat tilt, welded, size 1	PIECES	1	5707292381286
135	0631960	Button head screw 6*6 black	PIECES	1	5707292139214
136	2500555	Cover f.hand bar Ø20 - 280MM	PIECES	2	5707292483522
137	0380182	Locking house, complete M10*1	PIECES	1	5707292357953
138	1310533	Covering stopper, black O6,4MM	PIECES	8	5707292374691
139	1310528	Covering stud, black 13MM	PIECES	8	5707292470348
146	0603522	Insex CH M5*22 RF	PIECES	2	5707292490988
147	1606300	Lock nut 5 MM	PIECES	2	5707292141354
155	0588014	Rustl.tightening spring 3*14	PIECES	1	5707292137753
156	2703000	Oval ferrule FO 30*15 black	PIECES	2	5707292144492
157	70777-54	Axel welded, size 1	PIECES	1	5707292381347
158*	5009501	190*50 Air wheel, right	PIECES	1	5707292483409
	5009503	190*50 Solid wheel, right	PIECES	1	5707292483423
159*	5009500	190*50 Air wheel, left	PIECES	1	5707292483393
	5009502	190*50 Solid wheel, left	PIECES	1	5707292483416
161	1103500	Fazet washer 12MM	PIECES	2	5707292140326
162	0612500	Insex screw counters. 6*25	PIECES	1	5707292138712
163	1125618	Screen washer FZ6*18*3MM	PIECES	1	5707292140494
165	0602187	Insex M6*6, FZB	PIECES	2	5707292527325
188	508911610	T-frame x:panda, serval mounting	PIECES	1	5707292484222
189	70208-54	Seat tilt painted, size 2	PIECES	1	5707292483263
231	0611600	Insex screw counters. 6*12	PIECES	4	5707292138705

*		Choose size			
---	--	-------------	--	--	--

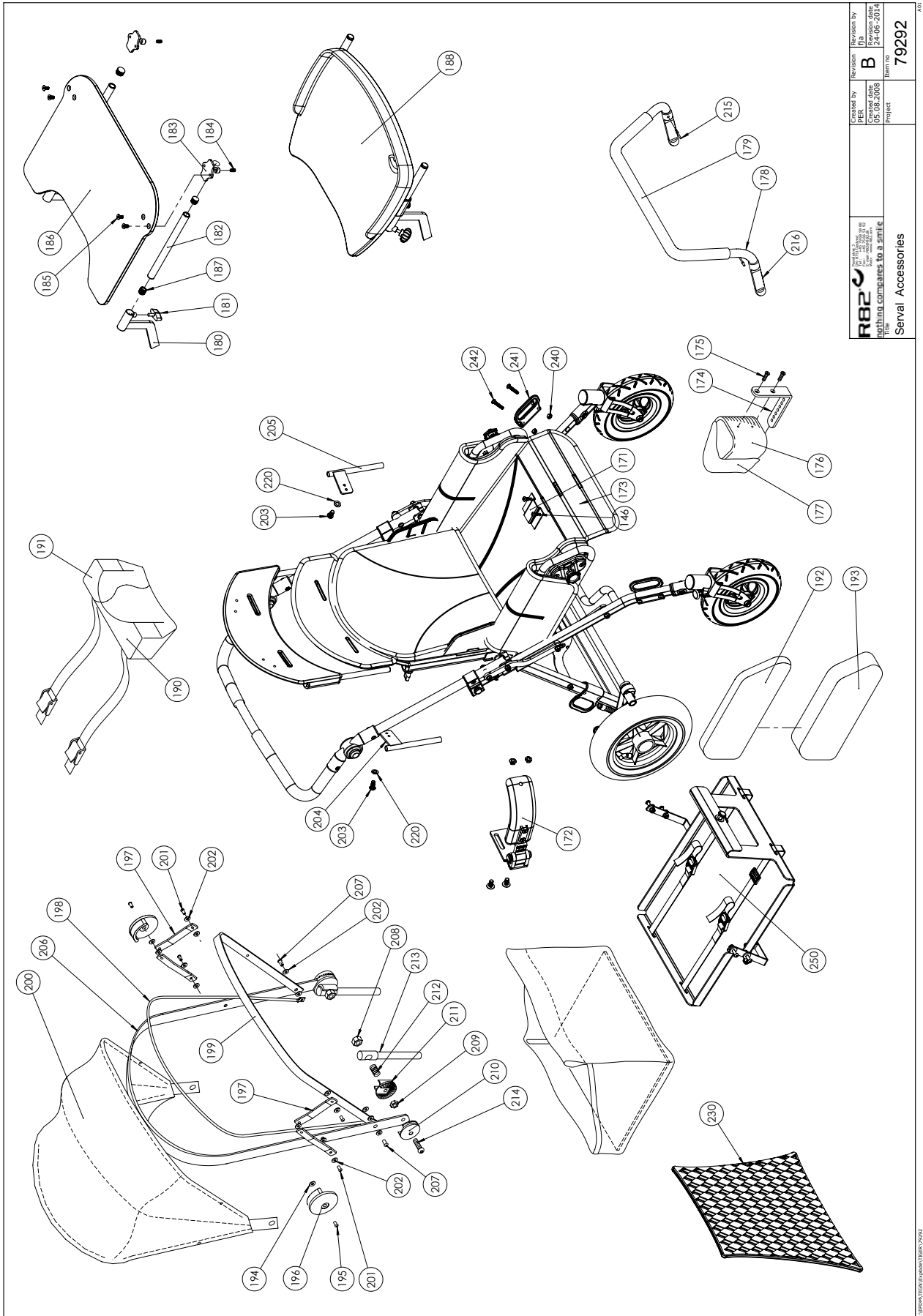

 R82	Created by PER	Revision B	Revision by JFB
	Date 01.08.2008	Date 12.08.2008	Date 12.08.2008
Project Serval Accessories		Item no. 79292	

Position	No.	Description	Quantity	Unit of Measure	Ean code
146	0641200	Button head screw 6*12	PIECES	4	5707292139252
157	4916000	30MM.black velcro,APLIX 800LOOP	METER	0,3	5707292151896
171	70230	Holder for pommel	PIECES	1	5707292101860
172	83111-160	Swing-away side support	PIECES	1	5707292437099
173	3470601	R82 handle M6*35MM screw	PIECES	1	5707292413918
174	70235	Angle mounting for pommel	PIECES	1	5707292101877
175	1811275	Cutting screw 4*30MM	PIECES	2	5707292141743
176*	6175111	Pommel size 1, black PUR	PIECES	1	5707292189103
	6175112	Pommel size 2, black PUR	PIECES		5707292189110
177*	89120-115	Cover for pommel, blue, size 1	PIECES	1	5707292386069
	89120-116	Cover for pommel, blue, size 2	PIECES		5707292386069
178*	70837	Brace, size 1	PIECES	1	5707292103086
	70836	Brace, size 2	PIECES		5707292103079
179*	2500630	Rubber 15*5*47CM, black, size 1	PIECES	1	5707292143808
	2500633	Rubber 15*5*47CM, black, size 2	PIECES		5707292143815
180	70861	Holder for tray	PIECES	2	5707292103321
181	3652501	R82 handle M6*16MM screw	PIECES	2	5707292413963
182	70866	Pipe ø16*1,5	PIECES	2	5707292103376
183	70255	Mounting for table	PIECES	2	5707292101976
184	0700072	Stop screw M6*12 MM	PIECES	2	5707292139825
185	0621200	Insex screw 6*12MM	PIECES	4	5707292138958
186	70864	Table top, size 1	PIECES	1	5707292103352
187	2501600	Ferrule D15*1,0 black	PIECES	4	5707292143969
190*	87445-115	Head support cushion, size 1	PIECES		5707292385840
	87445-215	Head support cushion, size 2	PIECES		5707292385857
191*	87446-115	Cover for head support cushion, size 1	PIECES	1	5707292385864
	87446-215	Cover for head support cushion, size 2	PIECES	1	5707292385871
192*	87477-1	Side cushion, 3 cm, size 1	PIECES	1	5707292254078
	87477-12	Side cushion, 2 cm, size 1	PIECES	1	5707292254085
193*	87477-2	Side cushion, 3 cm, size 2	PIECES	1	5707292254092
	87477-22	Side cushion, 2 cm, size 2	PIECES	1	5707292254108
194	1102900	Fazet washer 5 MM	PIECES	10	5707292140272
195	0506514	Stainless ferrule Ø5*18MM	PIECES	2	5707292137333
196	70287	Cover for hood	PIECES	2	5707292102171
197	70277	Arm for folding top	PIECES	4	5707292102119
198*	70871	Fitting 3 for folding top, size 1	PIECES	1	5707292103420
	70874	Fitting 3 for folding top, size 2	PIECES		5707292103451
199*	70870	Fitting 1 for folding top, size 1	PIECES	1	5707292103413
	70873	Fitting 1 for folding top, size 2	PIECES		5707292103444
200*	5026991	Cover for folding top, blue, size 1	PIECES	1	5707292157188
	5026993	Cover for folding top, blue, size 2	PIECES		5707292157201
201	0506519	Stainless blind rivet 4*12	PIECES	4	5707292137388
202	1102800	Fazet washer 4MM	PIECES	18	5707292140265
203	0650800	Button head screw 8*16 black	PIECES	2	5707292139368
204	70280	Mounting for hood, right	PIECES	1	5707292102133
205	70281	Mounting for hood, left	PIECES	1	5707292102140
206*	70869	Fitting 2 for folding top, size 1	PIECES	1	5707292103406
	70872	Fitting 2 for folding top, size 2	PIECES		5707292103437
207	0506518	Stainless blind rivet 4*16	PIECES	2	5707292137371

208	1606300	Lock nut 5 MM	PIECES	2	5707292141354
209	1760005	Compensation nut 8MM	PIECES	2	5707292141491
210	0450681	Plast washer for bar, flat	PIECES	2	5707292136480
211	0450693	Plast washer for bar, round	PIECES	2	5707292136497
212	0310155	Spring 1*7*15mm*8,5v	PIECES	2	5707292135674
213	70875	Mounting for folding top	PIECES	2	5707292103468
214	0640930	Button head M5*30 FZB	PIECES	2	5707292139221
215	0506509	Stainless rivet 4*7 MM	PIECES	2	5707292472441
216	71130-60	Black bush	PIECES	2	5707292223586
220	1102000	Planing washer 8MM	PIECES	2	5707292140227
230	84082	Shopping net	PIECES	1	5707292126511
240	1631060	Stainless lock nut 6MM	PIECES	8	5707292141446
241	82872	Belt fitting	PIECES	4	5707292373397
242	0659580	Button head screw 6*30 MM	PIECES	8	5707292139481

*		Choose size			
---	--	-------------	--	--	--