




# Spare part list - EN

## Molift Raiser Pro

***molift***<sup>®</sup>  
by Etac

SPL43001 - 2025-09-01 - Rev. 1.0

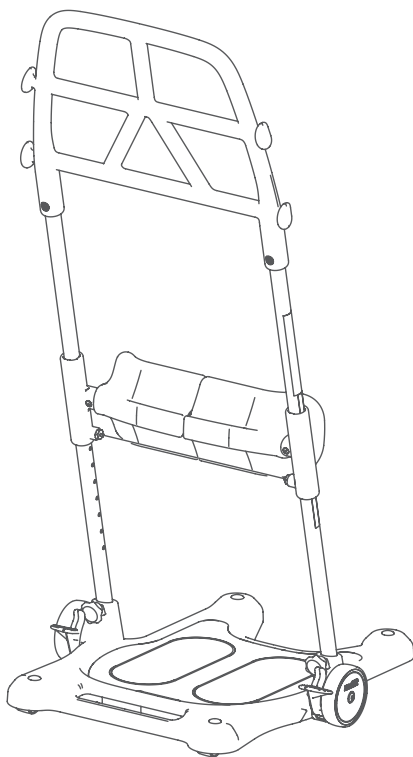

[www.etac.com](http://www.etac.com)



Fig.1


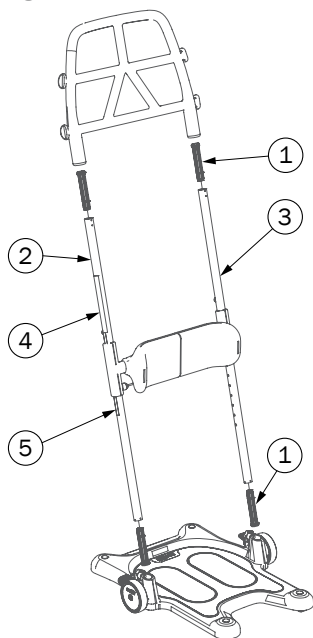


Fig.2

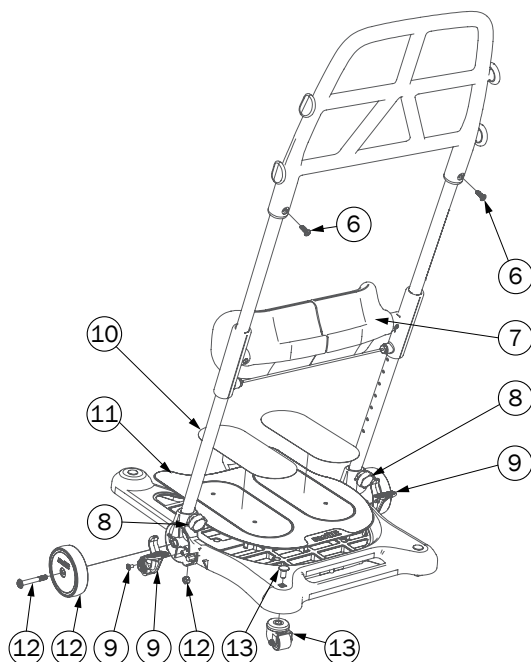

| Fig. | Pos. | Title                                              | Item no. |
|------|------|----------------------------------------------------|----------|
| 1    | 1    | Black plastic piece incl lock nut, 2 pairs         | 0990507  |
|      | 2    | Raiser PRO right pipe complete                     | 83361    |
|      | 3    | Raiser PRO left pipe complete                      | 83362    |
|      | 4    | User label, pipe                                   | 72117    |
|      | 5    | Height label, pipe                                 | 72118    |
| 2    | 6    | Screw set handle, 2 pcs                            | 28472    |
|      | 6    | Grease for brake pedal                             | 28473    |
|      | 6    | Socket for replacing castor (tool)                 | 28465    |
|      | 7    | Legsupport, complete                               | 83219    |
|      | 8    | Knobset, 2 pcs                                     | 28469    |
|      | 9    | Brake pedal set, left/right incl screws and grease | 28463    |
|      | 10   | Non-slip footboard kit                             | 1630060  |
|      | 11   | Footplate                                          | 28464    |
|      | 12   | Wheel 100 mm, 2 pcs incl screws and nuts           | 28462    |
|      | 13   | Castor wheel 35 mm, set of 4 incl screws           | 28461    |