

wombat

living size 3

Exploded views / Part lists

Rev. date (02.2020)

On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.



Contents

Wombat living Size 3

Gas/Pedal

Gas/Pedal3

Tilt on brace 5

Frame Gas/Pedal.....7

Power

Power..... 10

Power Tilt on Brace 12

Frame Powered 14

Lift/Tilt on Brace

Lift/Tilt on brace 16

Lift on Brace 18

Frame Lift on Brace 20

Subassemblies


Back 22


Seat 24

Seat/Cable Tilt..... 26

Footplate Size 1 and 2..... 28


Footplate Size 3 29

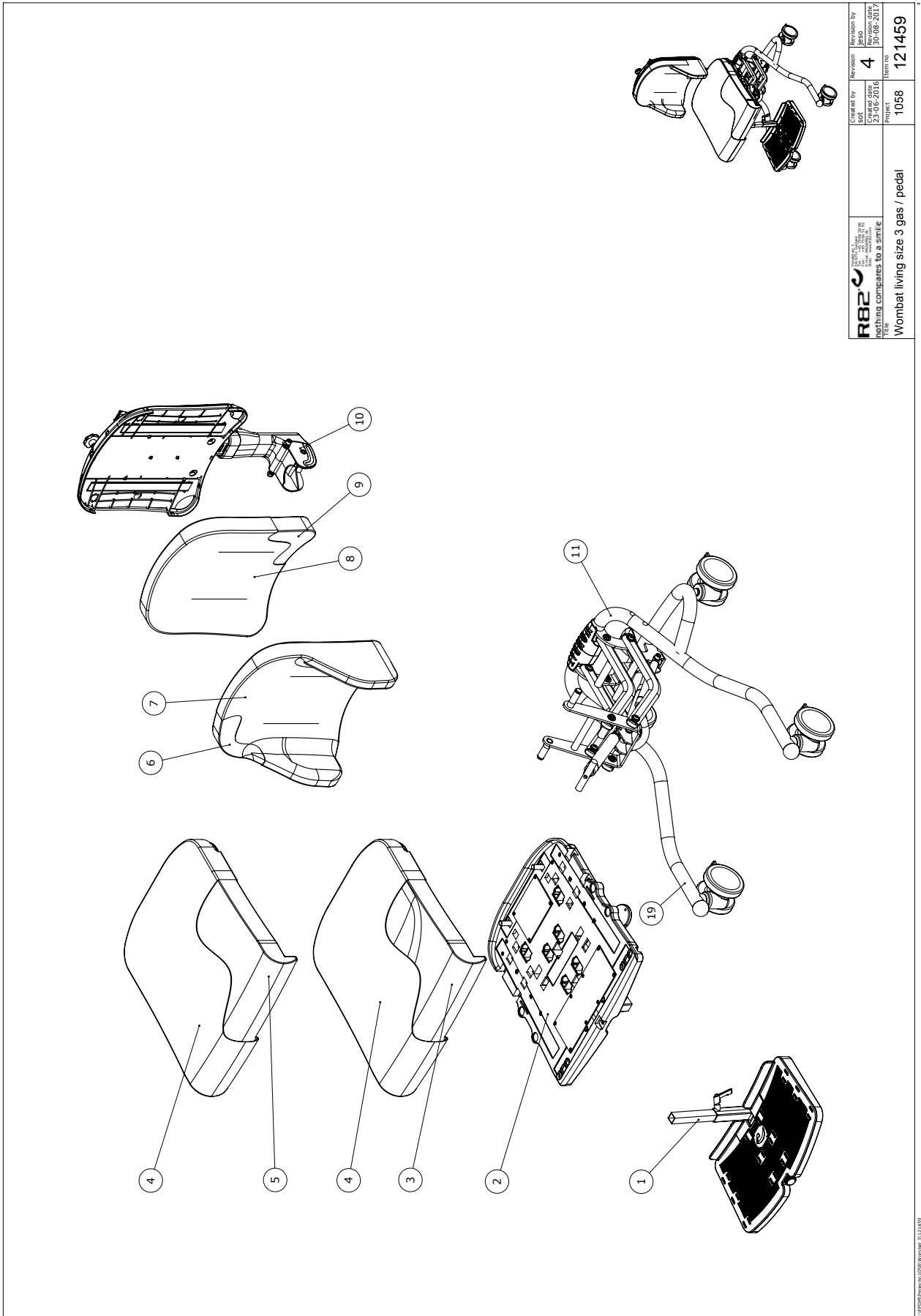

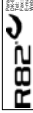
 R82 A/S Parallelvej 3 8751 Gedved Denmark Tel: +45 79 68 58 88 Fax: +45 75 66 51 92 E-mail: R82@R82.dk Web: etac.com	Created by SOT	Revised by JESD
	Change date 13-08-2014	Revision date 13-08-2017
Part name Wombat living size 3 gas / pedal	Project 1058	Item no. 121459

© 1999/2004/2008/2017 Wombat Living 3121459


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	121099-3	Footrest Wombat size 3	STK	1	5707292623034
2	121386	Wombat living seat str 3	STK	1	5707292623942
4 to 9	See Pricelist				
10	121391	Wombat Living back size 3	STK	1	5707292624161
11	See Page 7				

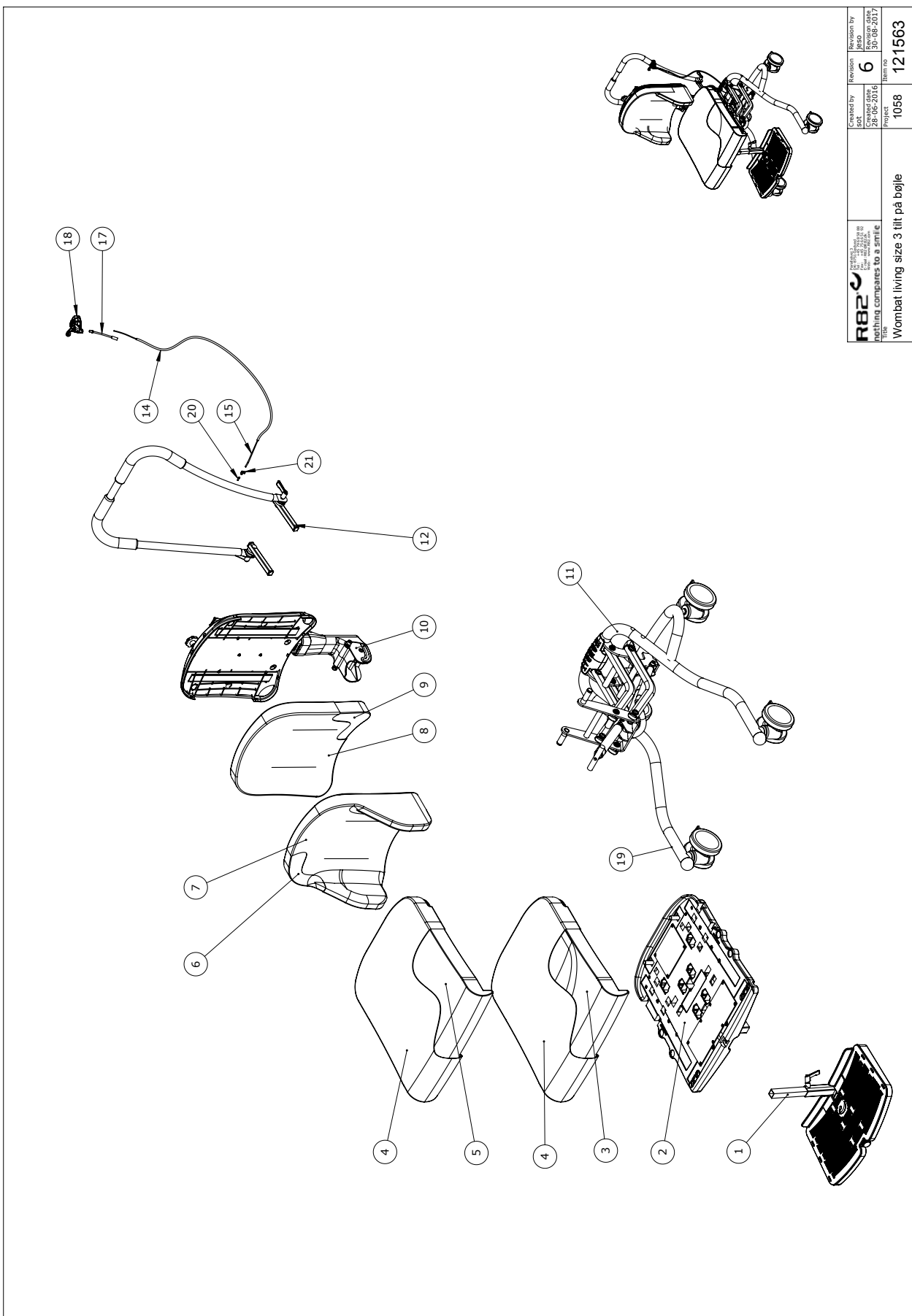

 nothing compares to a smile Title:	Created by: J. Jørgensen 28-05-2016	Revision by: J. Jørgensen 30-08-2017
	Project: 1058	Item no: 121563


C:\EPW\Projects\1058\Wombat 3\121563

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	121099-3	Footrest Wombat size 3	PCS	1	5707292623034
2	121435 TILTBØJ	Seat size 3 tilt on brace	PCS	1	5707292624581
3 to 9	See Pricelist				
10	121391	Wombat Living back size 3	PCS	1	5707292624161
11	See page 7				
12	121218	handbrace str 2 complete	PCS	1	5707292624178
14	5002995	Cable for tilt of seat	PCS	1	5707292542724
15	0371854-1	Soft inner cable for Stingray	PCS	1	5707292536945
17	0380060	Flexible v-brake pipe.	PCS	1	5707292425195
18	3503531	Gas pump lever with clamp	PCS	1	5707292413529
19	0494082	Etiket Wombat Living	PCS	1	5707292623294
20	0387000	End nipple for outer casing	PCS	1	5707292136091
21	0388010	Screwnippel 7*7mm	PCS	1	5707292136121
17	0380060	Flexible v-brake pipe.	STK	2	5707292425195
18	3503531	Gas pump lever with clamp	STK	2	5707292413529
20	0387000	End nipple for outer casing	STK	2	5707292136091
21	0388010	Screwnippel 7*7mm	STK	1	5707292136121

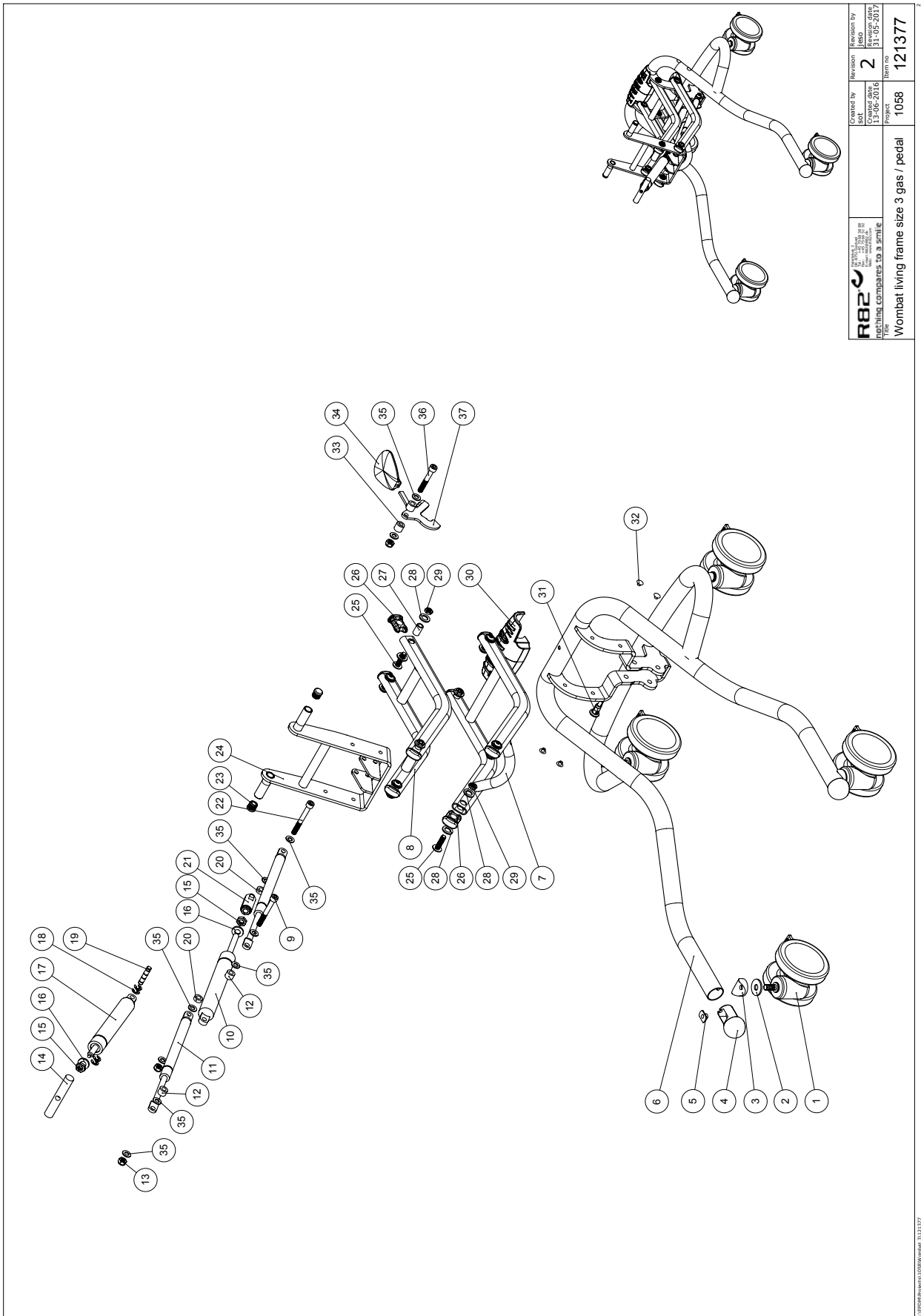


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	3006825	75mm wheel w/brake, black	PCS	4	5707292371393
1	3006938	100mm double wheel with brake	PCS	4	5707292574930
2	1108050	Washer M10, din9021	PCS	2	5707292140401
3	82894	Saddle washer	PCS	2	5707292387578
4+5	5082829-60	Mount of ball and nut	PCS	2	5707292621962
6	82725-10	Frame wombat size 3	PCS	1	5707292622549
6	82725-35	Frame wombat size 3	PCS	1	5707292622556
7	82790-10	Carryarm white ral 9010	PCS	1	5707292622471
7	82790-35	Carryarm Antrazit grey	PCS	1	5707292622495
8	82921-10	Carryarm, Wombat hvid	PCS	1	5707292621993
8	82921-35	Carryarm, Wombat Antrazitgrå	PCS	1	5707292622037
9	603665	Insex screw 8*65	PCS	1	5707292138286
10	2205300	Bloc-o-lift gasspring 5251hs	PCS	1	5707292143389
11	2202100	LIFT-O-MAT 5078UO 250N	PCS	2	5707292400468
12	1350021	Distance piece 11*15*8,2mm	PCS	2	5707292140890
13	1607000	Lock nut 8mm, large pack	PCS	3	5707292141378
14	82729	Shaft	PCS	1	5707292400086
15	1761000	Counter nut 10*1,0 fine thread	PCS	2	5707292141538
16	1310712	Nylon washer, nature	PCS	2	5707292356345
17	2205255	Bloc-o-lift 5085ux 500n fixed	PCS	1	5707292398154
18	1145070	Lock washer 7mm	PCS	4	5707292140586
19	80924	Ø8*40 spec locking pin,wombat	PCS	1	5707292111401
20	1350000	Distancepart black 6*15*8,2 mm	PCS	2	5707292140876
21	6118822	Horizontal release	PCS	1	5707292186744
22	603700	Insex screw 8*70	PCS	1	5707292138293
23	2501600	Ferrule d15*1,0 black	PCS	2	5707292143969
24	82782-10	tipbracket White Ral9010	PCS	1	5707292622570
24	82782-35	tipbracket antrazit grey	PCS	1	5707292622594
25	663031	Button head 8*30, Wombat	PCS	8	5707292139610
26	82946	End piece 15*30, Wombat	PCS	8	5707292126016
27	82933	Bearing bushing, Wombat	PCS	8	5707292125880
28	1310710	Nylon washer	PCS	16	5707292140807
29	1750800	Lock nut, stainless M8	PCS	8	5707292141484
30	82949-60	Bumper for wombat, black	PCS	1	5707292619815
31	380183	Spec. Lockhouse.M10*1, 32mm	PCS	1	5707292136046
32	1310540	Covering pushbutton, black	PCS	4	5707292140692
33	1350023	Distance piece, black	PCS	2	5707292140913
34	3470335	Release button 4*8mm	PCS	1	5707292146632
35	1103200	Fazet washer 8mm	PCS	10	5707292140296
36	603600	Insex screw 8*55	PCS	1	5707292138262
37	82914	Pedal, Wombat	PCS	1	5707292125712
50	603550	Insex screw 8*50	PCS	1	5707292138255
50	1350000	Distancepart black 6*15*8,2 mm	PCS	2	5707292140876
51	1103200	Fazet washer 8mm	PCS	2	5707292140296
51	121699	Release head	PCS	1	5707292629746
52	1103200	Fazet washer 8mm	PCS	2	5707292140296
52	82848	Bush	PCS	2	5707292125484
52	2200050	Small release arm	PCS	1	5707292143075
53	1607000	Lock nut 8mm, large pack	PCS	3	5707292141378


51	121699	Release head	PCS	1	5707292629746,00
52	1103200	Fazet washer 8mm	PCS	2	5707292140296,00
52	82848	Bush	PCS	2	5707292125484,00
52	2200050	Small release arm	PCS	1	5707292143075,00
53	1607000	Lock nut 8mm, large pack	PCS	3	5707292141378,00

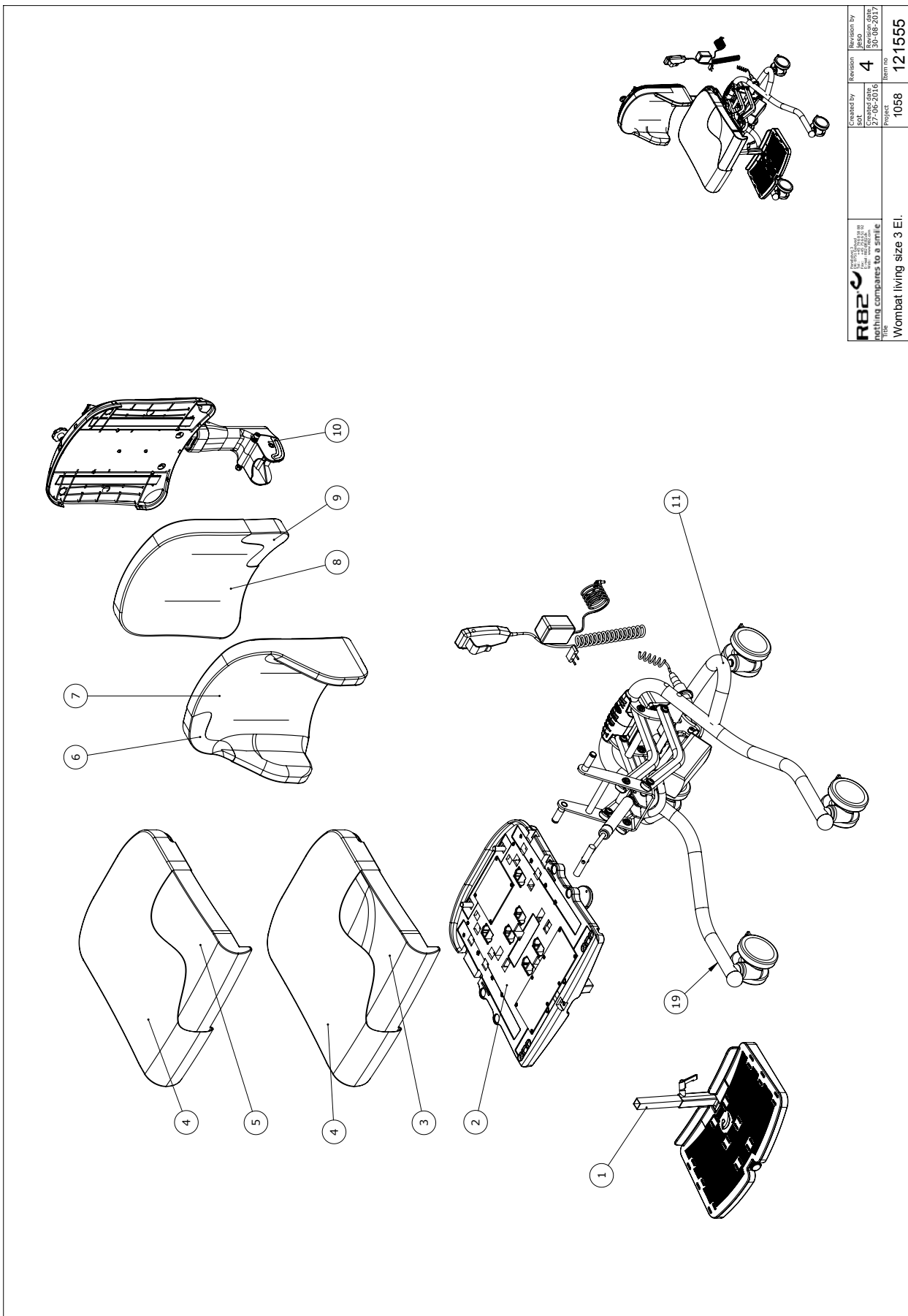

 nothing compares to a smile Title: Wombat living size 3 El.	Created by JYBA 27-05-2016	Revision JYBA 30-08-2017
	Project 1058	Item no. 121555


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	121099-3	Footrest Wombat size 3	PCS	1	5707292623034
2	121386	Wombat living seat str 3	PCS	1	5707292623942
4 to 9	See pricelist				
10	121391	Wombat Living back size 3	PCS	1	5707292624161
11	See Page 14 (121423)				

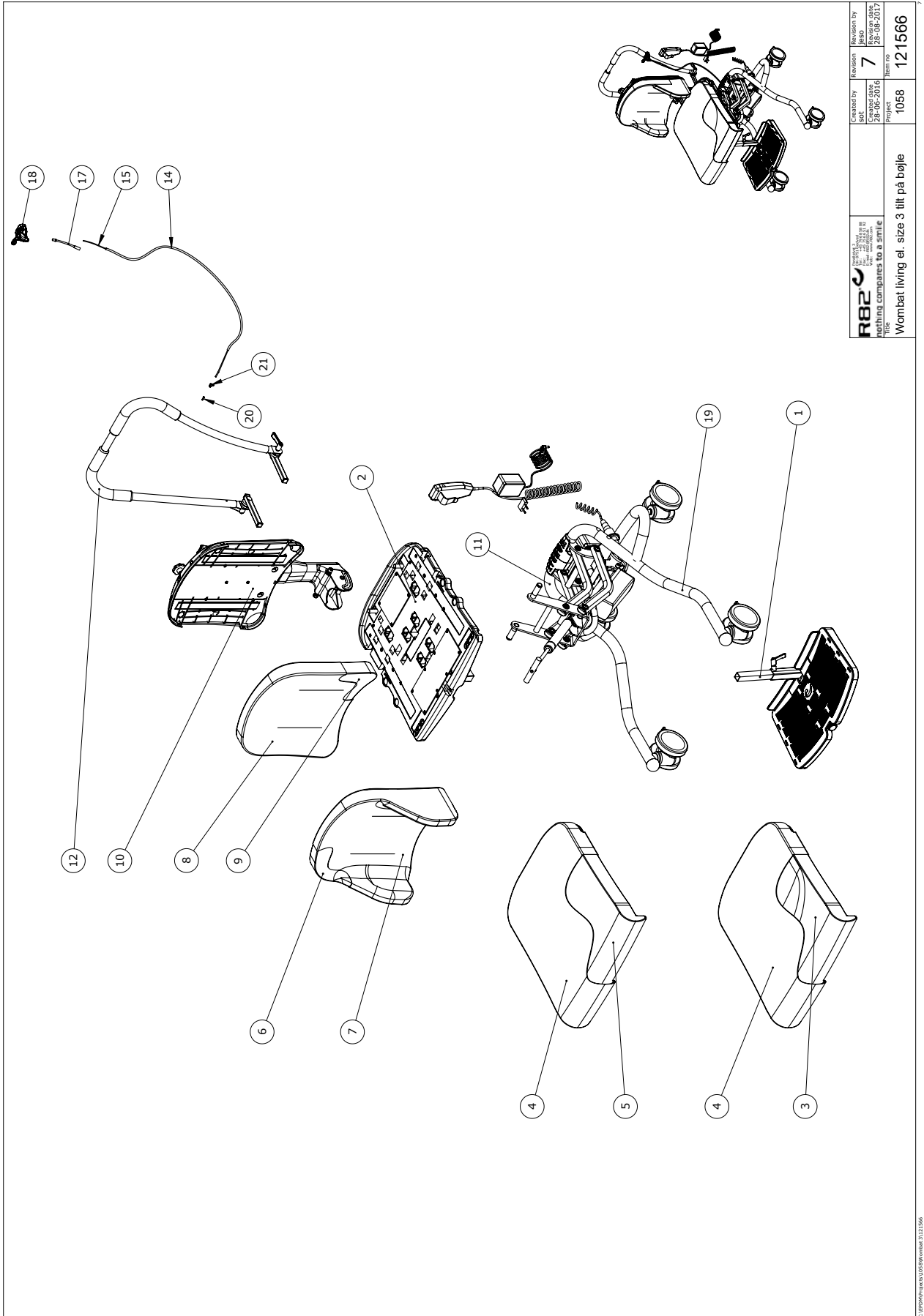


 nothing compares to a smile Title: Wombat living el. size 3 tilt på bøjle	Created by: JYBA Date: 28-06-2016 Project: 1058	Revision by: JYBA Date: 28-08-2017 Revision: 7 Item no: 121566
---	---	---


C:\EPM\Projects\1058\Wombat 3121566

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	121099-3	Footrest Wombat size 3	PCS	1	5707292623034
2	121435 TILTBØJ	Seat size 3 tilt on brace	PCS	1	5707292624581
3 to 9	See Pricelist				
10	121391	Wombat Living back size 3	PCS	1	5707292624161
11	See page 14 (121423)				
12	121218	Håndbøjle str 2 complete	PCS	1	5707292624178
14	5002995	Cable for tilt of seat	PCS	1	5707292542724
15	0371854-1	Soft inner cable for Stingray	PCS	1	5707292536945
17	0380060	Flexible v-brake pipe.	PCS	1	5707292425195
18	3503531	Gas pump lever with clamp	PCS	1	5707292413529
20	0387000	End nipple for outer casing	PCS	1	5707292136091
21	0388010	Screwnippel 7*7mm	PCS	1	5707292136121

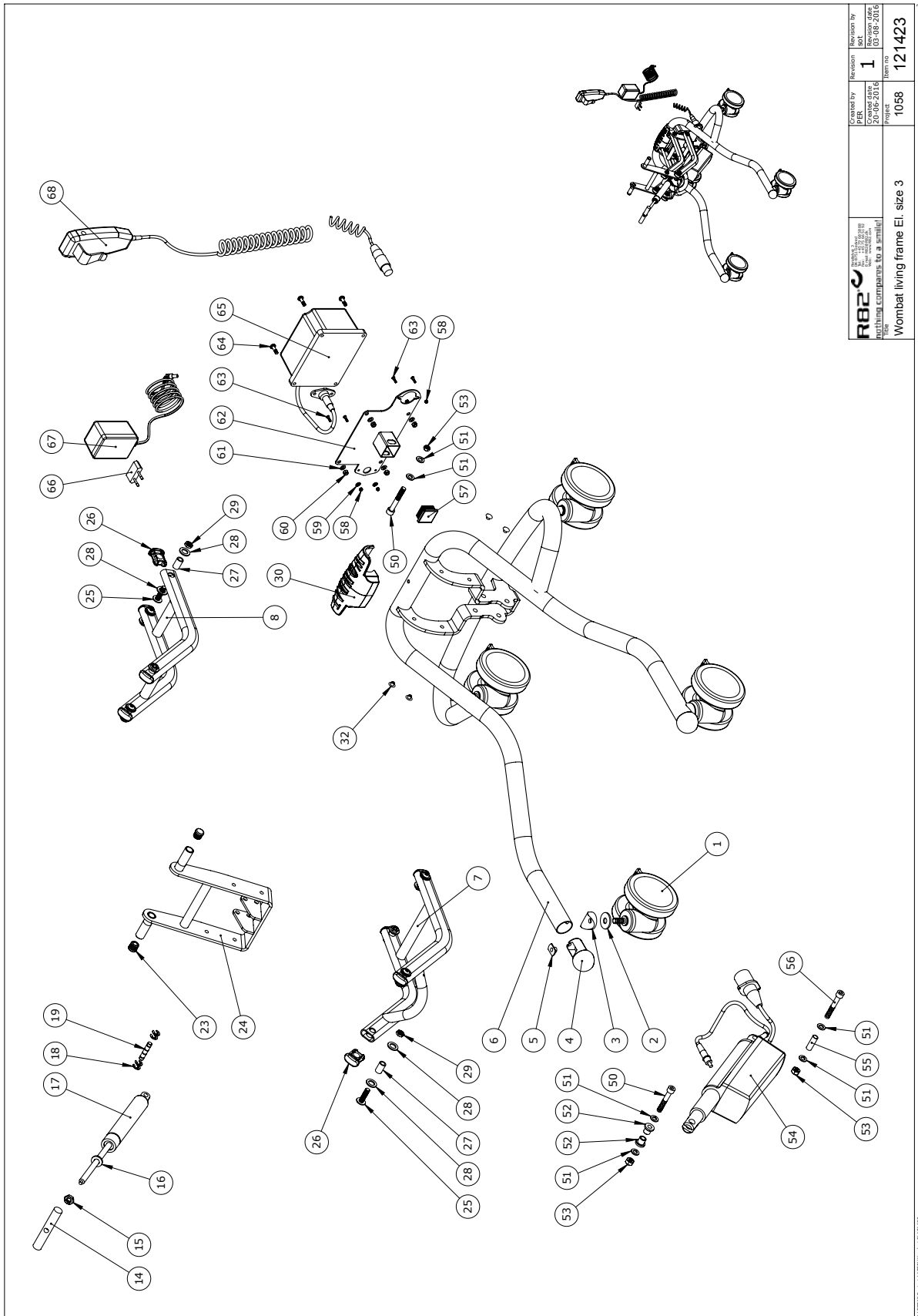

 R82 A/S Industrivej 11 5700 Svendborg Denmark Tel: +45 75 66 51 92 Fax: +45 75 66 51 92 E-mail: R82@R82.dk Web: etac.com	Covered by PER 1 5700 SVENDBORG DENMARK	Revision by SKT 02-09-2016 121423
	Wombat living frame EI, size 3 Project 1058 Item no. 121423	


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	3006938	100mm double wheel with brake	PCS	2	5707292574930
1	3006825	75mm wheel w/brake, black	PCS	4	5707292371393
2	1108050	Washer M10, din9021	PCS	2	5707292140401
3	82894	Saddle washer	PCS	2	5707292387578
4+5	5082829-60	Mount of ball and nut	PCS	2	5707292621962
6	82725-10	Frame wombat size 3	PCS	1	5707292622549
6	82725-35	frame wombat size 3	PCS	1	5707292622556
7	82790-10	Carryarm white ral 9010	PCS	1	5707292622471
7	82790-35	Carryarm Antrazit grey	PCS	1	5707292622495
8	82921-10	Carrie arm, Wombat white	PCS	1	5707292621993
8	82921-35	Carryarm	PCS	1	5707292622037
14	82729	Shaft	PCS	1	5707292400086
15	1761000	Counter nut 10*1,0 fine thread	PCS	2	5707292141538
16	1310712	Nylon washer, nature	PCS	2	5707292356345
17	2205255	Bloc-o-lift 5085ux 500n fixed	PCS	1	5707292398154
18	1145070	Lock washer 7mm	PCS	4	5707292140586
19	80924	Ø8*40 spec locking pin,wombat	PCS	1	5707292111401
23	2501600	Ferrule d15*1,0 black	PCS	2	5707292143969
24	82782-10	tipbracket White Ral9010	PCS	1	5707292622570
24	82782-35	tipbracket antrazit grey	PCS	1	5707292622594
25	0663031	Button head 8*30, Wombat	PCS	8	5707292139610
25	663031	Button head 8*30, Wombat	PCS	8	5707292139610
26	82946	End piece 15*30, Wombat	PCS	8	5707292126016
27	82933	Bearing bushing, Wombat	PCS	8	5707292125880
28	1310710	Nylon washer	PCS	16	5707292140807
29	1750800	Lock nut, stainless M8	PCS	8	5707292141484
30	82949-60	Bumper for wombat, black	PCS	1	5707292619815
32	1310540	Covering pushbutton, black	PCS	4	5707292140692
50	0603550	Insex screw 8*50	PCS	1	5707292138255
50	603550	Insex screw 8*50	PCS	1	5707292138255
51	1103200	Fazet washer 8mm	PCS	2	5707292140296
52	82848	Bush	PCS	2	5707292125484
52	1103200	Fazet washer 8mm	PCS	2	5707292140296
53	1607000	Lock nut 8mm, large pack	PCS	3	5707292141378
54	2222138	Actuator IA32	PCS	1	5707292589354
55	P-6108200	Plastrør p11 Ø10/Ø8 mm x30 mm	PCS	1	5707292585424
56	0603600	Insex screw 8*55	PCS	1	5707292138262
56	603600	Insex screw 8*55	PCS	1	5707292138262
57	2505000	Ferrule e30*301,0-2,0 int.blac	PCS	1	5707292144140
58	1606250	Self-locking nut 3mm	PCS	4	5707292408396
59	1102800	Fazet washer 4mm	PCS	2	5707292140265
60	1606300	Lock nut 5 mm	PCS	4	5707292141354
61	1102900	Fazet washer 5 mm	PCS	4	5707292140272
62	77576	Battery console highlow	PCS	1	5707292490759
63	1902590	Crosshead screw M3*10, FZB	PCS	4	5707292408389
64	1902760	Cross-headed screw 5*16 mm	PCS	4	5707292142146
65	2222110	Batteryboks ba18027	PCS	1	5707292492258
66+67	2222161	Charger with plugs	PCS	1	5707292574046
68	2220600	Handset w/r82 print & charging	PCS	1	5707292492241

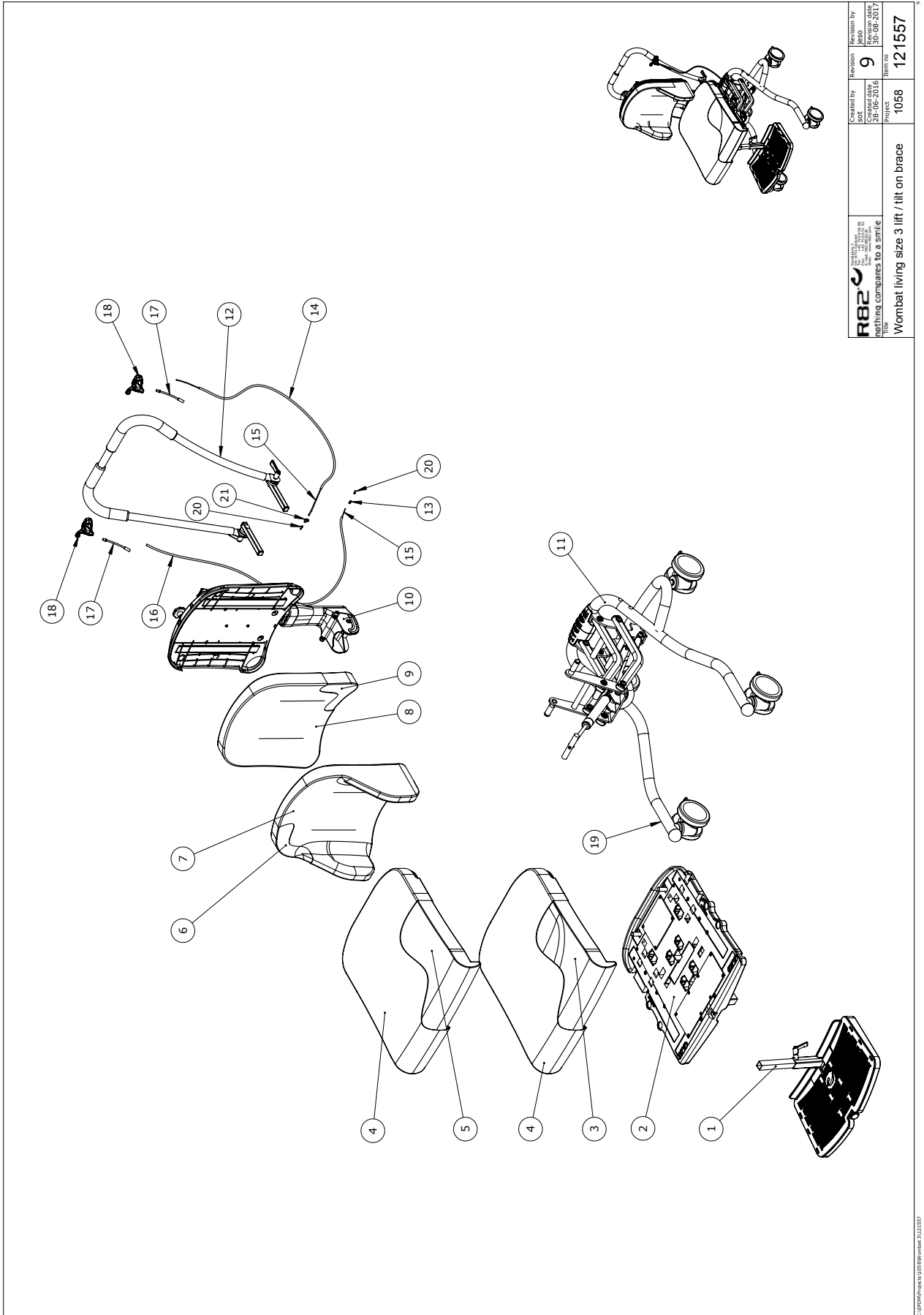

 Nothing compares to a smile	Created by Jaso	Revision by Jaso
	Date 28-05-2016	Revision 9
Title Wombat living size 3 lift / tilt on brace	Project 1058	Item no. 121557

C:\EPDF\projekter\1058\wombat 3\121557


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	121099-3	Footrest Wombat size 3	STK	1	5707292623034
2	121435 TILTbrace	Seat size 3 tilt on brace	STK	1	5707292624581
2	121386	Wombat living seat str 3	STK	1	5707292623942
2	121435-1 TILTBØ	Seat size 3 tilt on brace	STK	1	5707292805379
4 to 9	See Pricelist				
10	121391	Wombat Living back size 3	STK	1	5707292624161
10	121391-7	back size 3 with high back	STK	1	5707292784841
11	121526-10075	Frame Wombat lift brace hv 75 mm	STK	1	5707292624543
11	121526-10100	Frame Wombat lift brace hv 100mm	STK	1	5707292624451
11	121526-10100 CB	Frame Wombat lift brace hv 100mm	STK	1	5707292624567
11	121526-35075	Frame Wombat lift brace A.grå75mm	STK	1	5707292624550
11	121526-35100	Frame Wombat lift brace A.grå	STK	1	5707292624468
11	121526-35100 CB	Frame Wombat lift brace A.grå 100	STK	1	5707292624574
12	121218	Håndbracele str 2 complete	STK	1	5707292624178
13	0388006	Screwnipple 5*7,5mm	STK	1	5707292136114
14	5002995	Cable for tilt of seat	STK	1	5707292542724
15	0371854-1	Soft inner cable for Stingray	STK	2	5707292536945
16	5002951	Wire f/tiger,panther Ø200+10cm	STK	1	5707292154644
17	0380060	Flexible v-brake pipe.	STK	2	5707292425195
18	3503531	Gas pump lever with clamp	STK	2	5707292413529
19	0494082	Etiket Wombat Living	STK	1	5707292623294
20	0387000	End nipple for outer casing	STK	2	5707292136091
21	0388010	Screwnippel 7*7mm	STK	1	5707292136121

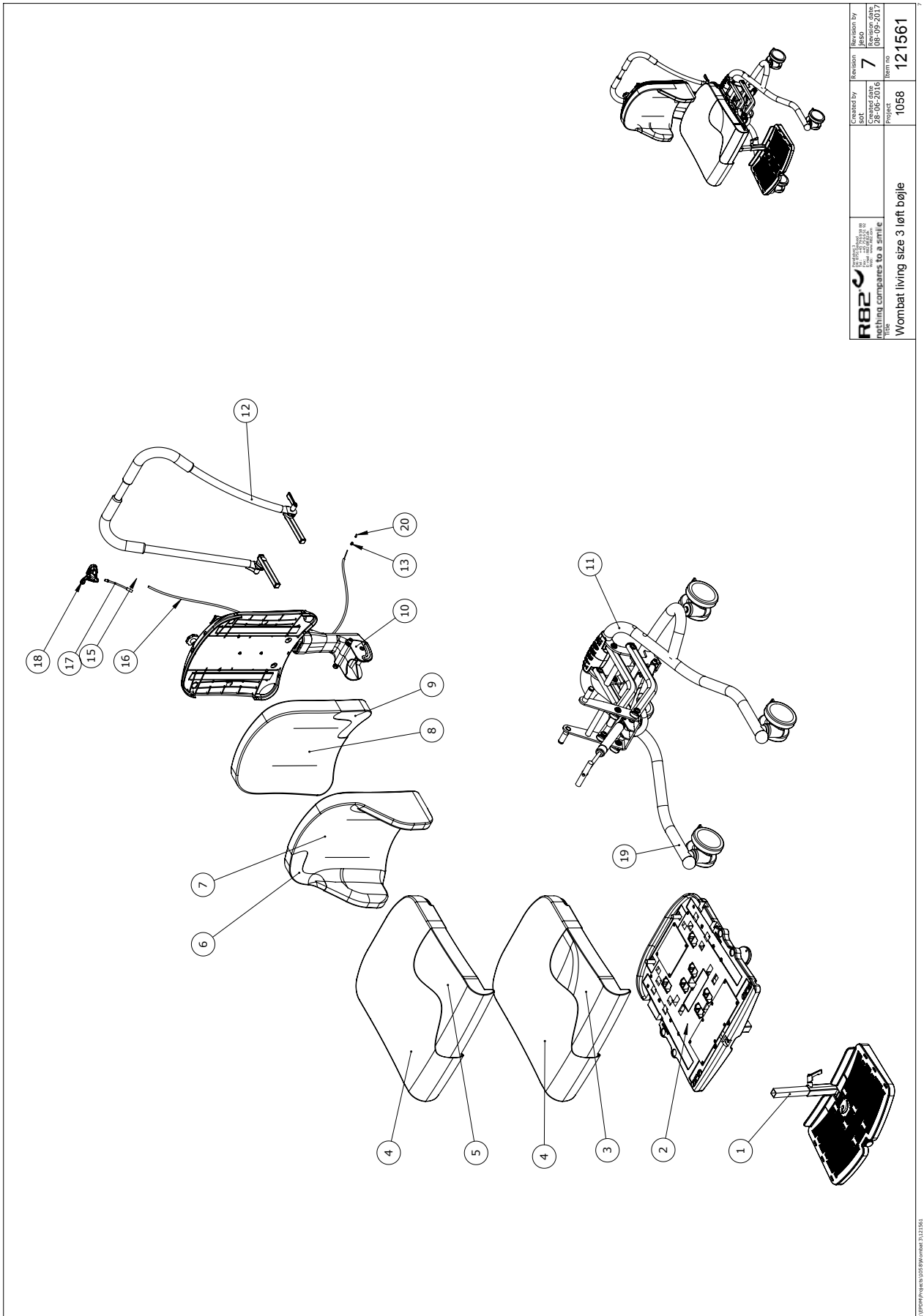

 nothing compares to a smile Title:	Created by: Jot:	Revision by: Jaso:
	28-06-2016	7
Project:	1058	Item no:
Wombat living size 3 left boiler	121561	08-09-2017


C:\EPW\Projects\1058\Wombat 3\121561


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	121099-3	Footrest Wombat size 3	STK	1	5707292623034
2	121386	Wombat living seat str 3	STK	1	5707292623942
3 to 9	See pricelist				
10	121391	Wombat Living back size 3	STK	1	5707292624161
11	121526-10100	Frame Wombat løft bøj hv 100mm	STK	1	5,70729E+12
11	121526-35075	Frame Wombat løf bøj A.grå75mm	STK	1	5,70729E+12
12	121218	Handbrace str 2 complete	STK	1	5,70729E+12
13	0388006	Screwnipple 5*7,5mm	STK	1	5707292136114
15	0371854-1	Soft inner cable for Stingray	STK	1	5707292536945
16	5002951	Wire f/tiger,panther Ø200+10cm	STK	1	5707292154644
17	0380060	Flexible v-brake pipe.	STK	1	5707292425195
18	3503531	Gas pump lever with clamp	STK	1	5707292413529
19	0494082	Etiket Wombat Living	STK	1	5707292623294
20	0387000	End nipple for outer casing	STK	1	5707292136091
13	0388006	Screwnipple 5*7,5mm	STK	1	5707292136114
14	5002995	Cable for tilt of seat	STK	1	5707292542724
15	0371854-1	Soft inner cable for Stingray	STK	2	5707292536945
16	5002951	Wire f/tiger,panther Ø200+10cm	STK	1	5707292154644
17	0380060	Flexible v-brake pipe.	STK	2	5707292425195
18	3503531	Gas pump lever with clamp	STK	2	5707292413529
20	0387000	End nipple for outer casing	STK	2	5707292136091
21	0388010	Screwnippel 7*7mm	STK	1	5707292136121

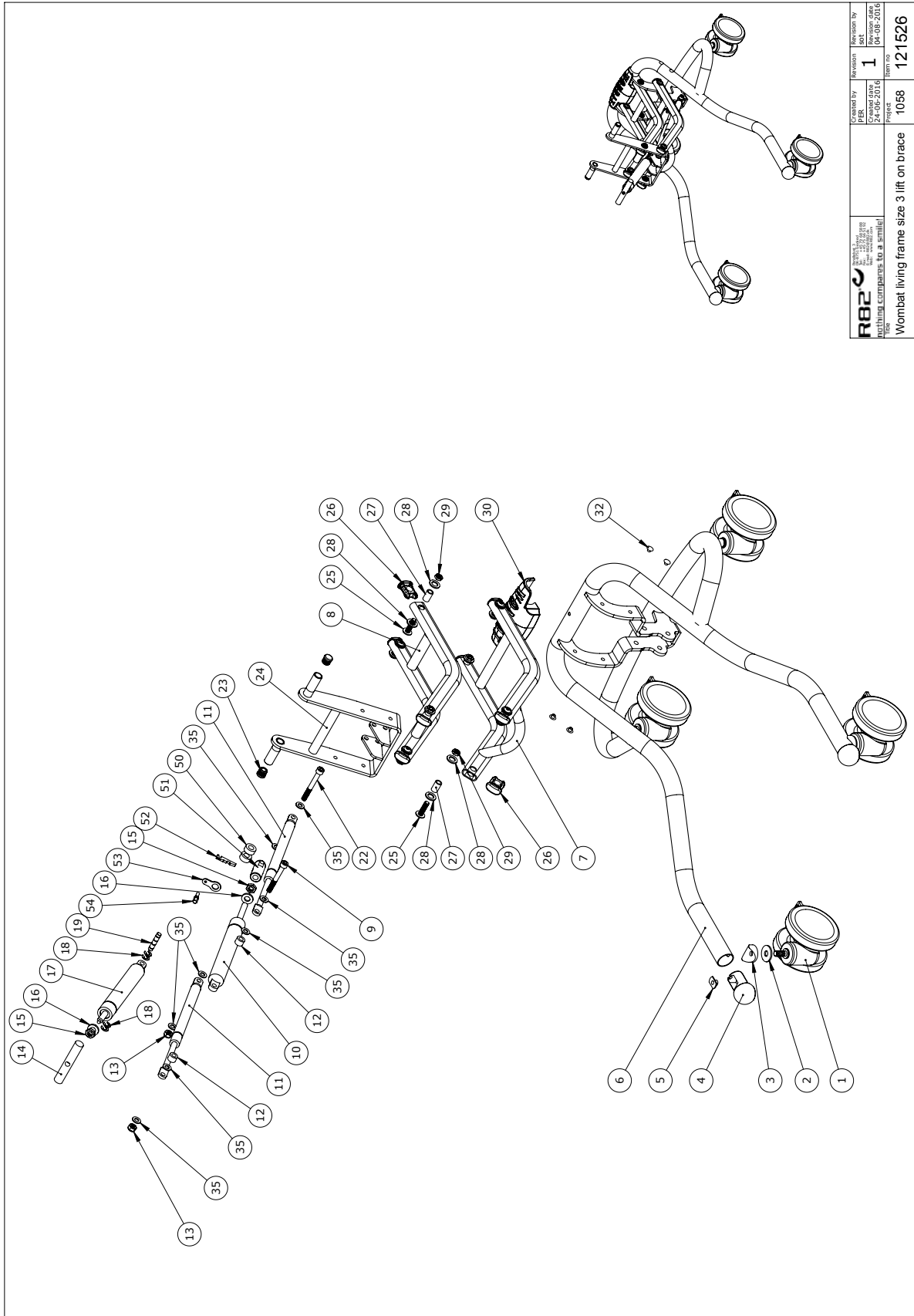

 R82 A/S Industrivej 3 8751 Gedved Denmark Net: www.r82.dk E-mail: R82@R82.dk Tel: +45 79 68 58 88	Covered by PER 54 400 010 6	Revision by SKT 1	Revision no. 121526
	Project 1058	Item no. 121526	Wombat living frame size 3 lift on brace


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	3006938	100mm double wheel with brake	PCS	2	5707292574930
1	3006825	75mm wheel w/brake, black	PCS	4	5707292371393
2	1108050	Washer M10, din9021	PCS	2	5707292140401
3	82894	Saddle washer	PCS	2	5707292387578
4+5	5082829-60	Mount of ball and nut	PCS	2	5707292621962
6	82725-35	frame wombat size 3	PCS	1	5707292622556
6	82725-10	Frame wombat size 3	PCS	1	5707292622549
7	82790-35	Carryarm Antrazit grey	PCS	1	5707292622495
7	82790-10	Carryarm white ral 9010	PCS	1	5707292622471
8	82921-35	Carryarm	PCS	1	5707292622037
8	82921-10	Carrie arm, Wombat white	PCS	1	5707292621993
9	0603665	Insex screw 8*65	PCS	1	5707292138286
10	2205300	Bloc-o-lift gasspring 5251hs	PCS	1	5707292143389
11	2202100	Lift-o-mat 5078uo 250nl	PCS	2	5707292400468
12	1350021	Distance piece 11*15*8,2mm	PCS	2	5707292140890
13	1607000	Lock nut 8mm, large pack	PCS	3	5707292141378
14	82729	Shaft	PCS	1	5707292400086
15	1761000	Counter nut 10*1,0 fine thread	PCS	2	5707292141538
16	1310712	Nylon washer, nature	PCS	2	5707292356345
17	2205255	Bloc-o-lift 5085ux 500n fixed	PCS	1	5707292398154
18	1145070	Lock washer 7mm	PCS	4	5707292140586
19	80924	Ø8*40 spec locking pin,wombat	PCS	1	5707292111401
22	0603700	Insex screw 8*70	PCS	1	5707292138293
23	2501600	Ferrule d15*1,0 black	PCS	2	5707292143969
24	82782-35	tipbracket antrazit grey	PCS	1	5707292622594
24	82782-10	tipbracket White Ral9010	PCS	1	5707292622570
25	0663031	Button head 8*30, Wombat	PCS	8	5707292139610
26	82946	End piece 15*30, Wombat	PCS	8	5707292126016
27	82933	Bearing bushing, Wombat	PCS	8	5707292125880
28	1310710	Nylon washer	PCS	16	5707292140807
29	1750800	Lock nut, stainless M8	PCS	8	5707292141484
30	82949-60	Bumper for wombat, black	PCS	1	5707292619815
50	1350000	Distancepart black 6*15*8,2 mm	PCS	2	5707292140876
51	121699	Release head	PCS	1	5707292629746
52	2200050	Small release arm	PCS	1	5707292143075
53	82268	Wire mounting by pedal	PCS	2	5707292123039
54	0380000	Cable tighter M5*23	PCS	1	5707292135933

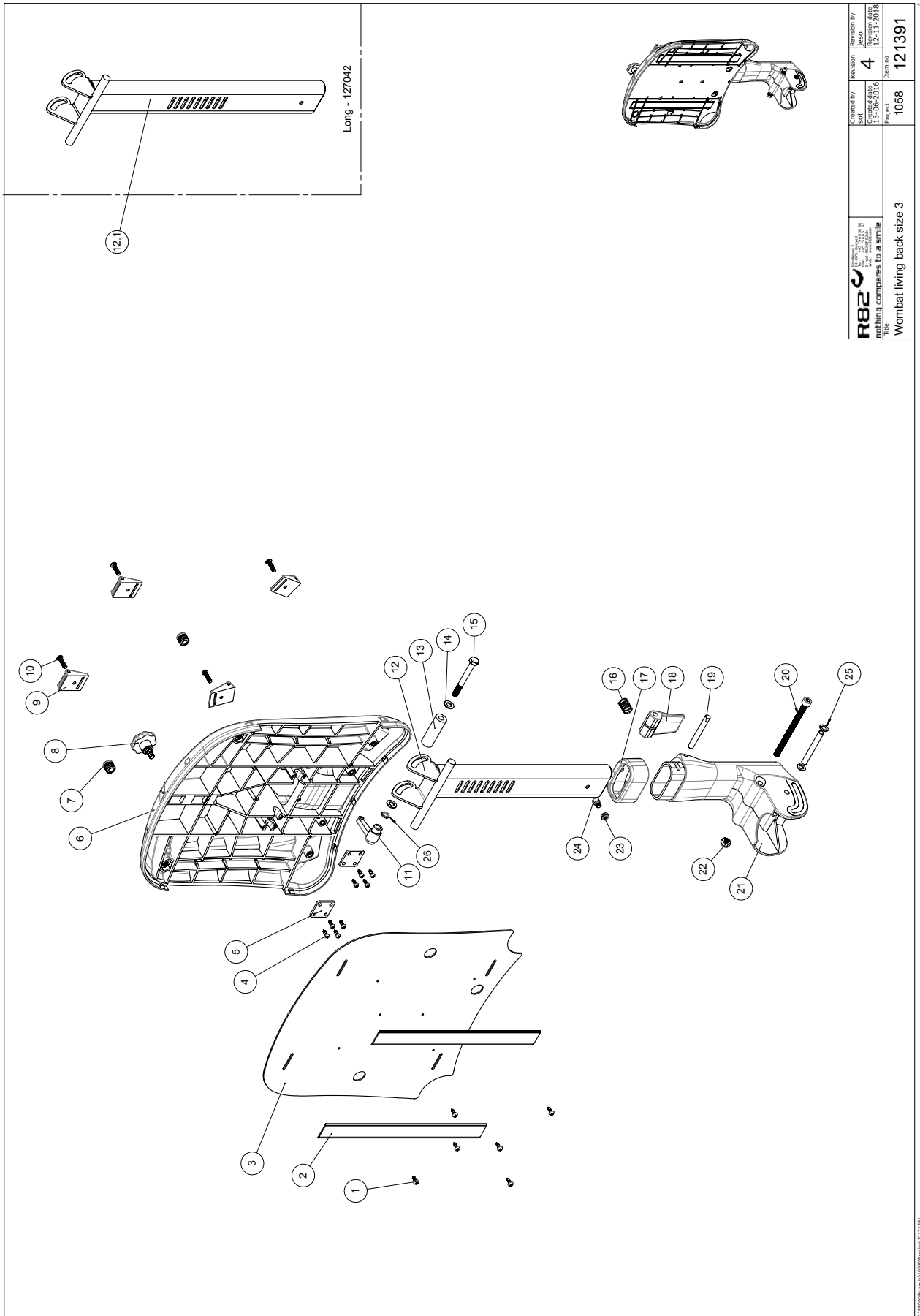

 Nothing compares to a smile Title	Created by Loh	Revision by Loh	Revision 4
	Created date 13-06-2016	Revision date 12-11-2018	Item no 121391
Product Wombat living back size 3		Item no 1068	

C:\R82\revs\105\Rev\modul 3\121391


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	1940010	Plastite 60, 2,9*6,5	STK	6	5707292142573
2	4913040	30mm adhesive hook, black	MTR	0,62	5707292786517
3	82794	Backplate f. Wombat size 3	STK	1	5707292400116
4	1940032	Eco-sight z M3,5x12 rustproof	STK	8	5707292397584
5	82781	Mounting plate	STK	2	5707292398840
6	82751	Back mounted with screws	STK	1	5707292400123
7	2501500	Ferrule d12*0,5-1,5 int.black	STK	1	5707292143952
8	3503001	R82 handle M8x15mm screw	STK	1	5707292413925
9	0445025	25mm press buckle, black	STK	4	5707292465160
10	0640920	Button head M5*20 FZB	STK	4	5707292381033
11	3502300	Tightening handle 65mm M8 bush	STK	1	5707292146885
12	82765	Backpile	STK	1	5707292398918
12.1	127042	Profile long Size 3	STK	1	5707292868848
13	82747	Shaft	STK	1	5707292400130
14	1103200	FAZET WASHER 8MM	STK	2	5707292140296
15	1907100	Steel bolt 8*70mm	STK	1	5707292142467
16	0310163	Compression spring 1,3*12rustpr	STK	1	5707292402790
17	82796-60	Lockcover, black	STK	1	5707292619846
18	82795-60	LOCKGRIP, BLACK	STK	1	5707292619839
19	0530005	Cyl.pin Ø8*66 mm	STK	2	5707292397805
20	0603975	Insex screw 8*110	STK	1	5707292138330
21	82797	Monteringsbeslag sæde/ryg	STK	1	5707292400048
22	1607000	Lock nut 8mm, large pack	STK	1	5707292141378
23	1606500	Lock nut 6 mm	STK	1	5707292141361
24	0602188	Insex M6*8, FZB	STK	1	5707292138071
25	0683925	Washer 8x14x1	STK	2	5707292467980
26	1100495	Springwasher M8	STK	1	5707292458674

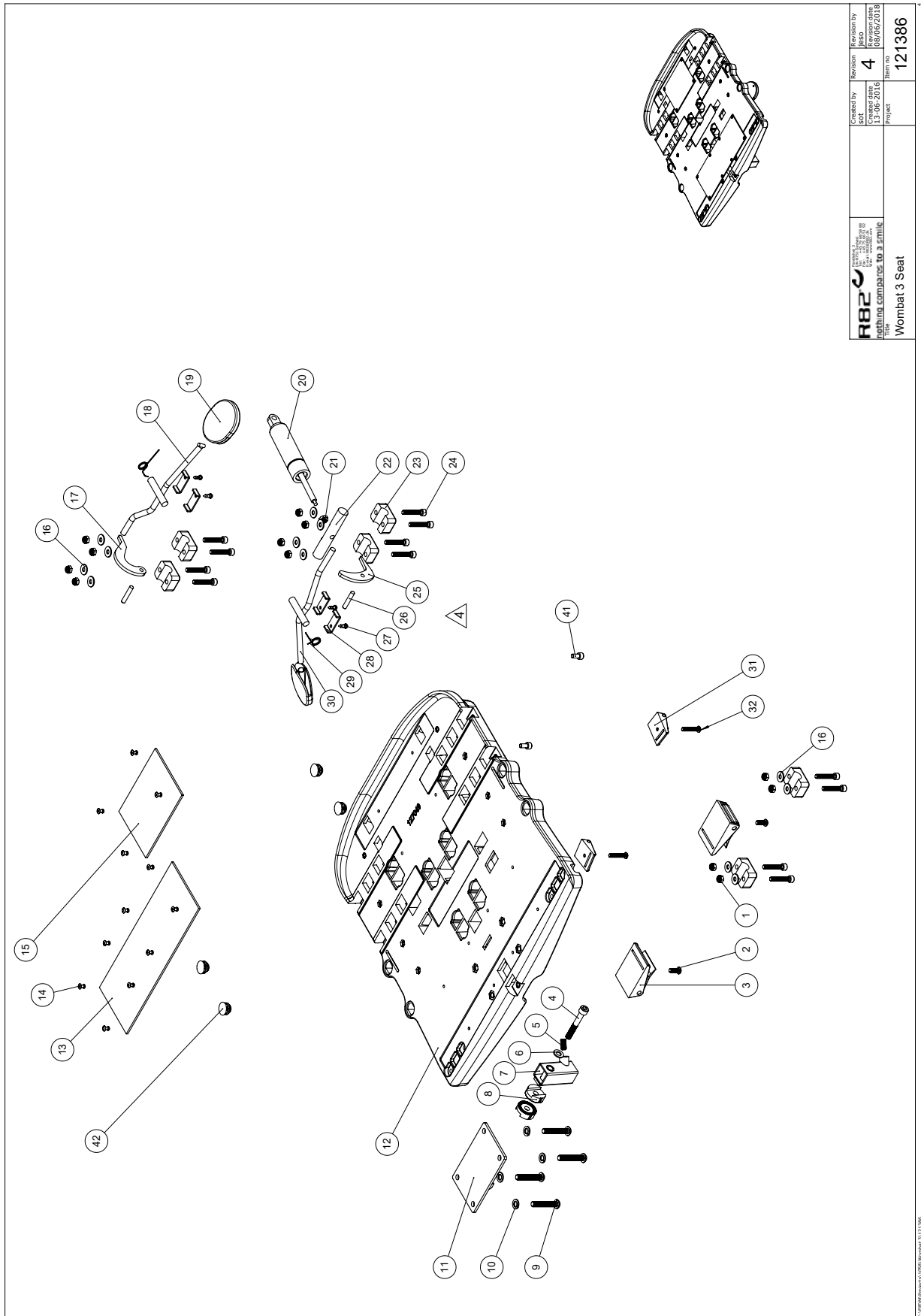


 R82	Created by 13-06-2016	Revision 4	Revision by 08/06/2018
	Project Wombat 3 Seat	Item no 121386	

C:\ERP\Projects\005\Wombat 3.13.186


Position	No.	Description	Quantity	Unit of Measure	Ean code
1	1606500	Lock nut 6 mm	STK	12	5707292141361
2	0641600	Button head screw 6*16	STK	2	5707292139269
3	0442070	Fixlock clamp 40 mm w/hole	STK	2	5707292136398
4	0603660	Insex screw 8*60	STK	1	5707292138279
5	0701000	Stop screw with crater 8*10	STK	1	5707292139870
6	1103200	Fazet washer 8mm	STK	1	5707292140296
7	82735	Fix for footrest	STK	1	5707292399335
8	74061	Tooth washer Ø30	STK	2	5707292373816
9	0661145	Button head 8*45 FZB	STK	4	5707292139535
10	1101000	Center spring washer 8mm	STK	4	5707292140173
11	82734	Plate complete	STK	1	5707292398932
12	82739-1	Seat- Wombat Size3 Living	STK	1	5707292631220
13	4935300	100mm black velcro hook	MTR	0,27	5707292152107
14	0506509	RIVET 4*10, STAINLESS	STK	12	5707292472441
15	4935300	100mm black velcro hook	MTR	0,15	5707292152107
16	1106000	Washer 6mm	STK	12	5707292140371
17	82728	Release disk	STK	1	5707292398864
18	82769	GRIP FOR SEATTIP	STK	1	5707292398871
19	3470336	Release button 5*12mm,round	STK	2	5707292397607
20	2203322	Bloc-o-lift 684074 200n solid	STK	1	5707292398147
21	1760800	Compensating nut 8*1,0 finep.t	STK	1	5707292141521
22	82741	Shaft	STK	1	5707292400109
23	2521840	Pipe holder f/16mmsquare,black	STK	6	5707292399236
24	0602435	Insex screw 6*35	STK	12	5707292138132
25	82746	RELEASE DISK BACKTIP	STK	1	5707292398895
26	0530005	Cyl.pin Ø8*66 mm	STK	2	5707292397805
27	1940025	Pan head screw 4*10 PZ FZB	STK	4	5707292545374
28	112307	Lockplate	STK	4	5707292591722
29	0310519	Twist spring	STK	2	5707292400321
30	82772	Grip for backtip	STK	1	5707292398857
31	0445025	25mm press buckle, black	STK	2	5707292465160
32	0640930	Button head M5*30 FZB	STK	2	5707292139221
41	0602200	Insex screw 6*16	STK	2	5707292138095

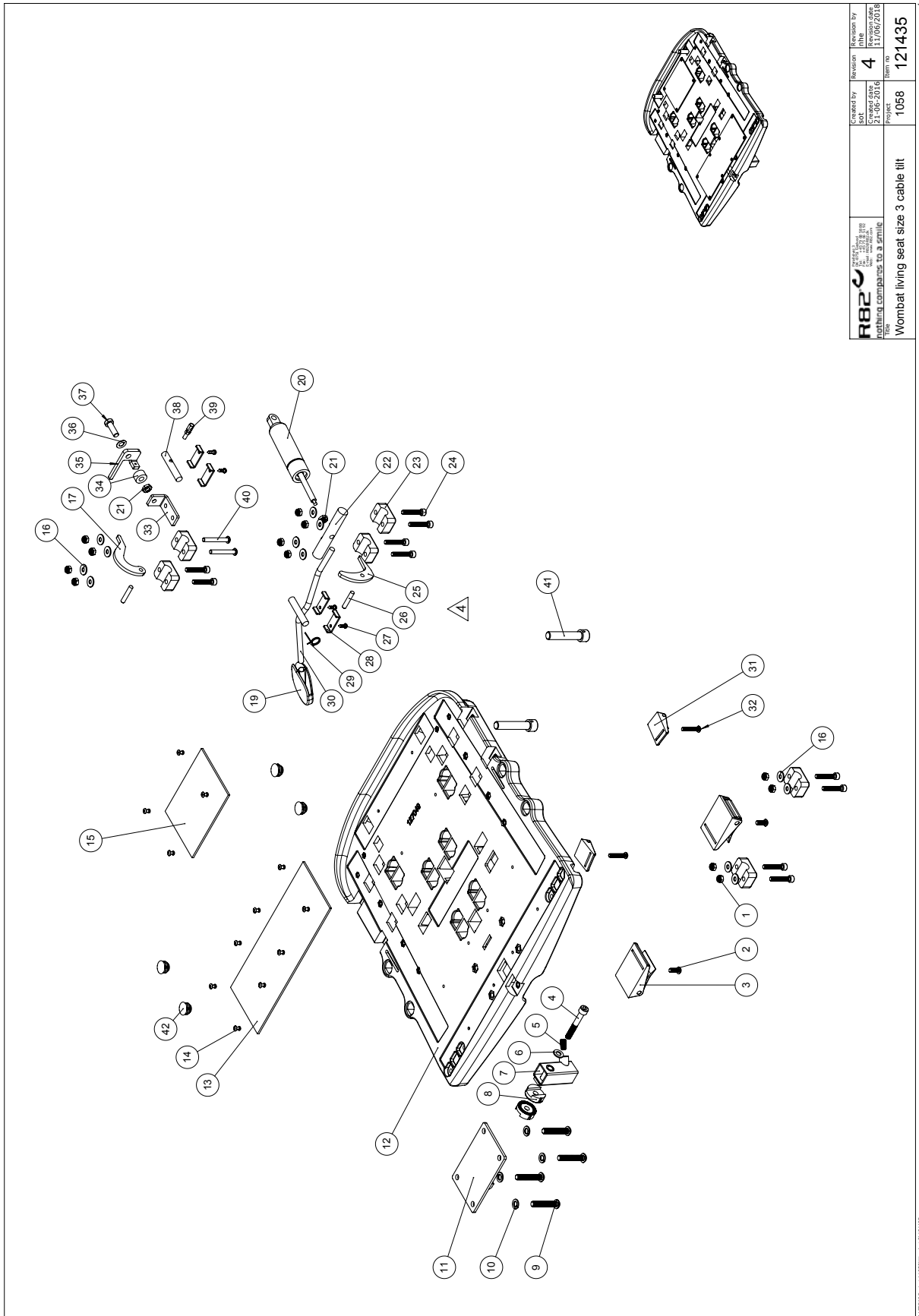

 R82 A/S Parallelvej 3 8751 Gedved Denmark	Created by SOT	Revision 4	Revisjon 4
	Drawing date 21/06/2016	Revisjon dato 21/06/2016	Part no. 1058
Wombat living seat size 3 cable lift			Part no. 121435


© 2016 R82 A/S. All rights reserved. 121435

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	1606500	Lock nut 6 mm	STK	12	5707292141361
2	0641600	Button head screw 6*16	STK	2	5707292139269
3	0442070	Fixlock clamp 40 mm w/hole	STK	2	5707292136398
4	0603660	Insex screw 8*60	STK	1	5707292138279
5	0701000	Stop screw with crater 8*10	STK	1	5707292139870
6	1103200	Fazet washer 8mm	STK	1	5707292140296
7	82735	Fix for footrest	STK	1	5707292399335
8	74061	Tooth washer Ø30	STK	2	5707292373816
9	0661145	Button head 8*45 FZB	STK	4	5707292139535
10	1101000	Center spring washer 8mm	STK	4	5707292140173
11	82734	Plate complete	STK	1	5707292398932
12	82739-2	Seat- Wombat Size3 Living	STK	1	5707292804754
12	82739-1	Seat- Wombat Size3 Living	STK	1	5707292631220
13	4935300	100mm black velcro hook	MTR	0,27	5707292152107
14	0506509	Rivet 4*10, stainless	STK	12	5707292472441
15	4935300	100mm black velcro hook	MTR	0,15	5707292152107
16	1106000	Washer 6mm	STK	12	5707292140371
17	82728	Release disk	STK	1	5707292398864
19	3470336	Release button 5*12mm,round	STK	1	5707292397607
20	2203322	Bloc-o-lift 684074 200n solid	STK	1	5707292398147
21	1750800	Lock nut, stainless M8	STK	1	5707292141484
22	82741	Shaft	STK	1	5707292400109
23	2521840	Pipe holder f/16mmsquare,black	STK	6	5707292399236
24	0602435	Insex screw 6*35	STK	12	5707292138132
25	82746	RELEASE DISK BACKTIP	STK	1	5707292398895
26	0530005	Cyl.pin Ø8*66 mm	STK	2	5707292397805
27	1940025	Pan head screw 4*10 PZ FZB	STK	4	5707292545374
28	112307	Lockplate	STK	4	5707292591722
29	0310519	Twist spring	STK	2	5707292400321
30	82772	Grip for backtip	STK	1	5707292398857
31	0445025	25mm press buckle, black	STK	2	5707292465160
32	0640930	Button head M5*30 FZB	STK	2	5707292139221
33	121427	Bracket	STK	1	5707292624116
34	1350100	Distance part white 8,2*18*10	STK	1	5707292140937
35	121430	Release plate	STK	1	5707292624109
36	1310645	Washer, nature Ø16*8*0,5mm	STK	1	5707292446015
37	0603535	Insex low head M8*25	STK	1	5707292575319
38	121426	Pin for cable	STK	1	5707292624093
39	0380000	Cable tighter M5*23	STK	1	5707292135933
40	0659600	Button head M6*45, stainless	STK	2	5707292356826
41	0602200	Insex screw 6*16	STK	2	5707292138095
42	2502510	Plug 20*2,0-2,5 inside black	STK	4	5707292557759

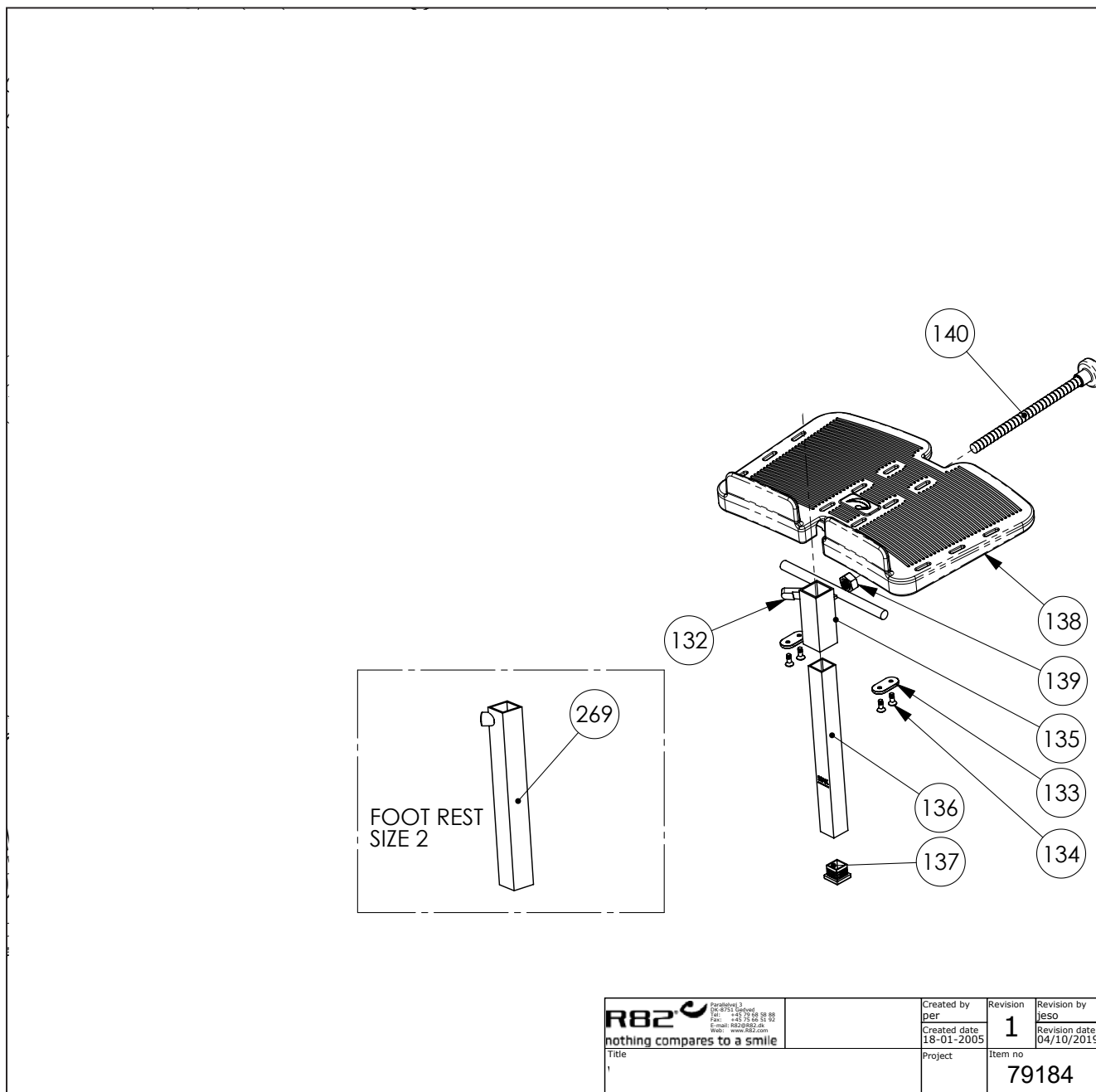

 nothing compares to a smile	Parallelvej 3 DK-8751 Gedved Tel: +45 79 68 58 88 Fax: +45 79 68 51 92 E-mail: R82@R82.dk Web: www.R82.dk	Created by per Created date 18-01-2005	Revision 1	Revision by jeso Revision date 04/10/2019
	Title 1	Project	Item no 79184	

Position	No.	Description	Quantity	Unit of Measure	Ean code
132	3502516	Tightening handle 45mm M8*16	PCS	1	5707292146915
133	82950	Hollow plate 16*32, Wombat	PCS	2	5707292126030
134	1940017	Sheet metal screw uhx4,2*13fzb	PCS	4	5707292142597
135	82925	Support fitting	PCS	1	5707292125804
136	82927	Pipe for footplate, Wombat	PCS	1	5707292125828
137	2504500	Ferrule e20*1,0-2,0 int.black	PCS	1	5707292144126
138	82940	Footplate size.2, Wombat	PCS	1	5707292125958
138	82944	Footplate size 1, Wombat	PCS	1	5707292125996
139	1604000	Steel nut 10 mm	PCS	1	5707292141293
140	3502790	Hand grip 28mm, M10*196mm	PCS	1	5707292363152
140	3502780	Handle 28mm,M10*160mm thread	PCS	1	5707292363145
269	82737	Pipe fitting long, Wombat 3	PCS	1	5707292399366

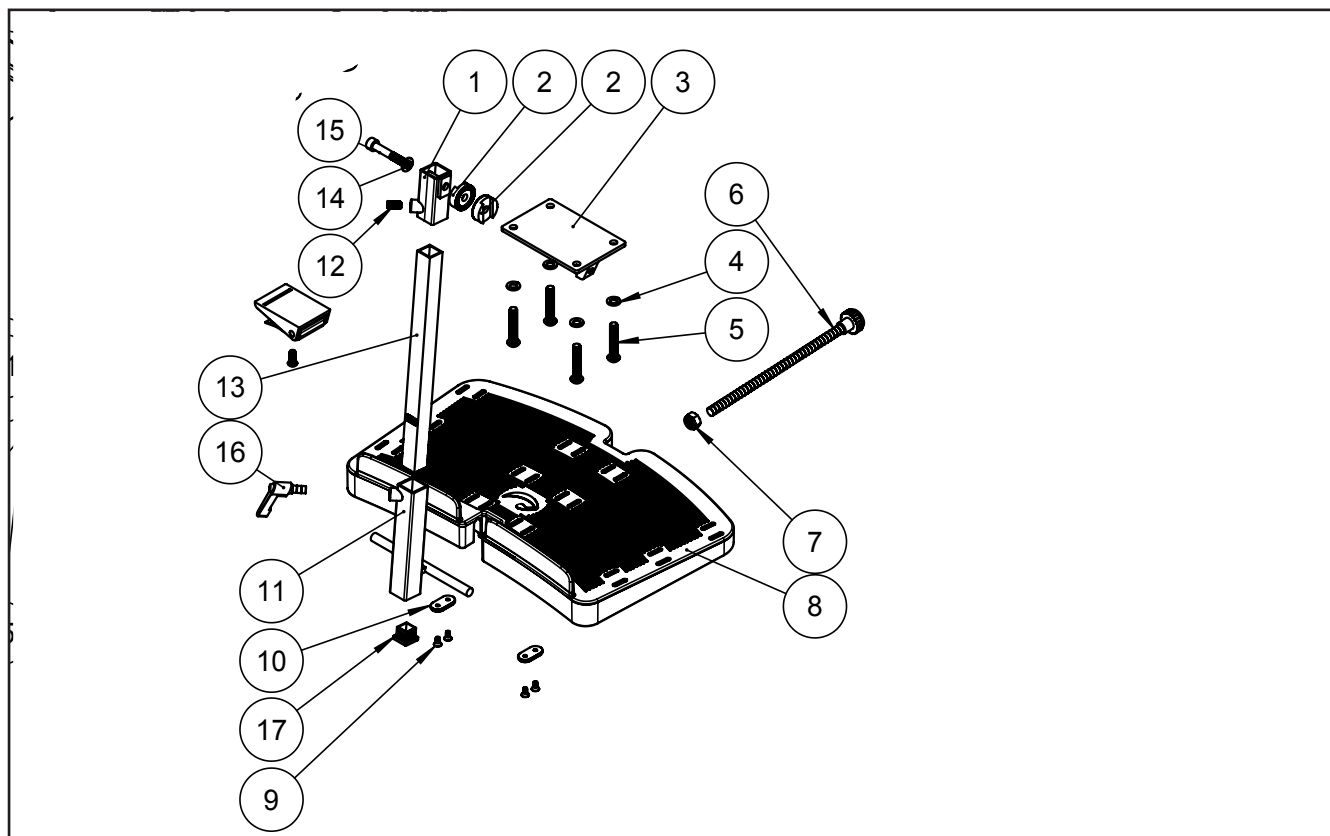

Position	No.	Description	Quantity	Unit of Measure	Ean code
1	82735	Fix for footrest	PCS	1	5707292399335
1	82733	Profile	PCS	1	5707292399014
1	82736	Profile footrest	PCS	1	5707292399007
2	82726	Plate	PCS	1	5707292398994
2	80915	Weldbushing m8 for squire tube	PCS	1	5707292359285
2	74061	Tooth washer Ø30	PCS	2	5707292373816
3	80915	Weldbushing m8 for squire tube	PCS	1	5707292359285
3	82738	Shaft, footrest	PCS	1	5707292399373
3	82734	Plate complete	PCS	1	5707292398932
4	82727	Tube	PCS	1	5707292399342
4	118565	Spacing Piece	PCS	1	5707292616593
4	1101000	Center spring washer 8mm	PCS	4	5707292140173
5	0661145	Button head 8*45 FZB	PCS	4	5707292139535
6	3502795	Knob 28mm,M10*??mm thread	PCS	1	5707292402110
7	1604000	Steel nut 10 mm	PCS	1	5707292141293
8	82752	Footplate size 3, Wombat	PCS	1	5707292401373
9	1940017	Sheet metal screw uhx4,2*13fzb	PCS	4	5707292142597
10	82950	Hollow plate 16*32, Wombat	PCS	2	5707292126030
11	82737	Pipe fitting long, Wombat 3	PCS	1	5707292399366
12	0701000	Stop screw with crater 8*10	PCS	1	5707292139870
13	82787	Rod for Wombat footrest 280 mm	PCS	1	5707292401472
14	1103200	Fazer washer 8mm	PCS	1	5707292140296
15	0603660	Insex screw 8*60	PCS	1	5707292138279
16	3502516	Tightening handle 45mm M8*16	PCS	1	5707292146915
17	2504500	Ferrule e20*1,0-2,0 int.black	PCS	1	5707292144126