

# wombat living and solo accessories

Exploded views / Part lists

Rev. date (2022.03)


*On the following pages you will find the parts used to assemble this product. Please note that some spare parts cannot be purchased individually but are part of a complete kit. Please also note that for some spare parts extra delivery time may occur.*



## Contents


|                                                                  |    |
|------------------------------------------------------------------|----|
| Pommel assembly, size 1 (79190) .....                            | 4  |
| Item numbers for Pommel assembly, size 1 (79190).....            | 4  |
| Pommel assembly, (121830) .....                                  | 5  |
| Item numbers for Pommel assembly (121830).....                   | 5  |
| Arm support, size 2&3 (126856) .....                             | 6  |
| Item numbers for Arm support, size 2&3 (126856) .....            | 7  |
| Arm support, size 2&3 (122042) .....                             | 8  |
| Item numbers for Arm support, size 2&3 (122042) .....            | 9  |
| Arm support, size 2&3 (126856) .....                             | 10 |
| Item numbers for Arm support, size 2&3 (126856) .....            | 11 |
| Wombat tray assembly (79148).....                                | 12 |
| Item numbers for Wombat tray assembly (79148) .....              | 12 |
| Tray (79185).....                                                | 13 |
| Item numbers for Tray (79185).....                               | 13 |
| Hip support (126874).....                                        | 14 |
| Item numbers for Hip support (126874) .....                      | 14 |
| Push handle complete size 1 (121058).....                        | 15 |
| Item numbers for Push handle complete size 1 (121058) .....      | 15 |
| Push handle complete size 2 (121218).....                        | 16 |
| Item numbers for Push handle complete size 2 (121218) .....      | 16 |
| Hand brace size 3 (121577).....                                  | 17 |
| Item numbers for Hand brace size 3 (121577) .....                | 17 |
| Back extencion for wombat size 3 (79196).....                    | 18 |
| Item numbers for Back extencion for wombat size 3 (79196) .....  | 18 |
| Lumba support straight, size 1-2 (128370).....                   | 19 |
| Item numbers for Lumba support straight, size 1-2 (128370) ..... | 19 |
| Lumba support straight, size 3 (128375).....                     | 20 |
| Item numbers for Lumba support straight size 3 (128375) .....    | 20 |
| Lumba support with sides, size 1 (127031) .....                  | 21 |
| Item numbers for Lumba support with sides, size 1 (127031) ..... | 21 |
| Lumba support with sides, size 2 (128279) .....                  | 22 |
| Item numbers for Lumba support with sides, size 2 (128279) ..... | 22 |
| Wombat central brake (79192) .....                               | 23 |
| Item numbers for Wombat central brake (79192) .....              | 23 |

|                                                        |    |
|--------------------------------------------------------|----|
| Anti-tip (79244).....                                  | 24 |
| Item numbers for Anti-tip (79244) .....                | 24 |
| Helper's handle (79185).....                           | 25 |
| Item numbers for Helper's handle (79185) .....         | 25 |
| Remote holder assembly (150952).....                   | 26 |
| Item numbers for Remote holder assembly (150952) ..... | 26 |
| Remote holder assembly (150860).....                   | 27 |
| Item numbers for Remote holder assembly (150860) ..... | 27 |

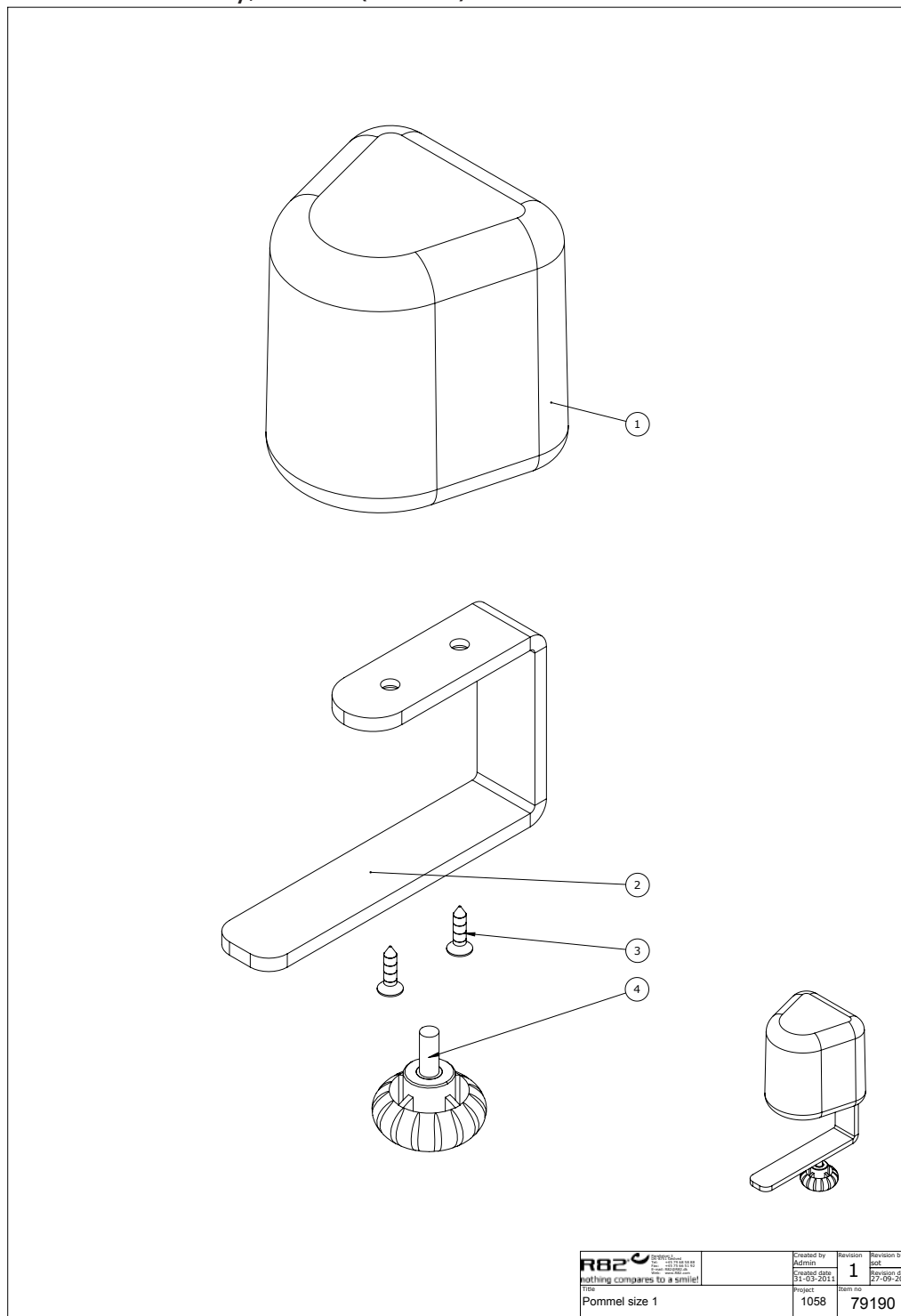**Pommel assembly, size 1 (79190)**

**Item numbers for Pommel assembly, size 1 (79190)**

| Position | No.     | Description              | Quantity | Unit of Measure | Ean code      |
|----------|---------|--------------------------|----------|-----------------|---------------|
| 1        | 6175111 | Pommel size 1, black pur | PCS      | 1               | 5707292189103 |
| 1        | 6175112 | Pommel size 2, black pur | PCS      | 1               | 5707292189110 |
| 1        | 6175114 | Pommel size 4, black pur | PCS      | 1               | 5707292189134 |
| 1        | 6175113 | Pommel sz. 3, black pur  | PCS      | 1               | 5707292189127 |
| 2        | 82978   | Pommel brace Wombat      | PCS      | 1               | 5707292126221 |
| 3        | 1811275 | Cutting screw 4*30mm     | PCS      | 2               | 5707292141743 |
| 4        | 3652501 | R82 handle M6*16mm screw | PCS      | 1               | 5707292413963 |

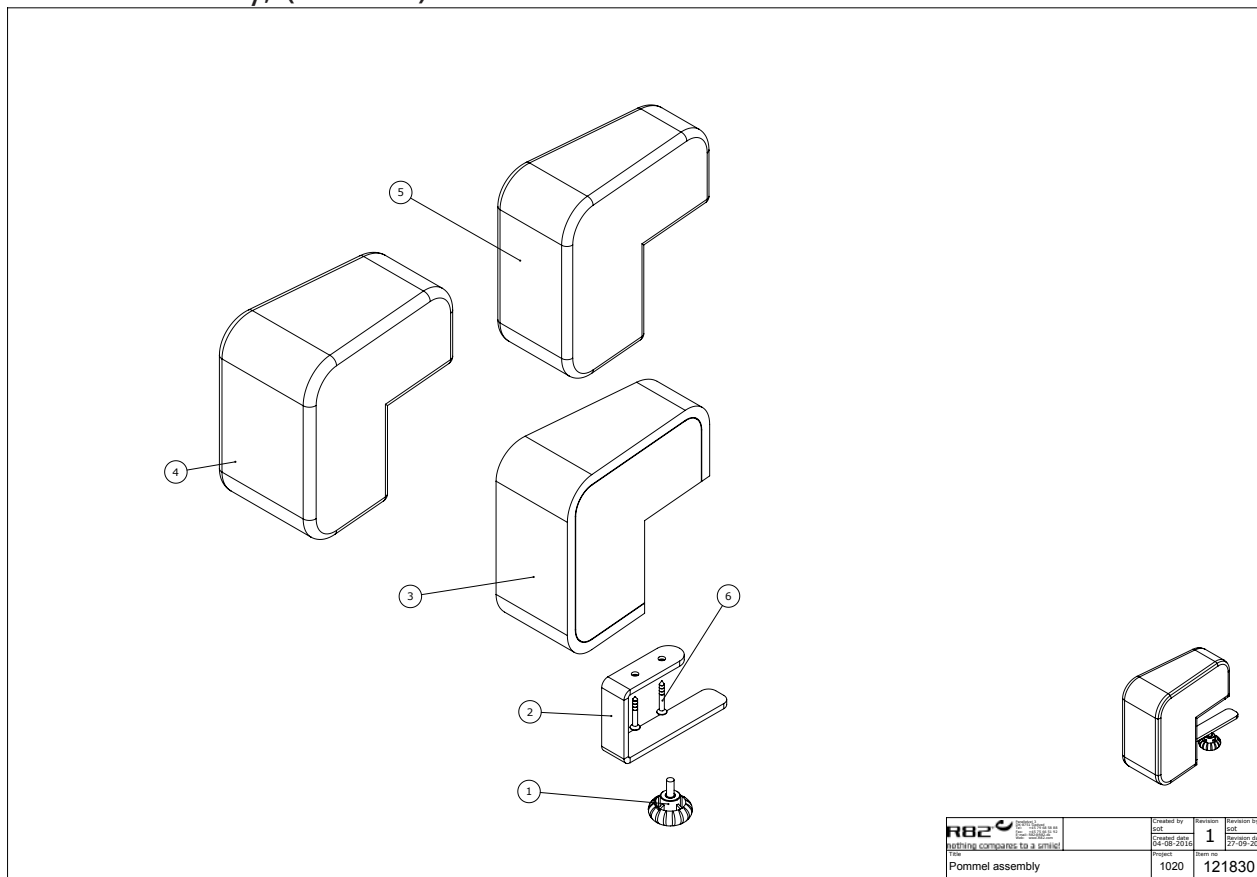
## Pommel assembly, (121830)



### Item numbers for Pommel assembly (121830)

| Position | No.         | Description                    | Quantity | Unit of Measure | Ean code      |
|----------|-------------|--------------------------------|----------|-----------------|---------------|
| 1        | 3652501     | R82 handle M6*16mm screw       | PCS      | 1               | 5707292413963 |
| 2        | 82978       | Pommel brace Wombat            | PCS      | 1               | 5707292126221 |
| 3        | 113633      | Pommel with foam               | PCS      | 1               | 5707292600882 |
| 4        | 95430-10160 | Cover f. knee abduction pommel | PCS      | 1               | 5707292600646 |
| 6        | 1811275     | Cutting screw 4*30mm           | PCS      | 2               | 5707292141743 |


## Arm support, size 2&3 (126856)

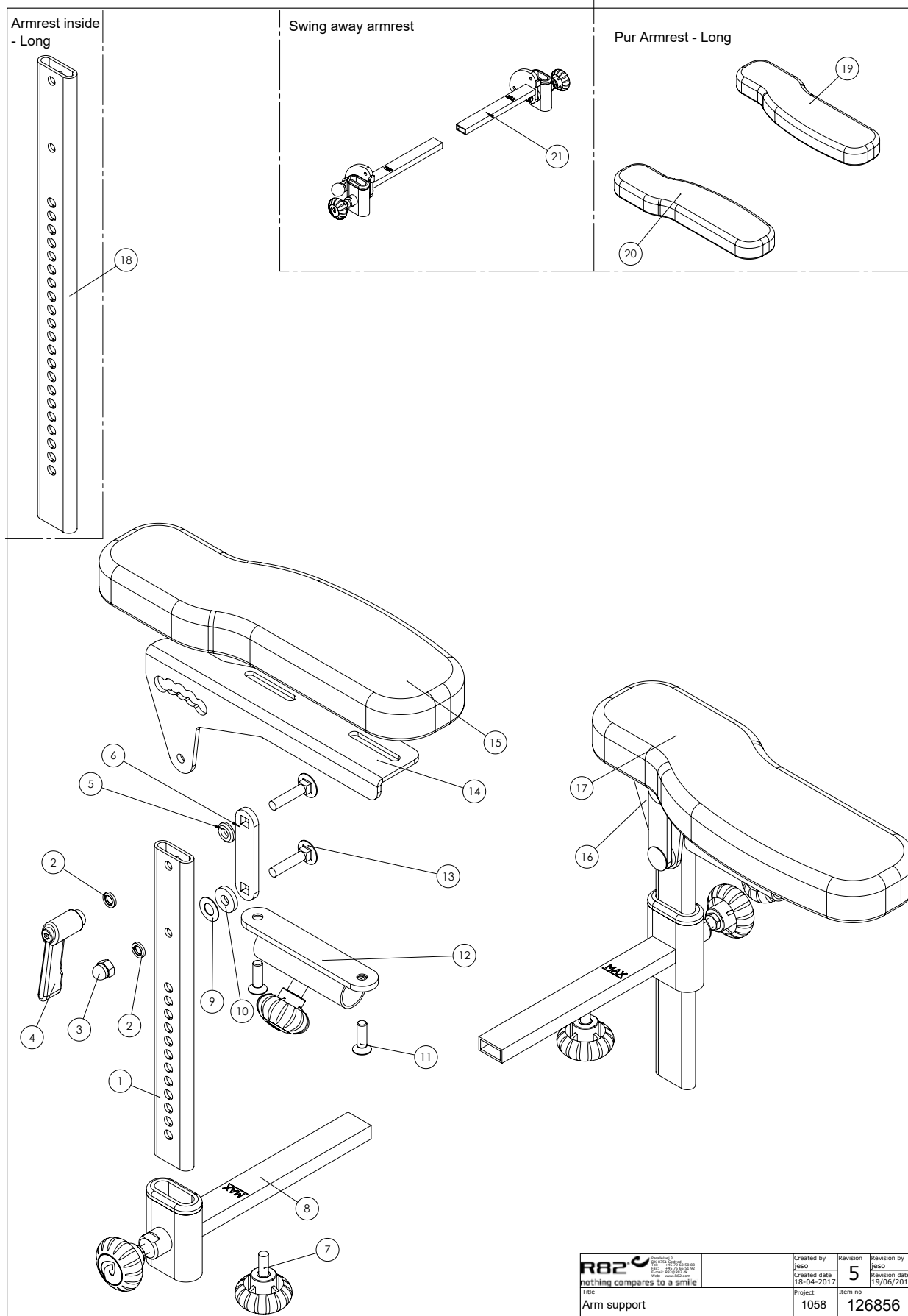

## Item numbers for Arm support, size 2&amp;3 (126856)

| Position | No.     | Description                    | Quantity | Unit of Measure | Ean code      |
|----------|---------|--------------------------------|----------|-----------------|---------------|
| 1        | 82963-1 | Innerprofil armrest Wombat Liv | PCS      | 2               | 5707292546890 |
| 2        | 1100900 | Spring washer M6 fzb, 6.1/9.9  | PCS      | 4               | 5707292459336 |
| 3        | 1780600 | Top nut 6mm                    | PCS      | 2               | 5707292141552 |
| 4        | 3502200 | Tightening handle 42mm M6 bush | PCS      | 2               | 5707292146861 |
| 5        | 128514  | Lock washer Ø6                 | PCS      | 2               | 5707292795687 |
| 6        | 128508  | Bracket for armrest            | PCS      | 2               | 5707292795670 |
| 7        | 3652501 | R82 handle M6*16mm screw       | PCS      | 2               | 5707292413963 |
| 8        | 5082960 | Hardware holder armrest        | PCS      | 2               | 5707292511157 |
| 9        | 1310645 | Washer, nature Ø16*8*0,5mm     | PCS      | 2               | 5707292446015 |
| 10       | 1125618 | Screenwasher fz6*18*3mm        | PCS      | 2               | 5707292140494 |
| 11       | 0611000 | Insex screw counters. 6*20     | PCS      | 4               | 5707292138682 |
| 12       | 5082975 | Hardware for table, mounted    | PCS      | 2               | 5707292511164 |
| 13       | 1905300 | Board bolt 6*25mm stainless    | PCS      | 4               | 5707292142252 |
| 14       | 128509  | Armrest right                  | PCS      | 1               | 5707292795663 |
| 15       | 6175154 | Pur armrest size 2&3, right    | PCS      | 1               | 5707292189240 |
| 16       | 128533  | Armrest left                   | PCS      | 1               | 5707292795656 |
| 17       | 6175156 | Pur armrest size 2&3, left     | PCS      | 1               | 5707292189257 |
| 18       | 82666-1 | Armrest, wombat, extended      | PCS      | 2               | 5707292546906 |

## Arm support, size 2&3 (122042)

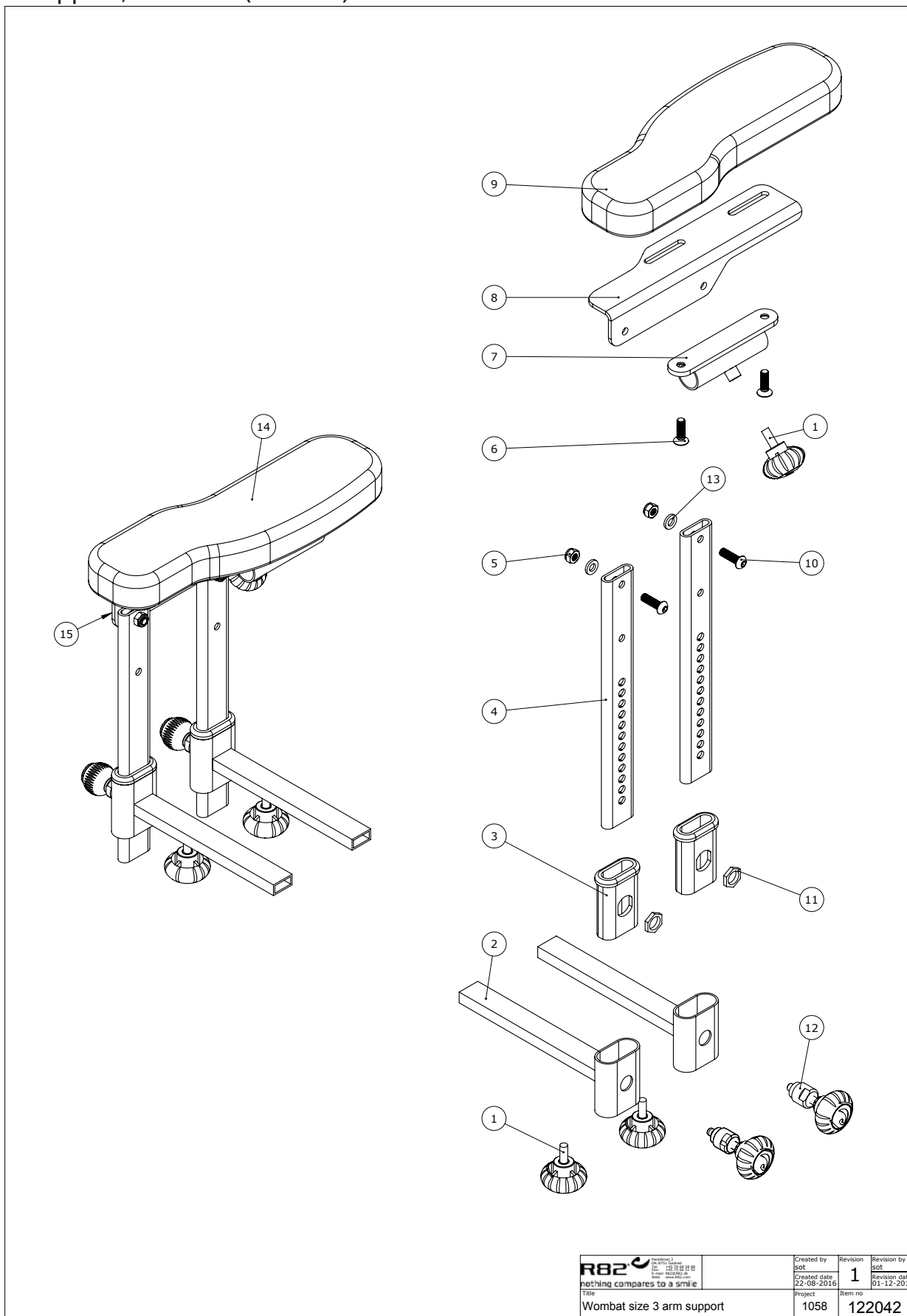


|                                                          |                                                                                 |                                  |                                                                                   |
|----------------------------------------------------------|---------------------------------------------------------------------------------|----------------------------------|-----------------------------------------------------------------------------------|
| <b>R82</b><br><small>nothing compares to a smile</small> | <small>Created by<br/>sot</small><br><small>Created date<br/>22-08-2016</small> | <small>Revision<br/>1</small>    | <small>Revision by<br/>sot</small><br><small>Revision date<br/>01-12-2017</small> |
|                                                          |                                                                                 |                                  |                                                                                   |
| <small>Title</small><br>Wombat size 3 arm support        | <small>Project</small><br>1058                                                  | <small>Item no</small><br>122042 |                                                                                   |



## Item numbers for Arm support, size 2&amp;3 (122042)

| Position  | No.     | Description                    | Quantity | Unit of Measure | Ean code      |
|-----------|---------|--------------------------------|----------|-----------------|---------------|
| 1         | 3652501 | R82 handle M6*16mm screw       | PCS      | 4               | 5707292413963 |
| 2-3-11-12 | 5082960 | Hardware holder armrest        | PCS      | 4               | 5707292511157 |
| 4         | 82666-1 | Armrest, wombat, extended      | PCS      | 4               | 5707292546906 |
| 4         | 82963-1 | Armlæn inv.wombat              | PCS      | 4               | 5707292546890 |
| 5         | 1606500 | Lock nut 6 mm                  | PCS      | 4               | 5707292141361 |
| 6         | 0611000 | Insex screw counters. 6*20     | PCS      | 4               | 5707292138682 |
| 1+7       | 5082975 | Hardware for table, mounted    | PCS      | 2               | 5707292511164 |
| 8         | 82718   | Armrest bracket left size 3    | PCS      | 1               | 5707292442727 |
| 8         | 82693   | Bracket left                   | PCS      | 1               | 5707292401762 |
| 9         | 6175163 | Armrest pu. Wombat sz 4. Right | PCS      | 1               | 5707292443151 |
| 9         | 6175156 | Pur armrest size 2&3, left     | PCS      | 1               | 5707292189257 |
| 10        | 0641620 | Button head screw 6*20         | PCS      | 4               | 5707292139276 |
| 13        | 1310640 | Washer, nature Ø16*10*0,5mm    | PCS      | 4               | 5707292140760 |
| 14        | 6175164 | Armrest pu. Wombat sz 4. Left  | PCS      | 1               | 5707292443168 |
| 14        | 6175154 | Pur armrest size 2&3, right    | PCS      | 1               | 5707292189240 |
| 15        | 82705   | Armrest right size 3           | PCS      | 1               | 5707292442734 |
| 15        | 82692   | Bracket right                  | PCS      | 1               | 5707292401755 |


## Arm support, size 2&3 (126856)

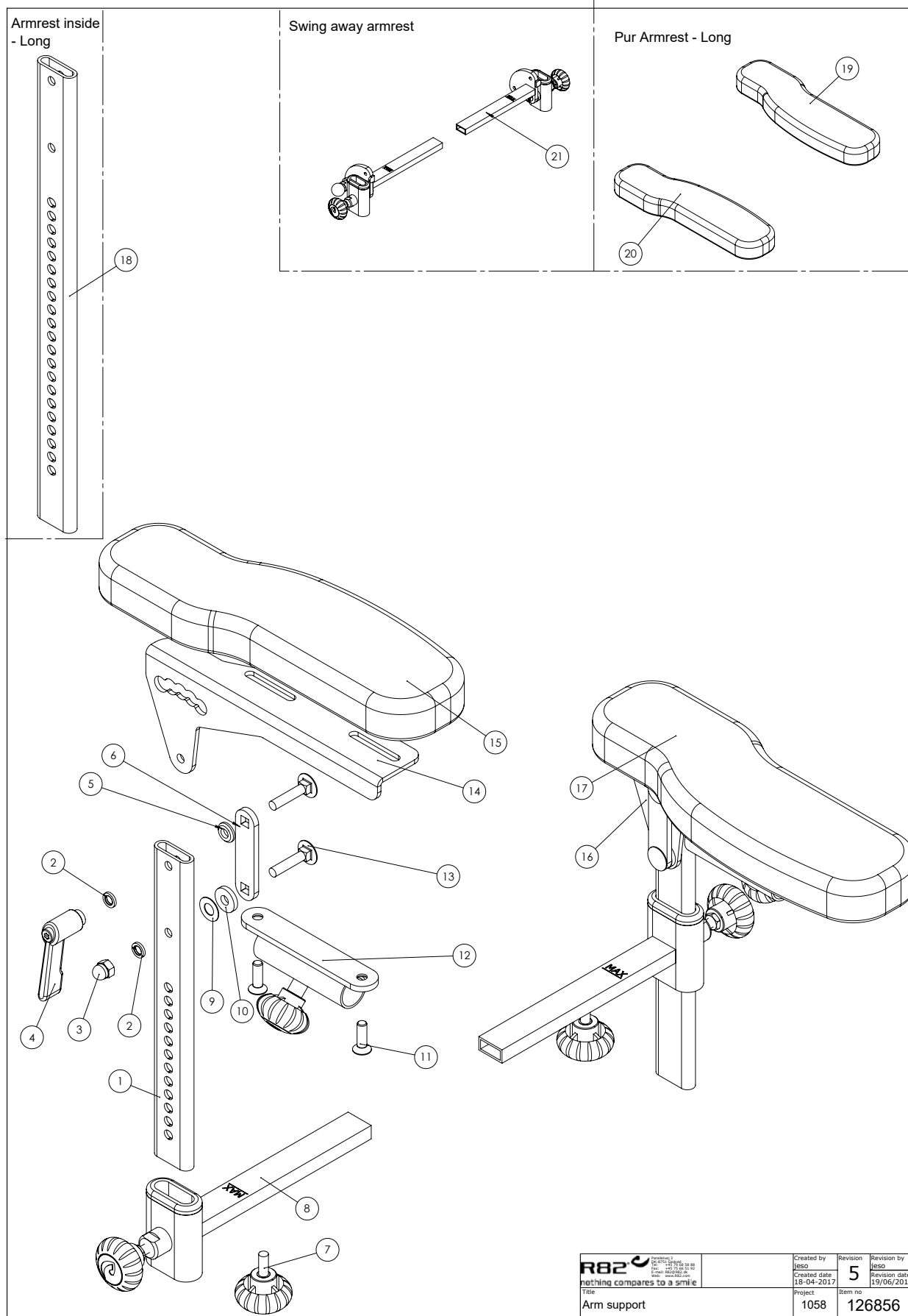

## Item numbers for Arm support, size 2&amp;3 (126856)

| Position | No.     | Description                    | Quantity | Unit of Measure | Ean code      |
|----------|---------|--------------------------------|----------|-----------------|---------------|
| 1        | 82963-1 | Innerprofil armrest Wombat Liv | PCS      | 2               | 5707292546890 |
| 2        | 1100900 | Spring washer M6 fzb, 6.1/9.9  | PCS      | 4               | 5707292459336 |
| 3        | 1780600 | Top nut 6mm                    | PCS      | 2               | 5707292141552 |
| 4        | 3502200 | Tightening handle 42mm M6 bush | PCS      | 2               | 5707292146861 |
| 5        | 128514  | Lock washer Ø6                 | PCS      | 2               | 5707292795687 |
| 6        | 128508  | Bracket for armrest            | PCS      | 2               | 5707292795670 |
| 7        | 3652501 | R82 handle M6*16mm screw       | PCS      | 2               | 5707292413963 |
| 9        | 1310645 | Washer, nature Ø16*8*0,5mm     | PCS      | 2               | 5707292446015 |
| 10       | 1125618 | Screenwasher fz6*18*3mm        | PCS      | 2               | 5707292140494 |
| 11       | 0611000 | Insex screw counters. 6*20     | PCS      | 4               | 5707292138682 |
| 12       | 5082975 | Hardware for table, mounted    | PCS      | 2               | 5707292511164 |
| 13       | 1905300 | Board bolt 6*25mm stainless    | PCS      | 4               | 5707292142252 |
| 14       | 128509  | Armrest right                  | PCS      | 1               | 5707292795663 |
| 15       | 6175154 | Pur armrest size 2&3, right    | PCS      | 1               | 5707292189240 |
| 16       | 128533  | Armrest left                   | PCS      | 1               | 5707292795656 |
| 17       | 6175156 | Pur armrest size 2&3, left     | PCS      | 1               | 5707292189257 |
| 18       | 82666-1 | Armrest, wombat, extended      | PCS      | 2               | 5707292546906 |
| 19       | 6175160 | Armrest pad,s.4,left,black     | PCS      | 1               | 5707292189271 |
| 20       | 6175158 | Armrest pad s.4,right,black    | PCS      | 1               | 5707292189264 |
| 21       | 140578  | Swing away armrest             | SET      | 1               | 5707292875334 |


## Wombat tray assembly (79148)

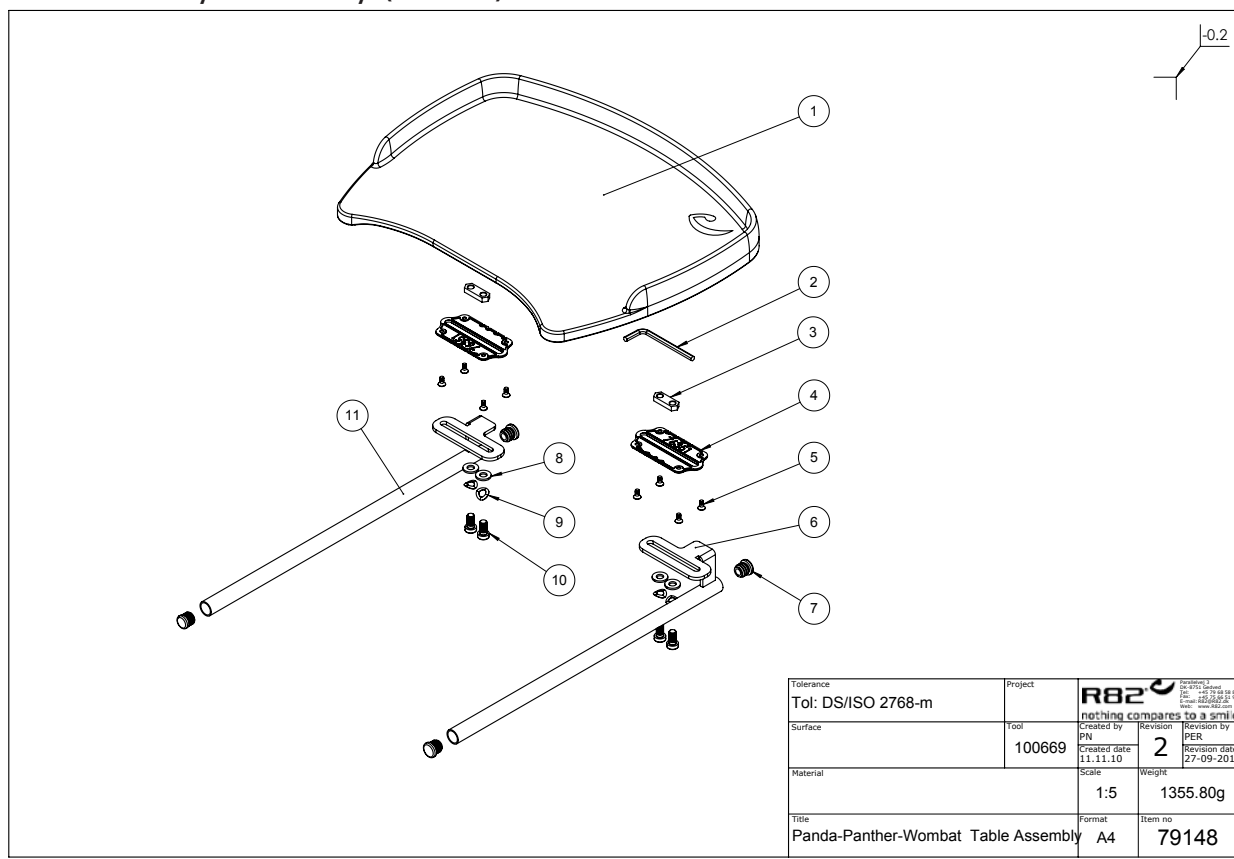

## Item numbers for Wombat tray assembly (79148)

| Position | No.     | Description                    | Quantity | Unit of Measure | Ean code      |
|----------|---------|--------------------------------|----------|-----------------|---------------|
| 1        | 76350   | Table for Stingray, Grey       | PCS      | 1               | 5707292541727 |
| 2        | 0589500 | Insex keys n5                  | PCS      | 1               | 5707292137869 |
| 3        | 77557   | Slide bar (M8x2)               | PCS      | 2               | 5707292544995 |
| 4        | 76351   | Mounting plate                 | PCS      | 2               | 5707292541741 |
| 5        | 1940011 | Plastite 60, 4*8               | PCS      | 8               | 5707292545978 |
| 6+7+11   | 5075364 | Brackets for table sz 4        | SÆT      | 1               | 5707292543608 |
| 8        | 1105080 | Stainless washer 8mm           | PCS      | 4               | 5707292140364 |
| 9        | 1100500 | Spring washer 8mm, stainless   | PCS      | 4               | 5707292140142 |
| 10       | 0661500 | Button head screw 8*16 stainl. | PCS      | 4               | 5707292139559 |


## Tray (79185)

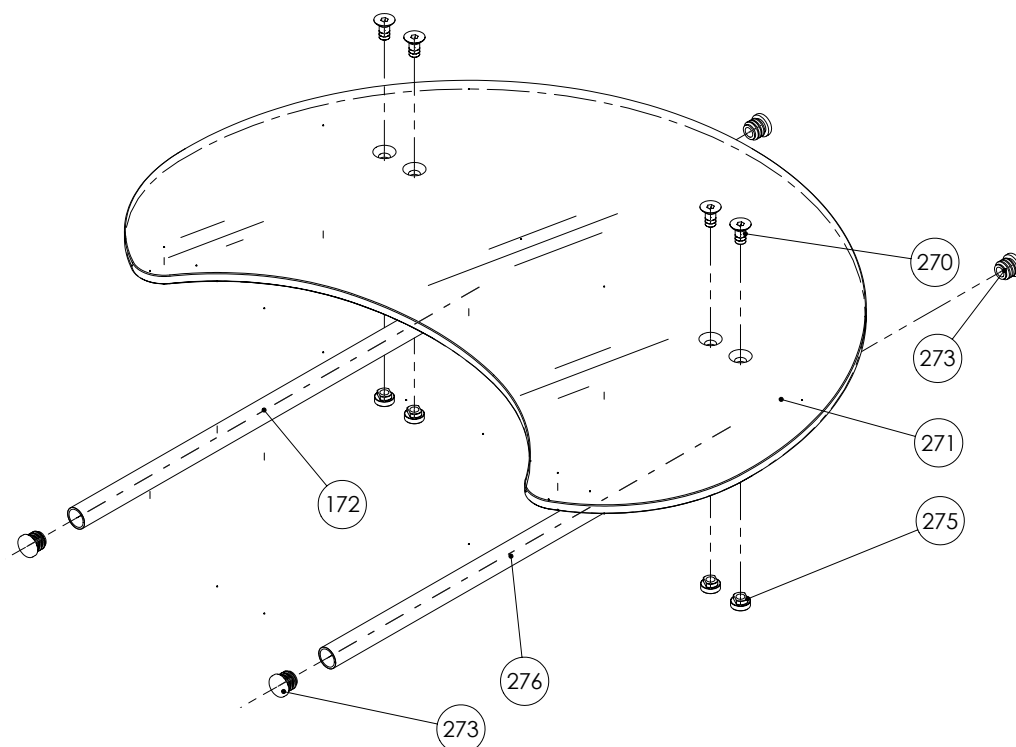

## Item numbers for Tray (79185)

| Position | No.     | Description                       | Quantity | Unit of Measure | Ean code      |
|----------|---------|-----------------------------------|----------|-----------------|---------------|
| 172      | 82865   | Mounting for tray, welded         | PCS      | 1               | 5707292125590 |
| 270      | 0624800 | Insex screw counters. 8*16        | PCS      | 4               | 5707292139054 |
| 271      | 82998   | Tray plate, 8 mm acryl (Size 1+2) | PCS      | 1               | 5707292126405 |
| 271      | 82695   | Tray plate, 8 mm acryl (Size 3)   | PCS      | 1               | 5707292402172 |
| 273      | 2502002 | Ferrule d18*0,8-2,5 black         | PCS      | 4               | 5707292433947 |
| 275      | 6118930 | Nut M8                            | PCS      | 4               | 5707292353825 |
| 276      | 82864   | Mounting for tray, welded         | PCS      | 1               | 5707292125583 |


## Hip support (126874)

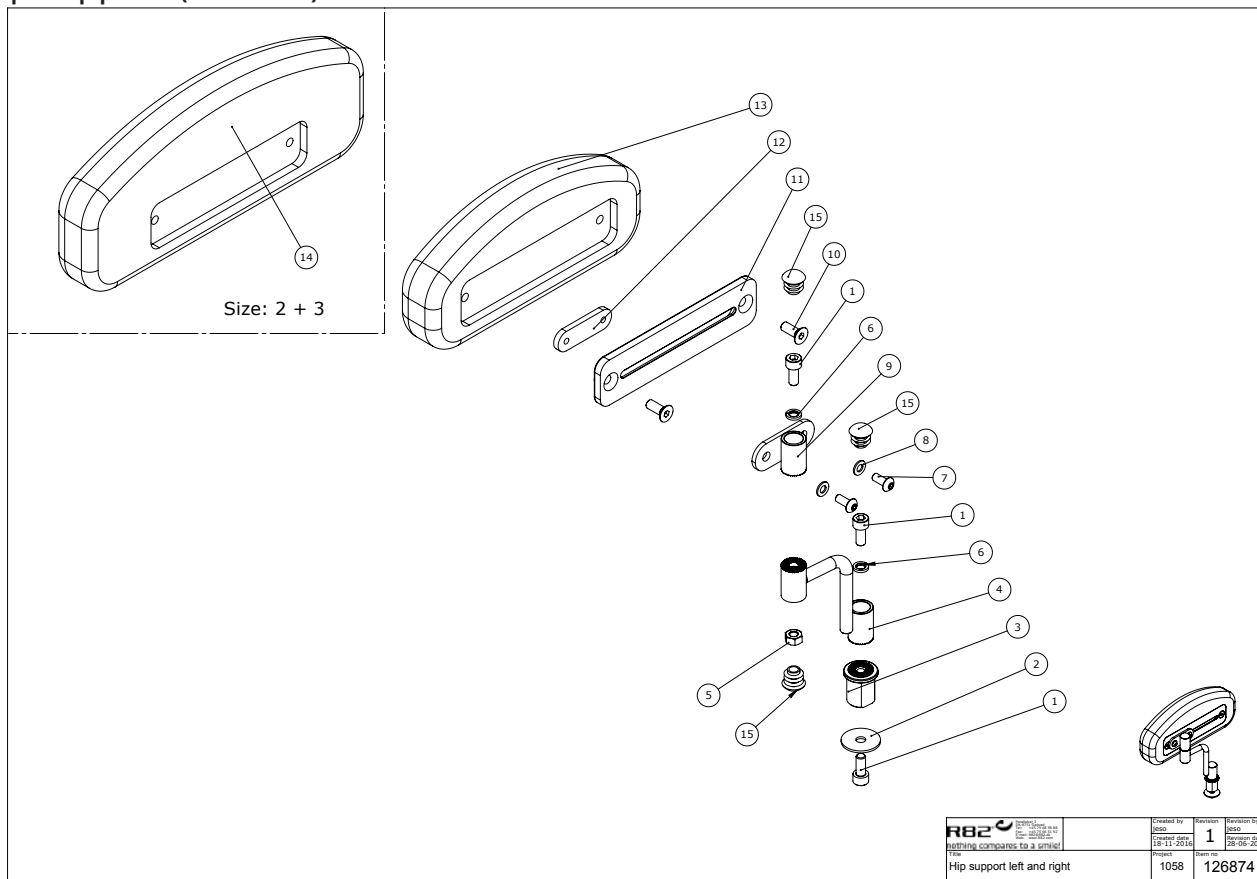

## Item numbers for Hip support (126874)

| Position | No.     | Description                    | Quantity | Unit of Measure | Ean code      |
|----------|---------|--------------------------------|----------|-----------------|---------------|
| 1        | 0602199 | Insex M6*14, FZB               | STK      | 6               | 5707292806000 |
| 2        | 1102775 | M 6,4*25*1,5 planskive         | STK      | 2               | 5707292785657 |
| 3        | 128356  | Tooth bracket                  | STK      | 2               | 5707292794727 |
| 4        | 126920  | Hip support assembly           | STK      | 2               | 5707292785558 |
| 5        | 1603000 | Steel nut 6 mm                 | STK      | 2               | 5707292141279 |
| 6        | 1100900 | Spring washer M6 fzb, 6.1/9.9  | STK      | 4               | 5707292459336 |
| 7        | 0658012 | Button head M5*12 FZB          | STK      | 4               | 5707292139382 |
| 8        | 1102902 | Washer 5mm, A2                 | STK      | 4               | 5707292411280 |
| 9        | 127886  | Tooth disc bracket for Hip Sup | STK      | 2               | 5707292786388 |
| 10       | 0621600 | Insex screw counters. 6*16     | STK      | 4               | 5707292138965 |
| 11       | 82953   | Plate for hip support          | STK      | 2               | 5707292126047 |
| 12       | 82974   | Threaded plate for hip support | STK      | 2               | 5707292126184 |
| 13       | 6175171 | Cushion size 1                 | STK      | 2               | 5707292189301 |
| 14       | 6175172 | Cushion size 2                 | STK      | 2               | 5707292189318 |
| 15       | 2501600 | Ferrule d15*1,0 black          | STK      | 6               | 5707292143969 |


## Push handle complete size 1 (121058)

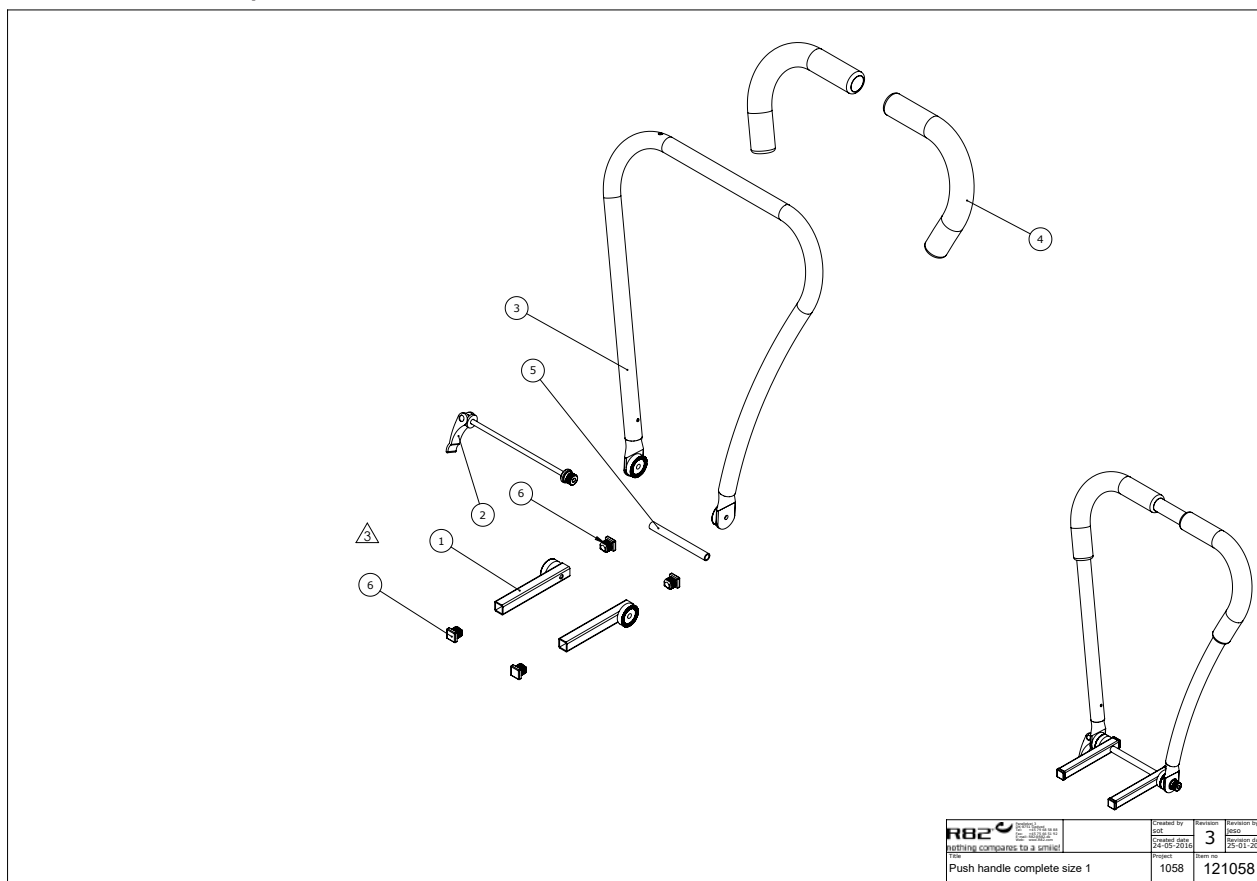

### Item numbers for Push handle complete size 1 (121058)

| Position | No.     | Description                    | Quantity | Unit of Measure | Ean code      |
|----------|---------|--------------------------------|----------|-----------------|---------------|
| 1        | 120409  | Bracket for handbrace size 1   | PCS      | 2               | 5707292621641 |
| 2        | 121061  | Handle size 1 angle adjustment | PCS      | 1               | 5707292622686 |
| 3        | 119439  | Handbrace size 1 Wombat        | PCS      | 1               | 5707292622112 |
| 4        | 2500520 | Tpe cover Ø20mm                | PCS      | 2               | 5707292401649 |
| 5        | 121080  | Ø10 tube                       | PCS      | 1               | 5707292623317 |

## Push handle complete size 2 (121218)

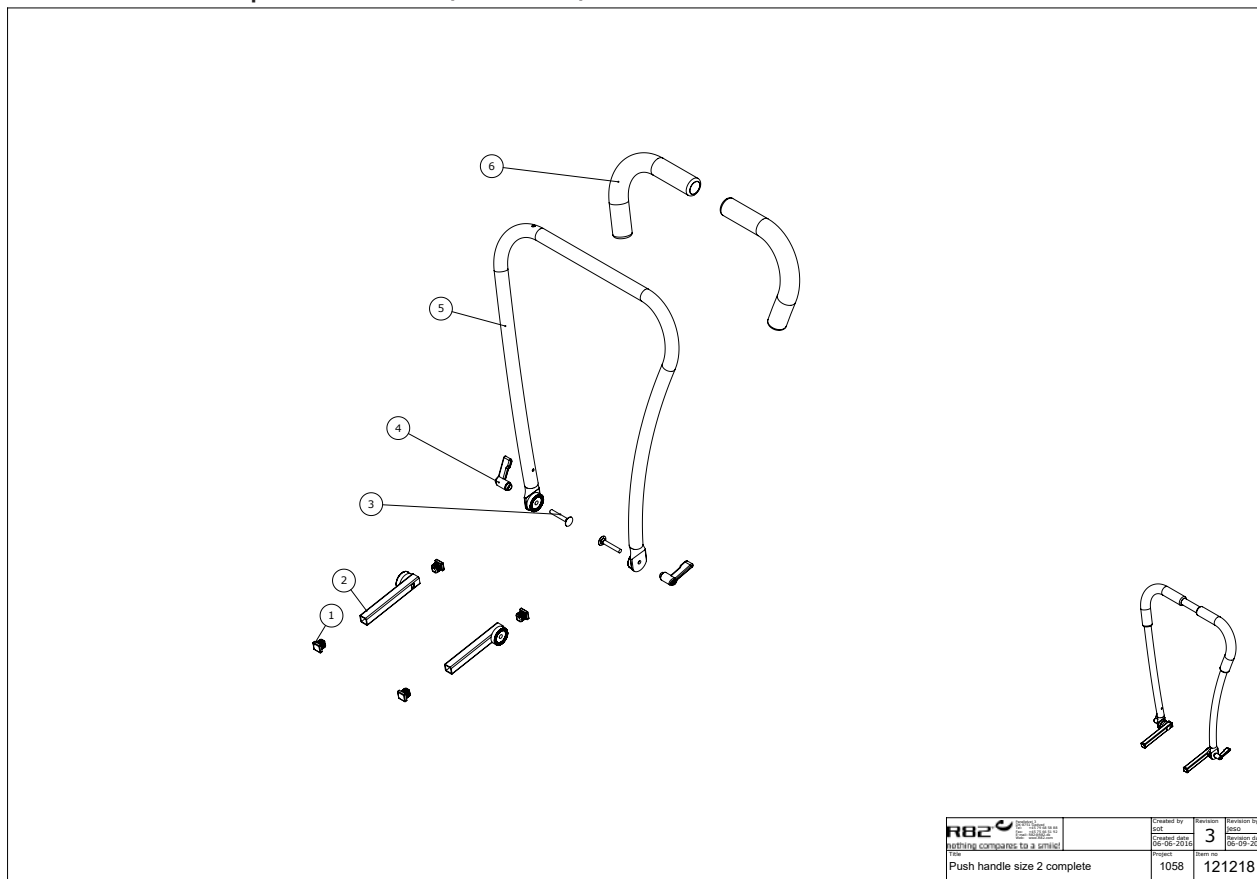


## Item numbers for Push handle complete size 2 (121218)

| Position | No.     | Description                    | Quantity | Unit of Measure | Ean code      |
|----------|---------|--------------------------------|----------|-----------------|---------------|
| 1        | 120410  | Bracket for handbrace size 2   | PCS      | 2               | 5707292621931 |
| 2        | 121070  | Handle size 2 angle adjustment | PCS      | 1               | 5707292622679 |
| 3        | 120635  | Handbrace size 2 Wombat svejst | PCS      | 1               | 5707292622181 |
| 4        | 2500520 | Tpe cover Ø20mm                | PCS      | 2               | 5707292401649 |
| 5        | 121219  | Ø10 tube 175mm                 | PCS      | 1               | 5707292623386 |
| 6        | 2504000 | Ferrule e15*1,0-2,0 int.black  | PCS      | 2               | 5707292144102 |




## Hand brace size 3 (121577)

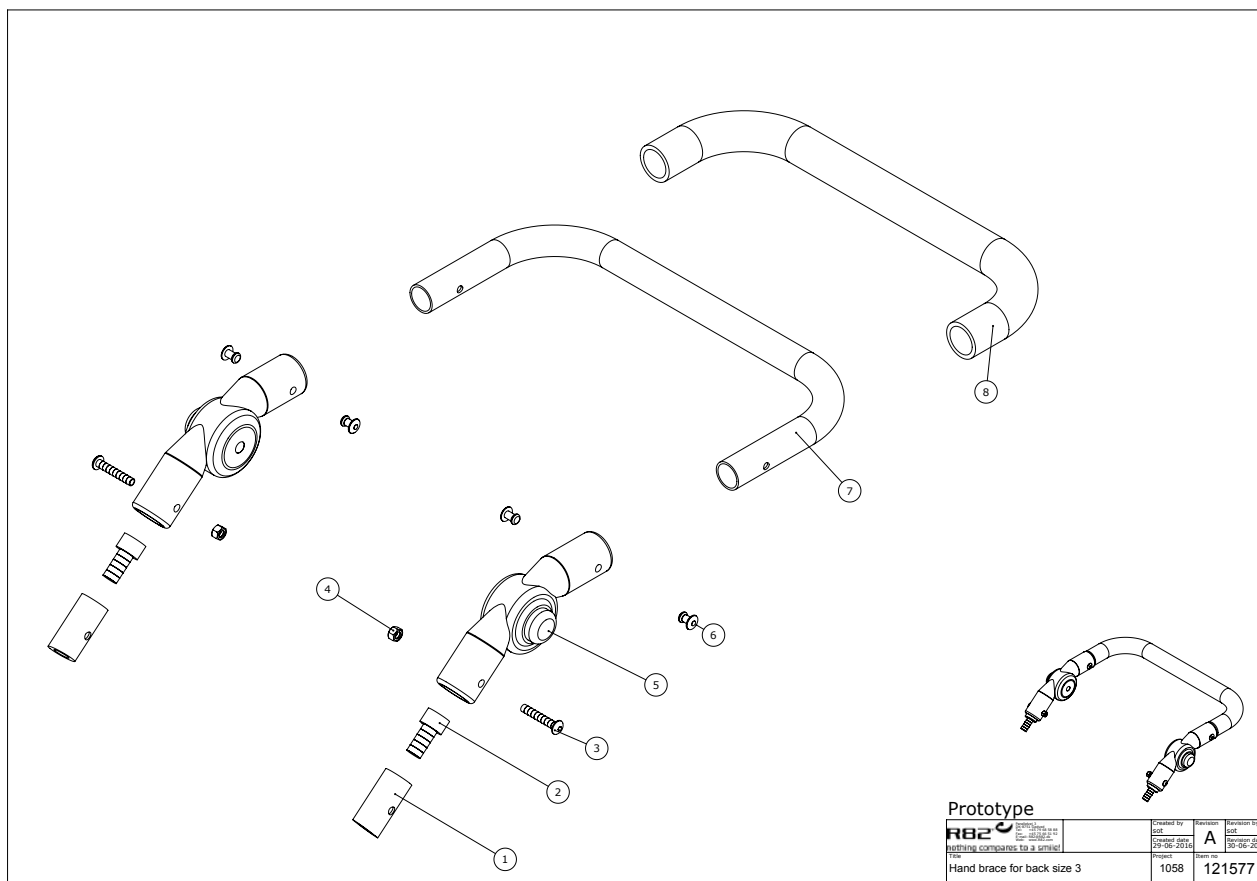

## Item numbers for Hand brace size 3 (121577)

| Position | No.     | Description                 | Quantity | Unit of Measure | Ean code      |
|----------|---------|-----------------------------|----------|-----------------|---------------|
| 1        | 80705   | Bush for handbrace          | PCS      | 2               | 5707292436900 |
| 2        | 0604000 | Insex screw 10*20           | PCS      | 2               | 5707292138361 |
| 3        | 0640935 | Button head M5*35 rf        | PCS      | 2               | 5707292485748 |
| 4        | 1606300 | Lock nut 5 mm               | PCS      | 2               | 5707292141354 |
| 5        | 0553090 | LOCKING JOINT Ø20MM         | PCS      | 2               | 5707292482235 |
| 6        | 0506508 | Rivet 5*14, stainless       | PCS      | 4               | 5707292359476 |
| 7        | 80706   | Tube for handbrace          | PCS      | 1               | 5707292436917 |
| 8        | 2500550 | Grip 700mm length for Ø20mm | PCS      | 1               | 5707292388520 |


## Back extension for wombat size 3 (79196)

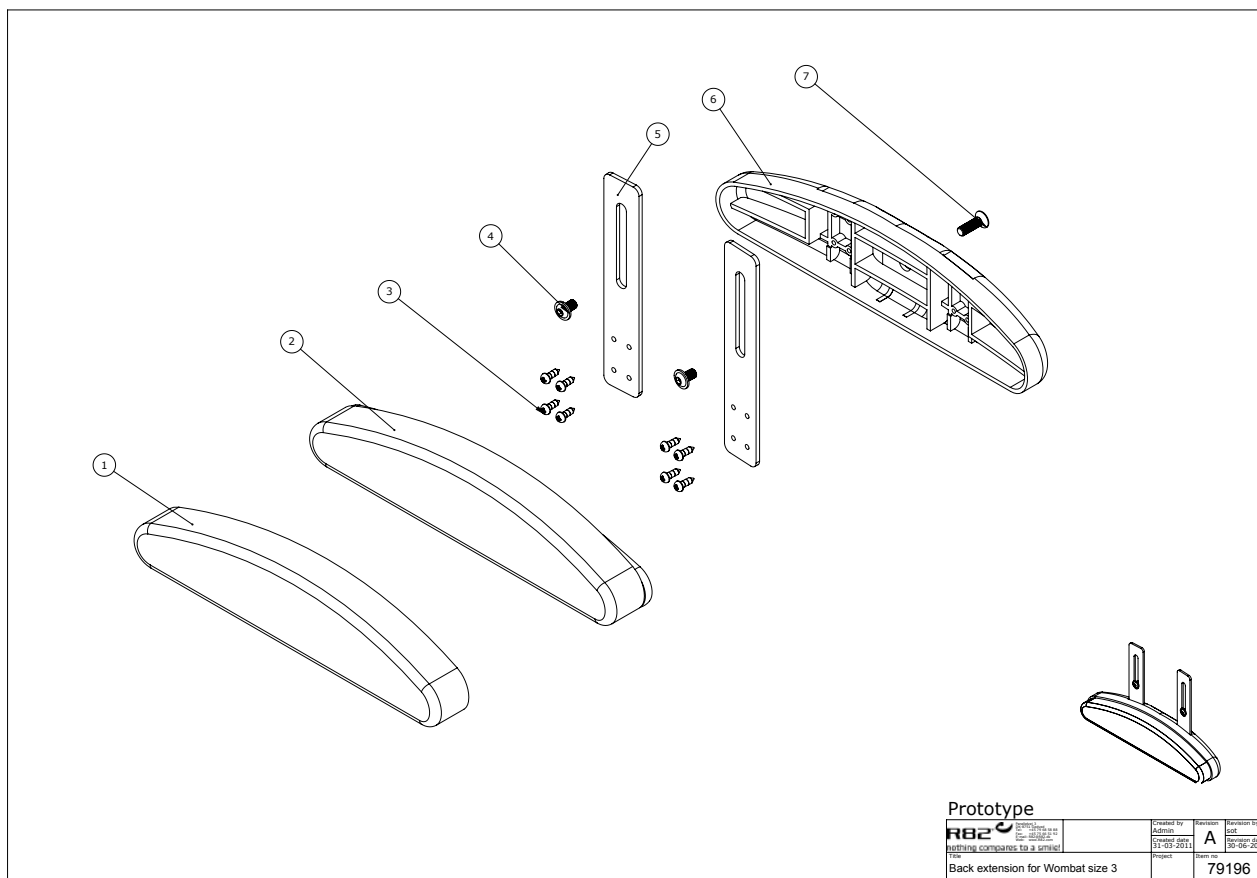

## Item numbers for Back extension for wombat size 3 (79196)

| Position | No.     | Description                       | Quantity | Unit of Measure | Ean code      |
|----------|---------|-----------------------------------|----------|-----------------|---------------|
| 1        | 121582  | Upholstery Lumbar support Wombat3 | PCS      | 1               | 5707292630032 |
| 2        | 5700483 | Upholstered lower back supp       | PCS      | 1               | 5707292399304 |
| 3        | 1940033 | ECO-SYN Z M3,5*10 RUSTFRI         | PCS      | 8               | 5707292422965 |
| 4        | 0659010 | Button head with flange M6*10     | PCS      | 2               | 5707292356819 |
| 5        | 82744   | MOUNTINGPLATE                     | PCS      | 2               | 5707292398901 |
| 6        | 82748   | LUMBAR SUPPORT                    | PCS      | 1               | 5707292400093 |
| 7        | 0612500 | Insex screw counters. 6*25        | PCS      | 1               | 5707292138712 |


## Lumba support straight, size 1-2 (128370)

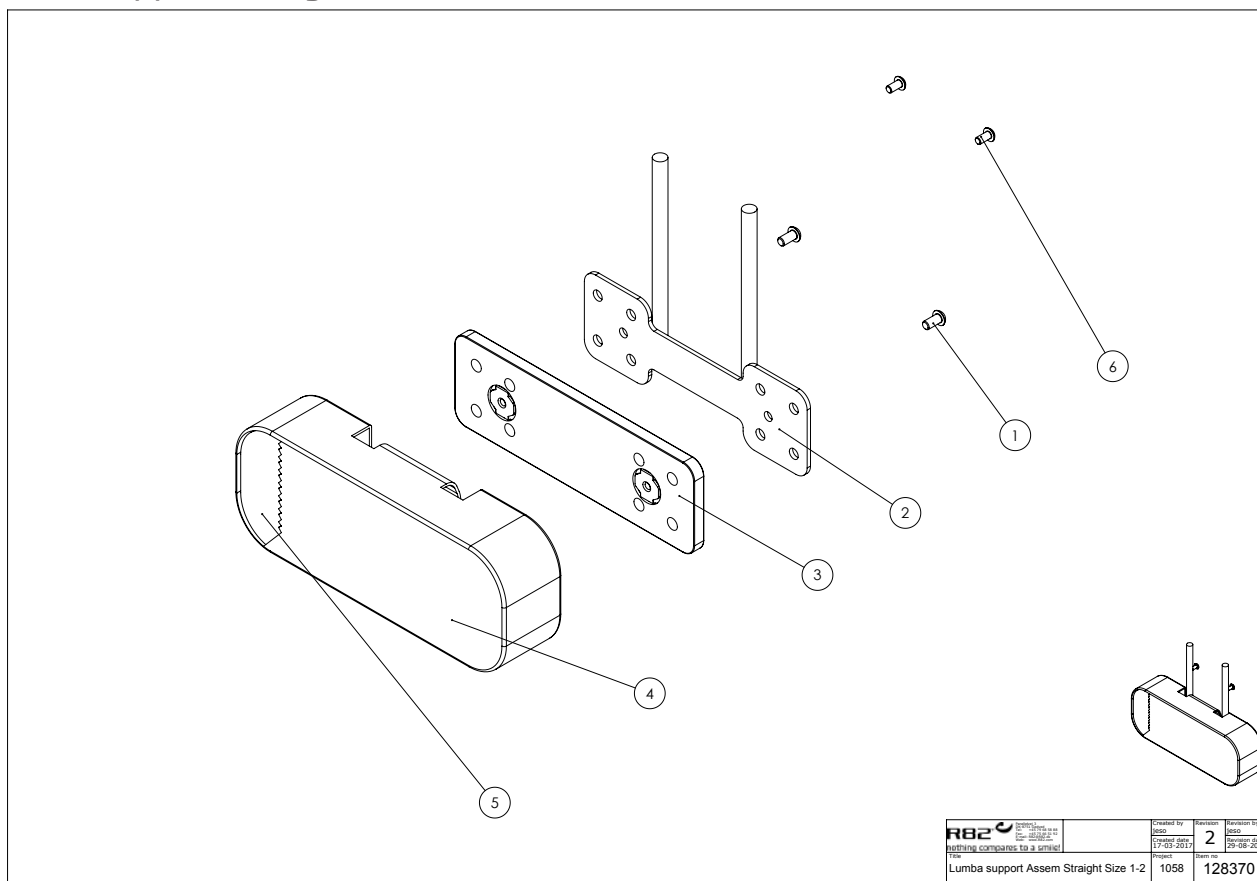

## Item numbers for Lumba support straight, size 1-2 (128370)

| Position | No.      | Description                    | Quantity | Unit of Measure | Ean code      |
|----------|----------|--------------------------------|----------|-----------------|---------------|
| 1        | 0641200  | Button head screw 6*12         | PCS      | 2               | 5707292139252 |
| 2        | 126965   | Lumbar support beslag          | PCS      | 1               | 5707292785589 |
| 3        | 127681   | Plate w yelo nut low back supp | PCS      | 1               | 5707292786975 |
| 4        | 128366   | Cover for sacrum Support zs1+2 | PCS      | 1               | 5707292806468 |
| 5        | 128372-1 | Foam straight size 1-2         | PCS      | 1               | 5707292807267 |
| 6        | 0640910  | Button head M5*10 FZB          | PCS      | 2               | 5707292380142 |


## Lumba support straight, size 3 (128375)

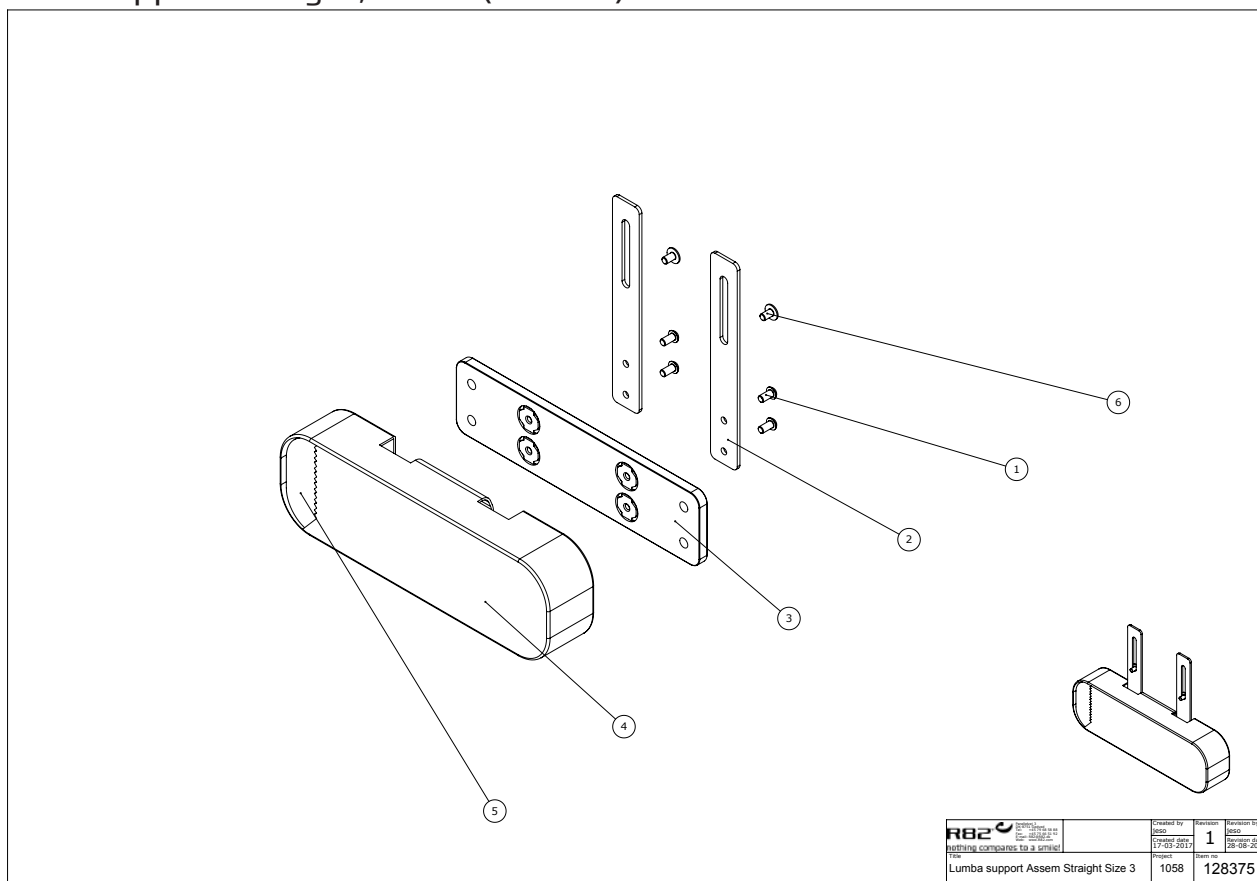

### Item numbers for Lumba support straight size 3 (128375)

| Position | No.      | Description                    | Quantity | Unit of Measure | Ean code      |
|----------|----------|--------------------------------|----------|-----------------|---------------|
| 1        | 0641200  | Button head screw 6*12         | PCS      | 4               | 5707292139252 |
| 2        | 127608   | Plate for lumbar support       | PCS      | 2               | 5707292787019 |
| 3        | 127677   | Plate w yelo not low back supp | PCS      | 1               | 5707292786968 |
| 4        | 128377   | Cover for sacrum Support size3 | PCS      | 1               | 5707292807311 |
| 5        | 128376-1 | Foam straight size 3           | PCS      | 1               | 5707292807274 |
| 6        | 0659010  | Button head with flange M6*10  | PCS      | 2               | 5707292356819 |


## Lumba support with sides, size 1 (127031)

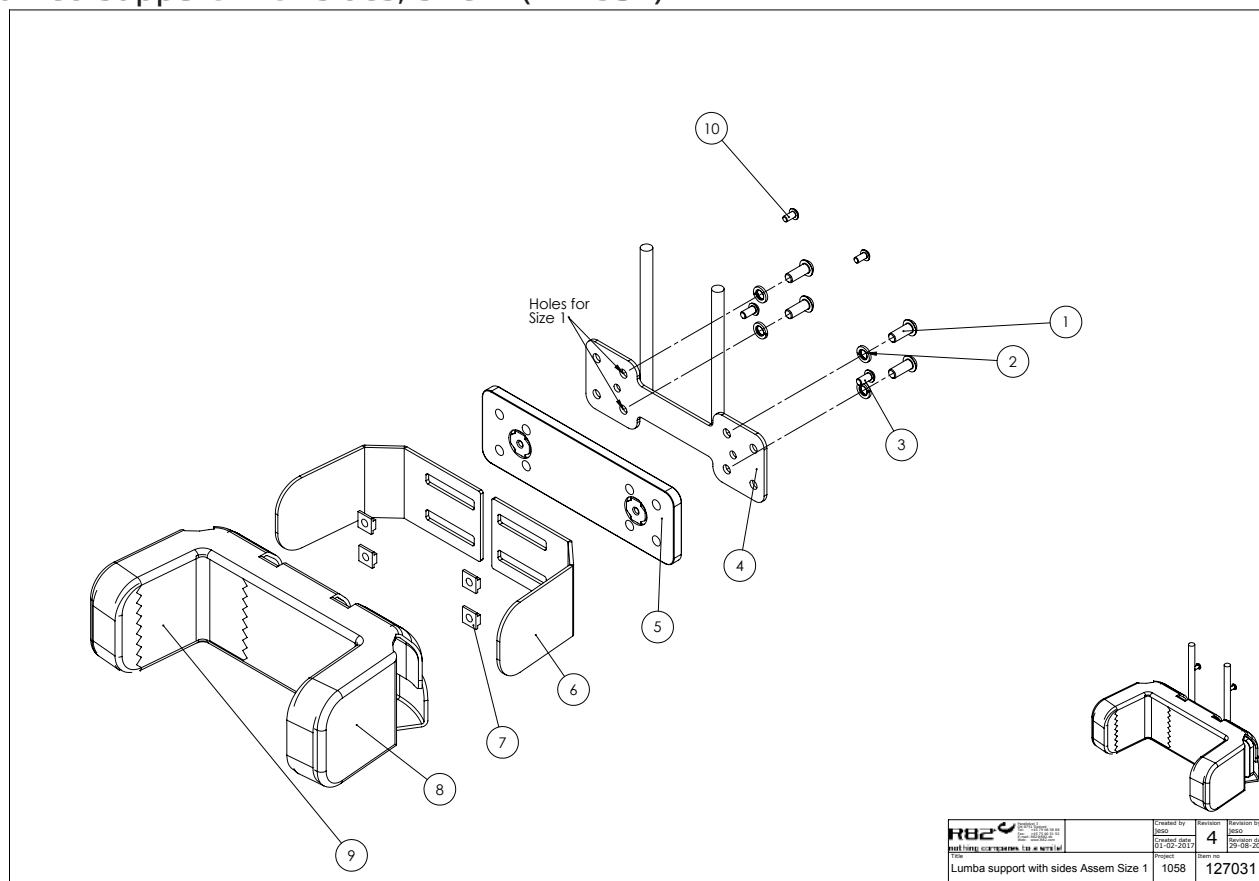

### Item numbers for Lumba support with sides, size 1 (127031)

| Position | No.      | Description                    | Quantity | Unit of Measure | Ean code      |
|----------|----------|--------------------------------|----------|-----------------|---------------|
| 1        | 0662000  | Button head screw 8*20 stainl. | PCS      | 4               | 5707292139573 |
| 2        | 1100500  | Spring washer 8mm, stainless   | PCS      | 4               | 5707292140142 |
| 3        | 0641200  | Button head screw 6*12         | PCS      | 2               | 5707292139252 |
| 4        | 126965   | Lumbar support beslag          | PCS      | 1               | 5707292785589 |
| 5        | 127681   | Plate w yelo nut low back supp | PCS      | 1               | 5707292786975 |
| 6        | 126967   | lumbar back support left       | PCS      | 2               | 5707292782502 |
| 7        | 75203-1  | M8 slide with threads          | PCS      | 4               | 5707292517593 |
| 8        | 128368   | Cover for sacrum Support size1 | PCS      | 1               | 5707292807328 |
| 9        | 128224-1 | Foam with sides size 1         | PCS      | 1               | 5707292807281 |
| 10       | 0640910  | Button head M5*10 FZB          | PCS      | 2               | 5707292380142 |


## Lumba support with sides, size 2 (128279)

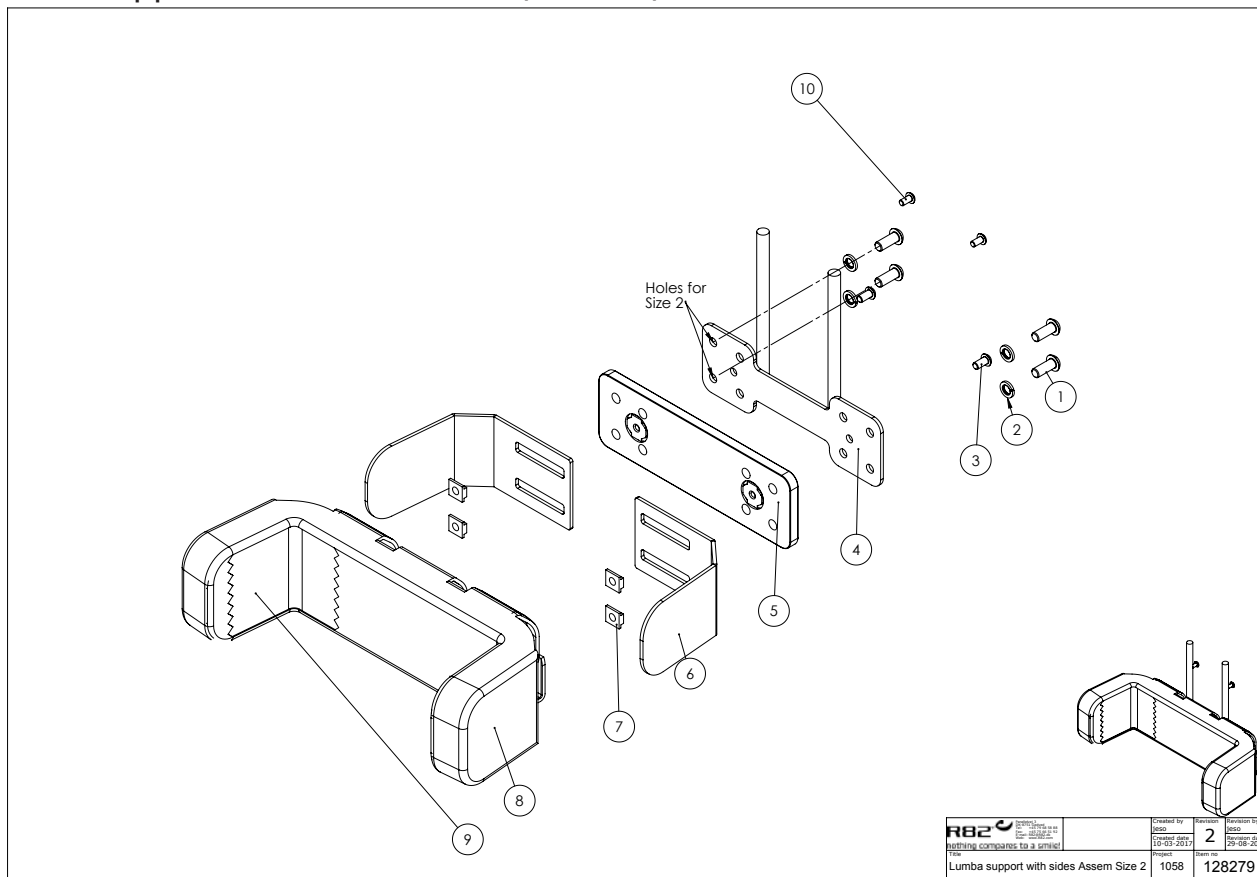

### Item numbers for Lumba support with sides, size 2 (128279)

| Position | No.      | Description                    | Quantity | Unit of Measure | Ean code      |
|----------|----------|--------------------------------|----------|-----------------|---------------|
| 1        | 0662000  | Button head screw 8*20 stainl. | PCS      | 4               | 5707292139573 |
| 2        | 1100500  | Spring washer 8mm, stainless   | PCS      | 4               | 5707292140142 |
| 3        | 0641200  | Button head screw 6*12         | PCS      | 2               | 5707292139252 |
| 4        | 126965   | Lumbar support beslag          | PCS      | 1               | 5707292785589 |
| 5        | 127681   | Plate w yelo nut low back supp | PCS      | 1               | 5707292786975 |
| 6        | 126967   | lumbar back support left       | PCS      | 2               | 5707292782502 |
| 7        | 75203-1  | M8 slide with threads          | PCS      | 4               | 5707292517593 |
| 8        | 128280   | Cover for sacrum Support size2 | PCS      | 1               | 5707292807335 |
| 9        | 128281-1 | Foamwidth sides size 2         | PCS      | 1               | 5707292807298 |
| 10       | 0640910  | Button head M5*10 FZB          | PCS      | 2               | 5707292380142 |


## Wombat central brake (79192)

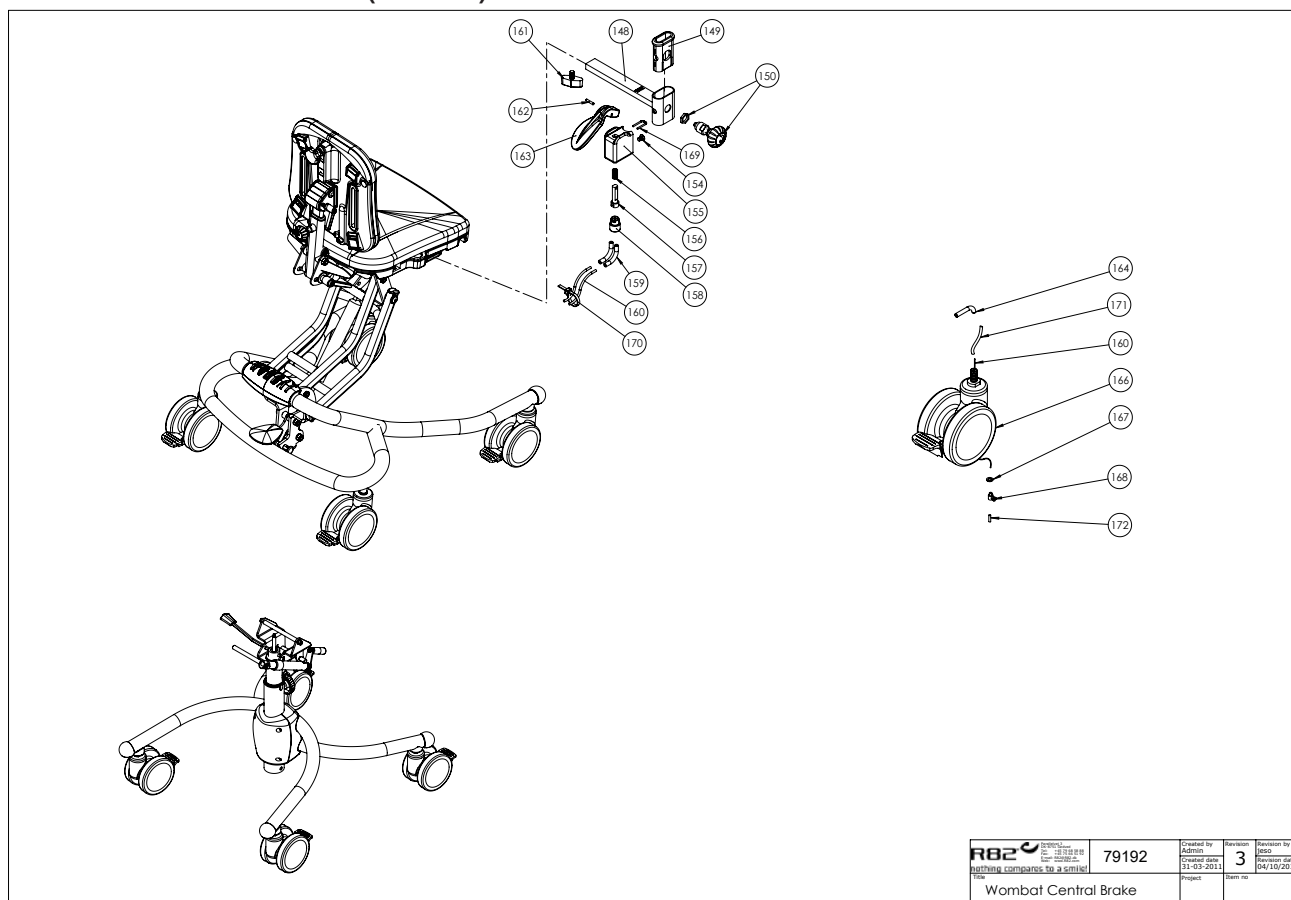

### Item numbers for Wombat central brake (79192)

| Position | No.     | Description                    | Quantity | Unit of Measure | Ean code      |
|----------|---------|--------------------------------|----------|-----------------|---------------|
| 154      | 0631809 | Insex uh. M5*5                 | PCS      | 1               | 5707292353337 |
| 155      | 70140   | Brakeholder, Rabbit            | PCS      | 1               | 5707292442833 |
| 156      | 0310129 | pressure spring 0,9*11*26mm*7v | PCS      | 1               | 5707292438782 |
| 157      | 72983   | Pin for brake                  | PCS      | 1               | 5707292438836 |
| 158      | 70139   | Plug                           | PCS      | 1               | 5707292101648 |
| 159      | 0389000 | Angled cable tightener, 45°    | PCS      | 2               | 5707292136152 |
| 160      | 0371850 | Strained wire 1,5-19 tr.1      | PCS      | 2               | 5707292135872 |
| 161      | 3652501 | R82 handle M6*16mm screw       | PCS      | 1               | 5707292413963 |
| 162      | 0529038 | Cyl.stift Ø3,0x18 mm.          | PCS      | 1               | 5707292439260 |
| 163      | 72964   | Brake grip for Central brake   | PCS      | 1               | 5707292499745 |
| 164      | 5082909 | Cable guide                    | PCS      | 2               | 5707292503107 |
| 166      | 3006950 | 100 mm double wheel w/central  | PCS      | 2               | 5707292145925 |
| 167      | 1102800 | Fazet washer 4mm               | PCS      | 2               | 5707292140265 |
| 168      | 0388010 | Screwnippel 7*7mm              | PCS      | 2               | 5707292136121 |
| 169      | 6108146 | Safety brace Ø2                | PCS      | 1               | 5707292185495 |
| 170      | 0235075 | Cable tape, black              | PCS      | 1               | 5707292135308 |
| 171      | 5002965 | Cable for central brake size 3 | PCS      | 2               | 5707292408846 |
| 171      | 5002964 | Outer casing f. Centralbreak   | PCS      | 2               | 5707292353689 |
| 172      | 0387000 | End nipple for outer casing    | PCS      | 2               | 5707292136091 |
| Modul    | 5082870 | Mounting bracket for armrest   | PCS      | 1               | 5707292539618 |

## Anti-tip (79244)

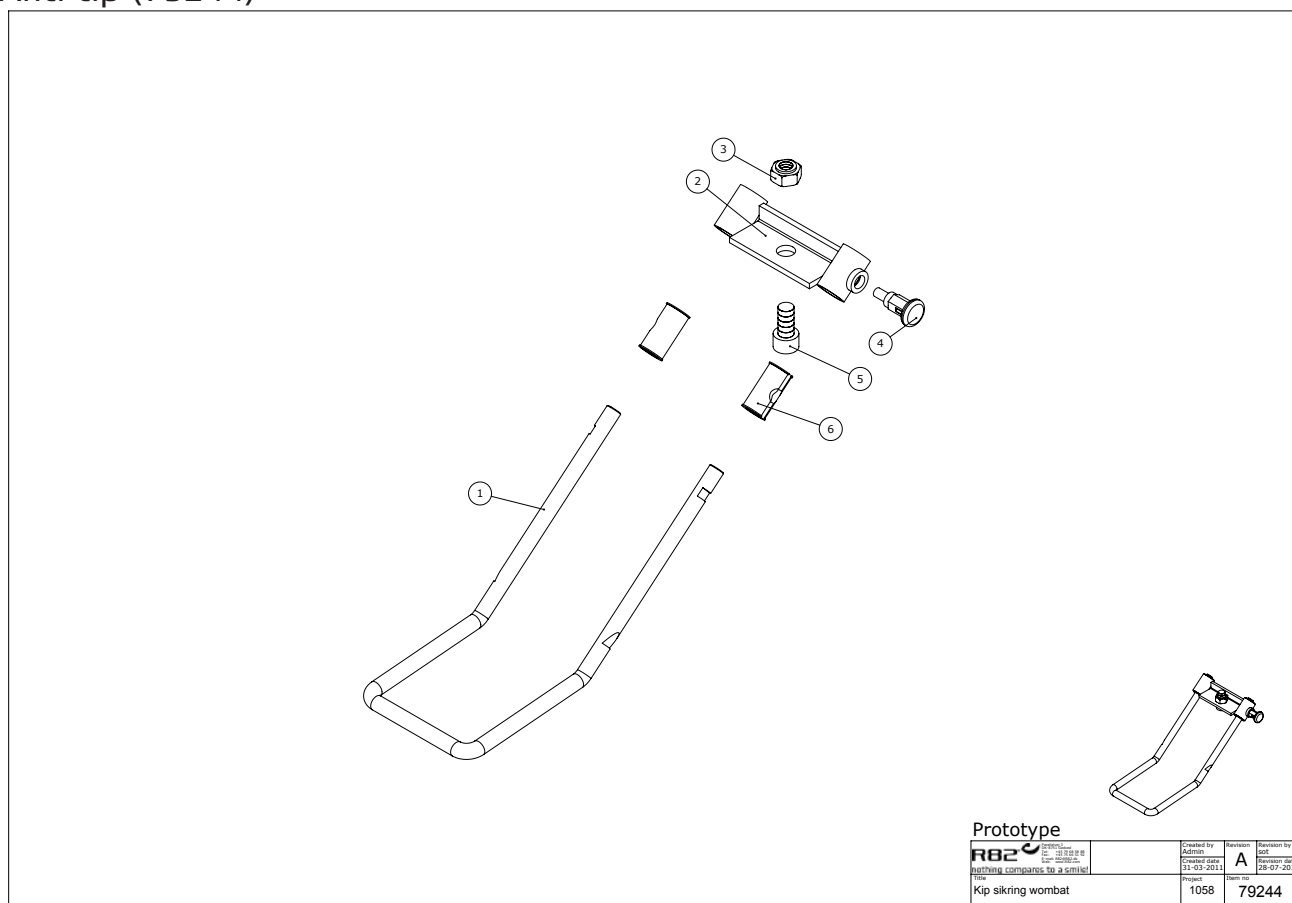


## Item numbers for Anti-tip (79244)

| Position | No.     | Description                       | Quantity | Unit of Measure | Ean code      |
|----------|---------|-----------------------------------|----------|-----------------|---------------|
| 1        | 121582  | Upholstery Lumbar support Wombat3 | PCS      | 1               | 5707292630032 |
| 2        | 5700483 | Upholstered lower back supp       | PCS      | 1               | 5707292399304 |
| 3        | 1940033 | ECO-SYN Z M3,5*10 RUSTFRI         | PCS      | 8               | 5707292422965 |
| 4        | 0659010 | Button head with flange M6*10     | PCS      | 2               | 5707292356819 |
| 5        | 82744   | MOUNTINGPLATE                     | PCS      | 2               | 5707292398901 |
| 6        | 82748   | LUMBAR SUPPORT                    | PCS      | 1               | 5707292400093 |
| 7        | 0612500 | Insex screw counters. 6*25        | PCS      | 1               | 5707292138712 |




## Helper's handle (79185)

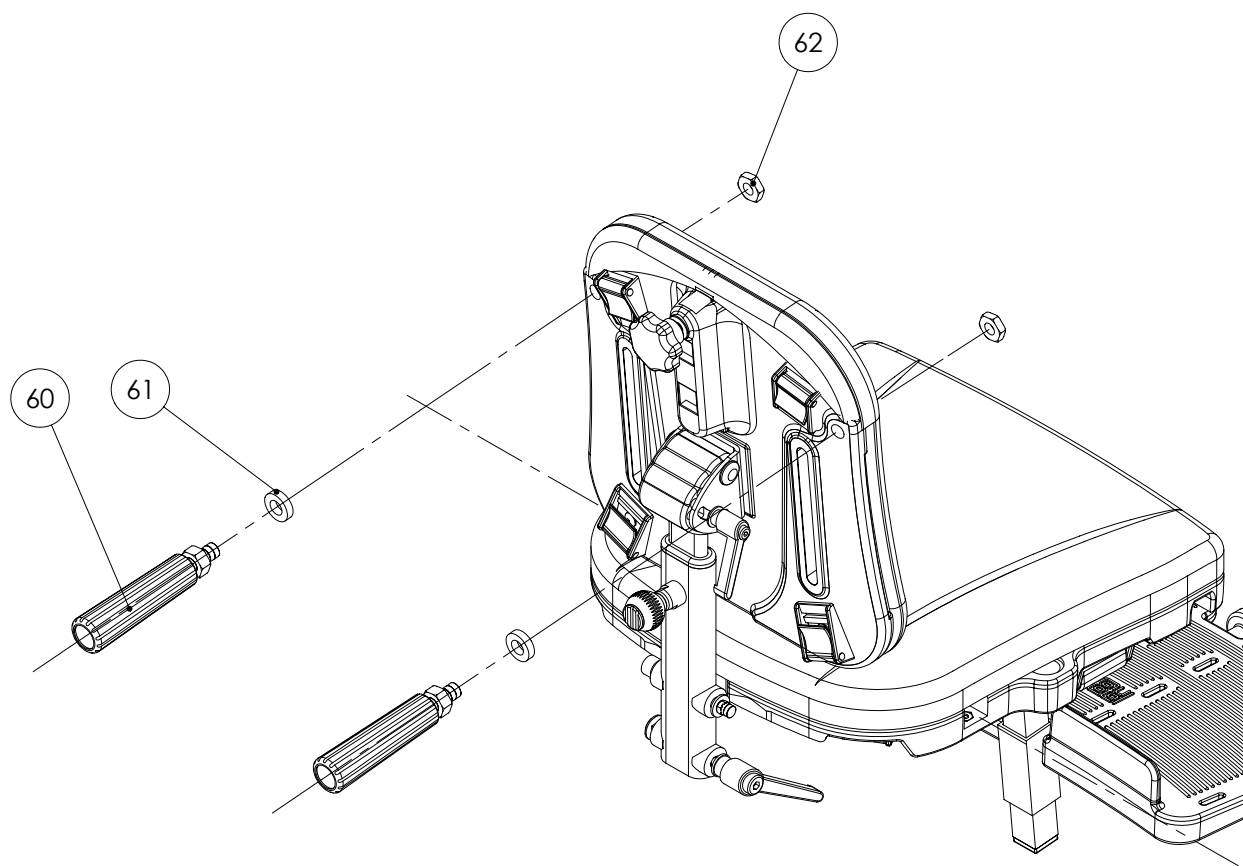

## Item numbers for Helper's handle (79185)

| Position | No.     | Description                   | Quantity | Unit of Measure | Ean code      |
|----------|---------|-------------------------------|----------|-----------------|---------------|
| 60       | 3805100 | Handle w/tilt 90*25 M10 black | PCS      | 2               | 5707292147264 |
| 61       | 1354520 | Spacer black 5*20*10,5mm      | PCS      | 2               | 5707292140982 |
| 62       | 1760010 | Compensating nut M10,fzb      | PCS      | 2               | 5707292141514 |


## Remote holder assembly (150952)

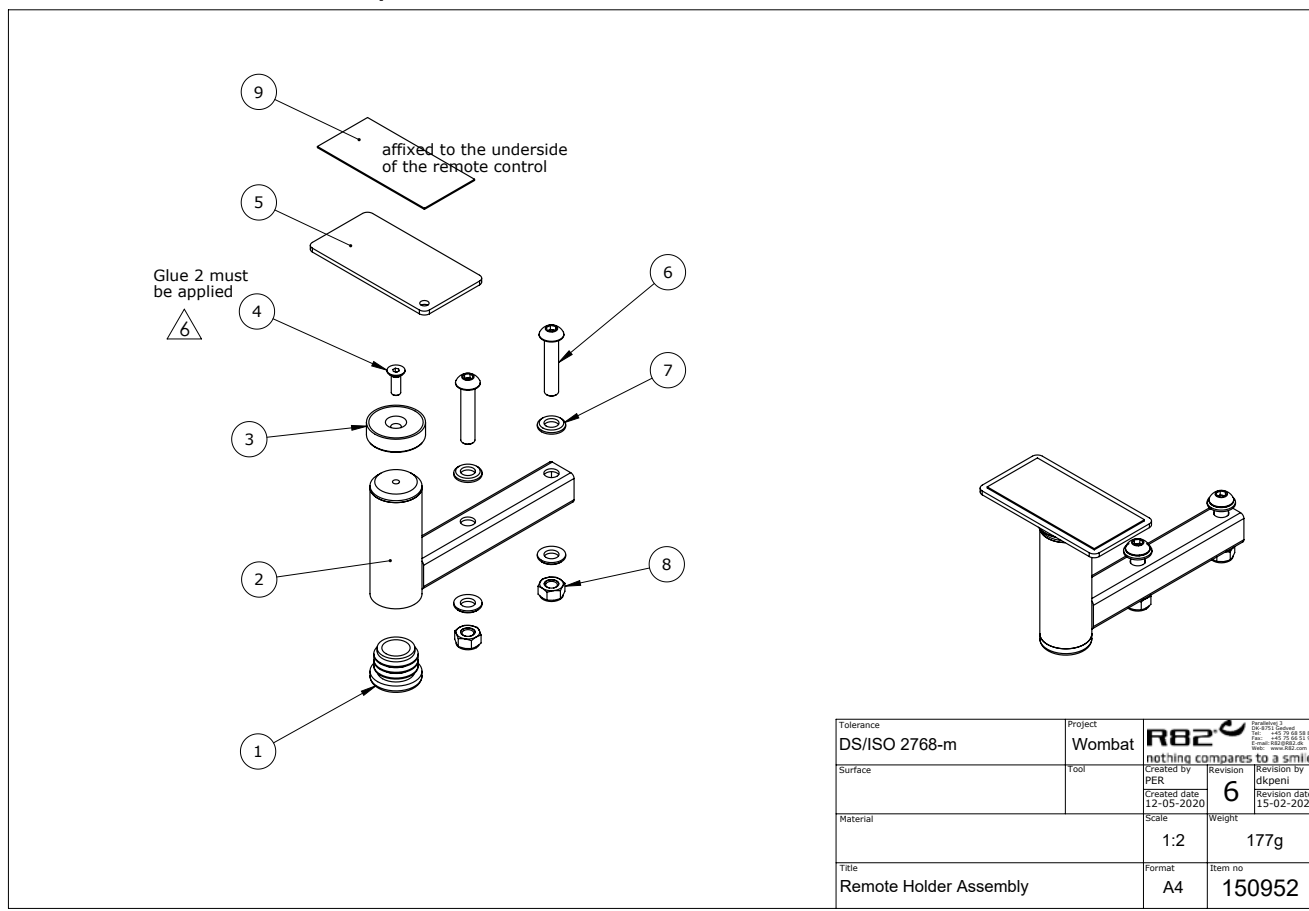

### Item numbers for Remote holder assembly (150952)

| Position | No.     | Description                | Quantity | Unit of Measure | Ean code      |
|----------|---------|----------------------------|----------|-----------------|---------------|
| 1        | 2503010 | Ferrule d22x1,8-3,5, black | 1        | PCS             | 5707292357632 |
| 2        | 150838  | Remote holder welded       | 1        | PCS             | 5707292917331 |
| 3        | 115700  | Magnet Ø25 x 7mm, N160     | 1        | PCS             | 5707292921123 |
| 4        | 0614012 | Insex base M4x12, FZB      | 1        | PCS             | 5707292138811 |
| 5        | 151774  | Plate for remote control   | 1        | PCS             | 5707292921697 |
| 6        | 0659580 | Button head screw 6x30 mm  | 2        | PCS             | 5707292139481 |
| 7        | 1103000 | Fazet washer 6mm           | 4        | PCS             | 5707292140289 |
| 8        | 1606500 | Lock nut 6 mm              | 2        | PCS             | 5707292141361 |
| 9        | 152135  | Tape for Plate             | 1        | PCS             | 5707292922229 |
|          |         |                            |          |                 |               |

## Remote holder assembly (150860)

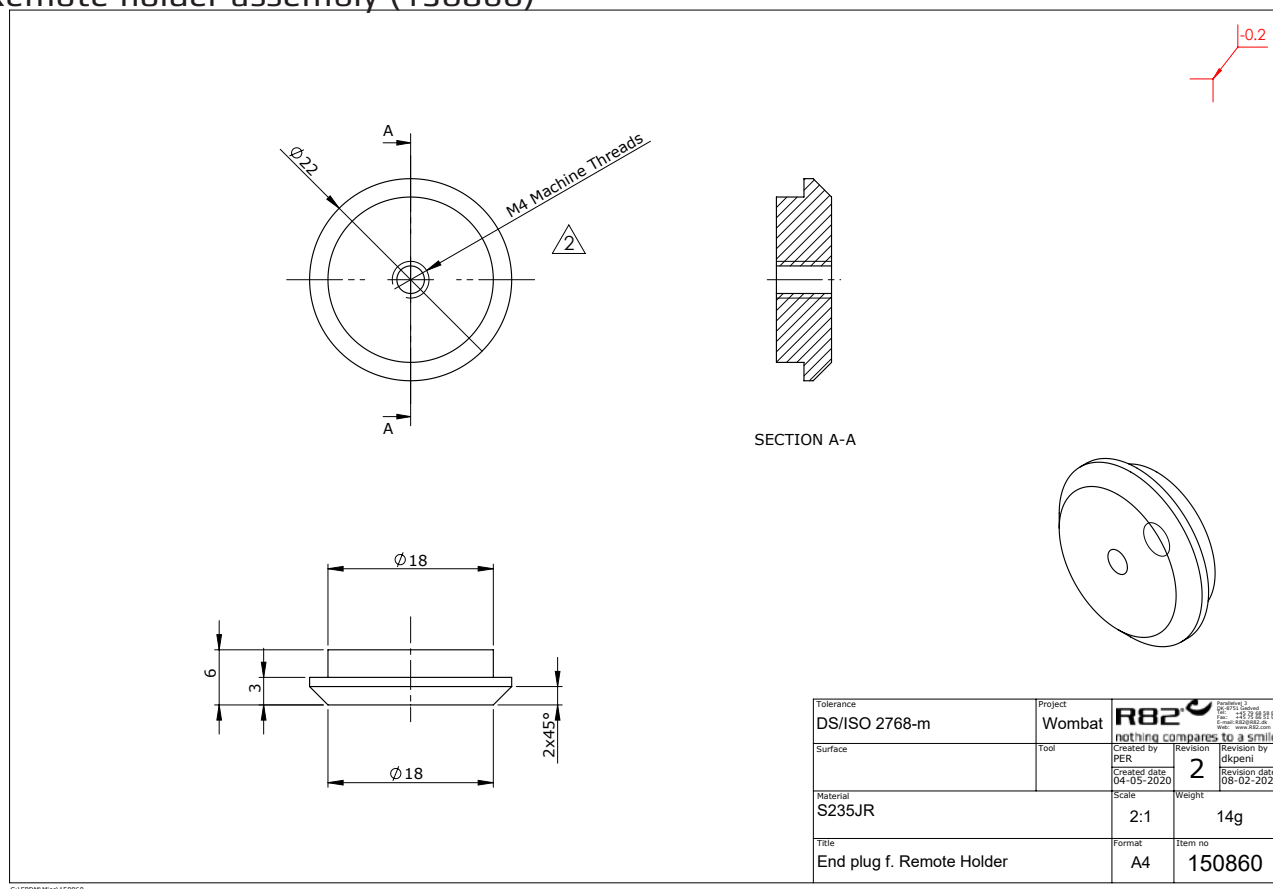

## Item numbers for Remote holder assembly (150860)

| Position | No.    | Description               | Quantity | Unit of Measure | Ean code      |
|----------|--------|---------------------------|----------|-----------------|---------------|
|          | 150860 | End plug f. Remote Holder | 1        | PCS             | 5707292917263 |