AIR SUPPLY




NO USER SERVICEABLE PARTS.

Only qualified service personnel shall perform repairs on the HoverTech International Air Supply.






Part Identification

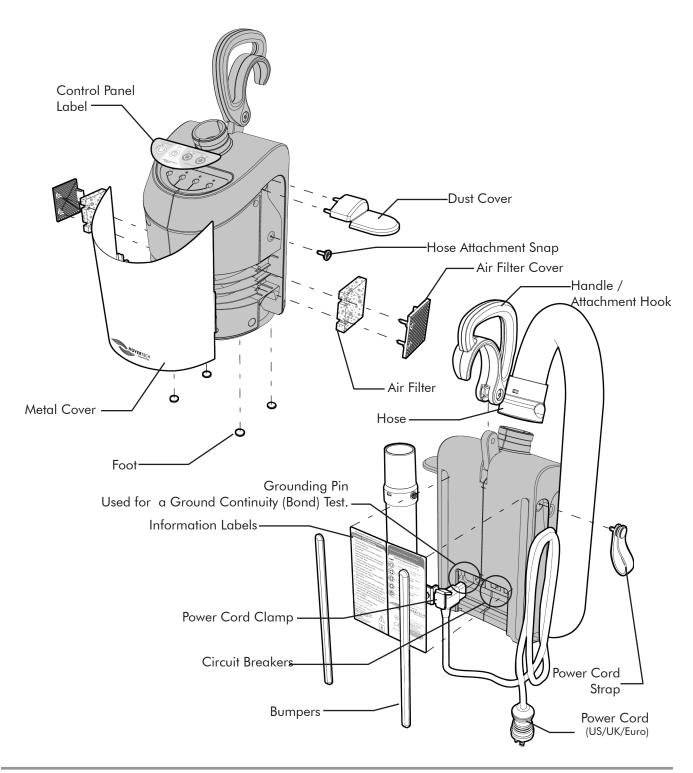





Power Cord / Clamp Replacement

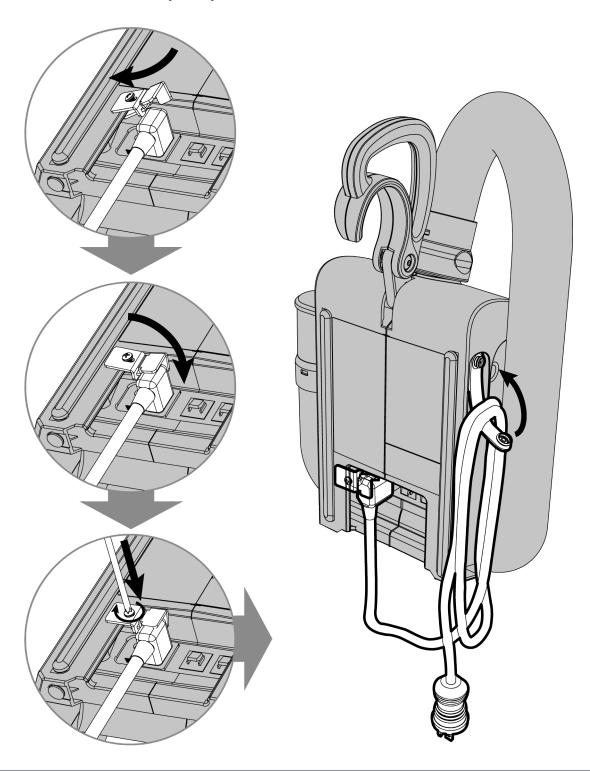






Handle Replacement

- 1. Remove the damaged handle by unscrewing the socket head screw from the barrel nut using two 5/32" allen wrenches as shown.
- 2. Attach the new handle by reversing the process. When tightening the screw be sure that the handle can rotate easily. The screw is treated with thread lock to secure it in place.

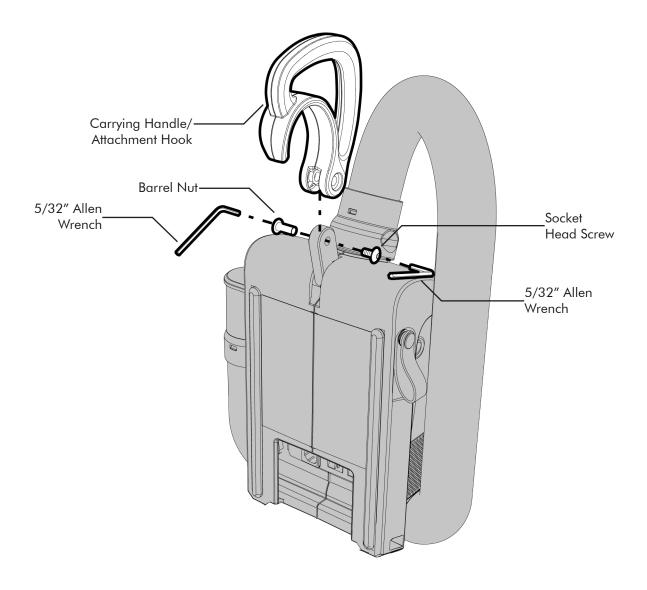






Feet or Bumper Replacement

- 1. The feet and bumpers are held in place by a self-adhesive backing. Use a small, flat bladed screwdriver to pry up an edge and gently remove the foot or bumper.
- 2. Clean surface thoroughly to remove any excess adhesive that may have been left behind. Apply the new part by removing the backing material and position as shown. Press firmly to ensure adhesion.

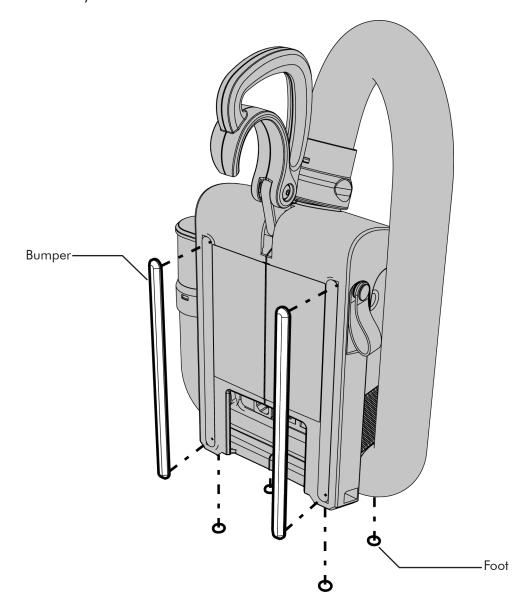






Hose Removal

- 1. Remove the damaged hose by lifting the dust cover slightly and unsnapping the hose from the side of the unit as shown.
- 2. Push the release button at the top of the unit to remove the hose.
- 3. Attach the new hose by reversing the process.

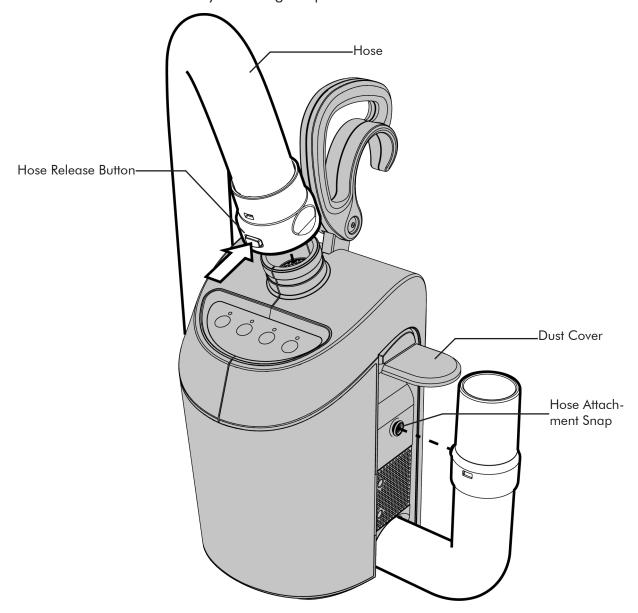






Air Filter and Air Filter Cover Replacement

To Remove Air Filter Covers

- 1. Disconnect hose from unit.
- 2. Remove the two phillips head screws on each side to detach the air filter covers.
- 3. Re-attach the new air filter covers and screws.

To Remove Air Filters

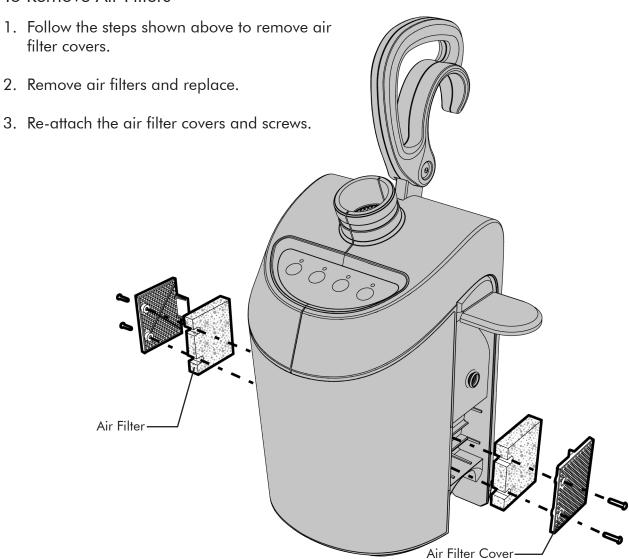






Dust Cover/Hose Attachment Snap Replacement

To Remove Dust Cover

- 1. Disconnect hose from unit.
- 2. Lift the cover "flap" to remove the 3 phillips head screws that attach the dust cover.
- 3. To replace dust cover, fold in the three flexible tabs. Then insert the screws one at a time starting with the top middle followed by the sides.

To Remove Hose Attachment Snap

- 1. Disconnect hose from unit.
- 2. Remove the phillips head screw and snap.

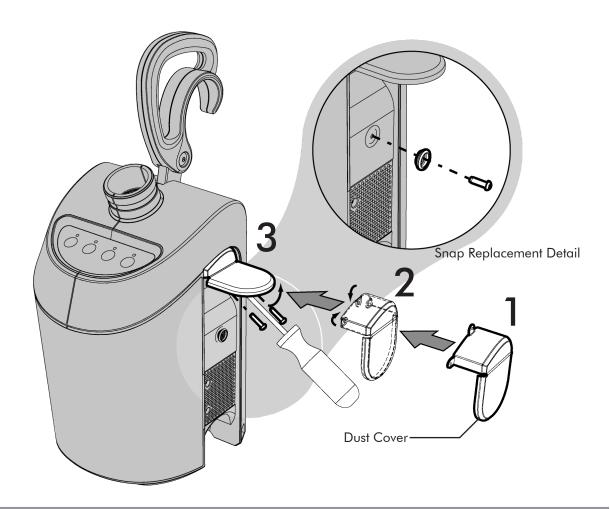






Metal Cover Replacement

- 1. Disconnect hose from unit.
- 2. Remove the two phillips head screws on each side to detach the air filter covers. Remove the air filters.

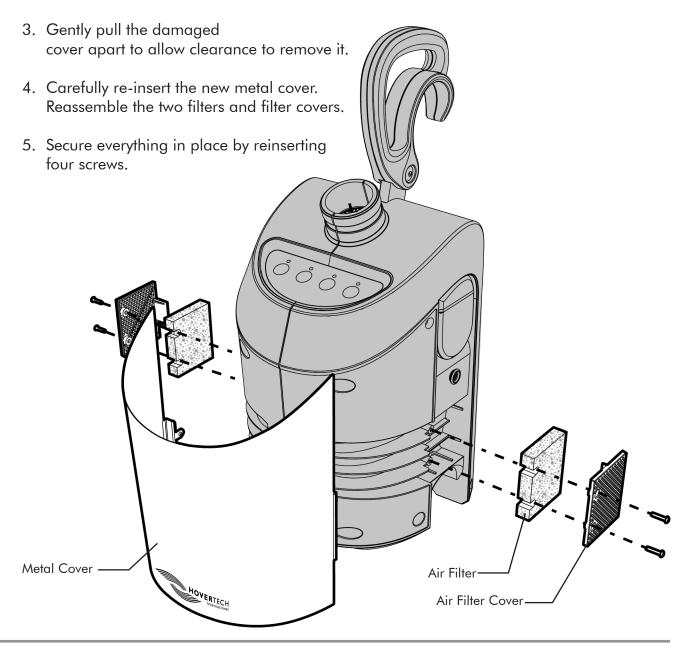






Cord Strap Replacement

- 1. Unsnap the strap and remove power cord.
- 2. Detach the damaged cord strap by removing the screw as shown.
- 3. Reattach strap by positioning in place and securing it with the screw provided.

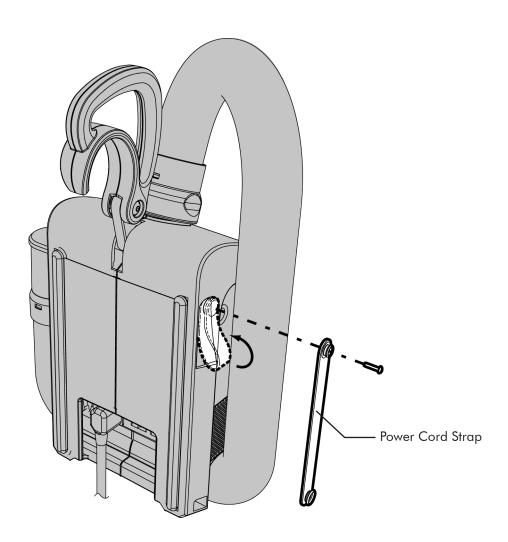



HoverTech Symbols






















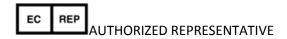

















SPARE PARTS

HoverTech Part Number	Description
HTA-AF	Air Filters (sold in pairs)
HTA-AFC	Air Filter Cover with screws (sold in pairs)
HTA-B/F	Bumpers and Feet (sold as a kit)
HTA-DC	Dust Cover w/ screws (kit)
HTA-H/AH	Handle/Attachment Hook
HTA-HAS	Hose Attachment Snap
HTA-Hose	Hose Assembly
HTA-MC	Metal Cover w/ screws
HTA-PCord	Power Cord
HTA-PCS	Power Cord Strap with screw



HoverTech International 4482 Innovation Way Allentown, PA 18109 www.HoverMatt.com Info@hovermatt.com

These products comply with the standards applicable for Class 1 products in the Medical Device Regulation (EU) 2017/745 on medical devices.



CEpartner4U , ESDOORNLAAN 13, 3951DB MAARN, THE NETHERLANDS.

www.cepartner4u.com

In case an adverse event in relation to the device, incidents should be reported to our authorized representative,
CEPartner4u. CEPartner4u will forward information to the manufacturer.



Etac AB Färögatan 33 164 51 Kista, Sweden www.etac.com