
User manual

Swift Freestanding

etac[®]
Creating Possibilities

1a19129 - 2024-01-02 - Version 2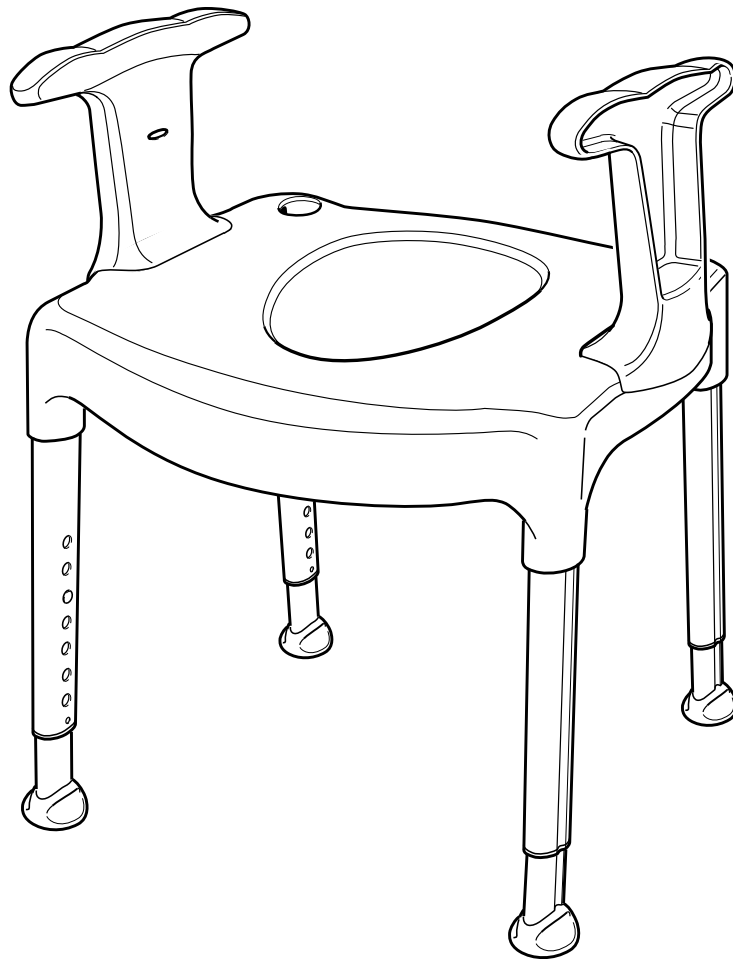


sv en da de it nl no bg cs el es et fi fr hr is lt ms pl pt sk ar


www.etac.com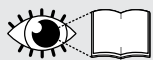

A

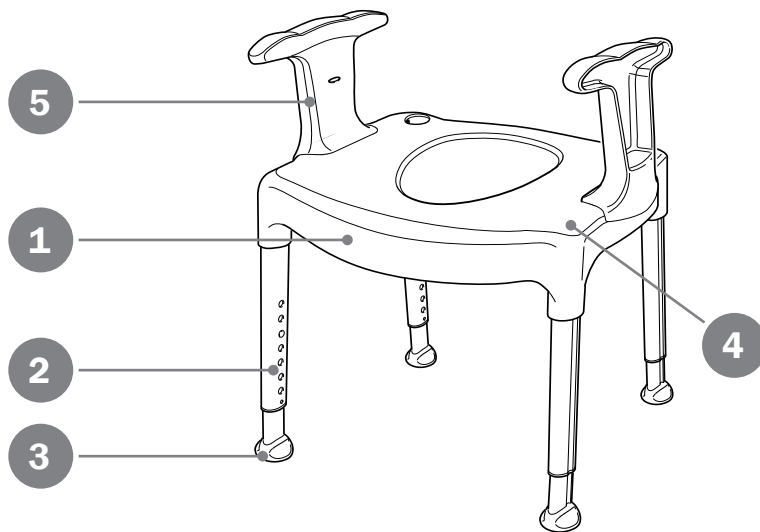B

C

6

UDI *Bar code GS1-128*
 (01)XXXXXXXXXXXXXXXX(11)YYMMDD(21)XXXXX

1	XXXXXXX				
2	XXXXXXX	REF	XXXXXXXXXX	CE	MD
	SN	XXXXXX		Etac Supply Center AB Långgatan 12 SE-334 33 Anderstorp YYYY-MM-DD	
	3	4		5	


D

Etac Swift Freestanding	210 mm 8 1/4"	450 mm 17 3/4"	540 mm 21 1/4"	530-550 mm 20 3/4"-21 3/4"	540 x 410 mm 21 1/4" x 16 1/4"	420-570 mm 16 1/2" - 22 1/2"	4,1 kg 9 lbs	300 x 215 mm 11 3/4" x 8 1/2"	160 kg



PP	Alu	TPE	PE


E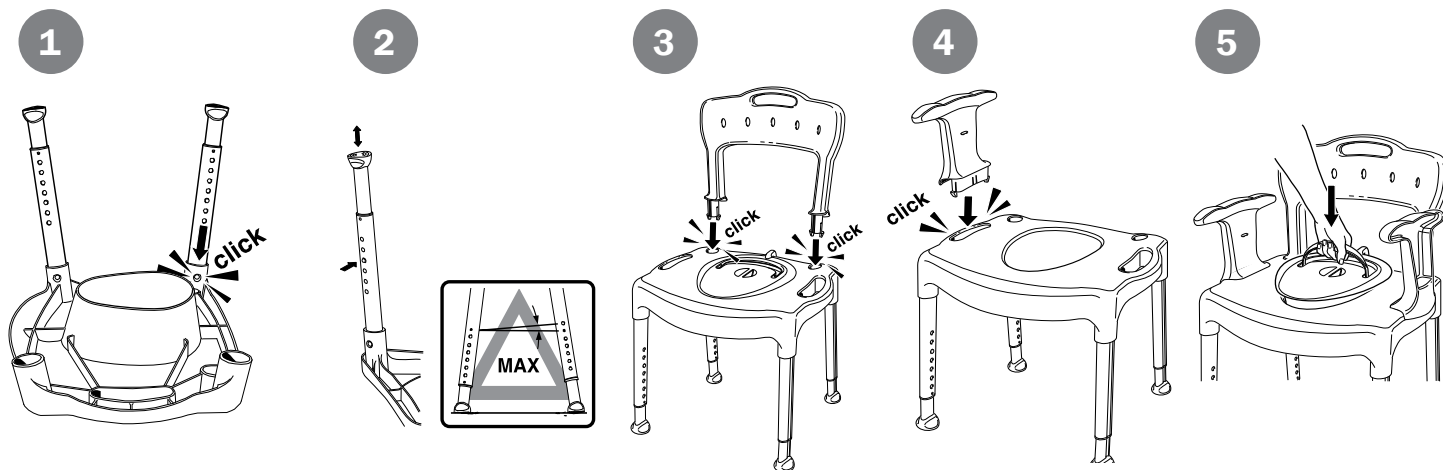

F

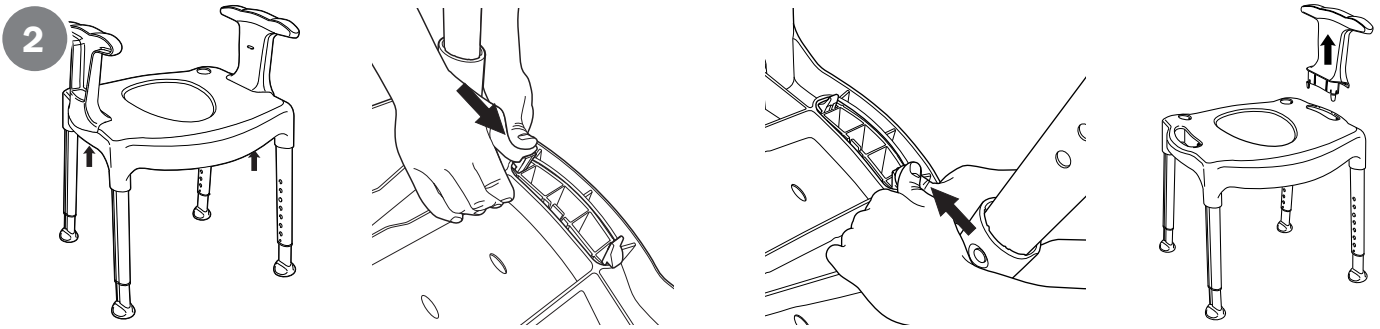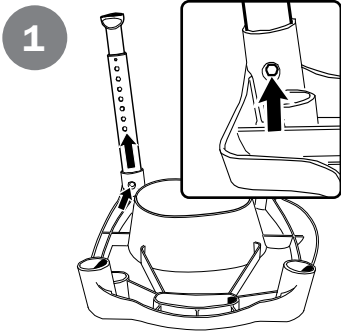G

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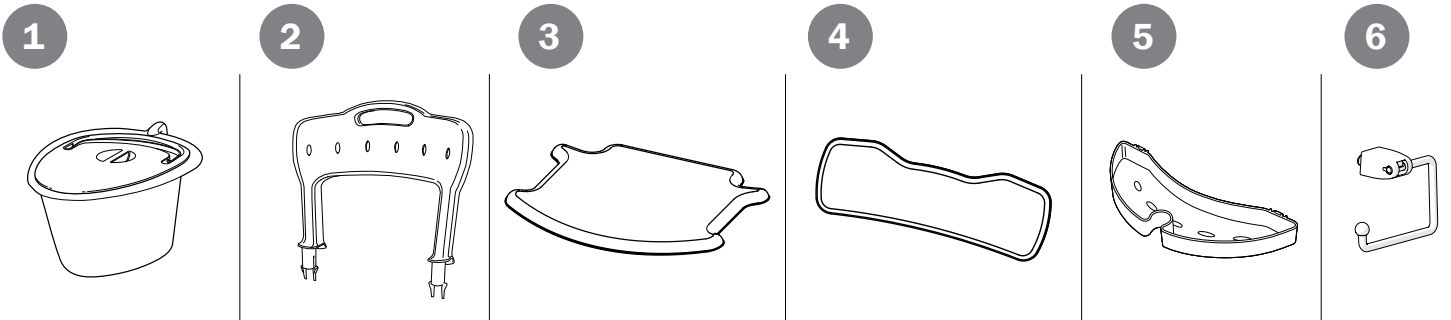Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at www.etac.com. You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

In the manual, the User is the person sitting in the chair. The carer is the person helping the user.

Device description

Swift Freestanding is a raised toilet seat with adjustable telescopic legs and detachable arm supports to be placed over the toilet. It can be used with one, two or without arm supports. If the arm supports are not used, the opening enables a stable grip and support.

Intended purpose

Swift Freestanding (hereafter also referred to as "the device" or "the product"), is a medical device intended for alleviation of or compensation for a functional impairment due to an injury or disability. The device is designed to facilitate access to the toilet for people with restricted functional ability. The raised seats and/or arm supports intend to provide stability and support when sitting down and standing up before and after performing toilet needs.

Intended user group

The user group for the device is based on the individual's functional ability and not on a specific diagnosis, health condition or age. It is intended for individuals with a height of 146 cm or more, or a mass of 40kg or more.

Secondary users of the device are caregivers providing assistance and clinicians/technicians who set up the device.

Intended environment

The device is intended for indoor use at home environments or institutions and is suitable for use in bathrooms, but not swimming pools or similar corrosive environments.

Intended application

The device is intended for short-term and long-term use and can be applied several times per day.

The device is intended to be used in contact with intact skin. The device is intended for refurbishment and reuse.

Expected service life

The expected service life is 10 years. For complete information regarding the service life of the device, see www.etac.com.

Indications

Indication for use is disability, including, but not limited to, an individual with fluctuating balance issues, physiological and/or functional impairment, and/or a history of falls.

Contraindications

There are no known contraindications.

Warnings

Warnings describing a risk element for a specific action or setting of the device

can be found in the relevant section.

 Do not stand on the device

 Ignoring installation or assembly instructions may result in injury.

 A defective device must not be used.

 When moving sideways, do not lean on just one armrest.

Declaration of conformity

The device meets the requirements of the Medical Device Regulation (EU) 2017/745.

The device is tested and meets the requirements of the EN ISO 10993-1, SS EN 12182.

Pre-purchase information

Pre-purchase information can be found at www.etac.com.

Adverse event

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Customization

is everything that goes beyond the manual's instructions and settings. Device specially adapted by the customer must not retain Etac's CE marking. Etac's warranty expires. If there is any uncertainty regarding the execution of the customization, consult Etac. Device coupled with another device may not retain either device's CE mark unless combination agreements exist. Contact Etac for current information.

Device description [Figur A](#)

1. Seat **2.** Leg **3.** Ferrule **4.** Product label **5.** Arm support

Symbols [Figur B](#)

Symbols in the manual and on the device:

1. Warning, precaution or limitation. **2.** Useful advice and tips. **3.** Material for recycling **4.** A height difference of more than one hole is not allowed for chair legs. **5.** When moving sideways, do not lean on just one armrest. **6.** Maximum user weight (see technical data) **7-8.** Cleaning (see maintenance) **9.** Product weight

Explanation item no label: [Figure C](#)

1. Product name **2.** Product description **3.** Serial number **4.** Item number **5.** Date of manufacture **6.** Barcode acc. to GS1-128 GTIN-14 and serial number*

*The device's date of manufacture can be read from the barcode on the device. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

Technical data [Figure D](#)

Installation [Figure E](#)

 A height difference of more than one hole is not allowed for chair legs.

Dismantling [Figure F](#)

1. Leg **2.** Arm support

Maintenance

Clean the device with a solvent-free detergent which has a pH of 5-9, or with a 70% disinfectant solution. Can be decontaminated at max. 85°C if so required.

Constituent materials are corrosion-resistant.

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Storage

The device should be stored indoors in a dry place at a temperature above 5°C. If the device has been stored for a long time (more than four months), its function must be checked by an expert before use.

Accessories [Figure G](#)

1. Pan Swift Commode incl. lid **2.** Back support **3.** Soft seat **4.** Soft back **5.** Soap dish/Lid support/Shower head holder **6.** Toilet paper holder

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Etac Ltd.
Promefa AG