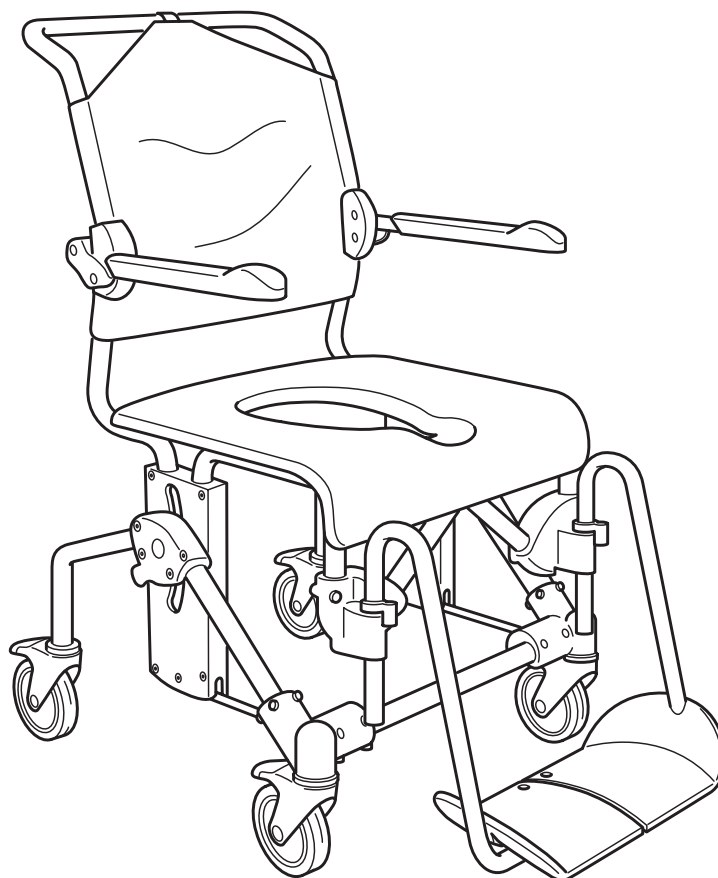
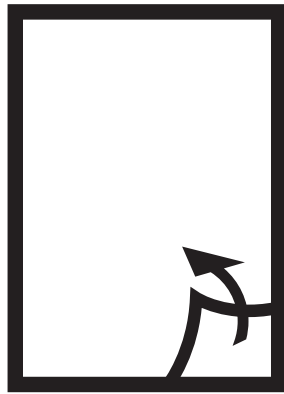
74873D

11-01-13 

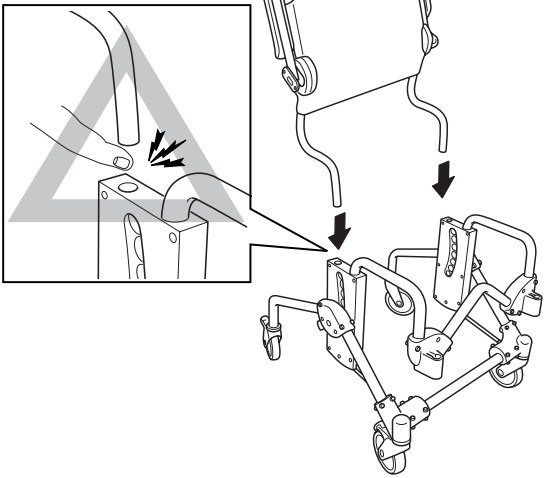
SV Manual
NO Bruksanvisning
DA Betjeningsvejledning
FI Käyttöopas
EN Manual
DE Gebrauchsanweisung
FR Manuel
NL Handleiding
IT Manuale
ES Manual


A


1


2



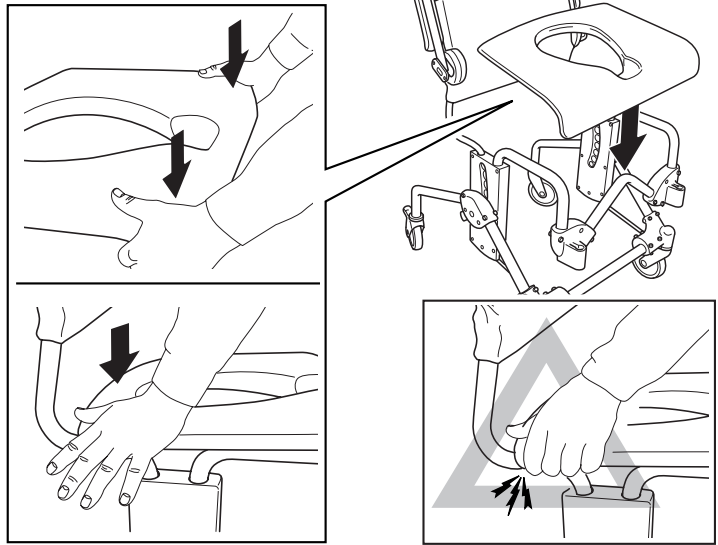
3

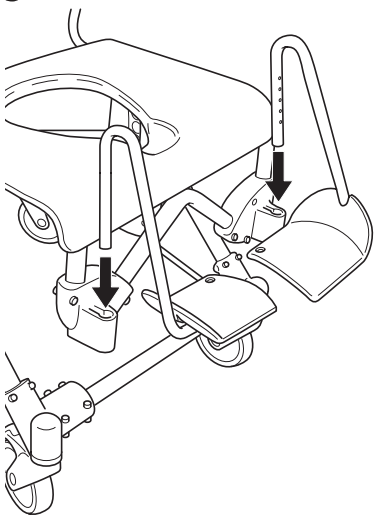
4

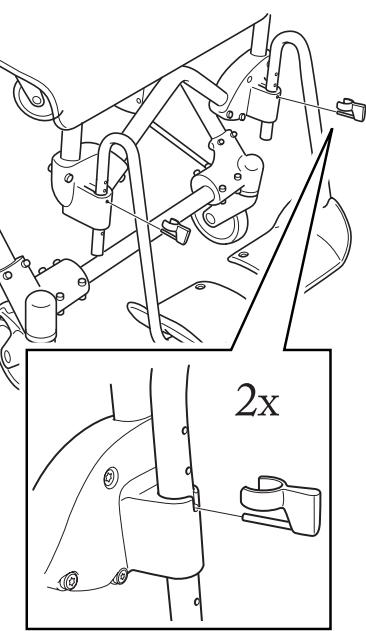5

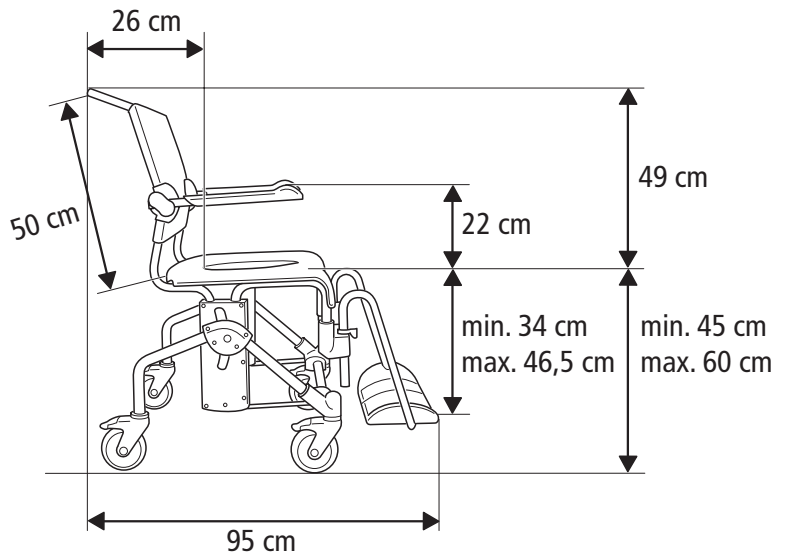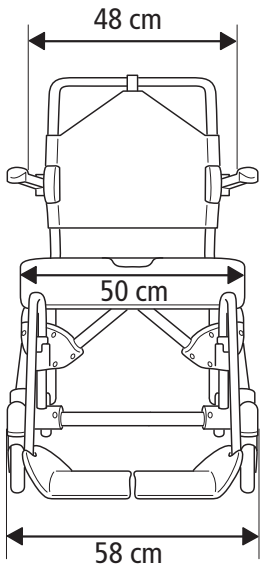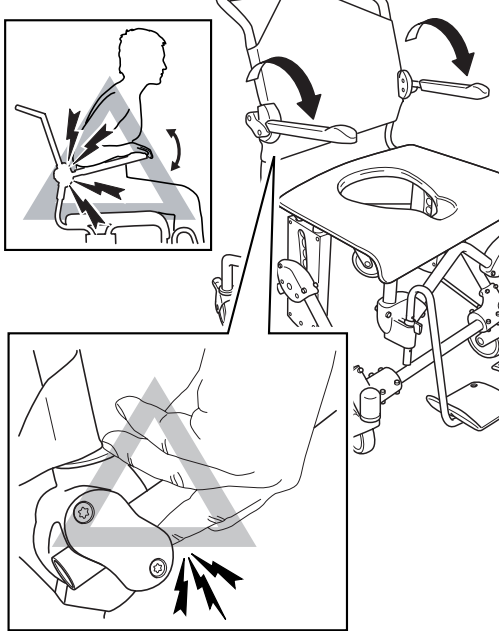
B

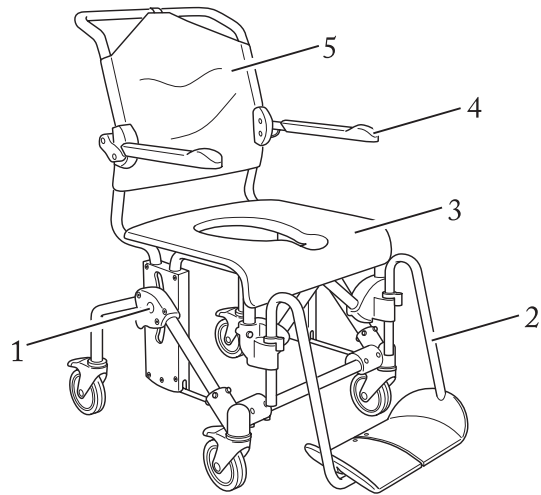
C

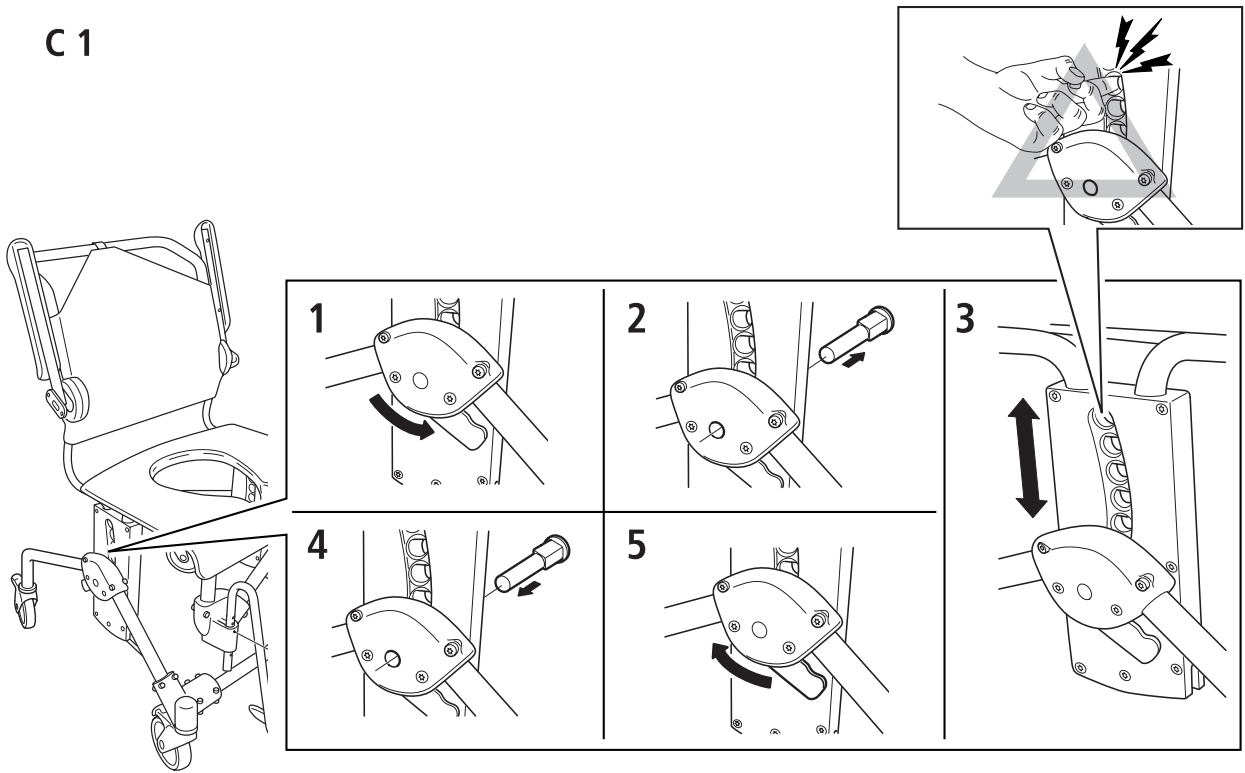
C1

C2

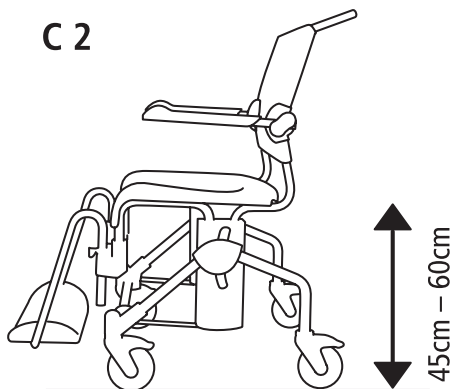
C3

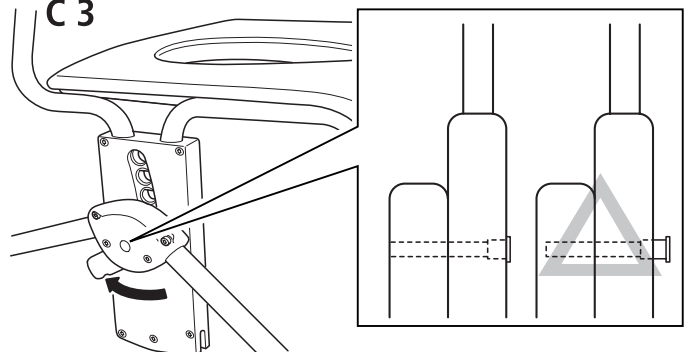
D
























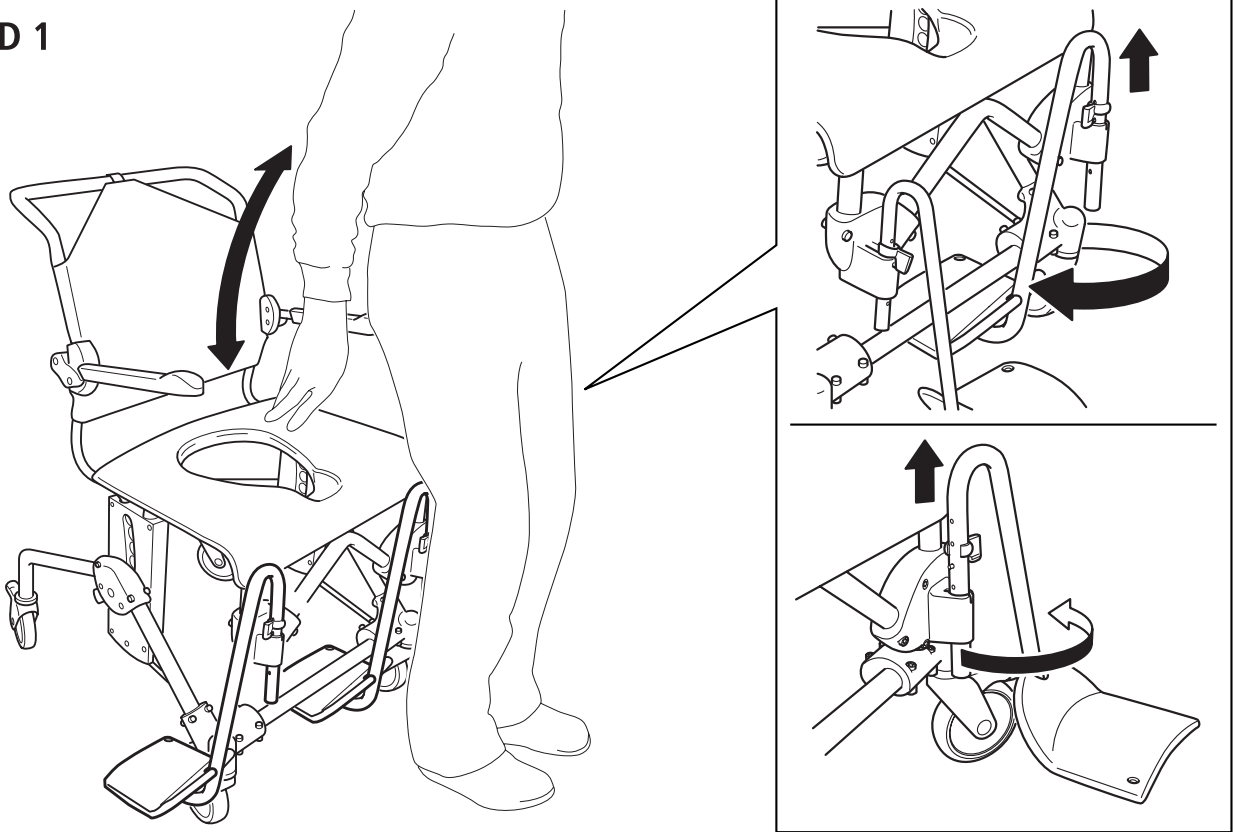
D 1

D 2

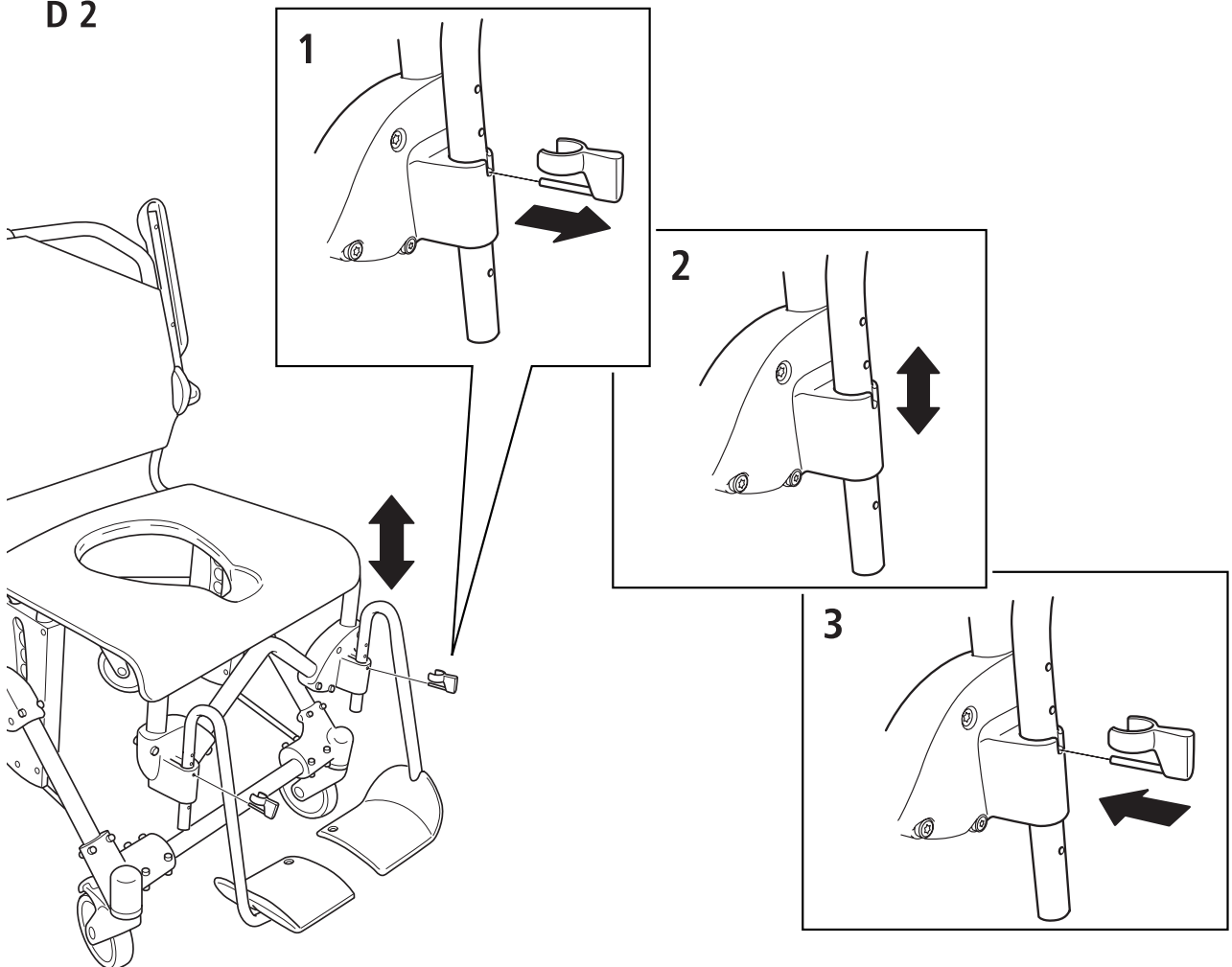E

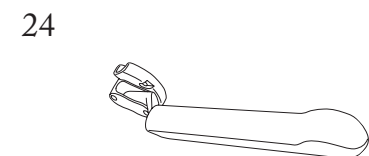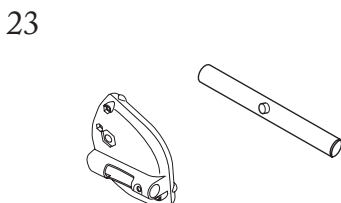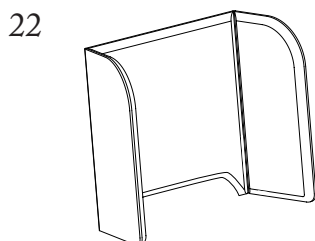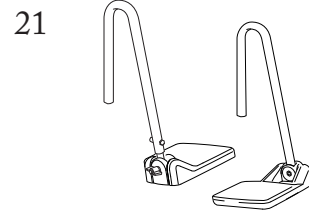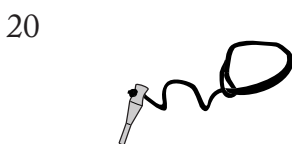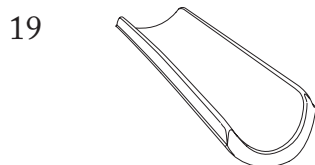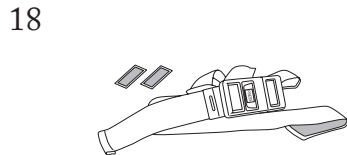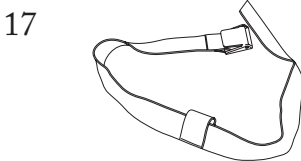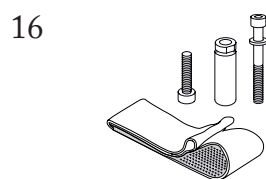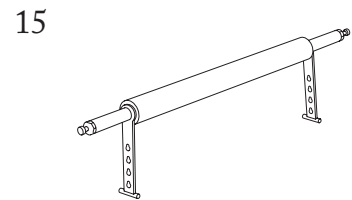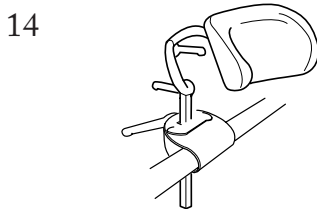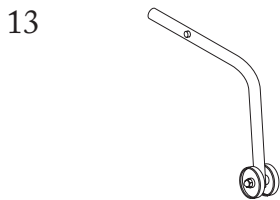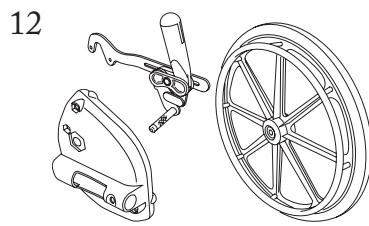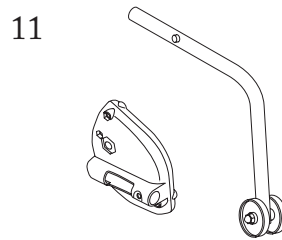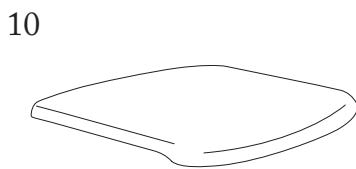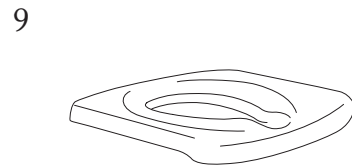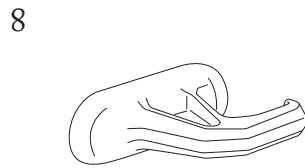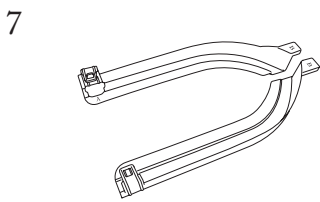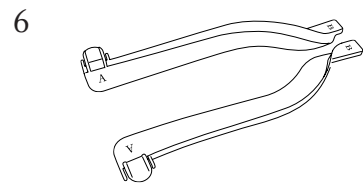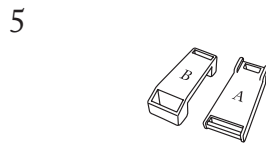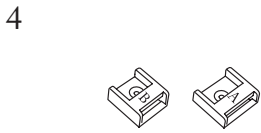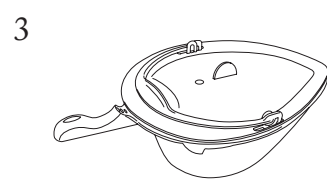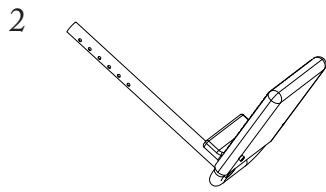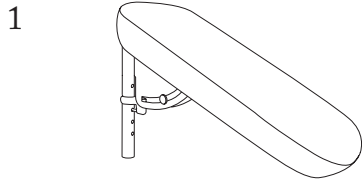SV Innehåll

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General information

Thank you for choosing a shower chair from Etac.

In order to avoid injury during handling and use, please read through this manual first.

In the manual, the user is the person sitting in the chair. The assistant is the person operating the chair.

Swift Mobil is a shower and toilet chair intended to make hygiene care easier for people with reduced mobility and their carers.

Swift Mobil has been tested by a testing institute and complies with the requirements stated in EN ISO 10993-1 and SS EN 12182.

It is designed for use in showers, with hand basins and over toilets, as well as for transport to and from the sanitary room. Every component of the chair is designed to make things easier for users and assistants.

This symbol appears in the manual along with text. It indicates instances where the safety of the user or the assistant may be at risk.

Assembly/Dimension table

fig. A

Product description

fig. B

1 Bolt	4 Armrest
2 Legrest	5 Backrest upholstery
3 Seat	

Height adjustment

fig. C

The height can be adjusted with ease by positioning the bolt in the required position (fig. C 1).

Swift Mobil is height-adjustable between 45 and 60 cm (fig. C 2).

Check that both sides are similarly adjusted, otherwise stability will be reduced. Do not use the chair if the bolt (orange) is not locked (fig. C 3).

Legrest


fig. D

The assistant can easily raise and swivel the legrests to the side for the user to sit down in or get up out of the chair (fig. D 1). The footrests are detachable.

Various height adjustment options are available by moving the locking pin (fig. D 2). The legrests have 6 different height settings.



Transport

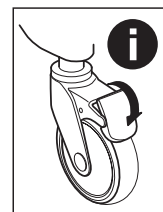1. Make sure that at least two of the wheels are locked whenever the user is sitting down in or getting up out of the chair.
2. The legrests should be swivelled to the side whenever the user is sitting down or getting up.

Negotiating thresholds:

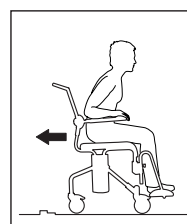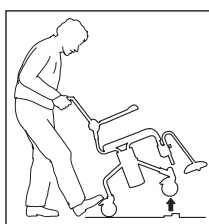1. Chair only: Place one foot on the rear chassis tube.
2. With a user: move the chair backwards. Ensure that the castors are facing in the direction of travel.

Warranty/maintenance

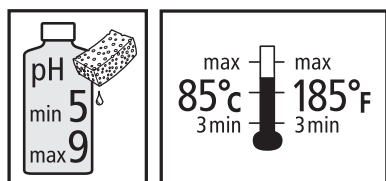
Warranty

The warranty period for material and manufacturing defects is 5 years,

Maintenance and reconditioning

To keep the chair in good and safe working order, you should regularly:

- Clean the chair with ordinary non-abrasive detergent. Rinse and dry.
- The product can be cleaned at 85 °C for 3 minutes.
- Keep the castors free of hair and soap deposits, etc. If your shower and toilet chair is not in perfect working order, contact your dealer.

Safety

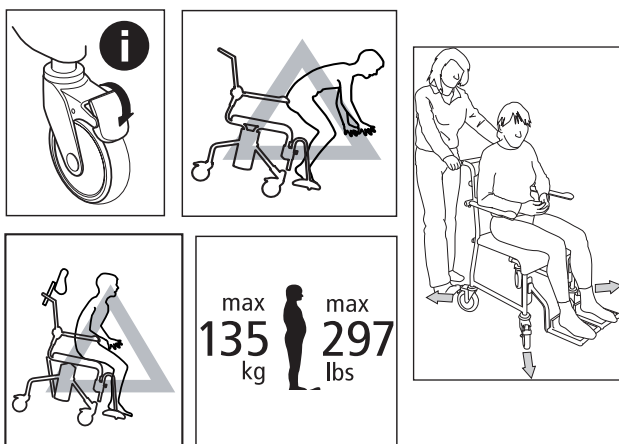
A defective chair should not be used.

Make sure that the wheels are locked whenever the user is sitting down in or getting up out of the chair.

Anti-tip stabilisers should be used if the user is a leg amputee.

Do not stand on the footrests, risk of tipping!
Seating position affects the stability.

Out-turned castors improve the chair's stability.
Swift Mobil can be used by people weighing a maximum of 135 kg.

Recycling

This product should be recycled as per national regulations.

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Etac Sverige AB
Etac AB (export)
Etac A/S