# Nova 500 NG

Mobil stå-løfter

Mobiler Aufrichtlifter

Mobile sit-to-stand lift

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|                                                      |                 |
|------------------------------------------------------|-----------------|
| <b>EN</b>                                            | <b>Contents</b> |
| General information .....                            | 20              |
| Safety instructions .....                            | 20              |
| Approved lifting slings/lifting gear .....           | 20              |
| Description .....                                    | 20              |
| Assembly .....                                       | 21              |
| Emergency stop/Emergency lowering/<br>Charging ..... | 21              |
| Use/Safety/Operation .....                           | 22-23           |
| Maintenance/Service/Warranty .....                   | 23              |
| Troubleshooting chart .....                          | 24              |
| Technical information .....                          | 25              |
| Recycling instructions .....                         | 26              |
| Accessories/Spare parts .....                        | 26              |
| Stockage/Transport .....                             | 26              |
| Notes .....                                          | 27              |

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Prod.nr 16030101

IPX4

Int: 10%, max 2 min/18 min  
Internally powered equipment

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## General information

EN

Thank you for choosing a sit-to-stand lift from Etac.

In order to avoid injury during handling and use, please read through the lift and sling manuals.

In this manual, the user is the person being lifted. The assistant is the person operating the sit-to-stand lift.

**This symbol appears in the manual along with text. It indicates important handling instructions to ensure the safety of the user or the assistant.**

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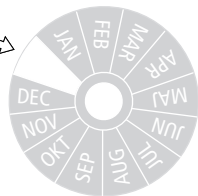
Tested

etac®

The sit-to-stand lift has been tested and complies with the requirements of the Medical Devices Directive for Class 1 products (MDD 93/42/EEC). The sit-to-stand lift complies with the requirements according to EN 60601-1, EN 60601-1-2, ISO 10535.

Etac continuously makes improvements to our products. We therefore reserve the right to make changes to our products without prior notice. The dimensions shown in the drawings or other material are for guidance only.

We cannot be held responsible for any printing errors or omissions.

**CE mark**

**Type B according to EN 60601-1**

**Read instructions before use**

**Recycle according to local regulations**

## Safety instructions

EN

- Etac's sit-to-stand lift may only be used for lifting people.
- The sit-to-stand lift may only be used by personnel who have been instructed in how to use the lift correctly.
- The sit-to-stand lift must not be used as a means of transportation.
- The sit-to-stand lift must not be loaded with more than 180 kg / 395 lbs.

## Approved lifting slings/lifting gear

EN

- Etac has its own range of lifting slings and lifting gear designed for the Etac lifting motor.
- Etac disclaims all responsibility for any defects or accidents that may occur in connection with the use of a lifting harness/gear of other manufacture.

## Description

EN

**Description**

**fig. A**

The sit-to-stand lift is intended for lifting people, with at least one of the assistants having been instructed in use of the lift, or when there is a supervisor present.

The sit-to-stand lift is designed for use in normal care environments.

Never use the sit-to-stand lift to lift anything other than what has been approved by the manufacturer.

The sit-to-stand lift is intended for use for people who have difficulty in moving from a sitting to a standing position but who can do so with assistance, and are able to stand once they have got up.

Once in a standing position they can be moved to a wheelchair or toilet, etc.

- |                                    |                          |                        |                                            |
|------------------------------------|--------------------------|------------------------|--------------------------------------------|
| 1. Lifting arm                     | 6. Electrical box        | 12. Emergency stop     | 18. Depth adjustment of lower leg support  |
| 2. Mast                            | 7. Battery module        | 13. Emergency lowering | 19. Height adjustment of lower leg support |
| 3. Base                            | 8. Hand control          | 14. Handle             | 20. Footrest                               |
| 4. Lift motor                      | 9. Power cord            | 15. Unhooking guard    | 21. Drainage hole                          |
| 5. Motor for base width adjustment | 10. Wheel with brakes    | 16. Lower leg support  |                                            |
|                                    | 11. Wheel without brakes | 17. Calf strap         |                                            |

# Assembly

EN

## Assembly

fig. B

Before assembly, check that the following parts have been supplied:

- **Mast**  
Battery module, motor, electrical box, Hand control
- **Base with motor for base width adjustment**

- **Footrest incl. knee support**
- **Power cord**

A 13 mm (½") spanner and a 5 mm (¼") Allen key are needed for assembly but are not included.

# Emergency stop/Emergency lowering/Charging

EN

## Emergency stop

fig. C

Movement of the sit-to-stand lift stops immediately when the red emergency stop button on the electrical box is pressed.

The lift must not be used if the emergency stop function is not working.

To release the emergency stop button after use, twist it a quarter turn as indicated by the arrows on the button. The motor can then be used again.

The emergency stop function must be checked regularly. Operate the lift without a load and press the emergency stop button – the lift should stop immediately.

**The emergency stop button must not be used as a start/stop button. When the lift is in use, the button must always be in the out position!**

EN

## Emergency lowering

fig. D

If the lift stops working while in use (for example, if the Hand control develops a fault), the user can always be lowered without any risk using the emergency lowering function.

The lift exerts no downward force during emergency lowering other than the weight of the user and the lifting arm. This reduces the risk of crushing injuries.

There are two ways of doing this:

1. On the electrical box there is a Button for electrical emergency lowering (fig. D 1).
2. The sit-to-stand lift has a mechanical emergency lowering function. Turn the top of the actuator and the lifting arm moves downwards when loaded (fig. D 2).

EN

## Battery – Charging

fig. E

The battery must be charged regularly. We recommend charging every night. The emergency stop button must not be depressed during charging.

The battery has no "memory" and its lifetime can be significantly shortened if it is allowed to discharge completely before its next charge. Connect the power cord to a wall socket.

**The sit-to-stand lift must not be used during charging. The battery must not be charged in a damp environment, e.g. a bathroom!**

There are two LEDs (indicator lights) on the electrical box.

- The bottom light is green and indicates handset activation (fig. E).
- The top light is yellow and indicates that charging is in progress. The indicator will blink during charging and change to a constant yellow light when fully charged (fig. E).

If the sit-to-stand lift does not function as expected, even though the battery is fully charged, this may be due to the battery's lifetime having been exceeded. If this is the case, the battery should be replaced.

Etac can supply a new battery.

# Use/Safety/Operation

EN

## Prior to use

The assistant is responsible for performing a visual check of the lift prior to use. If any problems are discovered, the lift must not be used until it has been repaired by a qualified technician. Damaged lifting slings must never be used.

### Before use, always check that:

- The lift has been assembled in accordance with the assembly instructions and that all parts are in place.
- All couplings and connections to the electrical box, frame and lifting arm are intact.
- The actuator and electrical box are not making any abnormal noises.
- The lift shows no signs of damage.
- The battery is fully charged.

### Before lifting, always check that:

- The lifting accessory is appropriate for the user's needs. This applies to type, size, material and configuration.
- The lifting accessories are free from damage.
- The lifting accessories are correctly and securely attached to the user so that no bodily harm can occur.
- The lifting accessory is correctly attached to the lifting equipment.
- The sling loops are correctly fitted to the sling bar hooks when they are extended. This should be checked before the user is lifted.

EN

## Lift

fig.F

- Extend the width of the wheel base prior to lifting using the button for base width adjustment on the Hand control.
- Raise and lower the lift by pressing the buttons gently.

**NEVER USE THE BRAKES when the user is being lifted or lowered.**

**NB! The wheels of the object the user is being lifted from or into must be in the locked position.**

- The assistant holds the Hand control and stands close to the user. This creates a sense of security.
- Work efficiently! For example, move the wheelchair to the sit-to-stand lift, not vice versa. Avoid long transfers during which the user is suspended in the sling.

**The sit-to-stand lift is not a means of transportation!**

EN

## Operation of the sit-to-stand lift

- We recommend that the sit-to-stand lift be used only for short transfers on level surfaces.
- The sit-to-stand lift should be pulled when moving. This makes movement as smooth as possible for the user, while at the same time, the assistant is subjected to the least possible strain.
- When the lift is to be turned, the assistant stands alongside the lift and allows it to turn "around its own axis". The assistant holds onto one of the lift handles with one hand and, with the other hand, gently pushes or pulls the lifting arm (closest to the unhooking guard).
- When moving the lift, it is recommended that the assistant uses his or her entire body rather than twisting the upper body or shoulders. The assistant must always ensure that lift movement is stable and that there is no risk of the lift tipping over.
- Push or pull the lift smoothly and carefully. Rapid movements are not only unpleasant for the user but in the event of a sudden stop could also constitute a safety hazard.
- Never attempt to lift the sit-to-stand lift. This can result in a risk of injury to both assistant and user.
- The brakes are primarily used for parking.
- The sit-to-stand lift can also be used for lifting from a sitting or a lying position. For these lifting situations, a sling that is normally used with patient lifts can be used. Please note that in such cases, the footrest and lower leg support must be removed and the lift must not have its brakes on.

## Use/Safety/Operation

EN

### Parking the sit-to-stand lift

fig. G

- Reset the wheel base to the parallel position using the button for base width adjustment on the hand control.
- Apply the brakes.

## Maintenance/Service/Warranty

EN

### Cleaning

The lift arm motor and the electrical box should only be cleaned using a lightly dampened cloth, due to the risk of moisture damaging the electronic components. The metal parts and handles can be cleaned using detergent or disinfectant.

The lift must not be exposed to running water!

**After each cleaning session, the emergency stop and emergency lowering functions must be checked!**

EN

### Annual service/inspection

If something on your lift is not working, please contact a qualified technician.

A defective lift must not be used.

For reconditioning or replacement of parts, original Etac parts or components of an equivalent quality should be used. These are specified in a spare part list.

If other parts are used, Etac bears no responsibility for function or safety.

- Etac stipulates that the lift must be inspected once a year, in accordance with Etac's service instructions for mobile lifts. This inspection must be performed by qualified personnel. An annual inspection of this kind is also stipulated in the standard ISO 10535.
- The owner is responsible for ensuring that an inspection is performed. Etac recommends that the owner take out a service agreement.

- If a service agreement has been taken out, the technician is responsible for ensuring that the inspection is performed correctly, and that an inspection report is created for the inspected lift.
- If the inspecting technician finds anything seriously wrong with the lift, he is obliged to ensure that the lift is taken out of service if the fault represents a safety risk.
- Service, repairs and other additional work on the lift between inspections must be noted in the inspection report.

EN

### Warranty - Service life

- Six-year warranty against manufacturing or material defects, battery one year. For complete warranty information, please refer to "Warranty Etac Lift Products" at [www.etac.com](http://www.etac.com)

Expected service life at normal use: approx. 10 years.

# Troubleshooting chart

EN

| Symptom                                                                                                                                                    | Possible cause                                              | Troubleshooting                                       | Corrective action                                                                       |
|------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------|-------------------------------------------------------|-----------------------------------------------------------------------------------------|
| The lift does not function.                                                                                                                                | No power                                                    | Emergency stop button depressed                       | Turn the emergency stop a quarter turn so that it springs out (fig. E).                 |
|                                                                                                                                                            |                                                             | Battery discharged or defective                       | Charge or replace battery                                                               |
|                                                                                                                                                            | Contact fault                                               | Check all contacts (motor, Hand control)              | Insert the contacts properly (fig. B).                                                  |
|                                                                                                                                                            |                                                             | Power cord connected to power outlet                  | Remove the power cord from the power outlet.                                            |
| Intermittent fault:<br>The lift only goes up or down (not in both directions) or the base width can only be adjusted in one direction (together or apart). | Fault in electrical box                                     | Hand control fault                                    | Replace the Hand control.                                                               |
|                                                                                                                                                            |                                                             | Fault in electrical box                               | Replace the electrical box.                                                             |
|                                                                                                                                                            | Inadequate power supply or fault in electrical box          | Battery discharged or defective                       | Charge or replace the battery                                                           |
|                                                                                                                                                            |                                                             | Loose connection in Hand control                      | Replace the Hand control.                                                               |
| Base width adjustment does not function.                                                                                                                   | Contact fault                                               | Check the contacts between the base and the mast      | Push the contacts together (fig. B).                                                    |
| Lift or base width movement is slow.<br>The light on the Hand control is on.                                                                               | Inadequate power supply                                     | Battery discharged or defective                       | Charge or replace the battery                                                           |
|                                                                                                                                                            |                                                             | Power cord or electrical box is defective             | Replace the power cord or electrical box.                                               |
| The lift is making an abnormal noise.                                                                                                                      | Motor fault or resistance at the motor's attachment joints. | The motor cannot move freely in its attachment joints | Loosen and check/adjust the tightness of the motor's attachment joints or contact Etac. |

# Technical information

## Technical information

### Etac sit-to-stand lift

|                                          |                                          |
|------------------------------------------|------------------------------------------|
| Dimensions .....                         | fig. H and J (*with 75 mm wheel, -30 mm) |
| Max load:.....                           | 180 kg (395 lbs)                         |
| Total lifts with 80 kg (175 lbs) .....   | about 80 (battery capacity)              |
| Sound volume : .....                     | 53 dBA                                   |
| Applied force hand control buttons ..... | max 5 N                                  |

### Base

|                                       |                       |
|---------------------------------------|-----------------------|
| Turning radius .....                  | approx. 110 cm (43½") |
| Total weight.....                     | 46 kg (101 lbs)       |
| Weight, base .....                    | 13 kg (28½ lbs)       |
| Weight, complete mast .....           | 23 kg (51 lbs)        |
| Footrest incl. lower leg support..... | 10 kg (22 lbs)        |

### Material

|               |                     |
|---------------|---------------------|
| Chassis:..... | Powder-coated steel |
|---------------|---------------------|

### Electrical data

|                             |                        |
|-----------------------------|------------------------|
| Intermittent operation..... | 10 %, max 2 min/18 min |
| Operating voltage .....     | 24V DC                 |
| Power supply.....           | Battery power          |
| Power consumption .....     | 10 A                   |

### Safety class

|                     |      |
|---------------------|------|
| Motor .....         | IPX4 |
| Electrical box..... | IPX4 |
| Hand control .....  | IPX4 |
| Battery.....        | IPX4 |

## Recycling instructions

EN

Recycling instructions

fig. I

This product may contain substances that can harm the environment if not recycled according to national directives.

1. Lead acid battery
2. Waste from electrical/electronic equipment
3. Mixed metals

## Accessories/Spare parts

EN

For accessories and spare parts, please visit [www.etac.com](http://www.etac.com)

## Storage/Transport

EN

For long time storage it is recommended that the emergency stop button is activated. The lifter can be stored and transported under the following conditions:

Pressure: 70 - 106 kPa

Humidity: 15 - 93 %

Temperature: -25 - 70 °C

The lifter is designed for use at standard room temperatures (+5 to +40°C). Following storage or transport at other temperatures leave the lifter in a room with a suitable temperature until it reaches a safe operating temperature

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Etac Sverige AB
Etac AB (export)
Etac A/S