

User manual


Flex

etac[®]
Creating Possibilities

75338 - 2023-10-23 - Version 6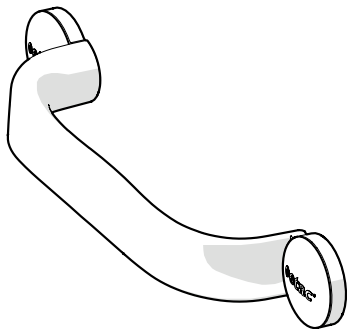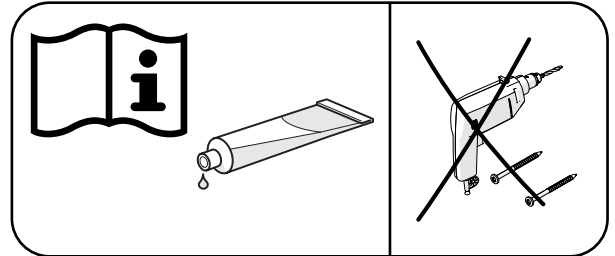

30 cm

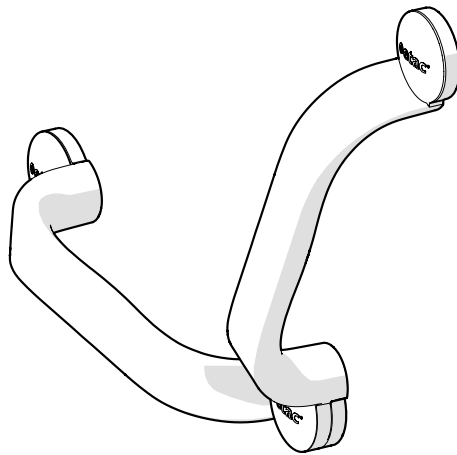60 cm

sv en da de it nl no bg cs el es et fi fr hr is lt ms pl pt sk ar

www.etac.com

MD CE




A


1

30 cm


60 cm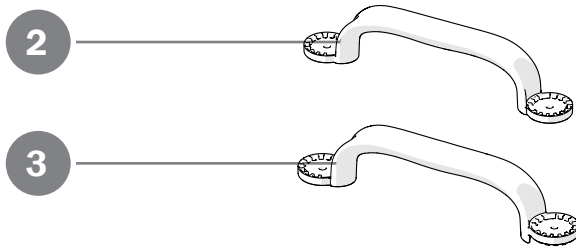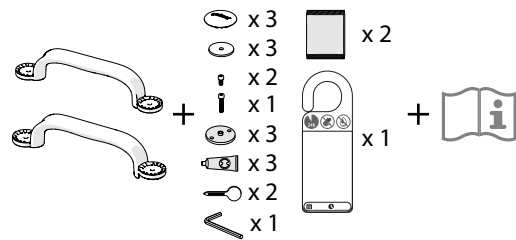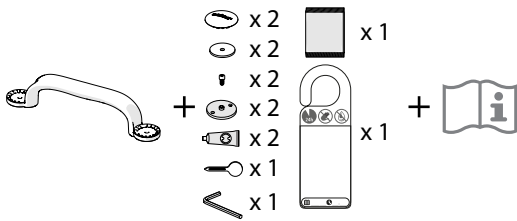


B

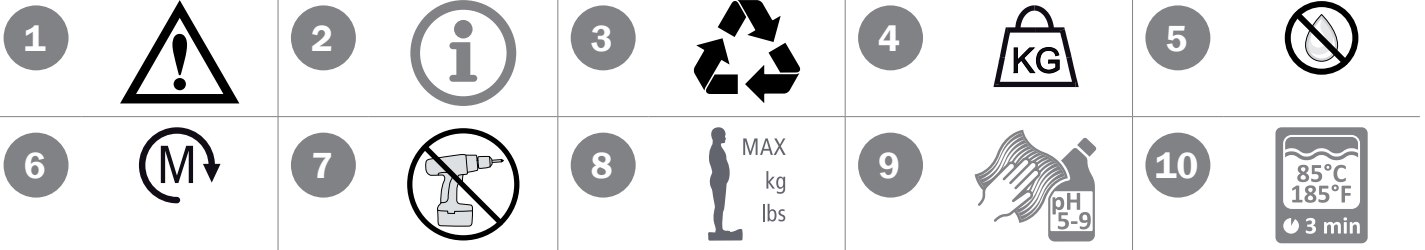C

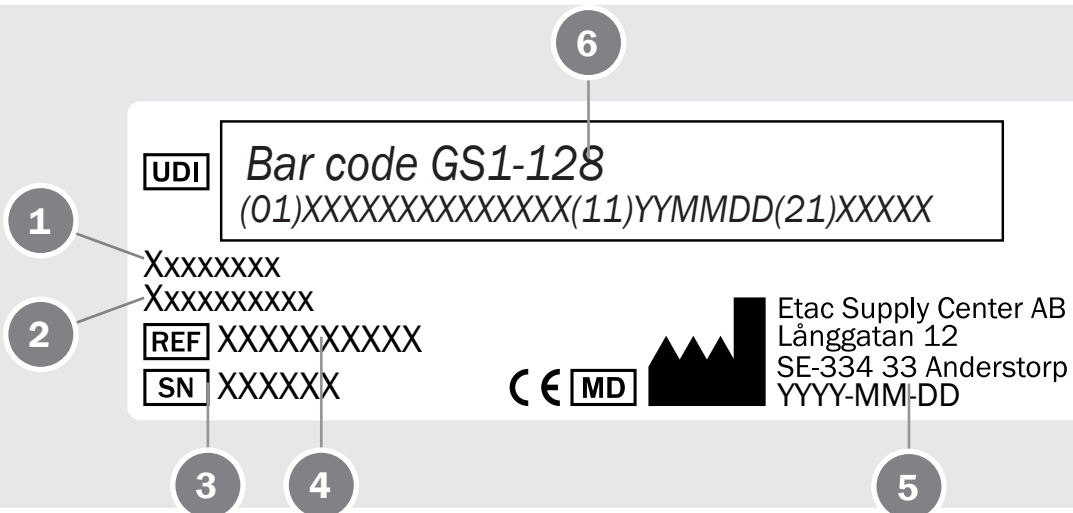D

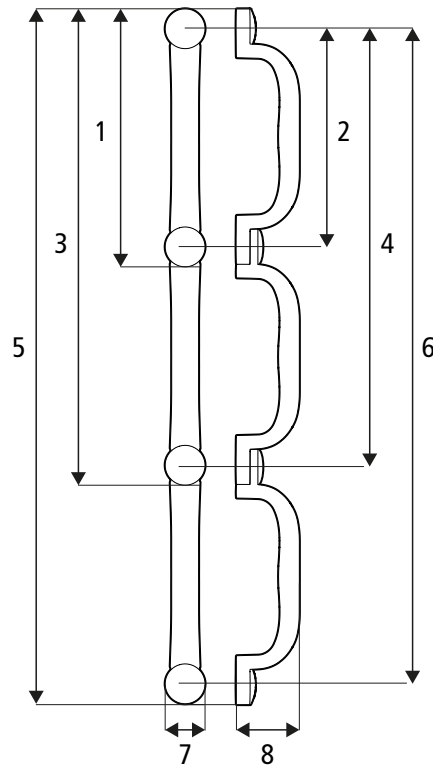B1: 355 mm (14")	B2: 300 mm (11¾")
B3: 655 mm (25¾")	B4: 600 mm (23½")
B5: 955 mm (37½")	B6: 900 mm (35½")
B7: 55 mm (2")	B8: 83 mm (3¼")

max
kg
100 kg
(220 lbs)





>PP GF<


0,35 kg
(0,77 lbs)

E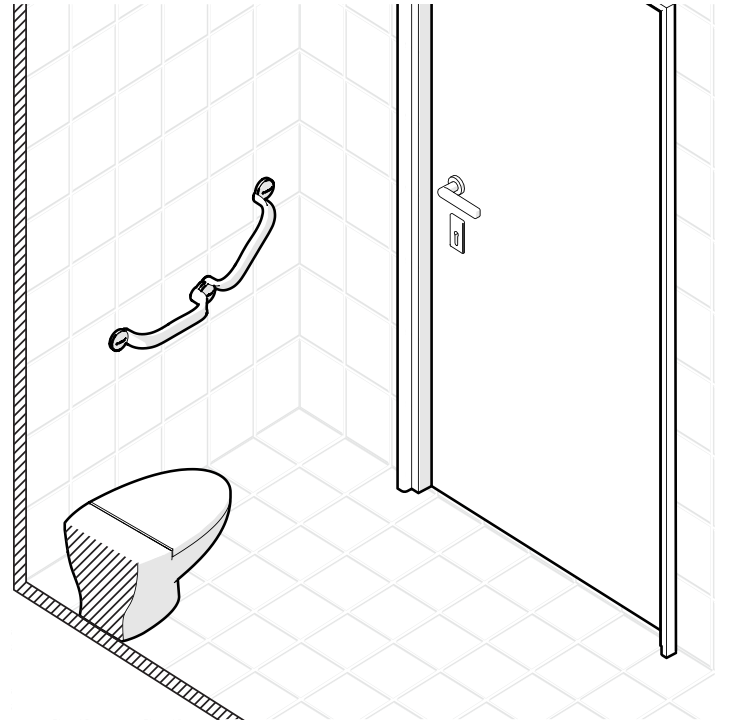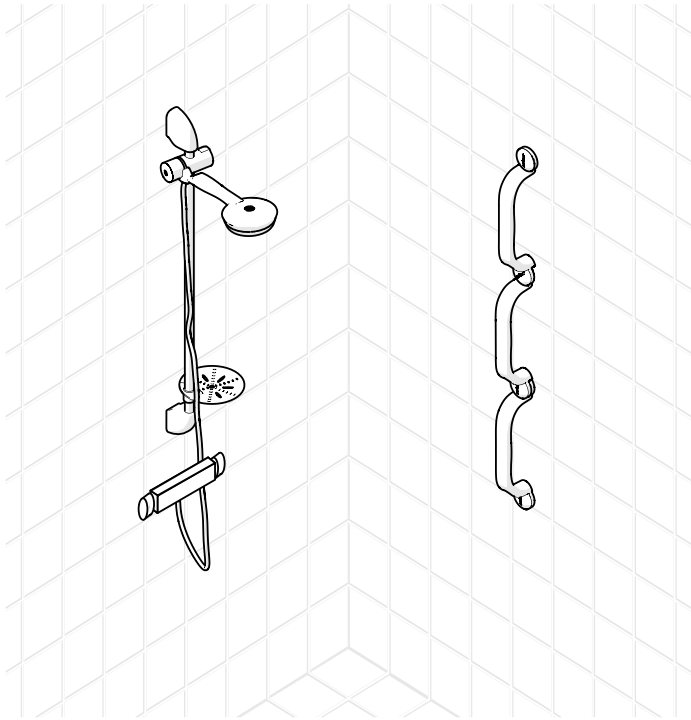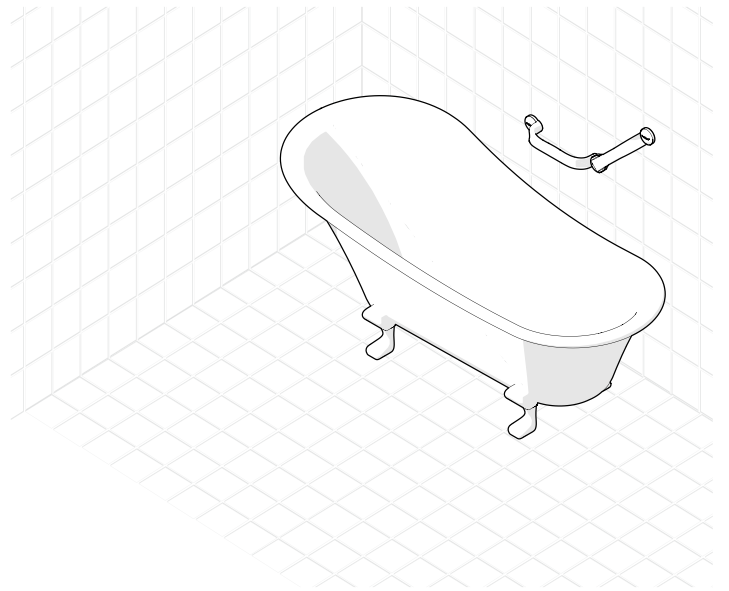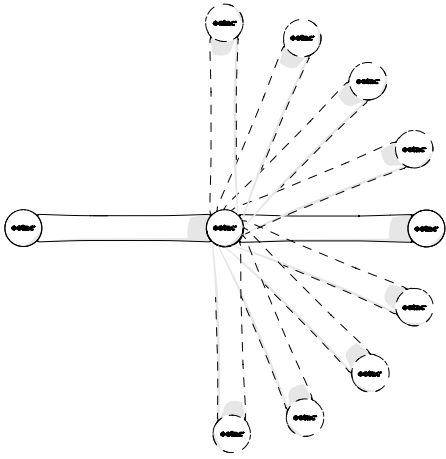

F


F 1

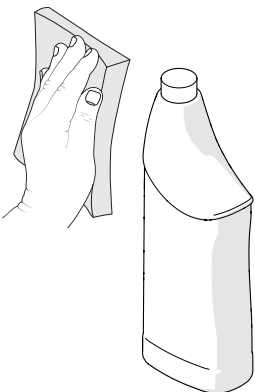F 2

F 3

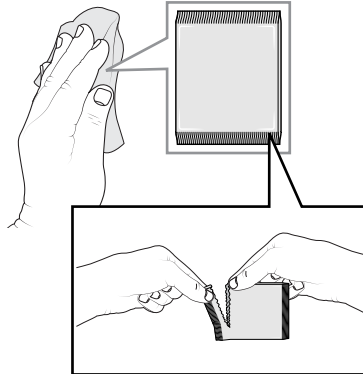F 4

F 5

15 min


G


30 cm


G 1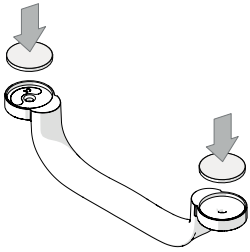


G 2

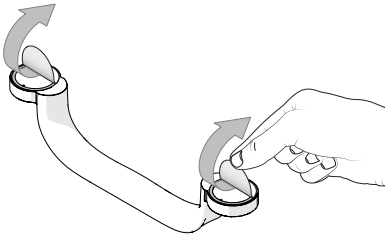
G 3



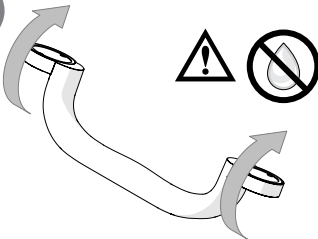
G 4

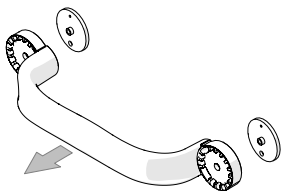
G 5


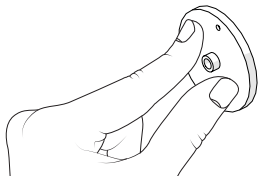
G 6


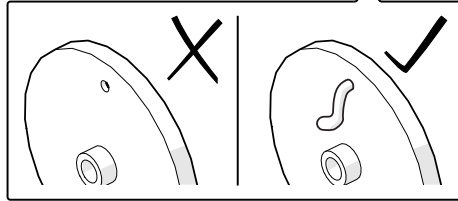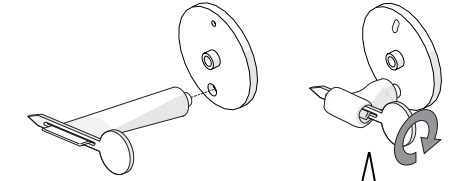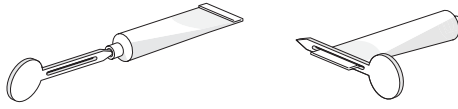
G 7

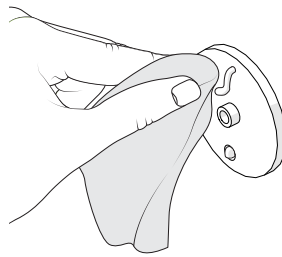G 8

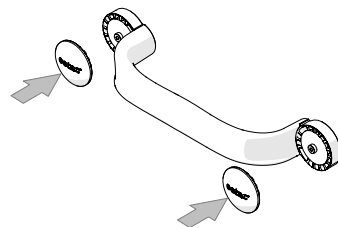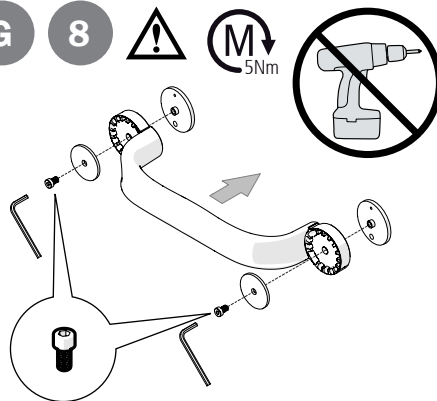
G 9

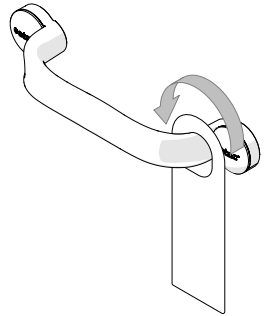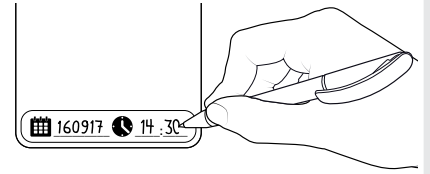
H

60/90 cm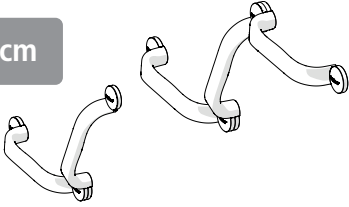


G 1 - 3


H 1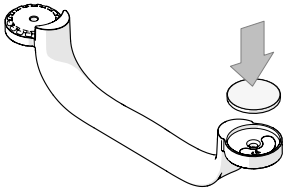


H 2

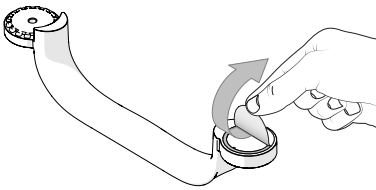
H 3

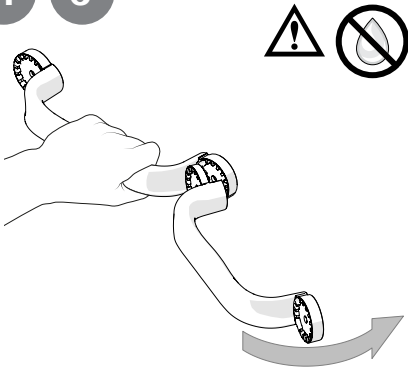
H 4

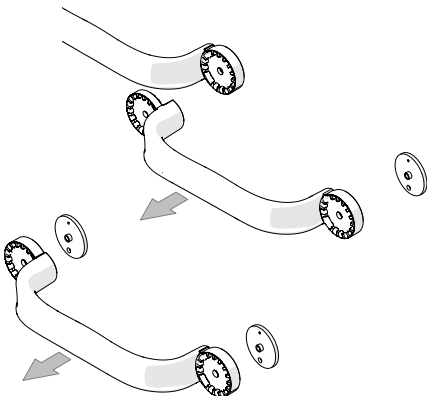
H 5


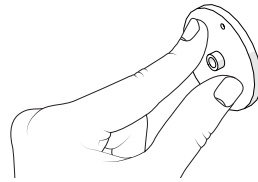
H 6


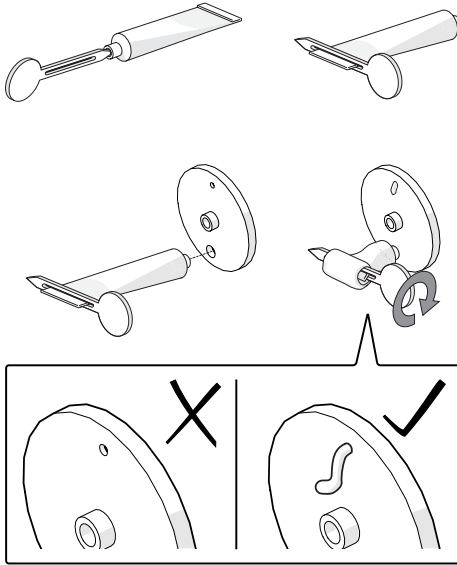
H 7


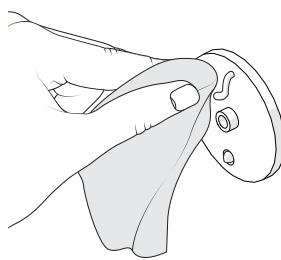
H 8

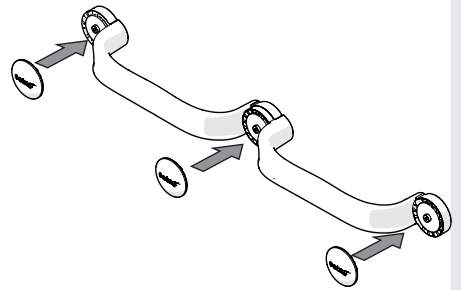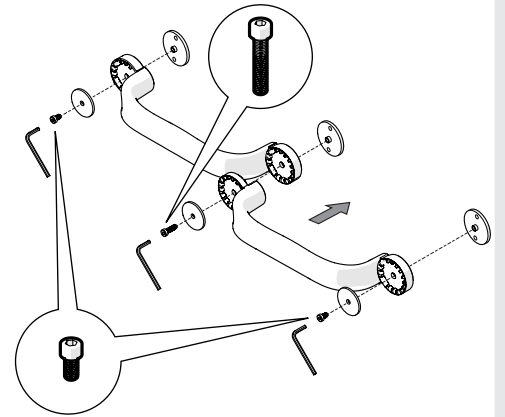H 9

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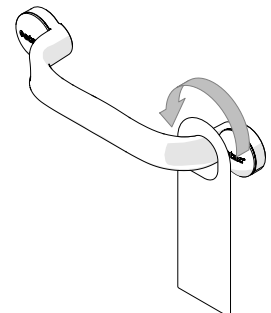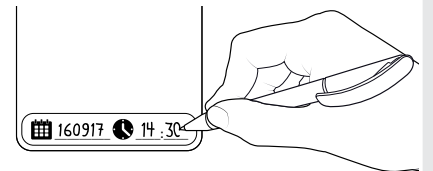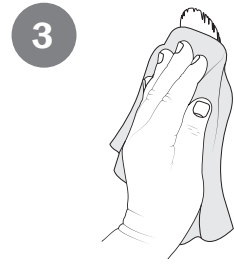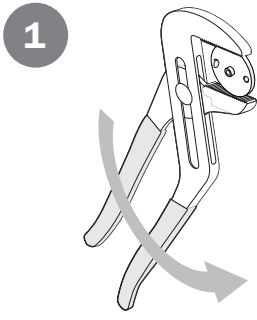English**General information**

Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at www.etac.com. You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

In the manual, the user is the person sitting in the chair. The carer is the person helping the user.

Device description

Flex is a wall-mounted grab rail with a stable and secure grip for the hand. The design makes it possible to add on as many sections as needed and in angles/positions that suit each individual need.

Intended purpose

Flex (hereafter also referred to as "the device" or "the product"), is an assistive device intended for alleviation of or compensation for a physical impairment due to an injury or disability. The device is designed to give a stable and secure grip when standing or making a transfer when using the shower, toilet or bath.

Intended user group

The user group for the device is based on the individual's functional ability and not on a specific diagnosis, health condition or age. It is intended for individuals with a height of 146 cm or more, or a mass of 40 kg or more. Secondary users of the device are caregivers providing assistance and clinicians/technicians who set up the device.

Intended environment

The device is intended for indoor use at home environments or institutions and is suitable for use in bathrooms, but not swimming pools or similar corrosive environments.

Intended application

The device is intended for short-term and long-term use and can be applied several times per day. The device is intended to be used in contact with intact skin.

The device is intended for refurbishment and reuse.

Expected service life

The expected service life is 10 years. For complete information regarding the service life of the device, see www.etac.com.

Indications

Indication for use is disability, including, but not limited to, an individual with fluctuating balance issues, physiological and/or functional impairment, and/or a history of falls.

Contraindications

There are no known contraindications.

Warnings

Warnings describing a risk element for a specific action or setting of the device can be found in the relevant section.

A defective or incorrectly mounted device should not be used. In the event of a fault, contact your prescriber or retailer.

Heated device can cause burns.

Ignoring installation or assembly instructions may result in injury.

Declaration of conformity

The device meets the requirements of the Medical Device Regulation (EU) 2017/745.

The device is tested and meets the requirements of the EN ISO 10993-1, SS EN 12182.

Pre-purchase information

Pre-purchase information can be found at www.etac.com.

Adverse event

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Customization

is everything that goes beyond the manual's instructions and settings. Device specially adapted by the customer must not retain Etac's CE marking. Etac's warranty expires.

If there is any uncertainty regarding the execution of the customization, consult Etac. Device coupled with another device may not retain either device's CE mark unless combination agreements exist. Contact Etac for current information.

Device description, content

of the delivery..... [Figure A](#)

- 1.** Content of the delivery
- 2.** Handle
- 3.** Extension

Symbols..... [Figure B](#)

Symbols in the manual and on the device:

- 1.** Warning, precaution or limitation.
- 2.** Useful advice and tips.
- 3.** Material for recycling.
- 4.** Product weight
- 5.** Surface must be dry
- 6.** Maximum tightening torque
- 7.** Electric screwdriver must not be used
- 8.** Maximum user weight (see technical data)
- 9-10.** Cleaning (see maintenance)

Explanation item no label:..... [Figure C](#)

- 1.** Product name
- 2.** Product description
- 3.** Serial number
- 4.** Item number
- 5.** Date of manufacture
- 6.** Barcode acc. to GS1-128 GTIN-14 and serial number*

*The device's date of manufacture can be read from the barcode on the device. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

Technical data..... [Figure D](#)

Positioning, installation..... [Figure E](#)

Etac Flex can be mounted in different positions. The additional handle can be mounted at angles of 22.5° between 0 and 90°. Etac Flex can be extended by an unlimited number of additional handles.

Mounting options

Etac Flex has been designed to withstand a maximum user weight of 100 kg (220 lbs) and should only be mounted onto a wall with sufficient load-bearing capacity.

Glue: The glue kit can be used to mount the handle on tiles, glass, stone or metal walls.

Inspect the surface to be sure it is completely flat at all fastening points. The bar should not rock or have gaps between the base and the wall.

If in doubt about the wall's load-bearing capacity or the suitability of mounting with glue, an assessment should be carried out by a qualified person.

Mounting on a wall with insufficient load-bearing capacity poses a danger to the user!
Do not glue the device onto concrete, plaster, wallpaper, veneer or any other covering.

Tubes with hardened glue should not be used.

Cleaning prior to mounting..... [Figure F](#)

F1-F5. Cleaning: Ensure that limescale deposits and other dirt is removed by cleaning the mounting surfaces. The following cleaning method is recommended:

- F1.** Use a sanding sponge with sandpaper surface (80). Clean the mounting surface using the sponge and a detergent (PH 5-9) suitable for the surface. Ensure that the surface is completely free of limestone residues and dirt. Note! Start by testing the sanding sponge with detergent on a less visible area to ensure that the surface is not scratched.
- F2.** Wipe off and dry the surface.
- F3.** Wash the surface with the wet wipe supplied to remove any residues from the detergent (the alcohol cleaning cloth should only be used for its intended purpose. It can only work adequately if opened directly before use).
- F4.** Wipe the surface with a clean, lint-free cloth.
- F5.** Wait until the surface is completely dry and has had time to adapt to room temperature. If you are not sure, wait for 15 minutes.

The surface must be dry

Other cleaning methods may be required to ensure that the mounting surface is clean.

Mounting

Flex 30..... [Figure G](#)

Flex 60..... [Figure H](#)

The glue's adhesive effect cannot be guaranteed if the mounting surface has not been fully cleaned.

Etac Flex must be mounted on a dry surface.

- G/H1.** Place the glue plate(s) in the handle with the protective tape outwards.
- G/H2.** Remove the protective tape.
- G/H3.** Press the glue plate(s) into place against the wall where the handle is to be positioned.

 The surface must be dry

G/H4. Carefully remove the handle(s).

G/H5. Press the glue plates into place to ensure that the tape adheres securely to the wall.

G/H6. Insert the tube of glue into the large hole and squeeze glue in until you see glue coming out of the smaller hole.

Do not press on the glue plate after it has been filled with glue.

G/H7. Wipe off any excess glue.

Do not use any other adhesive than provided with the device.

G/H8. Screw the handle(s) into place. Use the longer screws for the common fixing point(s) (60 cm).

 Do not use a screwdriver! Use the Allen key supplied.

G/H9. Indicate date and time when the device is ready for use on the accompanying installation tag and hang it on the device.

 Do not expose the device to any loads or moisture for a period of 24 hours after mounting. This gives the glue time to harden.

Removing glue [Figure I](#)

11. Twist the glue plate off the wall using adjustable pliers.

12. Scrape off any glue residues with a glass scraper.

13. Glue residues can be washed off using silicone remover.

Maintenance

Clean the device with a solvent-free detergent which has a pH of 5-9, or with a 70% disinfectant solution. Can be decontaminated at max. 85 °C if so required.

Constituent materials are corrosion-resistant. 5-year guarantee against material and manufacturing defects. For terms and conditions, see [www.etac.com](#).

Storage

The device should be stored indoors in a dry place at a temperature above 5 °C. If the device has been stored for a long time (more than four months), its function must be checked by an expert before use.

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Etac Ltd.
Promefa AG
Etac Supply Center AB