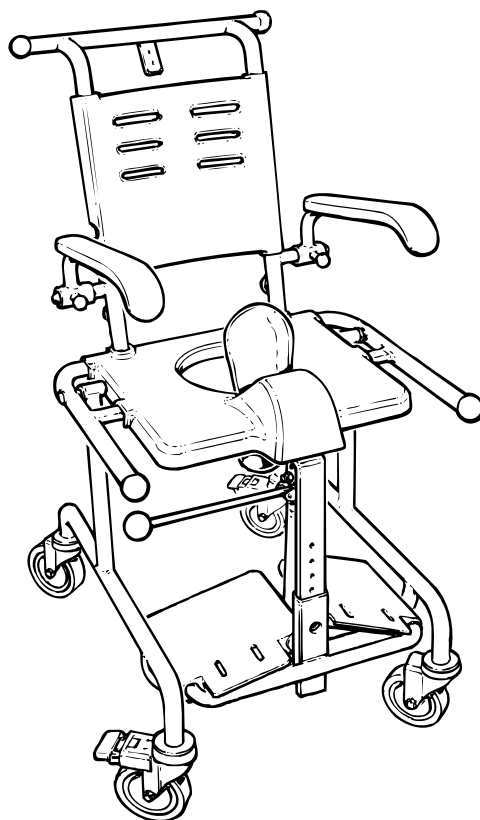
# Etac Tripp

Manual

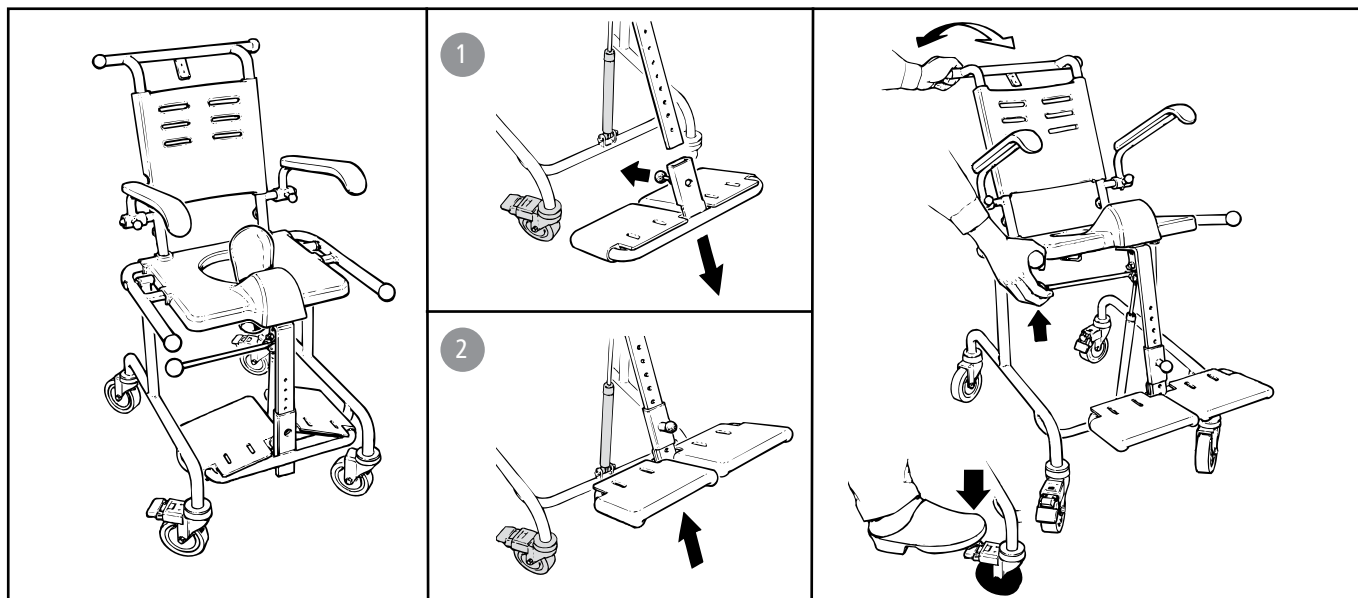
etac®

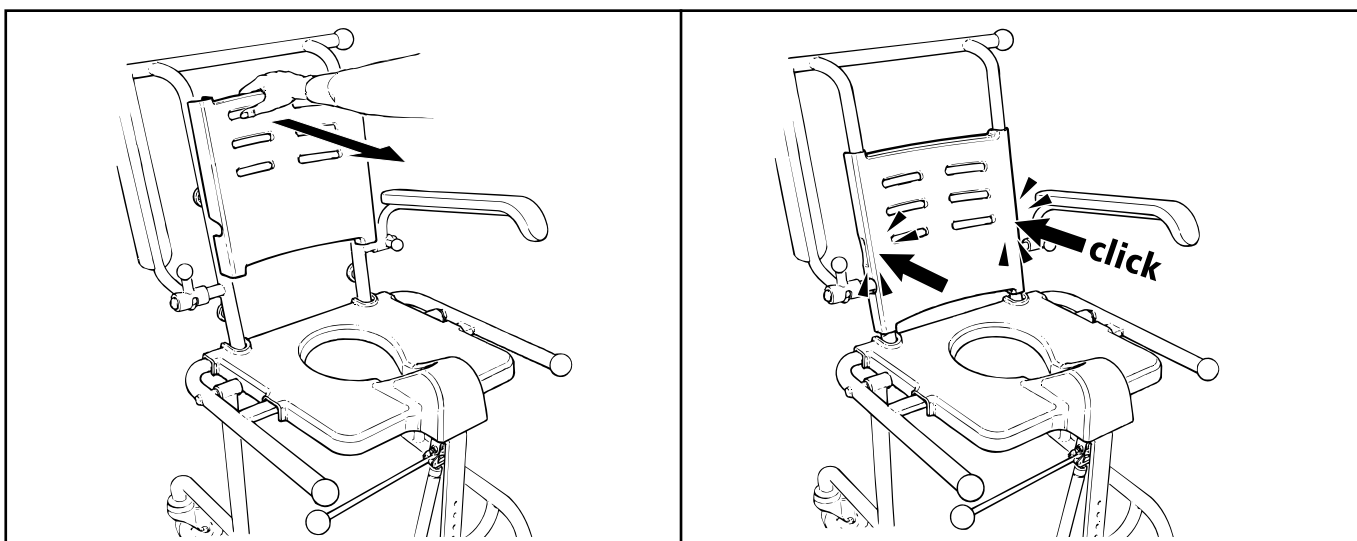
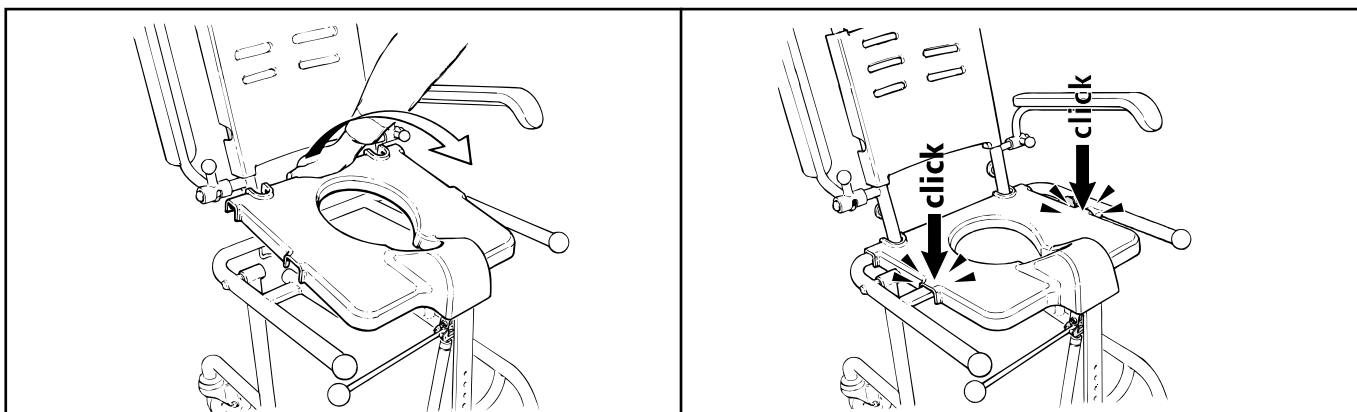
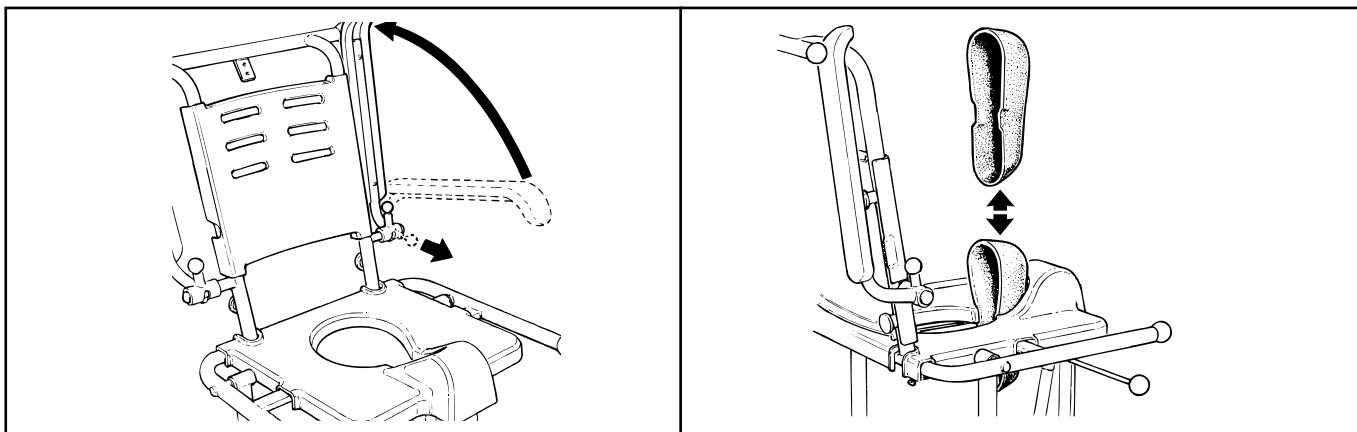
78056.7 23-01-18

sv no da en de fi fr nl it es pt




A

**B**

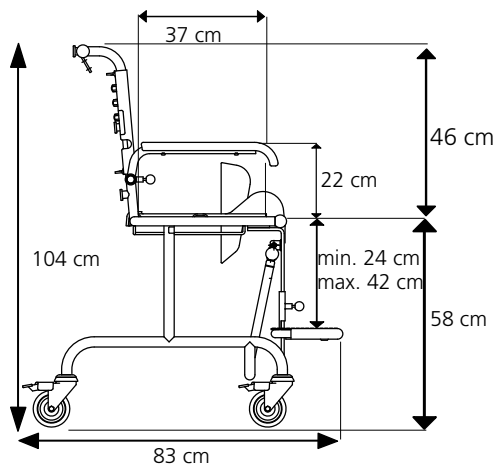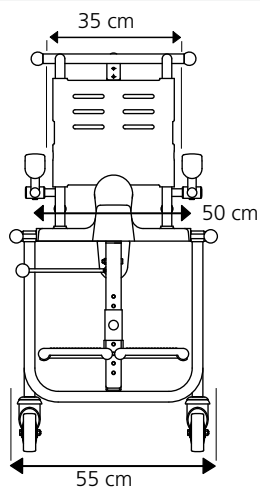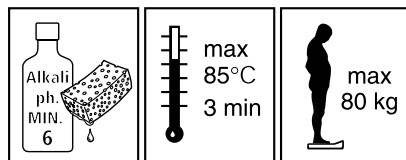**C**

17 kg

SV

## Svenska

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en

## English

Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at [www.etac.com](http://www.etac.com). You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

**Intended use**

The Tripp is a mobile shower and toilet chair designed for use in facilitating intimate hygiene for persons with restricted mobility.

The product conforms to the requirements of the Medical Devices Regulation (EU) 2017/745.

There are no known contraindications.

**Installation, use** ..... [Figure A](#)**Maintenance and safety** ..... [Figure B](#)

1. Clean/disinfect the chair with normal non-abrasive cleaning agents with a pH value of between 5-9, or with 70% disinfectant solution. Rinse and dry.  
The product is otherwise maintenance-free.
2. The product can withstand cleaning in a sterilizer at 85°C for 3 minutes. This applies for reconditioning.
3. Maximum user weight 80 kg.
4. Defective or incorrectly installed products must not be used. In case of damage, contact your dealer.
5. Heated chair can cause burns.

The materials used are corrosion-resistant.

The materials used are resistant to all general disinfectants.

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

**Guarantee, service life**

5-year guarantee against material and manufacturing defects. For terms and conditions, see [www.etac.com](http://www.etac.com).

Service life 7 years. For complete information regarding the service life of the product, see [www.etac.com](http://www.etac.com).

The product's date of manufacture can be read from the barcode on the product. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

**Storage**

The product should be stored indoors in a dry place at a temperature above 5°C. If the product has been stored for a long time (more than four months), its function must be checked by an expert before use.

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Etac Supply Center AB