
# Etac Swift Mobile Tilt Motor

## Manual

**etac**<sup>®</sup>

78543D 2019-06-20

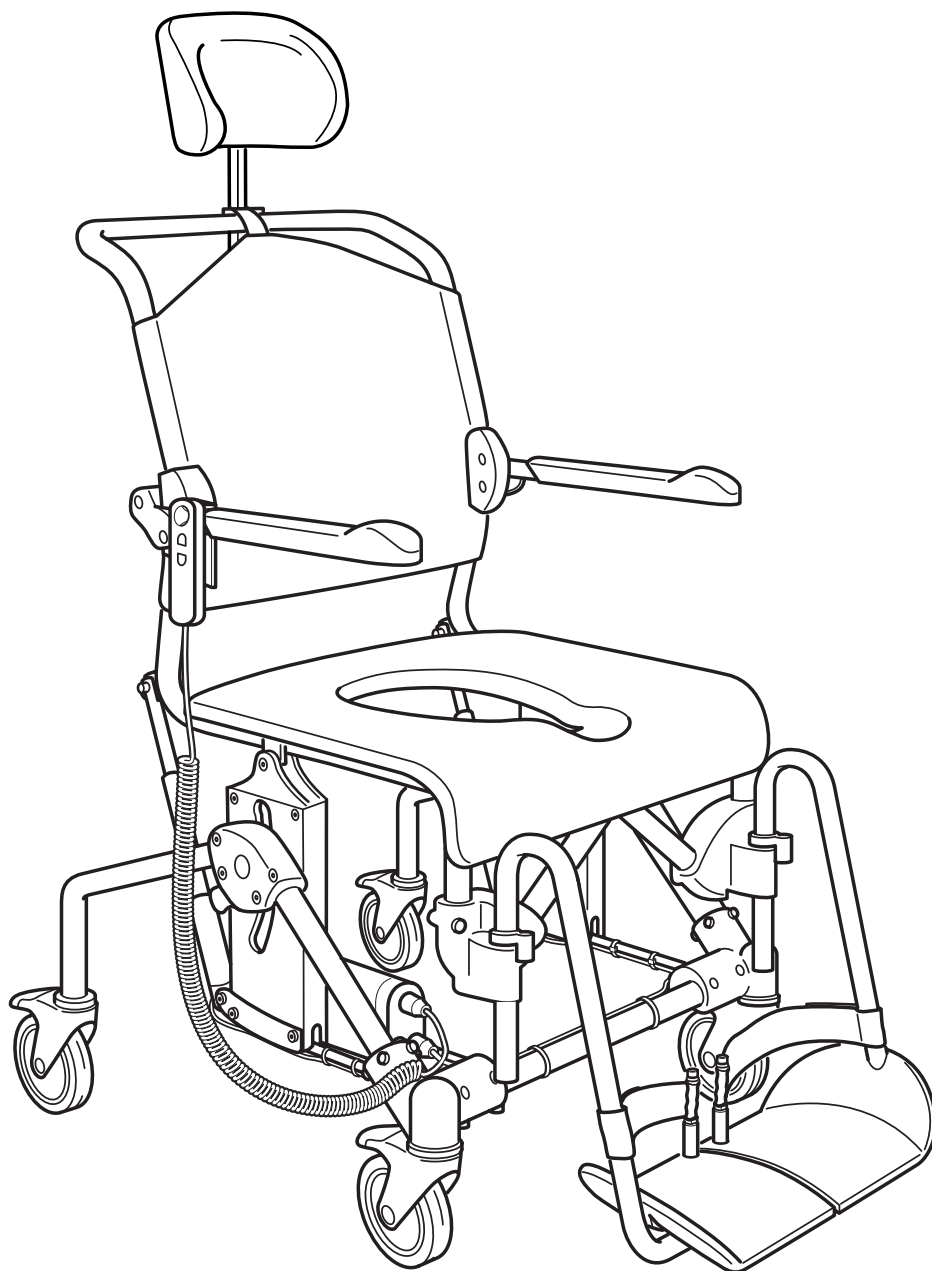CE

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# English

## General information

Thank you for choosing a shower chair from Etac.

In order to avoid injury during handling and use, please read through this manual first.

In the manual, the user is the person sitting in the chair. The assistant is the person operating the chair.

Etac Swift Mobile Tilt Motor is a shower and toilet chair intended to make hygiene care easier for people with reduced mobility and their carers.

Swift Mobil has been tested by a testing institute and complies with the requirements stated in EN ISO 10993-1 and SS EN 12182.

It is designed for use in showers, with hand basins and over toilets, as well as for transport to and from the sanitary room. Every component of the chair is designed to make things easier for users and assistants.

This symbol appears in the manual along with text. It indicates instances where the safety of the user or the assistant may be at risk.

The images which are referred to on each page are found at the start of the manual (pages 3 to 9). See Figures A to M for the corresponding text.

This product should be recycled as per national regulations.

## Product description

See Figure A

- |                    |                                |
|--------------------|--------------------------------|
| <b>A1.</b> Bolt    | <b>A4.</b> Armrest             |
| <b>A2.</b> Legrest | <b>A5.</b> Backrest upholstery |
| <b>A3.</b> Seat    | <b>A6.</b> Headrest            |

## Assembly

See Figure B

## Dimension table

See Figure C

## Warranty / maintenance

See Figure D

The warranty period for material and manufacturing defects is 5 years.

### Maintenance and reconditioning

To keep the chair in good and safe working order, you should regularly:

- D1.** Clean the chair with ordinary non-abrasive detergent. Rinse and dry.
- D2.** The product can be cleaned at 85 °C for 3 minutes.
  - Keep the castors free of hair and soap deposits, etc. If your shower and toilet chair is not in perfect working order, contact your dealer.
  - Charge the battery regularly.

## Transfers

See Figure E

### Over a threshold:

- E1.** Chair only, put one foot on the chassis tubing
- E2.** With occupant, pull the chair backwards.

### In stationary position:

- E3.** Always lock as many castors as possible when the chair is to be used in a stationary position.
- E4.** Out-turned castors improve the chair's stability.

### Transferring:

Assess the risks and take notes. You as a carer are responsible for the safety of the user. Aids should be used if the transfer is deemed as risky. Visit [www.etac.com](http://www.etac.com) for manual transfer aids.

- E3-E4.** It is important to ensure that the castors are locked and pointing outward before carrying out transfers to and from Clean.
  - The legrests should be swivelled to the side whenever the user is sitting down or getting up.
  - Fold up the armrest on the transfer side of the chair.
- E5-E7.** Carry out the transfer.

## Safety

See Figure F

- Defective chairs must not be used.
- Do not use outdoors, in saunas or immerse in water.

- F1.** Make sure the castors are locked when users sit down on or get up from the chair.
  - An anti-tip attachment must be used if the user is an amputee.
- F2.** Never stand on the foot supports as there is a risk of tipping!
- F3.** The sitting position has an impact on stability.
- F4.** Chair stability is improved if the castors point outward.
- F5.** See the maximum user weight for your commode chair in the table in the centre-spread section (page 15).

### Actuators/Batteries

- F6.** Must be recycled
- F7.** Reinforced/Dual insulation 2
- F8.** For indoor use

## Legrest

See Figure G

- G1.** The assistant can easily raise and swivel the legrests to the side for the user to sit down in or get up out of the chair. The footrests are detachable.
- G2.** Various height adjustment options are available by moving the locking pin. The legrests have 6 different height settings.

## Adjustment

See Figure H

### Height

- H1-H7.** The height is easily adjusted by inserting the bolt in the required position. See distances in table

Check that both sides have been adjusted to the same height as otherwise stability can be reduced. **H8.** Do not use the chair if the bolt (orange) is not locked.

### Anti-tip function

- H9.** The carer can easily regulate the seat angle using the handset. Make sure the castors are locked if the anti-tip function is used while the user is sitting in the chair. The seat unit can be tilted back 35° and forward 5°. We recommend the use of a head support.

# English

## Motor - Technical data

Operation voltage:	24 V
Current:	7A
Battery type:	NiMH
Charger in:	110 or 240 V AC
Charge time:	5-8 h
Swift Mobil Tilt Motor protection class:	IP 65
Charger protection class:	IP 65

## Charging the battery

See Figure J

The product must be dry and in a dry environment when managing the battery.

**J1.** Take the battery out of the control box and place it in the C3 charger.

When the battery is located in the charger the green light starts to flash until it is fully charged, at which time the light remains constantly green.

### Charging the battery

- The battery must be charged fully before the product is used for the first time.
- When starting to use the product the battery must be charged once a day or more frequently if necessary.
- When the product is not in use for a prolonged period the battery must be charged once a month.
- When fitting the battery make sure that the contact surfaces in the battery and control box are clean and dry. If a beep is heard the battery is fitted correctly and the chair is ready for use. If there are two buzzing sounds followed by one beep, the control box is in calibration mode and calibration must be completed before further use.
- If the battery is hot after a charge or the chair is overloaded when in use, the battery's overheating protection may be triggered. Remove the battery and allow it to cool down for about 30 minutes.
- If the battery is taken out of the control box, wait at least one minute before fitting a new battery.
- It is not permitted to use the chair without a battery in the control box.

## Motor

See Figure K

- K1.** Handset (Protection against water in line with IP 65), included.
  - K2.** Charger, included
  - K3.** Actuator (Protection against water in line with IP 66)
  - K4.** Battery (Protection against water in line with IP 65)
  - K5.** Control unit (Protection against water in line with IP 65)
- Make sure the battery, actuator and the handset are connected properly to the control unit.

## Calibrating the actuators

See Figure L

**Model C3 (see label on back of handset):**

**L1a.** Both up and down buttons are pressed simultaneously until a beep followed by two buzzing sounds are heard (about 10-15 seconds). The control box is now in calibration mode.

**L2.** Calibration is carried out by holding the back button in until both actuators stop in the end position. Keep the button pressed in for an extra couple of seconds. The chair is now calibrated and can be used.

**Model C4 (see label on back of handset):**

**L1b.** Press the up and down button on the handset. Do so until the actuators stop and then release the button.

**L2.** Press and hold the up and down button again (approx. 10

seconds) until the actuators move a short distance and then stop. Now release the up and down button. The LED on the control box flashes green in short, quick bursts during this step. A short beep and a longer solid green flashing LED confirm a successful calibration.

## Fault-finding chart

Faults	Possible cause	Solution
A signal is heard when using chair	The battery is almost flat	Charge the battery
If tilting does not work	The cables may not be connected or the battery is flat.	Insert the cables, charge the battery.
If tilting does not work	The actuators may need to be calibrated.	Calibrate the actuators.
If tilting does not work	The battery is overheated.	Remove the battery and allow it to cool down for at least 30 minutes.

## Accessories

See Figure M

- M1.** Leg and calf rest, angle-variable
- M2.** Footrest
- M3.** Commode pan
- M4.** Pan-holder attachments
- M5.** Attachments for low pan-holder mounting
- M6.** Commode pan holders
- M7.** Bucket holders
- M8.** Splash guard
- M9.** Comfort seat
- M10.** Seat cover
- M11.** Anti-tip stabilisers including bracket
- M12.** Anti-tip stabilisers excluding bracket
- M13.** Crossbar 605 mm
- M14.** Chest belt
- M15.** Positioning belt
- M16.** Hemiplegic armrest
- M17.** Armrest lock
- M18.** Angle adj. footrest
- M19.** Soft back pad
- M20.** Tilter
- M21.** Widening kit
- M22.** Soft Comfort seat
- M23.** Soft Comfort seat - narrow opening
- M24.** Comfort padding
- M25.** Trunk support
- M26.** Armrest cushion, pair
- M27.** ROHO pressure-relieving cushion
- M28.** Pan
- M29.** Lid with handle
- M30.** Amputee legrest
- M31.** Soft cushion - standard opening
- M32.** Soft cushion - oval opening
- M33.** Castor wheel directional lock

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Etac UK Limited
Etac A/S
Snug Seat, Inc.
Etac Supply Center AB