Etac Stapel

Manual

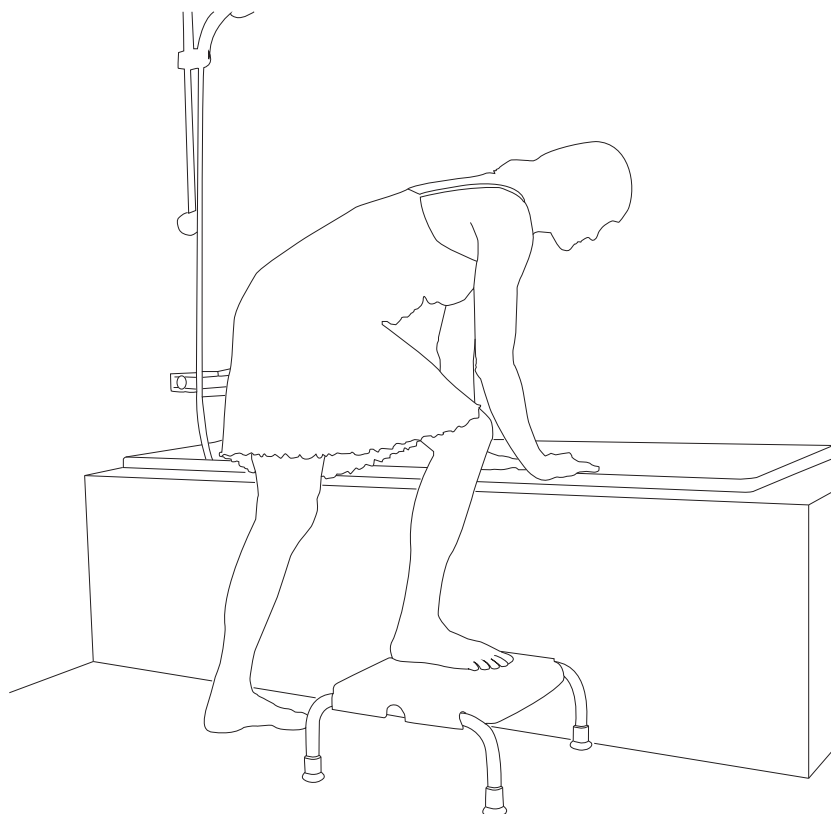
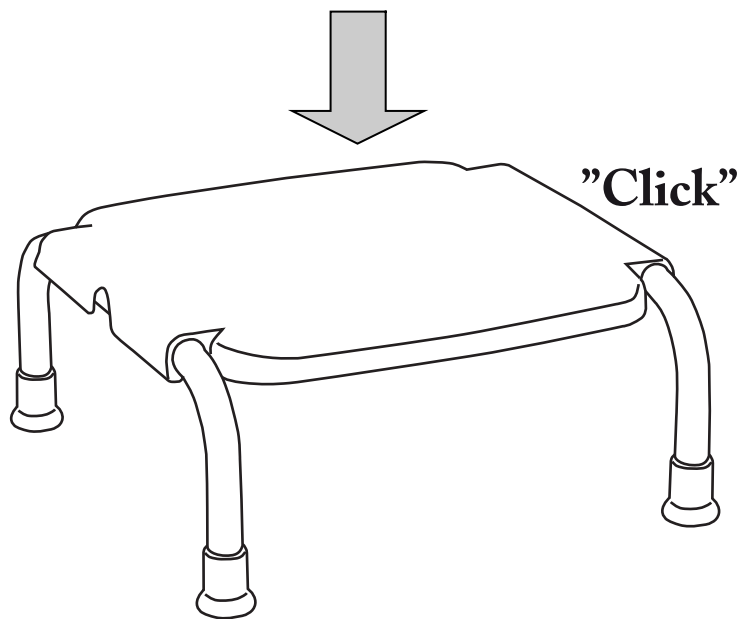
etac®

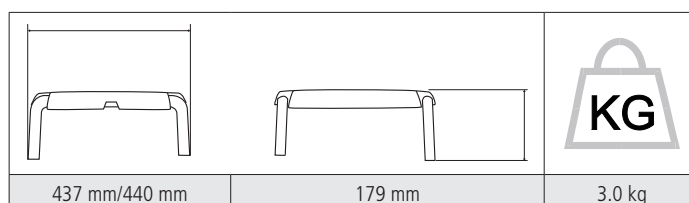
78643D 21-05-05

sv no da en de fi fr nl it es pt

CE

A

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Tekniske data..... *Figur D*

en**English****Intended use**

The Etac Stapel is a stool designed for use as an additional step for climbing into and out of the bath, and as a footstool for use with a toilet, for example, by persons with restricted mobility.

The product conforms to the requirements of the Medical Devices Regulation (EU) 2017/745.

Installation, use..... *Figure A*

Maintenance and safety..... *Figure B*

Max user weight: 130 kg.

Cleaning: Clean the product using a solvent-free cleaning agent with a pH value of 5-9 or with a 70% disinfectant solution.

Decontamination: Can be decontaminated at max. 85° C/185 °F for 3 minutes if required.

The materials used are corrosion-resistant.

The materials used are resistant to all general disinfectants.

Warning: A defective product must not be used.

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Guarantee, service life

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Service life 10 years. For complete information regarding the service life of the product, see www.etac.com.

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Etac Ltd.
Etac A/S
R82, Inc.
Etac Supply Center AB