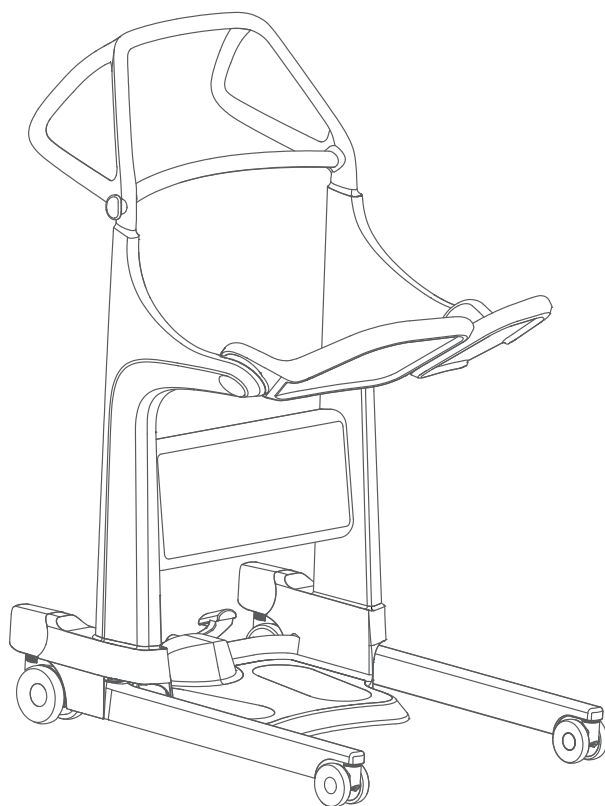
User manual

Molift Transfer Pro

molift[®]

by Etac

BM70199 - 2024-02-09 - Rev. 1.0

www.etac.com

	Symbols	4
	Device overview	8
	General	10
	English	10
	Svenska	14
	Norsk	18
	Dansk	22
	Suomi	26
	Deutsch	30
	Nederlands	34
	Français	38
	Italiano	42
	Español	46
	Handling	50
	Rising	51
	Lowering	53
	Warnings	55
	Information	57
	Technical data	58

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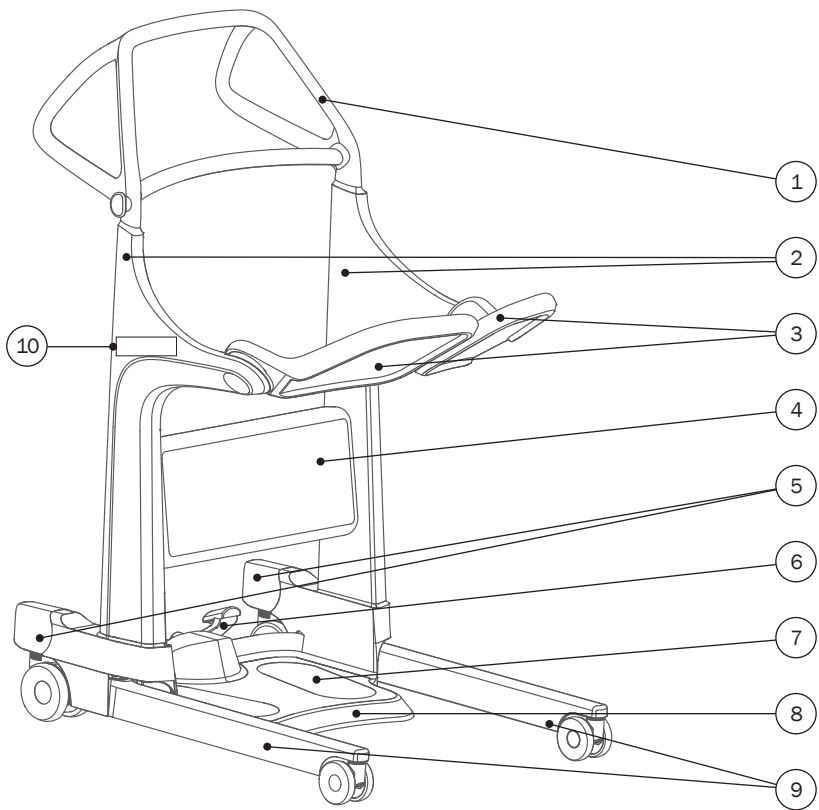
General

Thank you for choosing an Etac device.

This manual must be read thoroughly before you start using the device as it contains important safety information and gives instructions on how to use the device correctly. Visit www.etac.com for all relevant product documentation, for example the user manual, maintenance and recycling instructions. You can select your language via the “International” and “Local websites” link.

You can also consult the QR code on the device for further information and instructions.

Copyright

This document is the property of Etac A/S. The content shall not be modified in whole or part without prior approval of Etac A/S.

Disclaimer

Etac continuously makes improvements to our devices and information. We therefore reserve the right to modify our devices and instructions for use without prior notice. Consult www.etac.com to find the latest version of the documentation.

CE Declaration of Conformity

The device mentioned in this manual and the different models hereof comply with the Medical Device Regulation (EU) 2017/745.

The device is classified as a Class I medical device.

Product standards

The device has been tested and complies with the requirements applicable for this type of device in the below mentioned standards:

- ISO 21856:2022 Assistive products — General requirements and test methods.

Device description

Etac supplies a wide selection of sit-to-stand devices. These devices assist people with impaired mobility to be transferred between two sitting positions. See www.etac.com for a complete overview of sit-to-stand devices.

Molift Transfer Pro is a sit-to-stand device that is used for seated transfers; e.g. between bed, wheelchair, chair, toilet or similar. It is designed for easy maneuvering, and the ergonomic handle is designed with multiple grip options for both user and carer. The spacious design of the handle allows easy access to the sink when washing hands. The stable seat pads enhance a comfortable and safe transfer for both user and carer. The leg support has a soft padding and the angle adapts to the shins. The base plate has a sloped edge and a low step-in height making it easy to place the feet. The design of the leg adjustment allows the device to be placed close to the sitting area.

Benefits

The clinical benefits of Molift Transfer Pro are:

- Reduces work-related injuries or strain of carer.
- Enables early rehabilitation.
- Comfortable and safe seated transfers.
- Supports the user to use their physical ability.
- Enables single-handed care.

Warranty

Please refer to www.etac.com/support.

If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

Notice

Any serious incident that has occurred in relation to the device should be reported to your distributor/representative and the national competent authority within your region in a timely manner. The distributor/representative will forward the information to the manufacturer.

Pre-sales information

Please visit www.etac.com.

Materials

Powder coated aluminum, Plastic, Steel, Polyuretan foam.

Introduction

The person referred to as “the user” is the person who is standing on the product. “The carer” is the person who manoeuvres the product.

Intended purpose

The sit-to-stand aid is an assistive device intended for alleviation of, or compensation for, a functional impairment due to an injury or disability. The device is designed for an individual lacking the ability to stand up and transfer themselves to another sitting position to/from a bed, a wheelchair, a chair, a toilet or similar due to reduced mobility or physical strength.

Intended user

The device can be used for children, adults and elderly people with a need for support to stand up and be transferred seated between sitting positions.

Inteded environment

For indoor use on horizontal surfaces in acute care, long-term care and home care.

Indications

For users who are in need of support to stand up and be transferred seated between sitting positions; e.g. people suffering from pain, reduced range of motion in their joints, stiffness and muscle weakness. Not an exhaustive list.

The user group for the device is based on individual health and mobility function and not on a specific diagnosis or age.

Contraindications

There are no known contraindications.

Precautions

The user should be able to follow instructions, grip and pull up their own body weight and maintain a good core stability.

Training requirements

Only carers who have read and understood the user manual are allowed to use the device.

Safety information

The safety of both user and carer must never be jeopardized – if you suspect that the device has a malfunction you must put it in quarantine marked “out of order” and call your local distributor/representative.

Safety signs, safety measures and warnings symbols are given in the context relevant for the actual handling and/or operating situation – it is mandatory the given instruction is followed to avoid dangerous/hazardous situations.

Signs, symbols and instructions placed on this device must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the device. Immediately replace or repair illegible or damaged signs, symbols, and instructions. Contact your distributor/representative for instructions.

Patient transfer

Transfer of a user will always pose a certain risk. Therefore, a risk assessment must always be carried out before the transfer is performed.

Warnings:

- Never use the device for transportation of goods.
- Never use the device for self-transportation.
- Always transfer the user in a calm and stable speed.
- Never leave a user unattended in the device.
- Never transfer the user in a standing position.
- The device is not intended to be used for prolonged periods of time.
- Never use the device in the shower.
- Never use the device for gait training.

Risk assessment

Do a risk assessment of the device, the individual user and the transfer situation, and document your analysis. You as the carer are responsible for the safety of the user.

Plan the transfer in advance to ensure that it is performed as safe and smooth as possible.

Remember to work ergonomically according to local recommendations.

Ensure the user is able to remain seated on the seat pads and maintain feet placed on the base plate.

The carer has to be able to handle the weight of the user.

To avoid serious injury, the user shall avoid extended reaching or leaning movements.

The device

Always check – before each use:

- that the user has the required ability for the device. If needed use any suitable accessories.
- that the device is suitable for the user and the transfer situation.
- that the device does not have visible damage.
- that the wheels on the wheelchair or bed are locked during the transfer.
- that the device is locked when the user is moving from sitting to standing and vice versa.
- if the inspection reveals any safety risks, the device must immediately be taken out of service.
- never use a faulty or damaged device as it can cause personal injury. Contact your local distributor/representative for further instructions.

Materials

All used materials which are to be in direct contact with skin are biologically assessed and evaluated according to relevant standards. However, users/carers who are allergic to any of the materials are recommend not to use the device.

Accessories

User manuals supplied with accessories must be read and understood before use. This includes any limitations in use and combinations. Information about accessories can be found on www.etac.com.

Spare parts

A list of spare parts is available at www.etac.com.

Combinations

When combining the device with other devices or accessories, the combination must not change the intended purpose of the devices or modify the devices in such a way that compliance with the applicable requirements may be affected. The person/company responsible for the combination of the devices must ensure that the applicable requirements are fulfilled.

Assembly and supplied parts

Consult the assembly guide on www.etac.com.

Safety measures

The base plate has visible feet contours and anti-slip surface for extra safety.

The central brake provides auditive feedback and is easily accessible as it locks both wheels by activating only one of the brake pedals. When activated a red marking becomes visual. The central brake is marked with a (P) on the device.

The triangular marking on the side of the seat pad must minimum reach the front edge of the seat before the user sits down. (See illustration no. 02 on page no. 53). If the triangular marking does not reach the front edge of the seat, the transfer must under no circumstances be completed. If the triangular marking reaches the front edge of the seat, make a risk assessment to make sure that the user sits down far enough into the seat.

Service, cleaning and maintenance

Maintenance and service

Maintenance and inspections of the device are wholly the responsibility of the owner of the device. Failure to maintain a device in accordance with the instructions may compromise the safety of the user and/or carer and may affect the expected lifetime of the device.

Inspection of the device must be carried out regularly.

Check:

- that the device does not have visible damage.
- that frames and handle are well fitted and screws are tightened.
- that labels are readable.
- that the leg adjustment and wheels move smoothly.
- that the brake works.

Expected lifetime

The device has an expected lifetime of 10 years or 60.000 transfer cycles under normal use. The lifetime of the device may vary depending on usage frequency, loads and washing/drying frequency and conditions.

Cleaning

Clean on a regular basis. Clean surfaces with a damp cloth using an appropriate pH-neutral detergent. Do not use solvents or strong liquids as this may damage surfaces on the device. For disinfection when needed; use isoPropyl alcohol or similar. Avoid abrasive cleaning products.

Remove hair and filth from the wheels to ensure the wheels rotate freely and the brake functions properly.

Make sure not to damage or remove labels when cleaning.

Reconditioning

Separate reconditioning document is available at www.etic.com.

Transportation, storage and disposal

Transportation and storage

When the device is not in use, it can be stored under the following conditions:

Keep packaging away from children

Keep away from fire

The device can be stored and transported under temperatures between -30 - 40 °C.

It is recommended to store the device with activated brake.

Disposal

The device contains no hazardous substances that require special disposal. Check if there are options available for recycling in an environmentally friendly way and follow the national provisions that apply. Dispose of the device according to the national provisions that apply.

Recycling

See the recycling instructions on www.etic.com, and follow the national provisions that apply.

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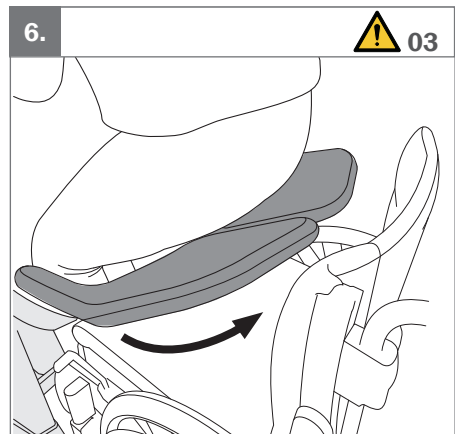
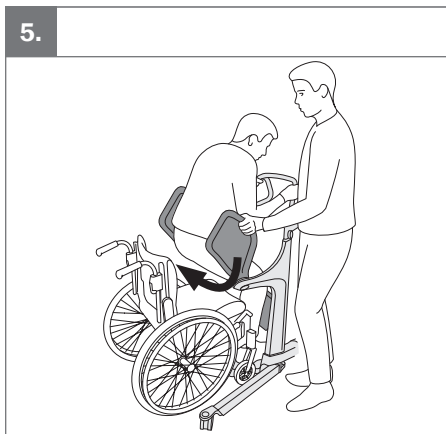
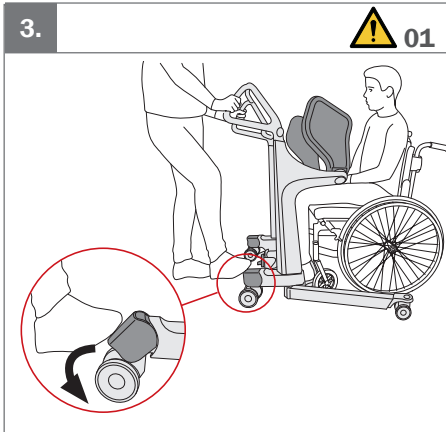
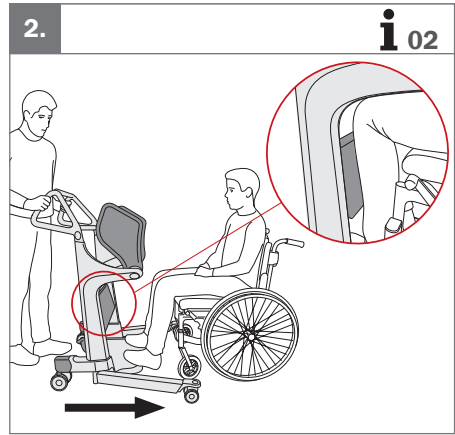
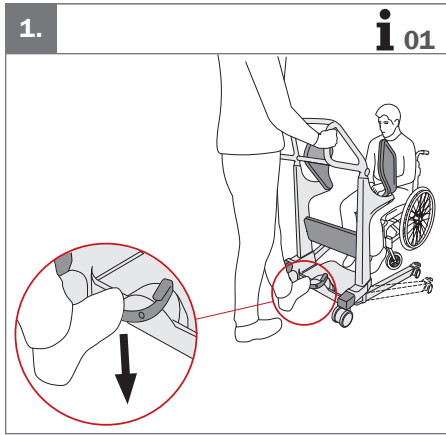
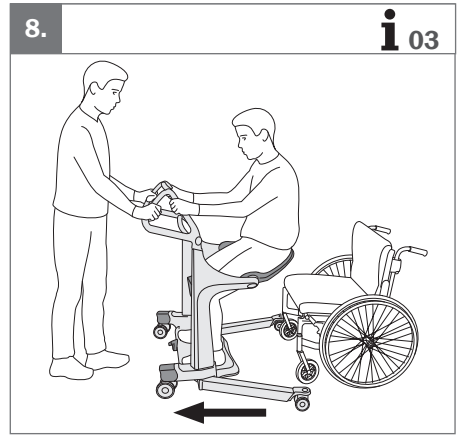
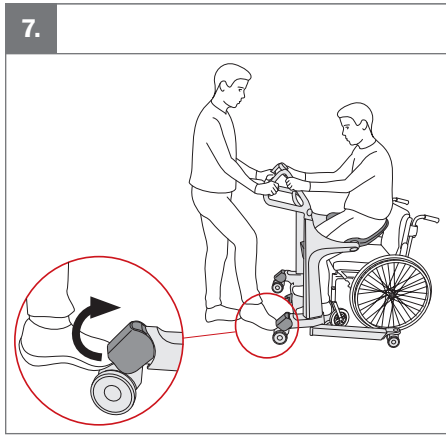
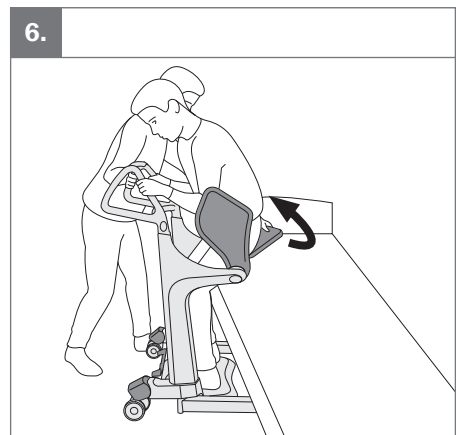
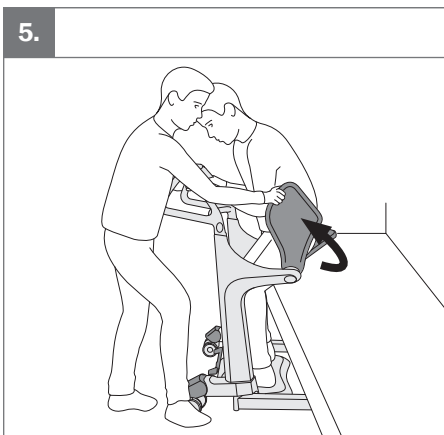
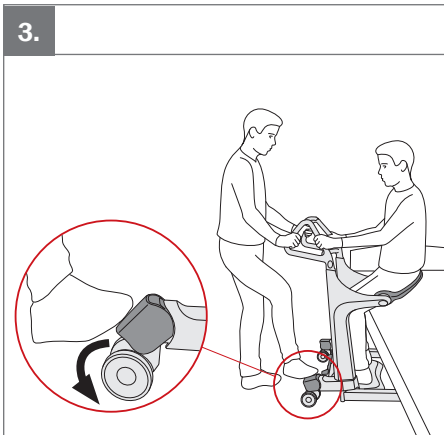
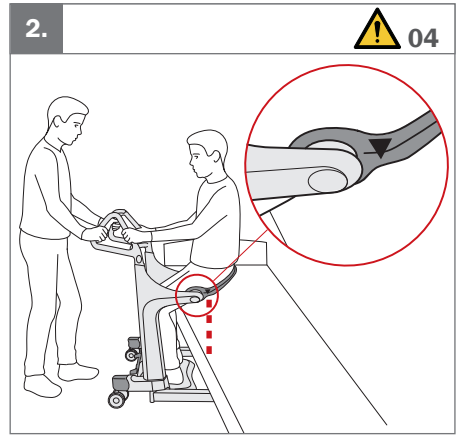
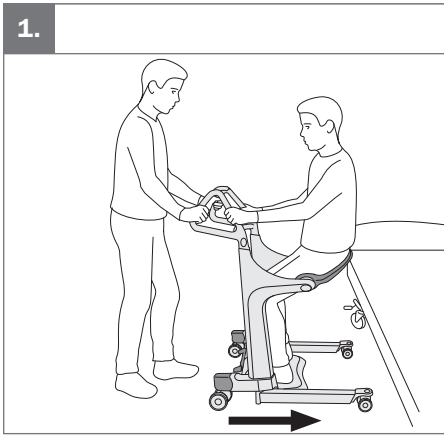
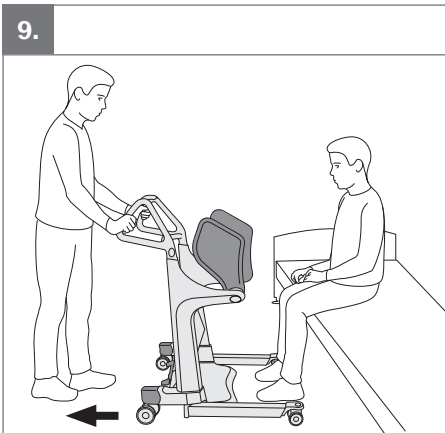
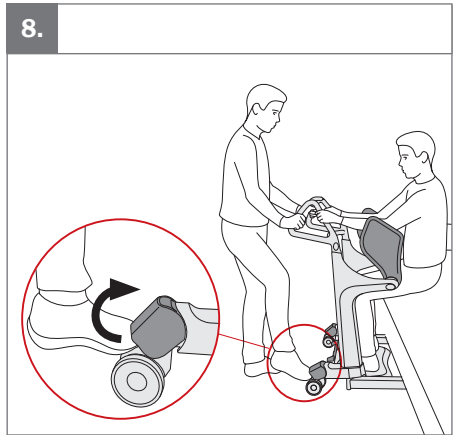
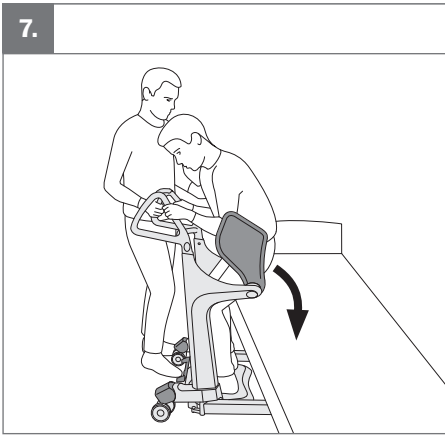
57

**EN:** Warnings

01. The wheels must always be locked when the user moves between sitting and standing position.
02. The user should always hold on to the handles during transfer.
03. The user must remain standing until both seat pads are rotated down.
04. Ensure that the marking on the seat pad minimum aligns with the seat's edge before the user sits down.
05. Always rotate both seat pads up before the user sits down.

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Etac Ltd.