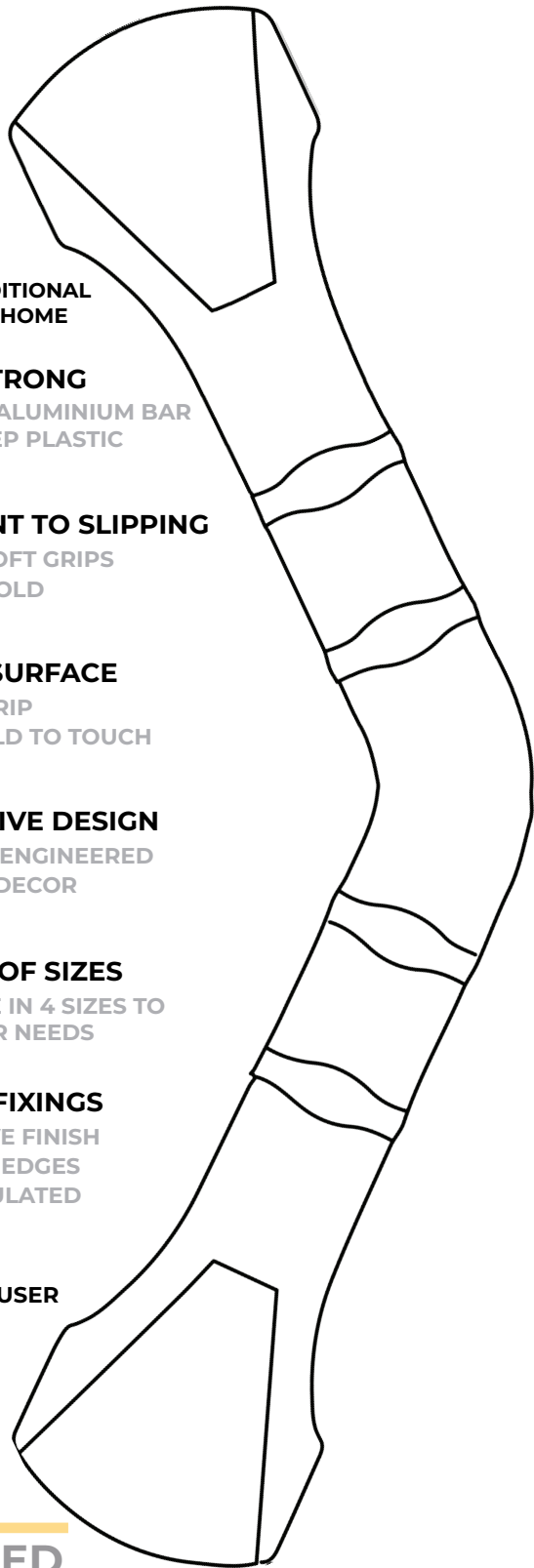
HANDY GRAB BAR

PROVIDES ADDITIONAL SAFETY IN THE HOME

- **EXTRA STRONG**
INTERNAL ALUMINIUM BAR
EXTRA-DEEP PLASTIC
- **RESISTANT TO SLIPPING**
SPECIAL SOFT GRIPS
EASY TO HOLD
- **RIBBED SURFACE**
EASY TO GRIP
NEVER COLD TO TOUCH
- **ATTRACTIVE DESIGN**
SUPERBLY ENGINEERED
SUITS ALL DECOR
- **VARIETY OF SIZES**
AVAILABLE IN 4 SIZES TO
MEET YOUR NEEDS
- **HIDDEN FIXINGS**
ATTRACTIVE FINISH
NO SHARP EDGES
FULLY INSULATED

⚠ **MAXIMUM USER WEIGHT**

ANGLED

40 cm: 80210040

A **HANDY GRAB BAR** IS AN INVALUABLE SAFETY AID FOR EVERYONE, ESPECIALLY THOSE WHO MAY BE UNSTEADY AND WOULD BENEFIT FROM A LITTLE EXTRA HELP

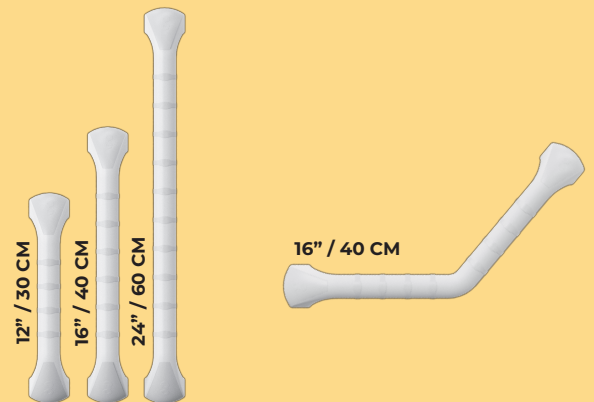
THE **HANDY GRAB BAR** CAN SERVE FOUR BASIC SAFETY FUNCTIONS:

- PUSHING UP
- PULLING UP
- STEADYING WHILE SITTING DOWN
- SECURITY WHILE TRANSFERRING POSITION

THE **HANDY GRAB BAR** RANGE:

STRAIGHT

ANGLED

⚠ ALWAYS ENSURE THAT GRAB BARS ARE SECURELY FIXED TO THE WALL WITH SUITABLE FIXINGS. FIXINGS WILL VARY DEPENDENT ON WALL SURFACE. CONSULT A QUALIFIED PERSON IF NECESSARY. WALL FIXINGS NOT INCLUDED.

ⓘ **SEE REVERSE FOR INSTRUCTIONS**
AVAILABLE IN THE FOLLOWING LANGUAGES:
📖 EN FR DE IT ES PT JP NL EL DK SE NO FI

🏢 **GORDON ELLIS & CO.**
TRENT LANE
CASTLE DONINGTON
DERBY. ENGLAND
DE74 2AT

EC REP
AUTHORISED REPRESENTATIVE:
ADVENA LTD. TOWER BUSINESS
CENTRE, 2ND FLR, TOWER
STREET, SWATAR, BKR 4013
MALTA

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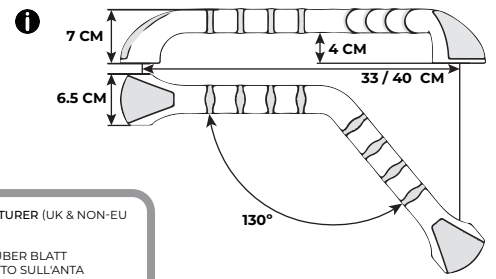
EN

1. Fitted correctly, angled rail must have one section horizontal to allow user extra leverage.
2. The rail may be fitted to most types of solid wall. If fitting to plasterboard ensure suitable fittings are used. If in doubt consult a tradesman.
3. Position the rail to a flat wall and mark the 4 screw positions.
4. To fix in position use no. 10 x 50mm (3/16" x 2") roundhead woodscrews with suitable wall plugs. Depending upon application other fittings may be suitable.
5. Locate the end caps to the rail and push down to fit.
6. Before first use check the rail is securely fitted to the wall.
7. Clean using non-scratch detergent.

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