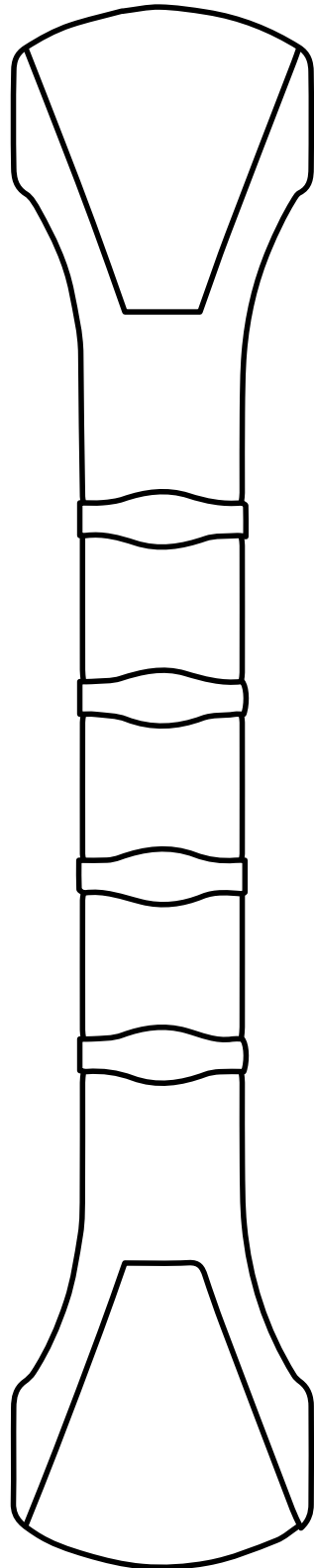
# HANDY GRAB BAR

## PROVIDES ADDITIONAL SAFETY IN THE HOME

- **EXTRA STRONG**  
INTERNAL ALUMINIUM BAR  
EXTRA-DEEP PLASTIC
- **RESISTANT TO SLIPPING**  
SPECIAL SOFT GRIPS  
EASY TO HOLD
- **RIBBED SURFACE**  
EASY TO GRIP  
NEVER COLD TO TOUCH
- **ATTRACTIVE DESIGN**  
SUPERBLY ENGINEERED  
SUITS ALL DECOR
- **VARIETY OF SIZES**  
AVAILABLE IN 4 SIZES TO  
MEET YOUR NEEDS
- **HIDDEN FIXINGS**  
ATTRACTIVE FINISH  
NO SHARP EDGES  
FULLY INSULATED

## ! MAXIMUM USER WEIGHT

## STRAIGHT

- 30 cm: 80210040
- 40 cm: 80210042
- 60 cm: 80210044

A **HANDY GRAB BAR** IS AN INVALUABLE SAFETY AID FOR EVERYONE, ESPECIALLY THOSE WHO MAY BE UNSTEADY AND WOULD BENEFIT FROM A LITTLE EXTRA HELP

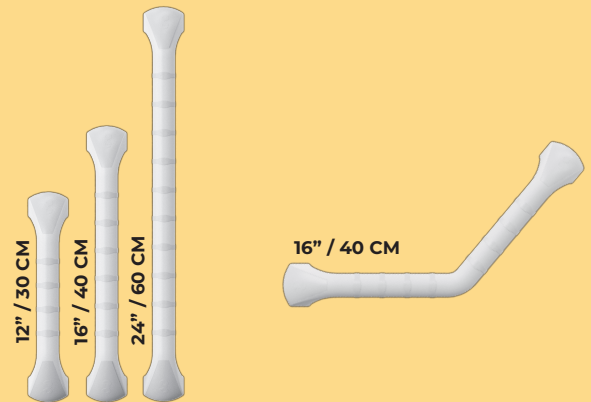
THE **HANDY GRAB BAR** CAN SERVE FOUR BASIC SAFETY FUNCTIONS:

- PUSHING UP
- PULLING UP
- STEADYING WHILE SITTING DOWN
- SECURITY WHILE TRANSFERRING POSITION

## THE HANDY GRAB BAR RANGE:

STRAIGHT

ANGLED

! ALWAYS ENSURE THAT GRAB BARS ARE SECURELY FIXED TO THE WALL WITH SUITABLE FIXINGS. FIXINGS WILL VARY DEPENDENT ON WALL SURFACE. CONSULT A QUALIFIED PERSON IF NECESSARY. WALL FIXINGS NOT INCLUDED.

**i** SEE REVERSE FOR INSTRUCTIONS  
AVAILABLE IN THE FOLLOWING LANGUAGES: MD

**i** EN FR DE IT ES PT JP NL EL DK SE NO FI

**GORDON ELLIS & CO.**  
TRENT LANE  
CASTLE DONINGTON  
DERBY. ENGLAND  
DE74 2AT

**EC REP**  
**AUTHORISED REPRESENTATIVE:**  
ADVENA LTD. TOWER BUSINESS  
CENTRE, 2ND FLR, TOWER  
STREET, SWATAR, BKR 4013  
MALTA

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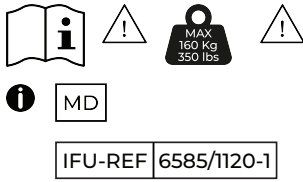
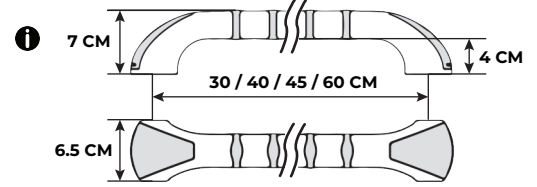
## EN

1. Fitted correctly, straight rails should be installed either horizontally or vertically.
2. The rail may be fitted to most types of solid wall. If fitting to plasterboard ensure suitable fittings are used. If in doubt consult a tradesman.
3. Position the rail to a flat wall and mark the 4 screw positions.
4. To fix in position use no. 10 x 50mm (3/16" x 2") roundhead woodscrews with suitable wall plugs. Depending upon application other fittings may be suitable.
5. Locate the end caps to the rail and push down to fit.
6. Before first use check the rail is securely fitted to the wall.
7. Clean using non-scratch detergent.

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