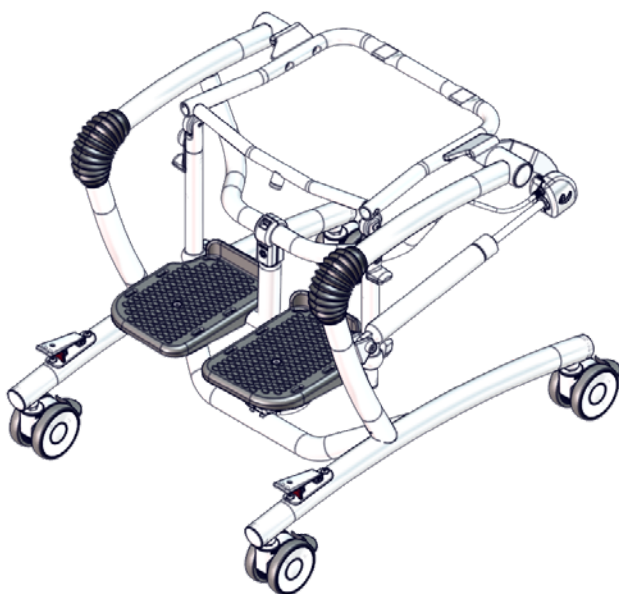
User manual

High-low bath frame

R82[®] 
by Etac

M1049 - 2024-02-21 - Rev. 8.0


www.etac.com

MD


	3	ENGLISH	32-34
	4	DEUTSCH	35-39
	5-6	NEDERLANDS	40-43
	7	DANSK	44-46
	8-9	NORSK	47-49
	10	SVENSKA	50-52
	11	SUOMI	53-55
	12-13	ESPAÑOL	56-59
	14	FRANÇAIS	60-63
	15	ITALIANO	64-67
	16-17	PORTUGUÊS	68-71
	18	PORTUGUÊS DO BRASIL	72-74
	19-29	РОССИЯ	75-79
	30-31	POLSKI	80-83
	32	ČESKY	84-86
	33	中國	87-89
	34	日本	90-94
	35	ΕΛΛΗΝΙΚΗ	95-99
			
			
			
			

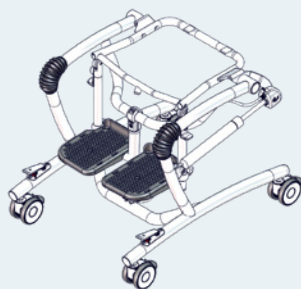High-low bath frame 1

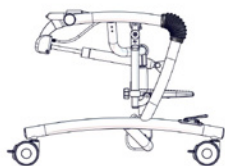
<37,5 kg
<82.6 lb

High-low bath frame 2

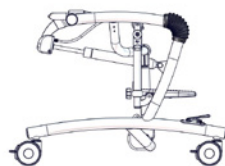
<53 kg
<116.8 lb

High-low bath frame 3

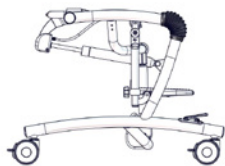
<64 kg
<141 lb






High-low bath frame 4
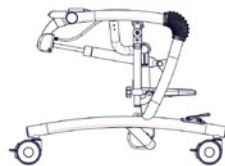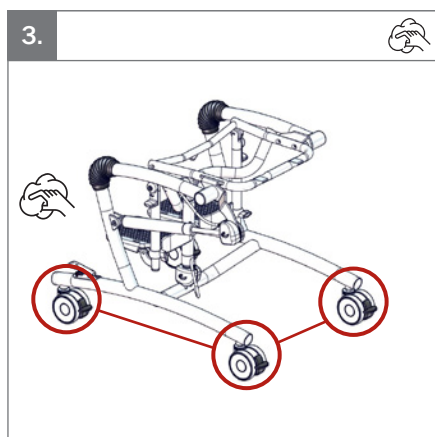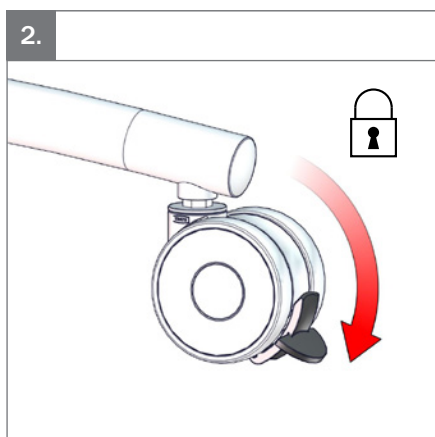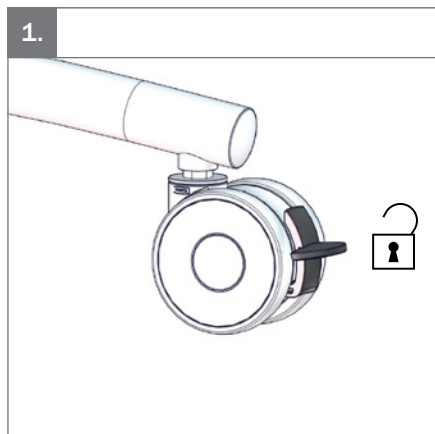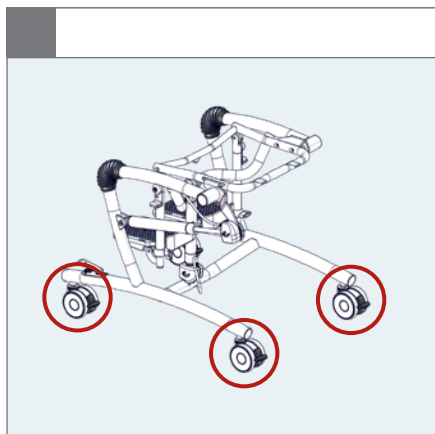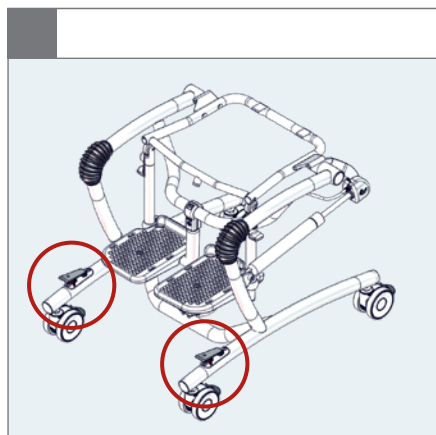
<74,5 kg
<164.2 lb








1.






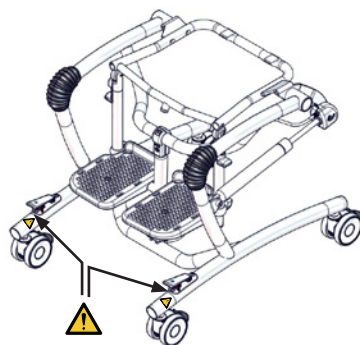
 05

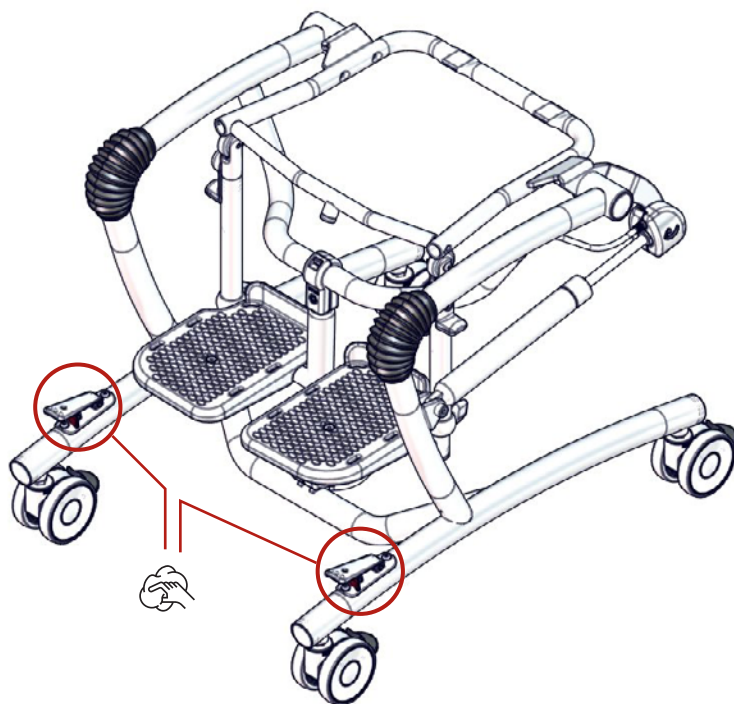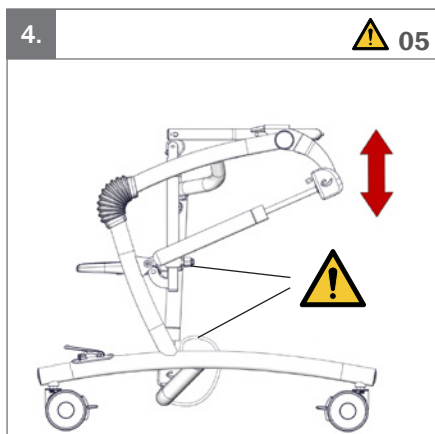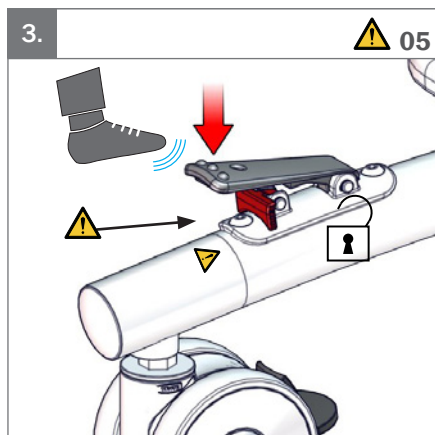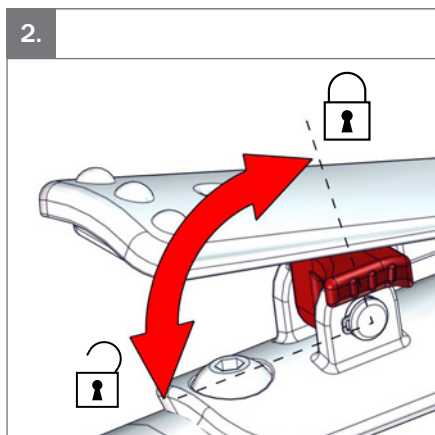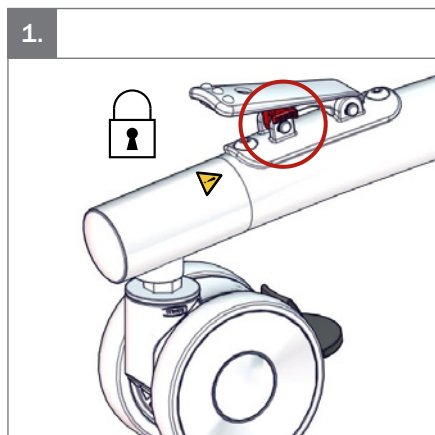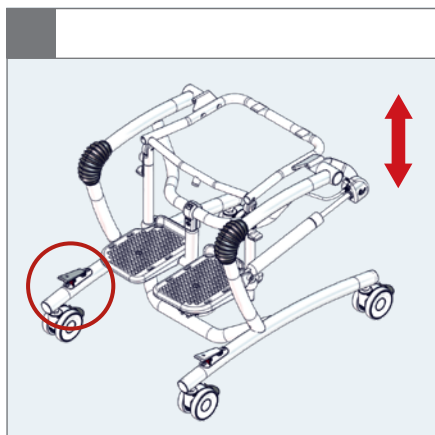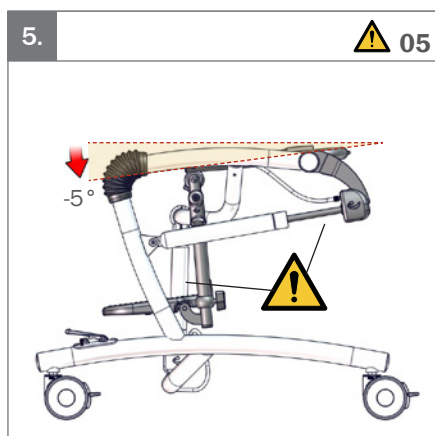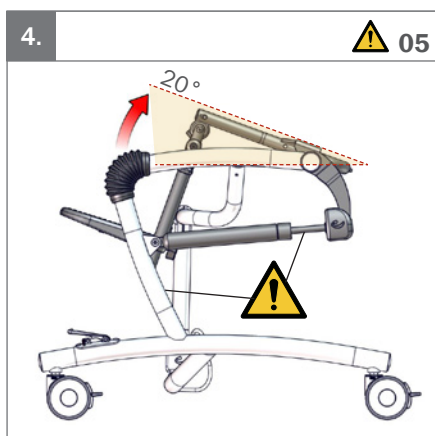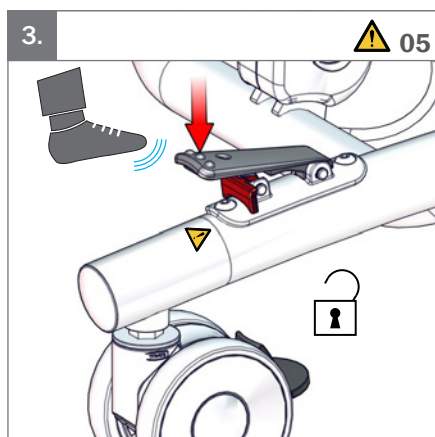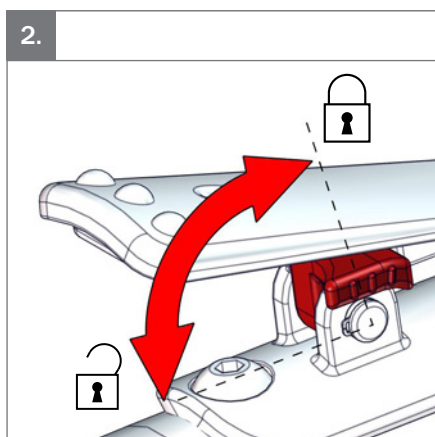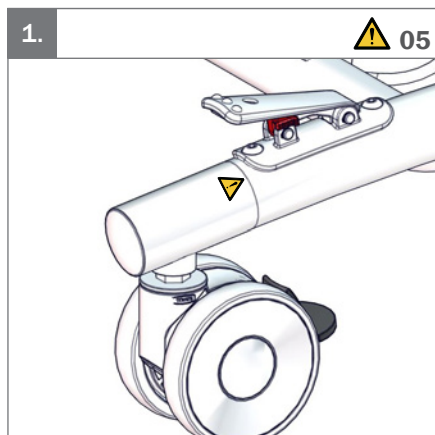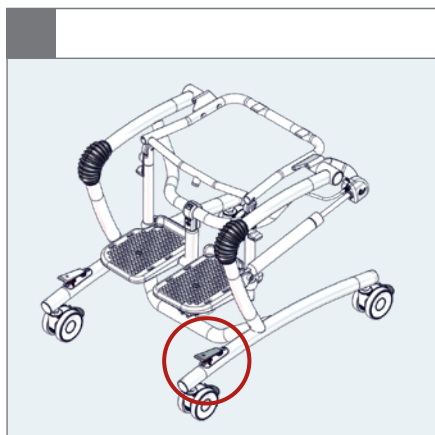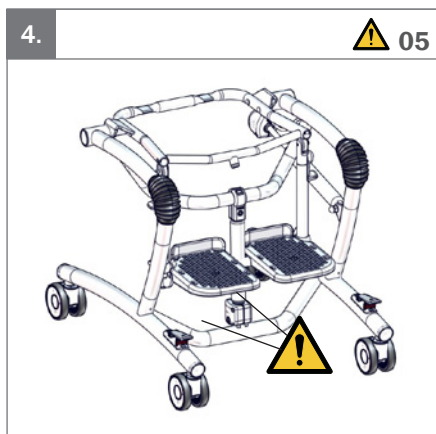
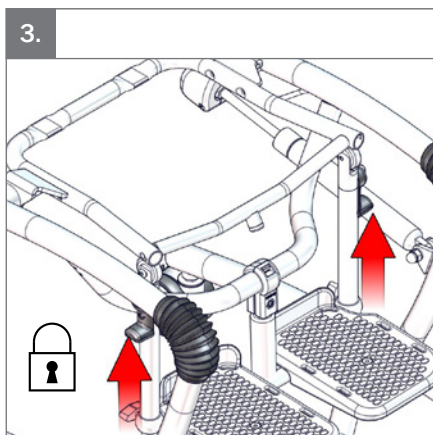
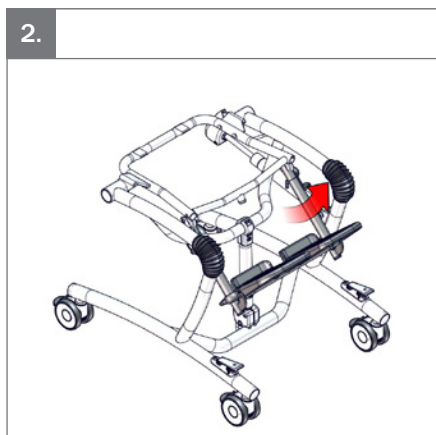
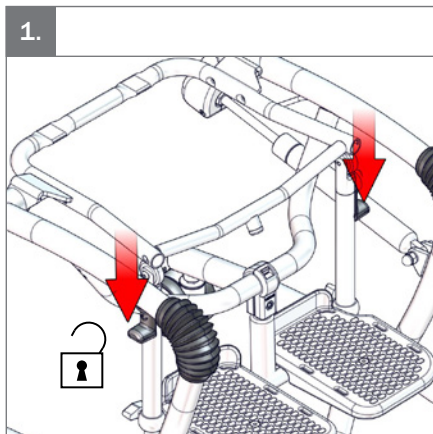
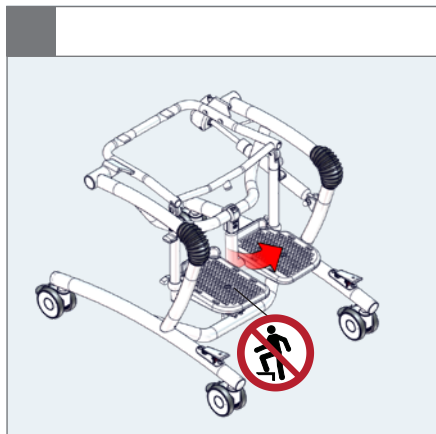
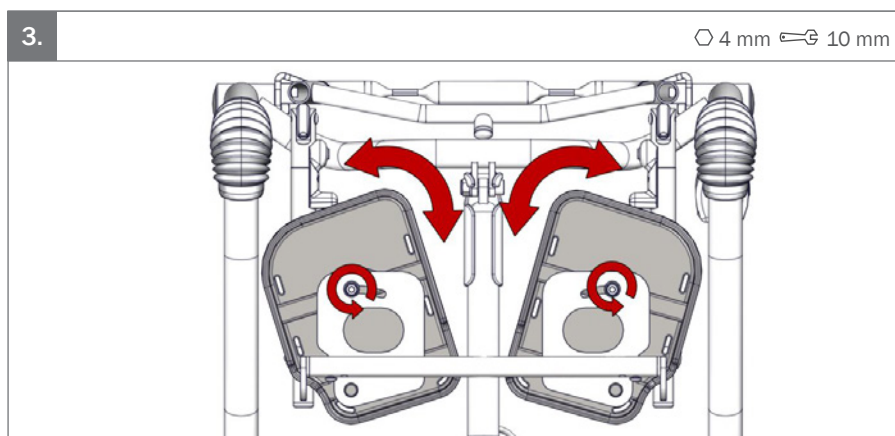
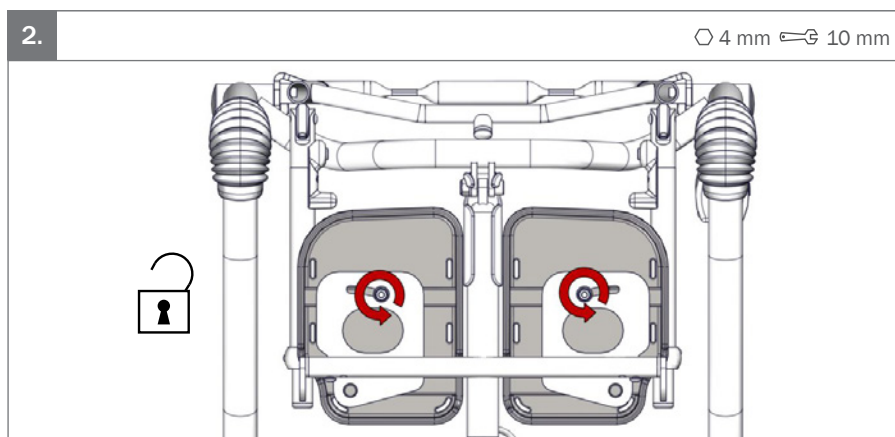
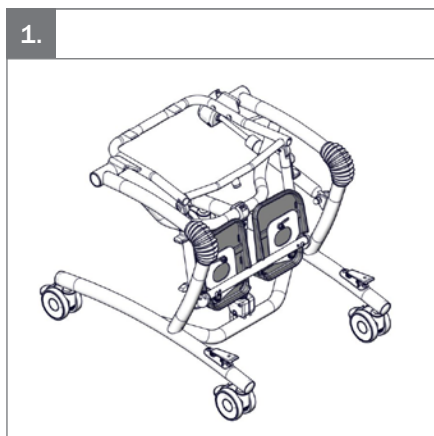
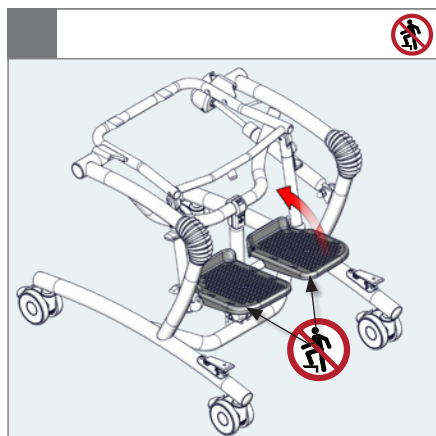
2.



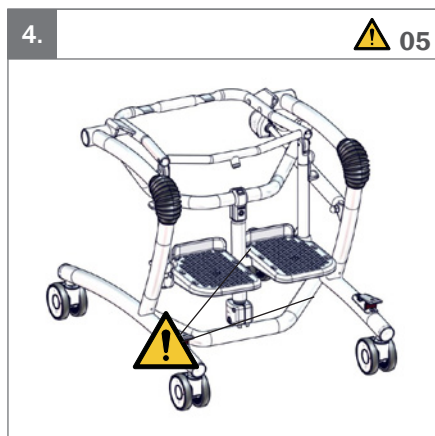
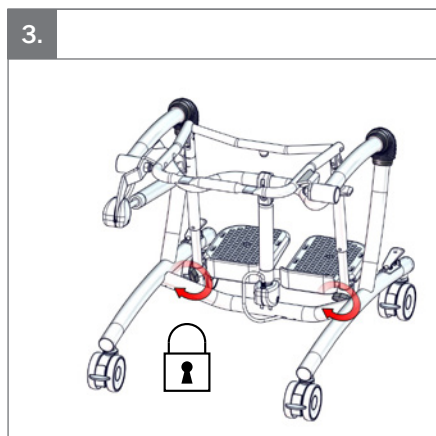
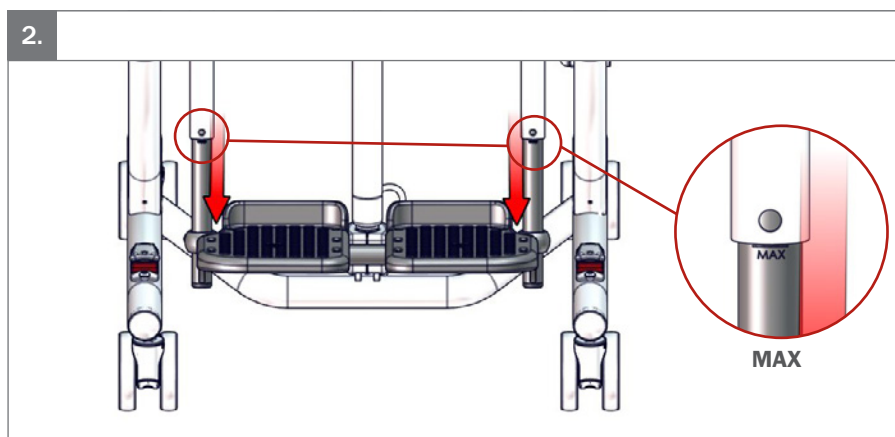
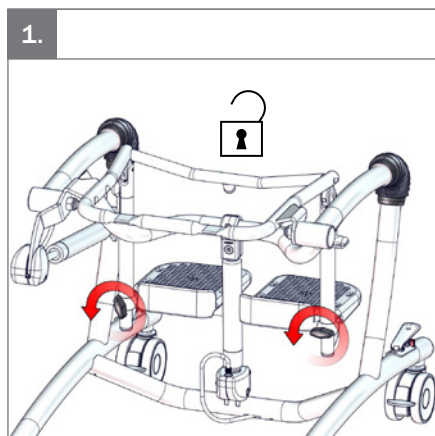
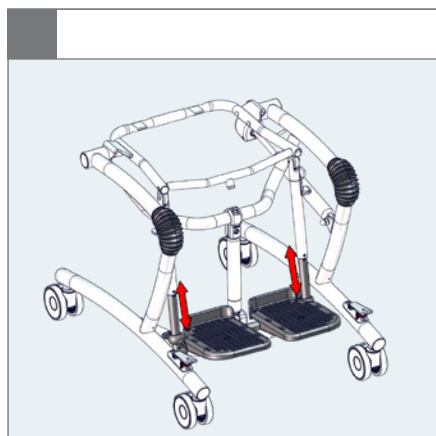
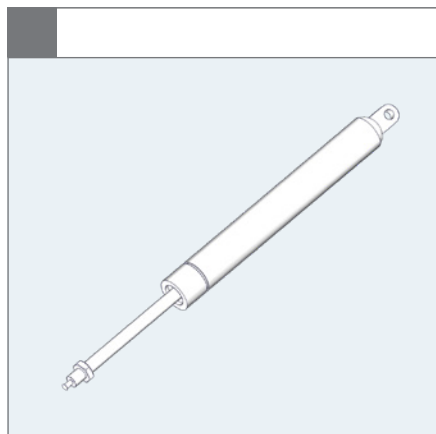
1.

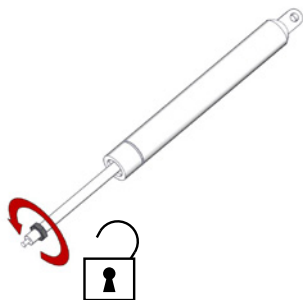17 mm


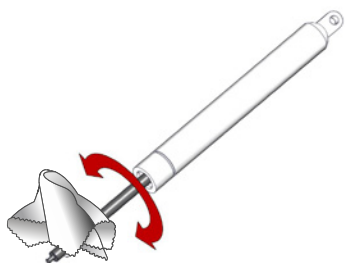
2.



3.

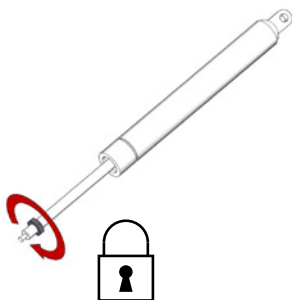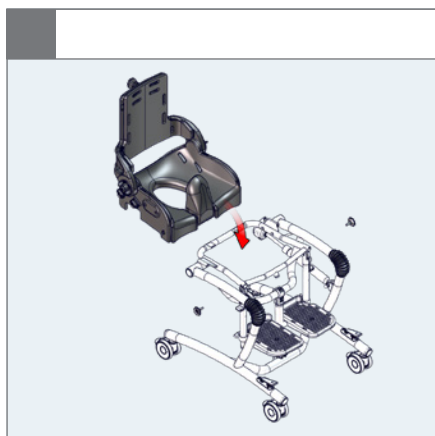17 mm


		 Size 1	 Size 2	 Size 3	 Size 4
 Size 1		✓	-	-	-
 Size 2		-	✓	-	-
 Size 3		-	-	✓	-
 Size 4		-	-	-	✓

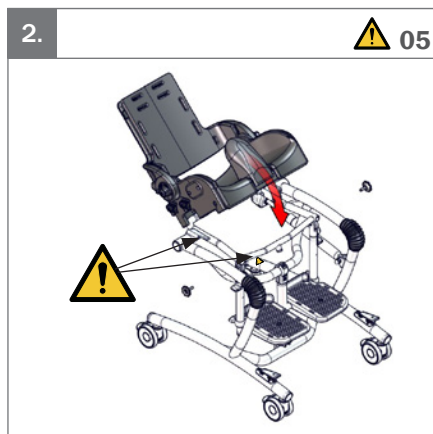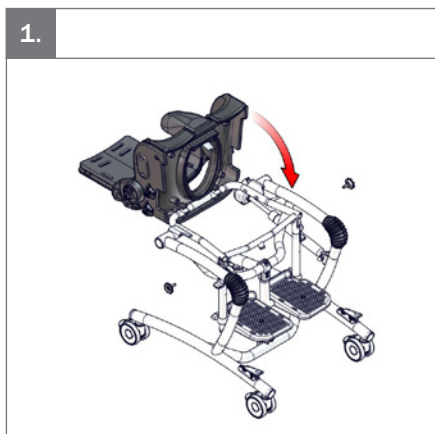
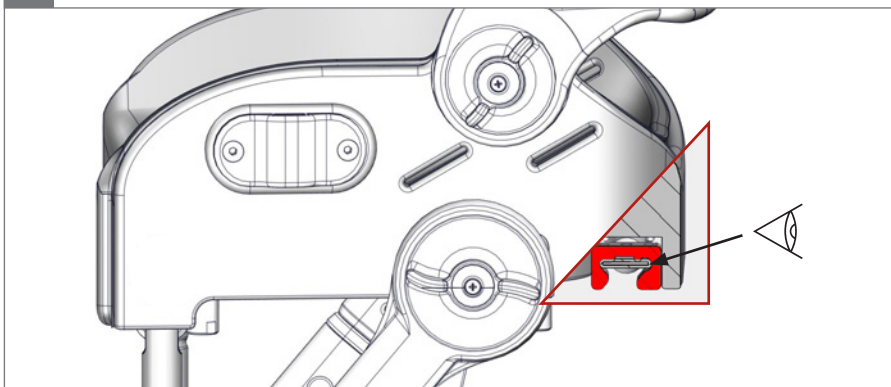
3.

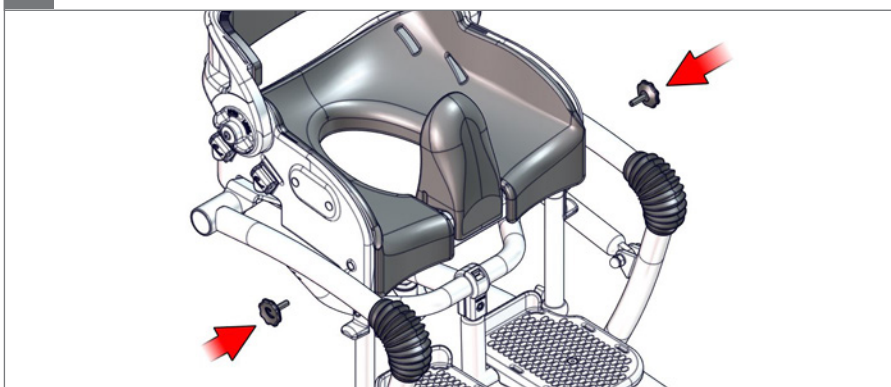4.





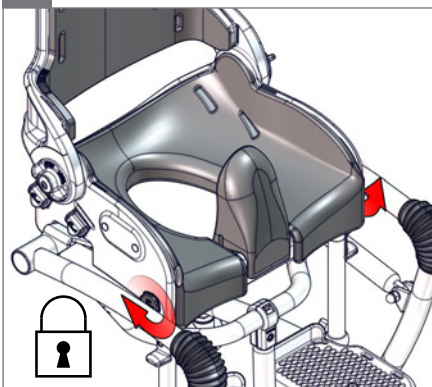
5.

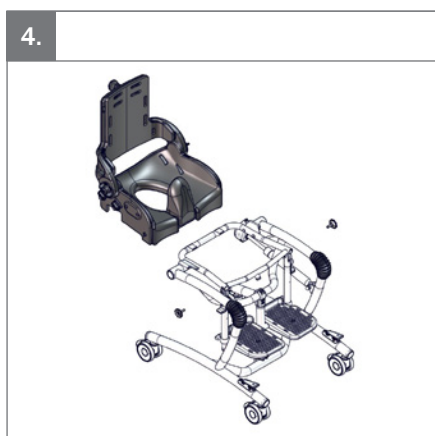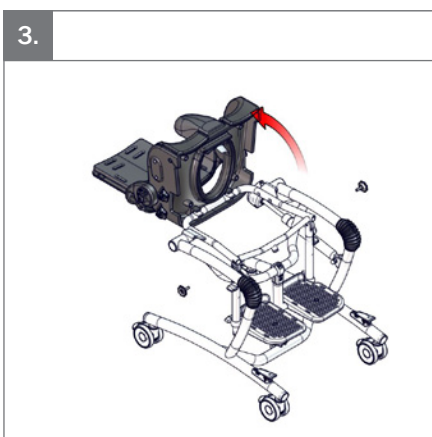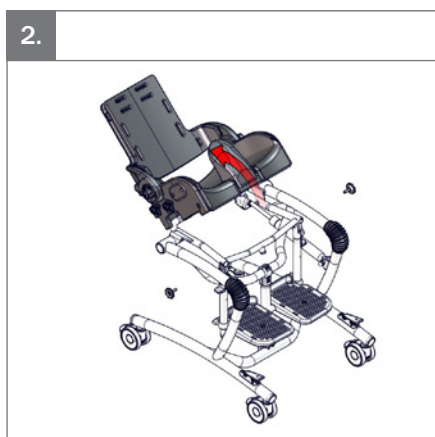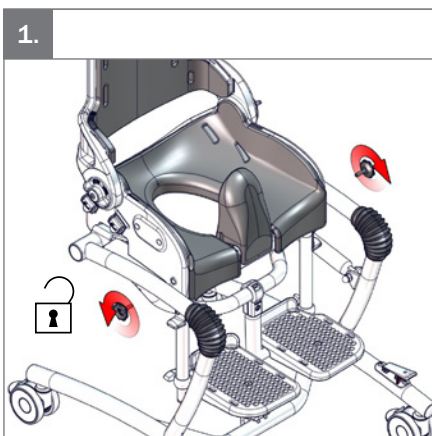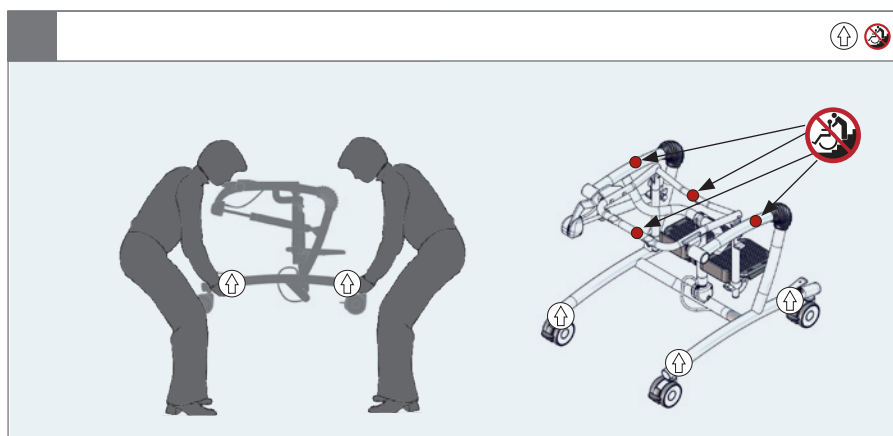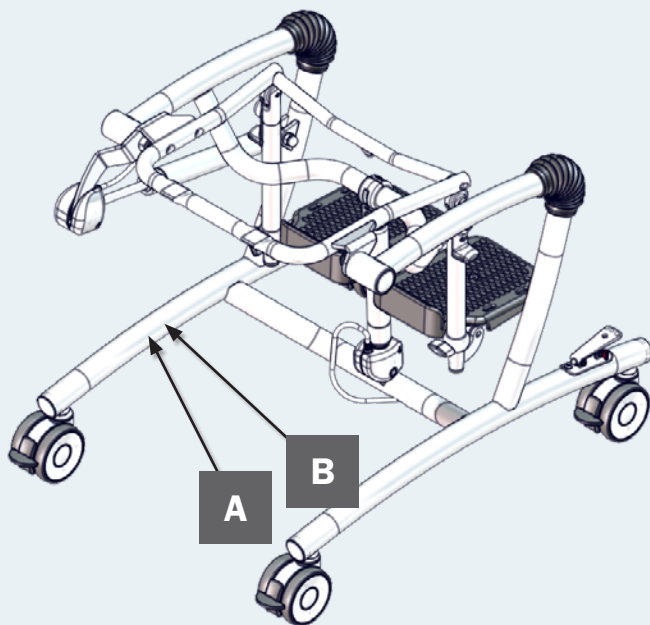
6.

A

R82
by Etac

R82 A/S, Parallelvej 3
DK-8751 Gedved
etac.com
Made in xxxxxxx

MD

CE UK CA

YYYY-MM-DD

B

Posting group
Product name, size x

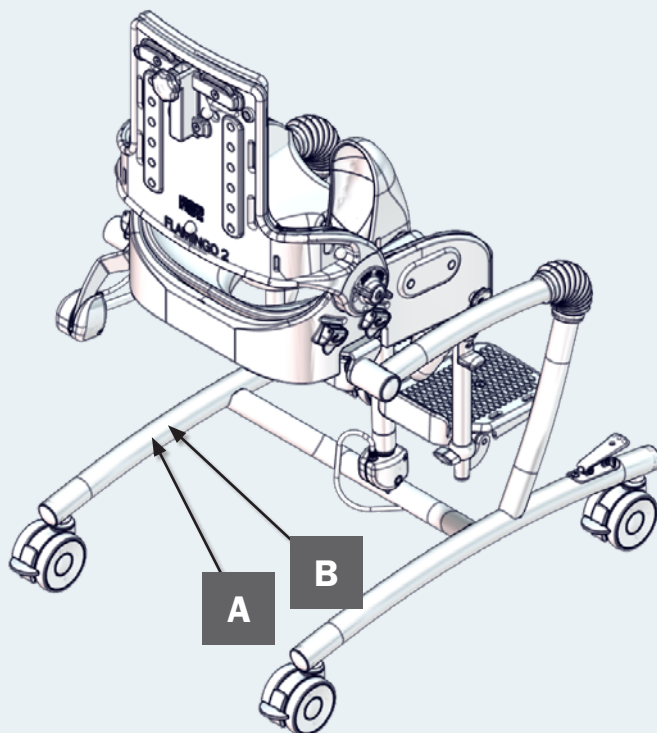
Max. load: xxx kg
Max. rated load: xxx kg
Mass device: xxx kg

DMR ver.: xxxx

SN xxxxxxxxxxxx

REF xxxxxxxxxxxx

(XX)XXXXXXXXXX(XX)XXXXXX(XX)XXXXXX

A

R82
by Etac

R82 A/S, Parallelvej 3
DK-8751 Gedved
etac.com
Made in xxxxxxx

MD

CE UK
CA

YYYY-MM-DD

B

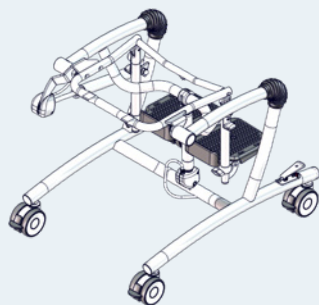
Posting group
Product name, size x

DMR ver.: xxxx

Max. user mass: xxx kg
Max. rated load: xxx kg
Mass device: xxx kg

SR xxxxxxxxxxxx
NEP xxxxxxxxxxxxxxxxxxxx

Barcode: (XX)XXXXXXXXXXXX(XX)XXXXXX(XX)XXXXXX

EN: Warning: This symbol appears in the User Guide along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger
 01 = Entrapment Hazards
 02 = Tilting Hazards
 03 = Transportation in motor vehicles
 05 = Squeezing Hazard

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
mm (inch)	1	2	3	4
 Overall width, seat	350 mm (13¾")	380 mm (15")	430 mm (17")	480 mm (19")
 Length, seat	260 mm (10¼")	315 mm (12½")	360 mm (14¼")	425 mm (16¾")
 Overall height, seat	400 mm (15¾")	430 mm (17")	480 mm (19")	510 mm (20")
 Effective seat width	240 mm (9½")	280 mm (11")	320 mm (12½")	340 mm (13½")
 Effective seat depth	250 mm (9¾")	290 mm (11½")	350 mm (13¾")	400 mm (15¾")
 Back width	200 mm (7¾")	220 mm (8¾")	260 mm (10¼")	280 mm (11")
 Backrest height	350 mm (13¾")	380 mm (15")	420 mm (16½")	470 mm (18½")
 Folded height, seat	240 mm (9½")	260 mm (10¼")	270 mm (10¾")	270 mm (10¾")
 Folded length, seat	370 mm (15")	400 mm (15¾")	430 mm (17")	460 mm (18")
 Backrest angle	30°	30°	30°	30°
 Overall width	500 mm (19¾")	500 mm (19¾")	570 mm (22½")	570 mm (22½")
 Length	640 mm (25¼")	640 mm (25¼")	760 mm (30")	760 mm (30")
 Overall height	860–1080 mm (33¾–42½")	920–1110 mm (36¼–43¾")	940–1170 mm (37–46")	1020–1200 mm (40¼–47¼")
 Seat surface height at front edge	530–740mm (20¾–29¼")	530–740mm (20¾–29¼")	530–740mm (20¾–29¼")	530–740mm (20¾–29¼")
 Footrest to seat distance	200–250 mm (7¾–9¾")	230–340 mm (9–13½")	300–420 mm (11¾–16½")	300–420 mm (11¾–16½")
 Leg to seat surface angle	90°–75°	90°–75°	90°–75°	90°–75°
 Seat plane angle	(-5°)–(20°) *	(-5°)–(20°)	(-5°)–(20°)	(-5°)–(20°)


* The -5° tilt is obtainable at 550 mm seat height above floor



kg (lb)	1	2	3	4
 Mass device, frame	11.5 kg (25.3 lb)	11.5 kg (25.3 lb)	12.5 kg (27.5 lb)	12.5 kg (27.5 lb)
 Mass device, seat	2,5 kg (5.5 lb)	3 kg (6.6 lb)	4 kg (8.8 lb)	4.5 kg (9.9 lb)
 Mass device	14 kg (30.8 lb)	14.5 kg (31.9 lb)	16.5 kg (36.3 lb)	17 kg (37.4 lb)
 Max user mass	35 kg (77.1 lb)	50 kg (110.2 lb)	60 kg (132.2 lb)	70 kg (154.3 lb)
 Max. load, operating force	35 kg (77.1 lb)	35 kg (77.1 lb)	55 kg (121.2 lb)	55 kg (121.2 lb)
 Max rated load	51 kg (112.4 lb)	66.5 kg (146.6 lb)	78.5 kg (173 lb)	89 kg (196 lb)

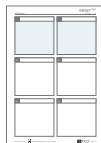9996097632

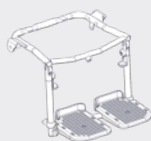9996097372

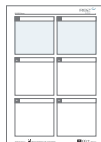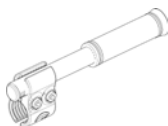9996097112

9996097375

ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User manual before use and save it for future reference.

Intended purpose

The High-low Bath Frame is a toilet/bathing frame with user friendly angle and height adjustment. The High-low Bath Frame is designed for Flamingo seats and can be placed over the toilet or used freestanding with a pan. The product is for indoor use in a bathroom environment.

Enclosed tools:

- 3+4+5 mm Allen key

CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12182

When using the product outdoors, then the product must be equipped with handbrakes. Handbrakes are optional and accessories for the product. The width of size 4 (700 mm) does not meet the requirement for maximum width of 650 mm for use in private homes.

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities

to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions. Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the User manual carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Transportation of the product whilst the user is in it is not allowed. It is allowed to use the product with the user safely strapped in it for transfers over short distances. Do not cross any obstacles during transfer.

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Ensure the castors are fully-functional and

secure before use

- Activate the brakes before placing the user into the product
- Ensure the stability of the product is maintained before placing the user in it
- We recommend you notify the user before angling the seat and back
- Before moving the chair, with or without a user seated, please ensure the seat is in horizontal position, the back in vertical position. Do not move an angled seat
- Always activate the security locks on the height adjustment and angling pedals once the desired position is found
- Adjust the frame height before placing a heavy user (50-70 kg)
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- Inspect the product and all of its accessories and replace any worn out parts before use
- Be careful not to get your fingers trapped, when placing the product over the toilet

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Reuse

This product and its accessories/components are suitable for reuse.

Refurbishment for reuse

Before reuse or changing the user, follow the service checklist, maintenance and washing instructions and carry out regular inspections.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

Service checklist:

Check and adjust the following as per their intended use:

- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- Adjustment levers for seat angle and height adjustment
- Adjustment levers lock
- Adjustment options for the foot rest, it is possible to lock the angle and fasten the telescopic tubes in position
- Castors are moving freely and all wheel locks/brakes are easily activated
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Check all fittings and buckles for signs of worn out parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to:

Wipe off any residue and dirt from the product,

using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use.

Carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

This product can be washed at 85° using a mild detergent for 3 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Materials

- Aluminium
- Rubber
- Plastic
- Stainless steel
- Fabric

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

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**Etac Ltd.**
**Gelbart AG**