

# Integrated Charger

## Accessory Mobile Lifters

**molift**<sup>®</sup>  
a part of Etac

BM03201 Rev A 2012-08-20

### Integrated Charger

The lifter can be charged without removing the battery from lifter with integrated charger. It can be factory mounted as an accessory to all Molift mobile lifters.

### Charging

Only Power cable should be disconnected from either wall power socket or from the charger when the lifter is not charging.


Do not disconnect DIN plug adapter. If it is disconnected, make sure it is reconnected careful and correctly.

### Charger Light

When the charger is connected to a power source such as a wall socket, the charger light will illuminate.

Description of charger light:

Charger light	Mode
Yellow	Ready for use, no battery connected
Yellow	Initialization
Red	Fast Charge
Green/Yellow	Top-off charge
Green	Trickle charge
Red/Green	Error

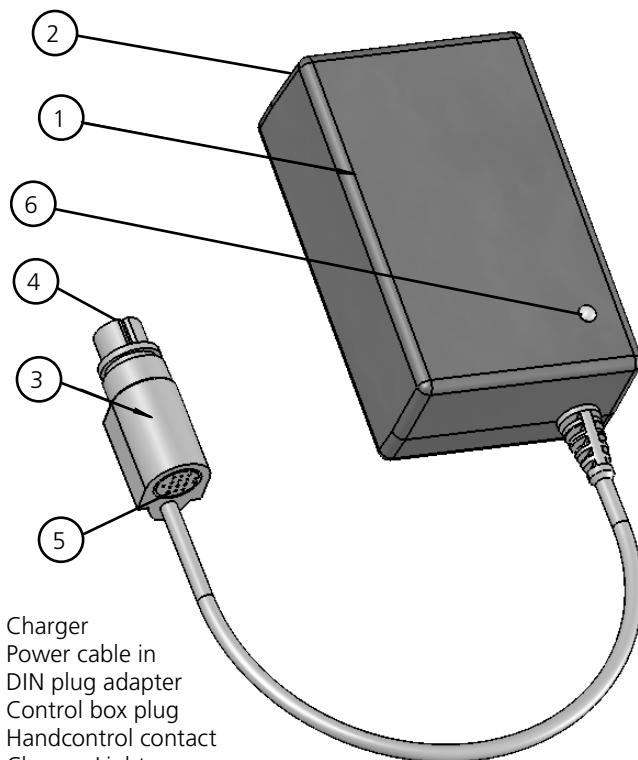

## Integrated charger on lifters



Mover 205  
Mover 300  
Partner 255



Mover 180  
Smart 150



Quick Raiser 1  
Quick Raiser 2  
Quick Raiser 2+

Etac AS,  
Etac Supply Gjøvik  
Hadelandsveien 2,  
2816 Gjøvik, Norway  
Tel +47 4000 1004  
molift@etac.com www.molift.com