

Molift Quick Raiser sandals

molift[®]
by Etac

EN - User manual

BM04601 Rev. D 2021-06-11



English manual

Contents

Important	1
About Molift Quick Raiser sandals	2
Mounting of Molift Quick Raiser sandals	2

Important

This user manual contains important safety instructions and information regarding the use of the lifter and accessories.

In this manual the user is the person being lifted. The assistant is the person operating the lifter.



Warning!

This symbol indicates important information related to safety. Follow these instructions carefully.



Read user manual before use!

It is important to fully understand the content of the user manual before attempting to use the equipment.

Visit www.etac.com to download documentation to ensure you have the latest version.

Declaration and warranty



CE declaration

The product, and its related accessories, described in this instruction for use (IFU), is in compliance with the regulation (EU) 2017/745 of 5. April 2017 – as a medical device, risk class I. Any serious incident that occurs in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Warranty notice



**Two-year warranty against defects in workmanship and materials of our products.
One-year warranty for batteries.
Please refer to www.etac.com for terms and conditions.**

Molift Quick Raiser sandals

About Molift Quick Raiser sandals

Molift Sandals is designed for use together with Molift Quick Raiser.



Read the operator manual carefully before use of lifter with accessories. All safety regulations applies.



Sandals are only for minor support of user's feet. User must have full control of leg mobility.

The sandals come with velcro tape mounted on the underside.



Mounting of Molift Quick Raiser sandals

Remove the anti-skid tape from the Quick Raiser on which the sandals shall be mounted.



Remove the tape from the underside on the velcro on the sandals.



Place the sandals on the chassis. The positioning may vary for different users.



Molift Quick Raiser sandals

Place the other sandal on the opposite side. Press both sandals against the chassis to ensure they are properly fastened.

The sandals can be removed and replaced, only the hard side of the velcro will remain on the Molift Quick Raiser chassis.



The sandals is now ready for use.



Etac A/S
Parallevej 3
DK-8751 Gedved
www.etac.com

molift[®]
by Etac