

Hard Case for Molift Smart - Part no 0990200



CE
- an accessorie from






Hard Case for Molift Smart

Fitting the Molift Smart in the hard case

- Fold the lifter completely as described in the lifter's operator manual.
- Remember to put the suspension in the hook and the cord around the legs and castors.
- Pick the lifter up and place it into the hard case upside down aligning the wheels of the lifter base and the hard case wheels.
- Make sure there are no loose parts, place any spare batteries and charger in a safe way, preferably in bottom section of lifter.

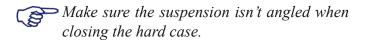

• Close the hard case, push the locks together.

Spare Parts

- 0990201 Wheel (2 3/4")
- 0990202 Curved Handle









