

Molift SmartBag

User manual

molift[®]
a part of Etac

BM09401 Rev. A 13.12.2016



CE

Packing

The Molift SmartBag is developed to easily transport the Molift Smart 150, in a car, on the train, an airplane or just for stowing the Molift Smart 150 when it is not in use.

1 Fold the Smart 150 according to the User manual, and place it in an upright position.



2 Open the Velcro on the Molift SmartBag and unfold.



3 Control that both the front wheels of the Molift Smart 150 is turned as shown on the picture



4 Pull the SmartBag over the Smart 150, with the grey side against the pushbar



Packing

5 Secure that the SmartBag is gently pulled over the two upper points of the suspension



6 Put the SmartBag with the Smart down on the floor



7 Turn and position the wheels on the Smart 150 as shown and activate the wheel brakes



8 Hold the column of the Smart 150 and pull the lid of the SmartBag forward. Lower the Smart 150 down into the bag



Packing

9 Place the battery and the battery charger in the lid



10 Close the bag



11 You are ready to go!



12 It is possible to carry the bag for two persons using the handlers in the bottom of the SmartBag



Unpacking

1 Put the SmartBag with the Smart 150 down on the floor



2 Open the zipper of the lid



3 Remove the battery and the battery charger from the lid



4 Hold the column of the Smart 150 and push the lid of the SmartBag under the bag.



Unpacking

5 Raise the bag with the Smart 150 to an upright position



6 Pull up the bag carefully and remove from the Smart 150



7 Roll up the SmartBag and secure with the velcrostrap, and...



8 You are ready to go!



Etac AS
Etac Supply Gjøvik
Hadelandsveien 2, 2816 Gjøvik, Norway
Tel +47 4000 1004
molift@etac.com www.etac.com

molift[®]
a part of Etac