
User manual

Molift Nomad

molift[®]
by Etac

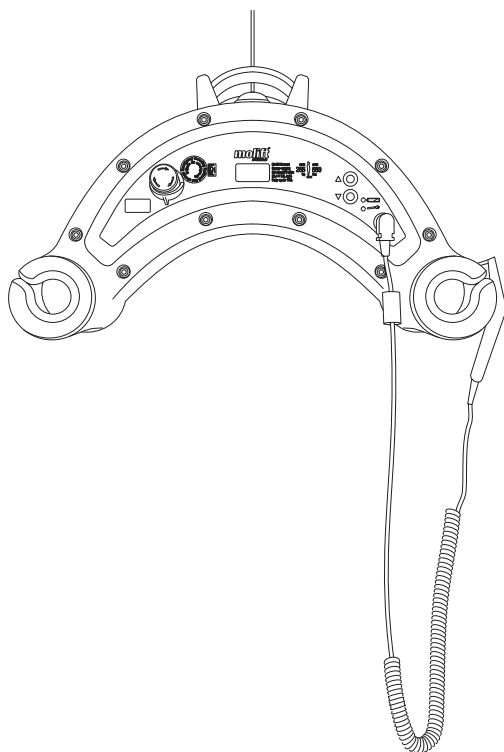BM15199 - 2026-08-12 - Rev. 1.0

www.etac.com

	Symbols	3
--	----------------	----------

	Device overview	9
--	------------------------	----------

	General	12
	English	12
	Svenska	18
	Norsk	24
	Dansk	30
	Suomi	36
	Deutsch	42
	Nederlands	49
	Français	56
	Italiano	63
	Español	70

	Handling	77
	Assembly	78

	Warnings	79
---	----------	----

	Information	80
---	-------------	----

	Checklist	81
---	-----------	----

	Emergency stop	83
---	----------------	----

	Information	84
---	-------------	----

	Rising	86
---	--------	----

	Lowering	88
---	----------	----

	Warnings	89
---	----------	----

	Information	91
---	-------------	----

	Room to room	92
---	--------------	----

	Warnings	94
--	----------	----

	Information	95
---	-------------	----

	Technical data	97
---	-----------------------	-----------

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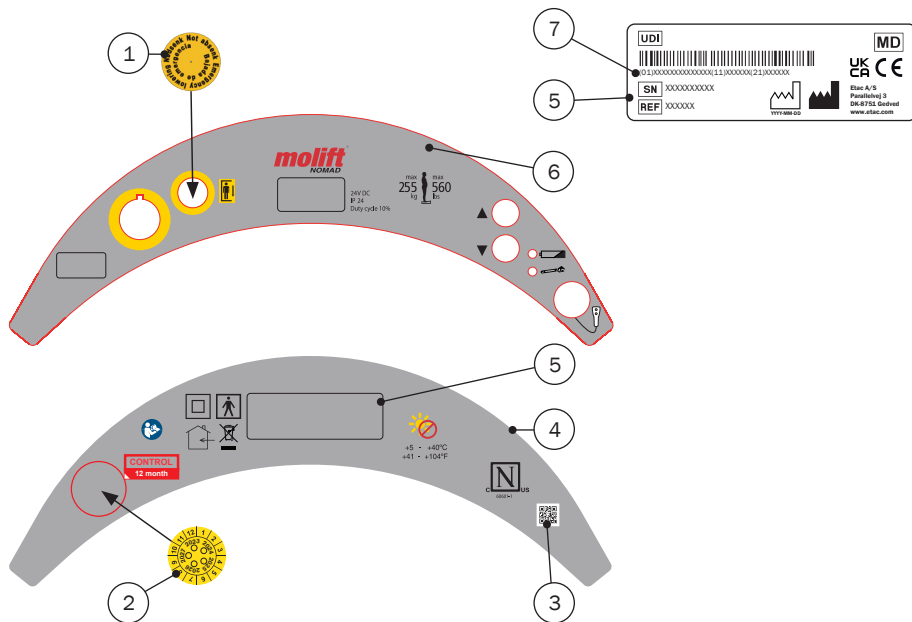
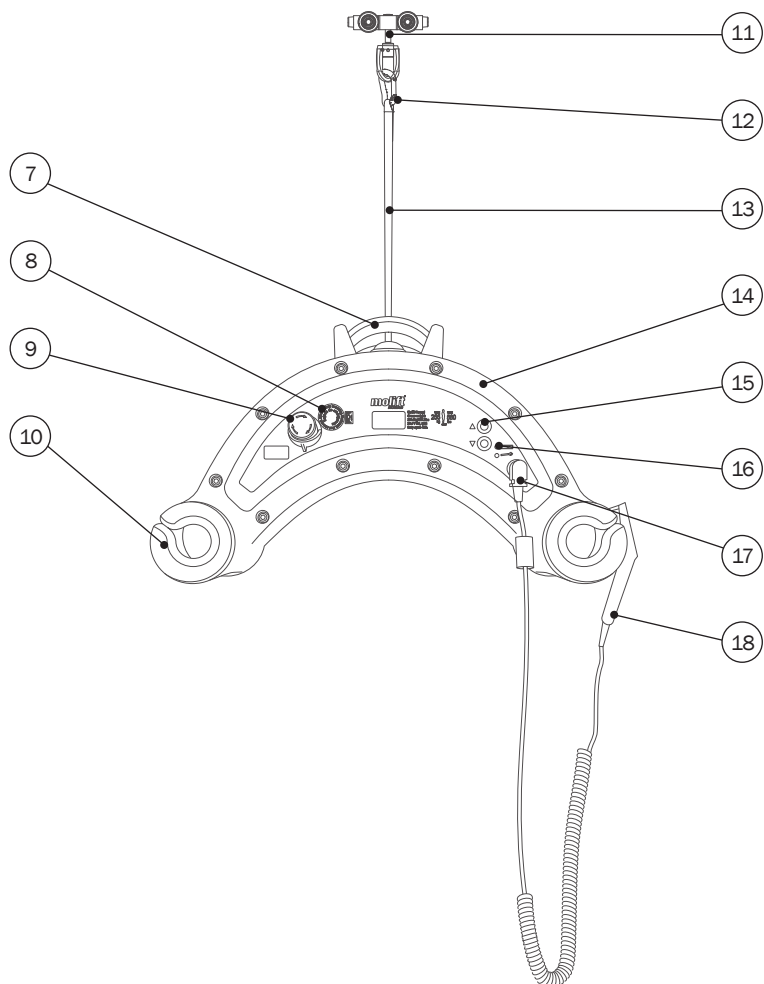
General

Thank you for choosing an Etac device.

This manual must be read thoroughly before you start using the device as it contains important safety information and gives instructions on how to use the device correctly. Visit www.etac.com for all relevant product documentation, for example the user manual, maintenance and recycling instructions. You can select your language via the “International” and “Local websites” link.

You can also consult the QR code on the device for further information and instructions.

Copyright

This document is the property of Etac A/S. The content shall not be modified in whole or part without prior approval of Etac A/S.

Disclaimer

Etac continuously makes improvements to our devices and information. We therefore reserve the right to modify our devices and instructions for use without prior notice. Consult www.etac.com to find the latest version of the documentation.

CE Declaration of Conformity

The device mentioned in this manual and the different models hereof comply with the Medical Device Regulation (EU) 2017/745.

The device is classified as a Class I medical device.

Product standards

The device has been tested and complies with the requirements applicable for this type of device in the below mentioned standards:

- ISO 10535:2021 Assistive products – Hoists for the transfer of persons – Requirements and test methods.
- IEC 60601-1:2005 Medical electrical equipment – Part 1: General requirements for basic safety and essential performance.

- IEC 60601-1-2:2014 Medical electrical equipment – Part 1-2: General requirements for basic safety and essential performance – Collateral Standard: Electromagnetic disturbances – Requirements and tests.

The product has been tested and approved by a third party according to standards IEC 60601-1, IEC 60601-1-2 and EN/ISO 10535:2021.

Warranty

Please refer to www.etac.com/support.

If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

Notice

Any serious incident that has occurred in relation to the device should be reported to your distributor/representative and the national competent authority within your region in a timely manner. The distributor/representative will forward the information to the manufacturer.

Pre-sales information

Please visit www.etac.com.

Device description

Etac supplies a wide selection of Ceiling Hoists for different types of transfer situations. Molift Nomad is a light-weight portable ceiling hoist intended for hoisting and transferring a person from/to bed, floor chair, wheelchair or toilet with a sling.

Molift Nomad is available in different variants.

Molift Nomad key features:

- Integrated lifting handle
- Electrical emergency lowering
- Integrated 4-point sling bar
- Powered by NiHM battery

See www.etac.com for a complete overview of possible variants.

Clinical benefits

The clinical benefits of the devices are:

- Reduced load or strain on carer.
- Reduced risk of work-related injury.

Materials

Aluminium, plastic, steel.

Introduction

Definitions

The person referred to as “the user” is the person who is transferred in the device. “The carer” is the person who manages the device.

Intended purpose

The Ceiling Hoists are assistive devices intended for alleviation of or compensation for a functional impairment due to an injury or disability. The devices are designed for an individual lacking the ability to stand up, reposition, move and/or transfer themselves due to reduced mobility or physical strength.

Intended user

The intended users of the devices are based on the users' functional ability and not on a specific diagnosis, health condition or age. Ensure the user complies with the SWL/user mass and that a clinical risk assessment is conducted.

No limitation in body height – for specific recommendation see size guide in the user manual for the relevant sling.

Secondary users of the devices are carers providing assistance and clinicians/technicians who set up the device.

Intended environment

The devices are intended for indoor use within a rail system in acute care, long term care and home care.

Intended application

The device is intended for ‘Transient’ use, meaning continuous use for less than 60 minutes.

The device is intended for use with intact skin only. The device is intended for refurbishment and re-use.

The device is intended for transfers to/from a bed, wheelchair, a chair, or similar.

Expected service life

The expected service life of Molift Nomad is 30.000 cycles or 10 years, whichever comes first.

Indications

Indication for use is disability, including but not limited to, users who are in need of support to be hoisted, transferred and/or mobilised. E.g. users with limited ability to move, balance and/or weight-bear on their legs.

Not an exhaustive list.

Contraindications

There are no known contraindications.

Precautions

Make sure the sling fits the user and the sling bar.

Training Requirements

Only carers who have read and understood the user manual are allowed to use the device.

Safety information

The safety of both user and carer must never be jeopardized – if you suspect that the device has a malfunction you must put it in quarantine marked “out of order” and call your local distributor/representative.

Safety signs, safety measures and warning symbols are given in the context relevant for the actual handling and/or operating situation – it is mandatory the given instruction is followed to avoid dangerous/hazardous situations.

Signs, symbols and instructions placed on this device must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the device. Immediately replace or repair illegible or damaged signs, symbols, and instructions. Contact your distributor/representative for instructions.

See additional warnings in the Handling section.

Patient transfer

Transfer of a user will always pose a certain risk. Therefore, a risk assessment must always be carried out before the transfer is performed.

Warnings:

- Never use the device for transportation of goods.
- Never leave a user unattended in the device.
- Never use the device in the shower.
- Always transfer the user at a calm and stable speed.
- Never transfer the user in a standing position.

Risk assessment

Do a risk assessment of the device, the individual user and the transfer situation, and document your analysis. You as the carer are responsible for the safety of the user.

Plan the transfer in advance to ensure that it is performed as safe and smooth as possible.

Remember to work ergonomically according to local recommendations.

The carer must be able to handle the weight of the user.

To avoid serious injury, the user shall avoid extended reaching or leaning movements.

The device

Always check – before each use:

- that the batteries are sufficiently charged (no light on hand control or hoist).
- that the service light is green, meaning no service is needed.
- that the device has no visible damage.

that the hoist is properly attached to the trolley and that the safety hook is locked.

that the device is suitable for the user and the transfer situation.

that the sling does not have visible damage or frays.

Inspection to be performed monthly:

- run the hoist through the complete installation, and check that it is running smoothly.
- verify that the hand control contact is firmly in place.
- check that the hand control cable has no damage.
- check that the lifting band has no damage. Remove dirt and dust from opening and check that the lifting band moves out and in easily.
- run hoist up and down to verify that it moves normally without any abnormal noise.

If the inspection reveals any safety risks, the device must immediately be taken out of service. Contact local service representative.

Safety measures

Molift Nomad is equipped with several safety devices to prevent personal injury or damage to equipment in case of incorrect use:

- An overload sensor prevents the hoist from hoisting if the weight of the user exceeds the SWL.
- A sensor will stop the hoisting if the lifting band is not tight, to avoid it curling up.

Materials

All used materials which are to be in direct contact with skin are biologically assessed and evaluated according to relevant standards. However, users/carers who are allergic to any of the materials are recommended not to use the device.

Accessories

User manuals supplied with accessories must be read and understood before use. This includes any limitations in use and combinations.

Information about accessories can be found on www.etac.com.

Spare parts

A list of spare parts is available on request. Contact your local service representative/dealer.

Device combinations and sling compatibility

When combining a medical device with other devices or accessories, the combination must not change the by manufacturer defined intended purpose of the devices or modify the devices in such a way that compliance with applicable requirements, or safe use of the device are affected.

We recommend the specified combination of Etac/Molift hoists and slings.

For combination with other manufacturers, visit www.etac.com for compatibility statements.

The person/organization responsible for the combination of the devices must ensure that the applicable requirements are fulfilled.

Any organization that purchases hoists and body-support units shall make sure, that the combination(s) are safe either by requiring compatibility documentation for the combination(s) from the manufacturer(s) or by performing compatibility testing themselves hereby transferring the responsibility for a safe combination to the organization.

Electric protection in combination with slings: Type BF Applied part.

If safe working load (SWL) differs between rails, hoist and sling, the lowest load should always be used.

The hoist is marked with Safe Working Load (SWL), this should not exceed rail systems max load capacity.

Electronics

Battery

Molift Nomad has a 26.4 V NiMH 2.6 Ah battery that is placed inside the chassis.

Recharge battery upon receipt, and then every six months, to avoid battery failure.

If the battery is discharged during a transfer, it has always enough power to lower the user again.

Used batteries must be disposed of as special waste according to local rules and regulations. Do not dispose in general waste.

The electrical system has a power save function which will turn off the electrical system 10 minutes without activity. All lights will turn off. The system is activated when pushing one of the operating buttons.

Motor

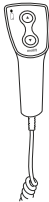
Motor: 24 V DC.

Lifting speed: Up: 50 mm/sec (75 kg). Down: 60 mm/sec without load.

Duty cycle

Molift Nomad should not run continuously for more than 2 minutes (with max load). After 2 minutes the hoist needs to rest for 18 minutes. The device has a maximum A-weighted sound power level: Lwa = 68.8 dB.

Hand control - Protection class: IP24

Overview	Icons	Description
		Battery light
		Operating buttons up/down


If the lifting band becomes loose during lowering, the hoist will automatically stop to prevent the band from being pulled in incorrectly. To re-tighten the lifting band, press the "Up" button.

Description of battery light

Battery light	Mode
No light	Ready for use
Green light	Battery must be charged
Red light	Charge battery immediately
Red light + sound	Charge battery immediately

If no hand control is connected, the battery light on the hoist will turn on when the battery level is low.

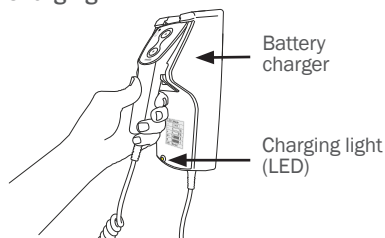Charging

The charger can be placed on the wall or a table and connected to the power outlet. The Charger must be placed or installed in a way that makes it easy to disconnect mains cable.

Description of charging light

Charging light	Mode
Yellow	Ready for use, no battery connected
Yellow	Initialization
Orange	Fast charge
Green/yellow	Top-off charge
Green	Trickle charge
Orange/green	Error

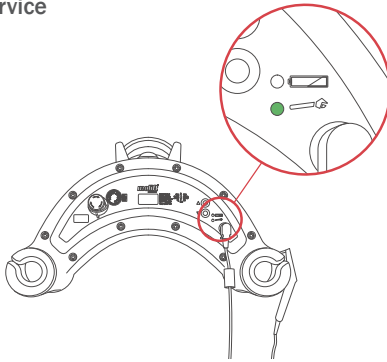
The hoist will not operate while charging.

If the emergency stop button is activated the batteries will not charge.

The battery will not achieve full capacity before it has been fully charged and discharged a couple of times.

The hoist can be operated without hand control. The hoist is then equipped with a special charger with a contact to be plugged in the hand control contact on the hoist when charging. Both chargers have the same functions.

Service

Service light codes:

Service light	Mode
No light	Power saving (stand by)
Green	No service is needed (ready for use)
Yellow	Order service
Red	Perform service
Red + sound	Perform service immediately
Flashing yellow	Overheating – the hoist will stop. Yellow flashing will stop when the hoist has cooled down sufficiently to operate again.

If no light is visible, press the up/down button to activate the electronics. The service lamp will then turn on.

Maintenance and cleaning

Maintenance, periodic inspections, and service of the device is the responsibility of the owner of the device.

Failure to maintain a device in accordance with the instructions may compromise the safety of the user and/or carer and may affect the expected lifetime of the device.

All maintenance, periodic inspections and service shall be stored in a logbook kept by the owner.

Periodic inspection scope

Periodic inspection shall be performed annually or more frequently if required by local requirements. See periodic inspection checklist at www.etac.com.

Service scope

Service must be carried out by authorized personnel. Service is needed when service LED is on. The service LED turns on after a certain number of lifts or after 5 years, whichever comes first.

Cleaning

Clean on a regular basis. Clean surfaces with a damp cloth using an appropriate pH-neutral detergent. Do not use solvents or strong liquids, this may damage surfaces on the hoist. For disinfection when needed; use isopropyl alcohol or similar. Avoid abrasive cleaning products. Clean contact and hand control with isopropyl alcohol to remove grease and dirt. Check emergency stop and emergency lowering after cleaning. The hoist should not be exposed to running water.

Make sure not to damage or remove labels when cleaning.

Transportation, storage and disposal

Transportation and storage

When the device is not in use, it can be stored under the following conditions:

Keep packaging away from children.

Keep away from fire.

Medical electrical equipment requires special precautions regarding electromagnetic compatibility (EMC). Interference may occur in the vicinity of the hoists electronics.

The hoist can be stored and transported under temperatures between -25 to +70 °C and relative humidity 15 - 93%.

Air pressure: 70 - 106 kPa.

The hoist is designed for use at standard room temperatures (+5 to +40 °C). Following storage or transport at other temperatures leave the hoist in a room with a suitable temperature until it reaches a safe operating temperature.

Disposal

The device contains no hazardous substances that require special disposal. Check if there are options available for recycling in an environmentally friendly way and follow the national provisions that apply. Dispose of the device according to the national provisions that apply.

Troubleshooting

Problem	Action
The hoist does not hoist.	Battery discharged. Charge the battery. Lifting band is untight. Tighten lifting band by lifting up. Emergency stop is activated (pushed in). Turn clockwise to deactivate. The hand control cable has disconnected. Reconnect hand control cable to the hoist. The hand control is defective. Use the operating buttons on the hoist and replace the hand control with a new. Defective electronics. Contact local service representative for service. The hoist is defective. Contact local service representative for service.
The hoist moves slowly or makes noise.	Brakes defective. Contact local service representative for service.
The hoist is not charging.	Emergency stop is activated. Reset emergency stop. Connection between charger and hand control is unstable. Check and clean the contact points on the charger and hand control. Connection between the hand control and the hoist is unstable. Ensure the hand control plug is properly inserted into the hoist.

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**EN:** Checklist after assembly

Make sure that:

- the device has no visible damage or missing parts.
- the batteries are sufficiently charged (no light on hand control or hoist).
- the hoist is properly attached to the trolley and that the safety hook is locked.
- the hoist functions correctly by running it up and down.
- if the cause of the failure that triggered emergency lowering is unknown, or if it resulted from a component malfunction, contact your local service representative.

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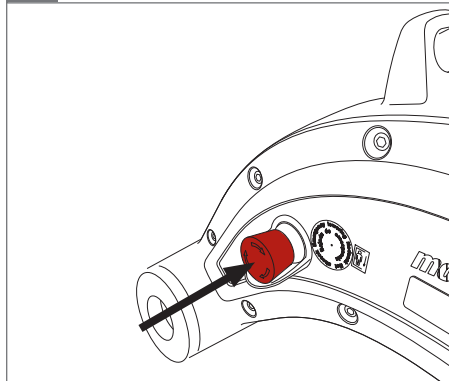
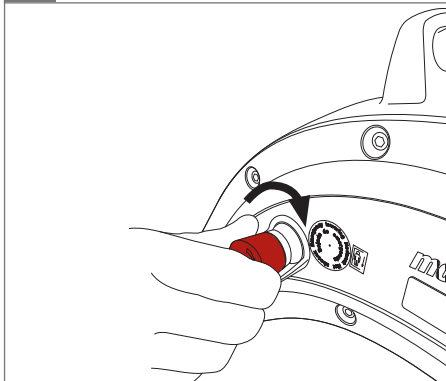
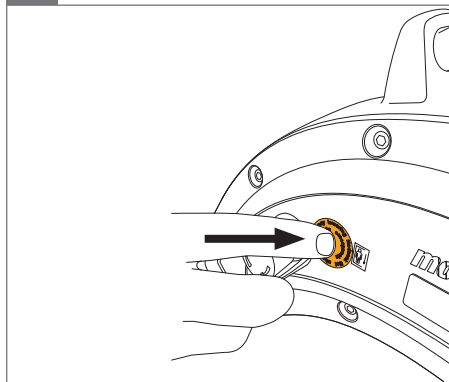
i 04+05+06

EN: Information

01. If the hoist does not move, charge the battery, and try the emergency lowering button again. Do not operate the hoist while it is connected to the charger.
02. Turn the button clockwise to reset.
03. Charging and emergency lowering do not work if the emergency stop is activated.
04. Puncture the seal and press the emergency lowering button. The hoist will make a smoothly movement downwards.
05. If the hoist does not move, charge the battery, and try the emergency lowering button again.
06. If none of the actions works, the user must be lowered manually. Call for assistance.

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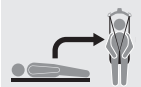
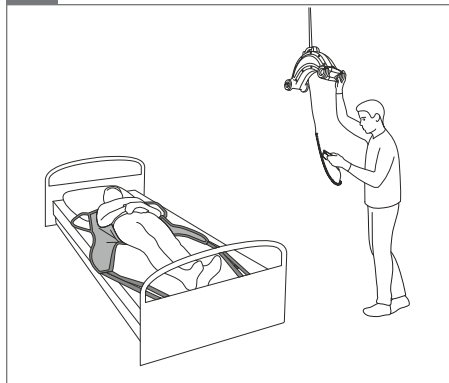
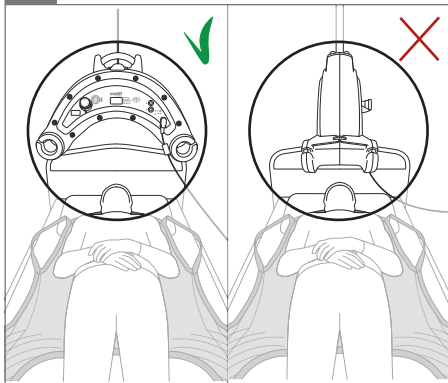
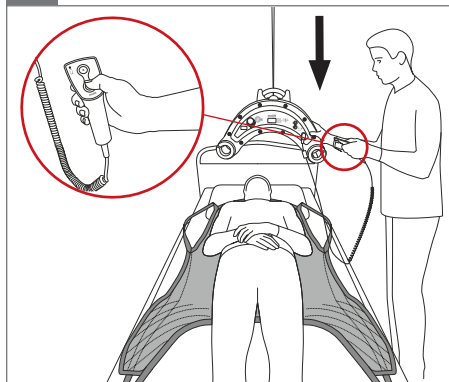
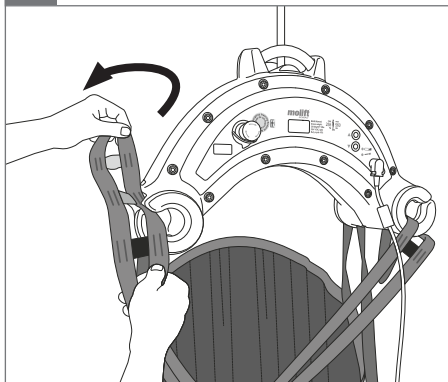
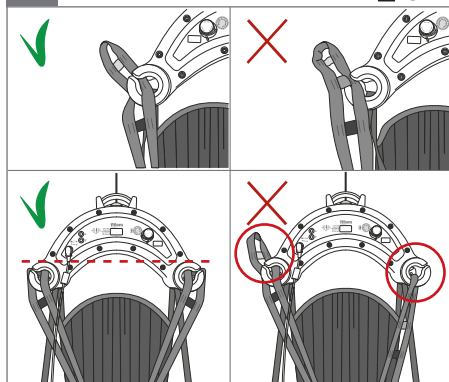
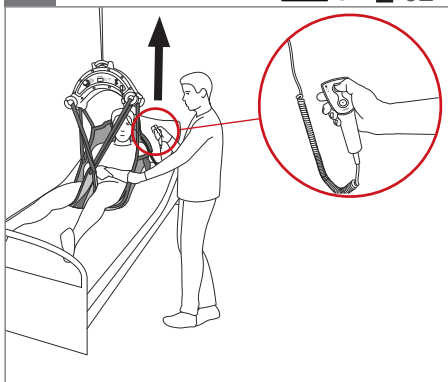
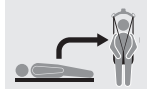
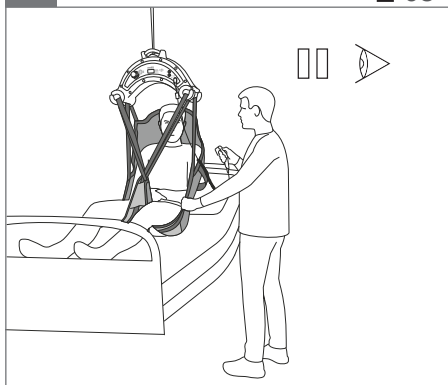
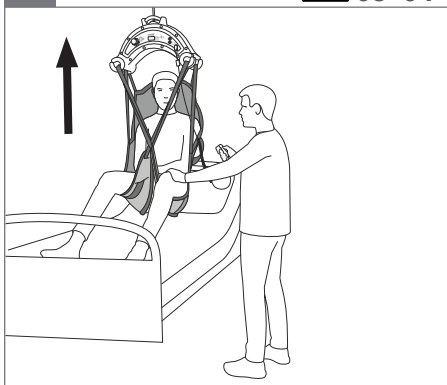
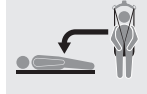
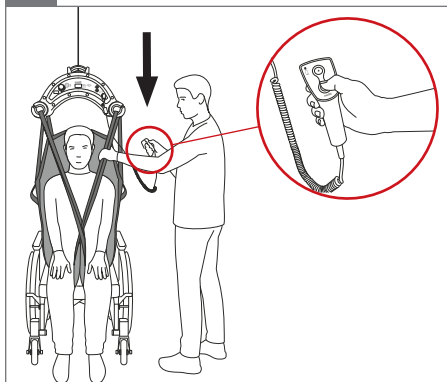
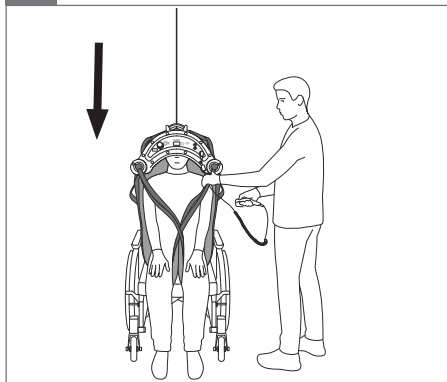
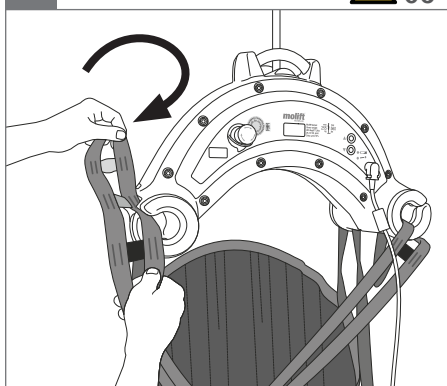
3.

**EN: Warnings**

01. Avoid damaging the device by preventing it from hitting the floor or walls.
02. Check that the lifting band don't curl when raising.
03. Be careful when transferring, the user may swing and hit surrounding objects.
04. Never hoist the user higher than necessary.
05. The hoist shall only be used for movement over short distances. It is not a replacement for wheelchair or similar.
06. Remove the sling gently to ensure the user does not roll out of the bed or wheelchair.

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EN: Information

01. When transferring from room to room, make sure a Molift Nomad trolley for climbing is installed.
02. Ensure a lifting strap with a safety hook is installed in the room you plan to transfer into.
03. Install the safety hook with the opening facing opposite the lifting band.
04. Make sure the lifting band is secured.
05. Ensure that the user does not hit the floor, and make sure the strap length is correctly. The lifting band exists in three different length.
06. Extend the lifting band by pulling it out of the Nomad.
07. Use the extension arm (accessory) to move the lifting band.
08. Take of the extra lifting band from the Nomad and leave it hanging.

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**Etac Ltd.**