
Molift UnoSling Toilet

molift[®]
by Etac


User manual

BM30299 Rev. A 2022-11-01

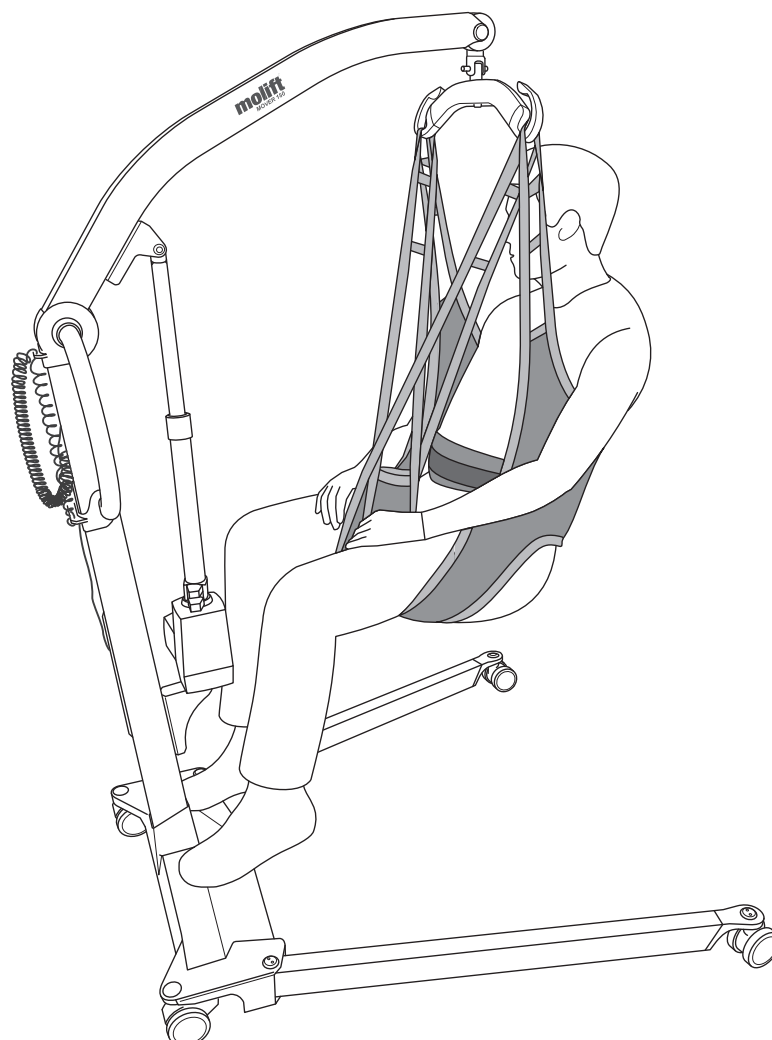MD

CE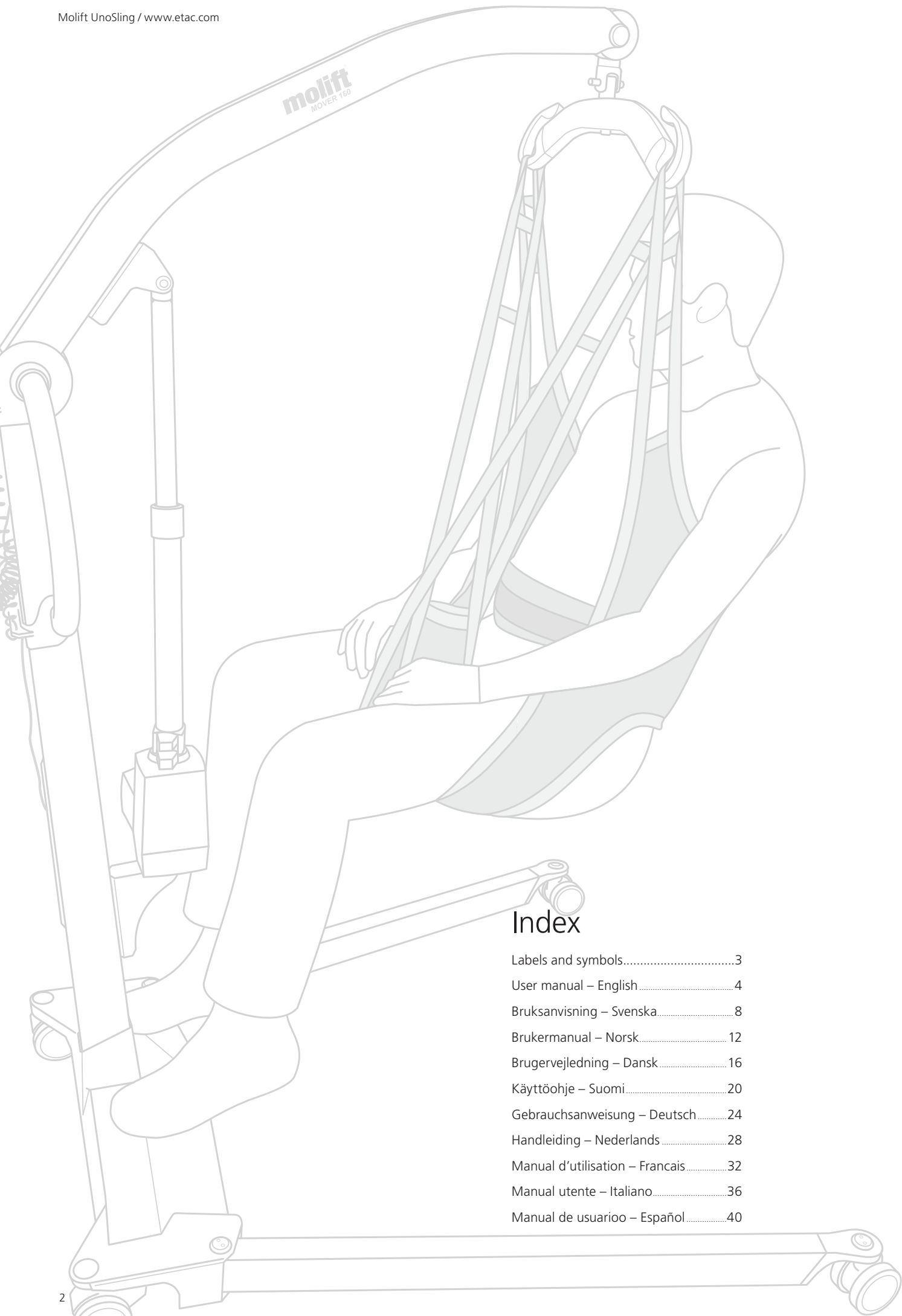

Index

Labels and symbols.....	3
User manual – English.....	4
Bruksanvisning – Svenska.....	8
Brukermanual – Norsk.....	12
Brugervejledning – Dansk.....	16
Käyttöohje – Suomi.....	20
Gebrauchsanweisung – Deutsch.....	24
Handleiding – Nederlands.....	28
Manual d'utilisation – Français.....	32
Manual utente – Italiano.....	36
Manual de usuario – Español.....	40

Labels and symbols/ Etiketter och symboler/ Etiketter og symboler/ Mærkater og symboler/ Tarrat ja symbolit/ Beschriftungen und Symbole/ Labels en symbolen/ Étiquettes et symboles/ Etichette e simboli/ Símbolos y etiquetas:

Molift UnoSling Toilet
 A patient specific, disposable sling
 Art. no: xxxxxxx
 DK HMI: xxxxxx

Size: XX

Material: Polypropylene, Polyester

Barcode according to GS1-128
 (01)xxxxxxxxxxxxxx(11)xxxxxx(21)xxxxxxxx
 (01):EAN number (11)YYMMDD(21):Serial number

Name: _____

UP/UPP OUT/UT

YYYYMM-DD

1 1

Date of first use

Year	2022	2023	2024	2025	2026	2027
Month	01	02	03	04	05	06
	07	08	09	10	11	12

Etac A/S
 Parallevej 3
 DK-8751 Gedved

MD

CE UK

M

Symbols:

 EN - Manufacturer
 SV - Tilverkare
 NO - Produsent
 DA - Producent
 FI - Valmistaja
 DE - Hersteller
 NL - Fabrikant
 FR - Fabricant
 IT - Produttore
 ES - Fabricante

 EN - This Way up, this side is outside
 SV - Denna sida upp, denna sida utåt
 NO - Denne siden opp, denne siden ut
 DA - Denne side op, denne side er ydersiden
 FI - Tämä puoli ylöspäin, tämä puoli ulospäin
 DE - Hier oben, diese Seite ist außen
 NL - Deze kant boven, dit is de buitenkant
 FR - Vers le haut, côté extérieur
 IT - Lato alto, lato esterno
 ES - Este lado hacia arriba, este lado fuera

 EN - CE marked
 SV - CE-märkt
 NO - CE-merket
 DA - CE-mærket
 FI - CE-merkintä
 DE - CE-konform
 NL - CE-markering
 FR - Marquage CE
 IT - Marchio CE
 ES - Marchio CE

 EN - max user weight
 SV - Max. brukarvikt (säker arbetsbelastning)
 NO - Maks. brukervekt (SWL)
 DA - Maks. brugervægt (sikker arbejdsbelastning)
 FI - Käyttäjän enimmäispaino (SWL)
 DE - Max. Benutzergewicht (Tragfähigkeit)
 NL - Max. gewicht gebruiker (veilige belasting):
 FR - Poids max. de l'utilisateur :
 IT - Peso massimo utente (carico di sicurezza)
 ES - Peso máximo del usuario (CTS)

 EN - Refer to user manual
 SV - Se manual
 NO - Se bruksanvisningen
 DA - Se brugervejledning
 FI - Katso käyttöohje
 DE - Lesen Sie die Bedienungsanleitung
 NL - Zie de bedienings-handleiding
 FR - Consultez le manuel d'utilisation
 IT - Fare riferimento al manuale dell'utente
 ES - Consulte el manual del usuario

 EN - Medical device
 SV - Medicinteknisk produkt
 NO - Medicinsk utstyr
 DA - Medicinsk udstyr
 FI - Lääkinnällinen laite
 DE - Medizinprodukt
 NL - Medisch hulpmiddel
 FR - Dispositif médical
 IT - Dispositivo medico
 ES - Producto sanitario

 EN - Date of manufacture. YYYY-MM-DD (year/month/date)
 SV - Tillverkningsdatum. ÅÅÅÅ-MM-DD (år/månad/dag)
 NO - Produksjonsdato. ÅÅÅÅ-MM-DD (år/måned/dag)
 DA - Produktionsdato. ÅÅÅÅ-MM-DD (år/måned/dag)
 FI - Valmistuspäivä. VVVV-KK-DD (vuosi/kuukausi/Päivä)
 DE - Herstellungsdatum. JJJJ-MM-DD (Jahr/Monat/Tag)
 NL - Productie-datum JJJJ-MM-DD (jaar/maand/dag)
 FR - Date de fabrication. AAAA-MM-DD (année/mois/jour)
 IT - Data di produzione. AAAA-MM-DD (anno/mese/giorno)
 ES - Fecha de fabricación. AAAA-MM-DD (año/mes/día)

 EN - Single patient - Multiple use
 SV - En patient - flera användningar
 NO - Kun til bruk på én pasient - flegangsbruk
 DA - Enkelt patient - flegangsbrug
 FI - Potilaskohtainen - voidaan käyttää useita kertoja
 DE - Einzelner Patient - mehrfach anwendbar
 NL - Eén patiënt - meervoudig gebruik
 FR - Un seul patient - à usage multiple
 IT - Singolo paziente - uso multiplo
 ES - Un solo paciente - uso múltiple

 EN - UKCA marked
 SV - UKCA-märkt
 NO - UKCA-merket
 DA - UKCA-mærket
 FI - UKCA-merkintä
 DE - UKCA-konform
 NL - UKCA-markering
 FR - Marquage UKCA
 IT - Marchio UKCA
 ES - Marchio UKCA

Wash symbols:

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
Molift UnoSling Toilet

The Molift UnoSling Toilet is a consumable for single patient use.

Etac supplies a wide selection of slings for different types of transfers. Molift UnoSling Toilet has a low back and a split leg support with a wide opening, facilitating dressing and undressing when using the toilet. The sling can be used for hoisting to and from a sitting position. It is approved for user weighing up to 350 kg. Molift UnoSling Toilet is available in sizes S – XL. Each are made of padded polyester with padded waist belt with non-slip material for a safer lift and transfer.

Molift UnoSling Toilet is not suitable for people with low muscle tone. Because of the sling's wide opening, there is a risk of the user slipping out of the sling.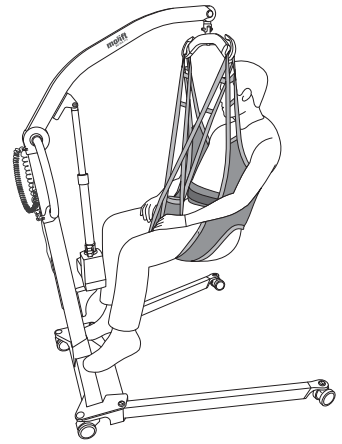

Definition

The person described as "the user" in this Instruction for Use is the person who is lying in the product. "The carer" is the person who applies the product.

Intended purpose

To be used with a hoist for lifting one specific user, who requires support to be lifted for a short time with easy access to toileting; e.g. to/ from a bed, a wheelchair, a chair, a toilet or similar.

Intended target group

Used for users who require support to be lifted for a short time with easy access to toileting; e.g. persons with limited ability to move and who require body support during transfers. Not an exhaustive list.

Intended environment

For indoor use in acute care, long term care, home care.

Indications

The device can be used for children, adults and elderly people when there is a need for a patient specific product.

The target group for the device is based on individual health and mobility function, and not a specific diagnose or age.

Contra indications

Make sure the sling fits the user.

Not suitable for users with low muscle tonus.

The user must have core- and head stability.

The sling is non-washable and should be disposed when dirty or when user leaves the facility.

The Molift UnoSling Toilet sling is a consumable for single patient use, it has a limited service life.


Write the user's name and the date on the product label before using the product.

The Molift UnoSling Toilet is manufactured from non-woven material. Ensure that no pointed or sharp objects come into contact with the sling. Avoid pulling hard on the sling when applying it to or removing it from the user.

The Molift UnoSling Toilet should be taken out of use:

- when it has become dirty or damp, or if there is a risk that the sling has been exposed to contamination;
- if it has been damaged;
- when the user no longer requires it.

The sling should be disposed of in accordance with applicable procedures for the health care facility.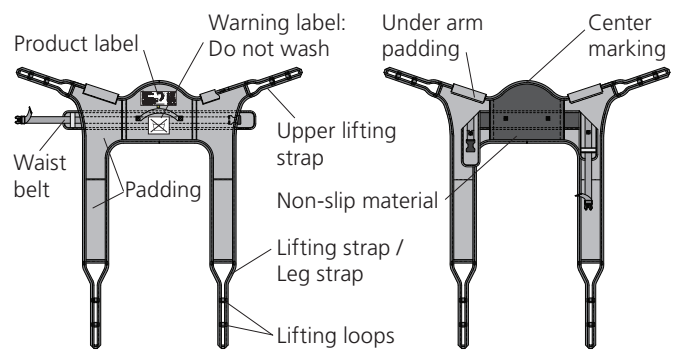

Things to remember


- Read through the manual before starting to use the sling.
- It is important to understand the contents of the user manual before using the equipment. Read the manual for both hoist and sling.
- Molift slings shall only be used to lift persons. Never use the sling to lift or move objects of any kind.
- Make a decision on whether one or more assistants are required. Plan the lifting operation in advance to ensure that it is as safe and smooth as possible. Remember to work ergonomically.
- Lifting and transferring a person always involves a certain risk. Only informed personnel should use the equipment described in this user manual. If the maximum load (SWL) differs between the sling bar and the sling, the lowest maximum load should always be used.
- Carry out a risk assessment and document it. As the assistant, you are responsible for the safety of the user.
- Never leave a user unattended in a lifting situation.
- Make sure that the wheels on the wheelchair, bed, etc. are locked during the movement.
- Remove the sling carefully to ensure that there is no sudden fall.

The following inspection should be carried out daily or before use


- The user must have the bodily functions required for the use of the sling.
- Make sure the sling is suitable for the hoist/sling bar to be used.
- Check that the sling has no visible damage or faults.

Sling checkpoints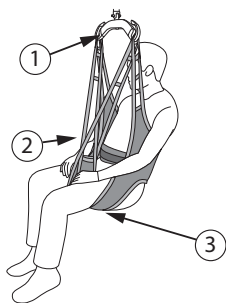


Checkpoint	Troubleshooting
1 Ensure the sling bar is away from the user's head	If not properly considered the user may hit his/her head in the sling bar.
2 Check that straps are crossed	If leg straps are not properly crossed the user risks falling.
3 Leg support is under the user's thighs without folds	Avoiding folds prevents pressure and discomfort for the user.

Lifting checkpoints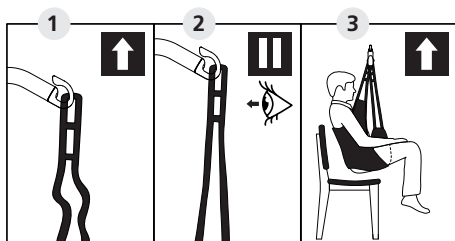

- 1 Check that the sling is correctly fitted around the user and that the strap loops are correctly fitted to the sling bar hooks.
- 2 Start lifting until the lifting straps are stretched without lifting the user. Ensure that all four loops of the sling are securely fastened to avoid the user slipping or falling out of the sling.
- 3 Lift user, and perform transfer.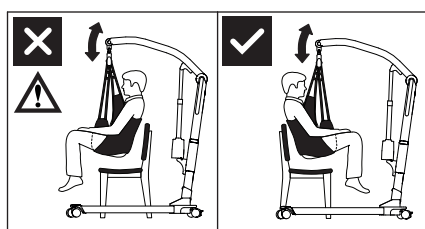


Be careful during movement, the user may swing during turns, stops and starts. Be careful when maneuvering close to furniture and similar to prevent the suspended user from colliding with these objects.

Avoid deep pile carpets, high thresholds, uneven surfaces or other obstacles that may block the castors. The lifter may become unstable if forced over such obstacles increasing the risk of tipping over.

Sling application – Sitting position

- 1 Ask the user to lean forward, support them if necessary with one hand. Place the sling approximately at waist height. Use the center marking line to align the center point of the sling with the user's spine.
- 2 Place the belt around the waist of the user, connect the buckle and tighten the belt. Make sure it is possible to access the relevant clothing. Arms must be placed on outside.
- 3 Pull the leg support forwards and position under the user's thighs.
- 4 Make sure the lifting straps are the same length on both sides and that the sling is placed under the user's thighs without any creases. Cross the leg loops by pulling one of the loops through the other and connect all 4 lifting straps to the sling bar. Make sure the lifting straps are fastened with equal length on the right and left side. Upper straps and leg straps can be mounted with different ladder loop.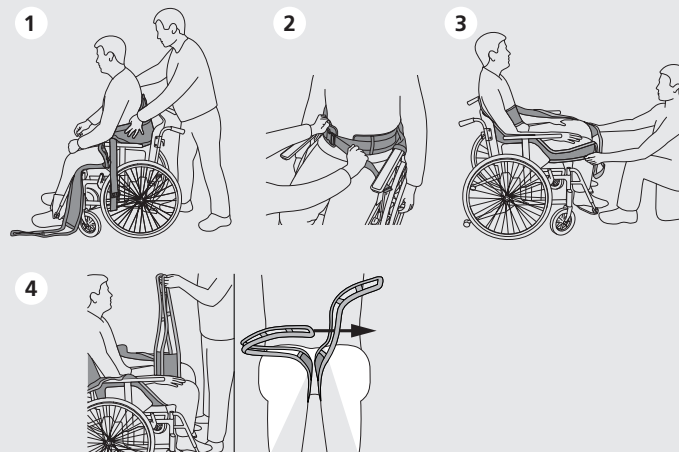


Lowering into sitting

Lift the user and remove their pants. Lower the user onto toilet or toilet chair. When transferring the user into a sitting position, the following techniques can be used for lowering the user as far back into the seat as possible (Use **A** or **B**):

- A:** Push gently on the users knees to guide them towards the backrest of the chair.
- B:** Tilt the seat backwards.

Lower the user into a sitting position. Release the lifting straps from the sling bar. It is possible to remove the leg support and adjust relevant clothing on the user when lowered into a sitting position. The sling can also be completely removed if necessary.

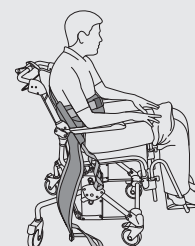Removal of sling - Sitting position

Release lifting straps from sling bar. Stand next to the chair and ask the user to lean forward. Support the user with one hand if necessary. Pull the sling away with one hand, and ask the user to lean back in the chair.

EN
SV
NO
DA
FI
DE
NL
FR
IT
ES

The Molift UnoSling Toilet, as described in this manual, is CE labelled and complies with the Council of Europe’s directive on medical devices MDR (EU) 2017/745, class 1, and has been tested and approved in accordance with the EN ISO 10535:2021 standard.

 This symbol indicates especially important safety information. Follow these instructions carefully.

 Do not wash. If the sling is washed, this symbol disappears and another symbol appears. The sling should then be disposed of.

 Do not use. If this symbol appears the sling has been washed and should be disposed of.

 The sling should not be stored in direct sunlight.

Safe Working Load (SWL): 350 kg (770 lbs)

Material: Polypropylene, polyester

2-point sling bar


	Size of sling		
	S	M-L	XL
Small 340 mm	✓	✓	✗
Medium 440 mm	✓	✓	✓
Large 540 mm	✗	✓	✓

4-point sling bar


	Size of sling		
	S	M-L	XL
Small 340 mm	✓	✓	✗
Medium 440 mm	✓	✓	✓
Large 540 mm	✗	✓	✓

Size in mm	A	B
S	330	840
M	330	1020
L	330	1150
XL	340	1320

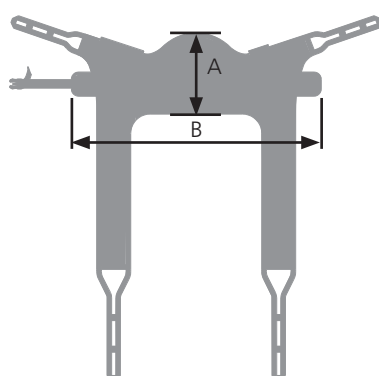Item no.	Sling model	Size
3090020	Molift UnoSling Toilet Single-patient sling, consumer item	S
3090030	Molift UnoSling Toilet Single-patient sling, consumer item	M
3090040	Molift UnoSling Toilet Single-patient sling, consumer item	L
3090050	Molift UnoSling Toilet Single-patient sling, consumer item	XL

Measurements in the table should be used as a reference. Correct size depends on the user’s weight, function and body shape. Measure the waist size and back height of the user in sitting position. Try the sling with most the correct measurements to make sure it fits.

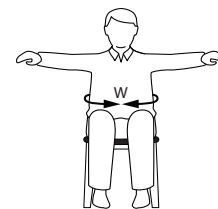Size	User weight - kg (lbs)	Waist (W)
S	25-50 (55-110)	76-100 cm
M	45-95 (99-210)	85-110 cm
L	90-165 (198-353)	94-120 cm
XL	160-350 (353-770)	115-140 cm

Visit www.etac.com to download documentation to ensure that you have the latest version.

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Etac Ltd, Unit 60