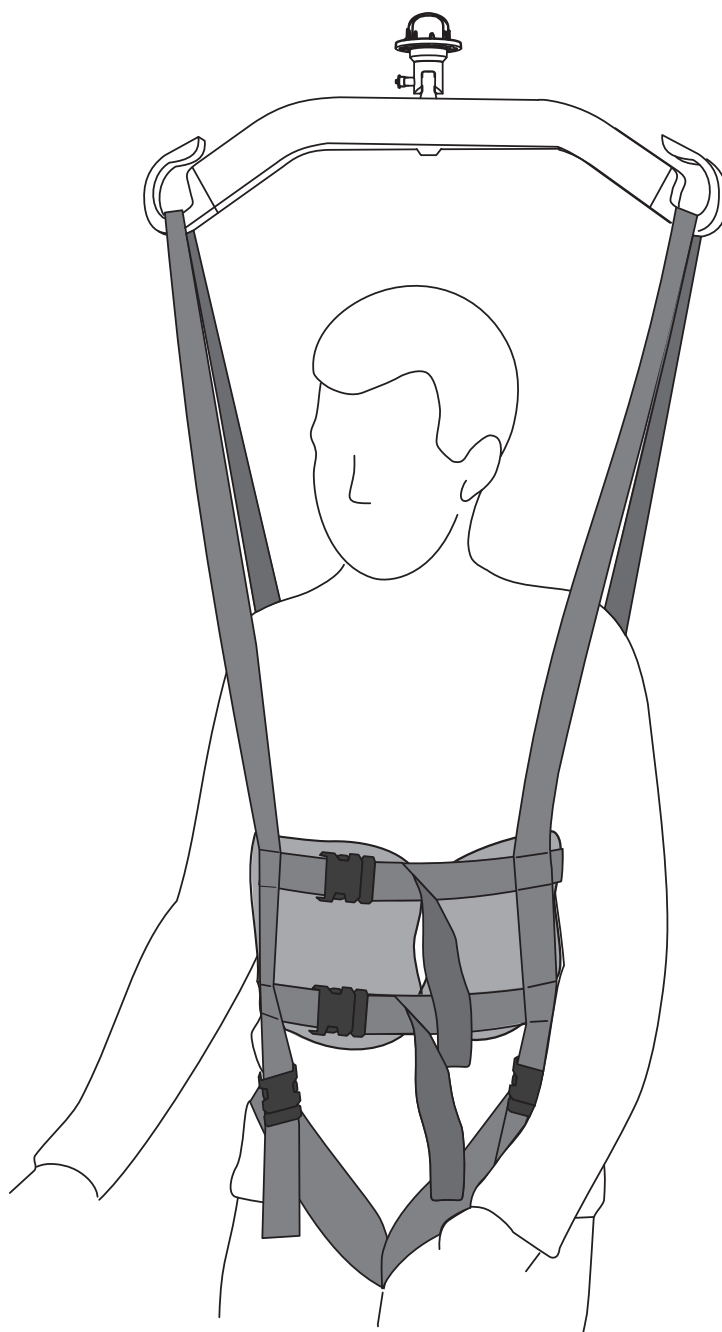
Molift UnoSling Ambulating Vest

molift[®]
by Etac

User manual

BM30499 Rev. A 2022-12-01

MD

CE

Index

Labels and symbols.....	3
User manual – English.....	4
Bruksanvisning – Svenska.....	8
Brukermanual – Norsk.....	12
Brugervejledning – Dansk.....	16
Käyttöohje – Suomi.....	20
Gebrauchsanweisung – Deutsch.....	24
Handleiding – Nederlands.....	28
Manual d'utilisation – Français.....	32
Manual utente – Italiano.....	36
Manual de usuario – Español.....	40

Labels and symbols/ Etiketter och symboler/
Etiketter og symboler/ Mærkater og symboler/
Tarrat ja symbolit/ Beschriftungen und Symbole/
Labels en symbolen/ Étiquettes et symboles/
Etichette e simboli/ Símbolos y etiquetas:

Symbols:

EN - Manufacturer
SV - Tilverkare
NO - Produsent
DA - Producent
FI - Valmistaja
DE - Hersteller
NL - Fabrikant
FR - Fabricant
IT - Produttore
ES - Fabricante

EN- max user weight
SV - Max. brukarvikt (säker arbetsbelastning)
NO - Maks. brukervekt (SWL)
DA - Maks. brugervægt (sikker arbejdsbelastning)
FI - Käyttäjän enimmäispaino (SWL)
DE - Max. Benutzergewicht (Tragfähigkeit)
NL - Max. gewicht gebruiker (veilige belasting)
FR - Poids max. de l'utilisateur
IT - Peso massimo utente (carico di sicurezza)
ES - Peso máximo del usuario (CTS)

EN - CE marked
SV - CE-märkt
NO - CE-merket
DA - CE-mærket
FI - CE-merkintä
DE - CE-konform
NL - CE-markering
FR - Marquage CE
IT - Marchio CE
ES - Marchio CE

EN - Medical device
SV - Medicinteknisk produkt
NO - Medisinsk utstyr
DA - Medicinsk udstyr
FI - Lääkinnällinen laite
DE - Medizinprodukt
NL - Medisch hulpmiddel
FR - Dispositif médical
IT - Dispositivo medico
ES - Producto sanitario

EN-Refer to user manual
SV-Se manual
NO - Se bruksanvisningen
DA - Se brugervejledning
FI - Katso käyttöohje
DE - Lesen Sie die Bedienungsanleitung
NL - Zie de bedienings-handleiding
FR - Consultez le manuel d'utilisation
IT - Fare riferimento al manuale dell'utente
ES - Consulte el manual del usuario

EN- Single patient - Multiple use
SV - En patient - flera användningar
NO - Kun til bruk på én pasient - flergangsbruk
DA - Enkelt patient - flergangsbrug
FI - Potilaskohtainen - voidaan käyttää useita kertoja
DE - Einzelnr Patient - mehrfach anwendbar
NL - Eén patiënt - meervoudig gebruik
FR - Un seul patient - à usage multiple
IT - Singolo paziente - uso multiplo
ES - Un solo paciente - uso múltiple

YYYY-MM-DD

EN - Date of manufacture. YYYY-MM-DD (year/month/date)
SV - Tillverkningsdatum. ÅÅÅÅ-MM-DD (år/månad/dag)
NO - Produksjonsdato. ÅÅÅÅ-MM-DD (år/månad/dag)
DA - Produktionsdato. ÅÅÅÅ-MM-DD (år/månad/dag)
FI - Valmistuspäivä. VVVV-KK-DD (vuosi/kuukausi/Päivä)
DE - Herstellungs-datum. JJJJ-MM-DD (Jahr/Monat/Tag)
NL - Productie-datum JJJJ-MM-DD (jaar/maand/dag)
FR - Date de fabrication. AAAA-MM-DD (année/mois/jour)
IT - Data di produzione. AAAA-MM-DD (anno/mese/giorno)
ES - Fecha de fabricación. AAAA-MM-DD (año/mes/día)

EN - Serial number
SV - Serienummer
NO - Serienummer
DA - Løbenummer
FI - Sarjanumero
DE - Seriennummer
NL - Serienummer
FR - Numéro de série
IT - Numero di serie
ES - Número de serie

EN - Article number
SV - Artikelnummer
NO - Artikkelnummer
DA - Artikelnummer
FI - Tuotenumero
DE - Artikelnummer
NL - Artikelnummer
FR - Numéro de série
IT - Numero articolo
ES - Número de artículo

EN- This Way up, this side is outside
SV - Denna sida upp, denna sida utåt
NO - Denne siden opp, denne siden ut
DA - Denne side op, denne side er ydersiden
FI - Tämä puoli ylöspäin, tämä puoli ulospäin
DE - Hier oben, diese Seite ist außen
NL - Deze kant boven, dit is de buitenkant
FR - Vers le haut, côté extérieur
IT - Lato alto, lato esterno
ES - Este lado hacia arriba, este lado fuera

EN - UKCA marked
SV - UKCA-märkt
NO - UKCA-merket
DA - UKCA-mærket
FI - UKCA-merkintä
DE - UKCA-konform
NL - UKCA-markering
FR - Marquage UKCA
IT - Marchio UKCA
ES - Marchio UKCA

Molift UnoSling Ambulating Vest
A patient specific, disposable sling

REF: xxxxxxxx Size: XX

SN: xxxxxxxx

DK HMI: xxxxxx

Name: _____

Barcode according to GS1-128
(01)xxxxxxxxxxxx(11)xxxxxxxx(21)xxxxxxxx
(01):EAN number (11)YYMMDD(21):Serial number

DO NOT WASH DO NOT BLEACH DO NOT TUMBLE DRY DO NOT IRON DO NOT DRY CLEAN

UPPER OUTPUT 1 XXX XXX

Material: Polypropylene, Polyester

MD Date of first use
Year: 2022 2023 2024 2025 2026 2027
Month: 01 02 03 04 05 06 07 08 09 10 11 12

Etac A/S - Parallelvej 3 - DK-8751 Gedved

Wash symbols:

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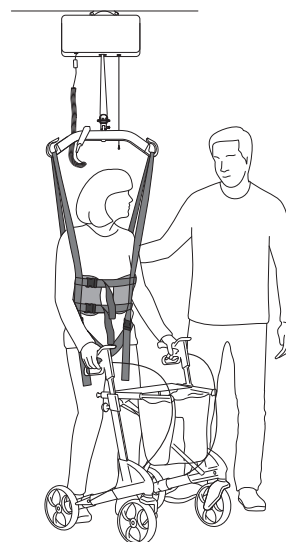
Molift UnoSling Ambulating Vest

The Molift UnoSling Ambulating Vest is a consumable for single patient use.

Etac supplies a wide selection of slings for different types of transfers. The Molift UnoSling Ambulating Vest is intended to be used for rehabilitation / mobilization to facilitate standing, walking and balance training. It is designed to be used with a 2-point sling bar. Molift UnoSling Ambulating Vest is intended for users who can follow instructions, have core balance and are able to take load on the legs. It is approved for users weighing up to 350 kg and is available in sizes S-XL.

The sling is placed around the user's waist and is secured with the belt buckles and possibly the groin straps. The groin straps have padding and prevents the sling from sliding up. The groin straps are removable. The sling has a non-slip lining inside the waist belt, which helps keep the sling in place.

Definition

The person described as "the user" in this Instruction for Use is the person who is lying in the product. "The carer" is the person who applies the product.

Intended purpose

To be used with a hoist for supporting one specific user, who requires support to stand and/or walk; e.g. for gait training or similar.

Intended target group

Used for users who require support to stand and/or walk; e.g. persons with limited ability to move and who require body support during transfers. Not an exhaustive list.

Intended environment

For indoor use in acute care, long term care, home care.

Indications

The device can be used for children, adults and elderly people when there is a need for a patient specific product.

The target group for the device is based on individual health and mobility function, and not a specific diagnose or age.

Contra indications

Make sure the sling fits the user.

The user should be able to take load on the legs and have core stability.

The sling is non-washable and should be disposed when dirty or when user leaves the facility.

The Molift UnoSling Ambulating Vest sling is a consumable for single patient use, it has a limited service life.

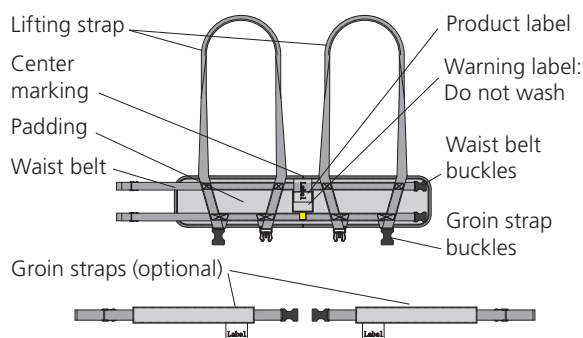
Write the user's name and the date on the product label before using the product.

The Molift UnoSling Ambulating Vest is manufactured from non-woven material. Ensure that no pointed or sharp objects come into contact with the sling. Avoid pulling hard on the sling when applying it to or removing it from the user.

The Molift UnoSling Ambulating Vest should be taken out of use:

- when it has become dirty or damp, or if there is a risk that the sling has been exposed to contamination;
- if it has been damaged;
- when the user no longer requires it.

The sling should be disposed of in accordance with applicable procedures for the health care facility.


Things to remember

- Read through the manual before starting to use the sling.
- It is important to understand the contents of the user manual before using the equipment. Read the manual for both hoist and sling.
- Molift slings shall only be used to lift persons. Never use the sling to lift or move objects of any kind.
- Make a decision on whether one or more assistants are required. Plan the lifting operation in advance to ensure that it is as safe and smooth as possible. Remember to work ergonomically.
- Lifting and transferring a person always involves a certain risk. Only informed personnel should use the equipment described in this user manual. If the maximum load (SWL) differs between the sling bar and the sling, the lowest maximum load should always be used.
- Carry out a risk assessment and document it. As the assistant, you are responsible for the safety of the user.
- Never leave a user unattended in a lifting situation.
- Make sure that the wheels on the wheelchair, bed, etc. are locked during the movement.
- Remove the sling carefully to ensure that there is no sudden fall.

The following inspection should be carried out daily or before use


- The user must have the bodily functions required for the use of the sling.
- Make sure the sling is suitable for the hoist/sling bar to be used.
- Check that the sling has no visible damage or faults.

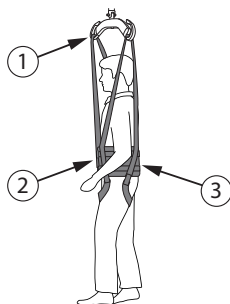Sling checkpoints

Checkpoint	Troubleshooting
1 Safety clearance to hoist – minimises the risk that the user will hit their head.	If the sling is too small, the head may come too close to the sling bar.
2 The sling does not fit the user.	Wrong sling size. Try another size.
3 The sling is sliding up the user's back.	The user is not able to take load on the legs/feet. Evaluate whether a sling intended for rehabilitation/mobilization training is right or if the user requires a passive lift. The user is wearing too slippery clothes. Change or remove some of the clothing. The sling has been raised without the user leaning forward or the belt not tightened properly. Lower the user, adjust the sling and start over.

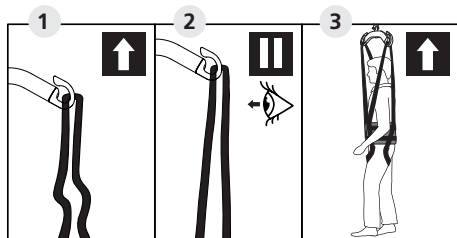
Lifting checkpoints

- 1** Check that the sling is correctly fitted around the user and that the strap loops are correctly fitted to the sling bar hooks.
- 2** Start lifting. Ensure that the lifting straps are stretched tense, and pause before continuing. Check that all lifting loops are attached correctly to the sling bar.
- 3** Continue lifting the user and carry out the planned training / rehabilitation activity.


Sling application

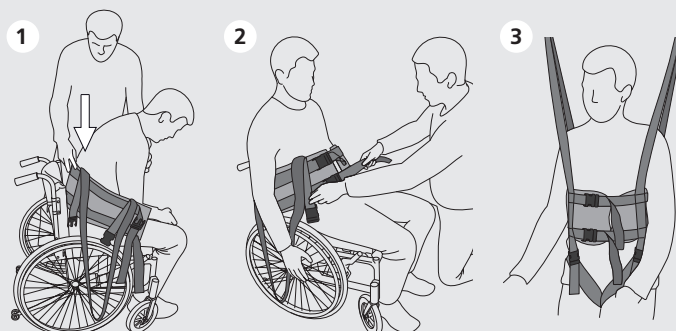- 1** Hold sling with one hand and use the other hand to support the user. Slide the folded sling down behind the user's back until it reaches the seat.
- 2** Fix the waist belt and tighten. The belt can be retightened after the patient is raised up.
- 3** The user's arm should be placed in between the lifting straps.


Groin strap (optional)

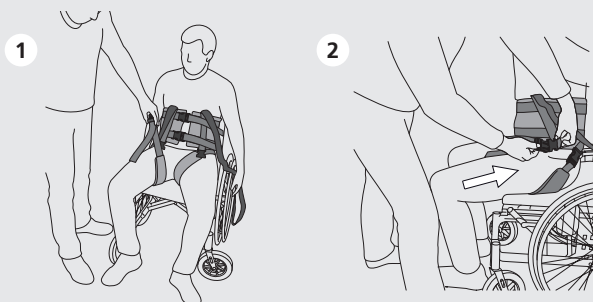- 1** Put the groin strap under the thigh.
- 2** Connect the buckles. The groin strap can be retightened after the patient is raised up.

Removing sling

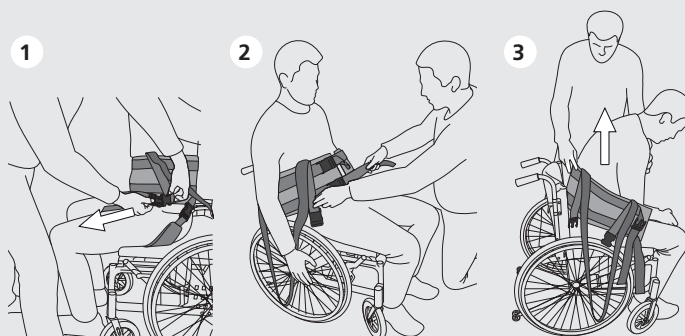- 1** Lower user into chair. Release straps from sling bar. Release groin strap buckles and remove groin strap.
- 2** Release waist belt buckles.
- 3** Use one hand to support and lean the user forward while removing the sling from the back with the other hand. Make sure not to pull too hard or fast, this may cause the user to fall out of the chair.

The Molift UnoSling Ambulating Vest, as described in this manual, is CE labelled and complies with the Council of Europe's directive on medical devices MDR (EU) 2017/745, class 1, and has been tested and approved in accordance with the EN ISO 10535:2021 standard.

This symbol indicates especially important safety information. Follow these instructions carefully.

Do not wash. If the sling is washed, this symbol disappears and another symbol appears. The sling should then be disposed of.

Do not use. If this symbol appears the sling has been washed and should be disposed of.

The sling should not be stored in direct sunlight.

Safe Working Load (SWL): 350 kg (770 lbs)


Material: Polypropylene, polyester

2-point sling bar


	Size of sling		
	S	M-L	XL
Small 340 mm	✓	✓	✗
Medium 440 mm	✓	✓	✓
Large 540 mm	✗	✓	✓

Size in mm	A	B
S	200	820
M	200	1000
L	210	1150
XL	210	1330

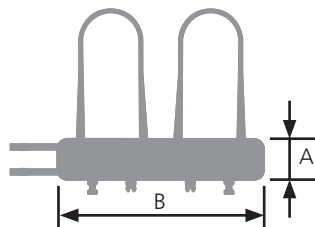Item no.	Sling model	Size
3070020	Molift UnoSling Ambulating Vest Single-patient sling, consumer item	S
3070030	Molift UnoSling Ambulating Vest Single-patient sling, consumer item	M
3070040	Molift UnoSling Ambulating Vest Single-patient sling, consumer item	L
3070050	Molift UnoSling Ambulating Vest Single-patient sling, consumer item	XL

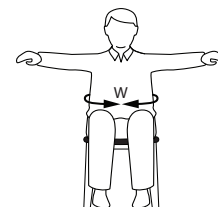Measurements in the table should be used as a reference. Correct size depends on the user's weight, function and body shape. Measure the waist size and back height of the user in sitting position. Try the sling with most the correct measurements to make sure it fits.

Size	User weight - kg (lbs)	Waist (W)
S	25-50 (55-110)	72-92 cm
M	45-95 (99-210)	90-110 cm
L	90-165 (198-353)	105-125 cm
XL	160-350 (353-770)	123-145 cm

Visit www.etac.com to download documentation to ensure that you have the latest version.

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Etac Ltd, Unit 60