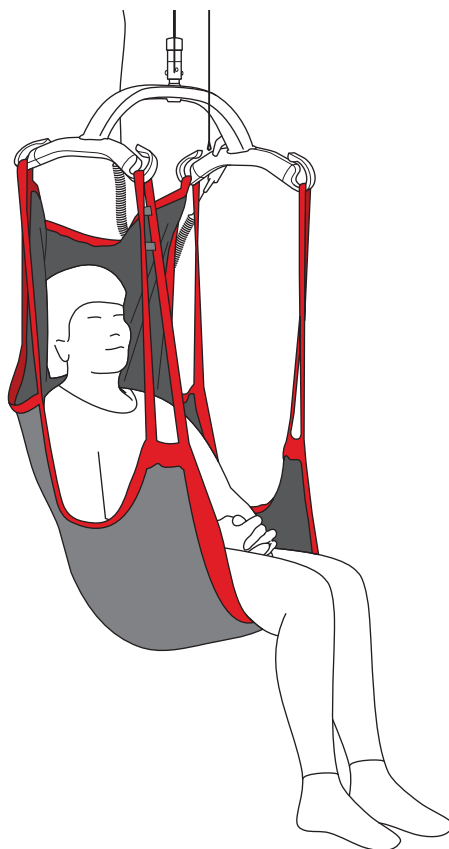
Brugermanual

Molift RgoSling Comfort HighBack

molift[®]
by Etac

BM40699 - 2024-08-13 - Rev. 5.0

www.etac.com

	Symbols	4
	Device overview	8
	General	10
	English	10
	Svenska	14
	Norsk	18
	Dansk	22
	Suomi	26
	Deutsch	30
	Nederlands	34
	Français	38
	Italiano	42
	Español	46
	Handling	50
	Lying application	51
	Sitting in situ	53
	Lying removal	54
	Warnings	55
	Information	57
	Combination list	59
	Sling measurements	59
	Size guide	59

EN: Warning: This symbol appears in the user manual along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger.

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EN: Information: This symbol appears in the user manual along with a number referring to supplementary information which can be found in the information section.

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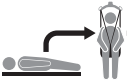
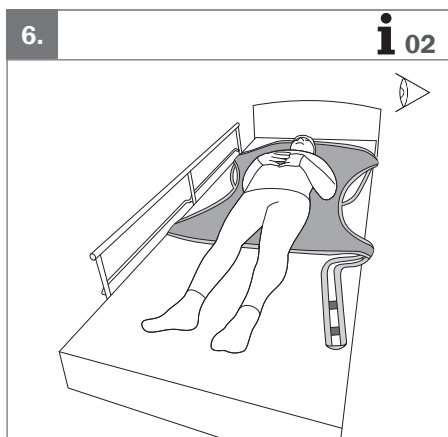
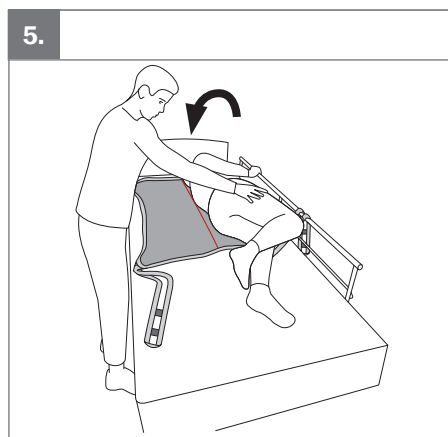
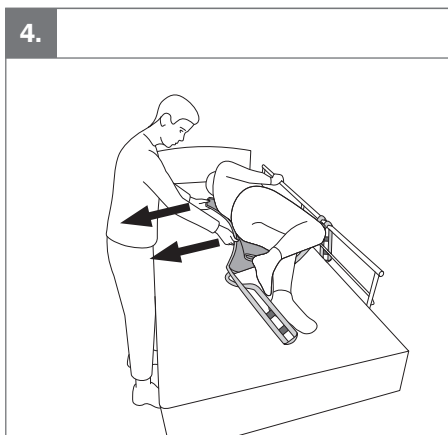
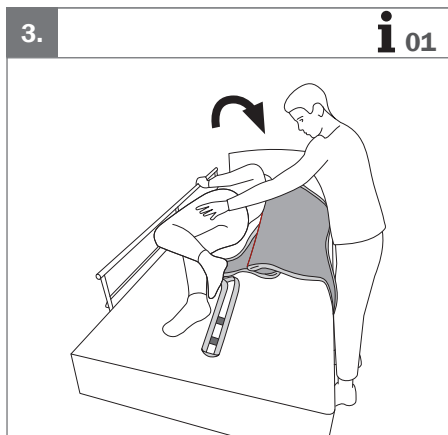
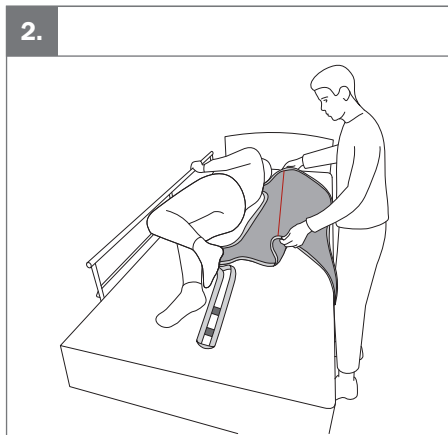
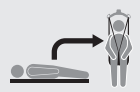
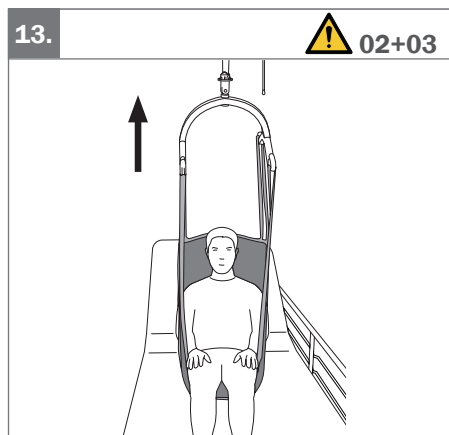
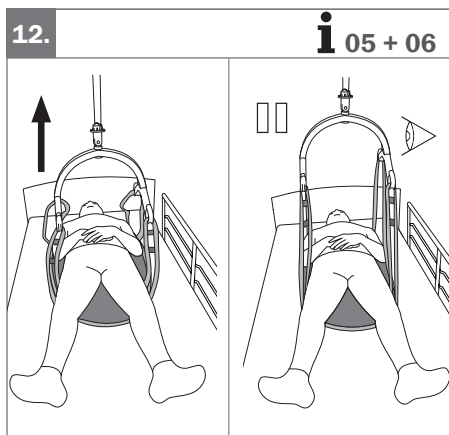
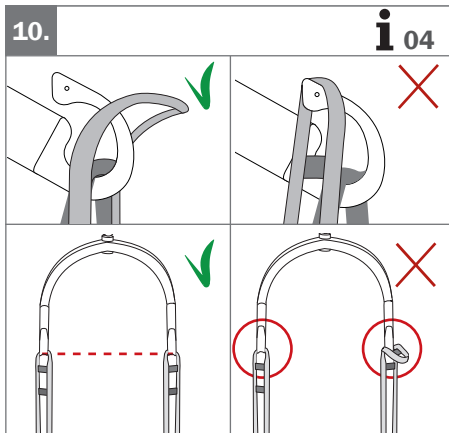
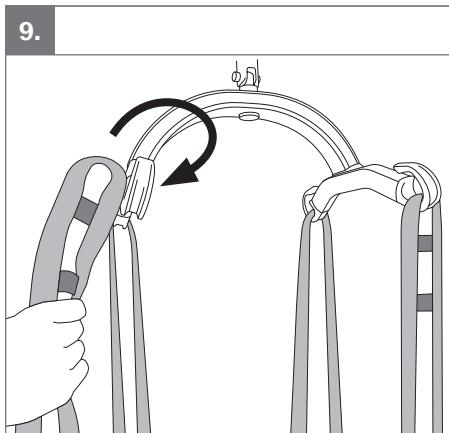
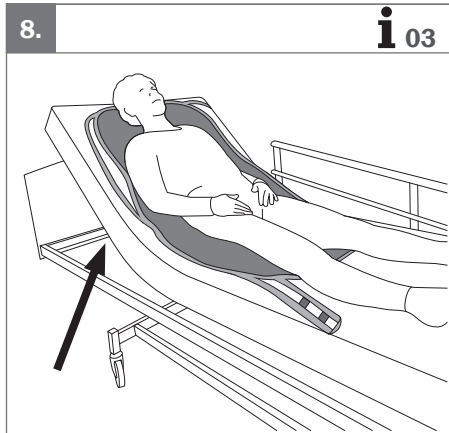
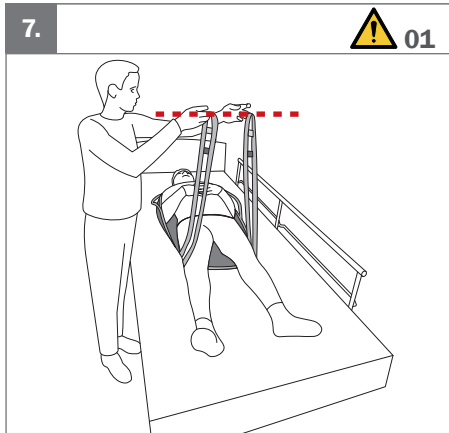
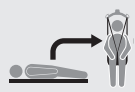
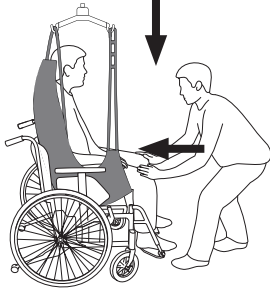
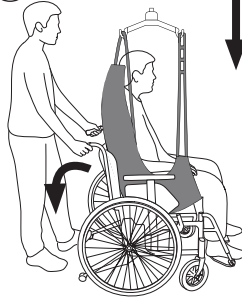
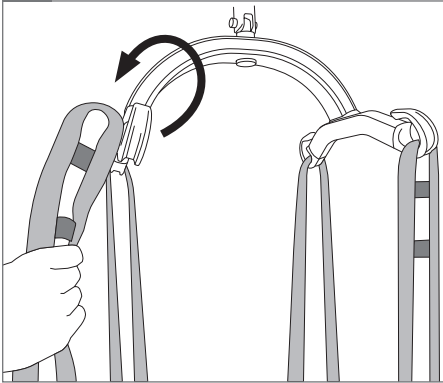
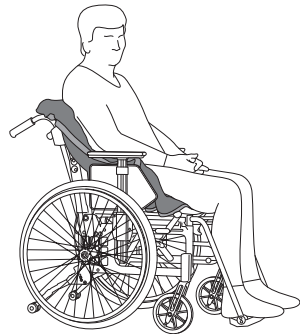
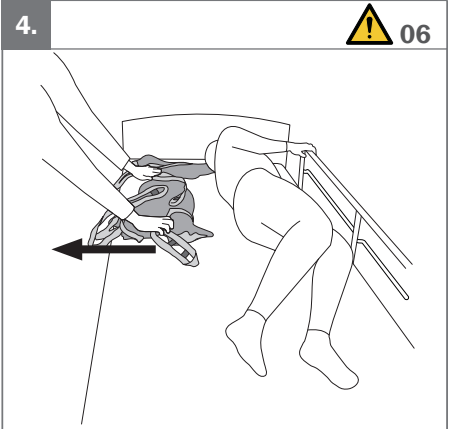
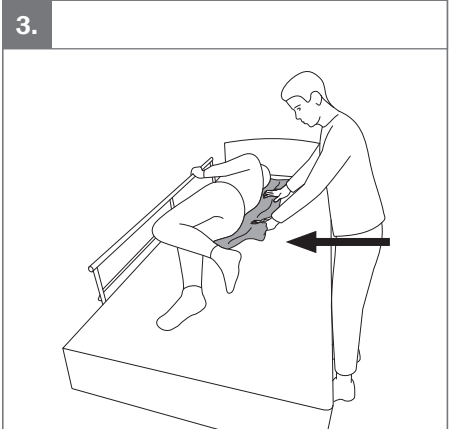
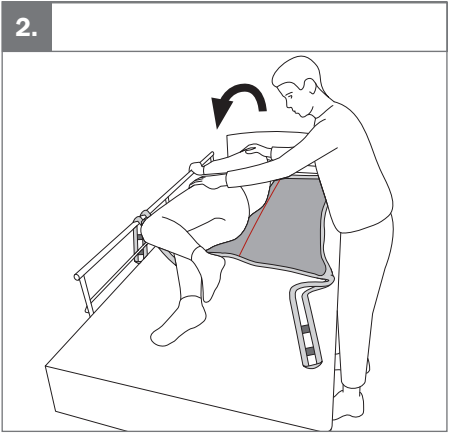
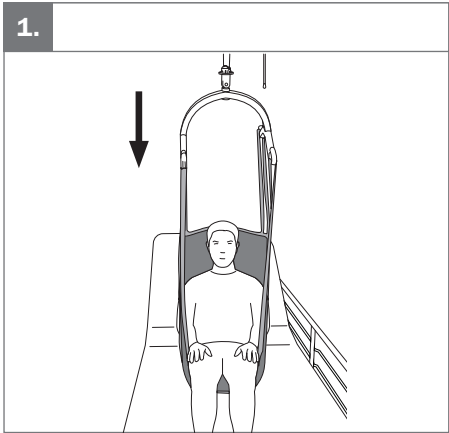
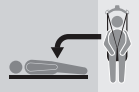
! 04+05

**EN:** Warnings

01. Do not cross the lower straps.
02. Be careful when transferring, the user may swing and hit surrounding objects.
03. Never hoist the user higher than necessary.
04. Do not try to remove the sling while the user is sitting on it.
05. Never hoist again without checking for correct placement of the sling, and adjust if required.
06. Remove the sling gently to ensure the user does not roll out of the bed.

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EN: Information

- Place the sling at the level of the user's knee pit. The centre marking should follow the midline of the user.
- Make sure the sling reaches the knee pit before moving on to next step.
- If possible, raise the backrest closer to a sitting position before hoisting from a lying position.
- Make sure the straps are fastened symmetrically on left and right side.
- Before performing the transfer – make sure that:
 - Sling reaches the user's knee pit.
 - Lower straps are not crossed.
 - If the user has been sitting or lying on the sling, check that the sling is still correctly positioned.
- Perform a safety stop when the straps are stretched tense. Check that all straps are correctly and symmetrically attached.
- Lower the user to sitting position using either method A, B or C.

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Etac Ltd.
Etac A/S