Instructions of Use

Clip Fixing Sling Range

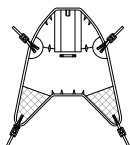
	Mesh		Parasilk		Silva-Superfine® with Coolmax®		Silva-Superfine® PLUS
	Polyester		Silva Quick -2-Apply®		Silva-Superfine® with Coolmax® Double layer		Silva-Superfine® PLUS Anti-Microbial

*Not all fabrics
available with all styles


HIGH EASY

 HY


AMPUTEE HIGH EASY ALL IN ONE

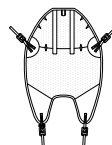 DA-HYA2


HIGH EASY TOILET SLING

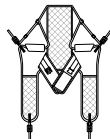 DA-ST3


TRANSPORTER

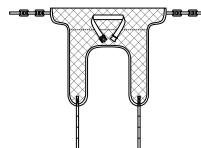 DA-AS100

RIGID STAND AID

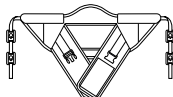 DA-AS112


DELUXE RIGID STAND AID

 DA-AS116


BLISS STAND AID






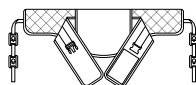
 BLISS







GENERAL STAND AID

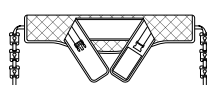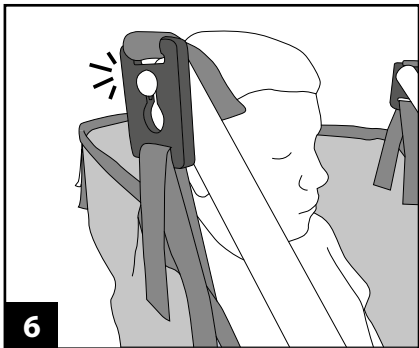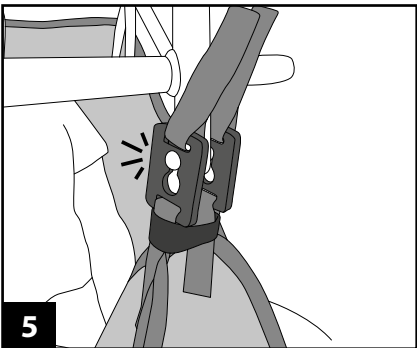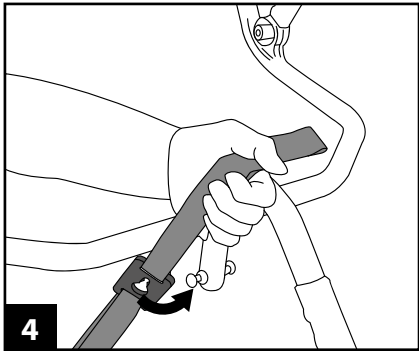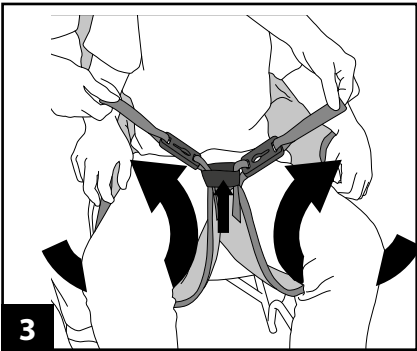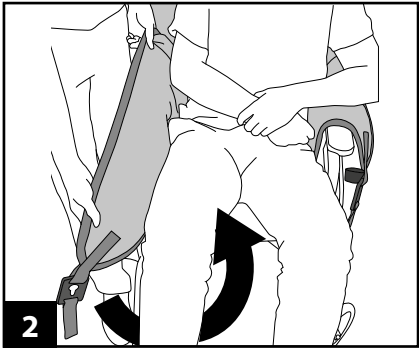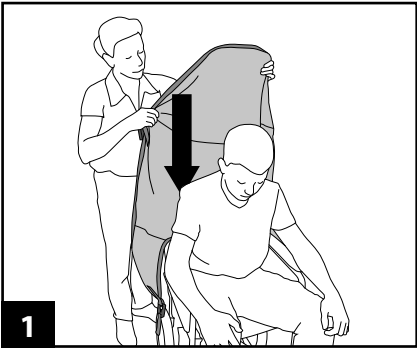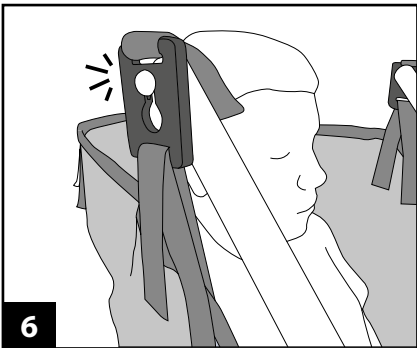
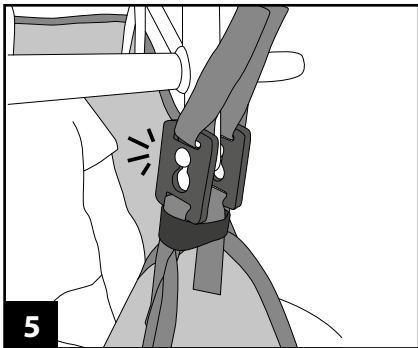
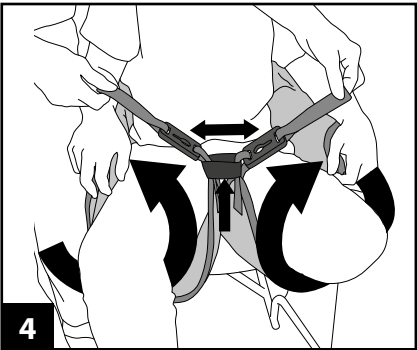
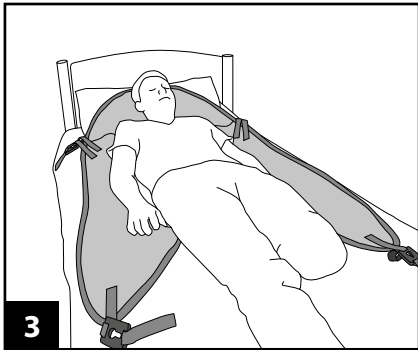
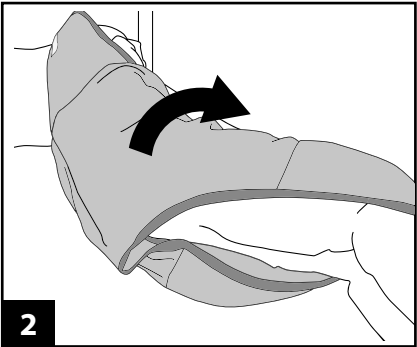
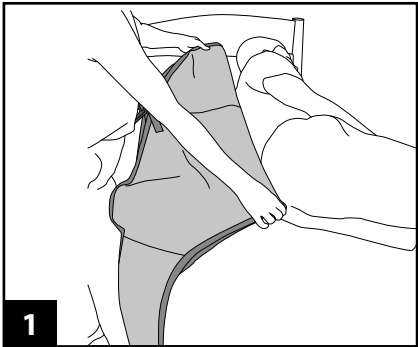
 D202












High Easy Toilet Sling | DA-ST3A

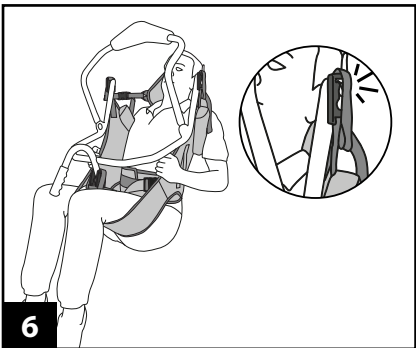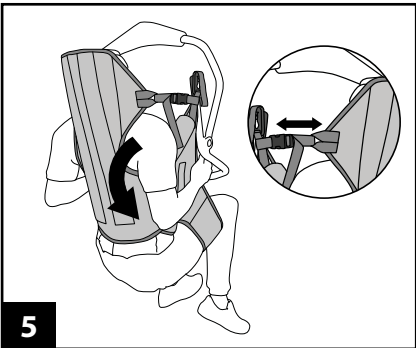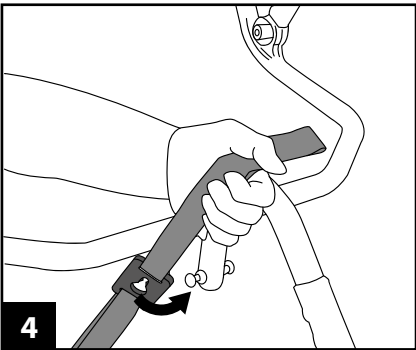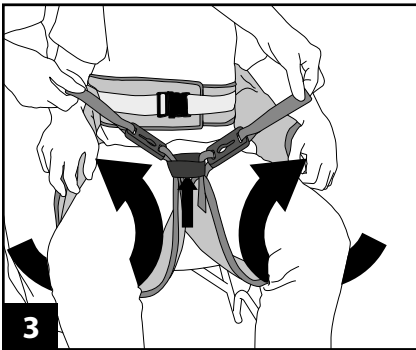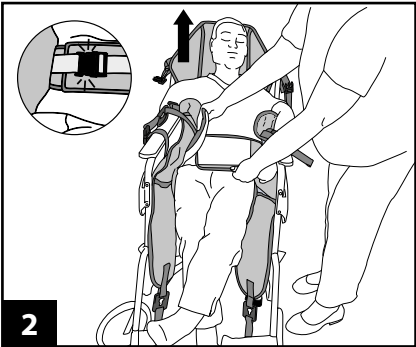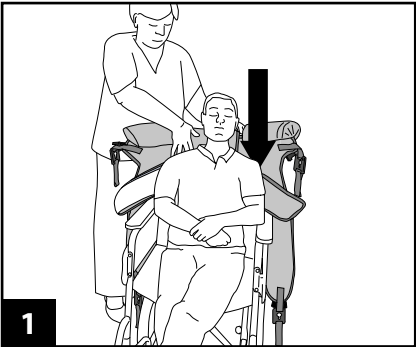Stand Aids | DA-AS112, DA-AS116, BLISS, D202

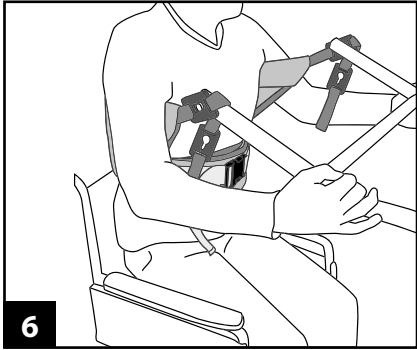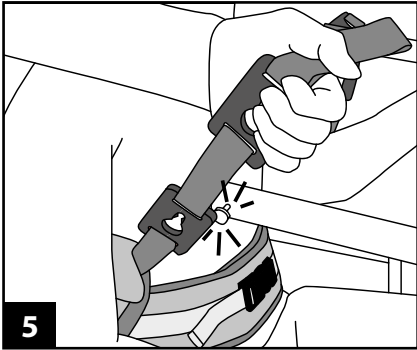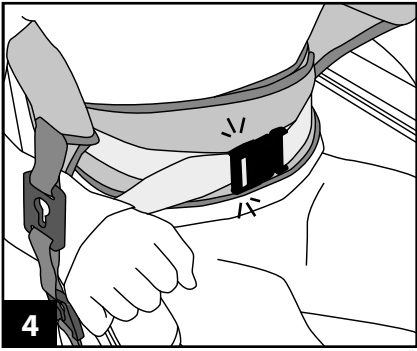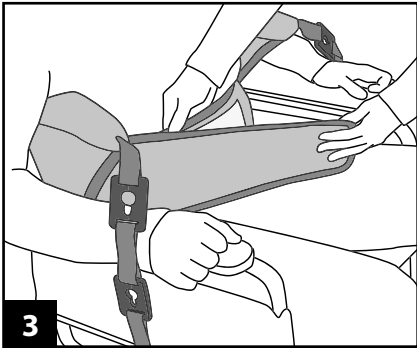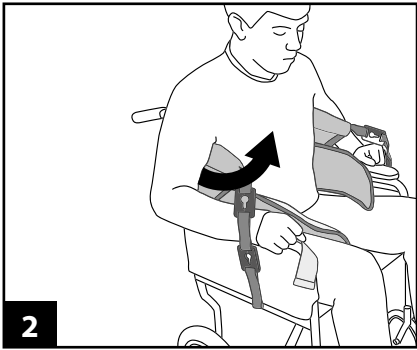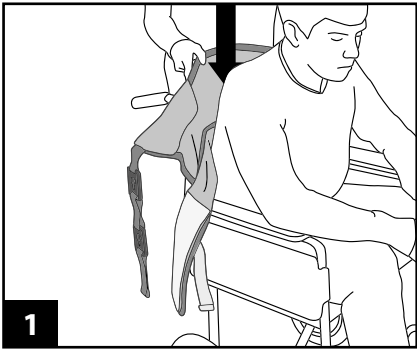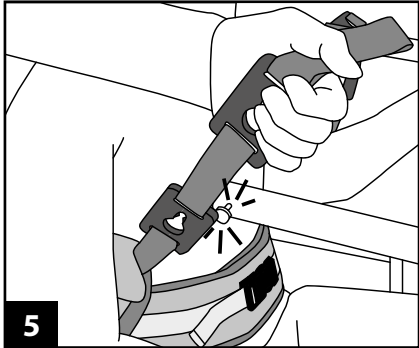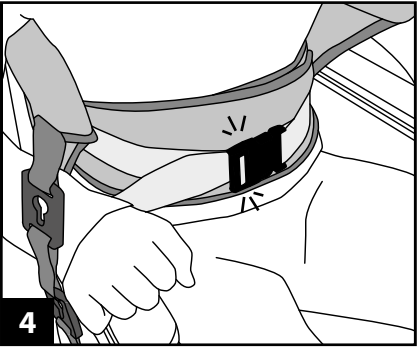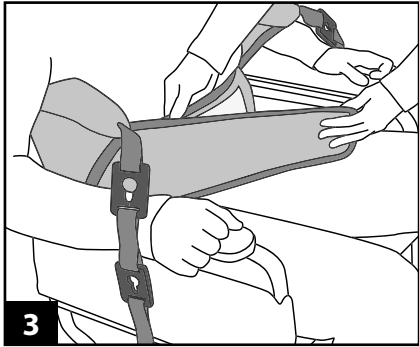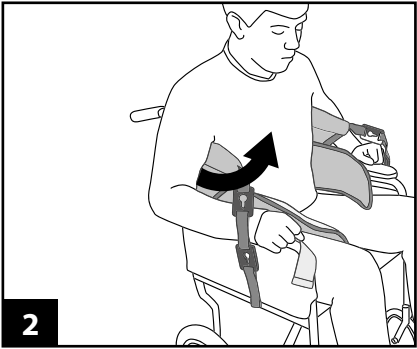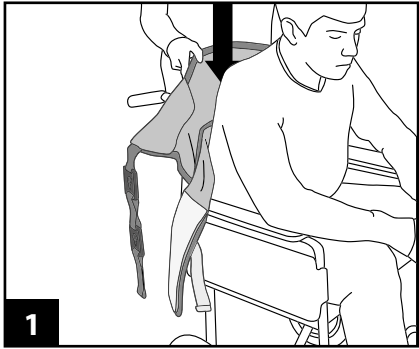For sizing guides and further information
of this product please visit
www.silvalea.com

Or for further help and advice call
+44 (0)1626 331 655

Product Care Instructions

Warning:

Do not attempt to lift the patient until all appropriate fixings are secured to the hoist spreader bar and you are satisfied that the sling is the correct style, size and fit.

Correct sizing of a sling is important as a sling that is too large could cause the patient to slip through the aperture.

Whilst every attempt is made to instruct a safe move you must conform to the current moving and handling procedures/regulations. If in any doubt, consult your Moving and Handling Advisor.

Service Label & Sling Checks:

All products should be checked each time they are used for any signs of deterioration.

The service label is a visual guide to the product's check, which should coincide with your 6-monthly or annual hoist inspections. This is in line with LOLER (Lifting Operations and Lifting Equipment Regulations).

A risk assessment must be carried out to ensure that the correct size and style of sling has been met before use.

Pre-use sling check list:

- braiding/tapes for deterioration and fraying
- material for holes/snagging
- compatibility with the hoist
- safe working load of both the hoist and sling /product (if applicable)
- label is legible

Remove from use immediately if there are any signs of deterioration to the sling or tapes or the label is illegible.

Warranty:

Silvalea offers a 12 month manufacturers warranty on all products unless otherwise stated. Scan QR or visit our website warranty page for more information.

Washing Guidelines:

- Follow laundry temperatures for washing and drying detailed on the product label.
- Always ensure that all Hook & Loop fastenings are fully closed before washing.
- All products should be laundered using non-biological detergents.
- Metal fixings are sometimes used in our products. Care should be taken to ensure that all products containing metal attachments are washed in a pillowcase or wash bag to prevent damage to the product or to washing equipment. Silvalea accept no liability for damage to laundry equipment.
- DO NOT IRON
- Always remove the head support bones prior to washing to prevent damage to the sling & bones
- Silva-Superfine with COOLMAX® mesh slings are provided with a wash bag. The wash bag MUST be used during laundering.

Product Care:

- Ensure that any Hook & Loop fastenings are closed when not in use. This will prevent unnecessary snagging of the sling material and prevent clogging of the Hook & Loop.
- Launder regularly.
- Store products in a suitable dry clean place.
- Silvalea will accept no liability for any additions/ adaptations made to our products that have NOT been carried out by ourselves. Doing so may cause serious injury or death to the user.

Drying Guidelines:

- Follow the recommended drying temperatures as stated on product label.
- Overheating a textile product can cause deterioration in the material components.
- Do not place the product on a convactor heater or steam pipes.

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Revision: 26/05/2021

Silverhills Buildings, Decoy Ind. Estate,
Newton Abbot, Devon, TQ12 5LZ, UK

ADVENA Ltd. Tower Business Centre, 2nd
Floor, Tower Street, Swatar, BKR 4013. Malta

Silvalea Ltd a division of **savaria**. Copyright 2021 © All rights reserved

Copyright & Design Rights: Silvalea. Silva-Quick-2-ApPLY® is a registered trademark of Silvalea Ltd. Silvalea Ltd is the exclusive licensee of the trademark. Silva-Superfine® is a registered trademark of Silvalea Ltd. Silvalea Ltd is the exclusive licensee of the trademark. Silva-Safety Slot® is registered trademark of D-ansermed Ltd. All rights reserved. COOLMAX® is a registered trademark of the LYCRA Company.

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Etac AB. Kista Science Tower
Silvalea Ltd silvalea.com