AXIOM | SP VISCO

AXIOM | SP FLUID

EN - USER INSTRUCTION MANUAL AND WARRANTY
ES - MANUAL DE INSTRUCCIONES Y GARANTÍA
FR - MANUEL D'INSTRUCTIONS DE L'UTILISATEUR ET GARANTIE
CZ - NÁVOD K POUŽITÍ A ZÁRUKA
DA - BRUGERVEJLEDNING OG GARANTI
DE - BEDIENUNGSANLEITUNG UND GEWÄHRLEISTUNG
FI - KÄYTTÖOHJE JA TAKUU
IS - NOTENDAHANDBÓK OG ÁBYRGÐ
IT - MANUALE CON LE ISTRUZIONI PER UTILIZZATORE E GARANZIA
NL - GEBRUIKSAANWIJZING & GARANTIE
NO - BRUKERHÅNDBOK OG GARANTI
PL - INSTRUKCJA OBSŁUGI I GWARANCJA
PT - MANUAL DE INSTRUÇÕES DO(A) UTILIZADOR(A) E GARANTIA
SL - NAVODILA ZA UPORABO IN GARANCIJA
SV - ANVÄNDARHANDBOK OCH GARANTI

EN- USER INSTRUCTION MANUAL AND WARRANTY	1
ES- MANUAL DE INSTRUCCIONES Y GARANTÍA.....	3
FR- MANUEL D'INSTRUCTIONS DE L'UTILISATEUR ET GARANTIE.....	6
CZ- NÁVOD K POUŽITÍ A ZÁRUKA.....	8
DA- BRUGERVEJLEDNING OG GARANTI	11
DE- BEDIENUNGSANLEITUNG UND GEWÄHRLEISTUNG.....	13
FI- KÄYTTÖOHJE JA TAKUU	17
IS- NOTENDAHANDBÓK OG ÁBYRGÐ.....	20
IT- MANUALE CON LE ISTRUZIONI PER UTILIZZATORE E GARANZIA.....	22
NL- GEBRUIKSAANWIJZING & GARANTIE	25
NO- BRUKERHÅNDBOK OG GARANTI	28
PL- INSTRUKCJA OBSŁUGI I GWARANCJA	30
PT- MANUAL DE INSTRUÇÕES DO(A) UTILIZADOR(A) E GARANTIA	33
SL- NAVODILA ZA UPORABO IN GARANCIJA	36
SV- ANVÄNDARHANDBOK OCH GARANTI	38

EN - INTRODUCTION

Thank you for purchasing an Axiom product!

WARNING: Do not use this without first reading this entire manual. Use without first reading this entire manual could result in serious injury to the user and/or attendant.

Ki Mobility recommends training by an Assistive Technology Professional (ATP) or other competent clinical or technical professional in the safe use of this chair prior to use.

If you have any questions or concerns about any aspect of this wheelchair, this manual or the service provided, do not hesitate to contact your authorized supplier, an authorized representative or Ki Mobility. Ki Mobility can be reached at:

(1)715-254-0991

Ki Mobility, LLC
5201 Woodward Drive
Stevens Point, WI 54481
USA

www.kimobility.com
(Includes Dealer locator and access to printed information)

Or via our Authorized EU Representative:

EU REP Etac Supply Center AB
Långgatan 12
334 33 Anderstorp
Sweden

Or via our UK Responsible Person:

UK CA Etac Ltd
Unit 60, Hartlebury Trading
Estate, Hartlebury
Kidderminster, DY10 4JD

TABLE OF CONTENTS

EN - INTRODUCTION.....	1
TABLE OF CONTENTS.....	1
AXIOM SP VISCO & SP FLUID.....	1
Cushion Features.....	1
Intended Use.....	1
Your Safety.....	1
Signal Words.....	1
Symbol Glossary.....	1
Check Your Skin Daily.....	1
General Warnings.....	1
Protecting Your Cushion.....	1
Obstructions.....	2
Cover Orientation.....	2
SET-UP, ADJUSTMENT AND CARE.....	2
Fitting Your Cushion and Wheelchair.....	2
Maintenance.....	2
Cleaning.....	2
Reassembling Your Axiom Cushion After Cleaning.....	2
Axiom SP Fluid - Bottoming Out.....	3
KI MOBILITY LIMITED WARRANTY.....	3
For 24 Months.....	3
For 6 Months.....	3
Limitations.....	3
What We Will Do.....	3
What You Must Do.....	3
Notice to Consumer.....	3

AXIOM SP VISCO & SP FLUID

IMPORTANT CONSUMER INFORMATION

NOTE: This manual contains instructions for use with the Ki Mobility Axiom SP Fluid and SP Visco cushion products. This information must be provided to the consumer of this product. Do not remove this manual before delivery to the consumer of this product.

Supplier: This manual must be given to the consumer of this product.


Consumer: Read the entire owner's manual prior to using this product and keep on-hand for future reference.

NOTE: Check all parts for shipping damage. In case of damage, DO NOT use. Contact the supplier or Ki Mobility for further instructions.

A. Cushion Features

1. Pleated 4-Way Stretch Inner Cover	2. Viscoelastic Foam Insert
3. Dual Density Polyurethane Foam Base	4. 4-Way Stretch Cover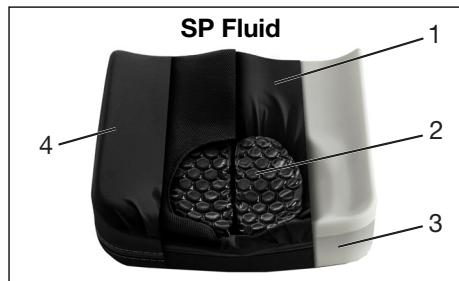

1. Pleated 4-Way Stretch Inner Cover	2. Hydrolite Fluid Bladder
3. Dual Density Polyurethane Foam Base	4. 4-Way Stretch Outer Cover

MAXIMUM USER WEIGHT FOR BOTH SP CUSHIONS:
158kg 350 lbs (158 kg)

B. Intended Use

Axiom Seating Cushions are intended to provide postural support, pressure distribution and improved sitting tolerance.

C. Your Safety

Any serious adverse events or injuries related to the use of your product or its accessories must be reported immediately if the incident directly or indirectly leads to serious health decline or death of the user or other person. Report any serious events and/or injuries to Ki Mobility and, if required by local regulation, the competent health authority where the user and/or other person legally reside.

Do not use this product without first reading this entire manual. BEFORE riding, you should be trained in the safe use of this product by an Assistive Technology Professional (ATP) or other competent clinical or technical professional. Ki Mobility manufactures many different products that might meet your needs. The recognized best practice for selecting a product is to consult with an ATP or other competent technical professional and an experienced clinical professional such as a physical therapist, occupational therapist or physician. Final selection of the type of product, options and adjustments rests solely with you and your technical and clinical professionals. The options you choose, and the set-up and adjustment of the product have a direct impact on its performance, stability and its ability to meet your needs.

D. Signal Words

Within this manual you will find what are referred to as "Signal" words. These words are used to identify and convey the severity of varying hazards. Before using this chair you, and each person who may assist you, should read this entire manual. Please note the Signal word and consider any notes, cautions or warnings. Make sure to follow all instructions and use your chair safely. The Signal word refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons. The "Warnings" are in three main categories, as follows:

NOTE: "Note" indicates a potentially hazardous situation which, if not avoided, could result in a decline in the performance of the chair or damage to your wheelchair.

CAUTION: "Caution" indicates a potentially hazardous situation which, if not avoided, could result in damage to your wheelchair and injury to you.

WARNING: "Warning" indicates a potentially hazardous situation which, if not avoided, could result in serious injury or death.

These signal words will be placed throughout the manual, where appropriate to highlight the hazardous situation. Refer to Section F. for hazardous situations that will apply to the general use of this wheelchair.

E. Symbol Glossary

Symbol	Definition
	Indicates the medical device manufacturer.
	Date of manufacture (YEAR/MM/DD).
	If presented on the medical device or packaging, it indicates the need for the user to consult the instructions for safety reasons such as cautions and warnings.
	Indicates the item is a medical device.
	Indicates the manufacturer's serial number so that a specific medical device can be identified.
	Indicates the authorized representative in the European Community.
	Indicates the manufacturer's declaration that the product meets the requirements of the applicable EC directives.
	Indicates the need for the user to consult the instructions for use.
	Indicates the need for the user to consult the listed website for instructions for use in an electronic format.
	Indicates a website where a user may obtain additional information about the medical product.

Symbol	Definition
	Indicates a specified maximum weight limit (lbs/kg).
	Indicates the entity importing the medical device into the locale.
	Indicates the manufacturer's batch code so that the batch or lot can be identified.
	Machine Wash, Hot
	Do Not Bleach
	Line dry or tumble, no heat
	Do Not Iron
	Do Not Dry Clean
	Indicates the manufacturer's declaration that the product meets the requirements of the applicable UKCA directives.

F. Check Your Skin Daily

These Axiom Seating and Positioning products are designed to improve pressure distribution and increase sitting tolerance. Whether the cushion provided will meet your individual needs cannot be assured in advance. The integrity of your skin can be affected by many aspects of your daily life and medical condition, including the use of this product. Be sure to follow any skin care regimens established by your clinician.

Users of this product should make sure their skin is inspected daily for changes. If you see changes, discontinue the use of this product immediately and consult with your clinician. A key clinical indicator of tissue breakdown is skin redness. If your skin develops redness, discontinue the use of this product immediately and consult your clinician. Be especially careful to inspect the areas related to the bony prominences of the pelvis such as your trochanters and ischial tuberosities.

WARNING: Failure to perform proper skin inspection or continuing to use a cushion after skin changes are present can affect your skin integrity, resulting in serious injury or death.

G. General Warnings

WARNING: Do not use or install this cushion without first reading and understanding this manual. Please contact your authorized Ki Mobility supplier or your clinician if you are unable to understand the Warnings and Instructions. Failure to do so could result in serious injury or death.

WARNING: The adhesive on hook and loop closures (commonly known as "Velcro") may degrade over time and reduce its ability to secure the cushion. This could cause you to fall from your wheelchair, resulting in serious injury.

WARNING: The installation of a cushion on a wheelchair could affect the center of gravity of the wheelchair. Changes in your center of gravity may affect your stability in your wheelchair, resulting in tipping over or falling from your wheelchair which may result in serious injury. Always review the instructions for use of your wheelchair to see if changes to the wheelchair may be needed to provide sufficient stability after adding a cushion.

WARNING: The integrity of your skin can be affected by many aspects of your daily life and medical condition, including the use of this product. Be sure to follow any skin care regimens established by your clinician. Consumers of this product should make sure their skin is inspected routinely for changes as directed by their clinician. Failure to do so could result in serious injury or death.

WARNING: Axiom SP Fluid - Do not leave the fluid cushion outside overnight at temperatures below 40°F (5°C). Allow the cushion to warm to room temperature before using. Sitting on extremely cold or hot surfaces can affect your skin integrity, resulting in serious injury or death.

H. Protecting Your Cushion

Avoid prolonged exposure to adverse environmental conditions. This may include excessive exposure to sunlight and temperatures, either hot or cold. This can result in the degradation of the performance of your cushions.

WARNING: Failure to protect your cushion may lead to changes that can affect your skin integrity, resulting in serious injury or death.

Although your cushion has been manufactured in accordance with its labeled flammability standards, you should avoid exposing your cushion to open flames or extreme heat sources such as a burning cigarette. This may cause your cushion to ignite or emit harmful gases.

⚠ WARNING ⚠

WARNING: Failure to protect your cushion from extreme heat may cause your cushion to ignite or emit harmful gases, resulting in serious injury or death.

I. Obstructions

To ensure your cushion works effectively, avoid placing obstructions between your bottom and your cushion. Obstructions may include, but are not limited to:

- Keys or other items in pockets
- Lift slings
- Underpads or Chucks


Obstructions will lessen the effectiveness of the product, potentially causing pressure or high pressure points.

⚠ WARNING ⚠

WARNING: Pressure points can affect your skin integrity, resulting in serious injury or death.

J. Cover Orientation

Your cushion was designed to be sat on in a specific orientation and it must be placed properly in your wheelchair with the cover oriented correctly with the foam base. It is labeled to indicate the proper orientation.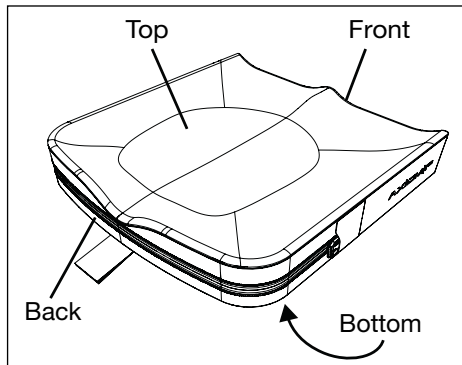


⚠ WARNING ⚠

WARNING: Failure to properly orient the cushion and cover can affect your skin integrity, resulting in serious injury or death.

SET-UP, ADJUSTMENT AND CARE

A. Fitting Your Cushion and Wheelchair

1. Initial Cushion Set-Up Without User in Seated Position
 - In determining the right cushion for you, ensure the dimensions are appropriate considering the width and length of your wheelchair sling or seat pan. If you are unsure, consult your authorized Ki Mobility supplier or your clinician.
 - Place the cushion in the chair with the material identification tags facing the rear of the chair and the cushion handle oriented toward the front of the chair. The non-skid texture should be facing the seat sling or seat pan on your wheelchair.
 - It is important that during normal use the cushion is retained in the wheelchair. It is also important that the cushion is not displaced during transfers that might cause you to slip or fall. Your cushion has hook and loop closures and a non-skid bottom to facilitate retention in a wheelchair. Ensure the cushion hook and loop closures align and mate with the wheelchair hook and loop closures. If your wheelchair sling or seat pan does not have hook and loop closures, or they do not align with the cushion, you should consult your authorized Ki Mobility supplier or your clinician to ensure the cushion is appropriate.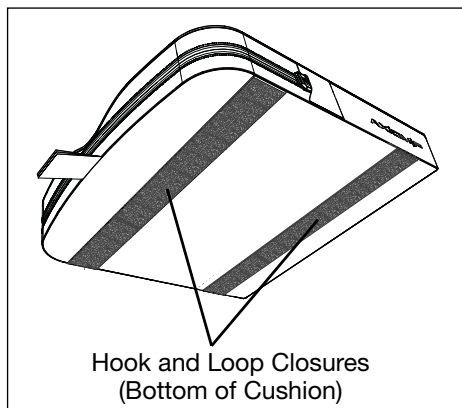

⚠ WARNING ⚠

WARNING: Failure to properly select a cushion that fits your wheelchair can affect your skin integrity which can result in serious injury or death.

⚠ WARNING ⚠

WARNING: Failure to properly orient the cushion and cover can affect your skin integrity which can result in serious injury or death.

⚠ WARNING ⚠

WARNING: Failure to properly select the cushion and cover that can be retained in the wheelchair can cause you to fall from your wheelchair, resulting in serious injury.

⚠ WARNING ⚠

WARNING: Avoid using the cushion on overly stretched seat sling upholstery. Stretched upholstery does not provide adequate support for the cushion and improper positioning may result. This can affect your skin integrity, resulting in serious injury or death.

2. Wheelchair Set-Up and Adjustment for your Cushion
 - a. Ensure hips are placed properly in the back of the wheelchair and cushion as directed by your authorized Ki Mobility supplier or your clinician.
 - b. Ensure your footrests are adjusted to the proper height, as directed by your authorized Ki Mobility supplier or your clinician.
 - c. Ensure your back support is properly positioned and adjusted for your seated posture, as directed by your authorized Ki Mobility supplier or your clinician.

⚠ WARNING ⚠

WARNING: Failure to ensure that the wheelchair is properly adjusted for use with the cushion can affect your skin integrity, resulting in serious injury or death.

B. Maintenance

Regular cleaning and maintenance may help extend the life of your cushion. You should inspect your cushion weekly. During cleaning, component inspection is recommended.

1. Check Cover for Tears and Excessive Wear
 - a. Remove cover and check for abnormalities (inside and out).
 - b. Check the foam base for breakdown or degradation.
 - Tears deeper than 0.25" (6.35 mm).
 - Loose granules of foam.
 - Permanent changes to the unloaded foam shape, such as flattening.
 - c. Inspect the fluid bladder (SP Fluid) for tears or punctures. Inspect the area around the bladder and the cover for the presence of a thick, white fluid which will indicate the presence of a leak.

⚠ WARNING ⚠

WARNING: If you find any abnormalities, such as the foam breaking down, discontinue the use of this product immediately and consult with your authorized Ki Mobility supplier or clinician. Failure to do so can affect your skin integrity, resulting in serious injury or death.

⚠ WARNING ⚠

WARNING: In the event of corrosive spills or exposure to bio-hazards, remove contaminants immediately or serious injury or death could occur. See the Cleaning section in this manual for instructions.

C. Cleaning

⚠ WARNING ⚠

WARNING: Cleaning, deodorizing and sanitizing, even using the method described here, can accelerate the natural degradation of the foam. The effect will vary depending on the chemicals used. Verify that the degradation has not progressed to the point the cushion should not be used, according to the instructions for use provided with the product, failure to do so can result in serious injury.

Care & Use Instructions	
	Machine Wash, Hot
	Do Not Bleach
	Line dry or tumble, no heat
	Do Not Iron
	Do Not Dry Clean

1. Cleaning the Cover

NOTE: Drying with heat may cause shrinking that affects the covers fit and function.

- a. Remove the outer cover from the foam base and zip closed.
- b. Machine wash in warm water and tumble dry - no heat. Do not iron. Note - Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach.
- c. To refit cover after washing, fit cover back over foam base. Ensure the back of the cover matches up to the back of the foam base. Note - If utilized in an institutional setting, write the residents name with a permanent marker on the content label of the cover. This will

ensure the return of the correct cover to its matching cushion after washing.

2. Cleaning or Deodorizing the Foam Base

NOTE: Do not immerse your foam base in any solution.

- a. Fill a spray bottle with detergent or deodorizer. Verify that the labeling for the manufacturer of the detergent or deodorizing indicates it is appropriate for this use. Follow the manufacturer's labeling for proper solution concentration and use.
- b. Lay the foam base in the bathtub or on a surface that can get wet.
- c. Spray the entire surface of the foam base including the sides. Flip it over and repeat on the bottom. Once you've finished spraying, let the solution set for 20 to 30 minutes.
- d. Wipe the solution off with a damp washcloth. Do not spray with water.
- e. Allow the material to air dry but avoid direct sunlight. Depending on how wet your foam base is, it may take 24 hours to several days to completely dry. Verify that no solution remains in the foam base before using.

3. Sanitizing the Foam Base

NOTE: Do not immerse your foam base in any solution.

- a. Fill a spray bottle with a 70% or greater alcohol solution.
- b. Remove the fluid bladders (SP Fluid).
- c. Lay the foam base in the bathtub or on a surface that can get wet.
- d. Spray the entire surface of the foam base including the sides. Flip it over and repeat on the bottom. Once you've finished spraying, let the solution set for 20 to 30 minutes.
- e. Wipe the solution off with a damp wash cloth. Do not spray with water.
- f. All the material to air dry but avoid direct sunlight. Depending on how wet your foam base is, it may take 24 hours to several days to completely dry. Verify that no solution remains in the foam base before using.

4. Cleaning or Deodorizing Fluid Pads (SP Fluid)

NOTE: Never immerse the fluid pads in any solution. Never clean with abrasive materials.

- a. Remove the cover from the foam base.
- b. Remove fluid pads from the inner cover.
- c. Fill a spray bottle with detergent or deodorizer. Verify that the labeling for the manufacturer of the detergent or deodorizing indicates that it is appropriate for this use. Follow the manufacturer's labeling for proper solution concentration and use.
- d. Spray and wipe the bladders with a clean dry cloth.
- e. Make sure the bladders are completely dry before reassembly.

5. Sanitizing the Fluid Pads (SP Fluid)

NOTE: Never immerse the fluid pads in any solution. Never clean with abrasive materials.


- a. Remove the cover from the foam base.
- b. Remove fluid pads from the inner cover.
- c. Fill a spray bottle with a 70% or greater alcohol solution.
- d. Spray and wipe the bladders with a clean dry cloth.
- e. Make sure the bladders are completely dry before reassembly.

D. Reassembling Your Axiom Cushion After Cleaning

Axiom SP Visco

After Proper Cleaning and Maintenance:


1. Insert the foam base into the cover with the label down and the zipper to the rear of the cushion.
2. Insert base plus inner cover into the outer cover. Ensure that white inner cover label is down and zipper is to the rear.
3. Close the zipper on your cushion ensuring that the zipper-pull ends up closed with the pull tucked into the zipper pocket cover.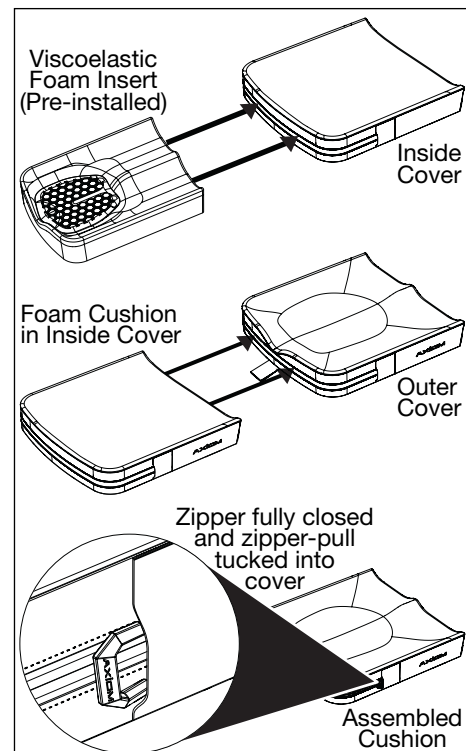

Axiom SP Fluid

After Proper Cleaning and Maintenance:

1. Insert foam base into the inner cover.
2. Install the fluid insert by mating the hook and loop closure sections from the fluid insert onto the hook and loop closure sections on the inner cover.
3. Install base/inner cover/fluid insert assembly into the outer cover.
4. Close the zipper on your cushion ensuring that the zipper-pull ends up closed with the pull tucked into the zipper pocket cover.

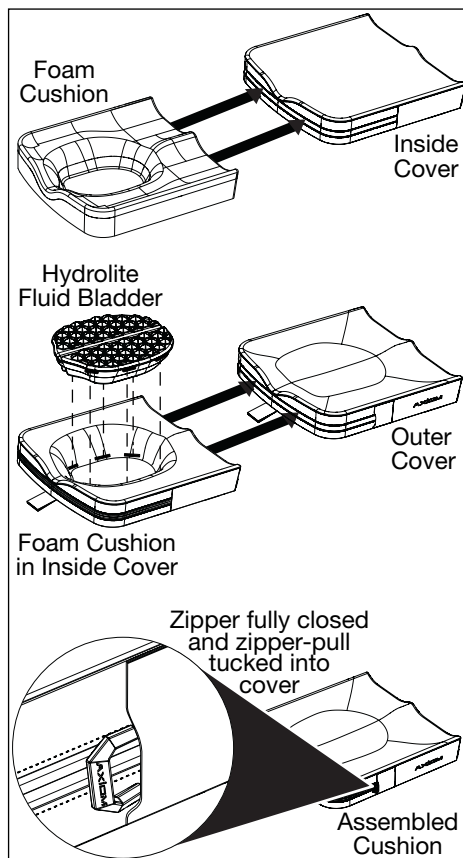WARNING

WARNING: Place the Axiom SP fluid pad on the inner cover ensuring that all hook and loop closure locations are aligned. A misalignment or creation of an edge could cause an unwanted pressure point which could lead to injury.

E. Axiom SP Fluid - Bottoming Out

WARNING

WARNING: If bottoming out is detected or you are unsure whether you are bottoming, discontinue the use of this product immediately and consult with your authorized Ki Mobility supplier or your clinician. Failure to do so can affect your skin integrity, resulting in serious injury or death.

Bottoming may occur on an Axiom SP Fluid cushion if you displace all the fluid underneath your pelvic bones or buttocks, which would leave you sitting on the foam base. It sometimes occurs on very thin individuals, people using recliner wheelchairs, people who have lost weight or have other changes in body size or shape, or people who slouch when sitting.

To check for bottoming out, first make sure all other instructions for fitting the cushion and wheelchair are complete. Then sit on the cushion, with the cover on and the zipper opened. For seven to ten minutes. Transfer up and off the cushion (or have someone help you transfer), trying not to disturb the fluid underneath you. Fold the back portion of the cover forward over the front, exposing the fluid pad. Push down in the depressions on the pad where your ischial tuberosities (seat bones) and coccyx (tailbone) were. You should have to push through at least 0.5" (1.3 cm) of fluid before you feel the firm cushion base below.

Changes to the cushion and changes in your body occur over time and you should check for bottoming out at least monthly.

KI MOBILITY LIMITED WARRANTY

The expected life of the frame is five years.

A. For 24 Months

Ki Mobility warrants the cushion to be free from defects in materials and workmanship for a period of 24 months. This limited warranty applies to:

- Axiom SP Cushion Base, Axiom SP Fluid Pad & Axiom SP Visco Cushion Base

For a period of 24 months (2 years) from the date of purchase, provided normal use, should a defect in materials or workmanship occur, Ki Mobility will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears, or burns.

B. For 6 Months

Ki Mobility warrants the cushion to be free from defects in materials and workmanship for a period of 6 months. This limited warranty applies to:

- Axiom SP Fluid & Axiom SP Visco Removable Outer and Inner Covers

For a period of 6 months from the date of purchase, provided normal use, should a defect in materials or workmanship occur, Ki Mobility will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns.

C. Limitations

1. We do not warrant:
 - a. Damage from neglect, accident, misuse, or from improper installation or repair.
 - b. Products modified without Ki Mobility's express written consent.
 - c. Damage from exceeding the weight limit.
2. This warranty applies in North America only. Check with your supplier to find out if international warranties apply.
3. This warranty is not transferable and only applies to the first consumer purchase of this product through an authorized Ki Mobility supplier.

D. What We Will Do

Our sole liability is to repair or replace covered parts. This is the exclusive remedy for consequential damages.

E. What You Must Do

1. Obtain from us, while this warranty is in effect, prior approval for return or repair of covered parts.
2. Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

F. Notice to Consumer

1. If allowed by law, this warranty is in place of any other warranty (written or oral, express or implied, including a warranty of merchantability, or fitness for a particular purpose).
2. This warranty gives you certain legal rights. You may also have other rights that vary from state to state.

Record your lot number here for future reference:

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U.S.A.