# Index

User manual – English .....	3
Bruksanvisning – Svenska.....	8
Bruksanvisning – Norsk .....	13
Brugervejledning – Dansk.....	18
Käyttöohjeet – Suomi.....	23
Gebrauchsanweisung – Deutsch ...	28
Gebruiksaanwijzing – Nederlands ...	33
Instruction d'utilisation – Français ...	38
Istruzioni per l'uso – Italiano .....	43
Instrucciones de uso – Español ....	48

See [www.etac.com](http://www.etac.com)

# General

## Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, please read this manual carefully.

The person described as “the user” in this “Instruction for Use” is the person who is lying or sitting on the product. The carers are the people who manoeuvre the product.

**This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the user or carer.**

These products comply with the standards applicable for Class 1 products in the European Council Directive MDD 93/42/EEC on medical devices.

At Etac we strive to continually improve our products and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations

and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

For further information on the Etac transfer range, see [www.etac.com](http://www.etac.com).

The product can be scrapped in accordance with national regulations.

## Symbols in general

Washing

pH level

Read the user manual

User mass limit = maximum rated load

Do not bleach

Wipe off

Warning

The product can be scrapped according to national regulations

Tumbledry

Never leave on the floor

Production year, month and day

Manufacturer  
Class 1 - European Council Directive MDD 93/42/EEC on medical devices

Iron

Risk of sliding down

Batch no./Lot no.

Do not dryclean

Keep out of the sun

## Intended use

Support arms during CT/MR/PET scanning.

## Intended environment

Acute care

# Practical handling

---

## Positioning

The scanner cushion can be positioned to best suit the individual user and to help maintain the user's position.

## Use

The user feels more comfortable and relaxed in the position.

# Notice

## Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling. Etac offers advice and training for carers. For further information, contact Etac.

Always use the correct manual handling techniques

Encourage the user to assist where possible

To ensure that the user feels safe and that every manual handling is smooth, always plan the manual handling in advance.

Never leave the product on the floor.

Always conduct a risk assessment, and ensure that the assistive product can be used for the individual user and, in combination with other devices, that it is safe for the user and carers.

If in any doubt - please contact Etac.

## Material and cleaning

### Material:

Plastazote LD-29

The product should be checked regularly, preferably each time it is used, and especially after washing.

Check that there is no damage to seams or fabric.

### Washing instructions:

This product can be wiped off with soapy water or cleaned with a solvent-free cleaning agent with a pH level between 5-9, or a 70% disinfection solution.

**Never try to repair a product yourself.  
Never use a defective product.**

## Products and accessories

---

Article no.	Product Description	Size (mm)
IM502	Scanner ArmSupport	L290xW510xH200

# Service information

## Inspection

**Thank you for choosing an Etac product.** If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid.

**Warranty:** Two-year warranty on material and manufacturing defects, provided that the product is used correctly. Before taking a product into service always conduct a risk assessment according to national and/or local legislation.

### First inspection:

- Is the packaging intact?
- Read the label on the packaging and check the article no. and product description.
- Check that Short Instruction is enclosed – updated User Manual/Instruction for use can be downloaded from [www.etac.com](http://www.etac.com) or contact Etac Customer Service or your local distributor.

- Check the label on the product - does it include article no., product description, lot/batch no., cleaning instructions and supplier name?

### Periodic Inspection:

- Make sure that your Manual transfer system is always in perfect condition.
- After wash – always check materials, stitching, handles, buckles.
- If the product shows signs of wear and tear, it must be removed from service immediately.

#### Visual inspection/Check the product:

Ensure that the materials, seams, stitching, handles, buckles are intact/faultless.

#### Stability test:

Try to bend the product and verify that it feels firm, stable and solid.

#### Mechanical load or stress/Test handles:

Pull hard on the handles in opposite directions and check the material, seams and stitching.

#### Test buckles:

Lock the buckle and pull in opposite directions, check the buckle and the stitching.

#### Test low/high friction:

Place the product on a firm surface or a bed, place your hands on the material and test the friction by pushing your hands down into the product.

Low friction – slides effortlessly

High friction – no sliding/movement

## Check

### Manual transfer aids with handles, straps and buckles:

### Manual transfer aids for sitting and lying transfer:

### Manual transfer aids with high or low friction:

The product can be scrapped in accordance with national regulations

Read the User Manual

If in doubt, please contact Etac Customer Service or local distributor for more information and guidance.

For more information about training and the use of these products, contact the Etac transfer experts – [www.etac.com](http://www.etac.com).

Class 1 - European Council  
Directive MDD 93/42/EEC  
on medical devices

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Etac AB
Etac Immedia A/S