Instruction for use

IMM1040_IFU.Version9.2023-02-13

GlideCushion

immedia
by Etac

Index

Illustrations.....	3
User manual – English	4
Bruksanvisning – Svenska.....	9
Bruksanvisning – Norsk	14
Brugervejledning – Dansk.....	19
Käyttöohjeet – Suomi.....	24
Gebrauchsanweisung – Deutsch ...	29
Gebruiksaanwijzing – Nederlands ...	34
Instruction d'utilisation – Français ...	39
Istruzioni per l'uso – Italiano	44
Instrucciones de uso – Español	49

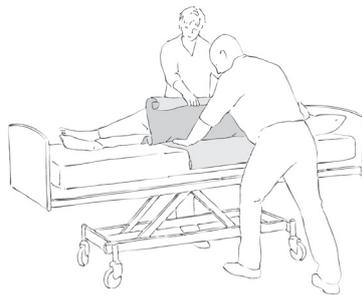
See www.etac.com

Illustrations

1

2

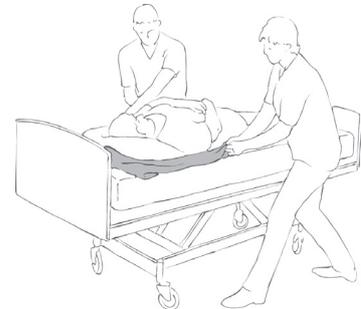
3

4

5

6

7

8

9

10

General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, please read this manual carefully.

The person described as “the user” in this “Instruction for Use” is the person who is lying or sitting on the product. The carers are the people who manoeuvre the product.

This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the user or carer.

These products comply with the standards applicable for Class 1 products in the Medical Device Regulation (EU) 2017/745 on medical devices.

At Etac we strive to continually improve our products and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

Expected lifetime: The device has an expected service life of 1 to 2 years under normal use. The service life of the device varies depending on usage frequency, loads and how often and how it is washed.

For further information on the Etac Patient handling, see www.etac.com.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Symbols in general

Washing	Manual Cleaning	Warning	Manufacturer
Do not bleach	Never leave on the floor	Production year, month and day	CE-marking
Tumbledry	Risk of sliding down	Batch no./Lot no.	Keep dry
Iron	Keep out of the sun	User mass limit = maximum rated load	Non sterile
Do not dryclean	Read the user manual	The product can be scrapped according to national regulations	Article number
Medical device	Do not add fabric conditioner	UKCA marked	

Intended use

GlideCushion can be used anywhere where it is beneficial to reduce friction at pressure points during manual handling:

Turning users in bed, pulling them higher up in the bed, getting in and out of bed, etc.

Intended environment

Acute care, Long-term care, Home care

Practical handling

Positioning

GlideCushion can be positioned under pressure points – where the user “clings” to the underlying surface.

Use

Positioning GlideCushion for turning:

Fold GlideCushion – roll the user slightly to one side and push GlideCushion in under the user’s body (Illustration 1). Turning the user will be easier if an absorbent sheet is used (Illustration 2). On the opposite side, GlideCushion can be folded out completely. If the user only has to be turned on their side, GlideCushion only has to be positioned under one hip. GlideCushion can also be positioned under the user’s thigh, so that the folded wing is facing the user. To unfold GlideCushion, unfold one wing at a time. The user can now be moved/turned as required. Alternative method: Fold GlideCushion loosely. Carer A grasps the sheet on Carer B’s side of the bed, and gently rolls the user. GlideCushion is pushed in under the heavy pressure points (most often shoulders and hips). Then the user is rolled back onto his/her back. If necessary, Carer B can roll the user as before while Carer A smoothens out GlideCushion (Illustration 2).

Turning without carer assistance:

By folding one arm over his/her chest and bending the leg on the same side, the user can turn onto their side without assistance (Illustration 10).

Turning with assistance from 1 or 2 carers:

The user can also be turned in this way: one of the carers stands with one foot forward and grasps the sheet using a “flourbag grip” or an “axe grip”. Then, with arms stretched at the level of the users shoulders and hips and using weight transference, the user is pulled to the edge of the bed (Illustration 3). The carer bends slightly at the knee, grasps the sheet holding his/her elbows close into the body. As the carer straightens up, the user turns on their side. If necessary, the carer on the opposite side can help by pushing on the sheet and GlideCushion, and supporting the user when he/she reaches the side position. Variant: Carer B rolls the user, while Carer A pulls on the sheet, thus turning the user on their side.

To remove GlideCushion:

The carer glides one hand between the two sheet layers, locates the corner on the opposite side and pulls it slowly towards him/herself.

Positioning GlideCushion higher up in the bed:

The user has slid down (Illustration 4). The user’s pillow and GlideCushion are placed under the user’s head. The carer pulls GlideCushion down until it is under the user’s shoulder blades (Illustrations 5 and 6). If the user can lift his/her hips (if possible while the foot end of the bed is raised slightly), the user’s feet can be positioned on a piece of non-slip fabric and the user will glide further up in the bed. If necessary, a carer can push gently on the user’s knee. A Sling or OneManSling can also be used to help the user glide higher up (Illustration 7). Alternatively a hoist can be used: Raise the user’s hips very gently from the sheet. The user will glide slowly to lie higher up in the bed. To remove GlideCushion: a carer pulls the corner towards the opposite side. The carer then grasps the corner and pulls slowly (Illustration 9). Self-supportive users, who find it difficult to turn/move in bed, can use GlideCushion without assistance. (Illustration 10).

In/out of bed:

When a user is to get in or out of bed, the distance between GlideCushion and the end of the bed should be not less than 15 cm.

Notice

Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling. Etac offers advice and training for carers. For further information, contact Etac.

Always use the correct manual handling techniques.

Encourage the user to assist where possible.

To ensure that the user feels safe and that every manual handling is smooth, always plan the manual handling in advance.

There is a risk that the user may slide off. Never leave the user alone on the edge of the bed.

Never leave the product on the floor.

Always conduct a risk assessment, and ensure that the assistive product can be used for the individual user and, in combination with other devices, that it is safe for the user and carers.

It is recommended that the guard rail is in place when the user is left on the glide system, unless a risk assessment has found that the user can be safely left on the glide system without the need of guard rails.

If in any doubt - please contact Etac.

Material and cleaning

Material:

Outer surface: 65% polyester/35% cotton or 100% nylon

Filling: Polyester (fibre)

Inner surface: Nylon.

The product should be checked regularly, preferably each time it is used, and especially after washing.

Check that there is no damage to seams or fabric.

**Never try to repair a product yourself.
Never use a defective product.**

Washing instructions:

Recommended to wash the product before use.

Do not use fabric softener.

This product can be cleaned with a solvent-free cleaning agent with a pH level between 5 and 9, or a 70% disinfection solution.

Products and accessories

Article no.	Product Description	Size (mm)
IM60/50	GlideCushion S	W600xL500
IM60/90	GlideCushion M	W600xL900
IM69/70	GlideCushion L	W690xL700
IM90/90	GlideCushion XL	W900xL900
IM6000N	GlideCushion Cover S	W600xL600
IM4035	Cover Disposable S. For IM60/90. 100 pcs/roll	W600xL1000

Set

Article no.	Product Description	Size (mm)
IM4303	GlideCushion S set Incl. IM60/50 + IM6000N	W600xL500

ILL.

GB

SV

NO

DA

FI

DE

NL

FR

IT

ES

Service information

Inspection

First inspection:

- Is the packaging intact?
- Read the label on the packaging and check the article no. and product description
- Check that Short Instruction is enclosed – updated User Manual/Instruction for use can be downloaded from www.etac.com or contact your Etac Customer Service or the local dealer.
- Check the label on the product - does it include article no., product description, lot/batch no., cleaning instructions and supplier name?

Periodic Inspection:

- Make sure that your Manual transfer system always is in a perfect condition.
- After wash – always check materials, stitching, handles, buckles.
- If the product shows signs of wear and tear, it must be removed from service immediately.

Visual inspection/Check the product:

Ensure that the materials, seams, stitching, handles, buckles are intact/faultless.

Stability test:

Try to bend the product verify that it feels firm, stable and solid.

Mechanical load or stress/Test handles:

Draw hard in the handles in opposite directions and control the material, seams and stitching. Test buckles:

Lock the buckle and draw in opposite directions, control the buckle and the stitching.

Test low/high friction:

Place the product on a firm surface or a bed, place your hands on the material and test the friction by pushing your hands down into the product.

Low friction – slides effortlessly

High friction – no sliding/moving

Check:

Manual transfer aids with handles, straps and buckles:

Manual transfer aids with for sitting and lying transfer and with high or low friction:

Production year, month and day

The product can be scrapped in accordance with national regulations

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Etac Ltd, Unit 60
Etac Immedia A/S