User manual

SatinSheet

immedia
by Etac

IMM1071_IFU - 2026-01-28 - Rev. 1.0

www.etac.com

Symbols

3

General

11

English	11
Svenska	16
Norsk	21
Dansk	26
Suomi	31
Deutsch	36
Nederlands	41
Français	46
Italiano	51
Español	56

Application suggestions

61

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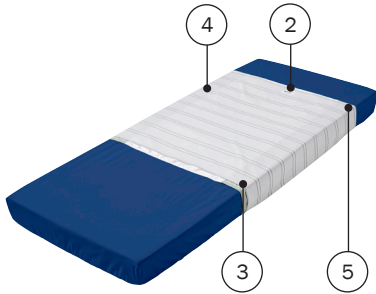
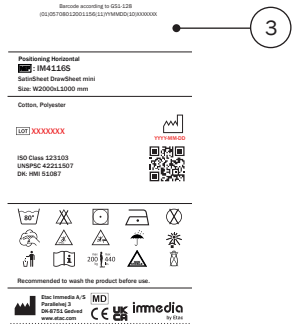
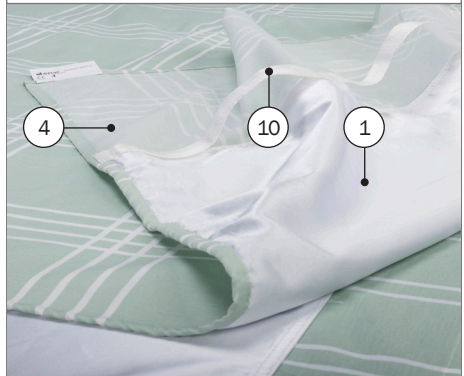
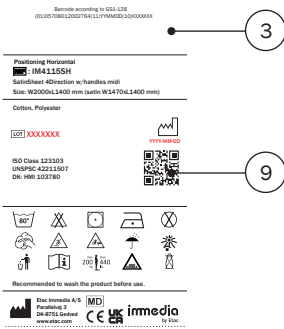
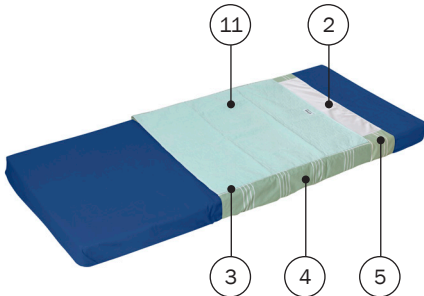
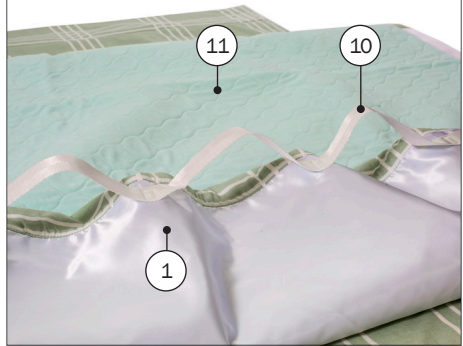
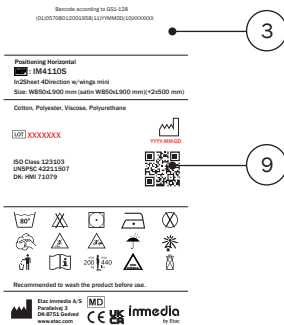
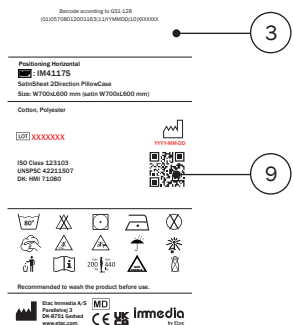
General

Thank you for choosing an Etac device.

This manual must be read thoroughly before you start using the device as it contains important safety information and gives instructions on how to use the device correctly. Visit www.etac.com for all relevant product documentation, for example the user manual, maintenance and recycling instructions. You can select your language via the “International” and “Local websites” link.

You can also consult the QR code on the device for further information and instructions.

Copyright

This document is the property of Etac Immedia A/S. The content shall not be modified in whole or part without prior approval of Etac Immedia A/S.

Disclaimer

Etac continuously makes improvements to our devices and information. We therefore reserve the right to modify our devices and instructions for use without prior notice. Consult www.etac.com to find the latest version of the documentation.

CE Declaration of Conformity

The device mentioned in this manual and the different models hereof comply with the Medical Device Regulation (EU) 2017/745.

The device is classified as a Class I medical device.

Product standards

The device has been tested and complies with the requirements applicable for this type of device in the below mentioned standards:

- ISO 21856:2022 Assistive products — General requirements and test methods.

- DS/EN ISO 20342-1:2019 Assistive products for tissue integrity when lying down - Part 1: General Requirements.

- EN 1021-1:2014 and EN ISO 12952-1:2010 resistance to ignition by cigarette (but not flame).

Warranty

Please refer to www.etac.com/support.

If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

Notice

Any serious incident that has occurred in relation to the device should be reported to your distributor/representative and the national competent authority within your region in a timely manner. The distributor/representative will forward the information to the manufacturer.

Pre-sales information

Please visit www.etac.com.

Device description

Etac supplies a wide selection of Positioning Horizontal devices for different types of transfer and positioning situations.

SatinSheet facilitates safe and comfortable movements in bed for independent to dependent users. SatinSheet supports turning, and frequent repositioning in lying positions or in need of mobilisation frequently.

SatinSheet consists of a BaseSheet, which features a low-friction satin panel in the centre. This satin area enables side-to-side movement, making it easier to turn the user in bed. SatinSheet BaseSheets are available in various sizes and models to suit different needs.

SatinSheet DrawSheet can be added for a partly dependent/independent user who needs assistance with movement. SatinSheet DrawSheets are designed to be used on top of the BaseSheet and do not work independently.

SatinSheet 4Direction DrawSheet are available in multiple sizes, models, and variants. These include options that allow movement in two or four directions, with or without handles. Drawsheet with handles are compatible with hoist attachment for assisted turning, and an incontinence version is also available. See www.etac.com for a complete overview of possible variants.

Clinical benefits

The clinical benefits of the devices are:

- Reduces the need for carers or carer's time
- Reduces work-related injuries or strain of carer
- Enhance user's well-being
- Reduces risk of shear or pressure injury
- Facilitates micro-and macro movements
- Enables users to remain independent
- Facilitates assisted movement
- Relieves pain
- Reduces the risk of infection
- Enables early rehabilitation

Introduction

Definitions

The person referred to as “the user” is the person who is transferred in or positioned with the device. “The carer” is the person who manages the device.

Intended purpose

The Positioning Horizontal are assistive devices intended for alleviation of, or compensation for, a functional impairment due to an injury or disability. The devices are designed for a user requiring reduction of friction at pressure points due to reduced mobility or physical strength.

Intended user

The intended users of the devices are based on the users' functional ability and not on a specific diagnosis, health condition or age. Ensure the user complies with the SWL/user mass and that a clinical risk assessment is conducted.

No limitation on height.

Secondary users of the devices are carers providing assistance and clinicians/technicians who set up the device.

Intended environment

The devices are intended for indoor use.

Intended application

The device is intended for ‘Long term’ use, meaning intended for continuous use for more than 30 days.

The device is intended for use with intact skin only.

The device is intended for refurbishment and re-use.

Expected service life

The expected service life of Positioning Horizontal are 2 years. The lifetime of the device may vary de-

Materials

Stribed material: 50% cotton/50% polyester.

Satin: 100% polyester.

Absorbing material: 100% polyester (micro-fiber), 80% polyester/20% viscose filling, polyurethane barrier and 100% nylon.

Absorption: 2200 ml/m².

Elastic band: 75% polyester, 25% elasthan.

pending on usage frequency, loads and washing/drying frequency and conditions.

Indications

Indication for use is disability, including, but not limited to, immobile or passive users, who are in need of support to be safely and comfortably positioned and turned in lying positions or in need of mobilisation or repositioned frequently to minimise the risk of pressure ulcers and other adverse events associated with immobility. Other medical conditions are patients with pain or discomfort, patients where rolling would be dangerous, pediatrics, geriatric, patients with dementia or psychiatric problems, and obese persons.

Contraindications

There are no known contraindications.

Precautions

Do not let the device block the user's airways.

If risk of pressure injury, do not lock the system after repositioning.

If risk of involuntary movements lock the system.

Before locking the glide system under the mattress, any creases should be smoothed to avoid decubitus.

It is recommended that the guard rail is in place when the user is left on the glide system, unless a risk assessment has found that the user can be safely left on the glide system without the need of guard rails.

Training requirements

Only carers and/or users who have read and understood the user manual are allowed to use the device.

Safety information

The safety of both user and carer must never be jeopardized – if you suspect that the device has a malfunction you must put it in quarantine marked “out of order” and call your local distributor/representative.

Safety signs, safety measures and warning symbols are given in the context relevant for the actual handling and/or operating situation – it is mandatory the given instruction is followed to avoid dangerous/hazardous situations.

Signs, symbols and instructions placed on this device must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the device. Immediately replace or repair illegible or damaged signs, symbols, and instructions. Contact your distributor/representative for instructions.

See additional warnings in the Handling section.

Patient transfer

Transfer of a user will always pose a certain risk. Therefore, a risk assessment must always be carried out before the transfer is performed.

The assistive device can be used for the individual user and in combination with other devices as long as it is safe for the user and carers.

Warnings:

- Pull/Push with a minimum of force.
- There is a risk that the user may slide off.
- Never leave the device on the floor.

Risk assessment

Do a risk assessment of the device, the individual user and the positioning, and document your analysis. You as the carer are responsible for the safety of the user.

Plan the positioning in advance to ensure that it is performed as safe and smooth as possible.

Remember to work ergonomically according to local recommendations.

Encourage the user to assist where possible.

The device

Warning:

- Make sure the BaseSheet fits the mattress.
- Make a risk assessment to secure right combination is used.
- DrawSheet should not be used without BaseSheet.
- When the carers have finished repositioning/moving the user, the glide system should, if required, be relocked unless the risk assessment states something else.

- Do not use the device as a sling.
- Keep handles and straps away from the user.

Materials

All used materials which are to be in direct contact with skin are biologically assessed and evaluated according to relevant standards. However, users/carers who are allergic to any of the materials are recommend not to use the device.

Combinations

When combining the device with other devices or accessories, the combination must not change the intended purpose of the devices or modify the devices in such a way that compliance with the applicable requirements may be affected. The person/company responsible for the combination of the devices must ensure that the applicable requirements are fulfilled.

Mounting/assembly

All SatinSheet models feature a “This Side Up” label to ensure correct orientation when placed on the bed. Additionally, each device includes an information label sewn into the fabric, providing guidance on proper usage.

- Place the SatinSheet BaseSheet with the satin low-friction surface facing upward.
- Ensure the high-friction surface is facing the mattress.
- If necessary, a drawsheet can be laid on SatinSheet, 4Direction DrawSheet are placed satin against satin.

Safety measures

Do not let the device block the user's airways.

Service, cleaning and maintenance

Maintenance and service

Maintenance and inspections of the device is the responsibility of the owner of the device. Failure to maintain a device in accordance with the instructions may compromise the safety of the user and/or carer and may affect the expected lifetime of the device.

Inspection of the device must be carried out regularly.

Check:

- that the materials, seams, stitching, and handles are intact/faultless.
- never to repair a device yourself.
- recommended to wash the product before use.

Cleaning

80 °
176F

Ironing improves the glide effect.

This device can be cleaned with a solvent-free cleaning agent with a pH level between 5 and 9, or a 70% disinfection solution.

Shrinks 5%.

Do not use fabric softener.

Reconditioning

SatinSheet is suitable for reuse after inspection and cleaning.

Periodic inspection

The device should be checked regularly, preferably each time it is used, and especially after washing.

Transportation, storage and disposal

Transportation and storage

When the device is not in use, it can be stored under the following conditions:

Keep packaging away from children.

Keep away from fire.

The device can be stored and transported under temperatures between -30 to +40 °C.

Disposal

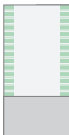
The device contains no hazardous substances that require special disposal. Check if there are options available for recycling in an environmentally friendly way and follow the national provisions that apply. Dispose of the device according to the national provisions that apply.

Recycling

See the recycling instructions on www.etac.com, and follow the national provisions that apply.

In-bed system selection guide

Level of functioning

	Level A	Level B	Level C	Level D	Level E
	For independent users, who turn themselves into bed.	For partly dependent users, who are able to bend knees and push off with feet.		For very dependent users, who need assistance moving legs/feet.	
Choose length of Immedia SatinSheet BaseSheet					
	SatinSheet BaseSheet Mini	SatinSheet BaseSheet Midi		SatinSheet BaseSheet Maxi	
Choose type of Immedia SatinSheet DrawSheet					
		SatinSheet 2Direction DrawSheet Mini or Midi	SatinSheet 4Direction DrawSheet Midi	SatinSheet 4Direction DrawSheet Maxi	
		If assistance is needed during sideways positioning (B).	If assistance is needed in all directions during transfers, but user is able to use feet (C).	If assistance is needed in all directions during transfers and the user is unable to use their feet, use handles for turning with a hoist (D/E).	

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EN: Information

01. BaseSheet for users who can turn independently but need reduced friction.
02. Before locking the glide system under the mattress, any creases should be smoothed by the carers pulling simultaneously and diagonally from both sides.
03. SatinSheet DrawSheet on top of SatinSheet BaseSheet if user needs assistance turning.
04. To lock the glide system after turning: Fold the excess material from DrawSheet under the mattress on both sides of the bed.
05. SatinSheet 4Direction DrawSheet on top of SatinSheet BaseSheet enables up/down and side-to-side movement. For repositioning, grasp the Drawsheet at hips and shoulders with an ergonomic grip. Two carers can assist, one by pulling the SatinSheet while one pushing at hips and shoulders.
06. To move higher in bed, pull at the shoulders using weight transfer or press on the mattress while adjusting with the Drawsheet.
07. With SatinSheet 4Direction with handles, the user can be turned using a hoist: attach handles to the sling bar, turn slowly, support the back with a cushion or wedge, then lower the hoist. Remove handles from the sling bar.
08. To prevent sliding when sitting up, fold part of the SatinSheet 4Direction under the buttocks and tuck it under the mattress at the hips for a braking effect.

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**Etac Ltd.**