User manual

4WayGlide

immedia
by Etac

IMM1072_IFU - 2026-04-07 - Rev. 1.0

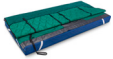
www.etac.com

Symbols

4

General

12

English	12
Svenska	17
Norsk	22
Dansk	27
Suomi	32
Deutsch	37
Nederlands	43
Français	48
Italiano	53
Español	58

Application suggestions

63

4WayGlide Application	64
-----------------------	----

4WayGlide Handling	66
--------------------	----

Warnings	70
----------	----

Information	71
-------------	----

EN: Warning: This symbol appears in the user manual along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger.

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EN: Information: This symbol appears in the user manual along with a number referring to supplementary information which can be found in the information section.

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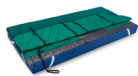
General

Thank you for choosing an Etac device.

This manual must be read thoroughly before you start using the device as it contains important safety information and gives instructions on how to use the device correctly. Visit www.etac.com for all relevant product documentation, for example the user manual, maintenance and recycling instructions. You can select your language via the “International” and “Local websites” link.

You can also consult the QR code on the device for further information and instructions.

Copyright

This document is the property of Etac Immedia A/S. The content shall not be modified in whole or part without prior approval of Etac Immedia A/S.

Disclaimer

Etac continuously makes improvements to our devices and information. We therefore reserve the right to modify our devices and instructions for use without prior notice. Consult www.etac.com to find the latest version of the documentation.

CE Declaration of Conformity

The device mentioned in this manual and the different models hereof comply with the Medical Device Regulation (EU) 2017/745.

The device is classified as a Class I medical device.

Product standards

The device has been tested and complies with the requirements applicable for this type of device in the below mentioned standards:

- ISO 21856:2022 Assistive products — General requirements and test methods.

- DS/EN ISO 20342-1:2019 Assistive products for tissue integrity when lying down - Part 1: General Requirements.

- EN 1021-1:2014 and EN ISO 12952-1:2010 resistance to ignition by cigarette (but not flame).

Warranty

Please refer to www.etac.com/support.

If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

Notice

Any serious incident that has occurred in relation to the device should be reported to your distributor/representative and the national competent authority within your region in a timely manner. The distributor/representative will forward the information to the manufacturer.

Pre-sales information

Please visit www.etac.com.

Device description

Etac supplies a wide selection of Positioning Horizontal devices for different types of transfer and positioning situations.

4WayGlide facilitates safe and comfortable movements in bed for partly independent to dependent users. 4WayGlide supports turning, and frequent repositioning in lying positions or in need of mobilisation frequently. The system consists of two components a 4WayGlide Mattress which is designed to be used on top of the NylonSheet and the system can be used for an independent/partly dependent/dependent users who needs assistance with movement.

NylonSheet, this base sheet features a low-friction nylon panel in the centre, that enables movement in all directions together with a 4WayGlide mattress. NylonSheet's are available in various sizes and models, with different numbers of locking zones to suit individual needs. NylonSheet's do not work independently. 4WayGlide Mattresses are available in multiple sizes, variants and models to suit different needs. All 4WayGlide Mattresses are equipped with handles that also allow for hoist attachment, enabling safe and efficient turning. 4WayGlide Mattress are also available in a wipeable version and an absorbent surface model known as TwinSheet4Glide.

See www.etac.com for a complete overview of possible variants.

Clinical benefits

The clinical benefits of the devices are:

- Reduces the need for carers or carer's time
- Reduces work-related injuries or strain of carer
- Enhance user's well-being
- Reduces risk of shear or pressure injury
- Facilitates micro-and macro movements
- Enables users to remain independent
- Facilitates assisted movement
- Relieves pain
- Reduces the risk of infection
- Enables early rehabilitation

Materials

NylonSheet:

100% nylon.

100% polyester with polyurethane coating.

Elastic band: 75% polyester, 25% elasthan.

Handles: 100% polyester.

4WayGlide Mattress:

Quilted 50% cotton/50% polyester, 100% polyester filling, and 100% nylon. Filling: 100% polyester.

Handles: 100% polyester. 100% Polyester mesh (Maxi).

Hook and Loop: 100% polyester.

TwinSheet4Glide:

100% polyester (microfiber), 80% polyester/20% viscose filling, polyurethane barrier and 100% nylon. Absorption: 2200 ml/m².

Handles: 100% polyester.

Hook and Loop: 100% polyester.

Introduction

Definitions

The person referred to as “the user” is the person who is transferred in or positioned with the device. “The carer” is the person who manages the device.

Intended purpose

The positioning horizontal are assistive devices intended for alleviation of, or compensation for, a functional impairment due to an injury or disability. The devices are designed for a user requiring reduction of friction at pressure points due to reduced mobility or physical strength.

Intended user

The intended users of the devices are based on the users’ functional ability and not on a specific diagnosis, health condition or age. Ensure the user complies with the SWL/user mass and that a clinical risk assessment is conducted.

No limitation on height.

Secondary users of the devices are carers providing assistance and clinicians/technicians who set up the device.

Intended environment

The devices are intended for indoor use.

Intended application

The device is intended for ‘Long term’ use, meaning intended for continuous use for more than 30 days.

The device is intended for use with intact skin only. The device is intended for refurbishment and re-use.

Expected service life

The expected service life of Positioning Horizontal are 2 years. The lifetime of the device may vary depending on usage frequency, loads and washing/drying frequency and conditions.

Indications

Indication for use is disability, including, but not limited to, immobile or passive users, who are in need of support to be safely and comfortably positioned and turned in lying positions or in need of mobilisation or repositioned frequently to minimise the risk of pressure ulcers and other adverse events associated with immobility. Other medical conditions are patients with pain or discomfort, patients where rolling would be dangerous, paediatrics, geriatric, patients with dementia or psychiatric problems, and obese persons.

Contraindications

There are no known contraindications.

Precautions

Do not let the device block the user’s airways.

If risk of pressure injury, do not lock the system after repositioning.

If risk of involuntary movements lock the system.

Position the user so that the seam does not risk causing pressure sores.

Before locking the glide system under the mattress, any unevenness/creases should be smoothed to avoid decubitus.

It is recommended that the guard rail is in place when the user is left on the glide system, unless a risk assessment has found that the user can be safely left on the glide system without the need of guard rails.

Keep handles and straps away from the user.

Training requirements

Only carers and/or users who have read and understood the user manual are allowed to use the device.

Safety information

The safety of both user and carer must never be jeopardized – if you suspect that the device has a malfunction you must put it in quarantine marked “out of order” and call your local distributor/representative.

Safety signs, safety measures and warning symbols are given in the context relevant for the actual handling and/or operating situation – it is mandatory the given instruction is followed to avoid dangerous/hazardous situations.

Signs, symbols and instructions placed on this device must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the device. Immediately replace or repair illegible or damaged signs, symbols, and instructions. Contact your distributor/representative for instructions.

See additional warnings in the Handling section.

Patient transfer

Transfer of a user will always pose a certain risk. Therefore, a risk assessment must always be carried out before the transfer is performed.

Warnings:

- Pull/Push with a minimum of force.
- There is a risk that the user may slide off.
- If risk of pressure injury, do not lock the system after repositioning.
- If risk of involuntary movements lock the system.
- Position the user so that the seam does not risk causing pressure sores.
- Keep handles and straps away from the user.

Risk assessment

Do a risk assessment of the device, the individual user and the positioning, and document your analysis. You as the carer are responsible for the safety of the user.

Plan the positioning in advance to ensure that it is performed as safe and smooth as possible.

Remember to work ergonomically according to local recommendations.

Encourage the user to assist where possible.

The device

Warning:

- Make sure the NylonSheet fits the mattress.
- Make a risk assessment to ensure the right combination is used.
- Mattress should not be used without NylonSheet.
- When the carers have finished repositioning/ moving the user, the glide system should, if required, be relocked unless the risk assessment states something else.

- If reduced freedom of movement in the upper body is required in the seated position, you can lock the glide system at both sides at the shoulders by fitting corner straps around the mattress.
- Do not use the device as a sling.

Materials

All used materials which are to be in direct contact with skin are biologically assessed and evaluated according to relevant standards. However, users/carers who are allergic to any of the materials are recommend not to use the device.

Accessories

User manuals supplied with accessories must be read and understood before use. This includes any limitations in use and combinations.

Information about accessories can be found on www.etac.com.

Combinations

When combining the device with other devices or accessories, the combination must not change the intended purpose of the devices or modify the devices in such a way that compliance with the applicable requirements may be affected. The person/company responsible for the combination of the devices must ensure that the applicable requirements are fulfilled.

Mounting/assembly

All NylonSheets and Mattres's feature a “This Side Up” label to ensure correct orientation when placed on the bed. See application section.

- Place the NylonSheet with the nylon low-friction surface facing upward.
- Ensure the high-friction surface is facing the mattress.
- Mattress is placed nylon against nylon.

Safety measures

Do not let the device block the user's airways.

Service, cleaning and maintenance

Maintenance and service

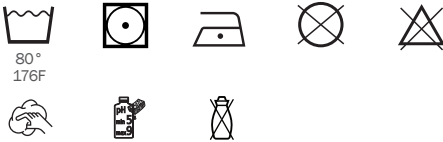
Maintenance and inspections of the device is the responsibility of the owner of the device. Failure to maintain a device in accordance with the instructions may compromise the safety of the user and/or carer and may affect the expected lifetime of the device.

Inspection of the device must be carried out regularly.

Check:

- that the materials, seams, stitching, and handles are intact/faultless.
- never to repair a product yourself.

Cleaning

Washing instructions: Recommended to wash the product before use. Shrinks 5%. Do not use fabric softener. This product can be cleaned with a solvent-free cleaning agent with a pH level between 5 and 9, or a 70% disinfection solution.

During washing and cleaning the hook and loop must be assembled or use the “Loop Washing Strap”.

Reconditioning

4WayGlide is suitable for reuse after inspection and cleaning.

Periodic inspection

The device should be checked regularly, preferably each time it is used, and especially after washing.

Transportation, storage and disposal

Transportation and storage

When the device is not in use, it can be stored under the following conditions:

Keep packaging away from children.

Keep away from fire.

The device can be stored and transported under temperatures between -30 to +40 °C.

Disposal

The device contains no hazardous substances that require special disposal. Check if there are options available for recycling in an environmentally friendly way and follow the national provisions that apply. Dispose of the device according to the national provisions that apply.

Recycling

See the recycling instructions on www.etcac.com, and follow the national provisions that apply.

In-bed system selection guide

Level of functioning

	Level A	Level B	Level C	Level D	Level E
	For independent users, who turn themselves in bed.	For partly dependent users, who are able to bend knees and push off with feet.		For very dependent users, who need assistance moving legs/feet.	
Choose length of Immedia SatinSheet BaseSheet					

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EN: Information

01. Secure NylonSheet Mini under the mattress with wings; non-slip back keeps it in place.
02. Center the 4WayGlide Mini on the sheet (nylon against nylon) with “this side up” labels aligned. Independent users can operate it themselves.
03. Secure NylonSheet Midi under the mattress with wings; non-slip back keeps it in place.
04. Center the 4WayGlide Midi on the sheet (nylon against nylon) with “this side up label” aligned. Partly depended user can help with their legs.
05. Place the 4WayGlide NylonSheet on the bed with “this side up” at the head.
06. Secure the elastic band around the bed mattress. Lock the glide system, so the non-slip material is visible if needed.
07. Fit the 4WayGlide or TwinSheet4Glide Long with “this side up” at the head (nylon-to-nylon); attach corner straps if needed.
08. Fit the 4WayGlide Twin or TwinSheet4Glide with “this side up” at the head (nylon-to-nylon); attach corner straps if needed.
09. Position the user in bed. If bedridden, slide the product under them like changing sheets. Ensure seam on 4WayGlide Twin/TwinSheet4Glide won't cause pressure ulcer.
10. 1 lock: To lock/unlock, one carer on each side pulls the nylon sheet handles upward toward the head end, or one carer can switch sides and do it alone.
11. Stop pulling when you feel resistant. One carer holds opposite handles while the other pulls the lock up past the hips; repeat on the other side or one carer can switch sides.
12. 2 locks: To lock/unlock, two carers pull the handles attached to the nylon sheet hanging over the head/foot end of the bed up/downward toward the head or foot end, or one carer can switch sides and do it alone.
13. 3 locks: To lock/unlock, the carer folds the nylon lock, pushes it under the user, the carer pulls the handles from the opposite side and fastens the hook and loop. Repeat for all three locks.
14. The system is unlocked; move the user by pulling handles at shoulders and hips under the 4WayGlide Mattress.
15. Sliding up: Unlock the system. Alternatively hold the handles at the shoulder and hips and pull the user towards you and reposition the user in the middle of the bed.
16. Single-handed care/“2 to 1”/1 carer/person centered care: Push diagonally at hips and shoulders toward the head end, then switch sides, grab handles, and center the user in the bed opposite side with the bed rail.
17. Use a hoist to turn the user: attach mattress handles to the sling bar, lift slightly to reduce weight, then raise until supine. Push or pull handles to position.
18. For personal care, keep the top part attached to the hoist and remove the lower part for access to the lower back, or support with example LeanOnMe Wedges, LeanOnMe Triangles or LeanOnMe Cushion.
19. After repositioning, lock the glide system if required, unless risk assessment says otherwise.
20. Supine positioning: If the user must stay on their side for rest or care, support with LeanOnMe Wedges, LeanOnMe Triangles or LeanOnMe Cushions placed as needed.
21. Before locking the glide system, any unevenness/wrinkles should be smoothed by the carers pulling simultaneously and diagonally from both sides.
22. Seated positioning: When head end is raised, prevent sliding by locking the system, using non-slip material, or using the knee bend. To limit upper-body movement, fastened corner straps.
23. Use the knee bend before raising the head end.
24. Now raise the head end without the user sliding down.
25. The “5 smiles” technique: For comfort, remove skin tension when raising head or feet. First Smile: adjust and reposition the pillow.
26. Second & third Smile: Relieve shoulder tension using an Immedia MultiGlide glove. Slide hand under user toward bed end or move mattress back and forth using handles.
27. Fourth and fifth Smile: Relieve skin tension under legs (Illustration 21); one leg at a time. Use Immedia MultiGlide Glove: slide hand under buttocks and move slowly toward bed end. Pull down in the trousers.
28. After releasing skin tension, support the user in a seated or lying position using LeanOnMe Positioning Cushions, LeanOnMe Wedges, or LeanOnMe Triangles for resting or hygiene situations.

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Etac Ltd.