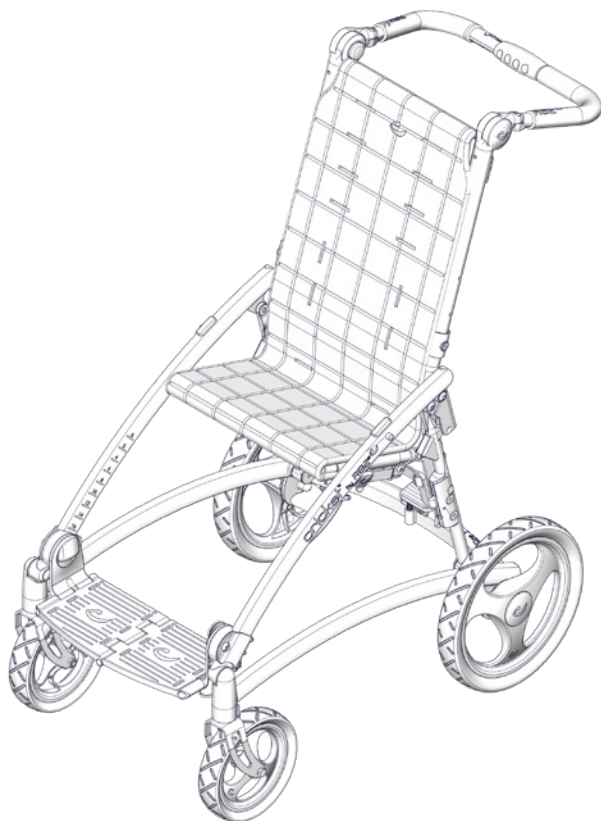
User manual

Cricket

R82[®] 
by Etac

M1046 - 2024-02-05 - Rev. 11

www.etac.com


	3	ENGLISH DEUTSCH	30 - 32 33 - 37
	4	NEDERLANDS DANSK	38 - 41 42 - 44
	5	NORSK SVENSKA	45 - 47 48 - 50
	6	SUOMI ESPAÑOL	51 - 53 54 - 57
	7	FRANÇAIS ITALIANO	58 - 61 62 - 65
	8	PORTUGUÊS PORTUGUÊS DO BRASIL	66 - 69 70 - 73
	9	РОССИЯ POLSKI	74 - 78 79 - 82
	10	CZ 中國	83 - 85 86 - 88
	11 - 12	日本 ΕΛΛΗΝΙΚΗ	89 - 93 94 - 98
	13		
	14 - 24		
	25		
mm (inch) kg (lb)	26 - 27		
	28 - 29		


   05








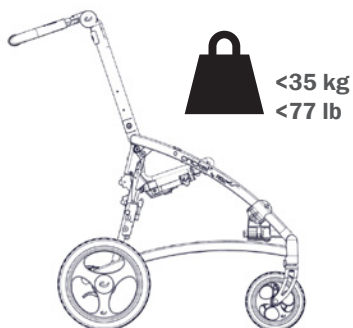
CRICKET 1


CRICKET 2

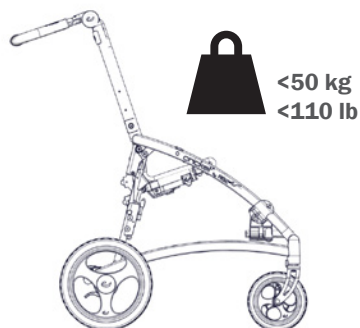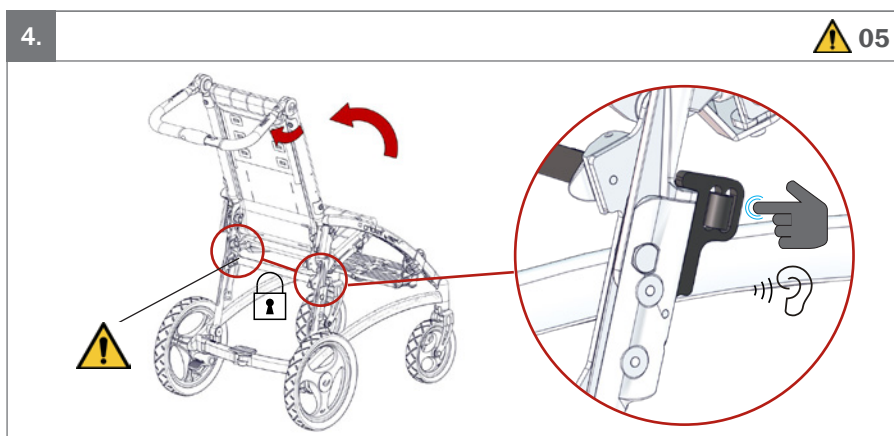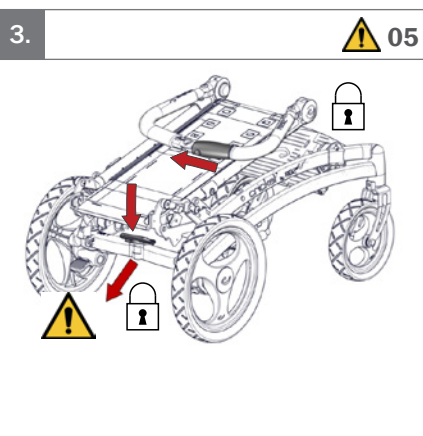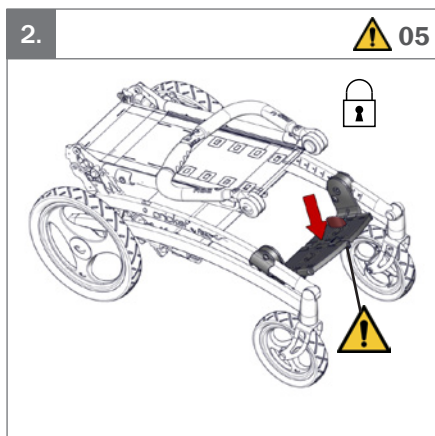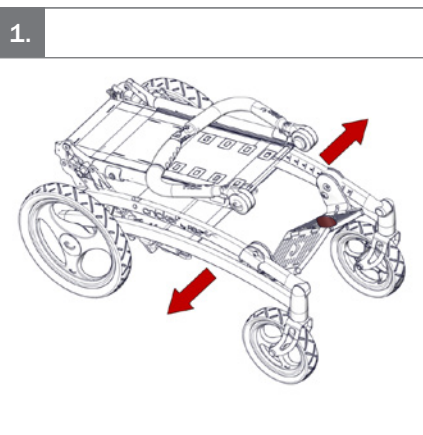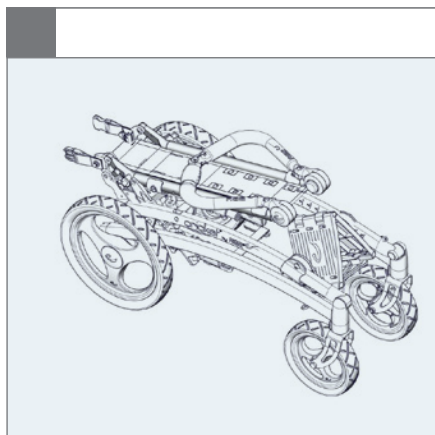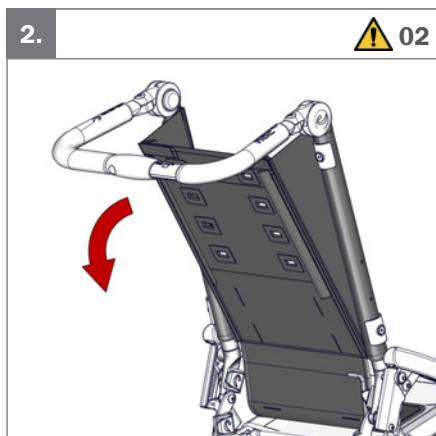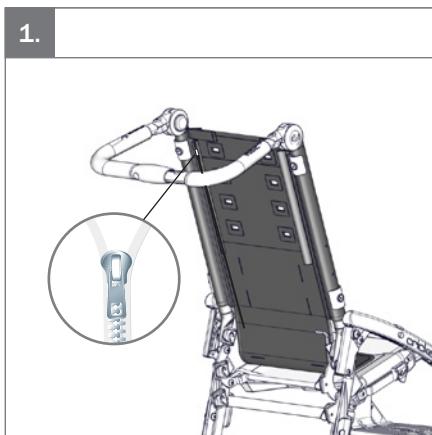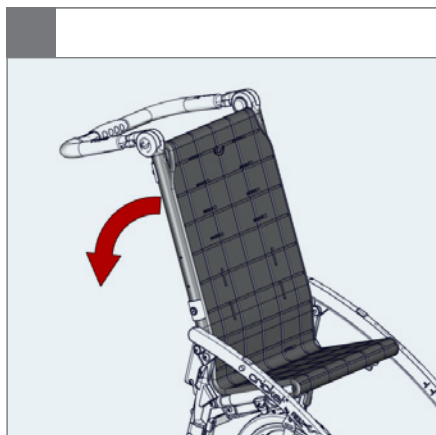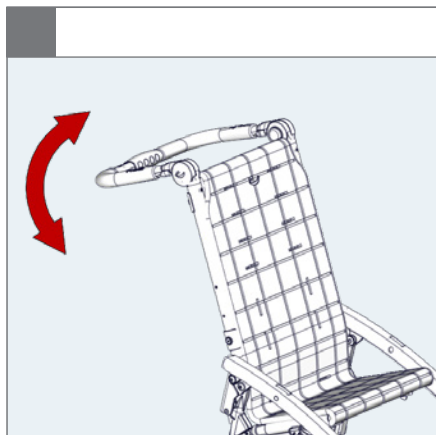1.

 05

2.

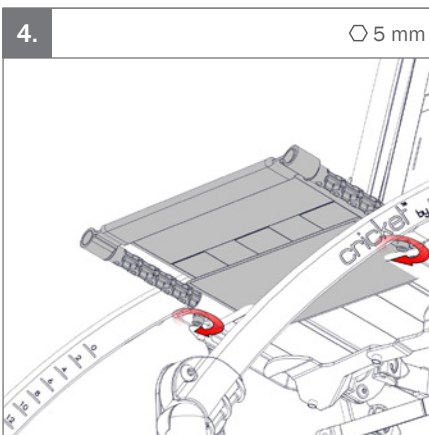
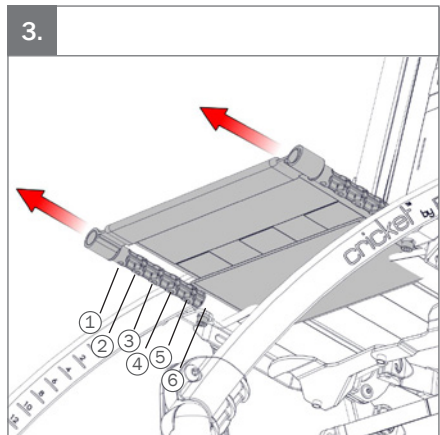
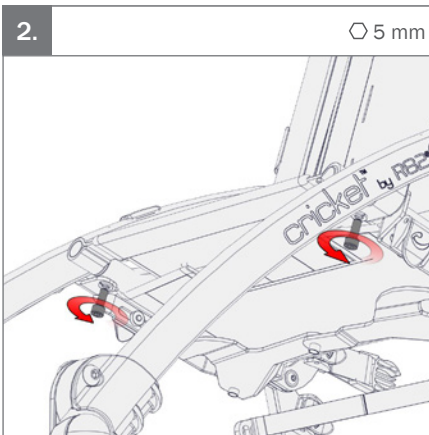
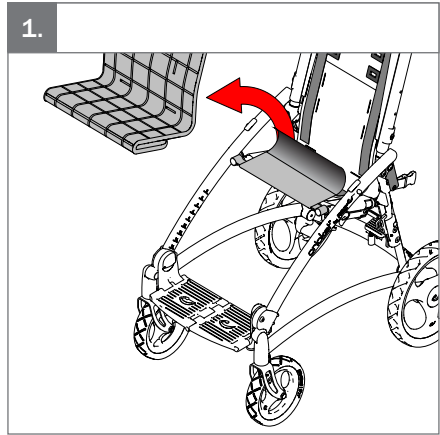
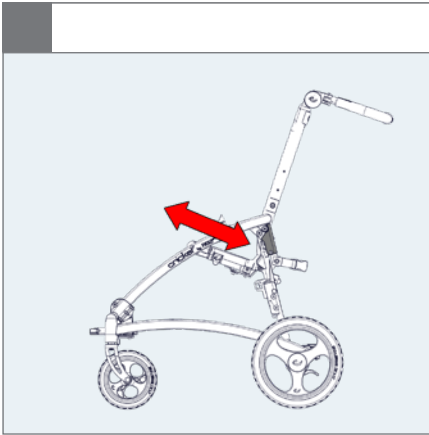
  05

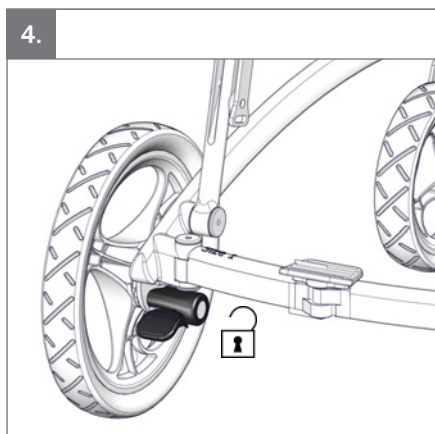
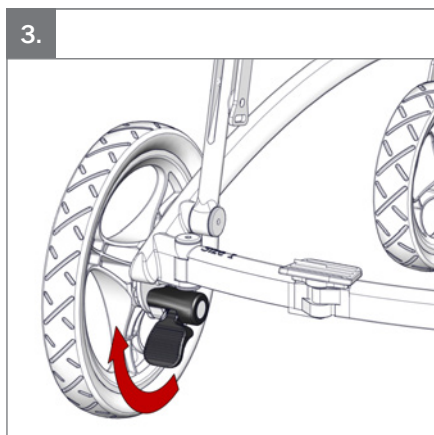
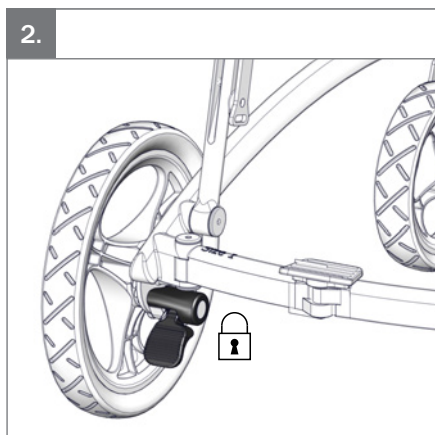
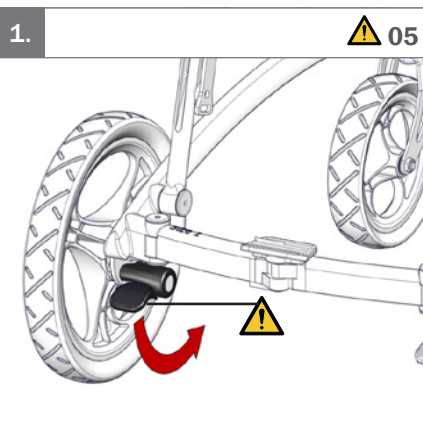
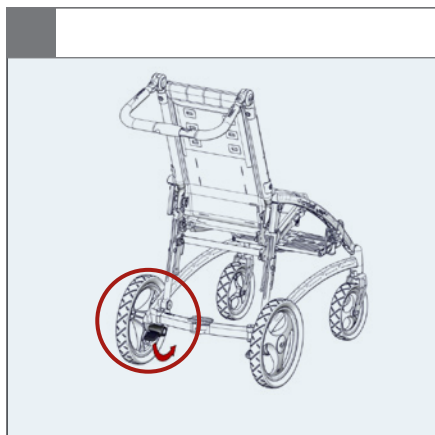
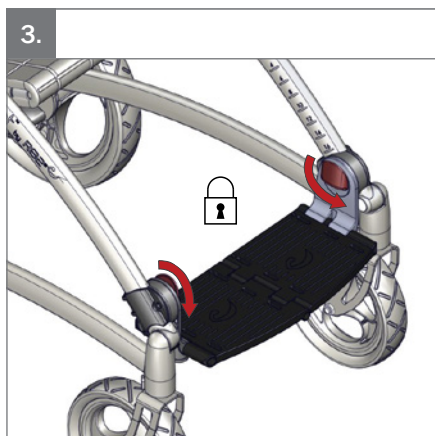
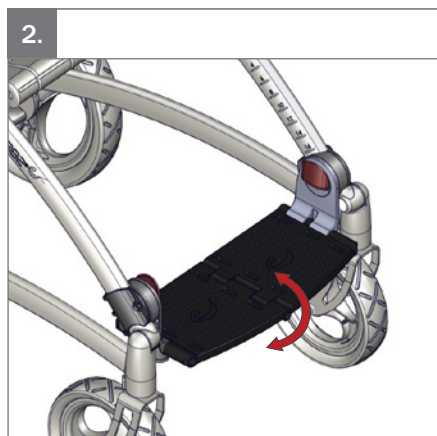
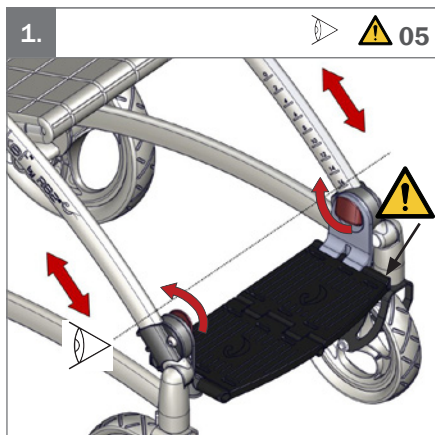
3.

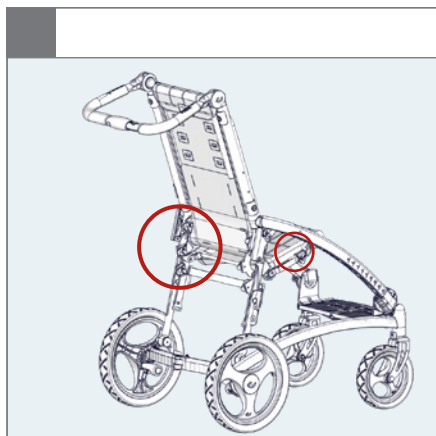
  05



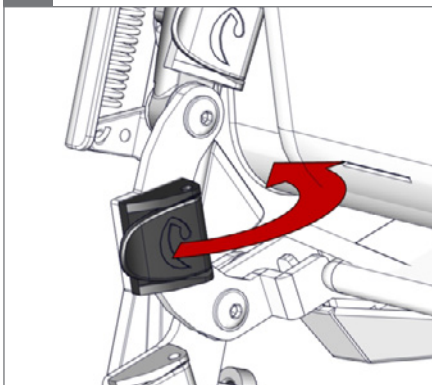
1.





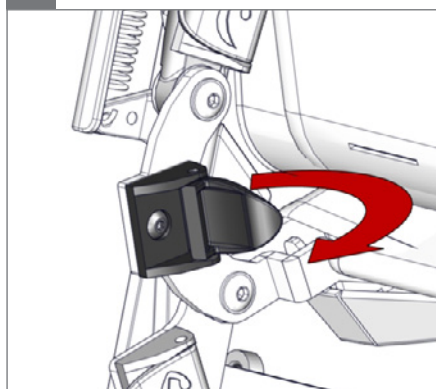
2.

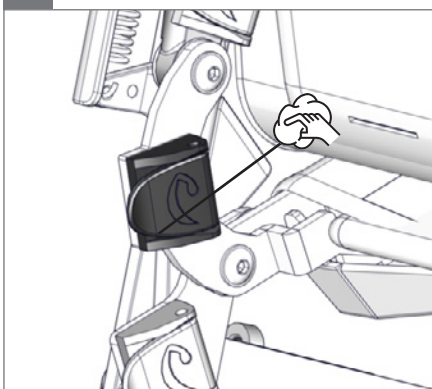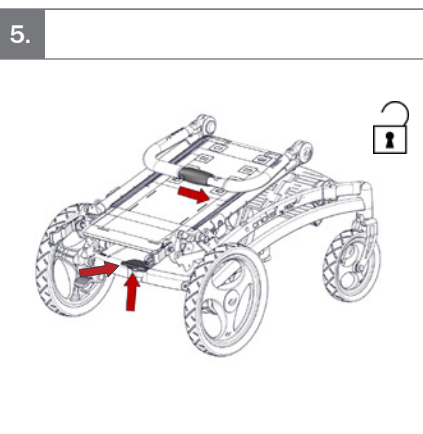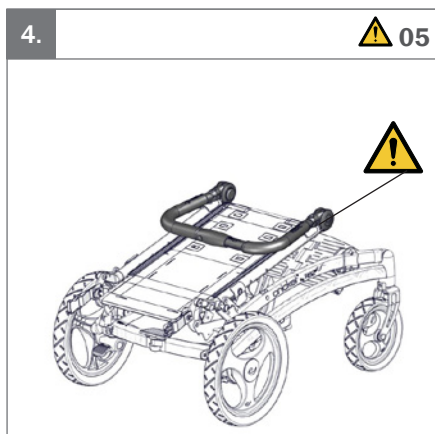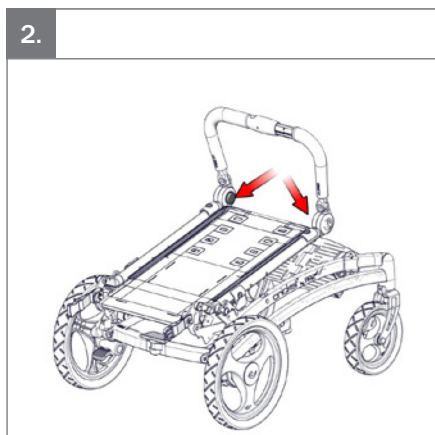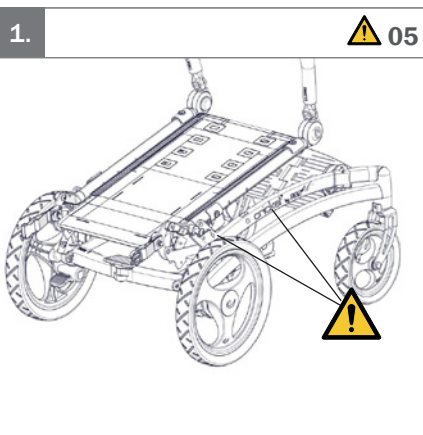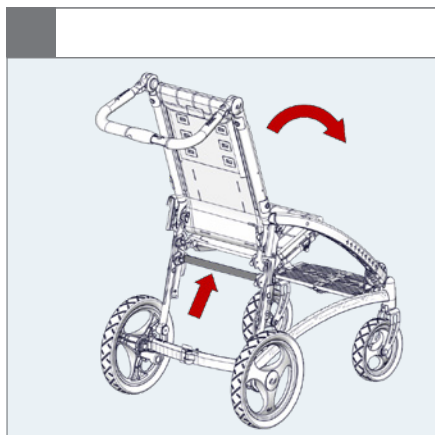
3.

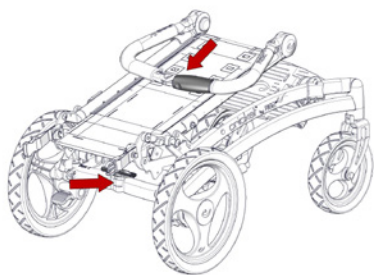
6.


7

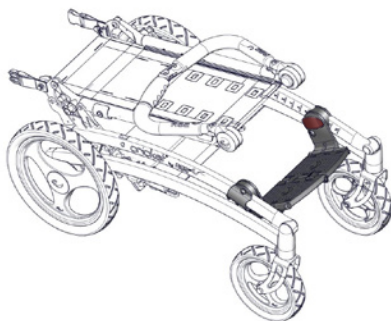
8.

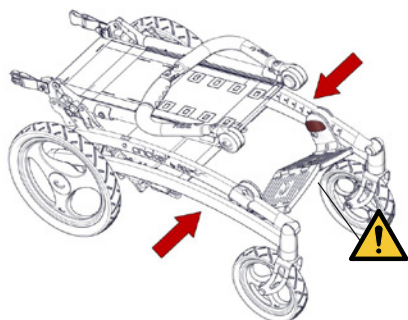
⚠ 05

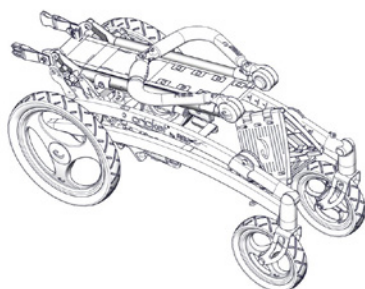
9.

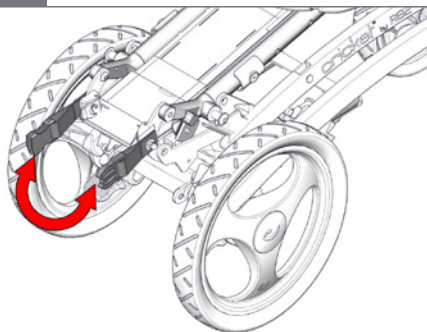
10.

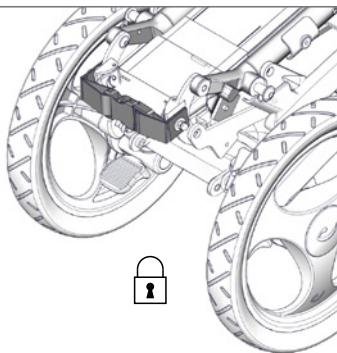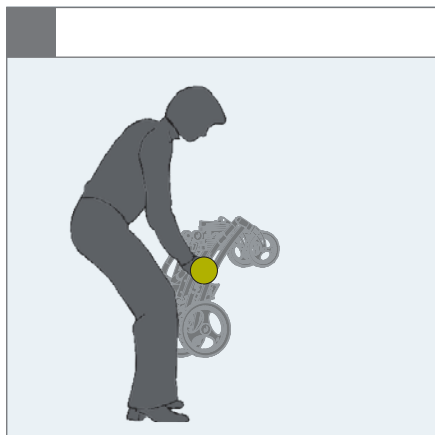
11.

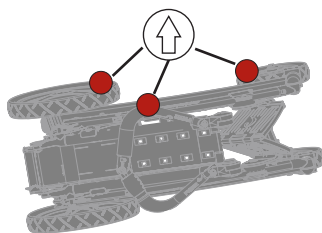1.

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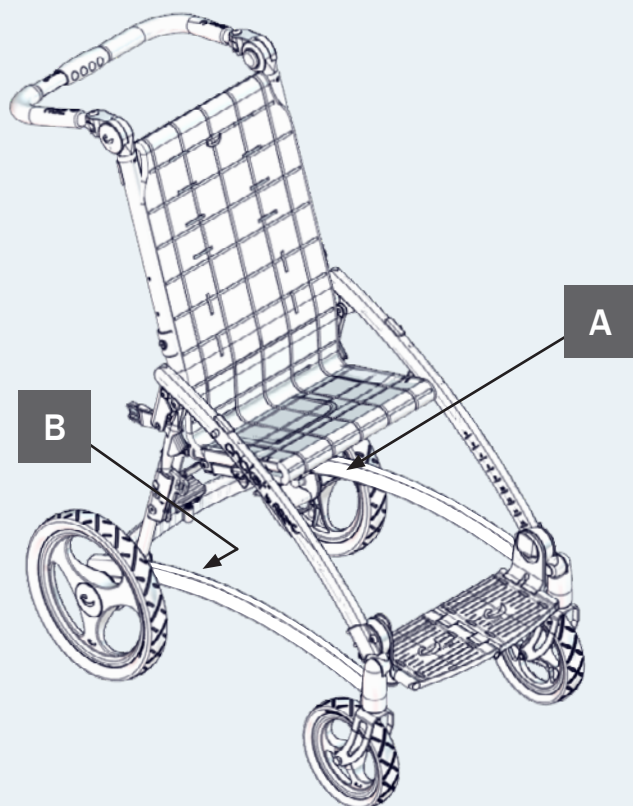
BR xxxxxxxxxxxx

REP xxxxxxxxxxxxxxxxxxxx

LDI (xx)xxxxxxxxxxxxxxxxxx(xx)xxxxxxxxxx


Max. user mass: xxx kg
Max. user mass: xxx kg
Max. rated load: xxx kg
Mass device: xxx kg

mm (inch) kg (lb)

	1	2
 Effective seat width	340 mm (13½")	395 mm (15½")
 Effective seat depth	210–310 mm (8¼–12¼")	310–430 mm (12¼–17")
 Backrest height	500 mm (19¾")	570 mm (22½")
 Footrest to seat distance	180–360 mm (7–14¼")	225–370 mm (8¾–14½")
 Leg to seat surface angle	360°	360°
 Seat surface height at front edge	500–530 mm (19¾–21")	540–580 mm (21¼–22¼")
 Effective back width	270 mm (10¾")	330 mm (13")
 Back width	340 mm (13½")	395 mm (15½")
 Overall width	525 mm (20¾")	575 mm (22¾")
 Length	750 mm (29½")	750 mm (29½")
 Overall height	1000 mm (39¾")	1050 mm (41¼")
 Backrest angle	110°–125°	110°–125°
 Seat plane angle	20°	20°
 Armrest to seat distance	160–190 mm (6¼–7½")	160–190 mm (6¼–7½")
 Front location of armrest structure	200 mm (7¾")	275 mm (10¾")
 Minimum turning radius	725 mm (28½")	763 mm (30")
 Folded width	355 mm (14")	355 mm (14")
 Folded height	485 mm (19")	485 mm (19")
 Folded length	760 mm (30")	760 mm (30")

mm (inch) kg (lb)


	 1	 2
 Max user mass	35 kg (77 lb)	50 kg (110 lb)
 Max user mass at transportation	35 kg (77.1 lb)	43 kg (94.5 lb)
 Mass device	12.4 kg (27.3 lb)	13.4 kg (29.5 lb)
 Max rated load	49.4 kg (108.9 lb)	65.4 kg (144.1 lb)
 Static stability sideways	10°	10°
 Static stability downhill	10°	10°
 Static stability uphill	10°	10°
Required width of right angled corridor	815 mm (32")	815 mm (32")
Front wheel diameter	38×180 mm (1½×7")	38×180 mm (1½×7")
Rear wheel diameter	38×280 mm (1½×11")	38×280 mm (11½×11")

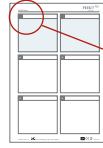
9996097270

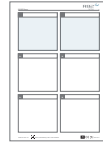9996097271

9996097272

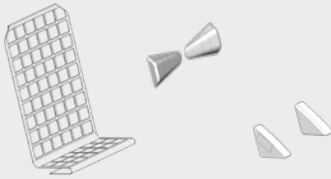9996097274

9996097275

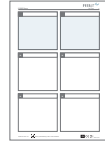
9996097276


9996097277



9996097278


9996097279

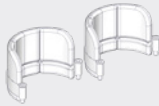9996097280

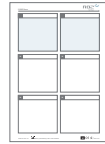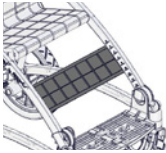9996097282

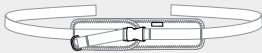9996097630

ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The Cricket is a foldable buggy which comes in 2 sizes. The seat depth can easily be changed, so the buggy can grow with the child. The accessory program make it possible to give the child extra support if needed. It is low weight and extremely compact when folded, which makes it a joy to bring it anywhere. The Cricket is suitable for users needing a simple foldable buggy, e.g. children with CP, GMFCS level 1-3. If mounted with labeled transport fittings the Cricket can be used for forward facing transport in motor vehicles with approved 4 point strap-type tiedown system and approved 3 point belt. For transportation the Cricket is suitable for children in the weight range of 18 kg to specified max user weight for transportation. The Cricket comes in 2 sizes.

Enclosed tools:

- 4, 5 and 6 mm Allen key

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12182

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed

of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read this "User manual" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Information about transportation can be found in the document "M1046 Transportation in Motor Vehicles" supplied with the product
- Ensure the stability of the product is maintained before placing the user in it
- We recommend you notify the user before angling the seat and back
- Ensure the wheels are fully-functional and secure before use
- In accordance with ISO 7176-19 Annex D, the product is evaluated regarding the ability to accommodate vehicle-anchored pelvic and shoulder belts. The device receives an overall score of "Good"
- The product is approved for transportation in vehicles according to ISO 7176-19. More information can be found in the document "M1460 Transportation in Motor Vehicles" supplied with the product.
- Inspect the product and all of its accessories and replace any worn out parts before use
- Be careful not to get your fingers squeezed
- This device is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1, EN 1021-2 and ISO 7176-16
- The device might lose its flame resistant characteristics if using other than original R82 cushions

Environment

- Observe the surface temperature on the product, before placing a user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product

- The plugs need to be mounted in the slots placed on each side of the Cricket when accessories are not mounted in the slots
- Shopping net needs to be emptied before folding the Cricket

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Reuse

This product and its accessories/components are suitable for reuse.

Refurbishment for reuse

Before reuse or changing the user, follow the service checklist, maintenance and washing instructions and carry out regular inspections.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

Service checklist:

Check and adjust the following as per their intended use:

- Folding/unfolding and locking mechanisms are working. Pay special attention to the hand-brace, footplate, rear lock and the locking mechanisms just beneath the seat to the rear
- The locking mechanisms for the rear wheels
- Footplate can be adjusted up and down the frame, angled and locked/unlocked in position
- Fittings and buckles for signs of worn out parts
- Castors are moving freely and all wheel locks/brakes are easily activated
- Before storage: Always deactivate wheel locks/brakes
- Do not place more than one user in the product
- Anti-tips can be fixed in working and resting position
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Product for cracks or signs for worn-out parts

- All labels on the product are intact
- Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to: Wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use.

Carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

Fabric and covers

The cover can be washed at max. 60°C using a mild detergent in a washing machine.

Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Materials

- Aluminium
- Plastic
- Rubber
- Fabric
- Steel

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

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**Etac Ltd.**
**Gelbart AG**
**R82 A/S**