<610 mm (24")

<70 kg (154 lb)

# Wombat Living User Manual

EN 12182

2022-12 - rev.12

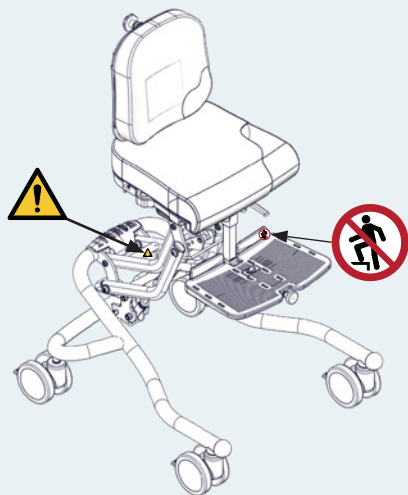
[www.etac.com](http://www.etac.com)

	3		26
	4		27-31
	5-9	mm (inch) kg (lb)	32
	10		33-35
	11-13	ENGLISH DEUTSCH	36-41 42-49
	14-16	NEDERLANDS DANSK	50-57 58-65
	17	NORSK SVENSKA	66-72 73-80
	18-19	SUOMI ESPAÑOL	81-88 89-96
	20	FRANÇAIS ITALIANO	97-104 105-112
	21	PORTUGUÊS PORTUGUÊS DO BRASIL	113-120 121-128
	22	РОССИЯ POLSKI	129-136 137-144
	23	ČESKY 中國	145-152 153-160
	24	日本 ΕΛΛΗΝΙΚΗ	161-168 169-176

### ⚠️ 01

### Wombat Living W/Power ⚠️ 01

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mm  
(inch)  
kg  
(lb)

## mm (inch) kg (lb)

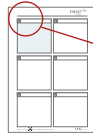
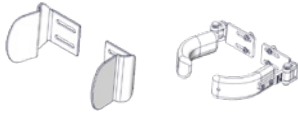
	 1	 2	 3
 Effective seat width	300 mm (11¾")	370 mm (14½")	420 mm (16½")
 Back width	250 mm (9¾")	300 mm (11¾")	350 mm (13¾")
 Effective back width, contoured cushion	170 mm (6¾")	230 mm (9")	270 mm (10½")
 Effective seat depth	180–310 mm (7–12¼")	250–380 mm (9¾–15")	320–500 mm (12½–19¾")
 Backrest height	280–340 mm (11–13½")	310–380 mm (12¼–15")	390–475 mm (15¼–18¾")
 Backrest height (Long Back)	360–430 mm (14¼–17")	395–460 mm (15½–18")	460–550 mm (18–21¾")
 Backrest angle	0°–28°	0°–28°	0°–33°
 Seat plane angle	(-15°)–(-20°)	(-15°)–(-20°)	(-10°)–(-22°)
 Footrest to seat distance	150–250 mm (6–9¾")	150–350 mm (6–13¾")	150–440 mm (6–17¼")
 Leg to seat surface angle	(-30°)–(30°)	(-30°)–(30°)	(-30°)–(30°)
 Seat surface height at front edge*	360–650 mm* (14¼–25½")	360–650 mm* (14¼–25½")	390–670 mm* (15¼–26¾")
 Overall width	550 mm (21¾")	550 mm (21¾")	610 mm (24")
 Total height	640–990* (25¼–39")	670–1030* (26½–40½")	780–1145* (30¾–45")
 Length	600 mm (23½")	600 mm (23½")	720 mm (28¼")
 Maximum user mass	45 kg (99.2 lb)	45 kg (99.2 lb)	70 kg (154.3 lb)
 Mass device	15.5 kg (34.1 lb)	15.5 kg (34.1 lb)	22 kg (48.5 lb)
 Maximum rated load	62.5 kg (137.7 lb)	62.5 kg (137.7 lb)	94 kg (207.2 lb)

## Accessories


 Seat width between hip supports	105–300 mm (4¼–11¾")	170–370 mm (6¾–14½")	215–420 mm (8½–16½")
Seat width, pelvis support w/sides	170–240 mm (6¾–9½")	230–300 mm (9–11¾")	290–360 mm (11½–14¾")
Width, pelvis support without sides	240 mm (9½")	240 mm (9½")	290 mm (11½")
Armrest to seat distance	65–175 mm (2½–7")	65–175 mm (2½–7")	65–175 mm (2½–7")
Armrest to seat distance, long rod and pad	65–265 mm (2½–10½")	65–265 mm (2½–10½")	65–265 mm (2½–10½")

\* Add 25 mm or 1 inch to measure when 100 mm castors are mounted


9996097300

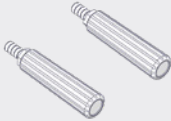
9996097301

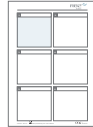
Size 1 & 2  
9996097302  
Size 3  
9996097298

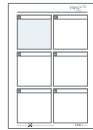9996097163

9996097317

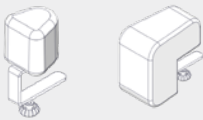
9996097305


9996097319

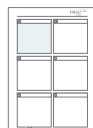9996097307

9996097308

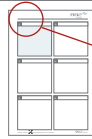9996097160

9996097122

9996097310

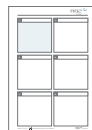
9996097731


9996097723


9996097313


9996097162

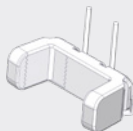
9996097299


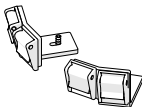
9996097286

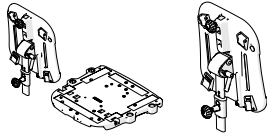9996097329


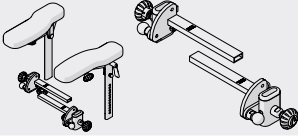
9996097337

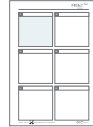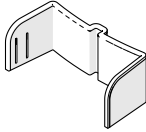9996097338

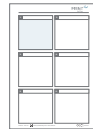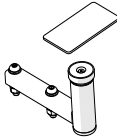9996097339

9996097342

9996097739

## ENGLISH

## General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and adults with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

### Intended purpose

The product is an activity chair from R82, available in 3 sizes. The product is available with gas-spring or an electric motor. At configuration, it is possible to choose between two ways to operate tilt-in-space and three ways to operate the height adjustment. All versions are delivered with a multi-adjustable seat, back and footrest. The product is suitable for users needing a practical indoor activity chair e.g. users with GMFCS level 2-4, users who need extra help while sitting. The product is for indoor use only.

### Contra indications

There are no known contra indications.

### Precautions

None

### Enclosed tools:

Wombat Living 1 + 2

- 3+4+5 mm Allen key

Wombat Living 3

- 3, 4, 5 and 6 mm Allen key
- 10 mm spanner

## CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12182

### Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

### Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

### R82 Warranty

[www.etac.com/support](http://www.etac.com/support)

### Presales information and instructions

Presales information and instructions are available at [www.etac.com](http://www.etac.com), under your product, in the documents section.

## Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

### Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user manual.
- Before the user is allowed to operate the remote control then the child's cognitive state must be assessed and considered to be sufficient to operate it
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Transportation of the product whilst the user is in it is not allowed. It is allowed to use the product with the user safely strapped in it for transfers over short distances. Do not cross any obstacles during transfer.
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Ensure that body parts are not trapped in openings for mounting of supports/ accessories.
- Do not let children and youngsters play with the handset
- It is important to remove the transport plastic bag before using the handset. Cables need to be placed in such a way that users or carers cannot trip and injure themselves.
- Be aware of any MAX mark indications before adjustment

### Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

### User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered
- The height and tilt functions can be operated by user under caregivers permanent supervision.
- When changing user, clean the product and disinfect it if necessary. Carry out necessary adjustments after cleaning.

### Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product
- Ensure the stability of the product is maintained before placing the user in it
- Ensure the castors are fully-functional and secure before use
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- The power unit does not have a safety stop
- Do only use the power unit within specified working limits
- Do not attempt to repair any part of the power unit.
- The footplate is for support only! The foot

support does not have a weight bearing function

- When the footplate is close to the floor and the castors, the castors may be unable to swivel
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1, EN 1021-2 and ISO 7176-16
- The product might lose its flame resistant characteristics if using other than original R82: Cushions
- This product is not to be occupied in a motor vehicle. The user must be positioned in a vehicle seat and the product must be stored in a luggage compartment
- The user must not occupy the product while it is being transported into or out of a vehicle by a lift
- Unoccupied device is suitable for land and air transport
- Details covering EMC can be found in the appendix "EMC details" supplied with the product

## Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

### Lifetime

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded. The device may be reissued within its lifetime.

### Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

### Service checklist:

Check and adjust the following as per their intended use:

- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- Frame is adjustable in height
- Adjustment options for the foot rest, it is possible to lock the angle and fasten the telescopic tubes in position
- Seat back is adjustable and can be locked in position
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

Repair or replace any damaged or worn out parts.

### Maintenance

Before every use it is recommended to:

Wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use carry out a visual inspection, for any damaged or worn-out parts.

### Washing

#### Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

### Before washing

Avoid degreasing of the piston rods, the actuators should be retracted to minimum stroke before washing.

- Remove all accessories, unplug all cables and make sure to dismount the battery, control-box and handset. Leave the actuators with the cables mounted on the product.
- Do NOT clean the power unit with a high pressure cleaner or steam cleaner.

### After washing

- Ensure correct placement of cables inserted into the control box when reinstalling the power unit system.
- Check that the handset functionalities works correct

### Fabric and covers

The cover can be washed at max. 60° C using a mild detergent in a washing machine.

Separate foam and cover before wash.

### Foam

The foam inserts must only be washed by hand. Use warm water and a mild detergent/soap without chlorine and let all parts dry before use.

During and after washing the foam, do not bend, squeeze or wring it, as it may negatively affect the shape.

### Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

### Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

### Materials

- Plastic
- Stainless steel
- Steel
- Polyurethan
- TPE

### Surface treatment

The following surface treatment processes

---

were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

### Power unit

For an optimum lifetime the power unit must be connected to the mains voltages as often as possible (e.g. every night). Charge the batteries 12 hours before use.

The batteries must be charged at least every 3rd month - otherwise there might be damages due to self-discharge. It is recommended to test the battery function at least once every year.

### Recharging the battery:

- Plug the charging lead into the handset
- Plug the charger into the electricity supply and switch ON. A yellow light on the charger will light up
- Charge the batteries until the lamp on the charger turns green

### Caregiver

- Do not leave children and youngsters alone with the handset
- Do not let children and youngsters play with the handset
- Do not leave the handset within reach of children and youngsters.
- It is important to remove the transport plastic bag before using the handset. Cables need to be placed in such a way that users or carers cannot trip and injure themselves.

### Product

- The power unit does not have a safety stop
- The system must not be sprayed directly with a highpressure cleaner
- Cleaning with a steam cleaner is not permitted
- Do only use the power unit within specified working limits
- It is recommended to check the handset and cable for damage and holes caused by violent handling at least once a year
- Do not submerge the handset in water
- Do not attempt to repair any part of the power unit.

### Technical Data

#### Battery - BA18

- Capacity 1.3 Ah - 24V DC
- Charging current is max. 500 mA
- IPX6

The batteries have a maximum life of 4 years but this can vary depending upon the conditions of use and charging practices

#### Battery charger - CH01

- Mains: 100 - 240VAC/50-60 Hz switch mode
- Charging voltage: 27.6 VDC
- Charging current: Max. 500 mA
- FRIWO Gerätebau GmbH, Model FW7218M/24

### Actuator - LA28/LA32

- Input voltage: 24V
- Noise level:  $\leq 45$  dB(A)
- IPX5
- Duty cycle: Max. 10% or 2 min. use then 18 min. without use

### Handset - HB 40

- IPX1

### Detachable parts

- Handset - HB 40
- Battery - BA 18

### Applied part

- Handset - HB 40

### IP-Rating table

IPX0	No protection
IPX1	Protection against vertically falling drops of water e.g. condensation
IPX2	Protection against direct sprays of water up to 15° from the vertical
IPX3	Protected against direct sprays of water up to 60° from the vertical
IPX4	Protection against water sprayed from all directions - limited ingress permitted
IPX5	Protected against low pressure jets of water from all directions - limited ingress
IPX6	Protected against temporary flooding of water, e.g. for use on ship decks - limited ingress permitted
IPX7	Protected against the effect of immersion between 15 cm and 1 m
IPX8	Protects against long periods of immersion under pressure

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## UKRP

Unit 60, Hartlebury Trading Estate,  
Hartlebury,  
Kidderminster, Worcestershire, DY10 4JB

DISTRIBUTOR:

M1058

Reserving our rights for printer's error or discontinued products  
Please find an updated edition on our website [etac.com](http://etac.com)  
The design and patents are all registered design and patents of R82.

© 2013 R82 A/S All rights reserved The R82 logo and the icons are registered trademarks of R82 A/S