User manual

Swan Curo

R82[®] 
by Etac

M1063 - 2024-01-15 - Rev. 3

www.etac.com

	3	ENGLISH DEUTSCH	28-30 31-36
	4	NEDERLANDS DANSK	37-41 42-44
	5	NORSK SVENSKA	45-47 48-50
	6	SUOMI ESPAÑOL	51-53 54-58
	7	FRANÇAIS ITALIANO	59-63 64-67
	8	PORTUGUÊS PORTUGUÊS DO BRASIL	68-72 73-76
	9	РОССИЯ POLSKI	77-82 83-88
	10	ČESKY 中國	89-91 92-94
	11	日本 ΕΛΛΗΝΙΚΗ	95-99 100-105
	11		
	13-23		
mm (inch) kg (lb)	24-26		
	27		

Swan Curo size 1

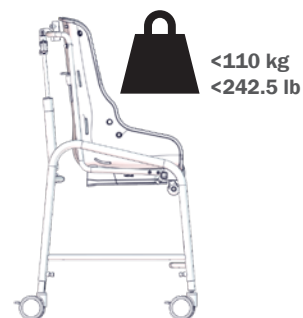
Swan Curo size 2

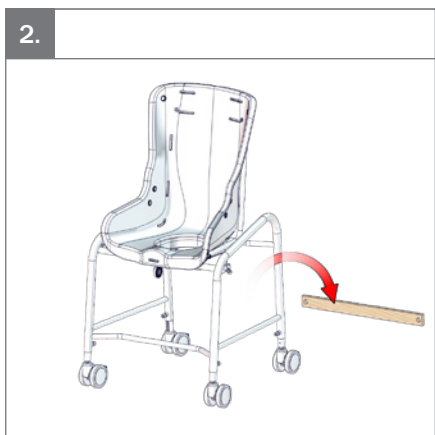
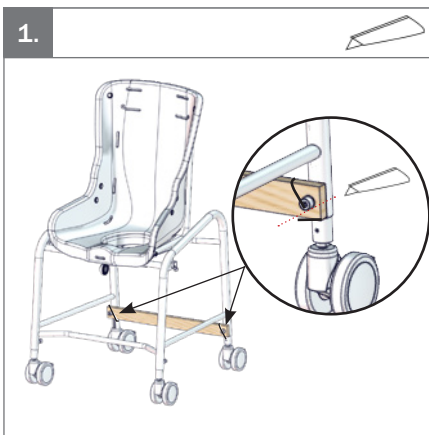
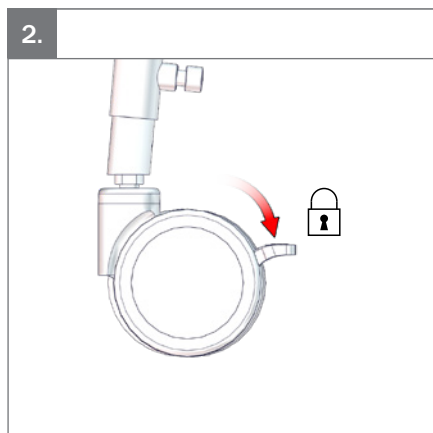
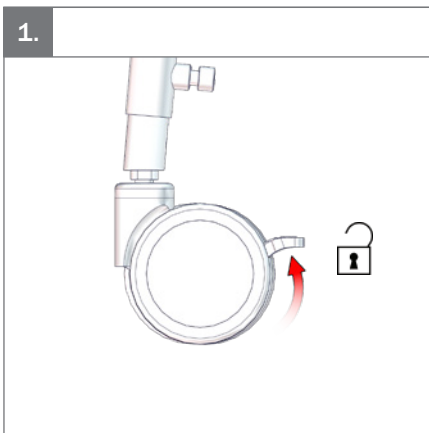
Swan Curo size 3

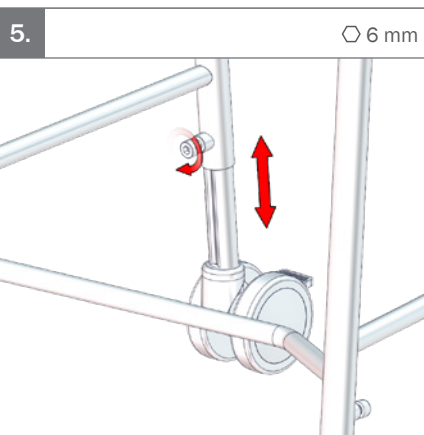
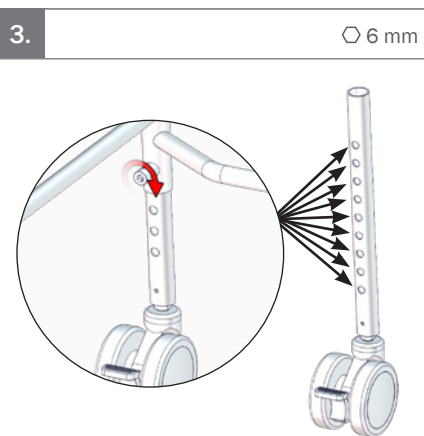
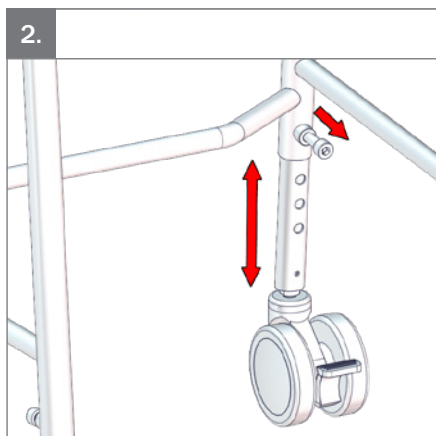
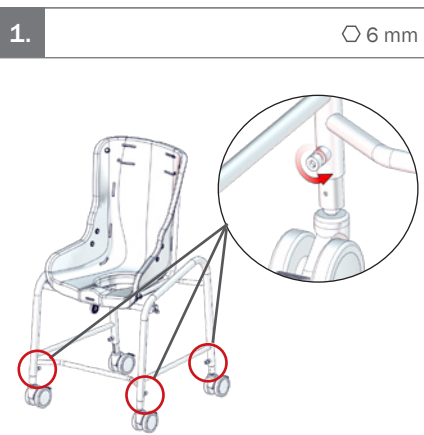
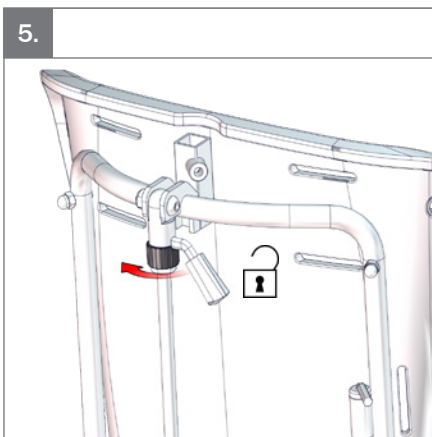
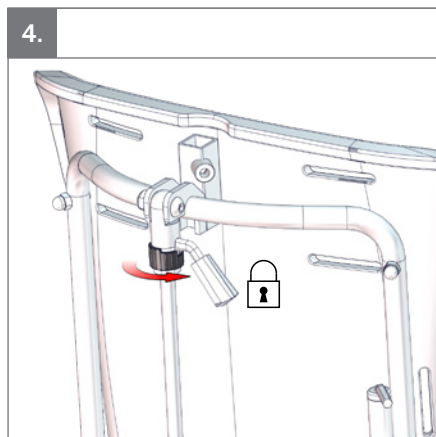
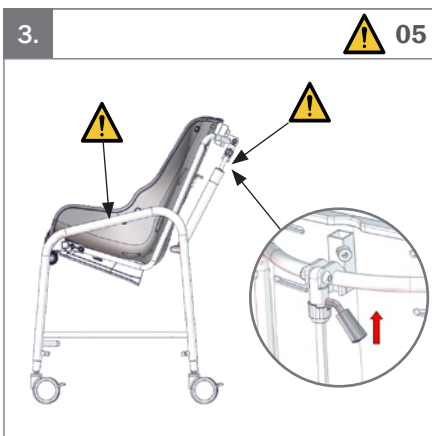
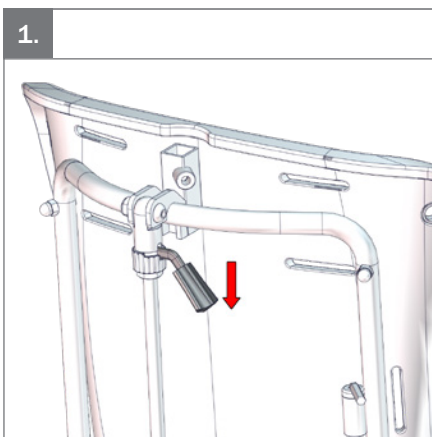
Swan Curo size 4






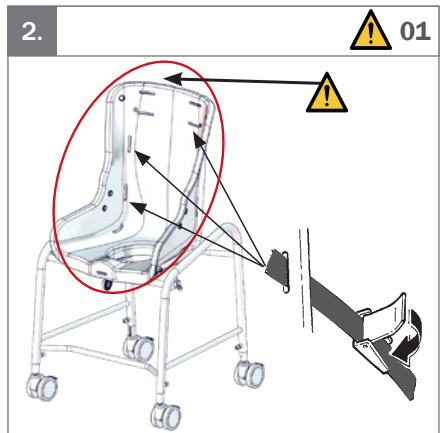
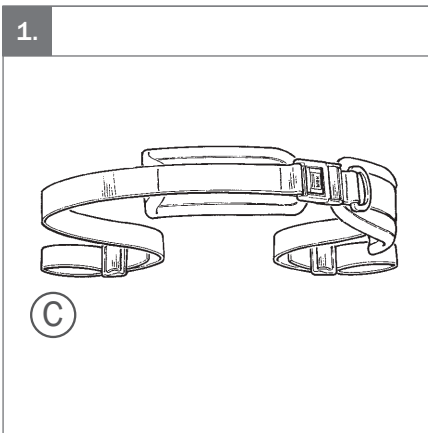
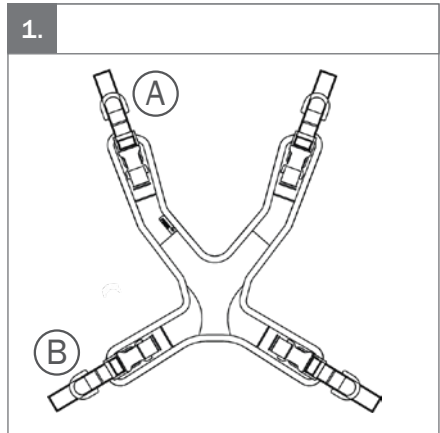
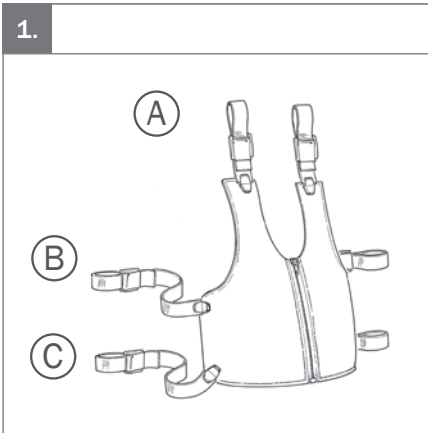
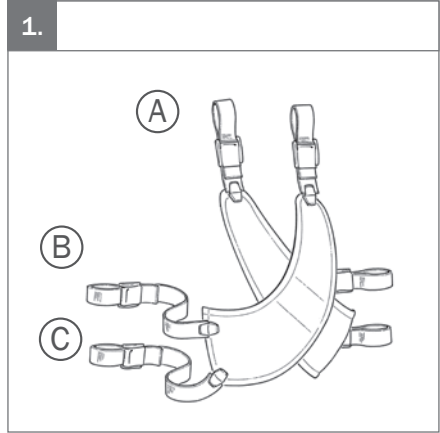
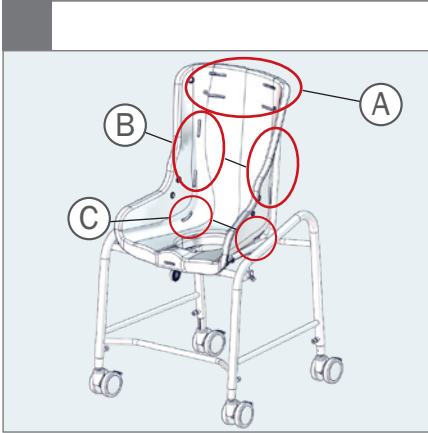
 05

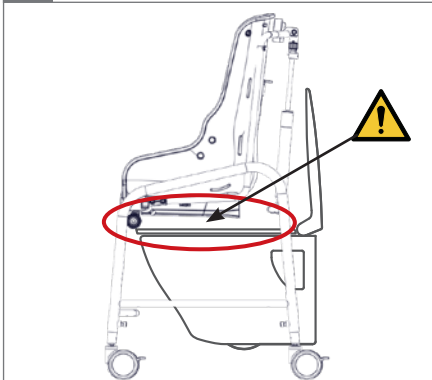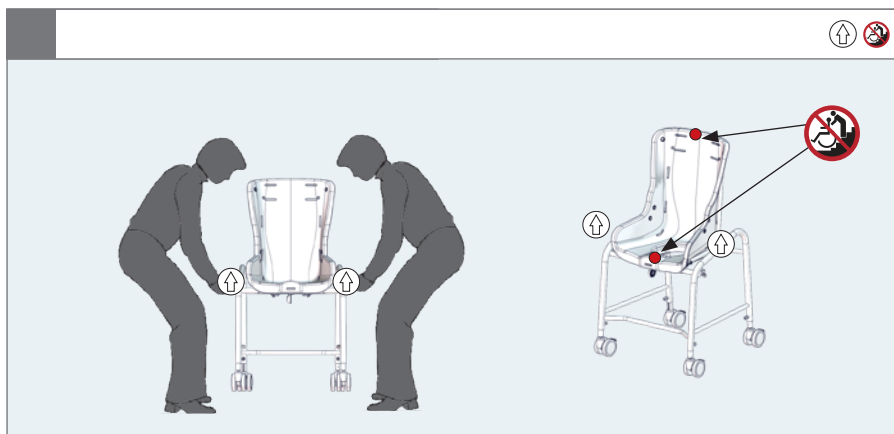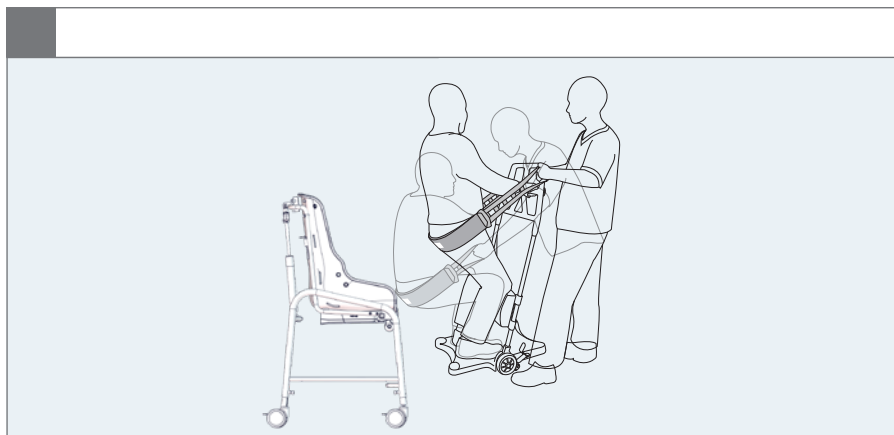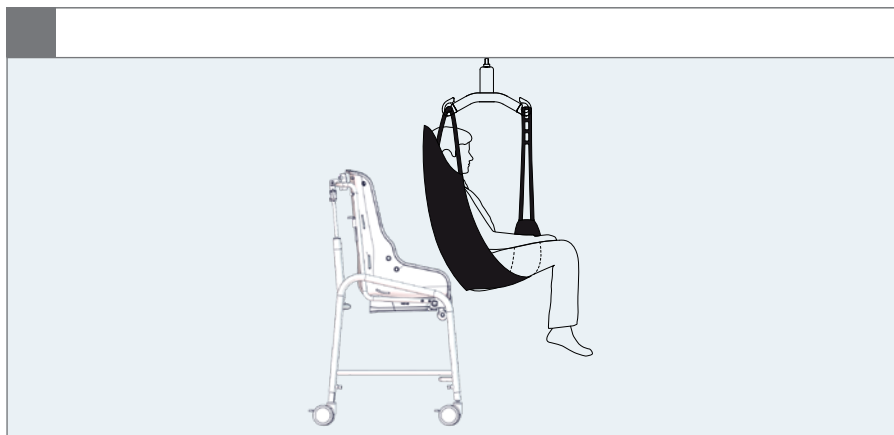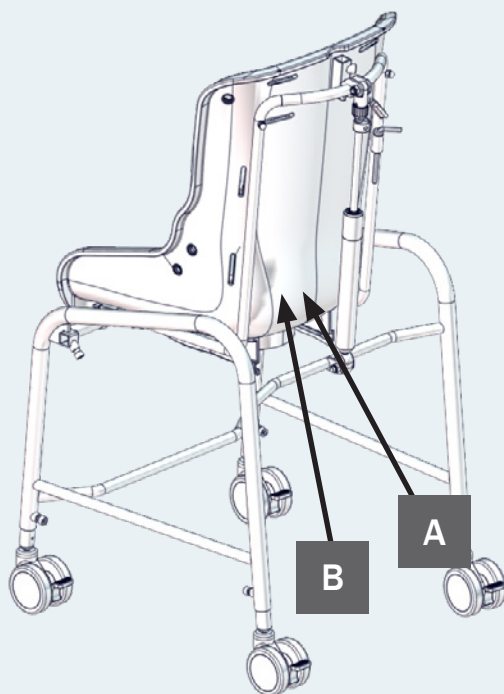A

R82[®] 
by Etac

R82 A/S, Parallelvej 3
DK-8751 Gedved
etac.com
Made in xxxxxxx

 
YYYY-MM-DD

B

Posting group
Product name, size x

DMR ver.: xxxx

 Max. user mass: xxx kg
 Max. rated load: xxx kg
 Mass device: xxx kg

USA (XX)XXXXXXXXXXXX(XX)XXXXXX(XX)XXXXXX

EN: Warning: This symbol appears in the User Guide along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger
01 = Entrapment Hazards
02 = Tilting Hazards
03 = Transportation in motor vehicles
05 = Squeezing Hazard

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
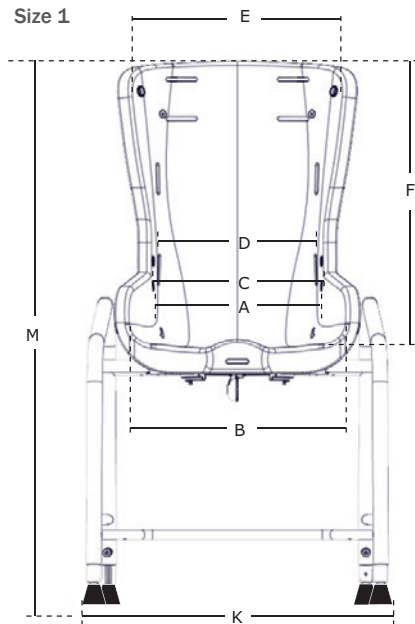
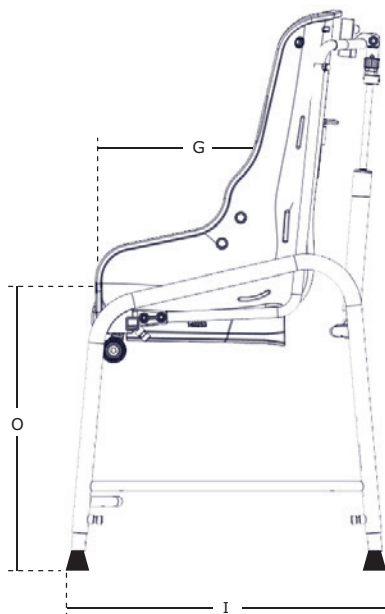
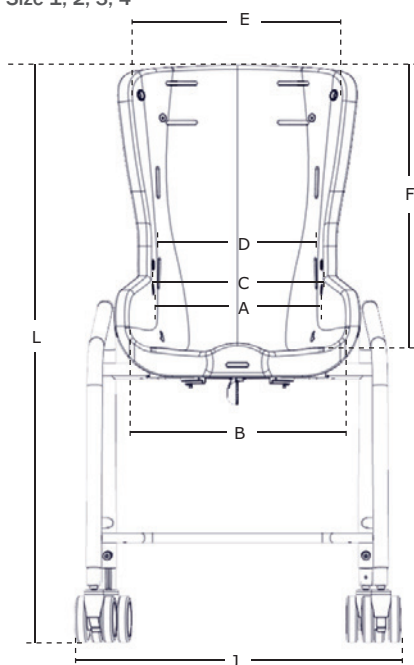
Size 1, 2, 3, 4

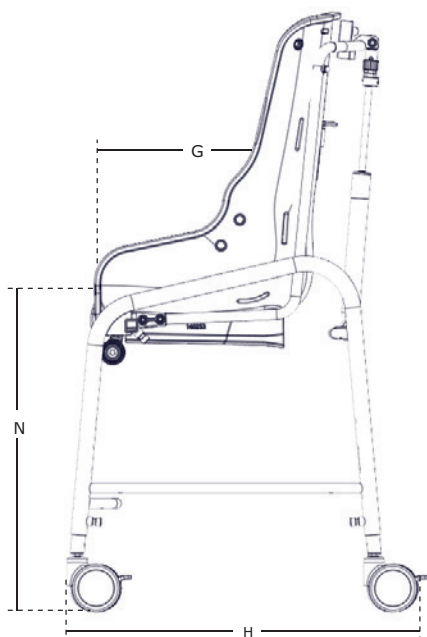mm
(inch)
kg
(lb)

mm (inch), kg (lb)		1	2	3	4
	Effective seat width (A)	220 mm (8¾")	260 mm (10¼")	320 mm (12½")	410 mm (16¼")
	Seat width, front (B)	320 mm (12½")	355 mm (14")	425 mm (16¾")	475 mm (18¾")
	Pelvis width (C)	240 mm (9½")	275 mm (10¾")	340 mm (13½")	420 mm (16½")
	Effective seat depth (G)	265 mm (10½")	310 mm (12¼")	400 mm (15¾")	450 mm (17¾")
	Backrest height (F)	380 mm (15")	470 mm (18½")	525 mm (20¾")	600 mm (23½")
	Seat surface height at front edge, with castors (N)	380–540 mm (15–21¼")	480–640 mm (19–25¼")	470–630 mm (18½–24¾")	465–625 mm (18½–24¾")
	Seat surface height at front edge, without castors (O)	290–450 mm (11½–17¾")	N/A	N/A	N/A
	Back width, top of backrest (E)	270 mm (10¾")	350 mm (13¾")	425 mm (16¾")	490 mm (19¾")
	Back width, bottom of backrest (D)	210 mm (8¾")	250 mm (9¾")	315 mm (12½")	400 mm (15¾")
	Overall width, with castors (J)	500 mm (19¾")	530 mm (20¾")	580 mm (22¾")	655 mm (25¾")
	Overall width, without castors (K)	460 mm (18")	N/A	N/A	N/A
	Minimum turning diameter	1410 mm (55½")	1640 mm (64½")	1790 mm (70½")	1890 mm (74½")
	Minimum distance for turning 180° forward in movement	1205 mm (47½")	1350 mm (53¼")	1475 mm (58")	1600 mm (63")
	Minimum distance for turning 180° allowing for reversing movement	705 mm (27¾")	820 mm (32¼")	895 mm (35¼")	945 mm (37¼")
	Length, with castors (H) outside (low/high) at 0° tilt	495 mm (19½")/ 555 mm (21¾")	625 mm (24½")/ 645 mm (25½")	680 mm (26¾")/ 715 mm (28¼")	680 mm (26¾")/ 715 mm (28¼")
	Length, without castors (I) outside (low/high) at 0° tilt	400 mm (15¾")/ 425 mm (16¾")	N/A	N/A	N/A
	Overall height, with castors (L)	760–920 mm (30–36¾")	960–1120 mm (37¾–44")	990–1150 mm (39–45¼")	1060–1220 mm (41¾–48")

mm (inch), kg (lb)		1	2	3	4
	Overall height, without castors (M)	670-830 mm (26½-32¾")	N/A	N/A	N/A
	Mass device, with castors	8.50 kg (18.8 lb)	10.50 kg (23.3 lb)	11.75 kg (26 lb)	12.75 kg (28 lb)
	Mass device, without castors	7.50 kg (16.5 lb)	N/A	N/A	N/A
	Maximum user mass	35 kg (77.3 lb)	60 kg (132.3 lb)	85 kg (187.5 lb)	110 kg (242.5 lb)
	Maximum rated load, with castors	46 kg (101.5 lb)	73 kg (161 lb)	99 kg (218.3 lb)	125 kg (275.5 lb)
	Maximum rated load, without castors	45 kg (99.3 lb)	N/A	N/A	N/A
	Aperture (hole dimensions)	DxW 185x125 mm (7¼x5")	DxW 220x150 mm (8¾x6")	DxW 280x180 mm (11x7")	DxW 330x180 mm (13x7")
	Aperture with inlay (hole dimensions)	DxW 175x100 mm (7x4")	DxW 210x125 mm (8¾x5")	DxW 270x155 mm (10¾x6")	DxW 320x155 mm (12½x6")
	Seat plane angle (tilt)	0° - 20°	0° - 20°	0° - 20°	0° - 18,5°

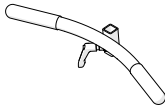
9996098105

9996098106

9996098107

9996098108

9996098109

9996098111

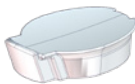
9996098112

9996098113

9996098114

9996098115

9996098135

ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

Assistive device for toileting, washing, bathing and showering for children and adults with disabilities.

Indications

The device can be used by children and adults with disabilities, who lack the ability to maintain a seated position without an assistive device or to compensate for the lack of ability to maintain a standing position while washing oneself or showering. The target group for the device is based on functional ability. It is not based on a specific health condition or age.

The user is the person sitting in the device. The caregiver is the person operating the device. Never leave the user unattended in this device. Ensure permanent supervision by a caregiver.

The device can be applied more times during the day. Between using the device, the user will be lying down, standing or sitting in another assistive device or a conventional product.

Contra indications

There are no known contra indications.

Precautions

Special attention should be given to the considerations about the configuration of the device and the use of accessories for users with limitation in their:

- Sensory functions and the sensation of pain consider extra checks of the pressure caused by the support provided by the device and its accessories.
- Ability to maintain a seated position, use the device with accessories (e.g. vests, belts, or a tray).
- Ability to maintain the position of their head, consider using accessories that provide support to the positioning of the head.

Application areas

- The product is for indoor use
- This equipment is for use in home care environment.

Enclosed tools

- 6 mm Allen key

CE Declaration of Conformity

This product conforms to the requirements of

the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12182
- ISO 17966

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions. In case an adverse event occurred in relation to the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user.
- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children.
- Always use the correct lifting techniques and aids for this purpose.
- Ensure the stability of the product is maintained before placing the user in it.
- This product contains small parts which could present a choking hazard if removed from their intended placing.
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver.
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier.
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible.
- Ensure that moving parts do not squeeze or trap body parts when adjusting them.
- Ensure that body parts are not trapped in openings for mounting of supports/ accessories.
- Be careful not to get body parts trapped, when placing the product over the toilet.
- Activate the brakes before placing the user into the product.
- The user must not occupy the product when it is lifted from the ground.
- All gas springs are to be kept locked, when the device is not being adjusted.
- Be aware of any MAX mark indications before adjustment.

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat or cold. If the surface temperature is above 41 °C

then allow the product to cool before use.

- For the purpose of safety and comfort, always check the water temperature before bathing the user.

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered.
- When changing user, clean the product and disinfect it if necessary. Carry out necessary adjustments after cleaning.

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children.
- Ensure the stability of the product is maintained before placing the user in it.
- Unoccupied device is suitable for land and air transport.
- The device is only designed to travel short distances within the same room.
- Resistance to ignition is not claimed for this product.
- The product might change its resistance to ignition characteristics if using other than original R82: Cushions.

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Reuse

This product and its accessories/components are suitable for reuse.

Refurbishment for reuse

Before reuse or changing the user, follow the service checklist, maintenance and washing instructions and carry out regular inspections.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

Service checklist:

Check and adjust the following as per their intended use:

- Accessories can be inserted and locked in the fittings in the top of the back, using the knob and handles mounted in the fittings.
- Seat is adjustable and can be locked in position.
- Fittings and buckles for signs of worn out parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact
- Check that all castors are moving freely and that all wheel locks/brakes are easily activated
- Nuts and bolts on the product are tightened

Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to:

Wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use.

Carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

This product can be washed at 85° using a mild detergent for 3 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Materials

- Stainless steel
- Polyurethan
- Plastic
- Aluminium
- Rubber

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

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**Etac Ltd.**
**Gelbart AG**