220-320 mm (8½-12½")

170-360 mm (6¾-14")

<50 kg (110 lb)

Nandu User Guide

EN 12182 + EN 12183

EN 1021-1 + EN 1021-2

2016.07-rev.03

CE
support.R82.org


	3	ENGLISH GERMAN	17 - 20 21 - 24
	4	NEDERLANDS DANSK	25 - 28 29 - 32
	5	NORSK SVENSKA	33 - 36 37 - 40
	6	SUOMI ESPAÑOL	41 - 44 45 - 48
	7	FRANÇAIS ITALIANO	49 - 52 53 - 56
	8	PORTUGUÊS PORTUGUÊS DO BRASIL	58 - 60 61 - 64
	9	РОССИЯ POLSKI	65 - 68 69 - 72
	10	ČESKY SLOVENSKY	73 - 76 77 - 80
	11	中國 日本	81 - 84 85 - 88
	14	ΕΛΛΗΝΙΚΗ	89 - 92
mm (inch)	15		
	16		


1.

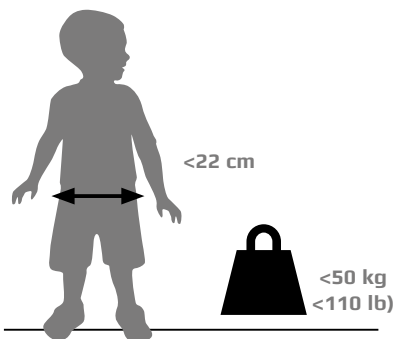
NANDU 1


NANDU 2

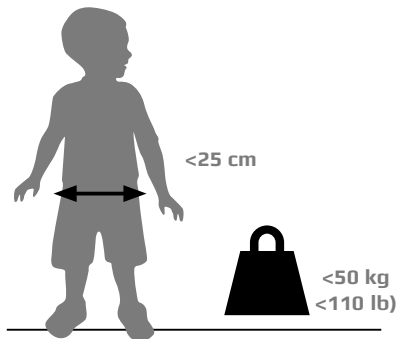NANDU 3

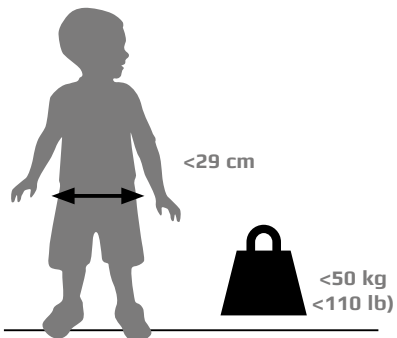NANDU 4

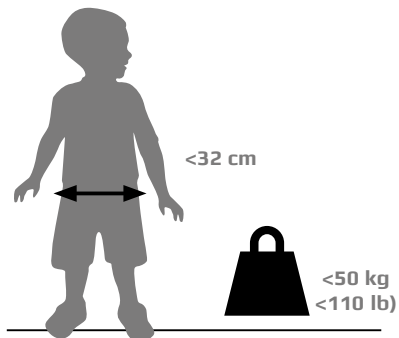
1.

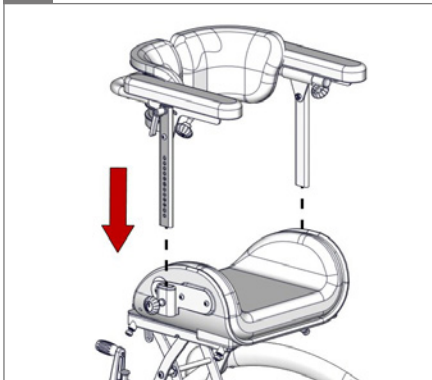
2.


3.

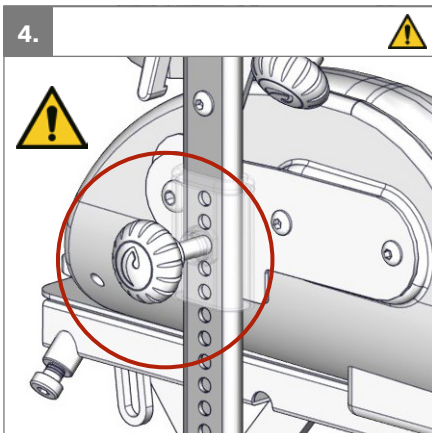
4.

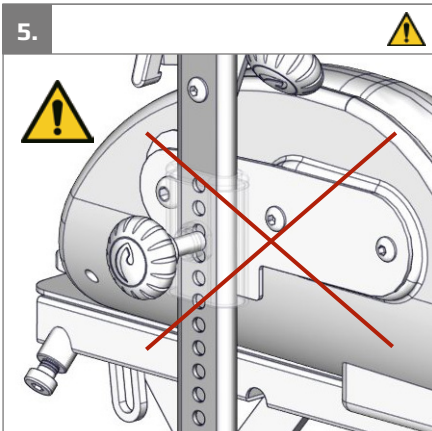5.

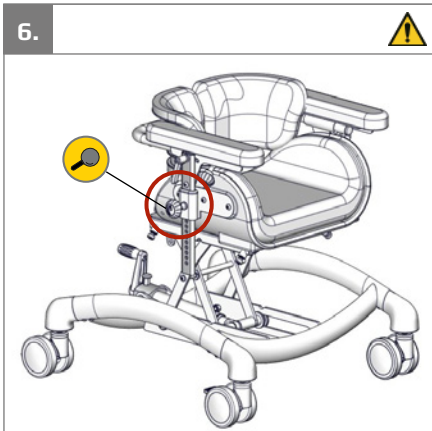
6.


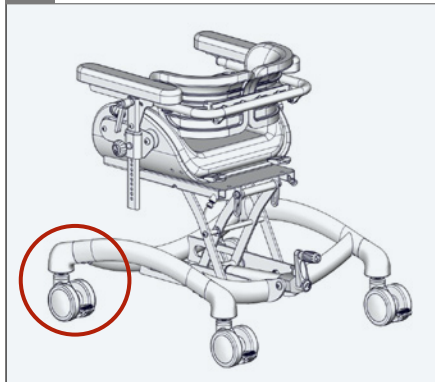
1.

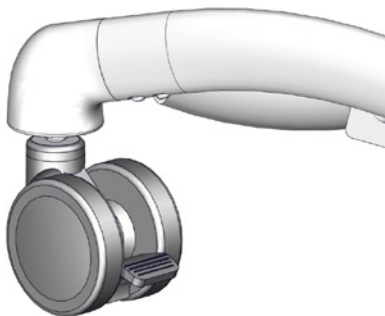
2.

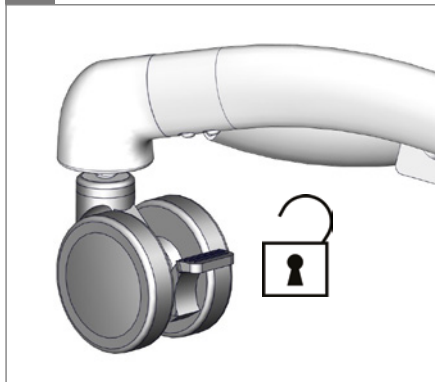
3.

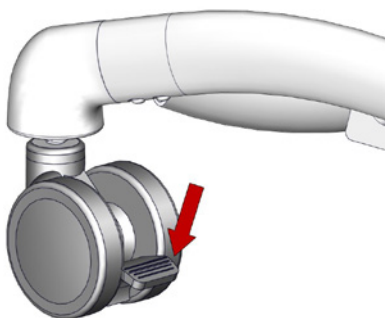
4.

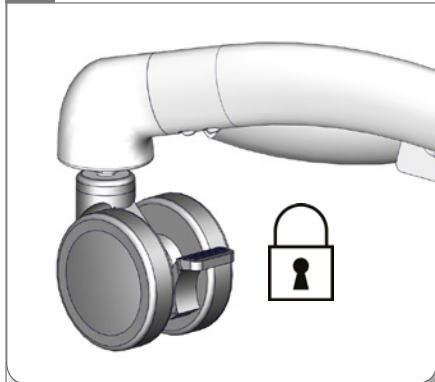5.

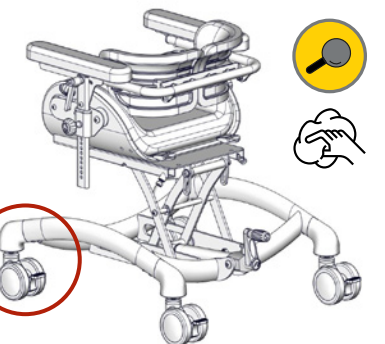
6.


1.

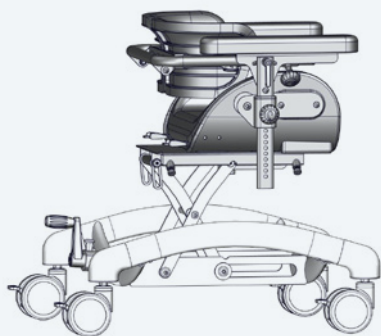0° - 8°


2.

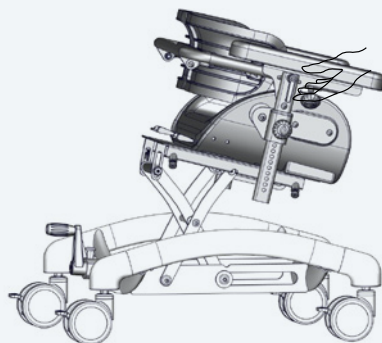0° - 8°

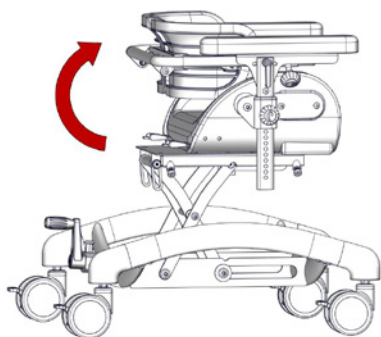
3.


4.

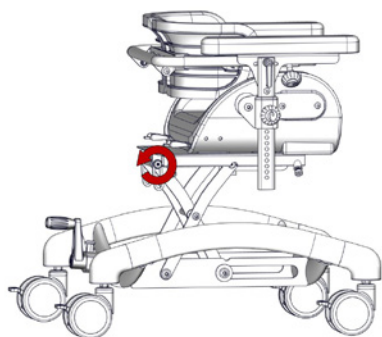⬡ 5 mm

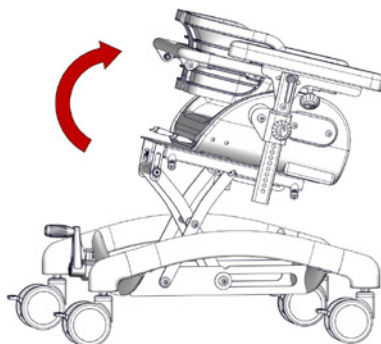
5.

6.

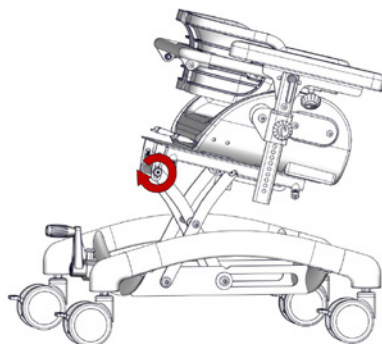⬡ 5 mm - 2,5 Nm

1.

0° - 20°

2.

0° - 20°

3.

4.

5.

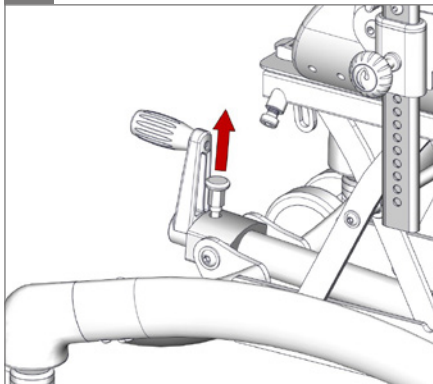
6.


1.


2.

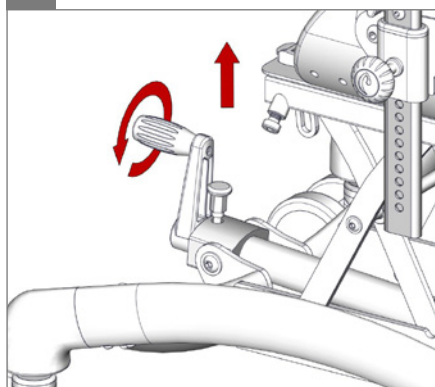
3.

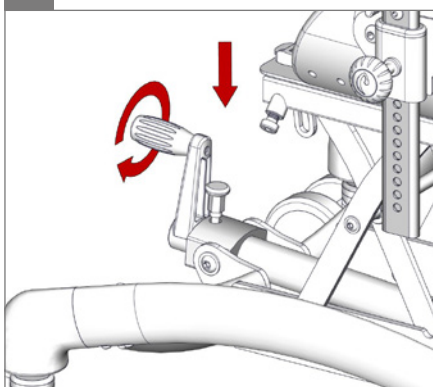4.

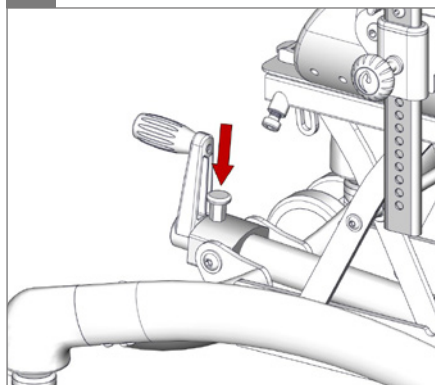5.

6.

1.

2.

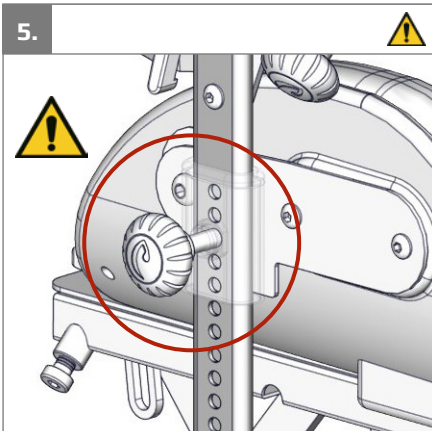
3.


4.

5.

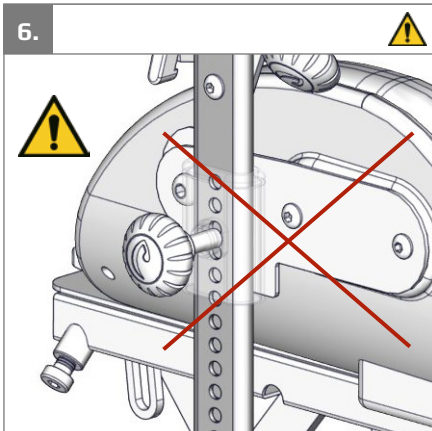6.


1.

2.

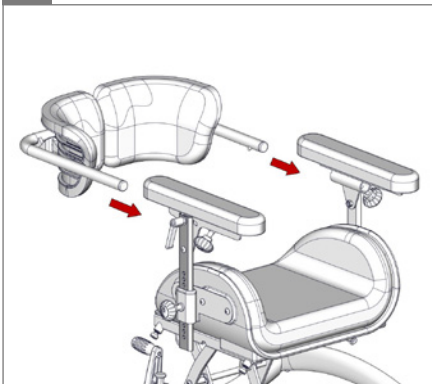
3.

4.

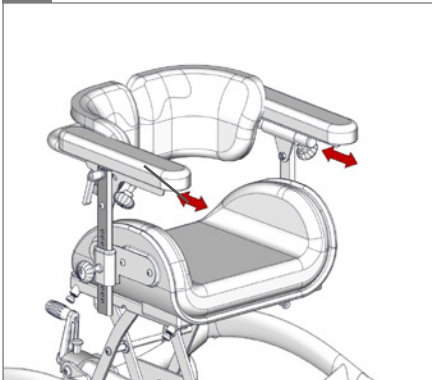5.

6.

1.

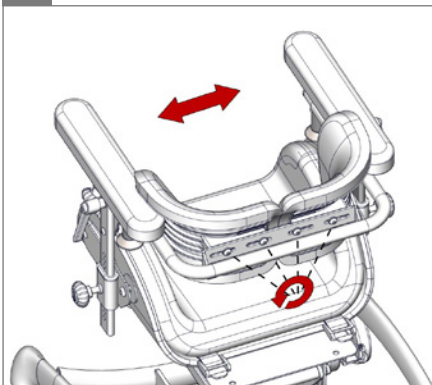
2.


2.

○ 5 mm


3.

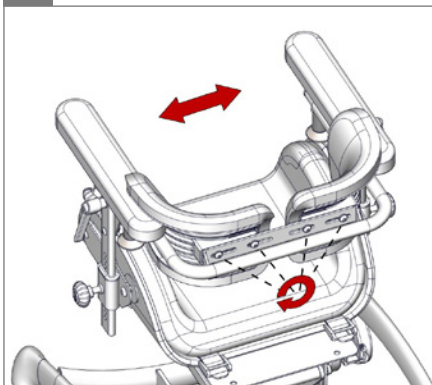○ 5 mm - 2,5 Nm


4.

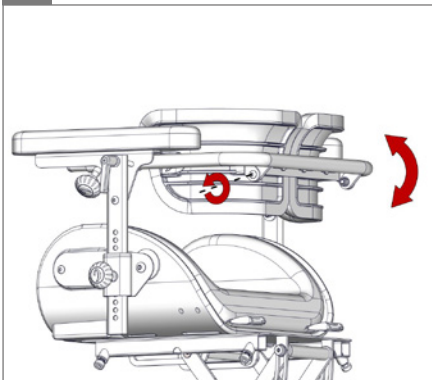○ 5 mm, ⚙ 10 mm

5.

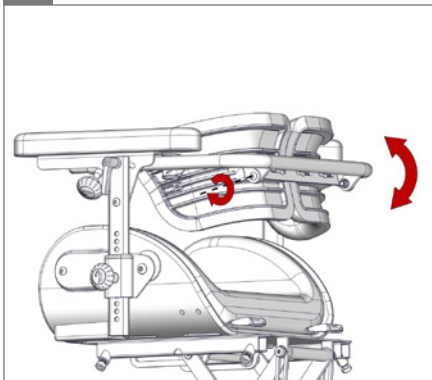○ 5 mm, ⚙ 10 mm - 2,5 Nm

1.

2.

3.

○ 5 mm - 2,5 Nm

4.

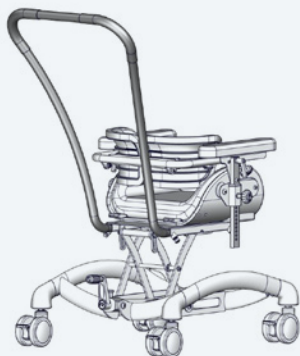
5.


6.

1.

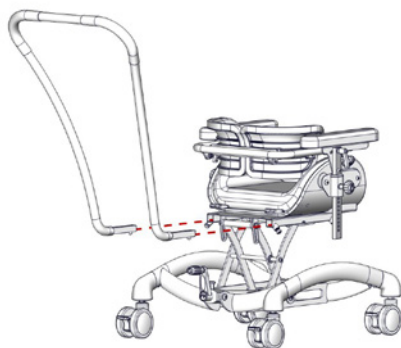
2.


3.

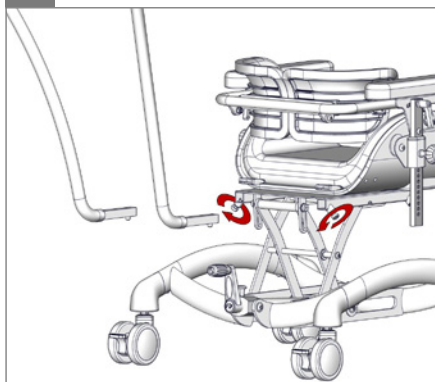○ 5 mm


4.

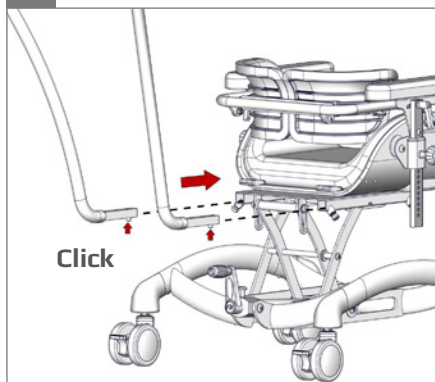Click


5.

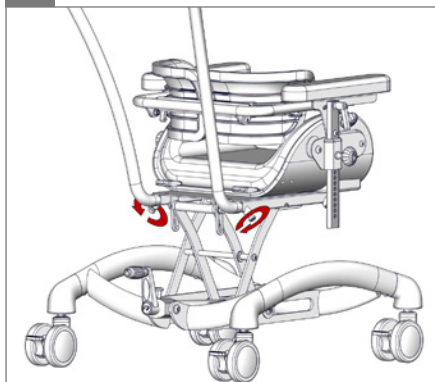○ 5 mm - 2,5 Nm

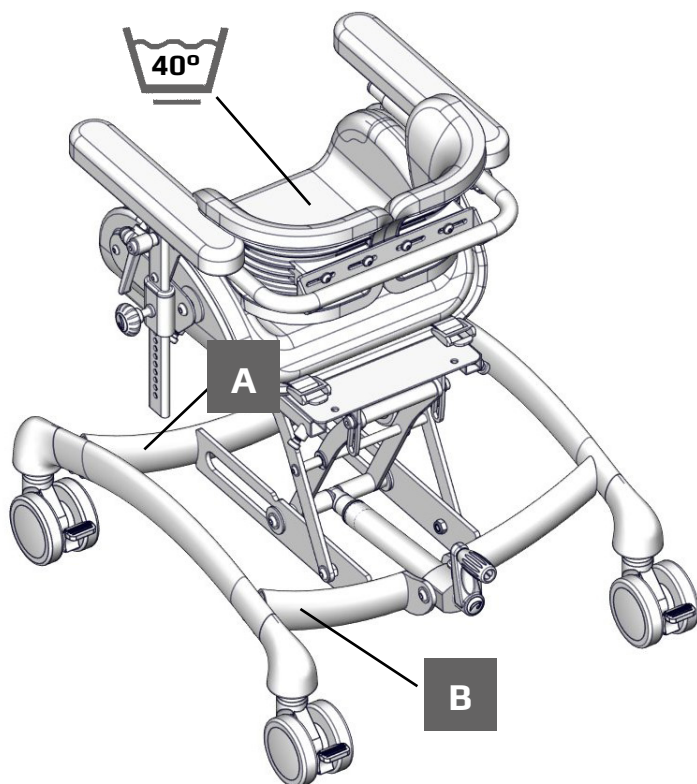6.

A

 XXXXXX Max load: xxx kg
 DMR ver.: XXXX
 Art. no.: XXXXXXXX
 Product: XXXXXXXX - Size X

(XX)XXXXXXXXXXXX(XX)XXXXXX(XX)XXXXXX

B


R82 

 Parallevej 3
 DK-8751 Gedved
 R82.org
 Made in Denmark YYY-MM-DD 


mm (inch)				
	 1	 2	 3	 4
	220 (8½")	250 (9¾")	290 (11½")	320 (12½")
	170-240 (6¾-9¼")	210-280 (8¼-11")	240-310 (9¼-12")	280-360 (11-14")
	245-390 (9½-15¼")	245-390 (9½-15¼")	335-480 (13-18¾")	335-480 (13-18¾")
	130-225 (5-8¾")	130-225 (5-8¾")	130-440 (5-17¼")	130-440 (5-17¼")
	500 (19½")	500 (19½")	500 (19½")	500 (19½")
	560 (21¾")	560 (21¾")	560 (21¾")	560 (21¾")
	435-580 (17-22½")	435-580 (17-22½")	565-710 (22-27¾")	565-710 (22-27¾")
	135-230 (5¼-9")	135-230 (5¼-9")	150-270 (5¾-10½")	150-270 (5¾-10½")
 Folded	500 (19½")	500 (19½")	500 (19½")	500 (19½")
 Folded	560 (21¾")	560 (21¾")	560 (21¾")	560 (21¾")
 Folded	435 (17")	435 (17")	565 (22")	565 (22")
	-20° -> 20°	-20° -> 20°	-20° -> 20°	-20° -> 20°
	8°	8°	8°	8°
	160°	160°	160°	160°
	13 kg (28,5 lb)	13,5 kg (30 lb)	14 kg (31 lb)	15 kg (33 lb)
	max. 50 kg (110 lb)	max. 50 kg (110 lb)	max. 50 kg (110 lb)	max. 50 kg (110 lb)
 max. 6°		 max. 6°	 max. 6°	

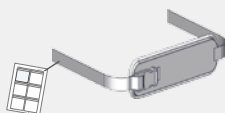9996097800


9996097801

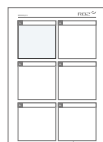9996097802

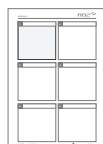9996097703

9996097118

ENGLISH

We appreciate your choice for a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product you must read this manual before use and save it for further reference.

EN

Intended use

The Nandu is a small seat for active children who need little support while seated. The seat is padded and adjustable for a perfect fit. The Nandu is available in four sizes, each with up to 15 cm continuous height adjustment and up to 8 degrees forward tilt. The back rest can be reversed to provide abdominal support when in the forward leaning position. The Nandu is constructed for one user at a time and is suitable for users needing a simple indoor seat, e.g. children with CP, GMFCS level 2-3.

Tools

Enclosed tools; 5 mm Allen key and 10 mm wrench.

Declaration of Conformity -

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings.

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or daily maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the manual. For further information, we refer to the R82 homepage/download.

The warranty can only be sustained if the R82 product is in use in the same country where it was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if non-original R82 parts/accessories are used, or if the product has not been repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the instructions for use (IFU). R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase. The warranty is given by R82 or, subsequently, an R82 dealer.

ENGLISH

Symbols

For indoor use

When the user is moved in the product from one room to another or to an outdoor area in case of an emergency, please ensure that the area is flat and without any obstacles. Never leave the user unattended in this product

Consult instructions

The latest versions of the instructions are always available on the R82 website and can be printed in larger sizes

Caution

Incorrect use of the product may cause serious damage to the product and the environment and may cause injury

Caution

Be careful not to get your fingers caught

Caution

Do not pull/lift the chair by the push brace, especially on stairs

Attention

Keep away from children

Inspection

Carry out inspections frequently

Cleaning

Keep clean to maintain optimum function

Disposal -

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

ENGLISH

Safety -

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Caregiver

Read all of the instructions carefully before use and keep them for future reference. Incorrect use of this product may cause serious injury to the user

Always use the correct lifting techniques and aids for this purpose

Never leave the user unattended in this product. Ensure permanent supervision by an adult

Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier

If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible

Environment

For the purpose of safety and comfort, always move the product away from the sun and allow to cool before use

Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery roads

When using on inclines, please refer to measurements

User

If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children

Activate the brakes before placing the user into the product

Ensure the stability of the product is maintained before placing the user in it

We recommend you notify the user before angling the seat and back

Ensure the castors are fully-functional and secure before use

Inspect the product and all of its accessories and replace any worn out parts before use

This product is not to be occupied in a motor vehicle. The user must be positioned in a vehicle seat and the product must be stored in a luggage compartment

The user must not occupy the product while it is being transported into or out of a vehicle by a lift

ENGLISH

Maintenance responsibilities table

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

Frequency	Description
1st Day	Read the instructions for use (IFU) carefully before use and store them in a safe place for future reference
	Store the enclosed tools on/at the product
Daily	Use a dry cloth to clean the product
	Visual inspection. Repair or replace any damaged or worn-out parts
	Ensure that all fasteners are in place and fitted correctly
	Check all fittings and buckles for signs of worn out parts
	Be aware of any max mark indications before adjustment
Weekly	Check that all castors are moving freely and that all wheel locks/brakes are easily activated
	Check that the angle mechanism functions correctly and that the safety lock is working correctly
	Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent and dry before use
Monthly	All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures
	Oil the swing-away parts. We recommend you use a professional lubricant system, e.g. Veidec Dry Lube
Annually	Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be sub-standard or faulty

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product. Contact your local dealer for instructions on repairs.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

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Supplier for:
Rest of the world

R82 UK Ltd.

Unit D4A,
Coombswood Business Park East
Coombswood Way,
Halesowen
West Midlands B62 8BH
England
Tel. +44 (0)121 561 2222
Fax +44 (0)121 559 5437
R82uk@R82.com
R82.co.uk
facebook.com/R82UK

Supplier for:
United Kingdom, Ireland

R82 GmbH

An den Wiesen 10
55218 Ingelheim
Germany
Tel +49 (0)6132 710 70
Fax +49 (0)6132 710 710
R82de@R82.com
R82.de
facebook.com/R82.de

Supplier for:
Germany, Switzerland, Austria

R82 Nederland B.V.

Tinweg 8
8445 PD Heerenveen
The Netherlands
Tel +31 (0)513 68 16 86
Fax +31 (0)513 68 19 84
R82nl@R82.com
R82.nl
R82.be

Supplier for:
The Netherlands, Belgium

R82 CZ, s.r.o.

Bohunicka 52
CZ-619 00 Brno
Czech Republic
Tel: +420 777 969 999
R82cz@R82.com
R82.cz
facebook.com/R82.cz

Supplier for:
Czech Republic, Slovakia

R82 Inc.

12801 E. Independence Blvd.
P. O. Box 1739
Matthews NC 28106
USA
Tel +1 800 336 7684
Fax +1 704 882 0751
us.info@R82.com
R82.com
facebook.com/R82Inc

Supplier for:
USA, Canada

R82 au Pty Limited

Unit 1, Level 2, 13 Lord Street
Botany NSW 2019
Australia
Tel +61 (02) 8213 6666
Fax +61 (02) 8213 6655
R82.com.au
facebook.com/R82Australia

Supplier for:
Australia

R82 China 中国

Room 920 Tower 2
China Central Place
79 Jianguo Road
Chaoyang District
Beijing 100025, China
Tel +86 10 5920 4399
Fax +86 10 5920 4222
R82cn@R82.com
R82.com.cn

Supplier for:
China

Find your local dealer

R82.org

DISTRIBUTOR: