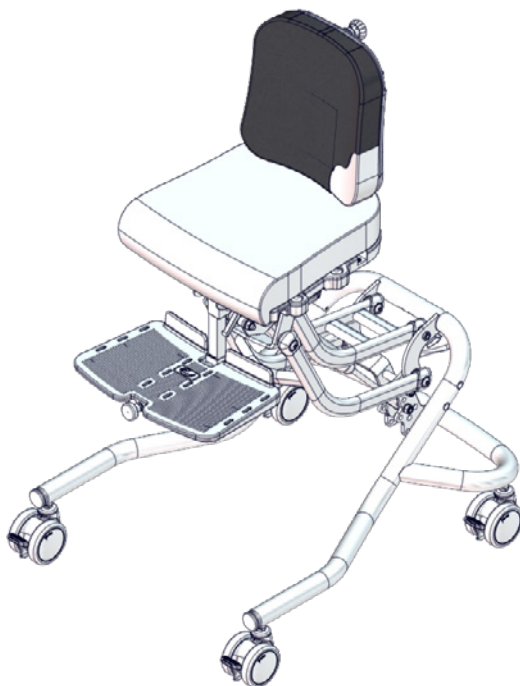
User manual

Wombat Living

R82[®] 
by Etac

M1075 - 2025-05-12 -Rev. 8.0


www.etac.com

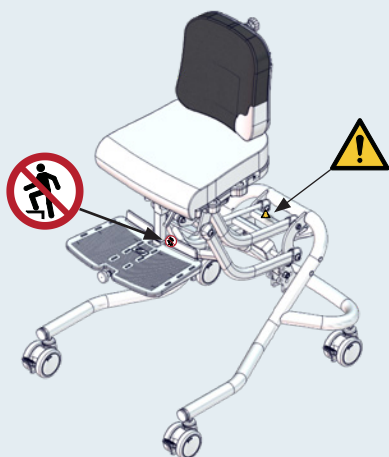
Wombat Living w/gas ⚠ 01

Wombat Living w/hydraulic ⚠ 01

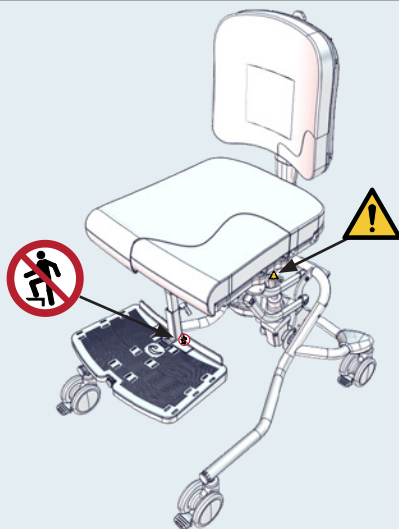
Wombat Living w/power ⚠ 01

01

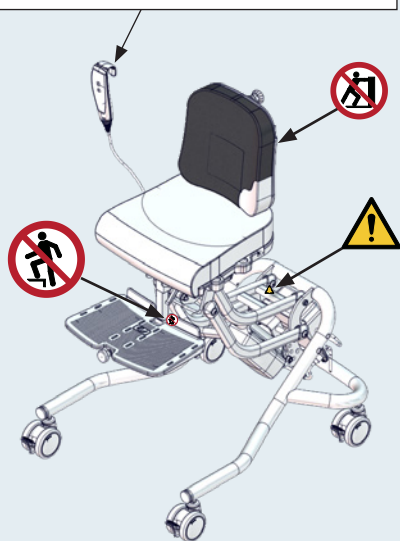The frame with power does not have a safety stop at the height adjustment function. Please remove the foot support before adjusting into lowest position.


Wombat Living w/tilt lever ⚠ 02

02

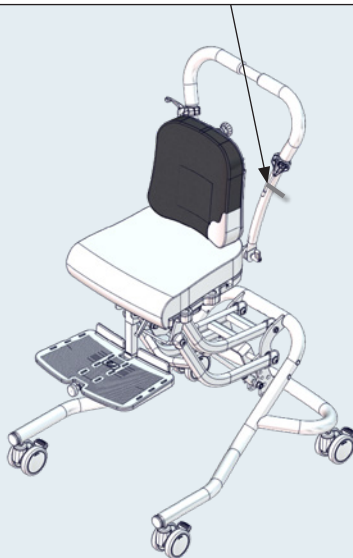Risk of loss of balance for the user when using the tilt function. The seat must only be tilted in a controlled manner


Wombat Living size 3 ⚠ 05






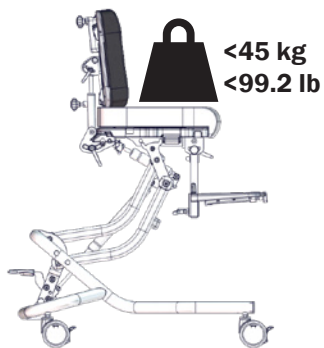
Wombat Living 1+2



Wombat Living 3

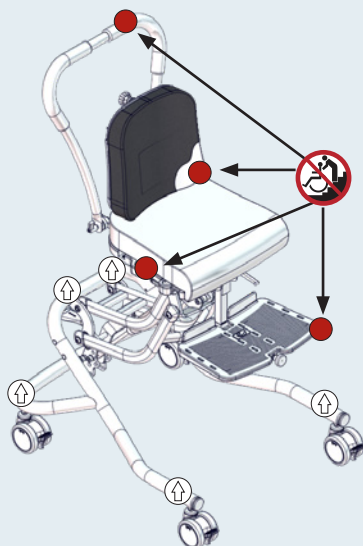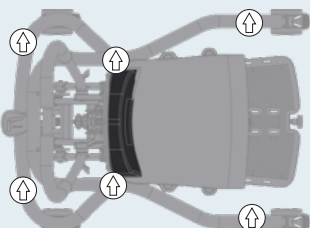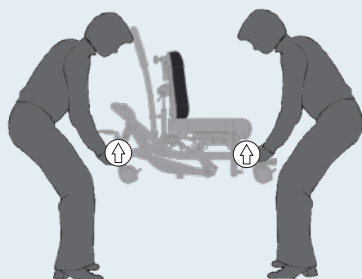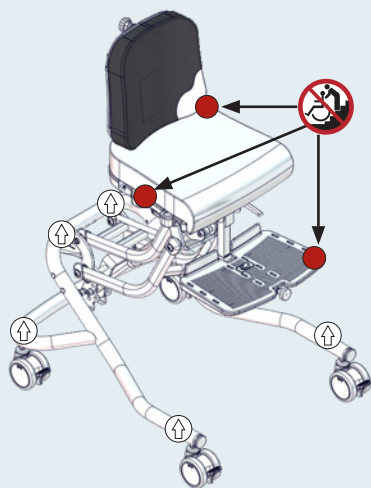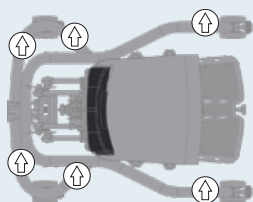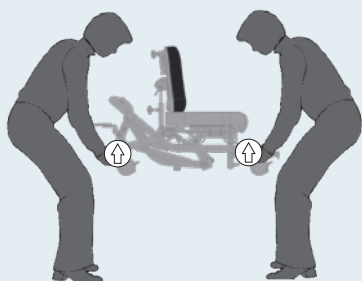
w/gas

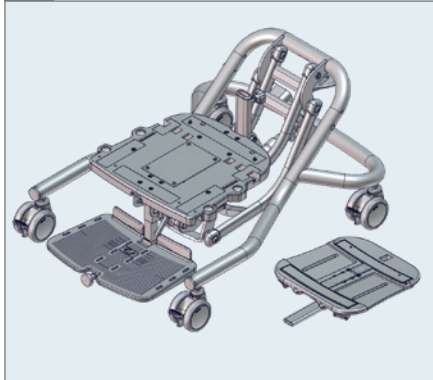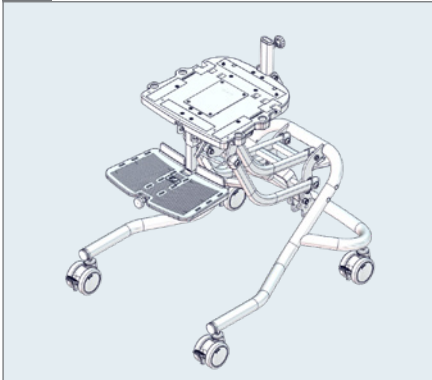
1.

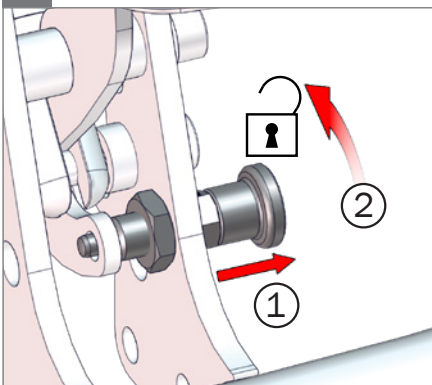Optional

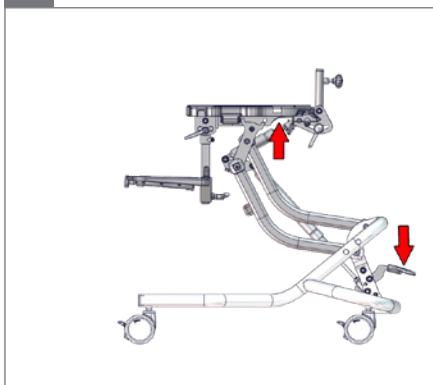
2.


3.

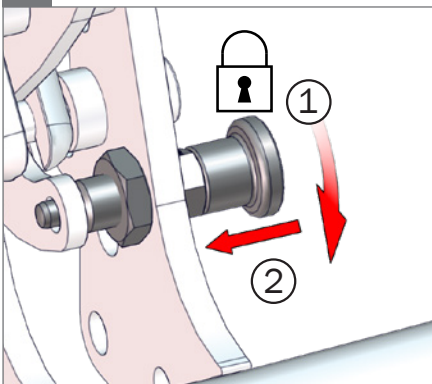Optional


w/gas

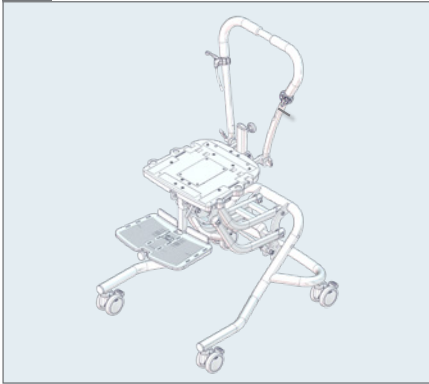
Size 1-2

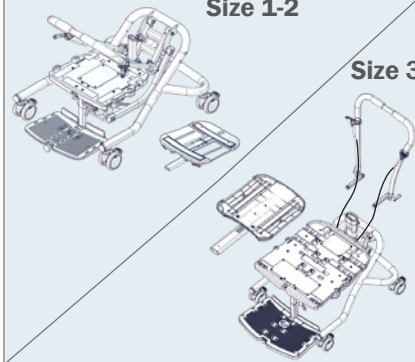Size 3

1.

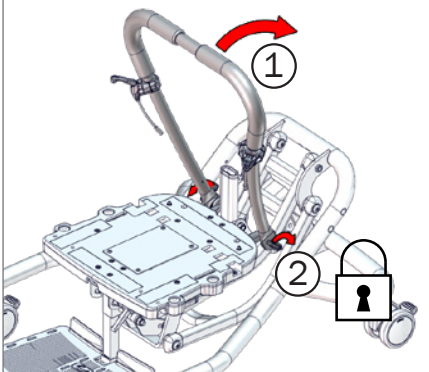Size 1-2


2.


3.

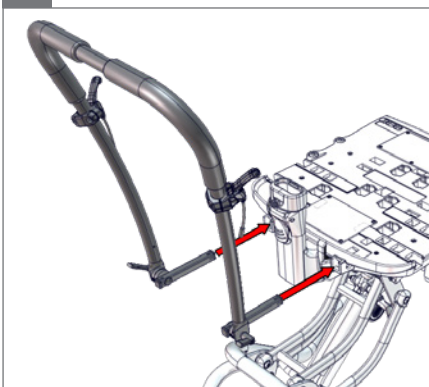Size 3


4.

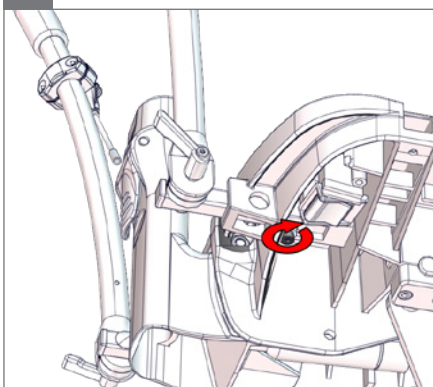Size 3 $\diamond$ 5 mm - 2,5 Nm


w/power

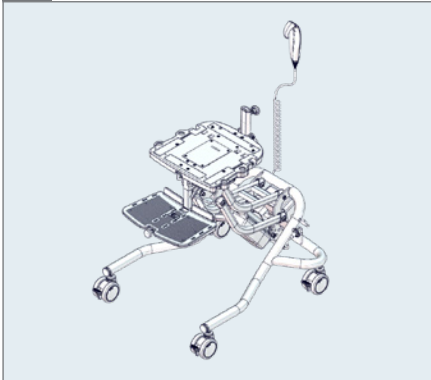
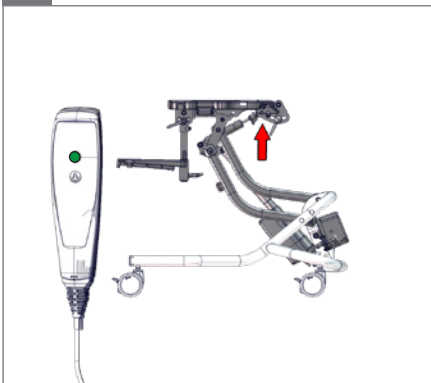
1.


w/hydraulic

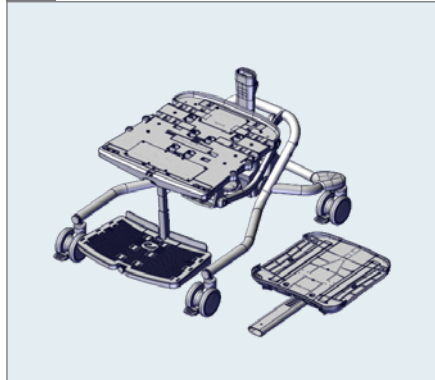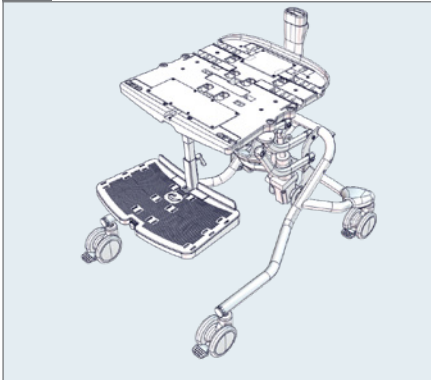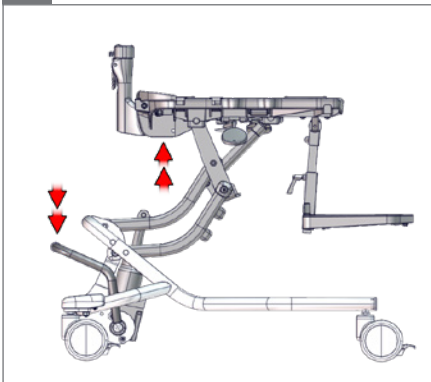
1.



Size 1-2

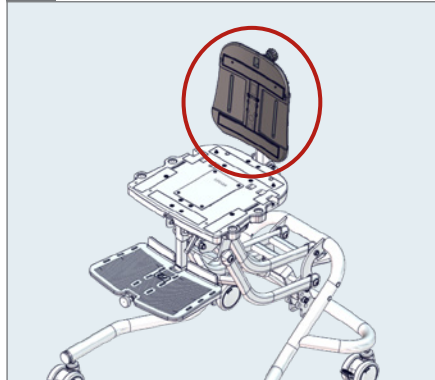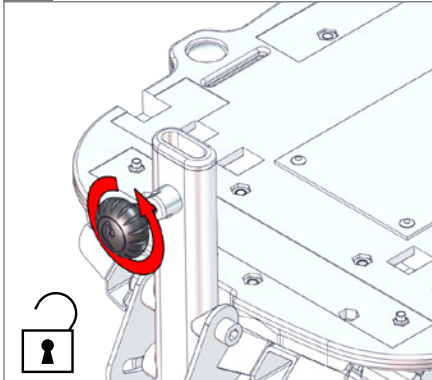
1.

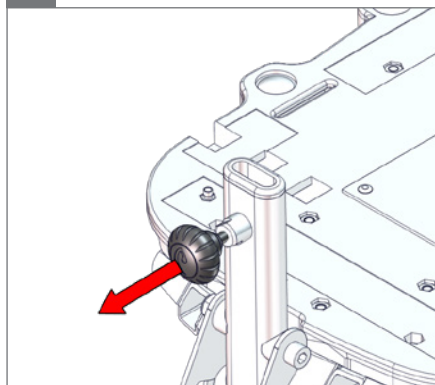
2.

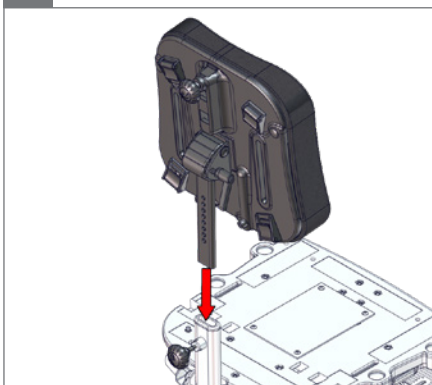
3.

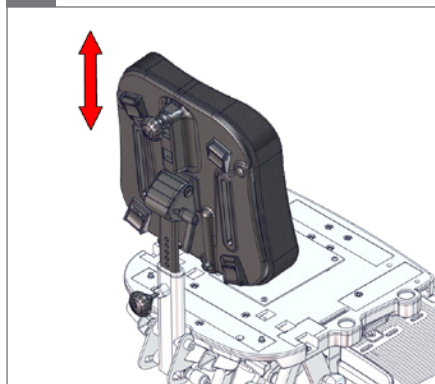4.

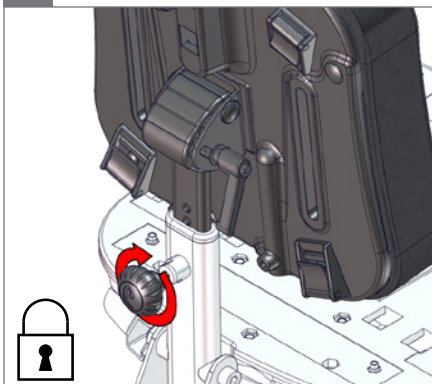
5.

Size 3

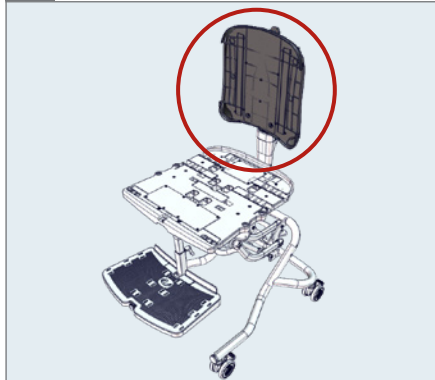
1.

⌀ 5 mm ⌀ 10 mm

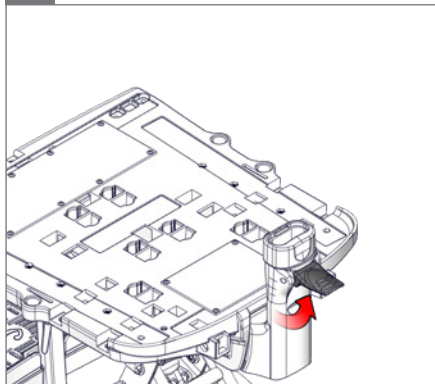
2.


3.

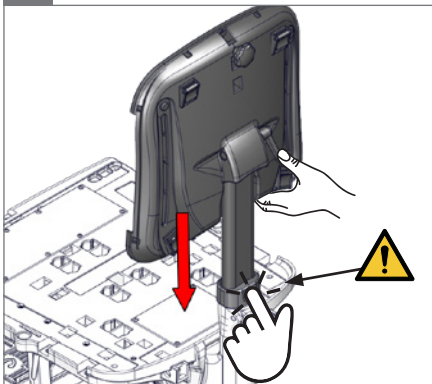⚠ 05





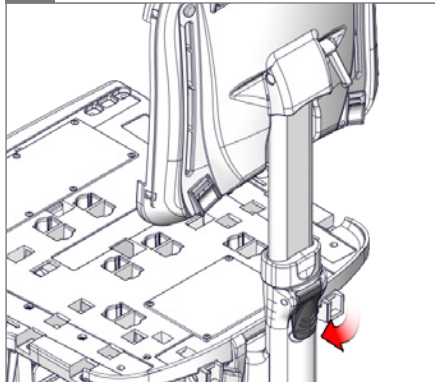
4.

5.

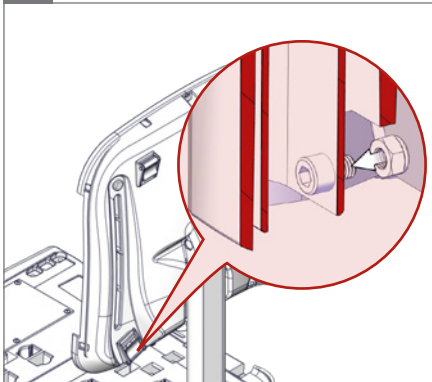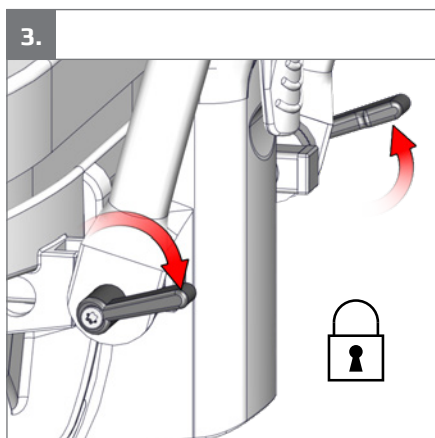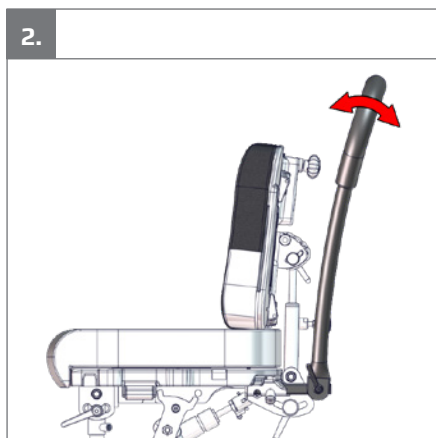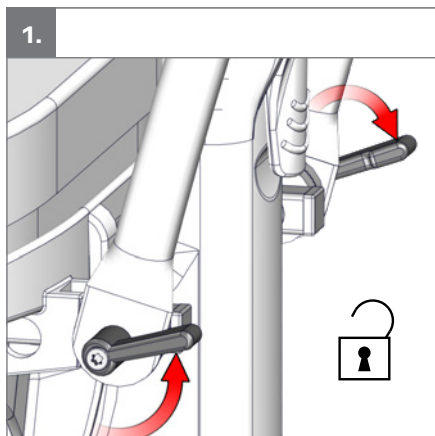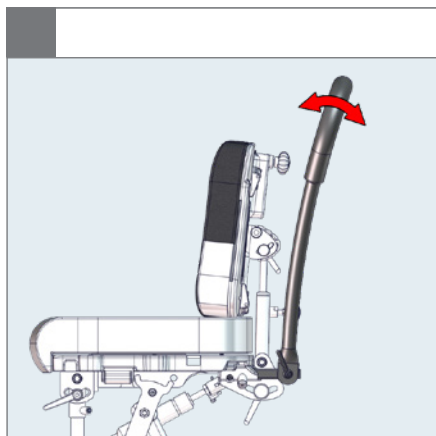⌀ 5 mm ⌀ 10 mm


A

w/gas

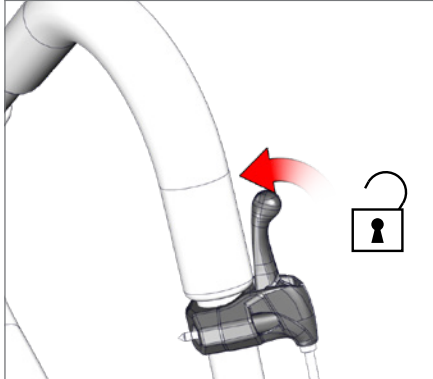
1.

2.

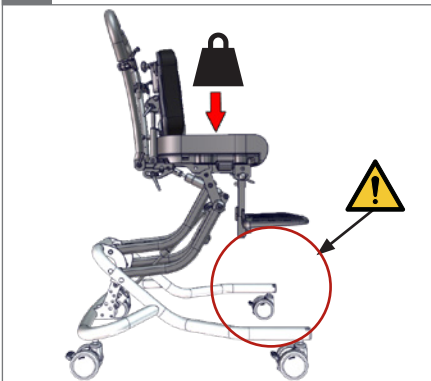⚠ 05

3.

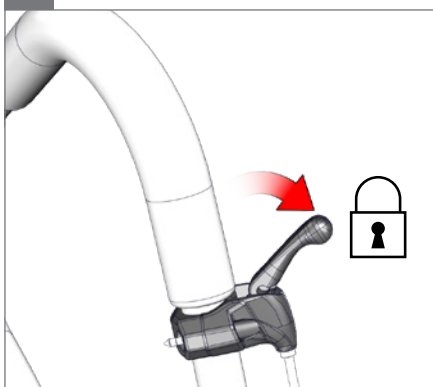
⚠ 05

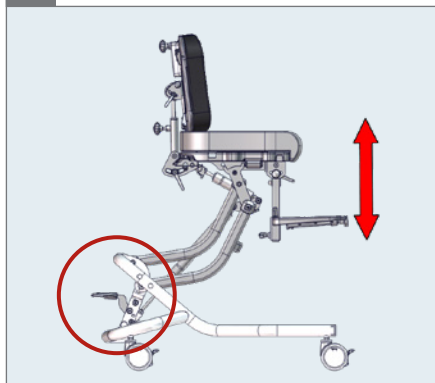
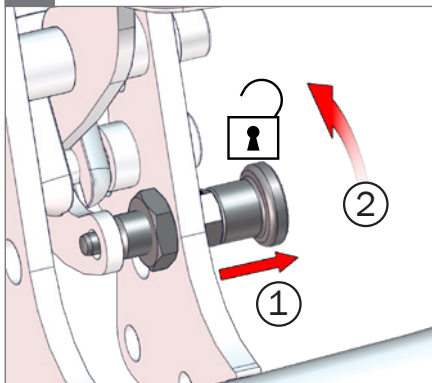
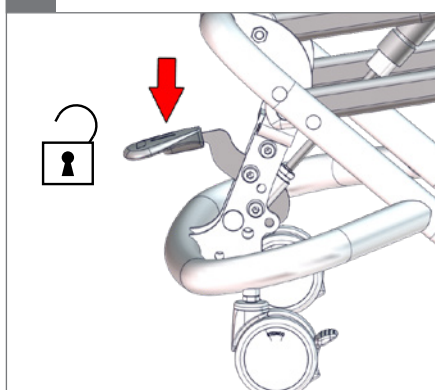
4.



5.


**B****w/gas****1.****Optional****2.****3.**

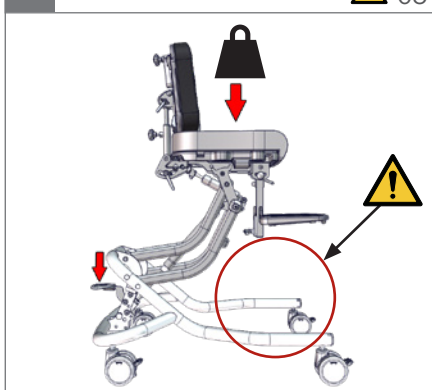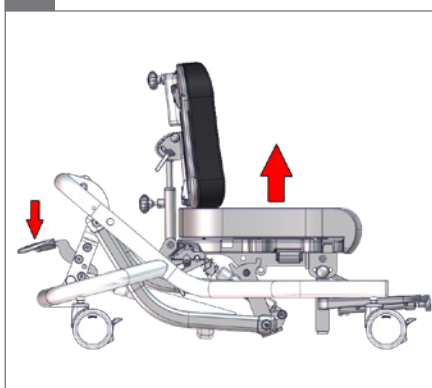05

**4.****5.**

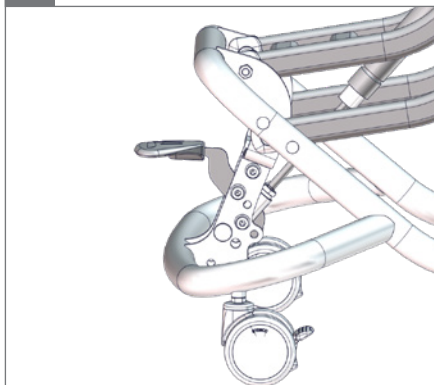
6.

7.

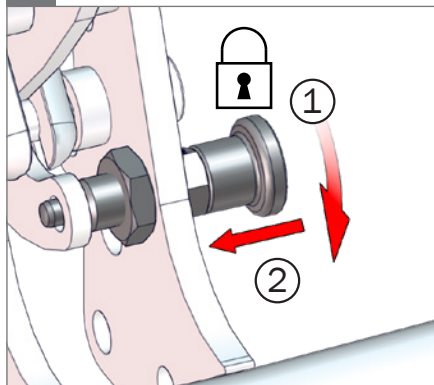
Optional


C

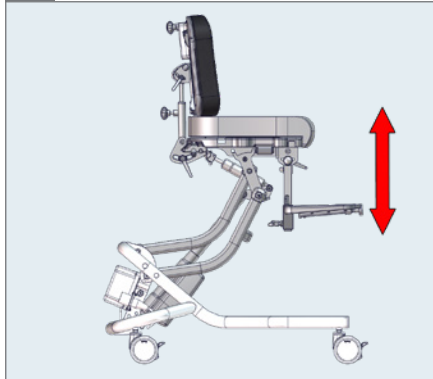w/power


1.

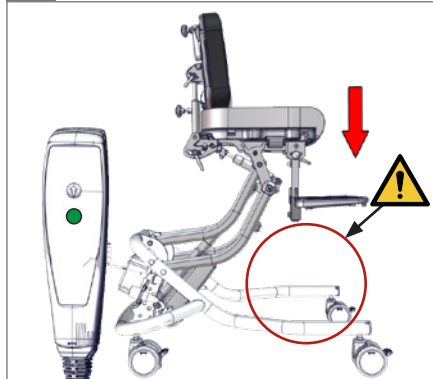⚠ 05

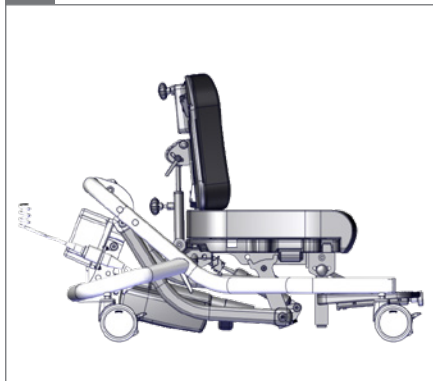2.


3.


D

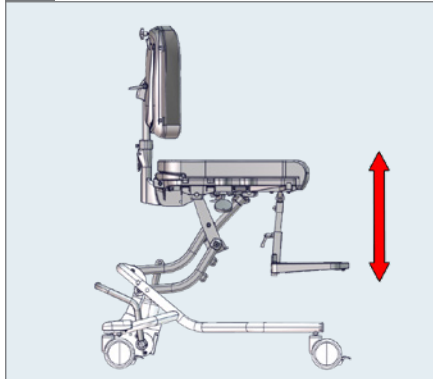w/hydraulic


1.

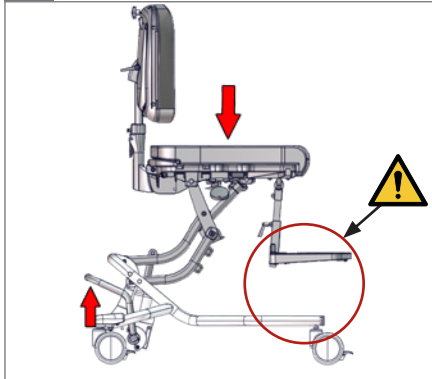
⚠ 05

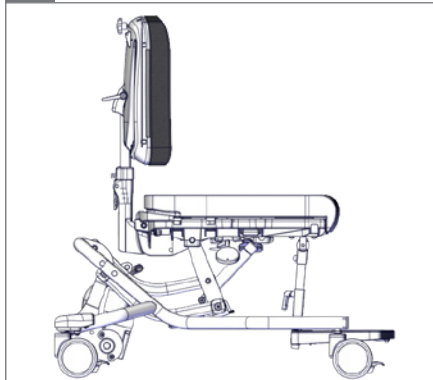2.

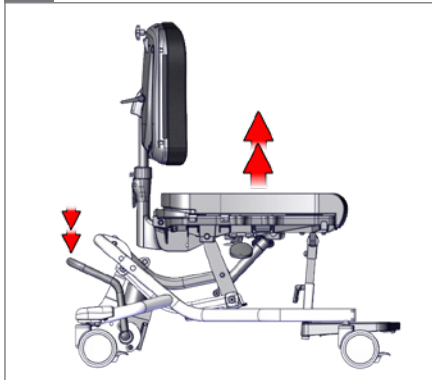
3.



Size 1+2

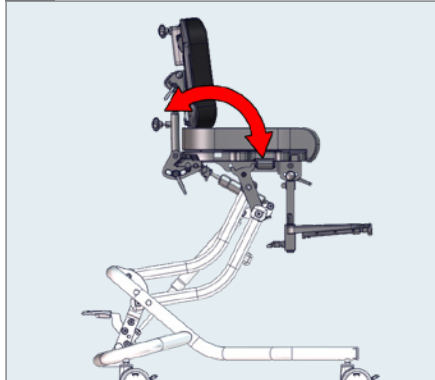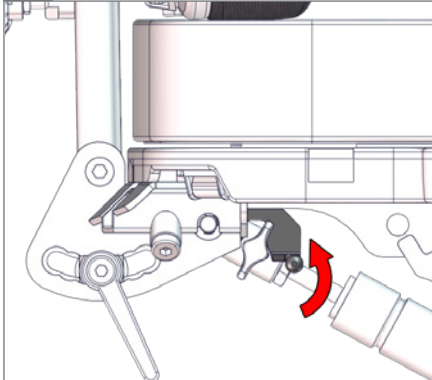
1.

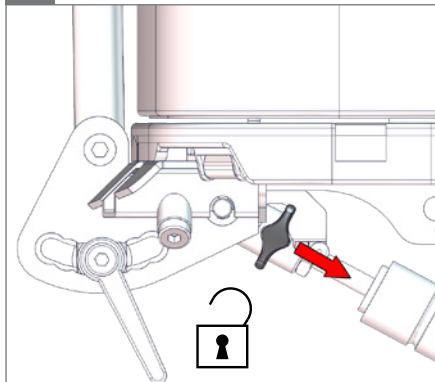
2.

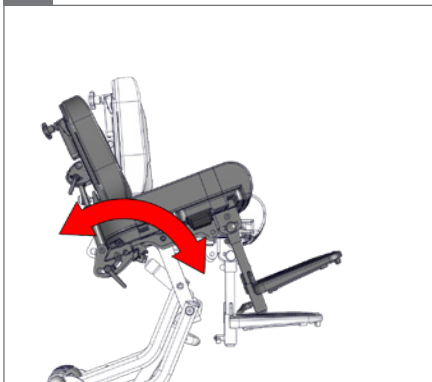
3.

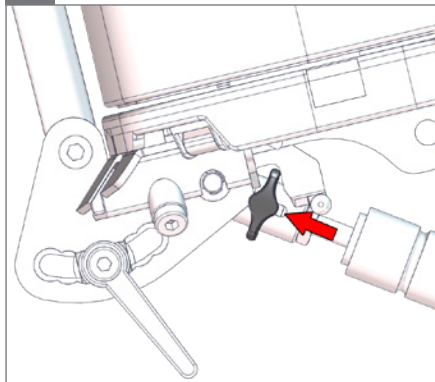4.

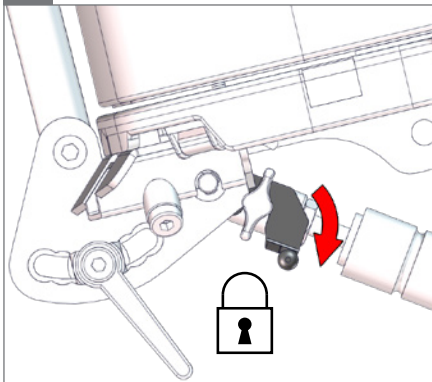5.



Size 3

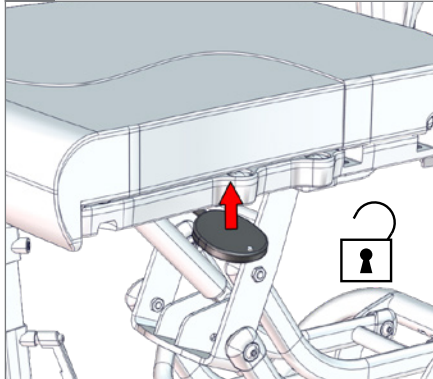
1.



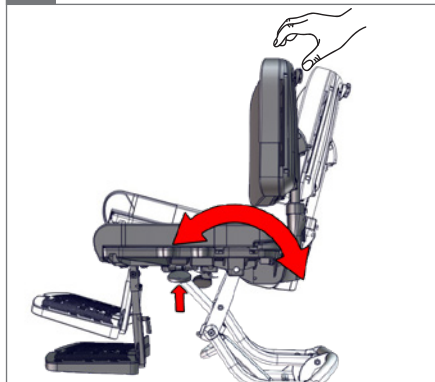
2.

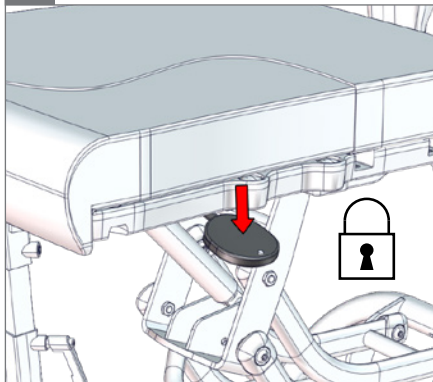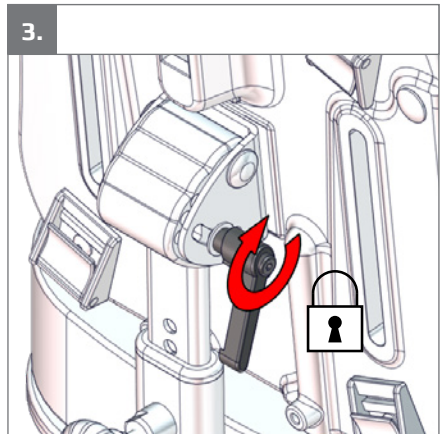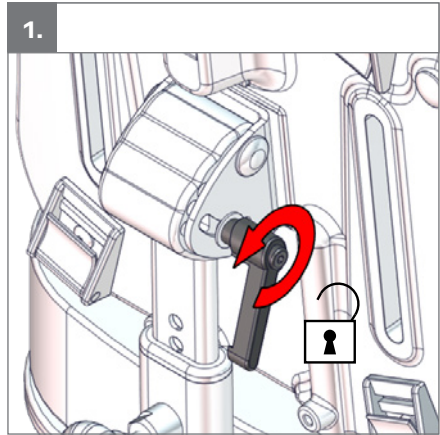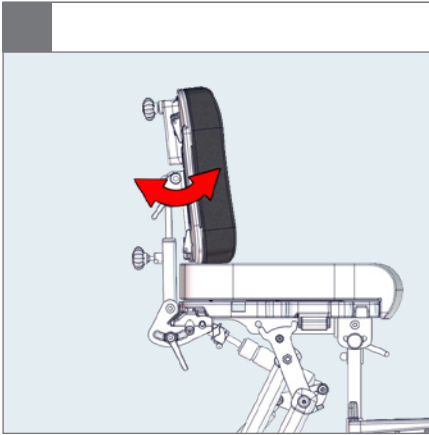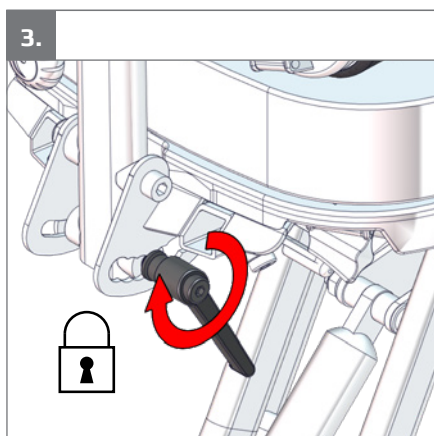
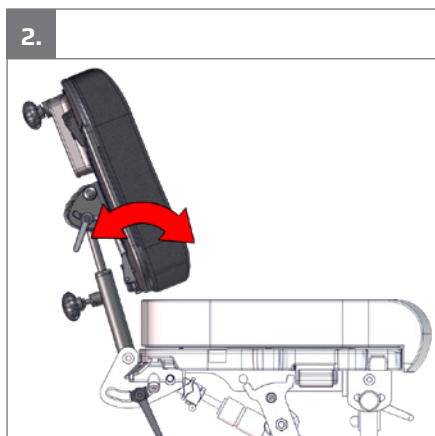
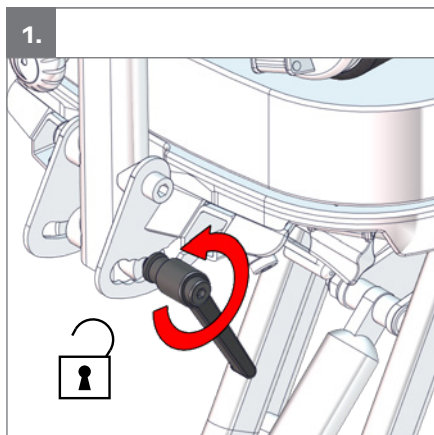
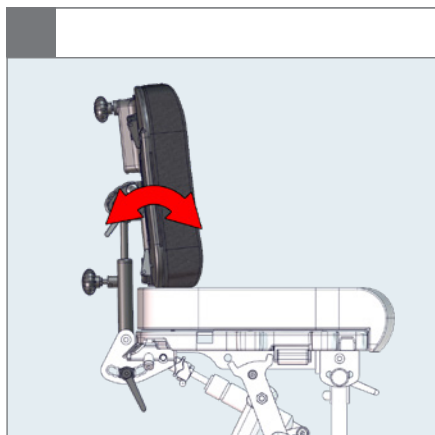
3.




Size 3

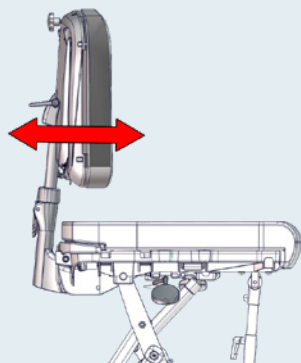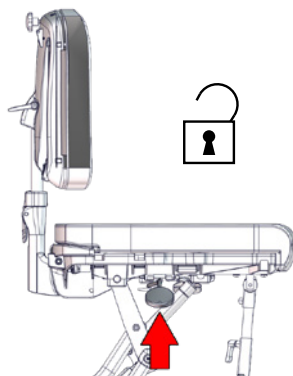
1.

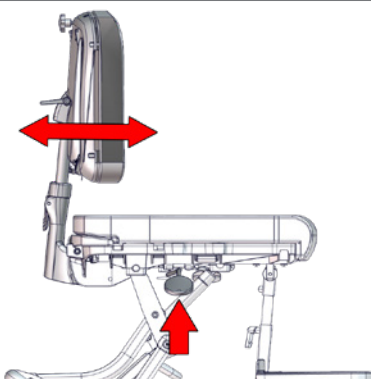
2.

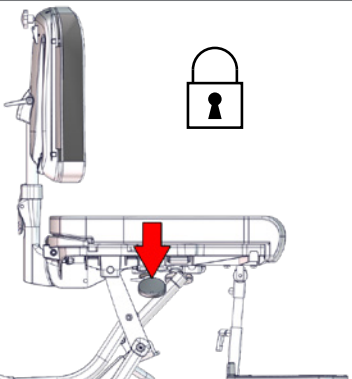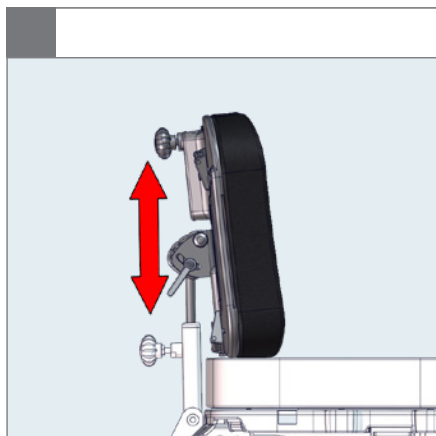
3.

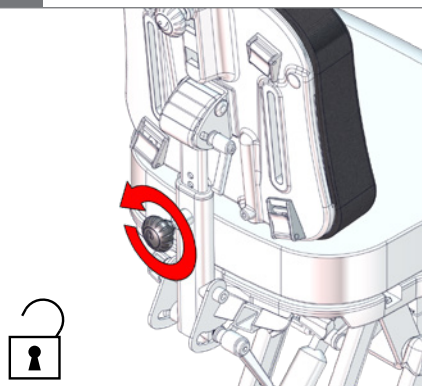
1.

2.

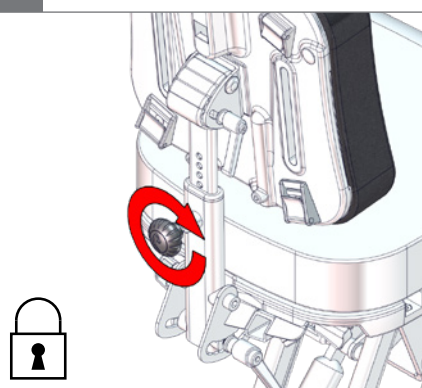
3.


Size 3

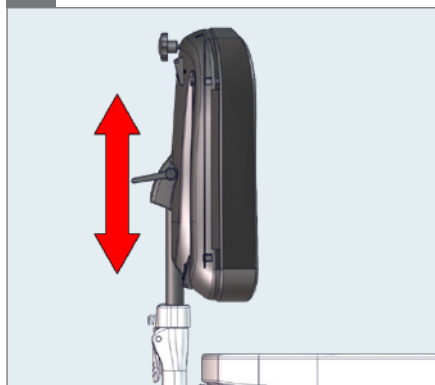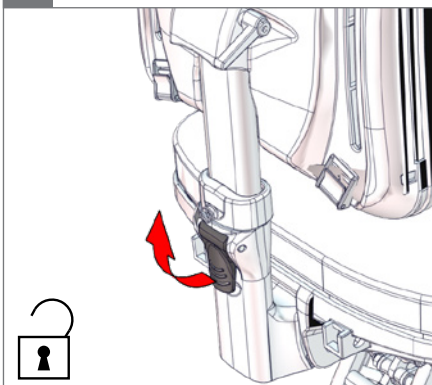1.







2.




⚠ 05

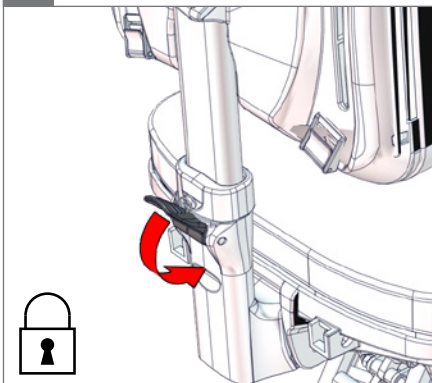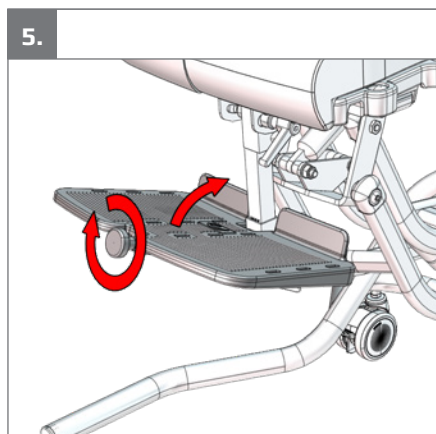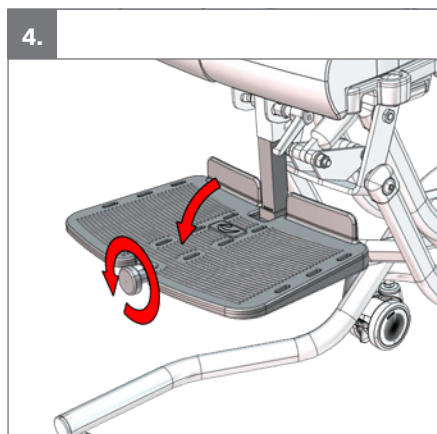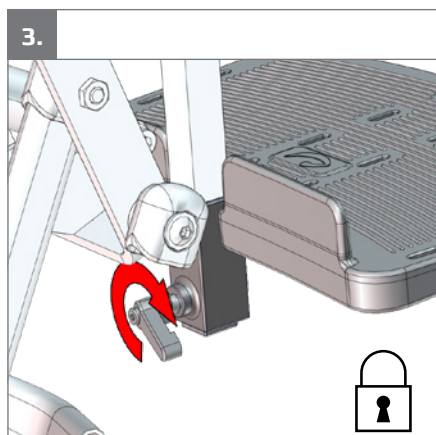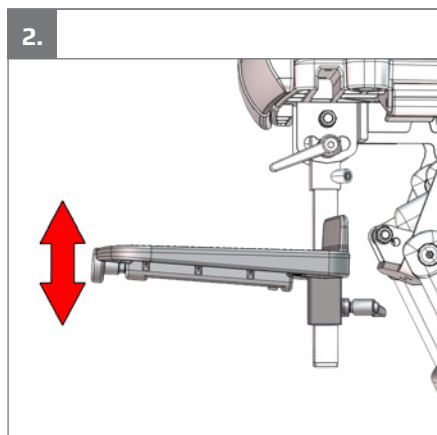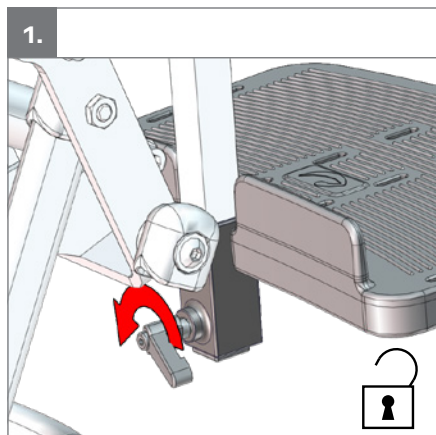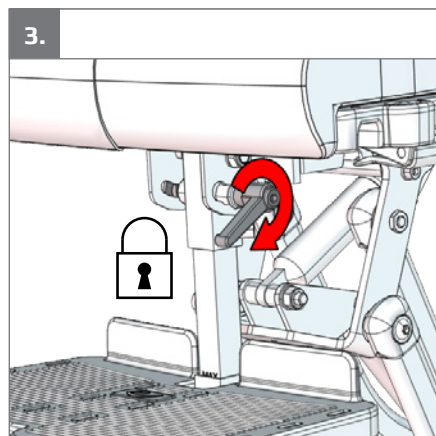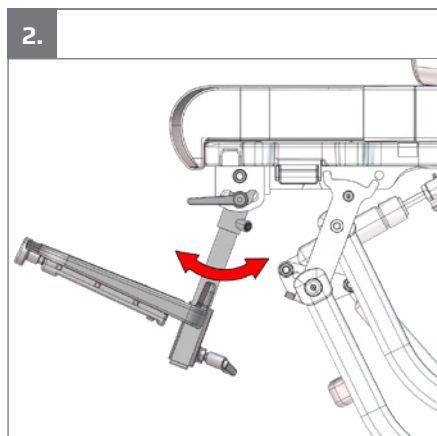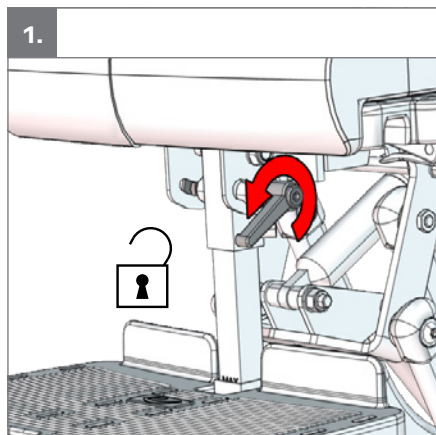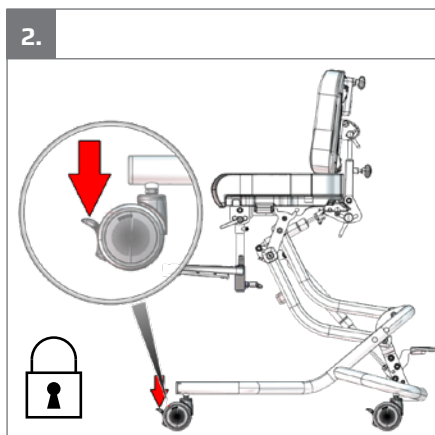
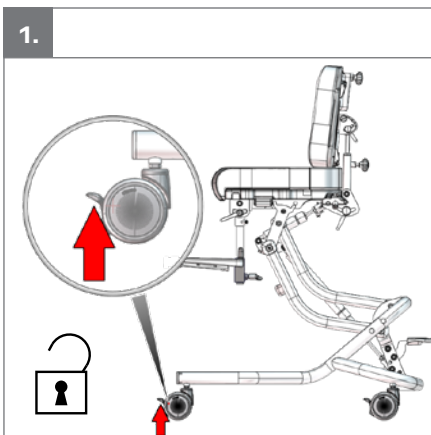
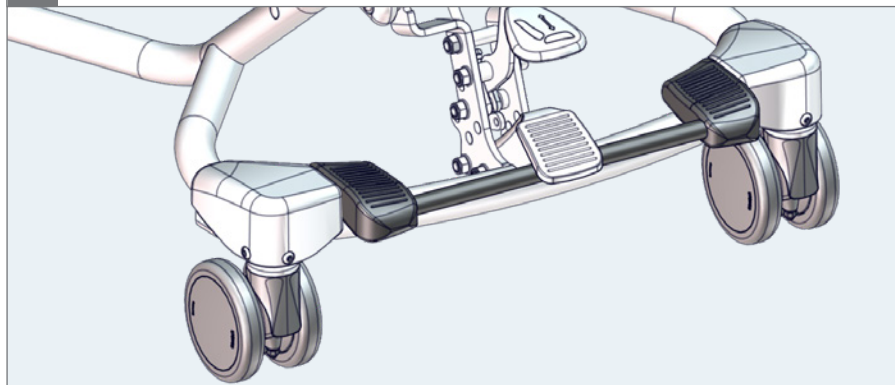
3.










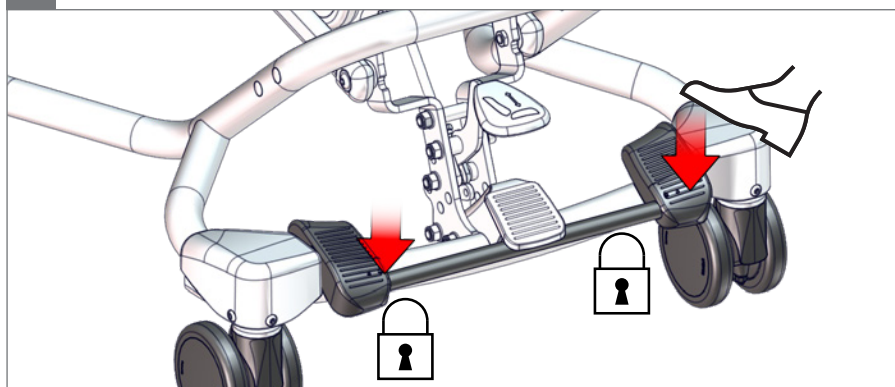
1.

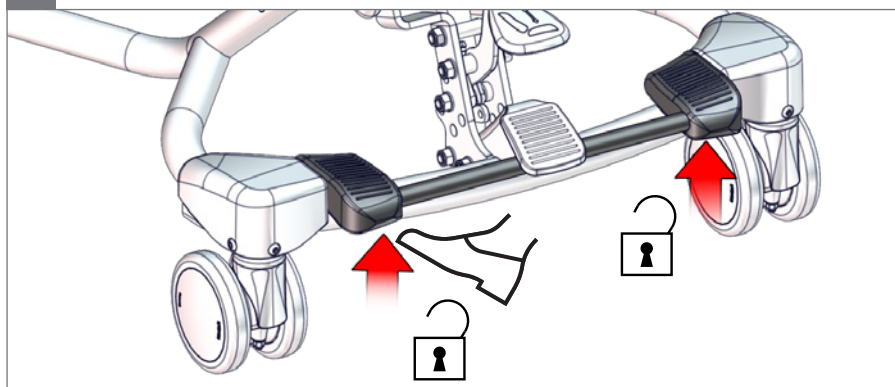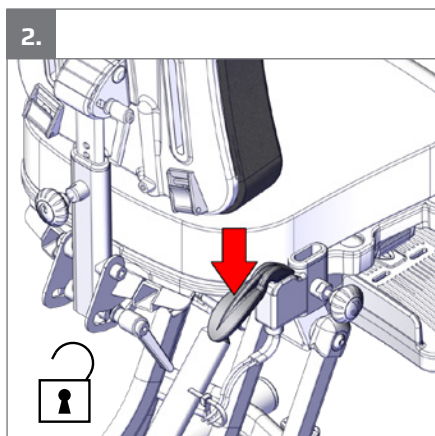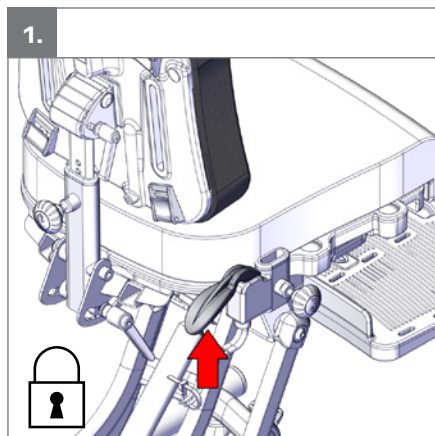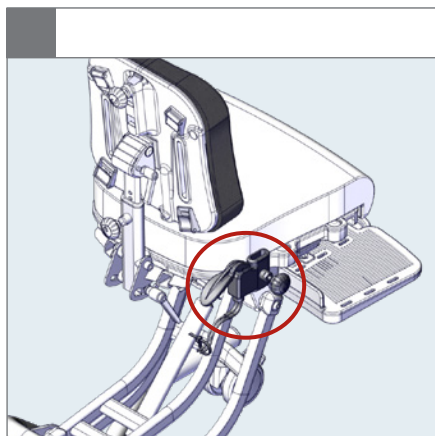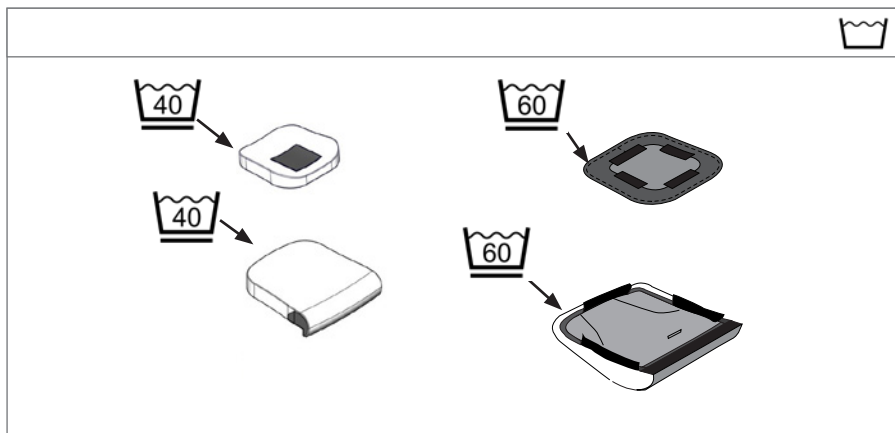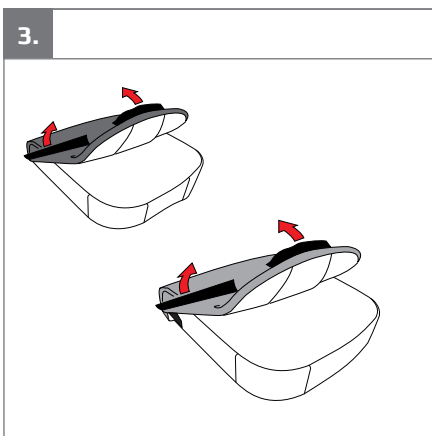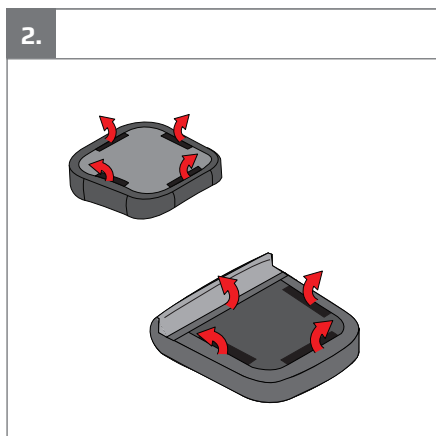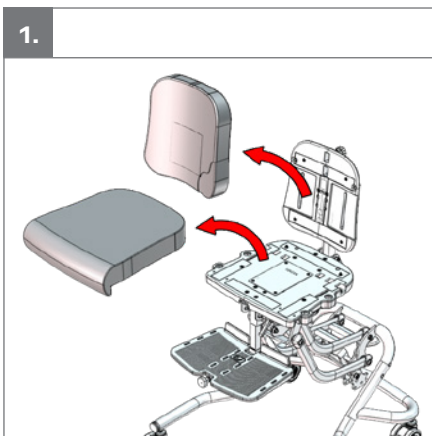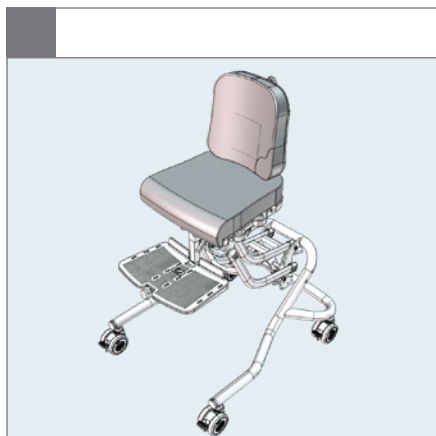2.



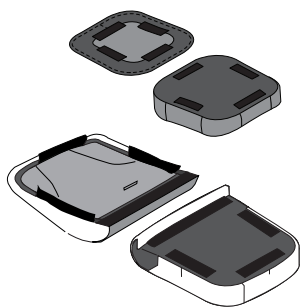
4.

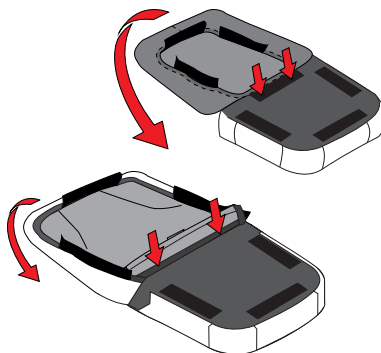
5.

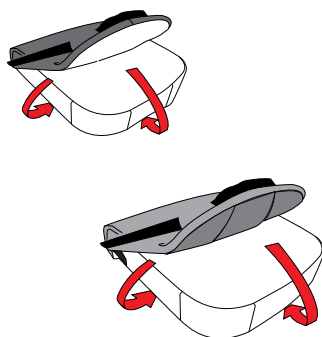
6.

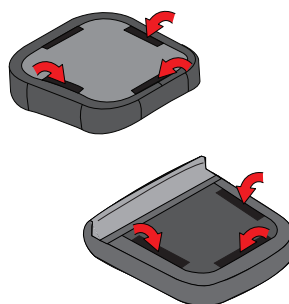
7.

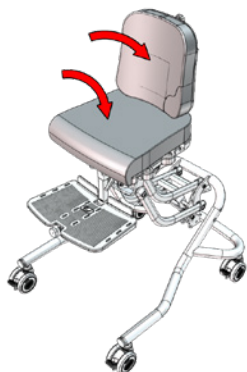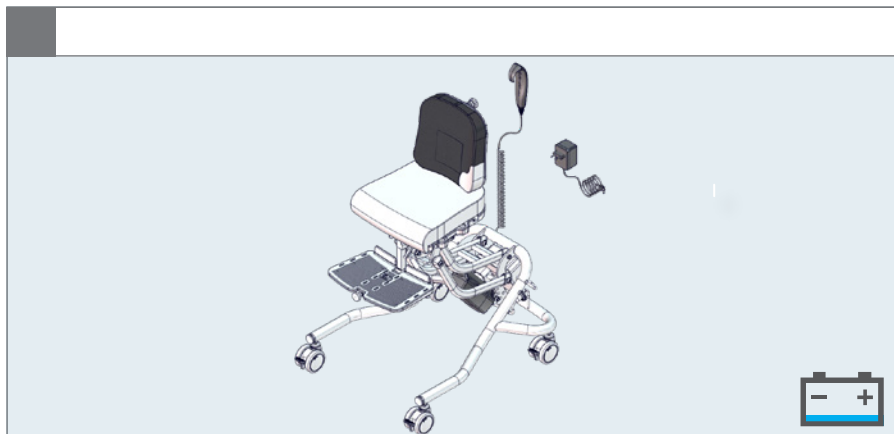
8.

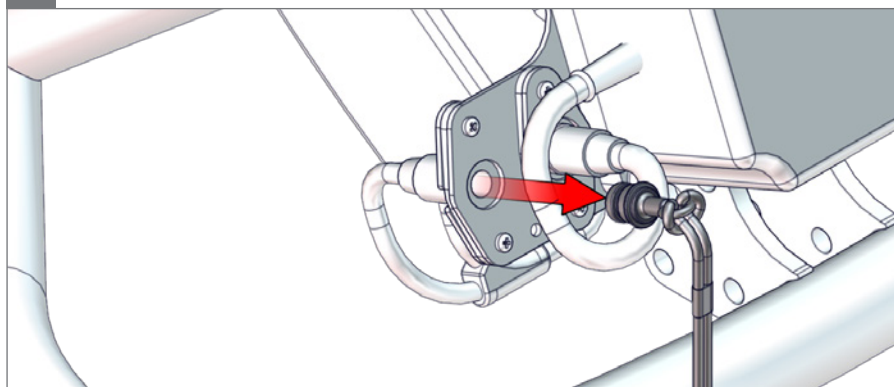
1.



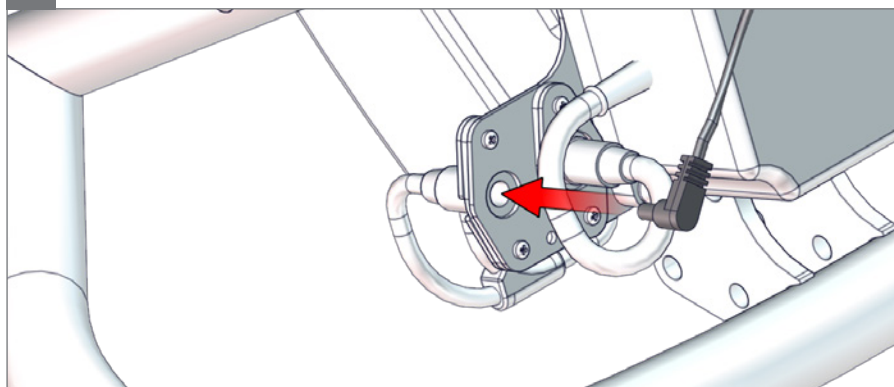
2.

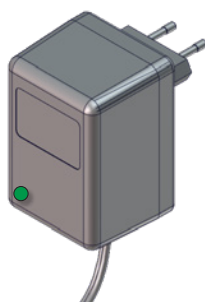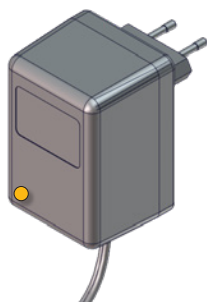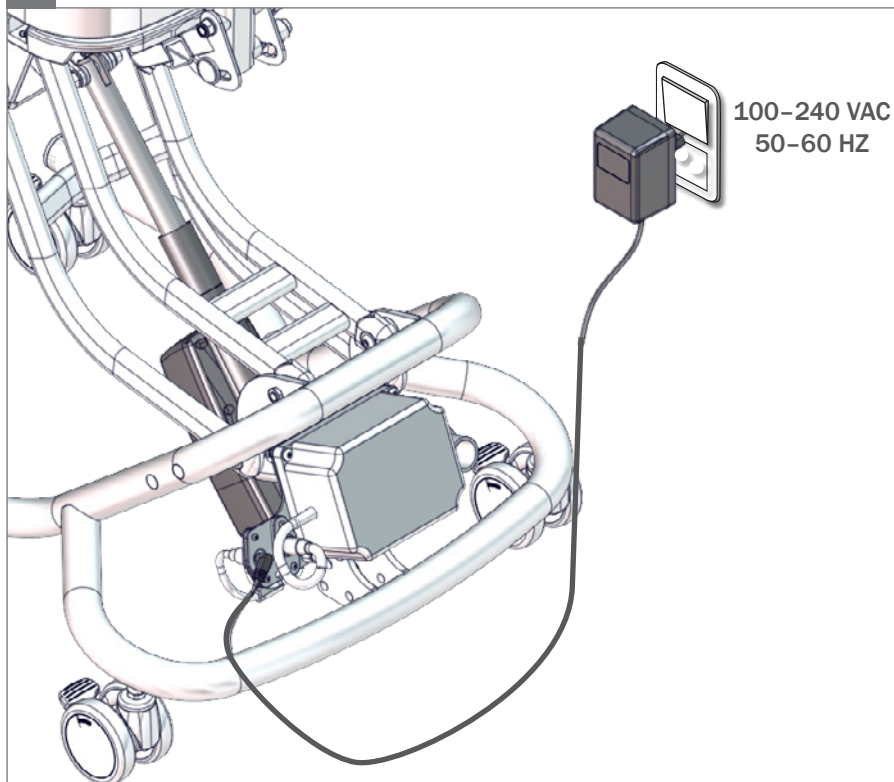
3.



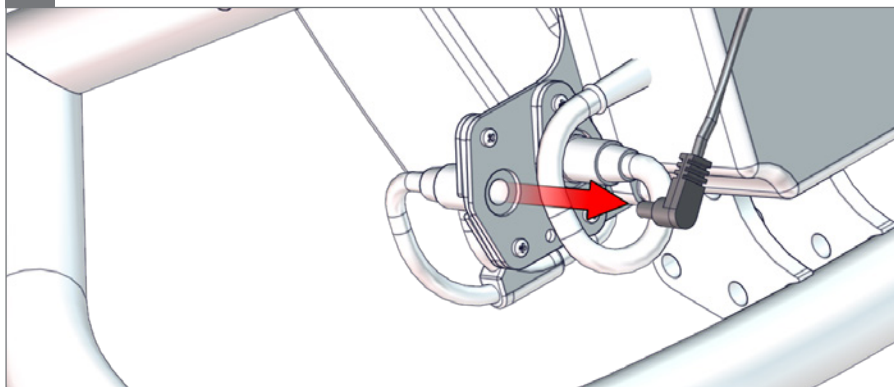
4.

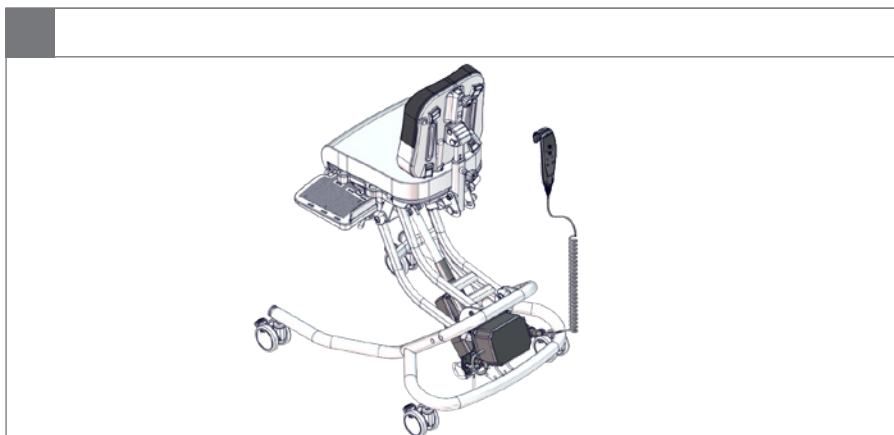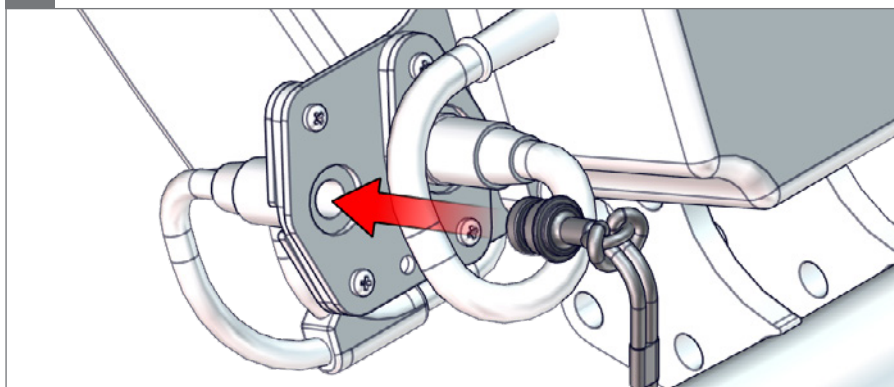
5.

Wombat Living w/gas & hydraulic

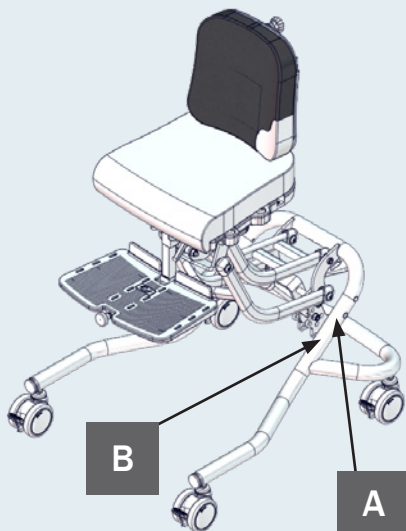Wombat Living w/power

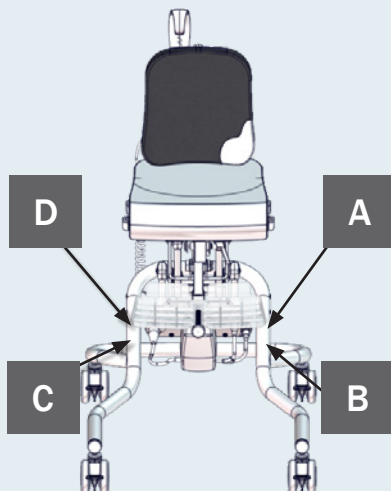A

R82
by Etac

R82 A/S, Parallevej 3
DK-8751 Gedved
etac.com
Made in xxxxxxxx

MD

CE UK CA

YYYY-MM-DD

B

Posting group
Product name, size x

DMR ver.: xxxxx

SN xxxxxxxxxxxx

Max. user mass: xxx kg
Max. rated load: xxx kg
Mass device: xxx kg

XXXXXXXXXXXXXXXXXXXX

XXXXXXXXXXXXXXXXXXXX

C

Storage and Transport	50°C	80%	1060 hPa
	-10°C	20%	700 hPa
Operating	40°C	80%	1060 hPa
	5°C	20%	700 hPa

DUTY CYCLE: 10% MAX 2 min./18 min.

D

IP 24

IP 24

EN: Warning: This symbol appears in the User Guide along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger
01 = Entrapment Hazards
02 = Tilting Hazards
03 = Transportation in motor vehicles
05 = Squeezing Hazard

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mm (inch) kg (lb)

	 1 Gas & power	 2 Gas & power	 3 Gas & power	 3 Hydraulic
 Effective seat width	300 mm (11¾")	370 mm (14½")	420 mm (16½")	420 mm (16½")
 Back width	250 mm (9¾")	300 mm (11¾")	350 mm (13¾")	350 mm (13¾")
 Effective back width, contoured cushion	170 mm (6¾")	230 mm (9")	270 mm (10½")	270 mm (10½")
 Effective seat depth	180–310 mm (7–12¾")	250–380 mm (9¾–15")	320–500 mm (12½–19¾")	320–500 mm (12½–19¾")
 Backrest height	280–340 mm (11–13½")	310–380 mm (12¼–15")	390–475 mm (15¼–18¾")	390–475 mm (15¼–18¾")
 Backrest height (Long Back)	360–430 mm (14¼–17")	395–460 mm (15½–18")	460–550 mm (18–21¾")	460–550 mm (18–21¾")
 Backrest angle	0° – 28°	0° – 28°	0° – 33°	0° – 33°
 Seat plane angle	(-10°) – (30°)	(-10°) – (30°)	(-10°) – (22°)	(-10°) – (22°)
 Footrest to seat distance	150–250 mm (6–9¾")	150–350 mm (6–13¾")	150–440 mm (6–17¾")	150–440 mm (6–17¾")
 Leg to seat surface angle	(-30°) – (30°)	(-30°) – (30°)	(-30°) – (30°)	(-30°) – (30°)
 Seat surface height at front edge*	230–610 mm* (9–24")	230–610 mm* (9–24")	330–630 mm* (13–24¾")	390–630 mm* (15¼–24¾")
 Overall width	550 mm (21¾")	550 mm (21¾")	550 mm (21¾")	610 mm (24")
 Total height	640–990* (25¼–39")	670–1030* (26½–40½")	780–1145* (30¾–45")	780–1145* (30¾–45")
 Length	650 mm (25½")	650 mm (25½")	650 mm (25½")	750 mm (29½")
 Maximum user mass	45 kg (99.2lb)	45 kg (99.2lb)	70 kg (154.3 lb)	70 kg (154.3 lb)
 Mass device	18 kg (39.8 lb)	18.5 kg (40.8 lb)	23.5 kg (51.8 lb)	26.2 kg (57.8 lb)
 Maximum rated load	65 kg (143.3 lb)	65.5 kg (144.4 lb)	95.5 kg (210.5 lb)	98.2 kg (216.5 lb)

Accessories

 Seat width between hip supports	105–300 mm (4¼–11¾)	170–370 mm (6¾–14½)	215–420 mm (8½–16½")	215–420 mm (8½–16½")
Seat width, pelvis support w/ sides	170–240 mm (6¾–9½")	230–300 mm (9–11¾")	290–360 mm (11½–14¼")	290–360 mm (11½–14¼")
Width, pelvis support without sides	240 mm (9½")	240 mm (9½")	290 mm (11½")	290 mm (11½")
Armrest to seat distance	65–175 mm (2½–7")	65–175 mm (2½–7")	65–175 mm (2½–7")	65–175 mm (2½–7")
Armrest to seat distance, long rod and pad	65–265 mm (2½–10½")	65–265 mm (2½–10½")	65–265 mm (2½–10½")	65–265 mm (2½–10½")

EN: *The measurements above are based on the use of 75 mm castors for the gas and power versions and 100 mm castors for the hydraulic version with the frame in the most basic configuration. Different settings may result in another set of measurements.

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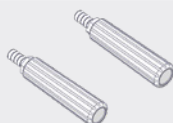
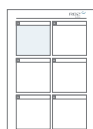
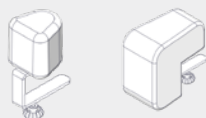
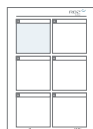
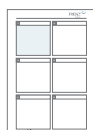
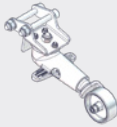
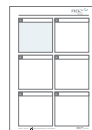
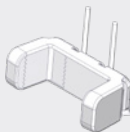
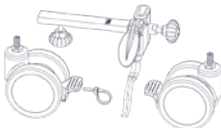
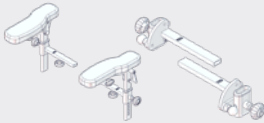
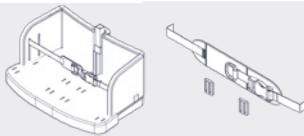
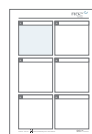
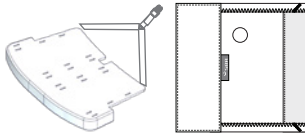
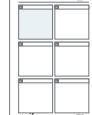
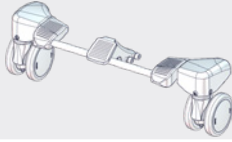
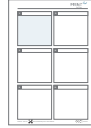
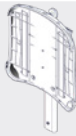
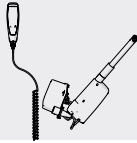
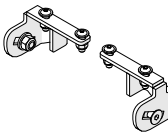
ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and adults with special needs. To fully benefit from the options offered by this product please read this User Manual before use and save it for future reference.

Intended purpose

Chair for children and adults with disabilities.

Indications

The device can be used by children and adults with disabilities, who lack the ability to maintain a seated position. The target group for the device is based on functional ability and not a specific health condition or age.

The user is the person sitting in the device. The caregiver is the person operating the device. Never leave the user unattended in this device. Ensure permanent supervision by a caregiver.

The use of the device will be discontinued multiple times during the day where the user will be lying down, standing or sitting in another assistive device or a conventional product.

Contraindications

There are no known contra indications.

Precautions

Special attention should be given to the considerations about the configuration of the device and the use of accessories for users with limitation in their:

- sensory functions and the sensation of pain consider extra checks of the pressure caused by the support provided by the device and its accessories.
- movement function (e.g. experience involuntary movements), consider using a device with a dynamic backrest.
- ability to maintain a seated position, use the device with accessories (e.g. side supports, vests, belts or arm supports and a tray).
- ability to maintain the position of their head, consider using accessories that provide support to the positioning of the head.

Application areas

For indoor use.

Enclosed tools:

Wombat Living 1 & 2

- 3, 4 & 5 mm Allen key

Wombat Living 3

- 3, 4, 5 & 6 mm Allen key
- 10 mm spanner

CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745).

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

Combinations

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Alignment of user weight/mass of the combined seat and base, approval for transportation in vehicles of both products (ISO16840-4 for seat and ISO7176-19 for base)
- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

The symbol of the crossed-out trash can on batteries or accumulators states that they must not be disposed of in household waste at the end of their service life. If batteries or accumulators contain mercury (Hg), cadmium (Cd) or lead (Pb), you will find the respective chemical symbol below the symbol of the crossed-out trash can. You are legally obliged to return old batteries and accumulators after use. You can do this free of charge at the retail store or at another collection point in your area. Addresses of suitable collection points can be obtained from your city or local

government.

Batteries can contain substances that are harmful to the environment and human health. Special care should be taken due to the special risks involved in handling lithium-containing batteries. The separate collection and recycling of old batteries and accumulators is intended to avoid negative effects on the environment and human health. Please avoid the generation of waste from old batteries as much as possible, i.e., by preferring batteries with a longer service life or rechargeable batteries. Please avoid littering of public spaces by not carelessly leaving batteries or battery-containing electrical and electronic equipment behind. Please consider ways to reuse a battery instead of disposal, for example by reconditioning or repairing the battery.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions. If an adverse event occurs in connection with the device, the incident must be reported to your local dealer and the national competent authority in a timely manner. The local dealer forwards the information to the manufacturer.

Caregiver

- Read the “User Manual” carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user.
- Always use the correct lifting techniques and aids for this purpose.
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver.
- Repairs/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier.
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user manual.
- Before the user is allowed to operate the remote control, the user's cognitive state must be assessed and considered to be sufficient to operate it.
- There is a risk of loss of balance for the user when using the tilt function. The seat must only be tilted in a controlled manner.
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible.
- Transportation of the product whilst the user is in it is not allowed. It is allowed to use the product with the user safely strapped in it for transfers over short distances. Do not cross any obstacles during transfer.
- Ensure that moving parts do not squeeze or trap body parts when adjusting them.
- Ensure that body parts are not trapped in openings for mounting of supports/accessories.
- Do not let children and youngsters play with the handset.
- It is important to remove the transport plastic bag before using the handset. Cables must be placed in such a way that users or carers cannot trip and injure themselves.
- Be aware of any MAX mark indications before adjustment.
- Use this product on a level surface without any obstacles. Be aware of reduced

maneuverability on slippery surfaces.

- When using on inclines, please refer to the page with measurements.
- Observe the surface temperature on the product, before placing a user in it. This goes especially for users with insensitive skin as they cannot feel heat or cold. If the surface temperature is above 41 °C then allow the product to cool before use.
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture.
- Only use the power unit within specified working limits.
- Do not expose the piston rod to scratches etc. - when adjusting the gas spring, it is important to place a cloth between the tool used and the piston rod.
- Ensure that the castors/wheels are fully functional and secure before use.
- Activate the brakes before placing the user into the product.
- Inspect the product and all of its accessories and replace any worn out parts before use.
- Ensure the stability of the product before placing the user in it.
- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children.

User

- The height and tilt functions can be operated by the user under the caregiver's permanent supervision.

Product

- The power unit does not have a safety stop.
- Do not attempt to repair any part of the power unit.
- The foot support does not have a weight bearing function and must only be used for support.
- When the foot support is close to the floor and the castors, the castors may be unable to swivel.
- This device is tested for resistance to ignition at level non-ignition in accordance with applicable standards.
- The product might lose its resistance to ignition if non-original R82 parts are used.

- This product must not be occupied in a motor vehicle. The user must be positioned in a vehicle seat and the product must be stored in a luggage compartment.
- The user must not occupy the product while it is being transported into or out of a vehicle by a lift.
- Unoccupied device is suitable for land and air transport.
- Details covering EMC can be found in the appendix "EMC details" supplied with the product.
- This product contains small parts which could present a choking hazard if removed from their intended placing.
- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size product with a higher max. load or a different R82 product should be considered.
- When changing user, clean the product and disinfect it if necessary. Carry out necessary adjustments after cleaning.

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the device. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this device in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Reuse

This device and its accessories/components are suitable for reuse.

Refurbishment for reuse

Before reuse or changing the user, follow the service checklist, maintenance and washing instructions and carry out regular inspections.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the device is inspected yearly, every time the device is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the device.

Service checklist

According to their intended use, check:

- that the nuts and bolts on the product are tightened
- the product for cracks or signs of worn out parts
- that all labels on the product are intact
- that all accessories, attachments and functions on the product are intact
- the adjustment options for the foot rest and that it is possible to lock the angle and fasten the telescopic tubes in position
- that the seat back is adjustable and can be locked in position
- that the seat plane is adjustable and stays in position after adjustment
- that the frame is adjustable in height
- that the powered versions are able to retain power after charging the battery

and adjust if necessary.

Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to:

Wipe off any residue and dirt from the device, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use.

Carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main device

This device can be washed at 60 °C using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the device. Unmount all accessories and wash them separately.

Tilt/rotate the device after washing to facilitate water drainage from within the device.

Before washing

Avoid degreasing of the piston rods, the actuators should be retracted to minimum stroke.

- Remove all accessories, unplug all cables and make sure to dismount the power unit.
- Do NOT clean the power unit with a high pressure cleaner or steam cleaner.

After washing

- Ensure correct installation of the power unit.
- Check that the handset functionalities works correctly

Fabric and covers

The cushion covers can be washed at 60 °C using a mild detergent in a washing machine. For the remaining fabric and covers, check each item's care label for specific instructions. Separate foam and cover before wash.

Foam

The following foam inserts can be washed at 40 °C using a mild detergent in a washing machine:

- Seat cushion foam inserts
- Back cushion foam inserts

During and after washing the foam, do not bend, squeeze or wring it, as it may negatively affect the shape.

It is not recommended to wash other foam parts.

Handwash

This device and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The device can be disinfected with a 70%

disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the device, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Materials

- Steel
- Stainless steel
- Plastic: ABS, PE, PA6+GF30, TPU, POM, PA6+GF15
- Polyurethan

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder
- coating or ED-coating
- Non-lacquered steel surfaces = Galvanized

Power unit

For an optimum lifetime the power unit must be connected to the mains voltages as often as possible (e.g. every night). Charge the batteries 12 hours before use.

The batteries must be charged at least every 3rd month - otherwise there might be damages due to self-discharge. It is recommended to test the battery function at least once every year.

Recharging the battery:

- Remove the plug from the power system and plug in the charging lead
- Plug the charger into the electricity supply and switch ON. A yellow light on the charger will light up
- Charge the batteries until the lamp on the charger turns green

Technical Data

Battery - BA18

- Capacity 1.3 Ah - 24V DC
- Charging current is max. 500 mA
- IPX6

The batteries have a maximum life of 4 years but this can vary depending upon the conditions of use and charging practices

Battery charger - CH01

- Mains: 100 - 240VAC/50-60 Hz switch mode
- Charging voltage: 27.6 VDC
- Charging current: Max. 500 mA
- FRIWO Gerätebau GmbH, Model FW7218M/24

Actuator - LA28/LA32

- Input voltage: 24V
- Noise level: ≤ 45 dB(A)
- IPX5
- Duty cycle: Max. 10% or 2 min. use then 18 min. without use

Handset - HB 30

- IPX6

Detachable parts

- Handset - HB 30
- Battery - BA 18

Applied part

- Handset - HB 30


Environmental conditions

- Operating temperature: $+5^{\circ}\text{C}$ to $+40^{\circ}\text{C}$
- Storage temperature: -10°C to $+50^{\circ}\text{C}$
- Relative humidity: 20% to 80%
- Atmospheric pressure: 700 to 1060 hPa

Ambient and surface temperature

The table below illustrates how much time it takes for the surfaces of the device to reach operating temperature from -10°C and $+50^{\circ}\text{C}$.

Surface temperature			
-10°C	to	$+5^{\circ}\text{C}$	108 min
$+50^{\circ}\text{C}$	to	$+40^{\circ}\text{C}$	129 min

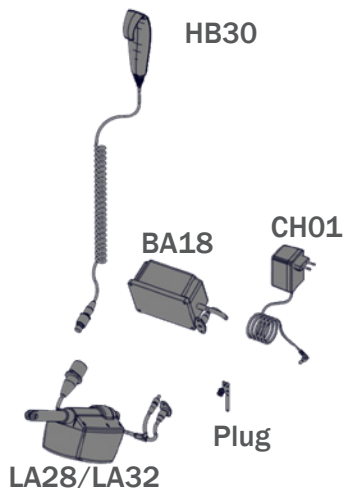IP-Rating table - First Digit: Solids

The first digit indicates the level of protection that the enclosure provides against access to hazardous parts (e.g., electrical conductors, moving parts) and the ingress of solid foreign objects

IP0X	No protection against contact and ingress of objects
IP1X	Any large surface of the body, such as the back of the hand, but no protection against deliberate contact with a body part.
IP2X	Fingers or similar objects.
IP3X	Tools, thick wires, etc.
IP4X	Most wires, screws, etc.
IP5X	Ingress of dust is not entirely prevented, but it must not enter in sufficient quantity to interfere with the satisfactory operation of the equipment; complete protection against contact.
IP6X	No ingress of dust; complete protection against contact.

Second Digit: Liquids

Protection of the equipment inside the enclosure against harmful ingress of water.

IPX0	No protection
IPX1	Protection against vertically falling drops of water e.g. condensation
IPX2	Protection against direct sprays of water up to 15° from the vertical
IPX3	Protected against direct sprays of water up to 60° from the vertical
IPX4	Protection against water sprayed from all directions - limited ingress permitted
IPX5	Protected against low pressure jets of water from all directions - limited ingress
IPX6	Protected against temporary flooding of water, e.g. for use on ship decks - limited ingress permitted
IPX7	Protected against the effect of immersion between 15 cm and 1 m
IPX8	Protects against long periods of immersion under pressure

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**Etac Ltd.**
**Gelbart AG**