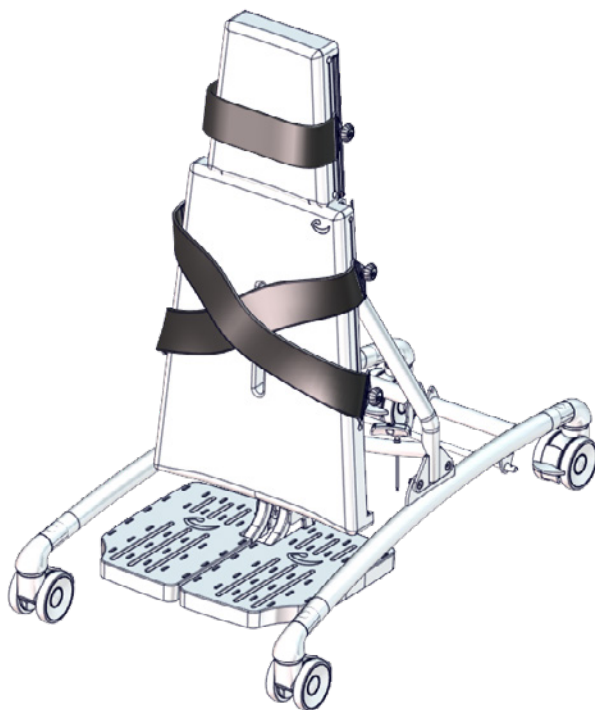
User manual

Caribou

R82®
by Etac

M1125 - 2024-10-15 - Rev. 13


www.etac.com

	3	ENGLISH DEUTSCH	24 - 30 31-38
	4 - 5	NEDERLANDS DANSK	39-46 47-53
	6	NORSK SVENSKA	54-60 61-67
	7-9	SUOMI ESPAÑOL	68-73 74-80
	10	FRANÇAIS ITALIANO	81-88 89-96
	11-12	PORTUGUÊS PORTUGUÊS DO BRASIL	97-103 104-111
	13	РОССИЯ POLSKI	112-119 120-127
	14	ČESKY 中國	128-135 136-143
	15	日本 ΕΛΛΗΝΙΚΗ	144-151 152-159
	16		
mm (inch) kg (lb)	17		
	18-21		
	22-23		

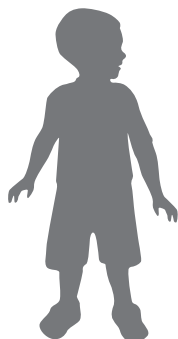
CARIBOU 1


<40 kg
<88.1 lb

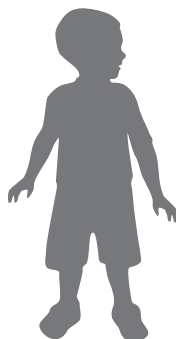CARIBOU 2


<60 kg
<132.2 lb

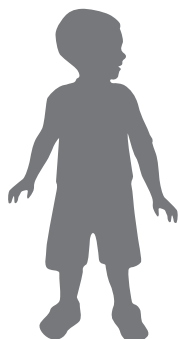CARIBOU 3







<80 kg
<176.3 lb

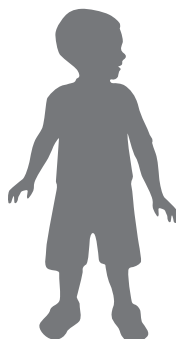CARIBOU 4

<100 kg
<220.4 lb

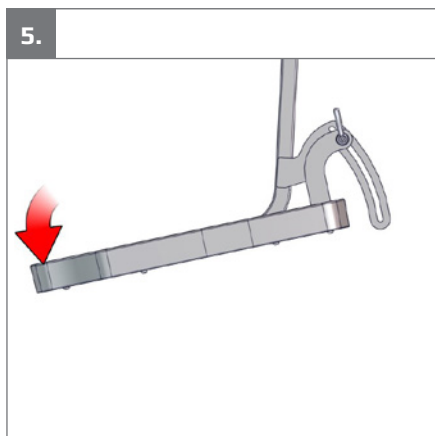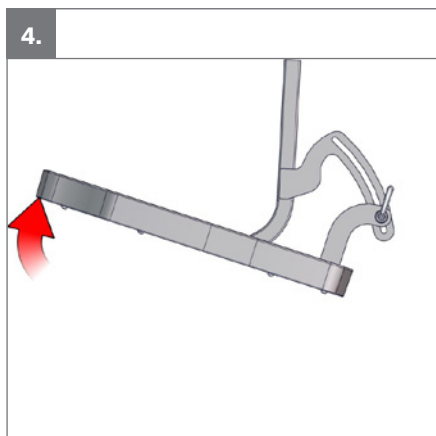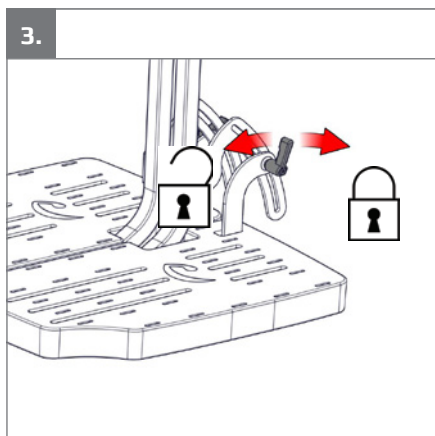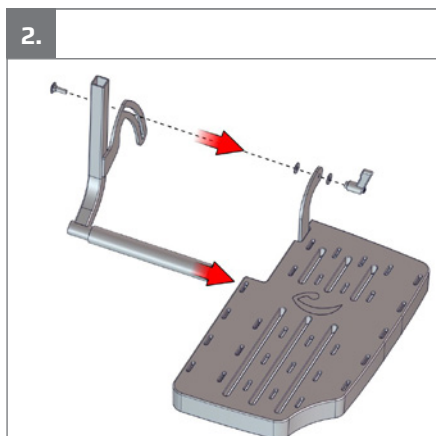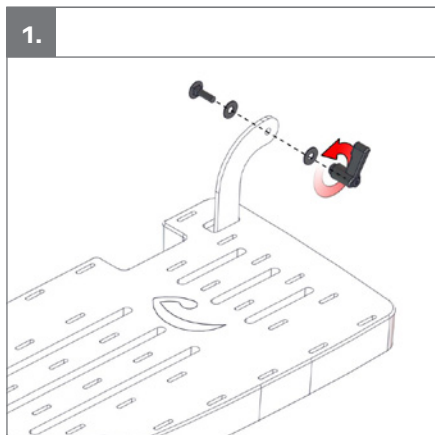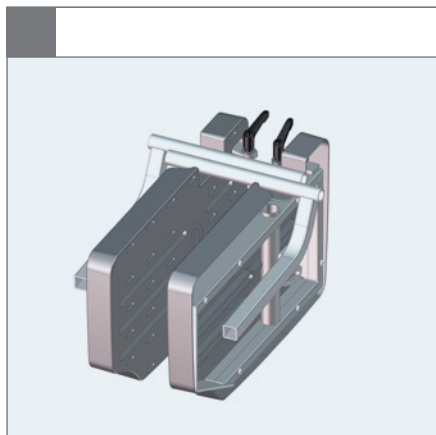1a

1b

2a

○ 5 mm

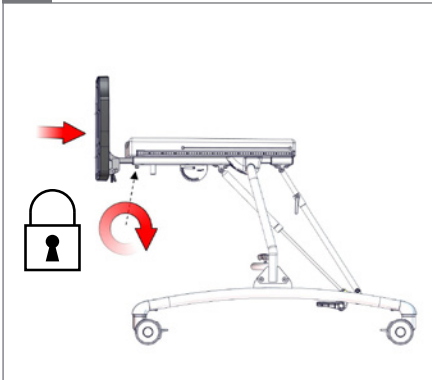
2b

○ 5 mm


3a


○ 5 mm









3b

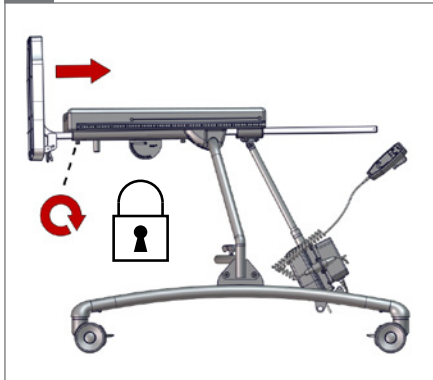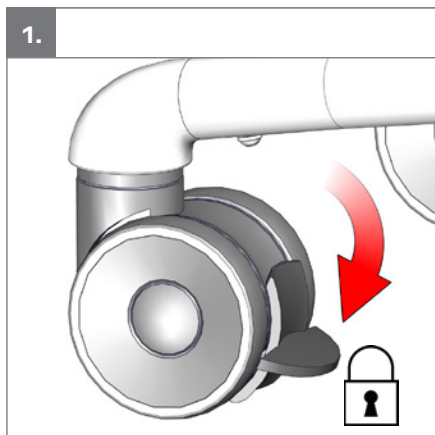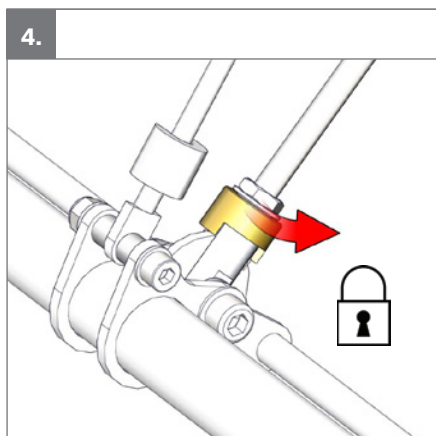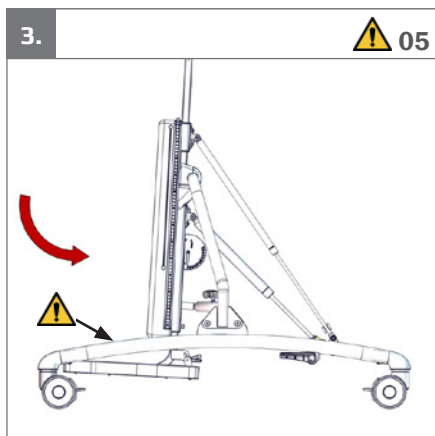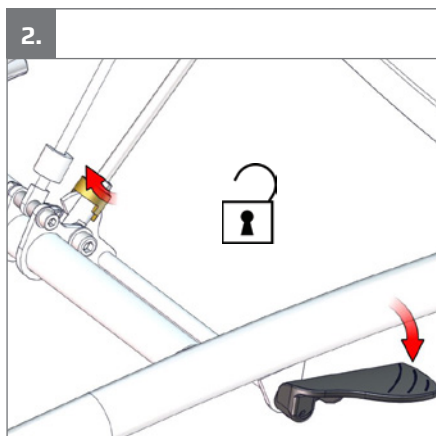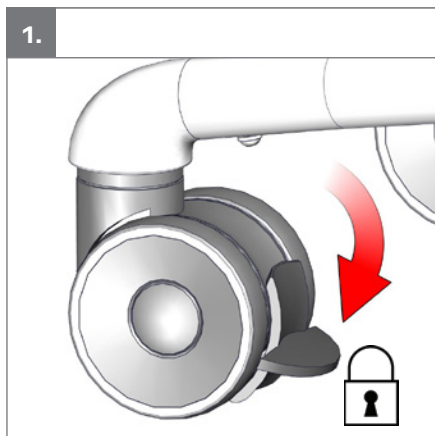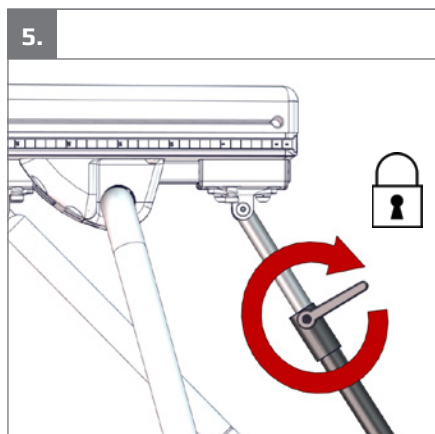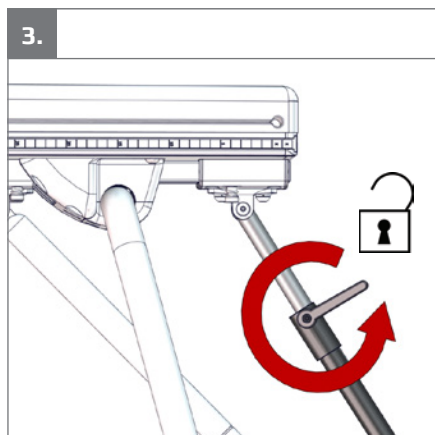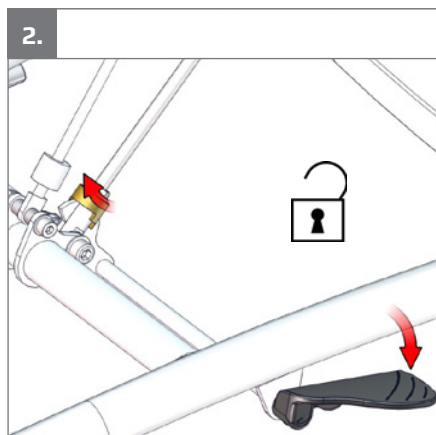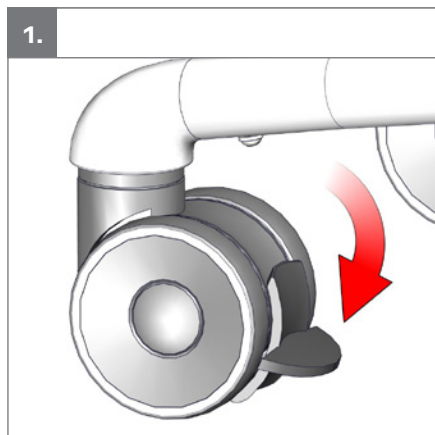○ 5 mm












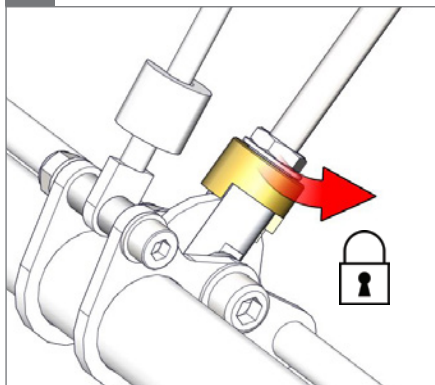
6.

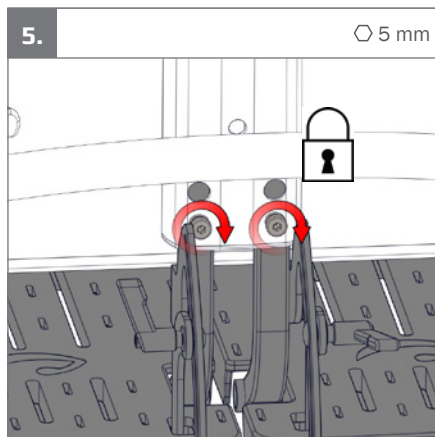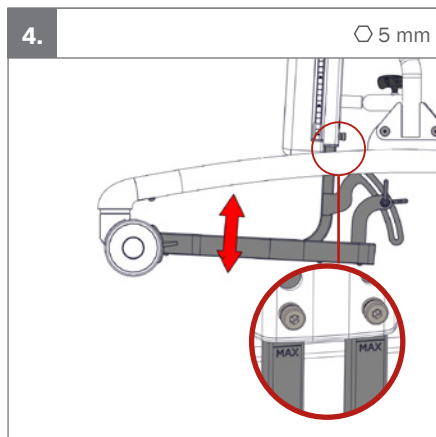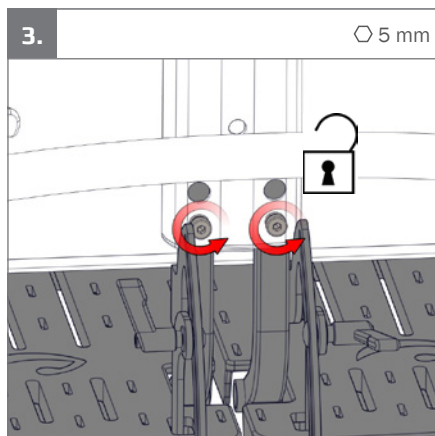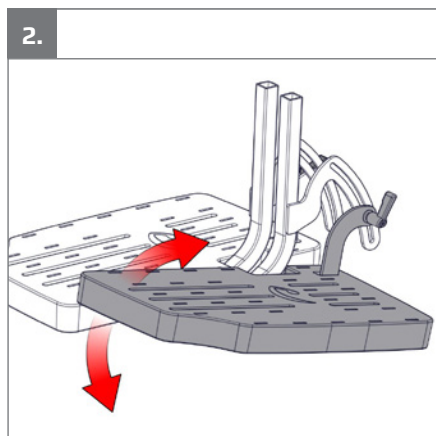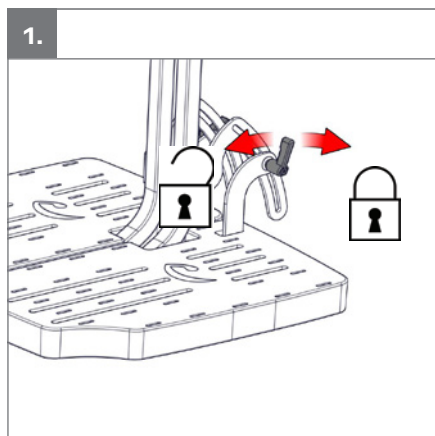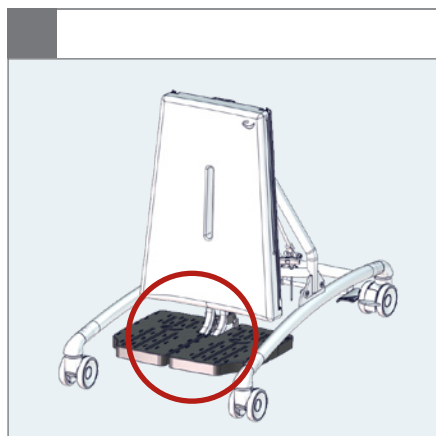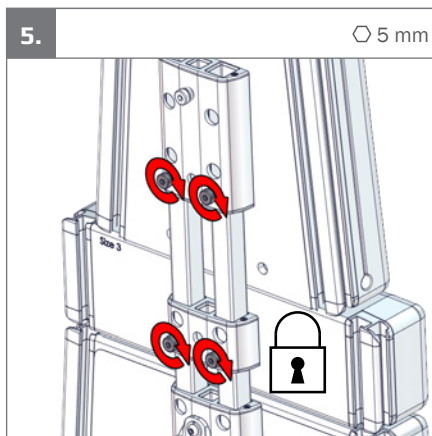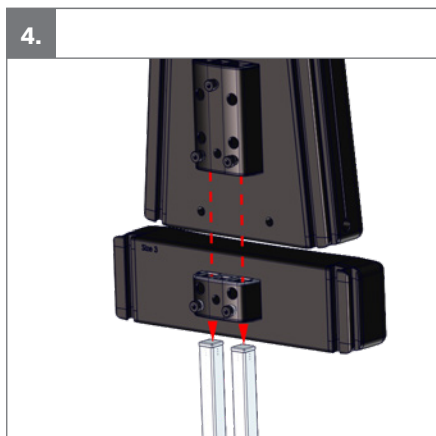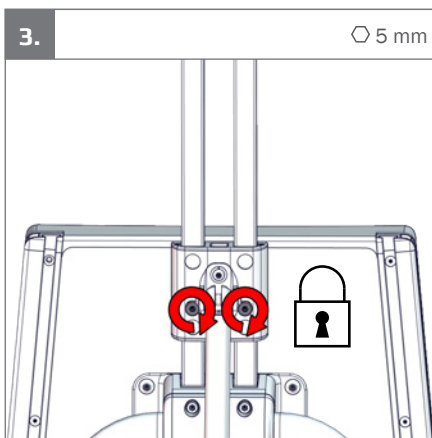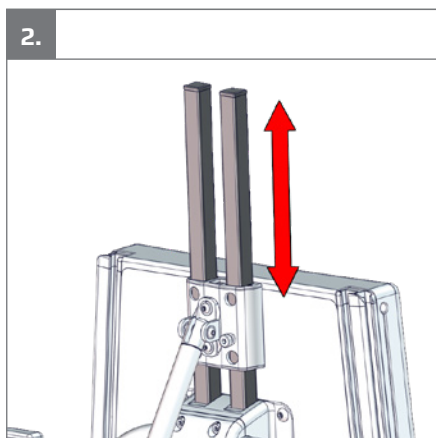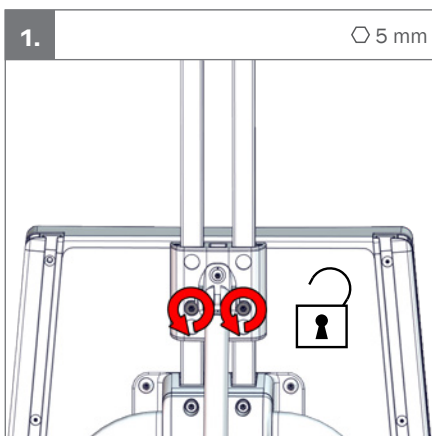7.













6.

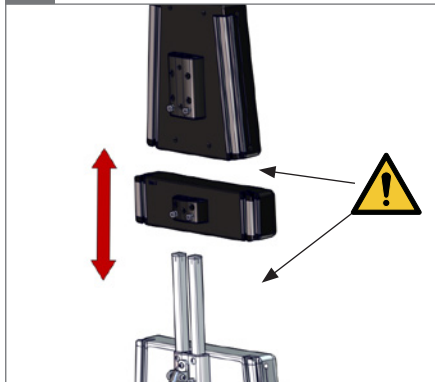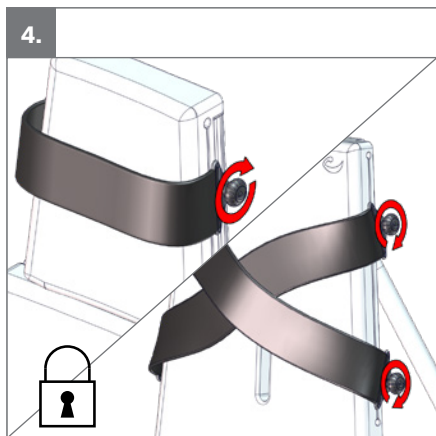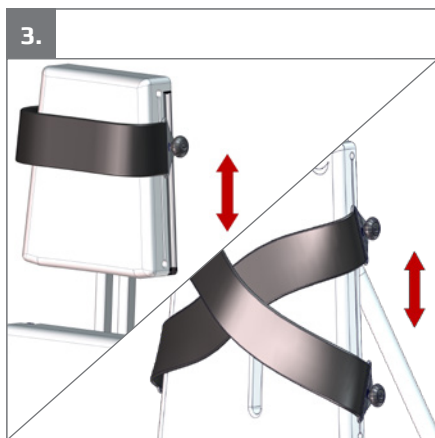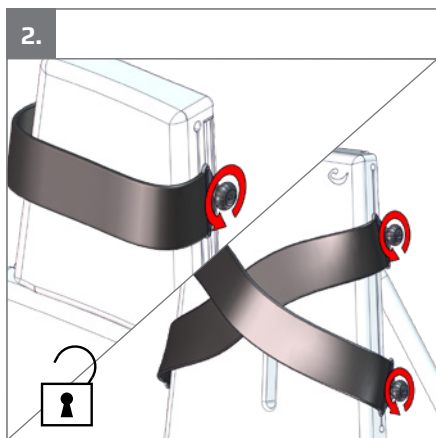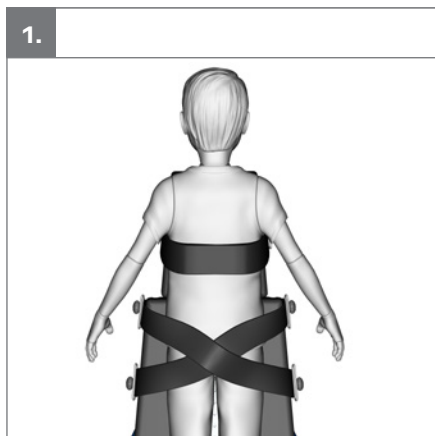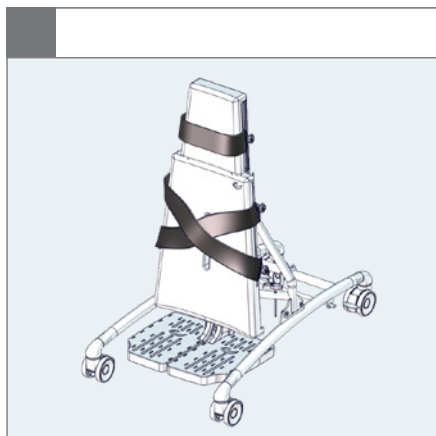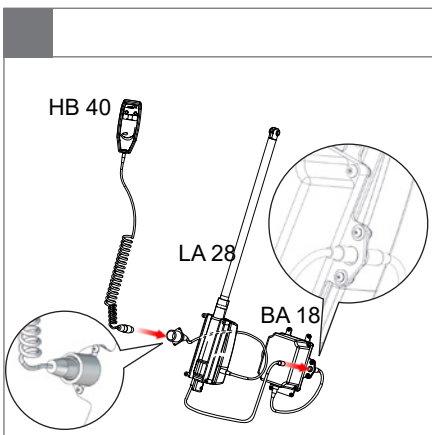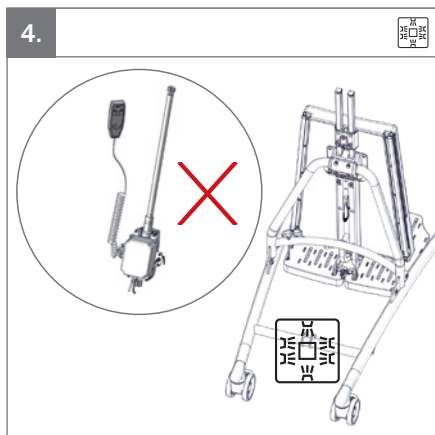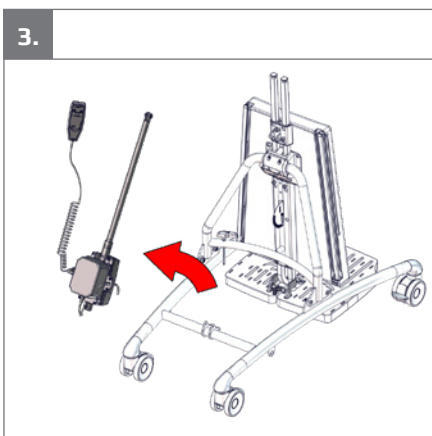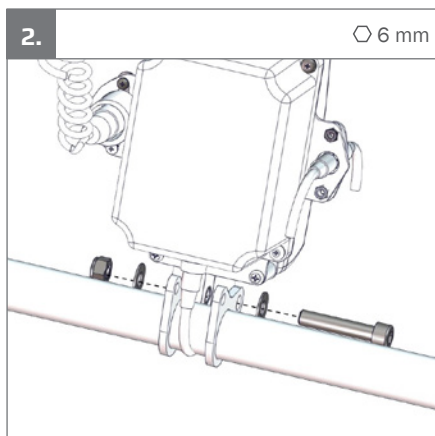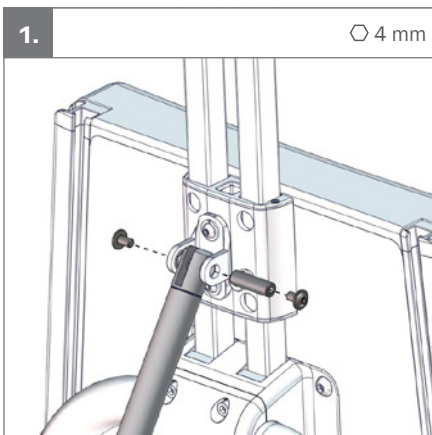
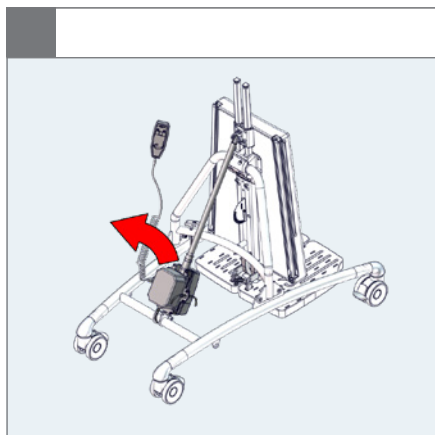
⚠ 05

Caribou w/Power

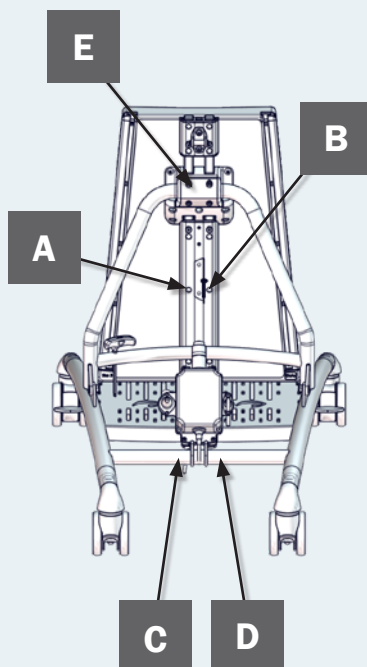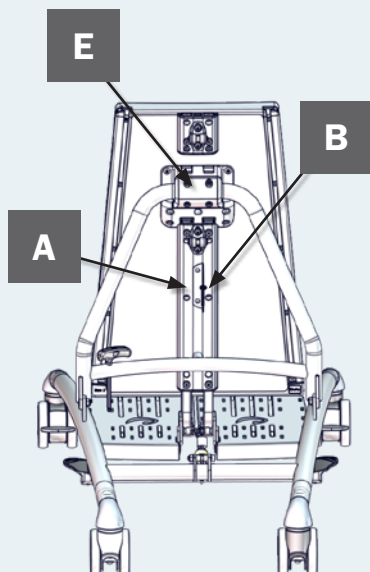A

R82
by Etac

R82 A/S, Parallelvej 3
DK-8751 Gedved
etac.com
Made in xxxxxxxx

MD

CE UKCA

YYYY-MM-DD

B

Posting group
Product name, size x

DMR ver.: xxxxx

SE xxxxxxxxxxxx

REF xxxxxxxxxxxxxxxxxxxx

Max. user mass: xxx kg
Max. rated load: xxx kg
Mass device: xxx kg

UEN (xx)xxxxxxxxxxxxxxxx(xx)xxxxxxxxxx

C

Storage and Transport	50°C	80%	1060 hPa
	-10°C	20%	700 hPa
Operating	40°C	80%	1060 hPa
	5°C	20%	700 hPa




DUTY CYCLE: 10% MAX 2 min./18 min.

D

IP 21

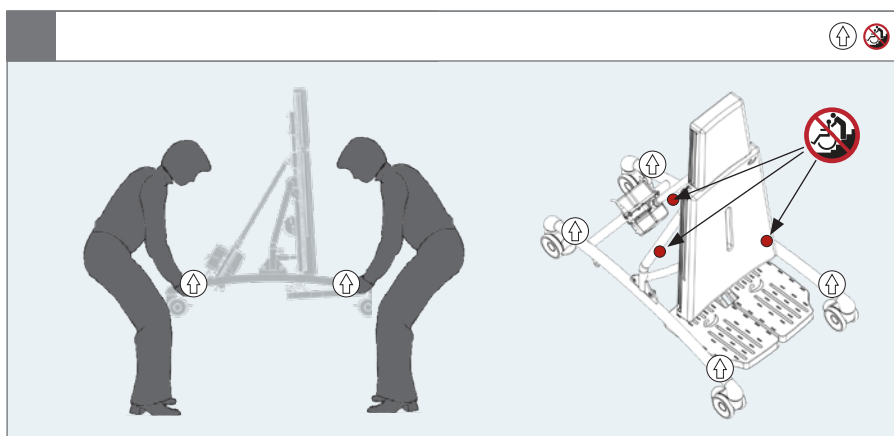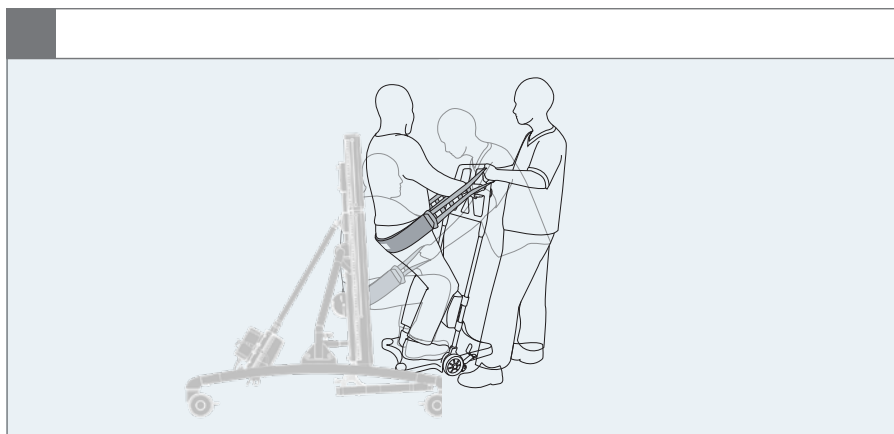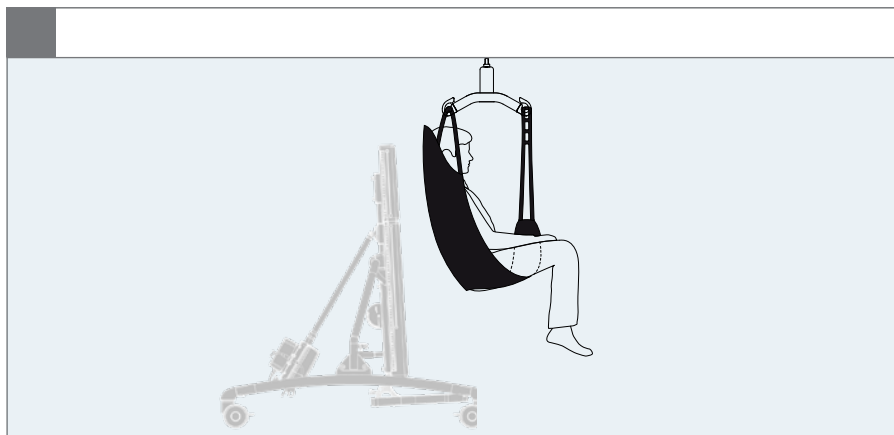E

mm (inch) kg (lb)				
	 1	 2	 3	 4
 Overall width	510 mm (20")	570 mm (22½")	650 mm (25½")	750 mm (29½")
 Length	750 mm (29½")	920 mm (36¼")	1110 mm (43¾")	1110 mm (43¾")
 Overall height, vertical	460–570 mm (18–22½")	530–640 mm (20¾–25¼")	680–790 mm (26¾–31")	880–990 mm (34¾–39")
 Overall height, horizontal	700 mm (27½")	700 mm (27½")	800 mm (31½")	800 mm (31½")
 Angling	0°–87°	0°–87°	0°–87°	0°–87°
 Foot plate angle	(-20°)–(20°)	(-20°)–(20°)	(-20°)–(20°)	(-20°)–(20°)
 Total mass	20.5 kg (45.1 lb)	24 kg (52.9 lb)	30.5 kg (67.2 lb)	35 kg (77.1 lb)
 Maximum rated load	62,5 kg (137.7 lb)	86 kg (189.6 lb)	112,5 kg (248 lb)	137 kg (302 lb)
 Total mass, with power	22.5 kg (49.6 lb)	25.5 kg (56.2 lb)	32.5 kg (71.7 lb)	37 kg (81.6 lb)
 Maximum rated load, with power	64.5 kg (142.1 lb)	87.5 kg (192.9 lb)	114.5 kg (252.4 lb)	139 kg (306.4 lb)
 Maximum user mass	40 kg (88.1 lb)	60 kg (132.2 lb)	80 kg (176.3 lb)	100 kg (220.4 lb)
 User height	850–1120 mm (33½–44")	980–1290 mm (38½–50¾")	1250–1560 mm (49¼–61½")	1530–1840 mm (60¾–72½")
 Static stability downhill	max. 5°	max. 5°	max. 5°	max. 5°
 Static stability uphill	max. 5°	max. 5°	max. 5°	max. 5°
 Sideways stability sideways	max. 5°	max. 5°	max. 5°	max. 5°

EN: Warning: This symbol appears in the User Guide along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger
01 = Entrapment Hazards
02 = Tilting Hazards
03 = Transportation in motor vehicles
05 = Squeezing Hazard

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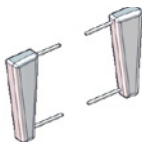
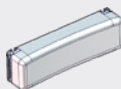
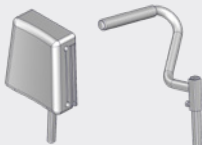
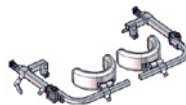
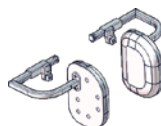
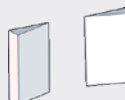
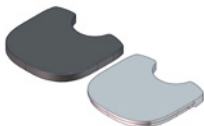
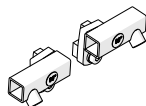
ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The device is a standing frame for children and adults with disabilities.

Indications

The device can be used by children and adults with disabilities, who experience limitations in their ability to maintain a standing position.

The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age.

The user is the person standing in the frame. The caregiver is the person operating the frame. Never leave the user unattended in this product. Ensure permanent supervision by a caregiver. The use of the device will be discontinued more times during the day where the user will be lying down or sitting in another assistive device or a conventional product.

Contra indications

There are no known contra indications.

Precautions

Special attention should be given to the considerations about the configuration of the product and the use of accessories for users with limitation in their:

- Sensory functions and the sensation of pain, consider extra checks of the pressure caused by the support provided by the product and its accessories.
- Movement function (e.g. experience involuntary movements).
- Ability to maintain a standing position, use the product with accessories (e.g. chest or back support).
- Ability to maintain the position of their head, consider to use accessories or to use a product that provide support to the positioning of the head.

Application areas

The product is for indoor use

Enclosed tools:

- 5 mm Allen key

CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12182

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

The symbol of the crossed-out trash can on batteries or accumulators states that they must not be disposed of in household waste at the end of their service life. If batteries or accumulators contain mercury (Hg), cadmium (Cd) or lead (Pb), you will find the respective chemical symbol below the symbol of the crossed-out trash can. You are legally obliged to return old batteries and accumulators after use. You can do this free of charge at the retail store or at another collection point in your area. Addresses of suitable collection points can be obtained from your city or local government.

Batteries can contain substances that are harmful to the environment and human health. Special care should be taken due to the special risks involved in handling lithium-containing batteries. The separate collection and recycling of old batteries and accumulators is intended to avoid negative effects on the environment and human health. Please avoid the generation of waste from

old batteries as much as possible, i.e., by preferring batteries with a longer service life or rechargeable batteries. Please avoid littering of public spaces by not carelessly leaving batteries or battery-containing electrical and electronic equipment behind. Please consider ways to reuse a battery instead of disposal, for example by reconditioning or repairing the battery.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions. Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user manual.
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Transportation of the product whilst the user is in it is not allowed. It is allowed to use the product with the user safely strapped in it for transfers over short distances. Do not cross any obstacles during transfer.
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Ensure that body parts are not trapped in openings for mounting of supports/ accessories.
- Do not let children and youngsters play with the handset
- It is important to remove the transport plastic bag before using the handset. Cables need to be placed in such a way that users or carers cannot trip and injure themselves.

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without

any obstacles. Be aware of reduced maneuverability on slippery surfaces

- When using on inclines, please refer to measurements

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered
- When changing user, clean the product and disinfect it if necessary. Carry out necessary adjustments after cleaning.

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product
- Ensure the stability of the product is maintained before placing the user in it
- Ensure the castors are fully-functional and secure before use
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- The power unit does not have a safety stop
- Do only use the power unit within specified working limits
- Do not attempt to repair any part of the power unit.
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1 and EN 1021-2
- The product might lose its flame resistant characteristics if using other than original R82 cushioning
- This product is not to be occupied in a motor vehicle. The user must be positioned in a vehicle seat and the product must be stored in a luggage compartment
- The user must not occupy the product while it is being transported into or out of a

vehicle by a lift

- Unoccupied device is suitable for land and air transport
- Details covering EMC can be found in the appendix "EMC details" supplied with the product

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Reuse

This product and its accessories/components are suitable for reuse.

Refurbishment for reuse

Before reuse or changing the user, follow the service checklist, maintenance and washing instructions and carry out regular inspections.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

Service checklist:

Check and adjust the following as per their intended use:

- For products with gasspring and/or telescopic tube, check that the locking mechanisms work. Also check that it is possible to tilt the basis plate horizontally and vertically.
- For products with power, recharge the battery and check that the handset works by tilting the basis plate to vertical and horizontal position
- Check that all castors are moving freely and that all wheel locks/brakes are easily activated
- The foot plate(s) is adjustable, both distance to basisplate and the angle mechanism must be checked
- All cushions are in good shape without cracks and or holes in the cover
- Nuts and bolts on the product are tightened
- The straps must be without cracks and not flossed, pay special attention to the area where the straps connect to the brackets.
- The plastic pieces connecting the frame to the basis plate is undamaged and without cracks. Pay special attention to the areas where fasteners are inserted

- The rails in the basis plate must be in good shape, it should be possible to slide a strap up and down the rail.
- All labels on the product are intact
- Check that all cables connecting the power system are in good shape.

Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to:

Wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

Before washing

Avoid degreasing of the piston rods, the actuators should be retracted to minimum stroke before washing.

- Remove all accessories, unplug all cables and make sure to dismount the battery, control-box and handset. Leave the actuators with the cables mounted on the product
- Do NOT clean the power unit with a high pressure cleaner or steam cleaner.

After washing

- Ensure correct placement of cables inserted into the control box when reinstalling the power unit system
- Check that the handset functionalities works correct

Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild

detergent/soap without chlorine and let it dry before disinfection.

Materials

- Stainless steel
- Aluminium
- Plastic
- Rubber
- Polyurethan

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

Power unit

For an optimum lifetime the power unit must be connected to the mains voltages as often as possible (e.g. every night). Charge the batteries 12 hours before use.

The batteries must be charged at least every 3rd month - otherwise there might be damages due to self-discharge. It is recommended to test the battery function at least once every year.

Recharging the battery:

- Plug the charging lead into the handset
- Plug the charger into the electricity supply and switch ON. A yellow light on the charger will light up
- Charge the batteries until the lamp on the charger turns green

Technical Data

Battery - BA18

- Capacity 1.3 Ah - 24V DC
- Charging current is max. 500 mA
- IPX6

The batteries have a maximum life of 4 years but this can vary depending upon the conditions of use and charging practices

Battery charger - CH01

- Mains: 100 - 240VAC/50-60 Hz switch mode
- Charging voltage: 27.6 VDC
- Charging current: Max. 500 mA
- FRIWO Gerätebau GmbH, Model FW7218M/24

Actuator - LA28

- Input voltage: 24V
- Noise level: ≤ 45 dB(A)
- IPX5
- Duty cycle: Max. 10% or 2 min. use then 18 min. without use

Handset - HB 40

- IPX1

Detachable parts

- Handset - HB 40
- Battery - BA 18

Applied part

- Handset - HB 40


Environmental conditions

- Operating temperature: $+5^{\circ}\text{C}$ to $+40^{\circ}\text{C}$
- Storage temperature: -10°C to $+50^{\circ}\text{C}$
- Relative humidity: 20% to 80%
- Atmospheric pressure: 700 to 1060 hPa

IP-Rating table - First Digit: Solids

The first digit indicates the level of protection that the enclosure provides against access to hazardous parts (e.g., electrical conductors, moving parts) and the ingress of solid foreign objects

IPOX	No protection against contact and ingress of objects
IP1X	Any large surface of the body, such as the back of the hand, but no protection against deliberate contact with a body part.
IP2X	Fingers or similar objects.
IPX3	Tools, thick wires, etc.
IPX4	Most wires, screws, etc.
IPX5	Ingress of dust is not entirely prevented, but it must not enter in sufficient quantity to interfere with the satisfactory operation of the equipment; complete protection against contact.
IP6X	No ingress of dust; complete protection against contact.

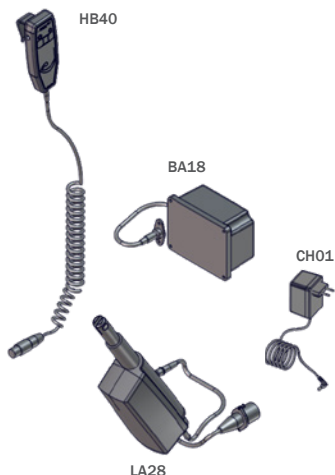Ambient and surface temperature

The table below illustrates how long time it takes for the surfaces of the device to reach operating temperature from -10°C or $+50^{\circ}\text{C}$.

Surface temperature			
-10°C	to	$+5^{\circ}\text{C}$	108 min
$+50^{\circ}\text{C}$	to	$+40^{\circ}\text{C}$	129 min

Second Digit: Liquids

Protection of the equipment inside the enclosure against harmful ingress of water.

IPX0	No protection
IPX1	Protection against vertically falling drops of water e.g. condensation
IPX2	Protection against direct sprays of water up to 15° from the vertical
IPX3	Protected against direct sprays of water up to 60° from the vertical
IPX4	Protection against water sprayed from all directions - limited ingress permitted
IPX5	Protected against low pressure jets of water from all directions - limited ingress
IPX6	Protected against temporary flooding of water, e.g. for use on ship decks - limited ingress permitted
IPX7	Protected against the effect of immersion between 15 cm and 1 m
IPX8	Protects against long periods of immersion under pressure

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**Etac Ltd.**
**Gelbart AG**