0 - 30°

<70 kg (154.3 lb)

Rabbit User Guide

EN 12182

2020-02-rev.8


	3	ENGLISH DEUTSCH	17 - 19 20 - 22
	4	NEDERLANDS DANSK	23 - 25 26 - 28
	5	NORSK SVENSKA	29 - 31 32 - 34
	6	SUOMI ESPAÑOL	35 - 37 38 - 40
	7 - 8	FRANÇAIS ITALIANO	41 - 43 44 - 46
	9	PORTUGUÊS PORTUGUÊS DO BRASIL	47 - 49 50 - 52
	10	РОССИЯ POLSKI	53 - 55 56 - 58
	11	ČESKY 中國	59 - 61 62 - 64
	11	日本 ΕΛΛΗΝΙΚΗ	65 - 67 68 - 70
	12 - 14		
mm (inch)	15		
	16		

1.

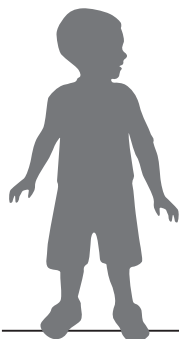
RABBIT 1


<40 kg
<88.1 lb

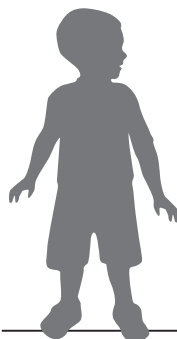RABBIT 2


<50 kg
<110.2 lb

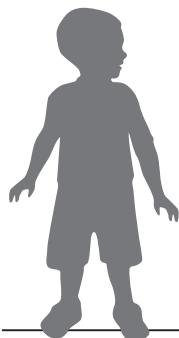RABBIT 3

<60 kg
<132.2 lb

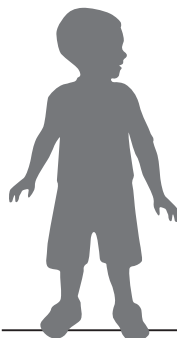
RABBIT 4

<70 kg
<154.3 lb

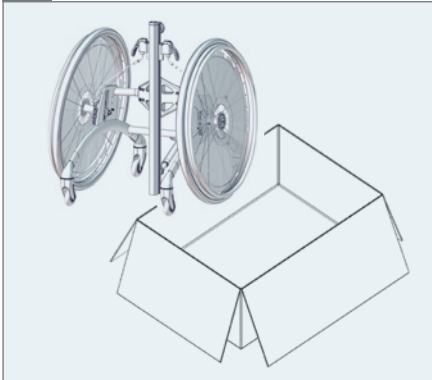
1.


2.

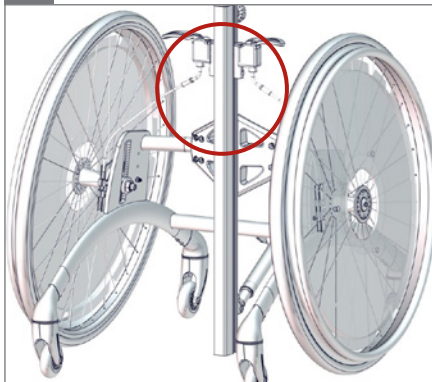
3.

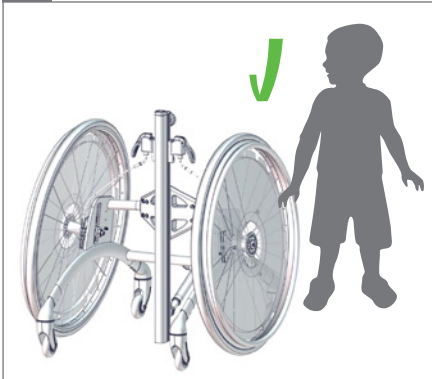4.

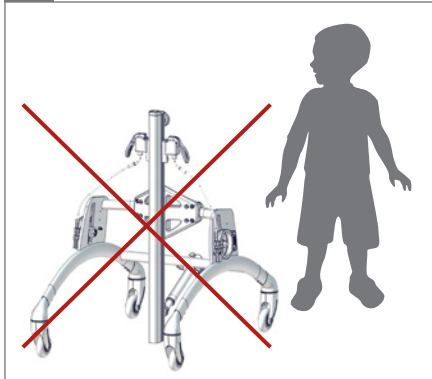
5.


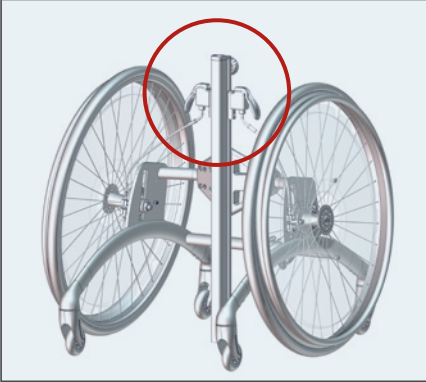
1.

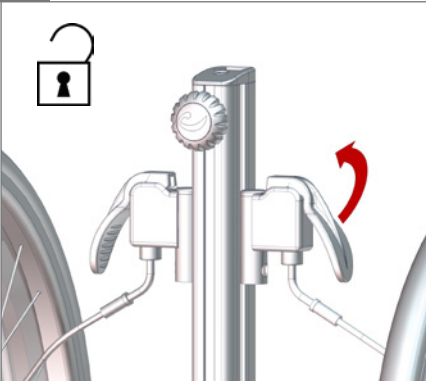
2.

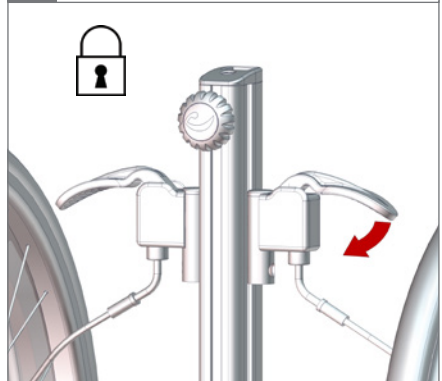
3.

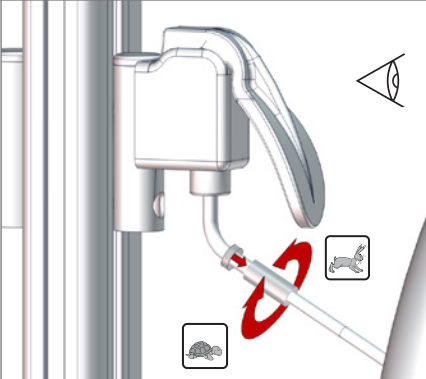4.

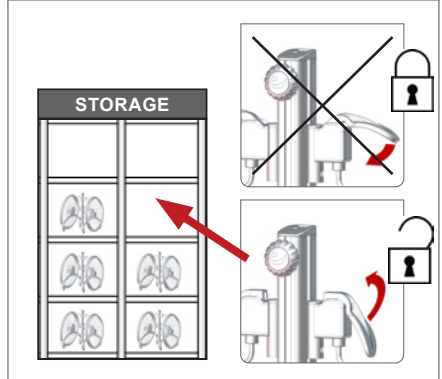
5.

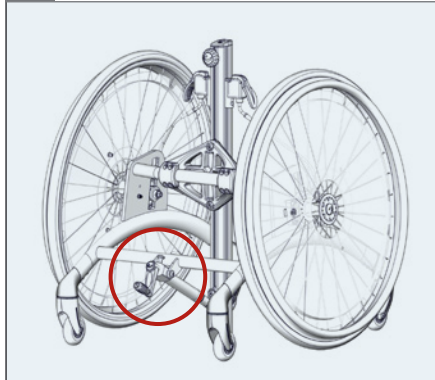
1.

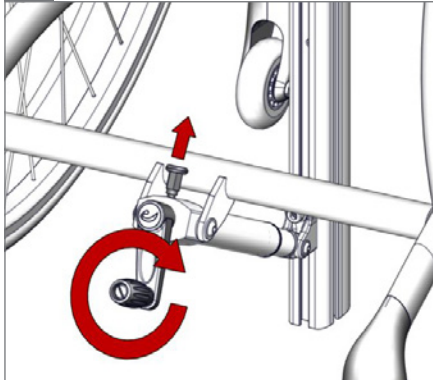
2.


3.

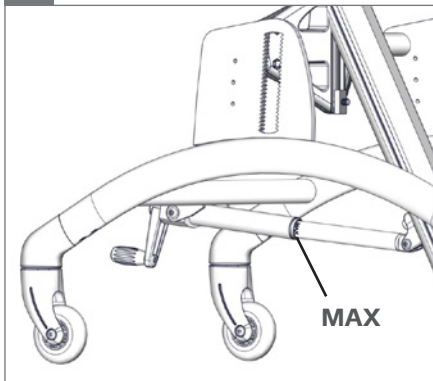
4.

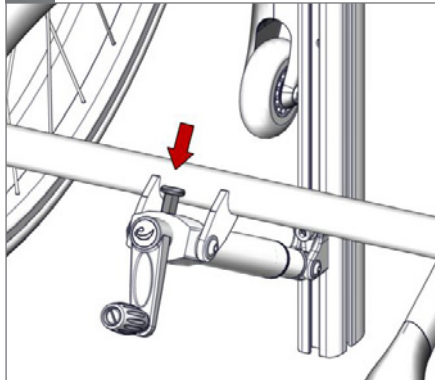5.

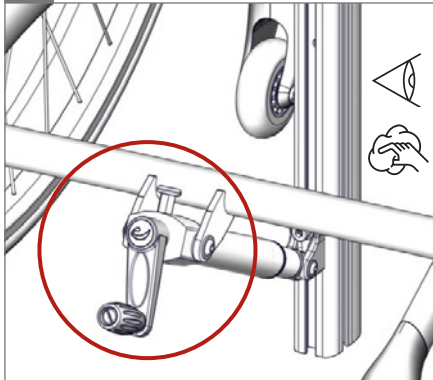
6.

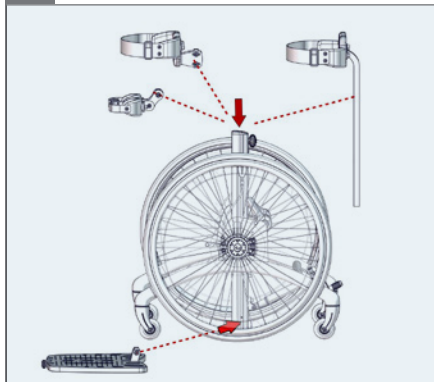
1.


2.

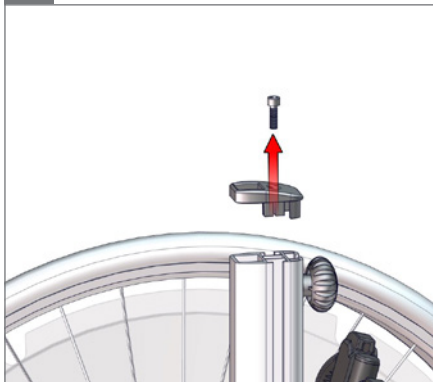
⬡ 5 mm

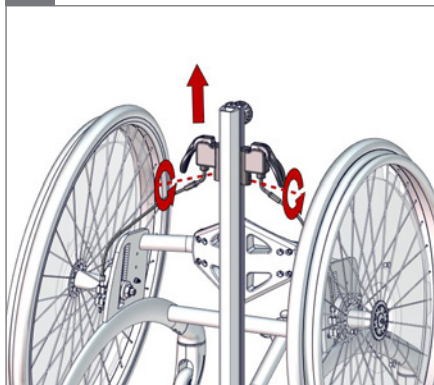3.

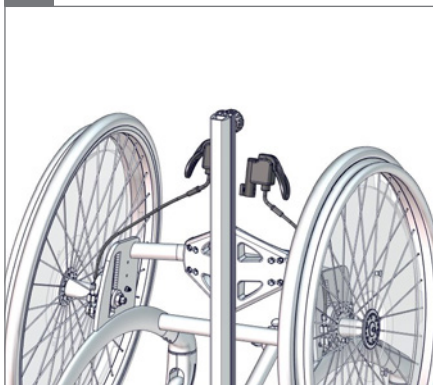
4.








5.

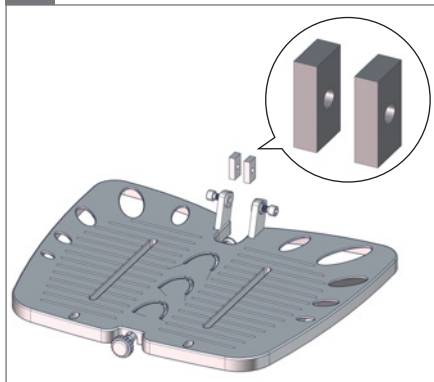⬡ 5 mm

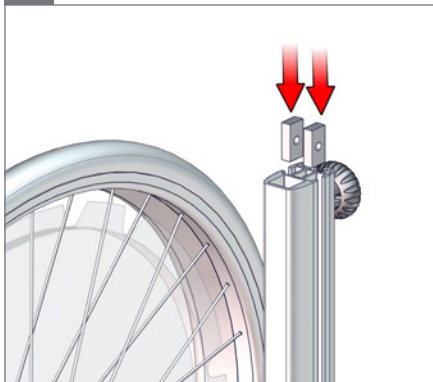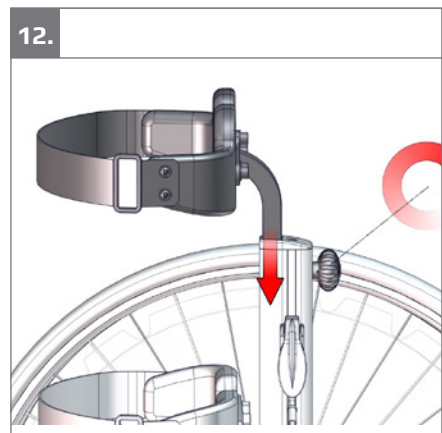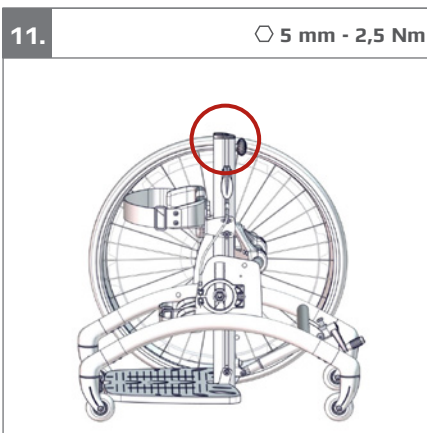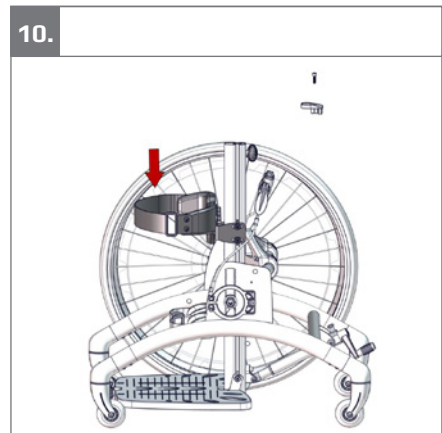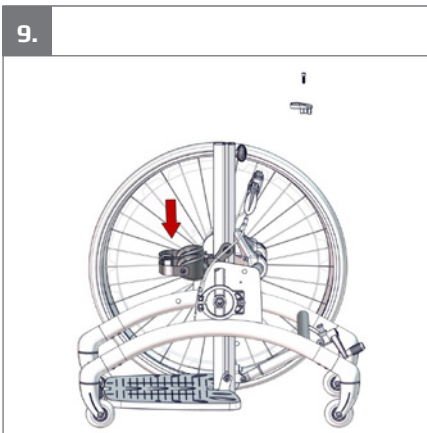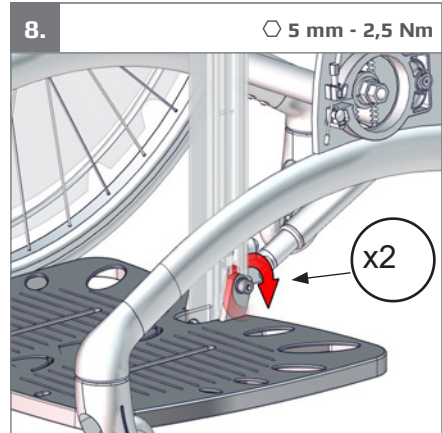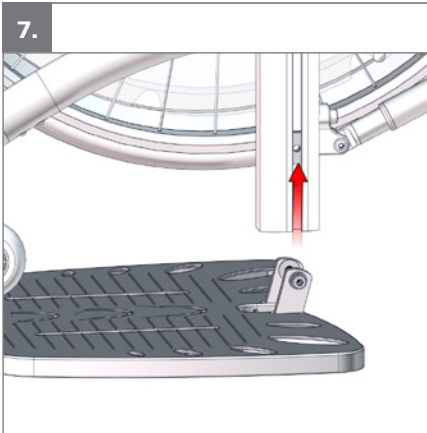
6.


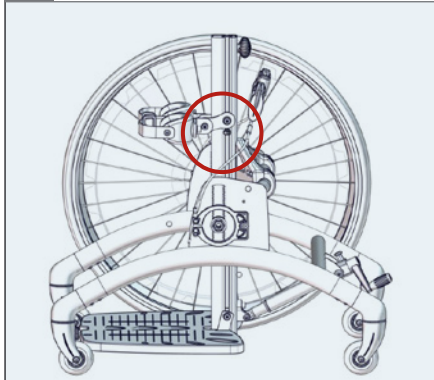
1.

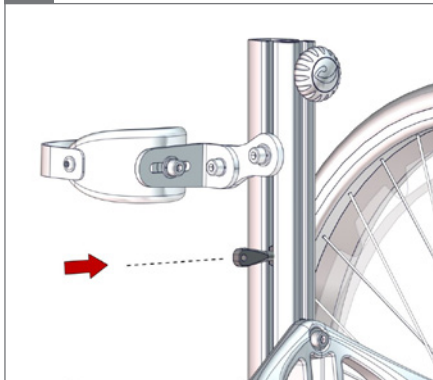
2.

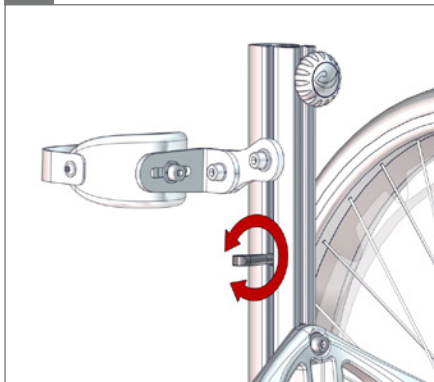
3.

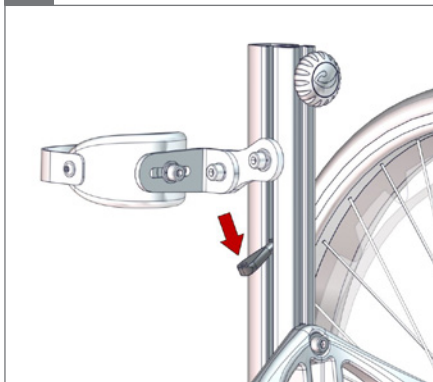
4.

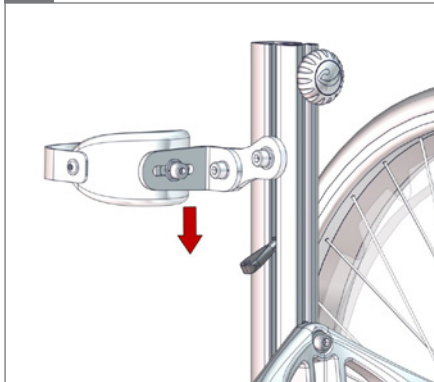5.

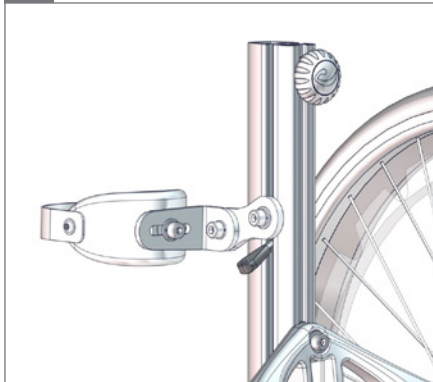
6.

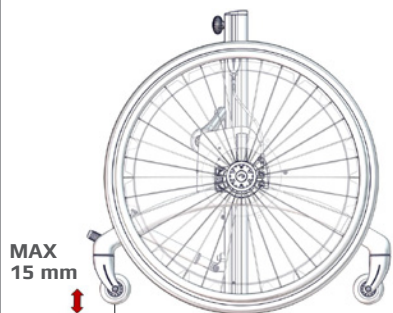
1.

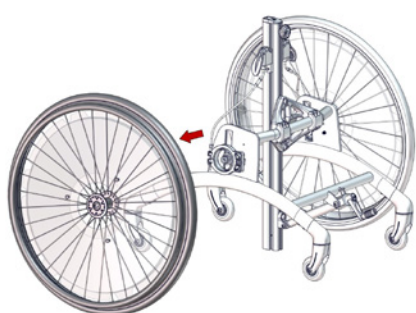
2.


3.

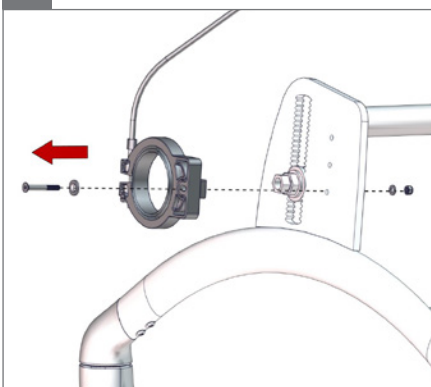3 mm 8 mm


4.

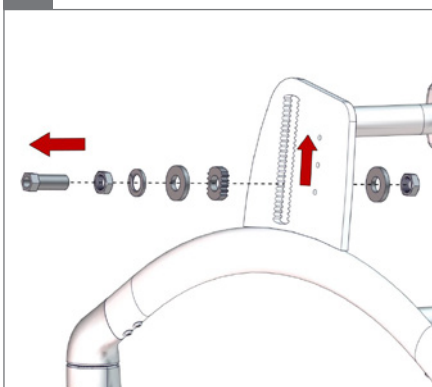17+24 mm


5.

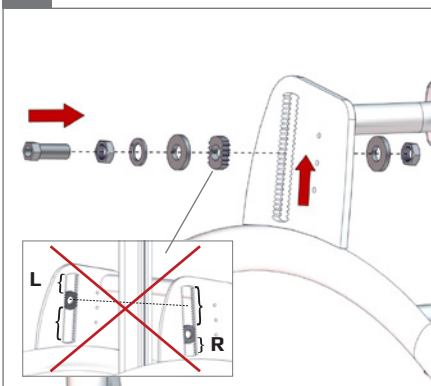
17+24 mm, 30 Nm

6.

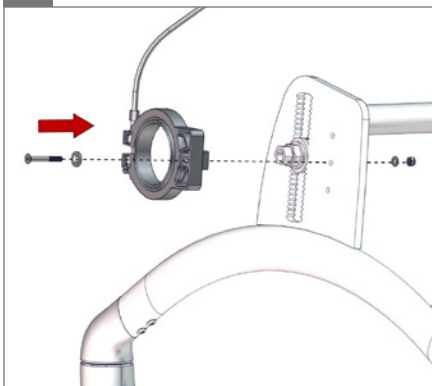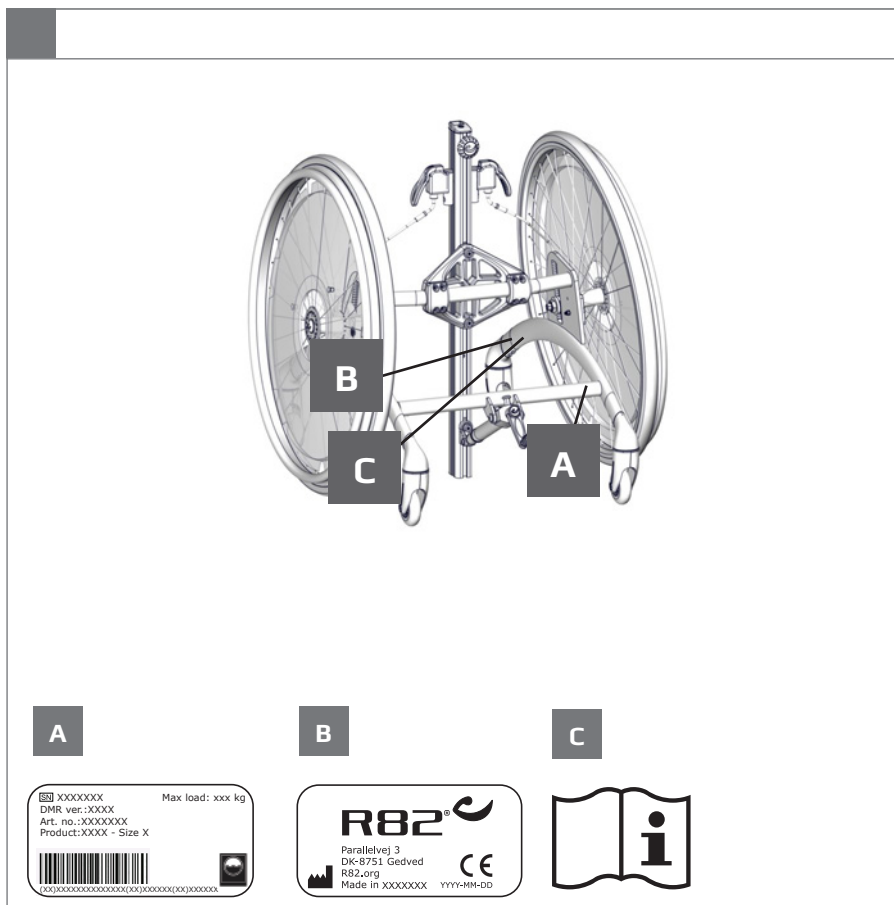3 mm 8 mm, 2,5 Nm

**EN:** For indoor use**DE:** Für den Innengebrauch**NL:** Voor binnenhuis gebruik**DK:** Til indendørs brug**NO:** For innendørs bruk**SV:** För inomhusbruk**FIN:** Sisäkäyttöön**ES:** Para uso en interiores**FR:** Utilisation intérieure**IT:** Per uso interno**PT:** Para utilização em interiores**BR:** Para uso interno**RU:** Для использования в помещении**PL:** Do użytku wewnątrz**CZ:** Pro vnitřní používání**SN:** 适用于室内使用**JP:** 屋内使用**GR:** Κατάλληλο για εσωτερική χρήση

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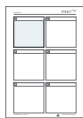
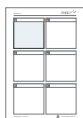
9996097810

ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The Rabbit is a mobile standing frame providing users the possibility of moving around and orientate oneself. The standing position enables the child to be an active participant, whilst stimulating senses and giving the child more independence. The Rabbit is suitable for many users e.g. children with CP, GMFCS level 3 - 4. The Rabbit comes in 4 sizes.

Enclosed tools:

- 5 mm Allen key
- 6 mm Allen key

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

- EN 12182

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the R82 homepage/download.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

Presales information

- support.r82.org

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user guide and who has been given the necessary instructions by a product advisor.
- Users who have difficulties reading the user guide can have someone read the user guide aloud to understand how to use the product.
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Ensure that moving parts does not squeeze or trap body parts when adjusting them

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product
- Ensure the stability of the product is maintained before placing the user in it
- Ensure the wheels are fully-functional and secure before use
- When changing wheels or camber parts always refer to the mounting guide 9996097649 for correct placement and configuration. The mounting guide is always available at r82.org/support.
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1 and EN 1021-2
- The product might lose its flame resistant characteristics if using other than original upholstery and/or accessories
- This product is not to be occupied in a motor vehicle. The user must be positioned in a vehicle seat and the product must be stored in a luggage compartment
- The user must not occupy the product while it is being transported into or out of a vehicle by a lift

Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference
- Store the enclosed tools on/at the product

Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts
- Be aware of any MAX mark indications before adjustment

Weekly

- Check that all castors are moving freely and that all wheel locks/brakes are easily activated
- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use
- Check that the angle mechanism functions correctly and that the safety lock is working correctly
- Disinfect the product. please check the disinfection section in this User Guide for more information

Monthly

- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

Warewashing machine

This product can be washed at:

- 60 °C in 10 minutes
- 85 °C in 3 minutes

...using a mild detergent in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.

Before washing

Remove the following items and clean them separately:

- Accessories
- Cushioning
- Spoke protectors

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

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- 5 mm Inbusschlüssel
- 6 mm Inbusschlüssel

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© 2017 R82 A/S All rights reserved The R82 logo and the icons are registered trademarks of R82 A/S
R82 Inc.
R82 au Pty Limited