User manual

High-low:xo
with power height adjustment

R82[®] 
by Etac

M1190 - 2023-12-13 - Rev. 4.0

www.etac.com

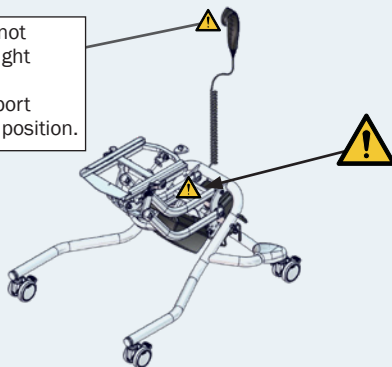
	3-4	ENGLISH DEUTSCH	52-60 61-69
	5	NEDERLANDS DANSK	70-78 79-86
	6-7	NORSK SVENSKA	87-94 95-103
	8	SUOMI ESPAÑOL	104-112 113-121
	9-11	FRANÇAIS ITALIANO	122-130 131-139
	12-15	PORTUGUÊS PORTUGUÊS DO BRASIL	140-148 149-157
	16-17	РОССИЯ POLSKI	158-166 167-175
	18-22	ČESKY 中國	176-184 185-193
	23	日本 ΕΛΛΗΝΙΚΗ	194-202 203-211
	24-25	ÍSLENSKA	212 - 219
	26		
	27-29		
	30		
	31-32		
	33-43		
mm (inch) kg (lb)	44-49		
	50-51		

 05


The frame with power does not have a safety stop at the height adjustment function. Please remove the foot support before adjusting into lowest position.

Size 1

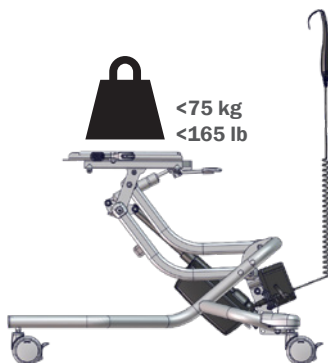
Size 2


Size 3


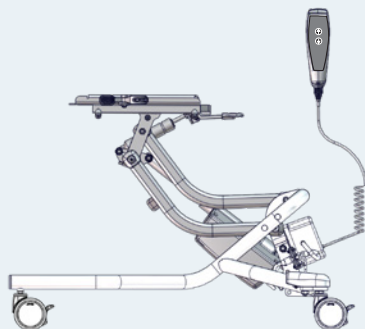
1.

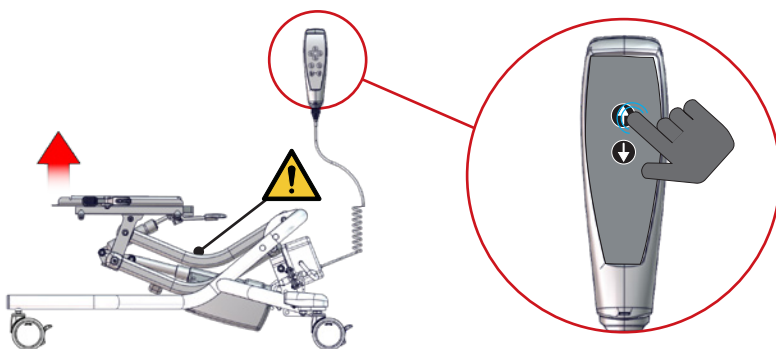 05

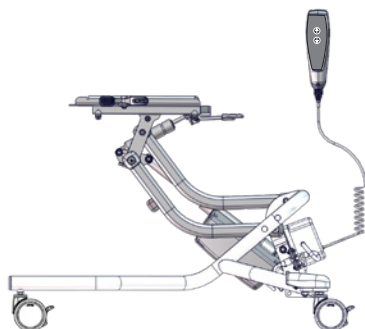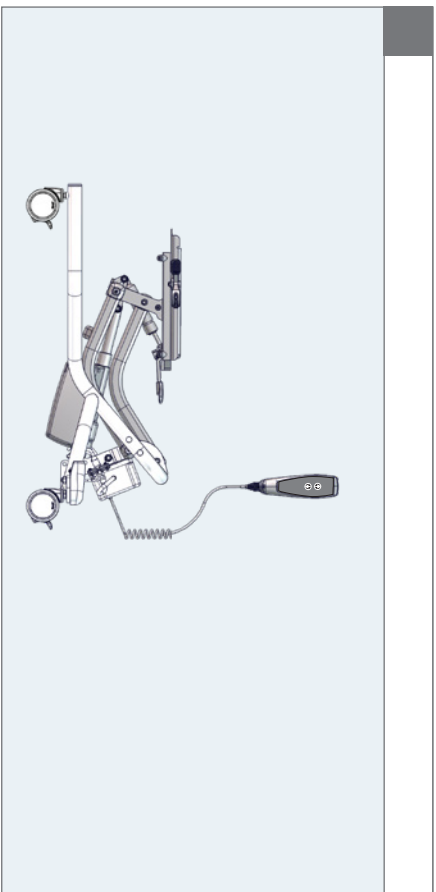
2.

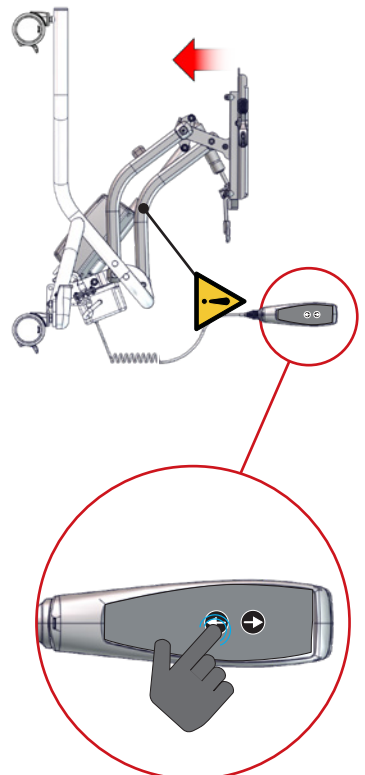
1.

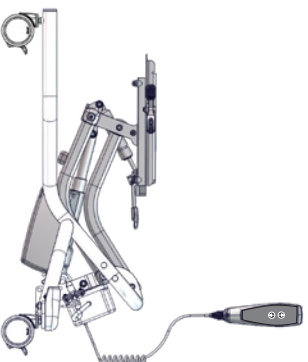2.



 05

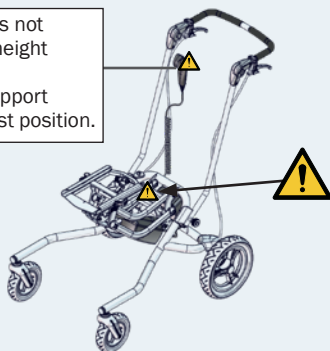The frame with power does not have a safety stop at the height adjustment function. Please remove the foot support before adjusting into lowest position.

Size 1

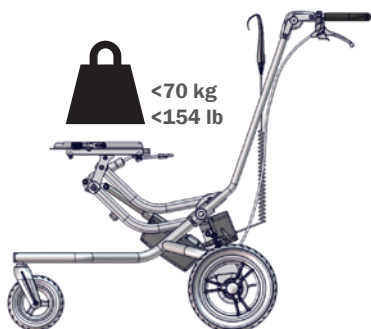
Size 2

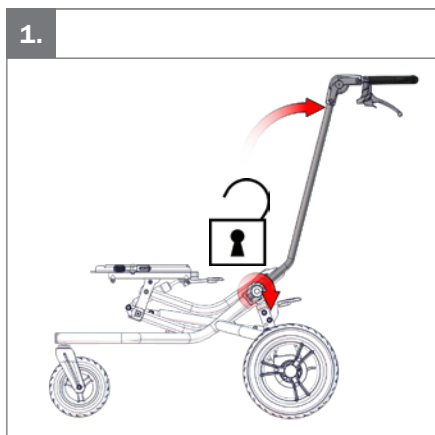
1.






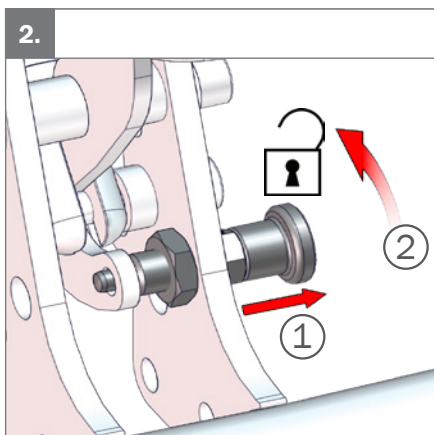
2.

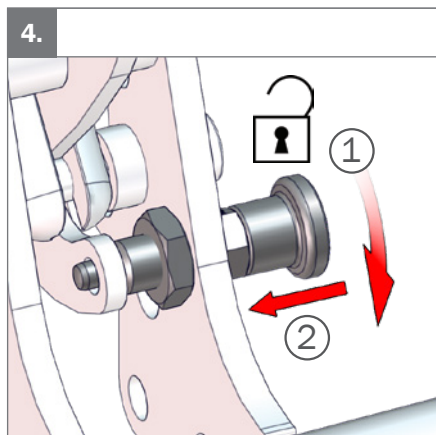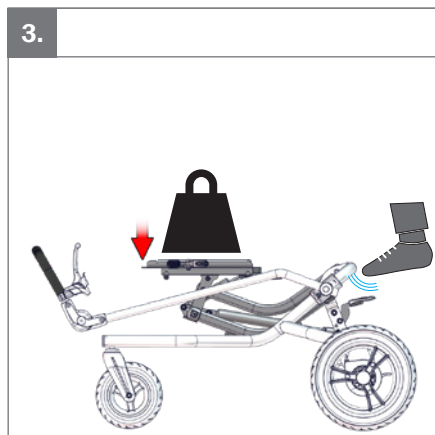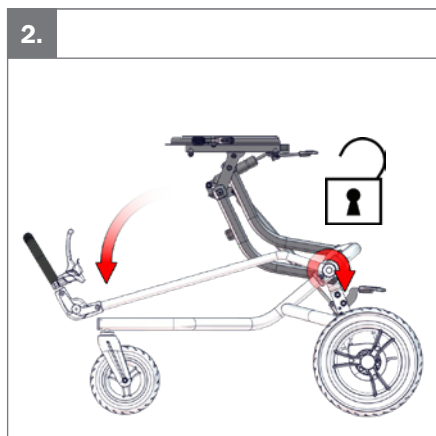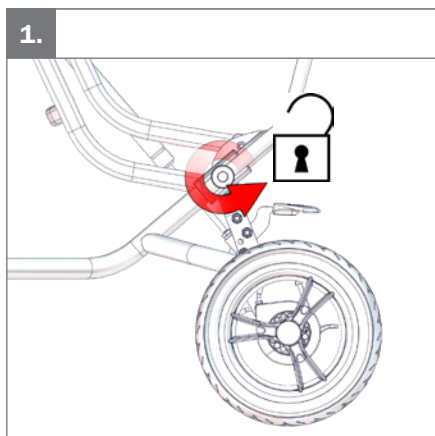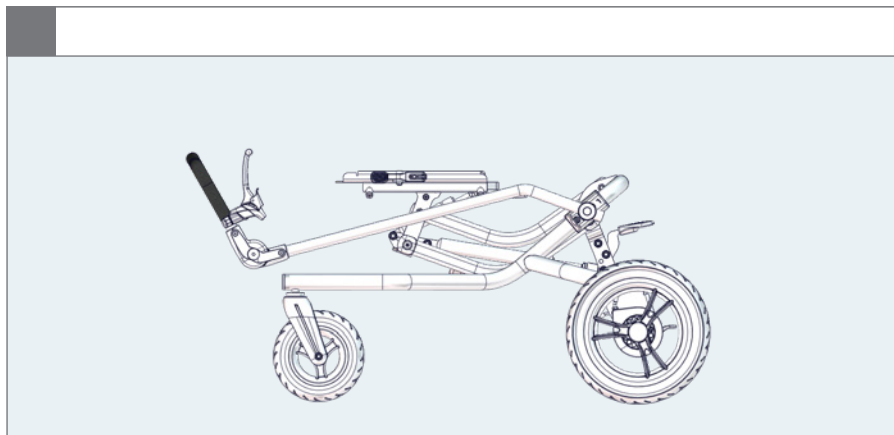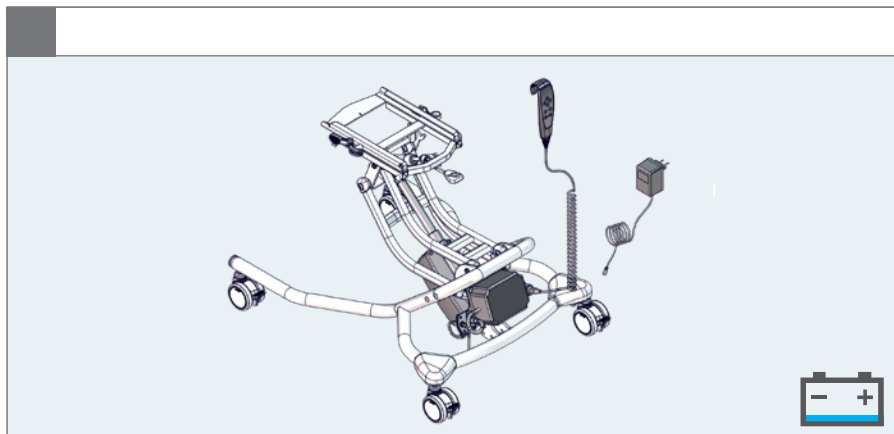
3.

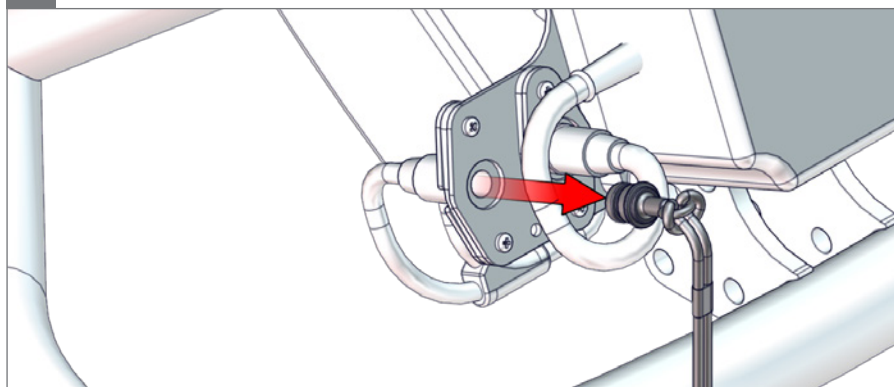
1.

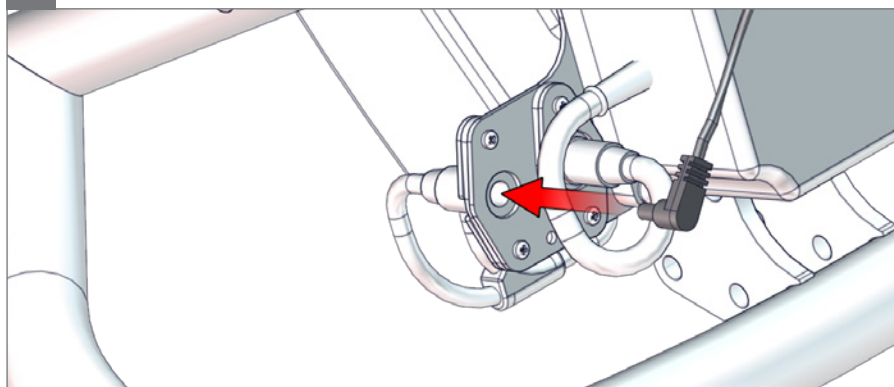
2.


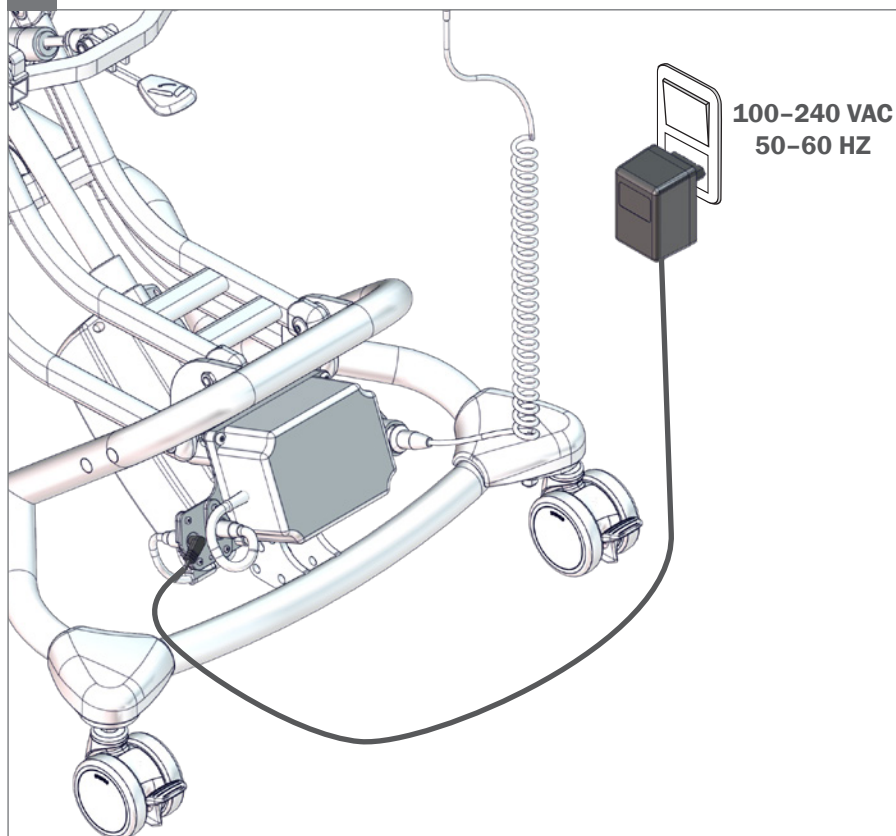
3.


100-240 VAC
50-60 HZ

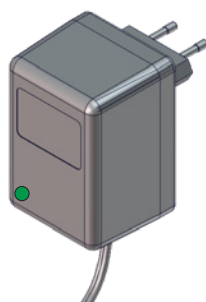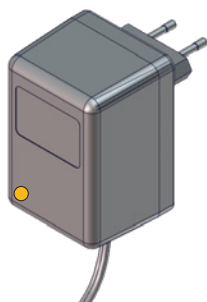
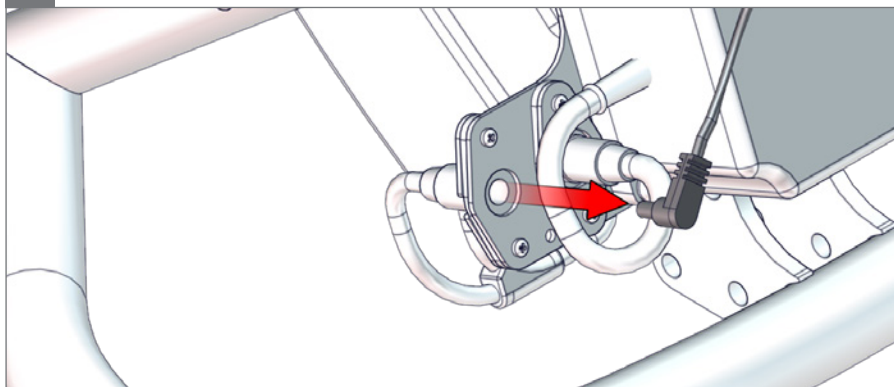
4.

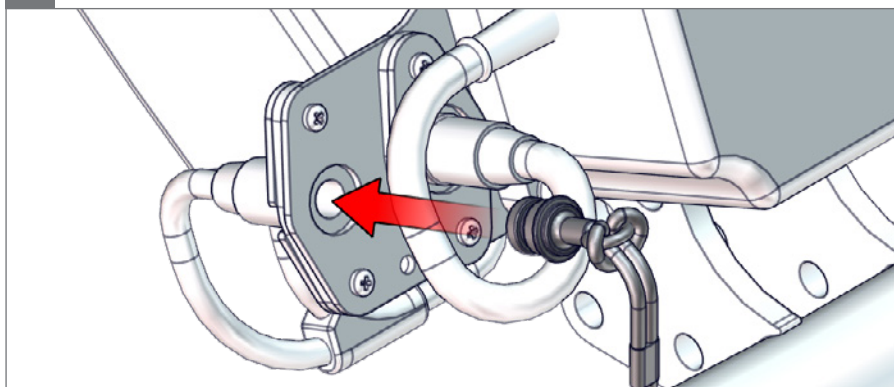5.

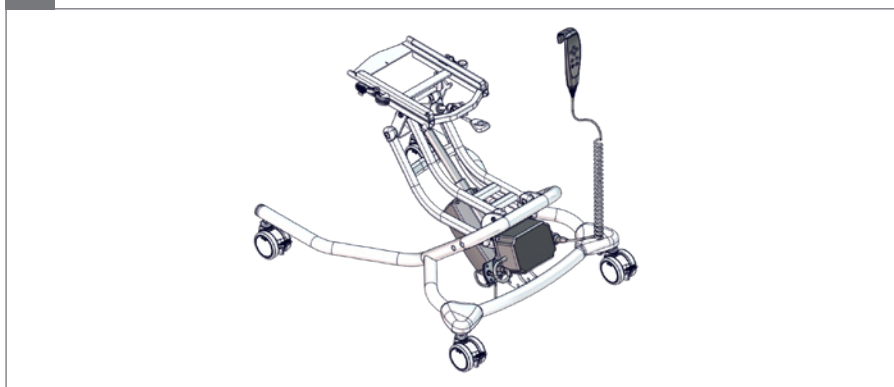
6.



Size 1 and 2

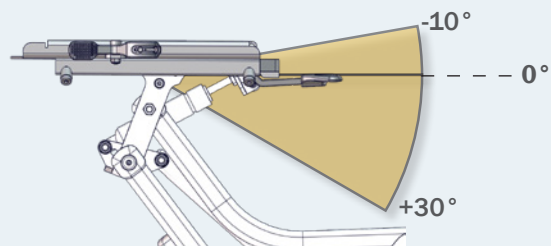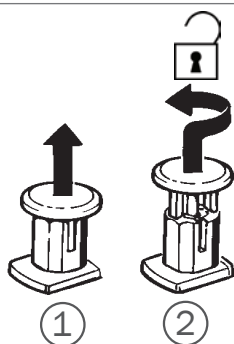
1.

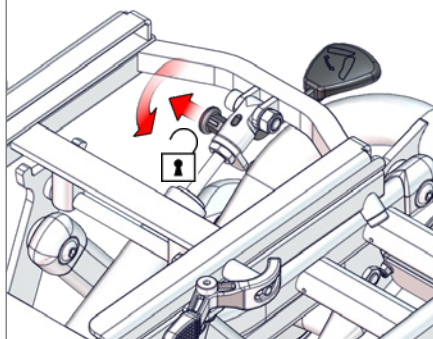2.

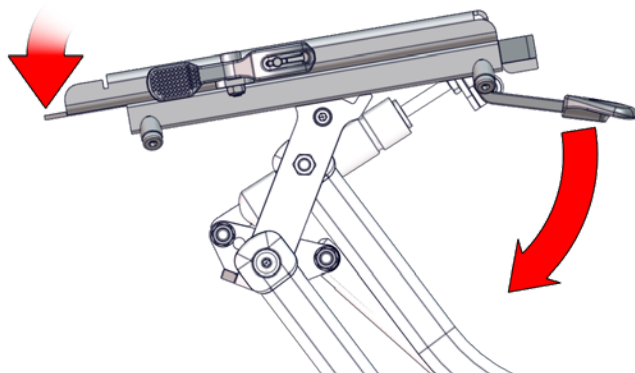
3.


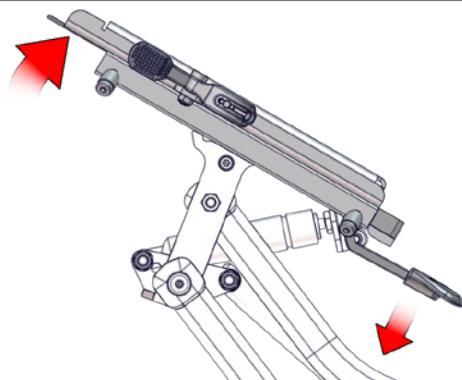
4.

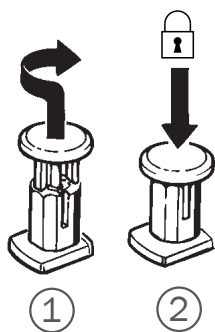
5.

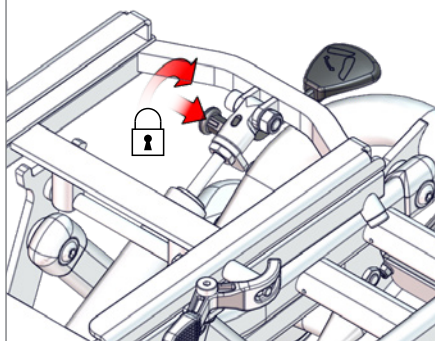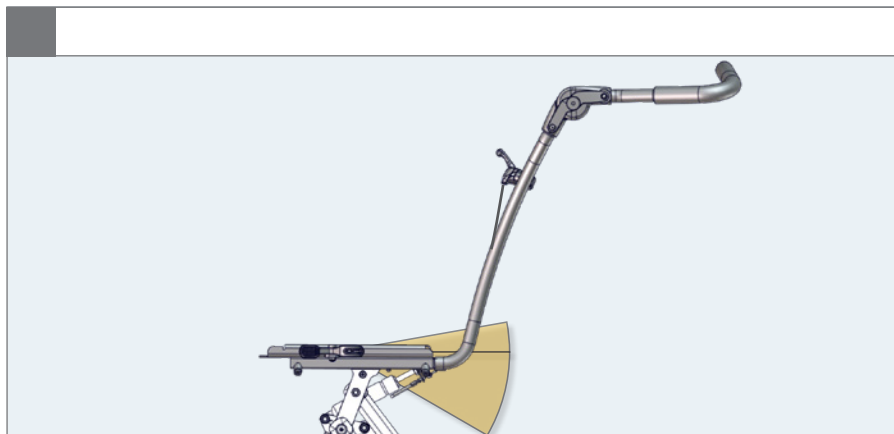
6.


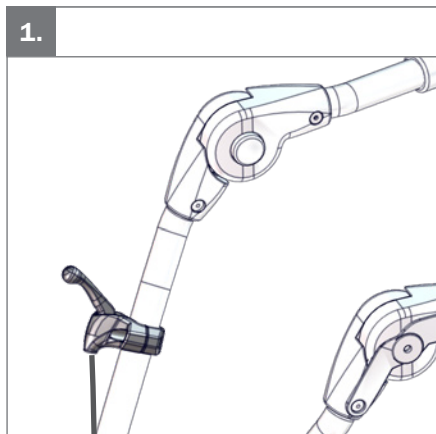
1.

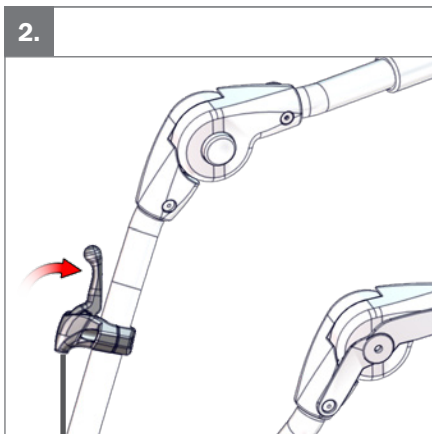2.

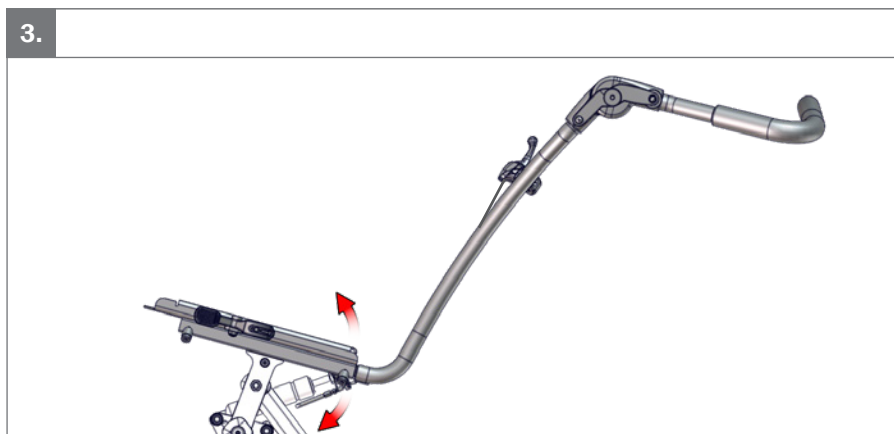
3.



Size 3

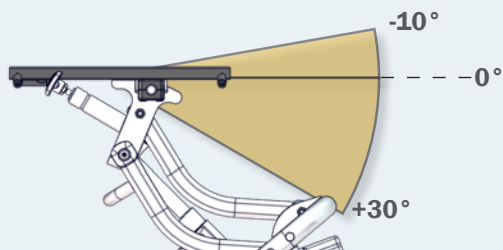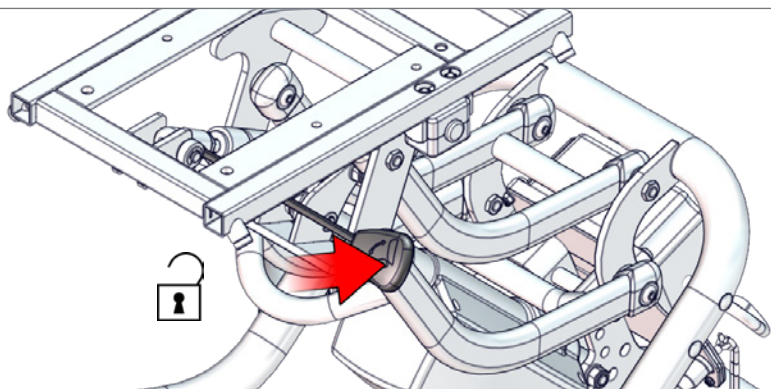
1.

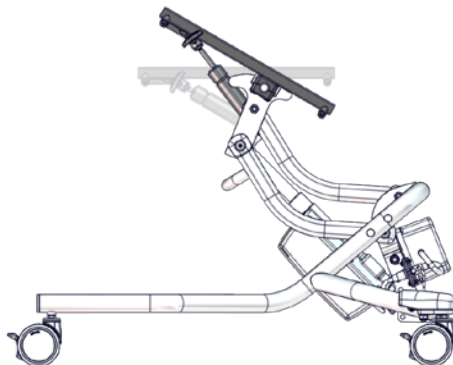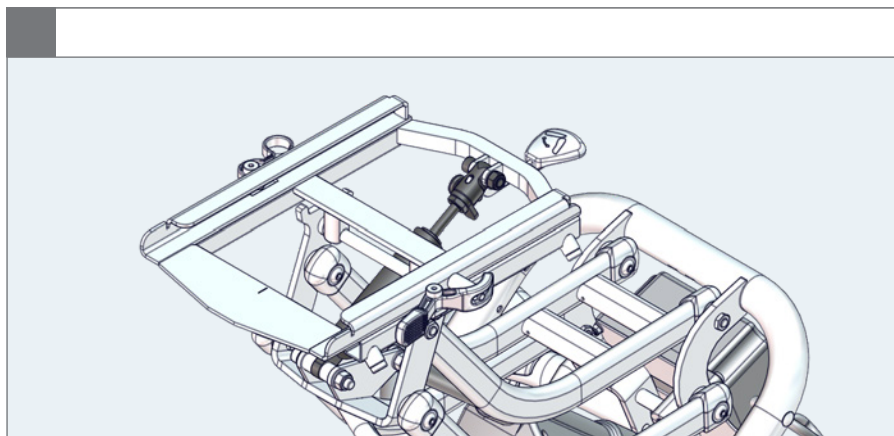
2.


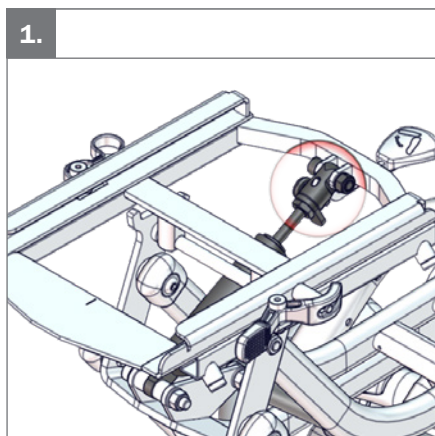
1.

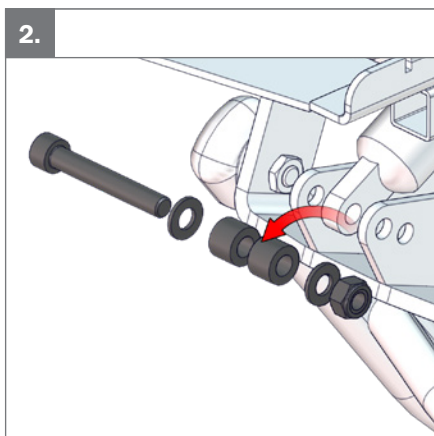2.

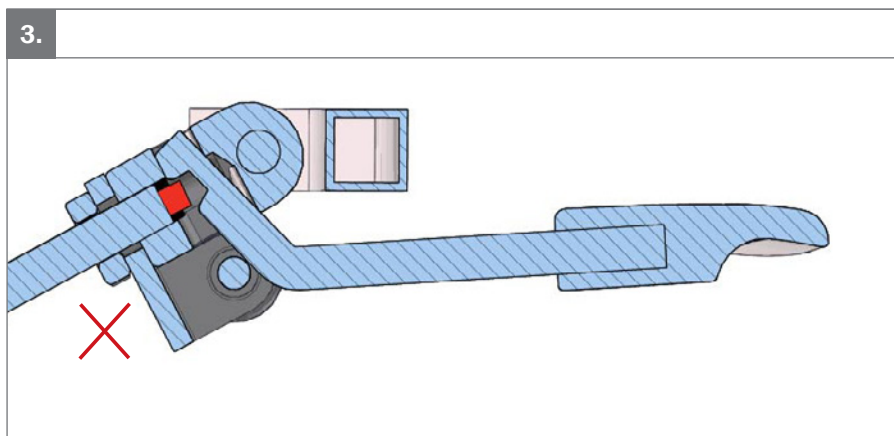
3.


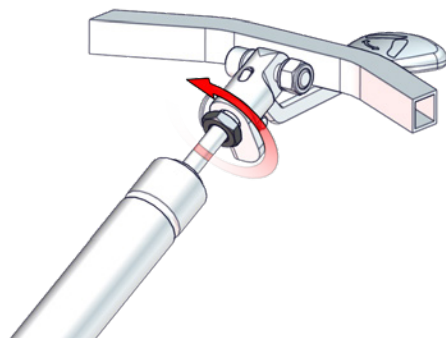
4.

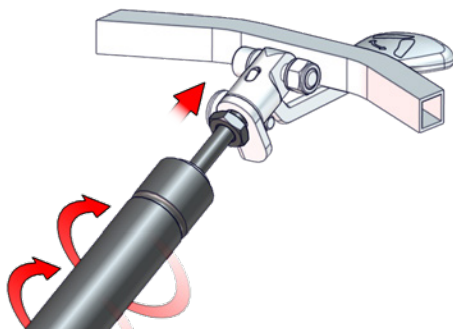
5.

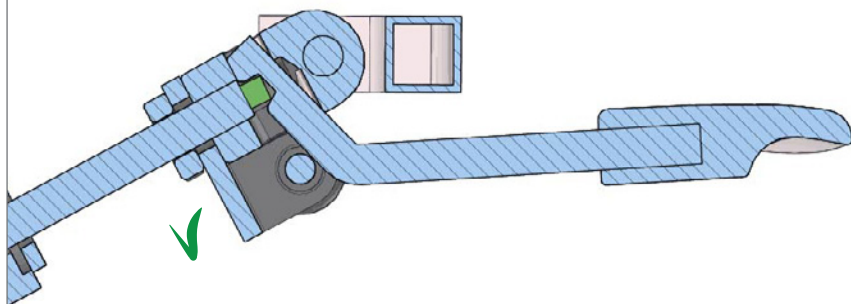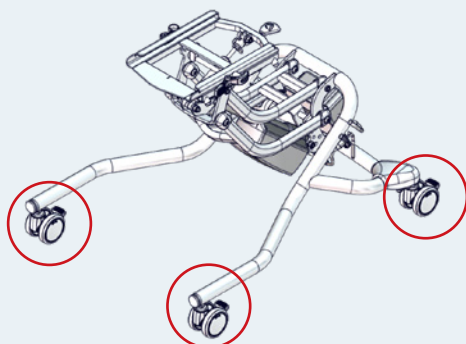
6.

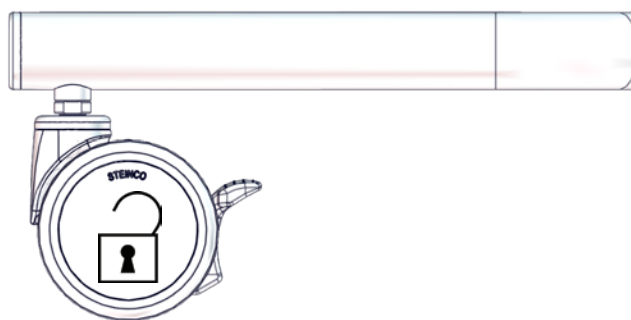
1.

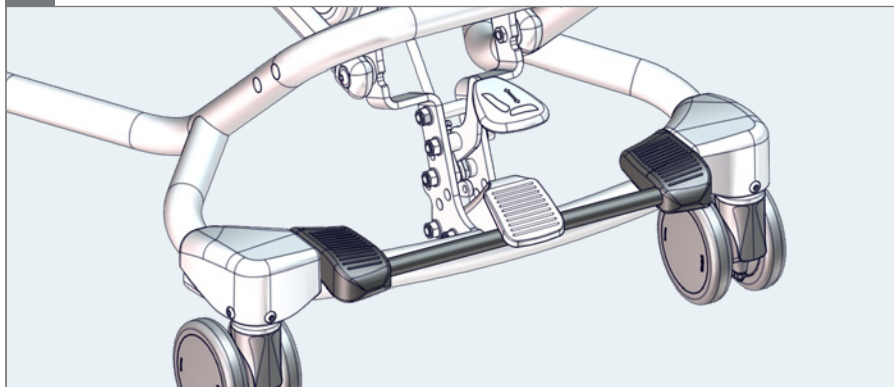
2.


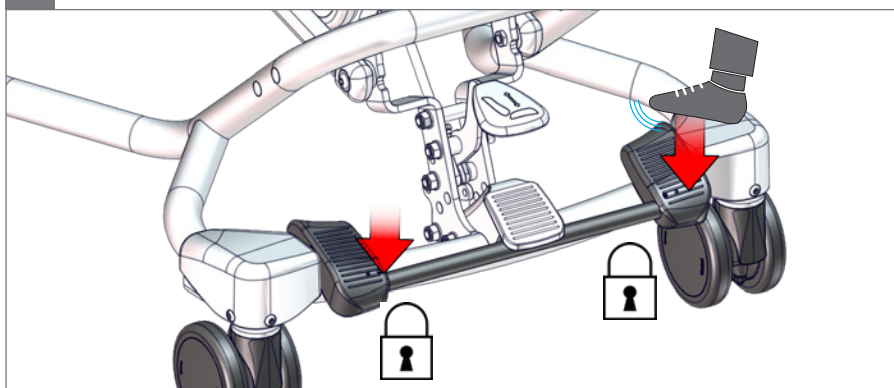
1.

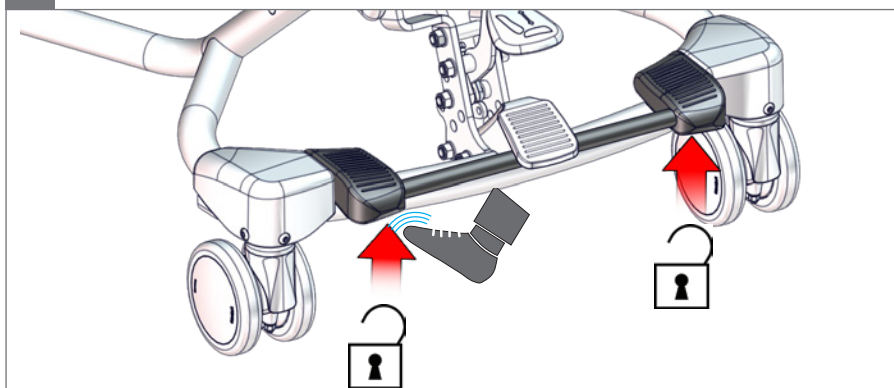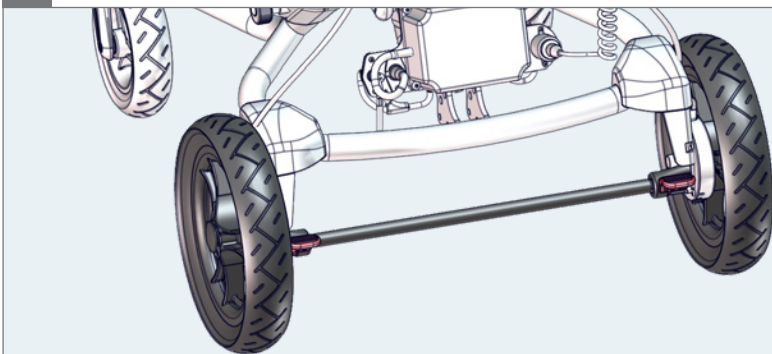
2.


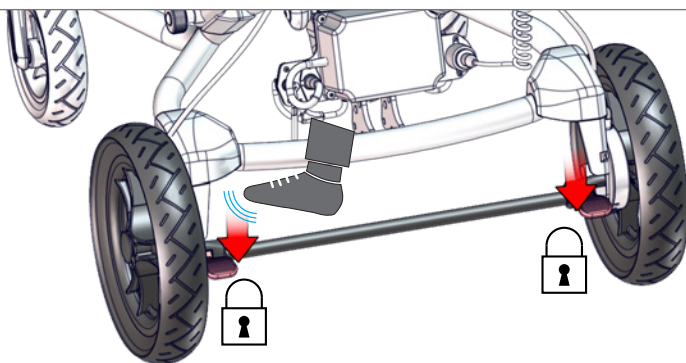
1.

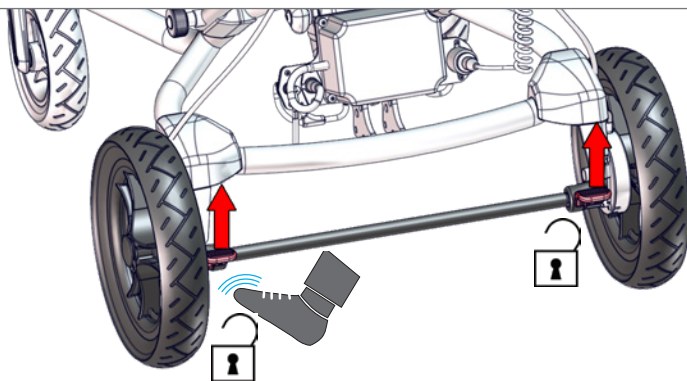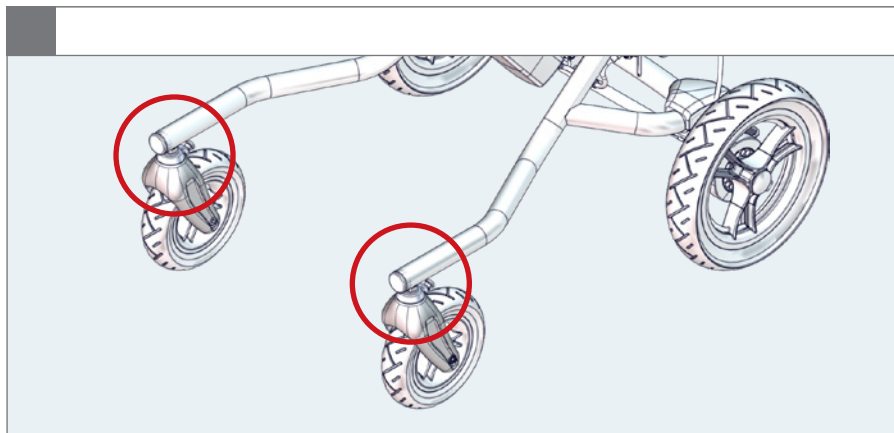
2.


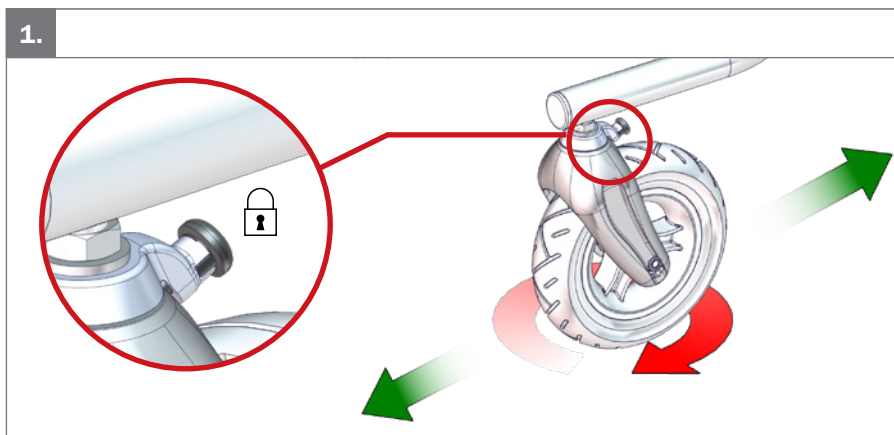
1.

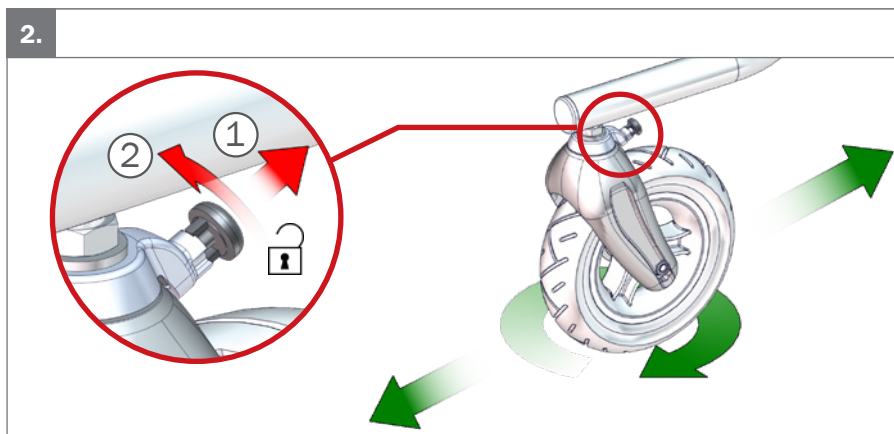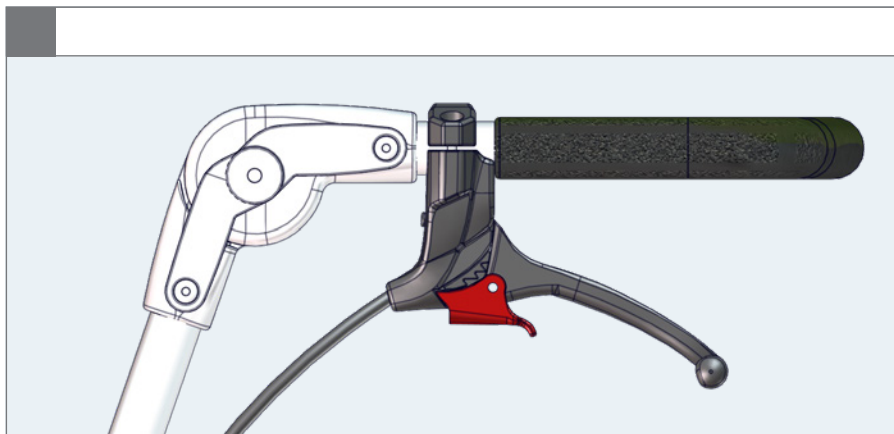
2.


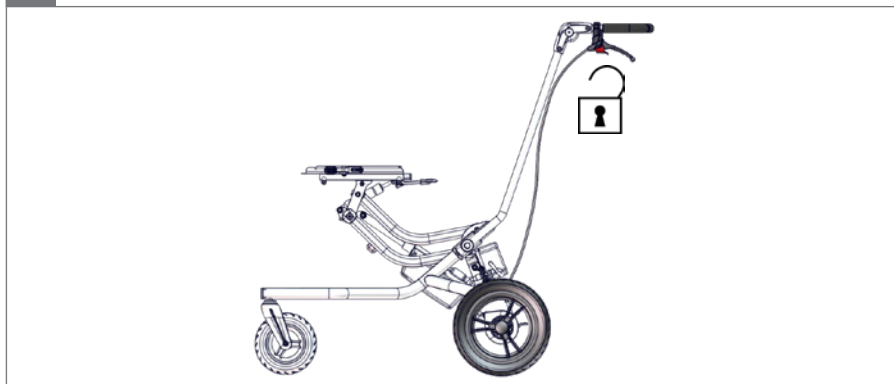
1.

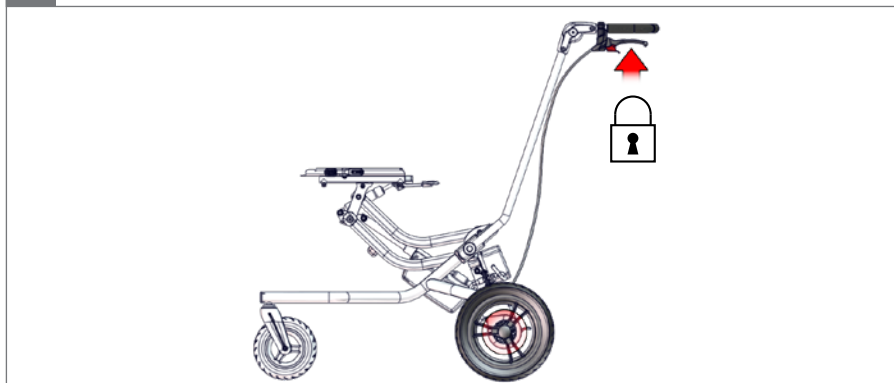
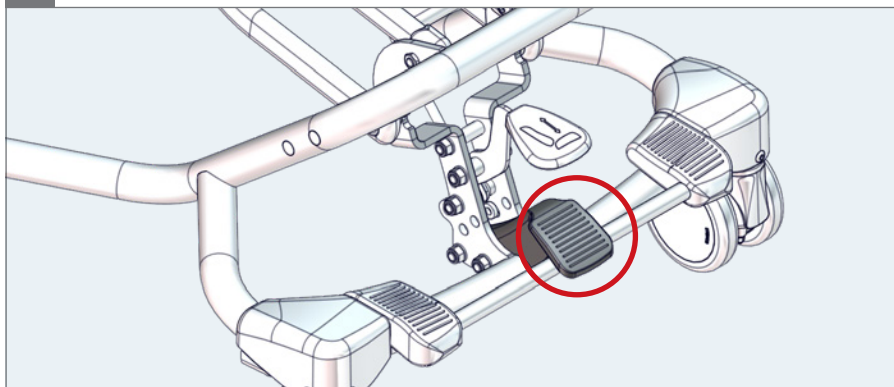
2.

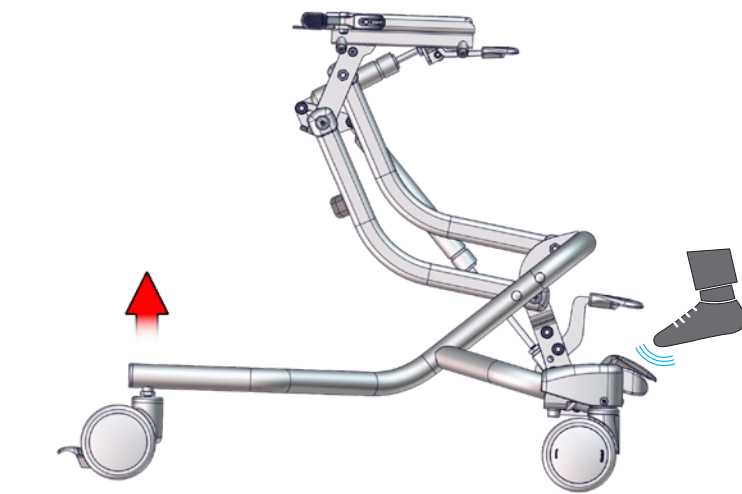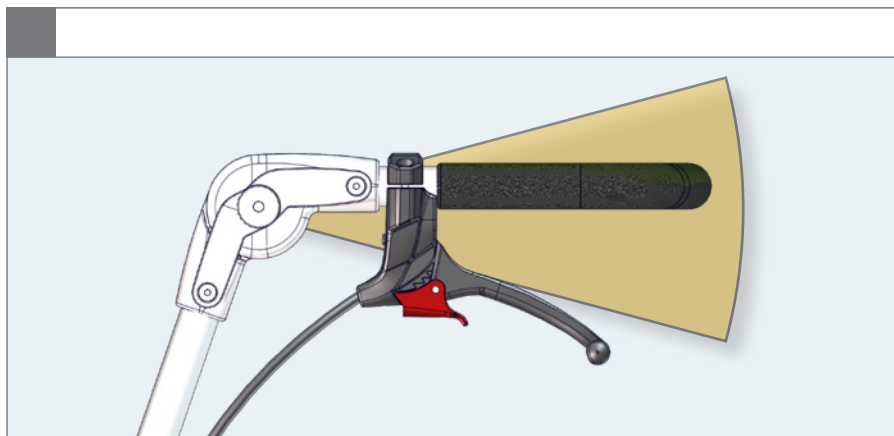
1.

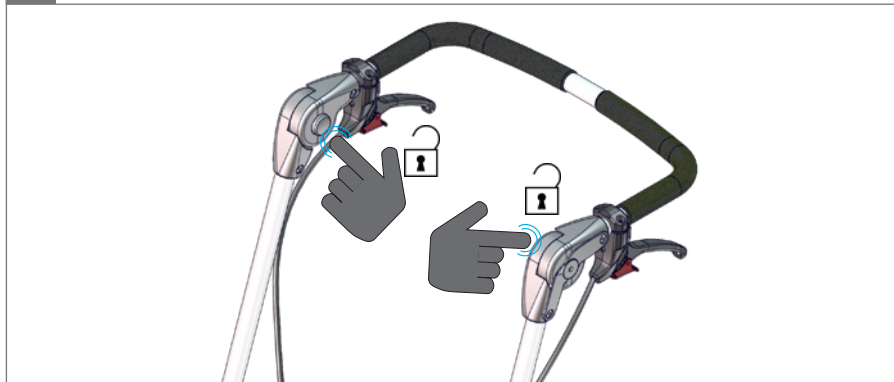1.

2.

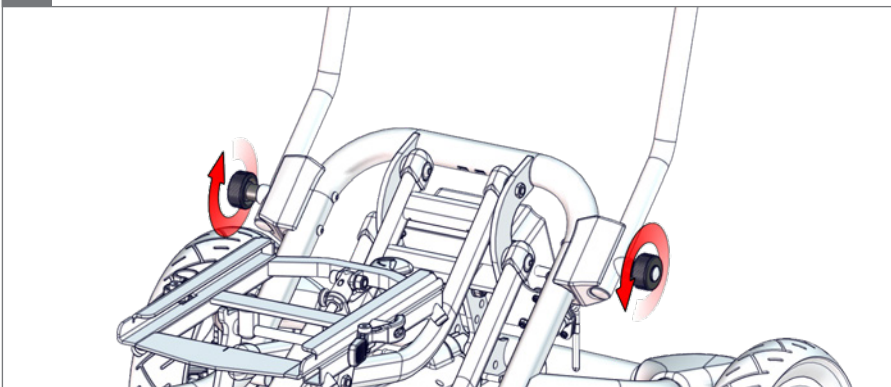
3.



1.

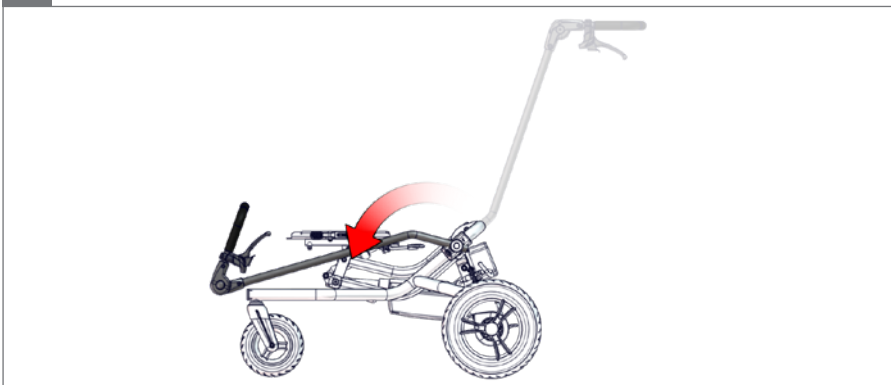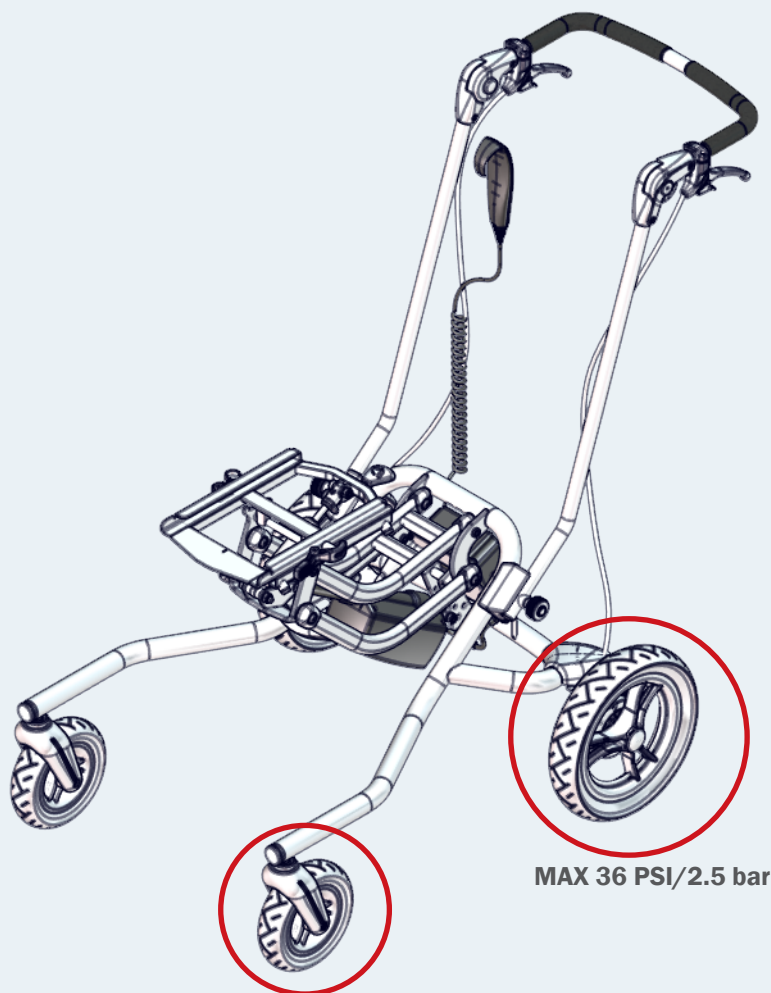2.


MAX 36 PSI/2.5 bar

MAX 36 PSI/2.5 bar


a.

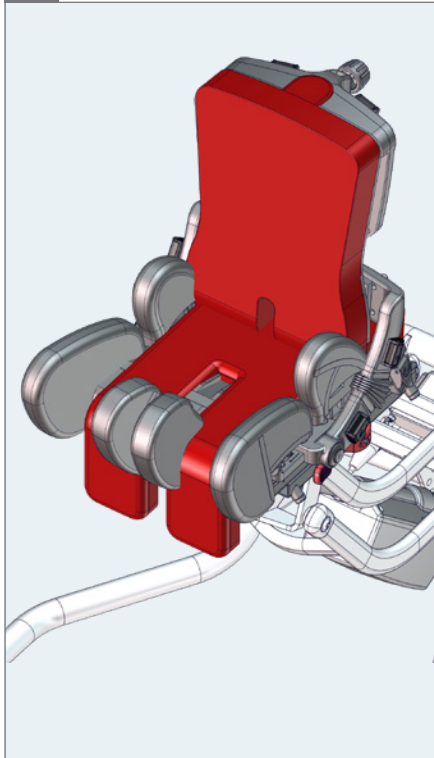Size 1 with x:panda size 1 - 2
Size 2 with x:panda size 3
Size 1 with x:panda shape size 1
Size 2 with x:panda shape size 2
Size 1 with Panda Futura size 1 - 4


b.

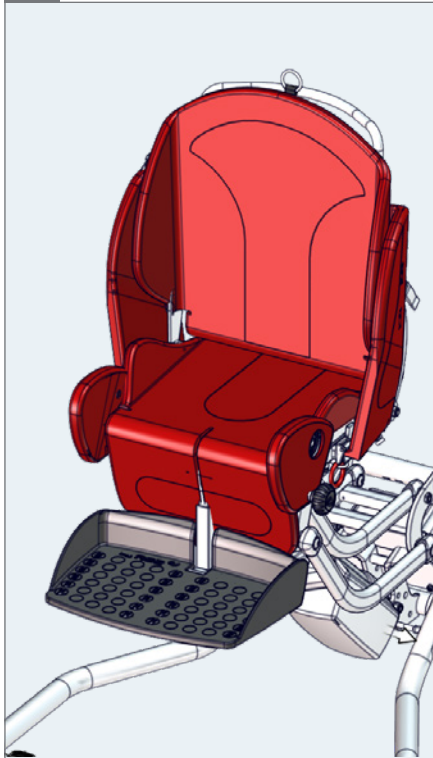
Indoor size 1 with Stingray size 1
Indoor size 2 with Stingray size 1 & 2

1a.

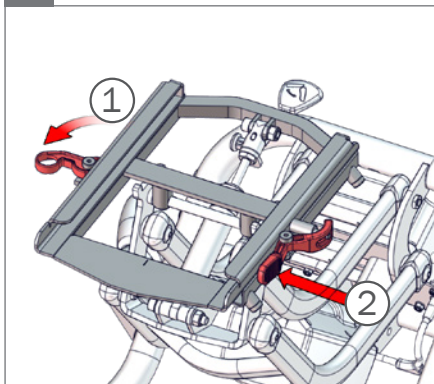
1b.

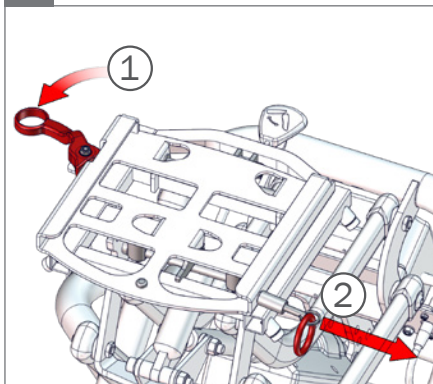
2.

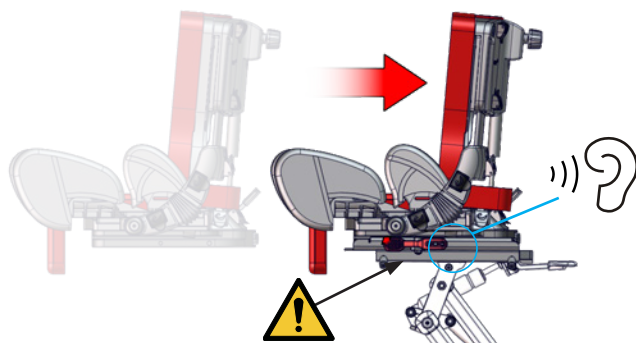
⚠ 05

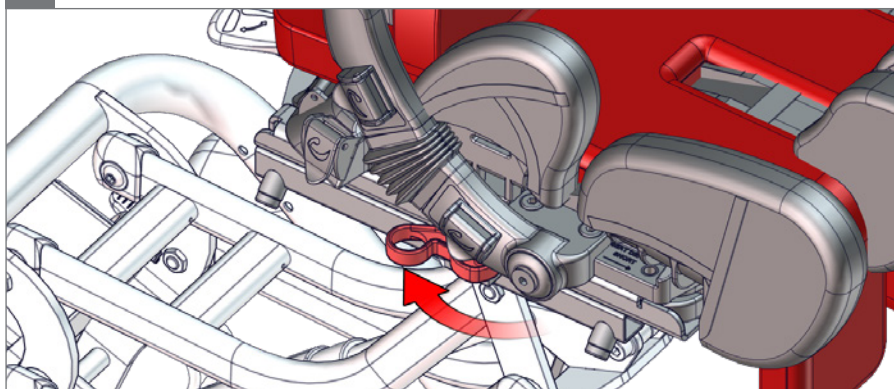3.

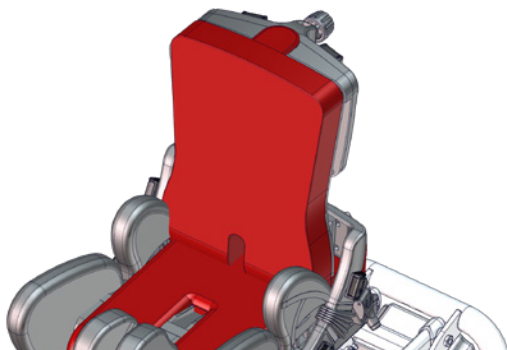
4.


Size 3 with x:panda size 4
Size 3 with x:panda shape size 3
Size 3 with Panda Futura 5

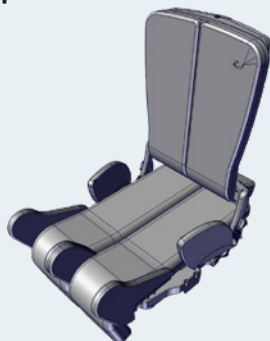x:panda size 4



+

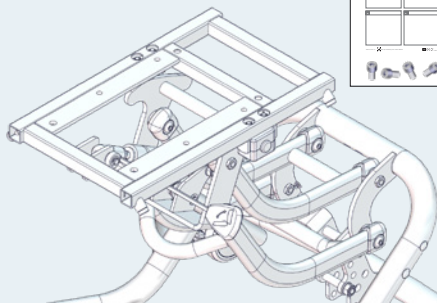x:panda shape size 3

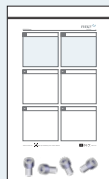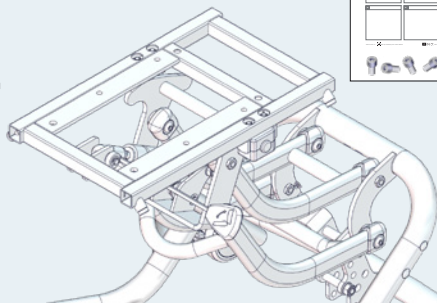
+




Panda Futura 5

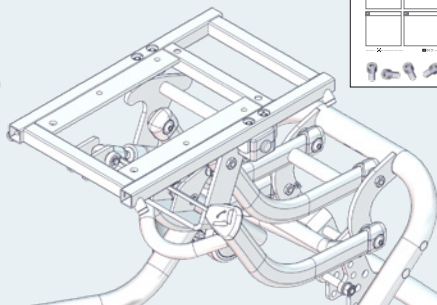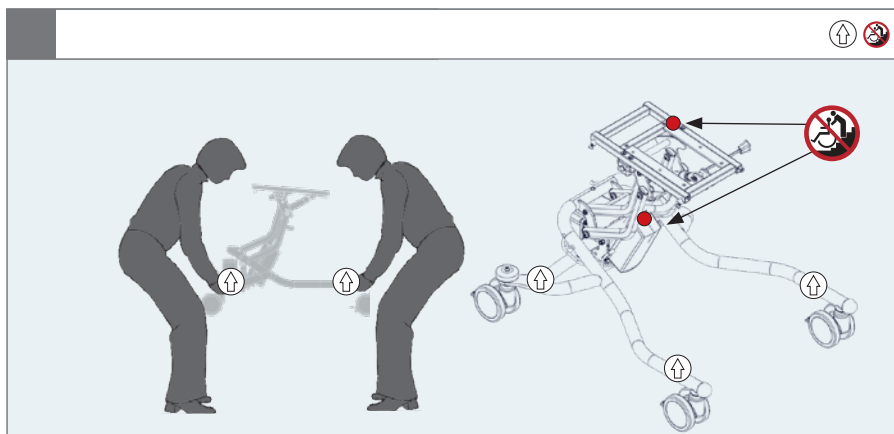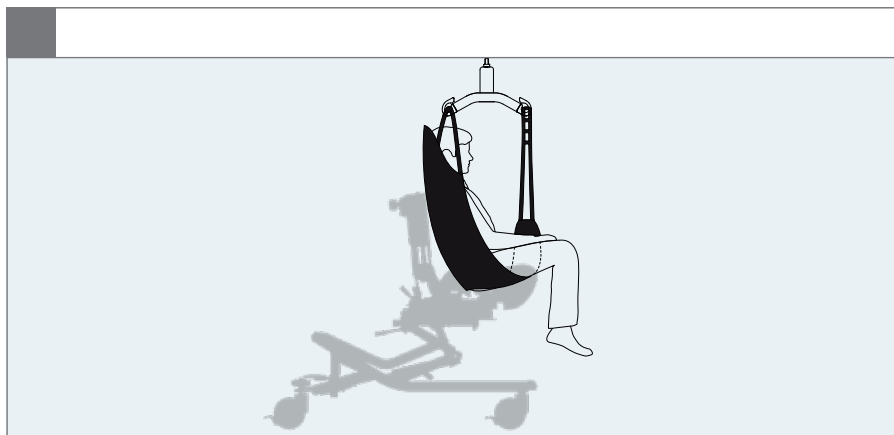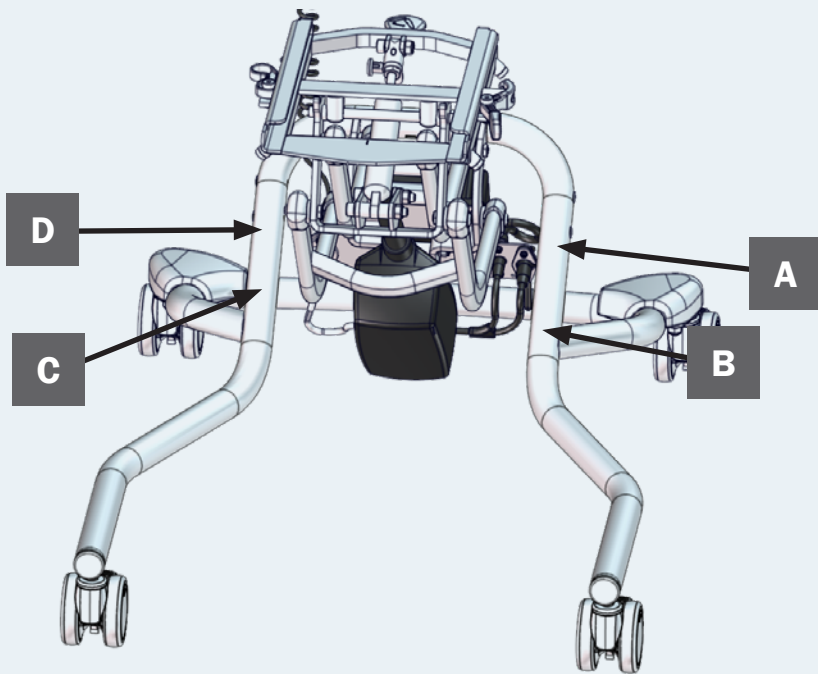+

A

R82
by Etac

R82 A/S, Parallelvej 3
DK-8751 Gedved
etac.com
Made in xxxxxxxx

MD

CE

UK

CA

YYYY-MM-DD

B

Posting group
Product name, size x

DMR ver.: xxxxx

Max. load: xxx kg

Max. rated load: xxx kg

Mass device: xxx kg

XXXXXXXXXXXXXXXXXXXX

XXXXXXXXXXXXXXXXXXXX

XXXXXXXXXXXXXXXXXXXX


C

Storage and Transport	50°C	80%	1060 hPa
	-10°C	20%	700 hPa
Operating	40°C	80%	1060 hPa
	5°C	20%	700 hPa

D

IP 24

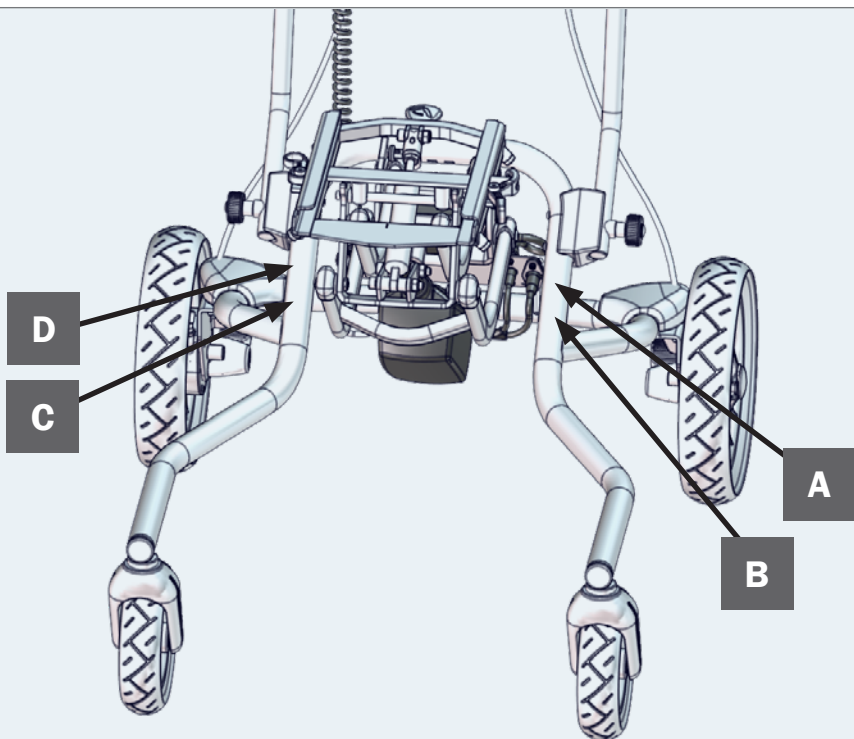DUTY CYCLE: 10% MAX 2 min./18 min.

A

R82[®]
by Etac

MD

R82 A/S, Parallelvej 3
DK-8751 Gedved
etac.com
Made in xxxxxxxx

CE UK CA

YYYY-MM-DD

B

Posting group
Product name, size x

DMR ver.: xxxxx

SN xxxxxxxxxxxxxxxx

Max. load: xxx kg
Max. load: xxx kg
Max. rated load: xxx kg
Mass device: xxx kg

XXXXXXXXXXXXXXXXXXXX

(xx)XXXXXXXXXXXXXXXXXXXXXX(XX)XXXXXXXX(XX)XXXXXX

C

Storage and Transport	50°C	80%	1060 hPa
	-10°C	20%	700 hPa
Operating	40°C	80%	1060 hPa
	5°C	20%	700 hPa

D

IP 24

DUTY CYCLE: 10% MAX 2 min./18 min.

EN: Warning: This symbol appears in the User Guide along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger
01 = Entrapment Hazards
02 = Tilting Hazards
03 = Transportation in motor vehicles
05 = Squeezing Hazard

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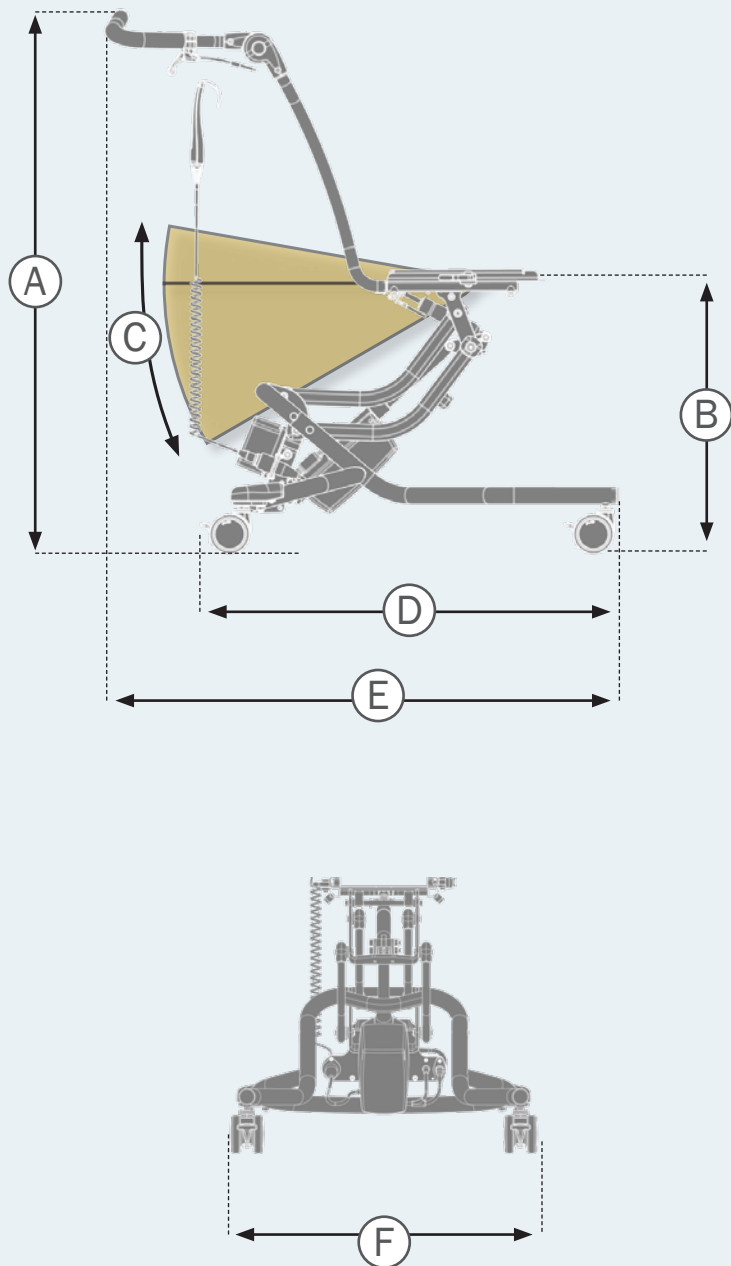
Indoor	1	2	3
 Overall height (A), to pushbrace with 75 mm castors	1100 mm (43 $\frac{3}{8}$ ")	1100 mm (43 $\frac{3}{8}$ ")	1100 mm (43 $\frac{3}{8}$ ")
 Overall height (A), to pushbrace with 100 mm castors	1130 mm (44 $\frac{1}{2}$ ")	1130 mm (44 $\frac{1}{2}$ ")	1130 mm (44 $\frac{1}{2}$ ")
 Overall height (A), to pushbrace with 125 mm castors	1170 mm (46 $\frac{1}{8}$ ")	1170 mm (46 $\frac{1}{8}$ ")	1170 mm (46 $\frac{1}{8}$ ")
 Seat surface height at front edge (B) (adapter plate) with 75 mm castors	170–580 mm (6 $\frac{3}{4}$ –22 $\frac{3}{4}$ ")	210–580 mm (8 $\frac{1}{4}$ –22 $\frac{3}{4}$ ")	270–530 mm (10 $\frac{3}{4}$ –20 $\frac{3}{4}$ ")
 Seat surface height at front edge (B) (adapter plate) with 100 mm castors	200–610 mm (7 $\frac{3}{4}$ –24")	240–610 mm (9 $\frac{1}{2}$ –24")	300–560 mm (11 $\frac{3}{4}$ –22")
 Seat surface height at front edge (B) (adapter plate) with 125 mm castors	240–650 mm (9 $\frac{1}{2}$ –25 $\frac{1}{2}$ ")	280–650 mm (11–25 $\frac{1}{2}$ ")	350–600 mm (13 $\frac{1}{2}$ –23 $\frac{1}{2}$ ")
 Seat plane angle (adapter tilt) (C)	(-10°) – (30°)	(-10°) – (30°)	(-10°) – (30°)
 Length (D)*	660 mm (27 $\frac{3}{4}$ ")	795 mm (31 $\frac{1}{4}$ ")	795 mm (31 $\frac{1}{4}$ ")
 Overall length (E)	670 mm (26 $\frac{3}{8}$ ")	830 mm (32 $\frac{3}{4}$ ")	830 mm (32 $\frac{3}{4}$ ")
 Overall width (F)	560 mm (22")	620 mm (24 $\frac{3}{8}$ ")	620 mm (24 $\frac{3}{8}$ ")
 Castors diameter	75/100/125 mm (3/4/5")	75/100/125 mm (3/4/5")	75/100/125 mm (3/4/5")
 Minimum turning radius	400 mm (15 $\frac{3}{4}$ ")	475 mm (18 $\frac{3}{4}$ ")	475 mm (18 $\frac{3}{4}$ ")
 Max. load	70 kg (154 lb)	75 kg (165 lb)	95 kg (209 lb)
 Mass device*	16 kg (35 lb)	17 kg (38 lb)	18 kg (42 lb)
 Max. rated load*	88 kg (194 lb)	94 kg (207 lb)	116 kg (256 lb)
 Static stability Sideways, downhill and uphill	10°	10°	10°

EN: *Measurements: The High-low:xo measurements above are based on the use of 75 mm castors with the frame in the most basic configuration. Different settings may result in another set of measurements. As we supply various types of accessories, the weight is listed without.

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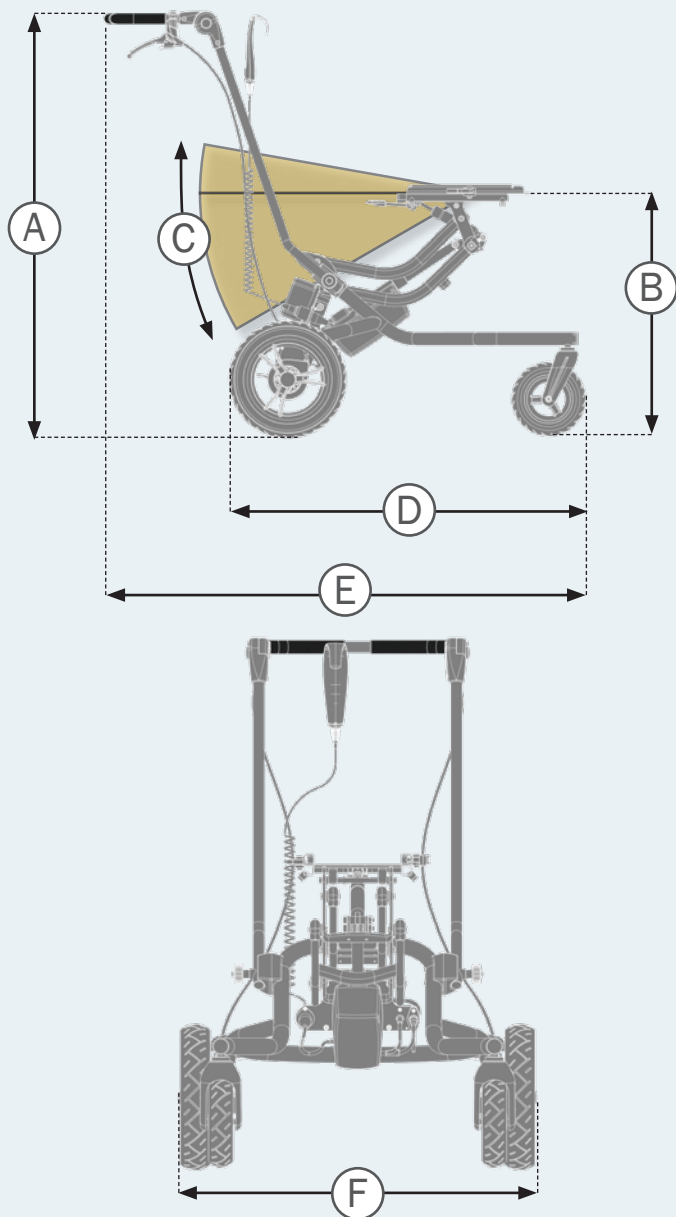
Outdoor	1	2
Overall height (A), to push brace	1045 mm (41¼")	1045 mm (41¼")
Seat surface height at front edge (B) (adapter plate)	300–720 mm (11¾–28¼")	345–720 mm (13½–28¼")
Seat plane angle (adapter tilt) (C)	(-10°) – (30°)	(-10°) – (30°)
Length (D)*	795 mm (31¼")	800 mm (31½")
Overall length (E)	1070 mm (42⅞")	1170 mm (46⅞")
Overall length with leg rest	1115 mm (43⅞")	1190 mm (46⅞")
Overall width (F)	660 mm (26")	700 mm (27½")
Foot support to adapter plate	50-290 mm (2-11½")	50-290 mm (2-11½")
Leg to seat surface angle	(-30°) – (30°)	(-30°) – (30°)
Front wheel h×w	190×50 mm (7½×2")	190×50 mm (7½×2")
Rear wheel h×w	290×60 mm (11½×2¼")	290×60 mm (11½×2¼")
Folded height	460 mm (18")	460 mm (18")
Minimum turning radius	875 mm (34½")	930 mm (36½")
Horizontal location of axle	200–210 mm (7¾–8¼")	255–265 mm (10–10½")
Max. load	70 kg (154 lb)	75 kg (165 lb)
Min. user mass in transportation	22 kg (49 lb)	22 kg (49 lb)
Max. user mass in transportation	57 kg (125 lb)	57 kg (125 lb)
Mass device*	25 kg (55 lb)	26 kg (57 lb)
Max. rated load*	97 kg (214 lb)	103 kg (227 lb)
Mass of the heaviest part	21 kg (46 lb)	22 kg (49 lb)
Pivot width	1180 mm (46½")	1250 mm (49¼")
Maximum slope (for use of parking brake)	7°	7°
Static stability Sideways, downhill and uphill	10°	10°

EN: *Measurements: The High-low:xo Outdoor measurements above are based on the use of standard solid wheels with the frame in the most basic configuration. Different settings may result in another set of measurements. As we supply various types of accessories, the weight is listed without.

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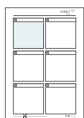
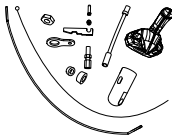
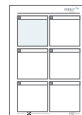
ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this user manual before use and save it for future reference.

Intended purpose

The High-low:xo is a frame for a seating system for children and adults with disabilities.

Indications

The device is used together with a seating system (i.e. a modular seating system like R82 x:panda or R82 Panda Futura). It can be used by children and adults with disabilities, who lack the ability to maintain a seated position and move around (Within the room or between rooms / outdoors) without an assistive device. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age.

The user is the person sitting in the device. The caregiver is the person operating the device. Never leave the user unattended in this device. Ensure permanent supervision by a caregiver.

The use of the device will be discontinued more times during the day where the user will be lying down, standing or sitting in another assistive device or a conventional product.

The outdoor version of the device is approved for transportation for children and adults in the weight range of 22 kg to specified max user weight for transportation.

Application areas

The product is for indoor and outdoor use

Contra indications

None

Precautions

None

Enclosed tools

None

CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- High-low:xo indoor: EN 12182
- High-low:xo outdoor: EN 12182 & EN 12183

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts

and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Alignment of user weight/mass of the combined seat and base, approval for transportation in vehicles of both products (ISO16840-4 for seat and ISO7176-19 for base)
- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

The symbol of the crossed-out trash can on batteries or accumulators states that they must not be disposed of in household waste at the end of their service life. If batteries or accumulators contain mercury (Hg), cadmium (Cd) or lead (Pb), you will find the respective chemical symbol below the symbol of the crossed-out trash can. You are legally obliged to return old batteries and accumulators after use. You can do this free of charge at the retail store or at another collection point in your area. Addresses of suitable collection points can be obtained from your city or local government.

Batteries can contain substances that are harmful to the environment and human health. Special care should be taken due to the special risks involved in handling lithium-containing batteries. The separate collection and recycling of old batteries and accumulators is intended to avoid negative effects on the environment and human health. Please avoid the generation of waste from

old batteries as much as possible, i.e., by preferring batteries with a longer service life or rechargeable batteries. Please avoid littering of public spaces by not carelessly leaving batteries or battery-containing electrical and electronic equipment behind. Please consider ways to reuse a battery instead of disposal, for example by reconditioning or repairing the battery.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

For information about product safety notices and product recalls, local dealer information are available at the website etac.com

Caregiver

- Read the “User Guide” carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user.
- Always use the correct lifting techniques and aids for this purpose.
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver.
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier.
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user guide.
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible.
- When adjusting the seat angle, keep a hand on the seat mounted on the frame to ensure slow angling.
- Ensure that moving parts does not squeeze or trap body parts when adjusting them.
- Ensure that body parts are not trapped in openings for mounting of supports/ accessories.
- Do not let children and youngsters play with the handset.
- It is important to remove the transport plastic bag before using the handset. Cables need to be placed in such a way that users or carers cannot trip and injure themselves.
- Always use the product in a moderate pace. Do not run with the product.
- Information about transportation can be found in the document “M1460 Transportation in Motor Vehicles” supplied with the product.
- Information about roadability can be found in the document “Roadability” supplied with the product.
- Information regarding frame and seat combinations including max. load in combination can be found at: etac.com.

- This product contains small parts which could present a choking hazard if removed from their intended placing.

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use.
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces.
- When using on inclines, please refer to measurements.

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered.
- Do not place more than one user in the product.
- When changing user, clean the product and disinfect it if necessary. Carry out necessary adjustments after cleaning.

Product

- Removable parts without use of tool: Foot support and seat. The removal will have adverse effect on the device.
- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children.
- Activate the brakes before placing the user into the product.
- Ensure the wheels are fully-functional and secure before use.
- The correct tyre pressure is printed on the tyre casing and listed in the section for tyre technical data.
- In case of a tyre puncture do not use the product until remedied.
- Ensure the stability of the product is maintained before placing the user in it.
- When the footplate is close to the floor and the castors, the castors may be unable to swivel.

- If the frame has to be used in the lowest position, remove the footrest before adjustment.
- Inspect the product and all of its accessories and replace any worn out parts before use.
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring.
- When mounting a Panda Futura seat size 4 on any High-low:xo frame, you must mount the anti-tip device that comes as an accessory to the product.
- The power unit does not have a safety stop.
- Do only use the power unit within specified working limits.
- Do not attempt to repair any part of the power unit.
- The user must not occupy the product when it is lifted from the ground.
- The product is approved for transportation in vehicles according to ISO 7176-19. More information can be found in the document "M1460 Transportation in Motor Vehicles" supplied with the product.
- In accordance with ISO 7176-19 Annex D, the product is evaluated regarding the ability to accommodate vehicle-anchored pelvic and shoulder belts. The device receives an overall score of "Good".
- The device conforms to ISO 7176-8, covering requirements and test methods for static, impact and fatigue strengths.
- Unoccupied device is suitable for land and air transport.
- Details covering EMC can be found in the appendix "EMC details" supplied with the product.
- Depending on the size, the device is intended for occupant mass groups I and II (ISO 7176-5).

Safety product

Due to variant and settings the wheelchair may become larger than the recommended dimensions. Compliance with recommended overall width and overall length ensures unhindered use of emergency escape routes. (See table 1)

Table 1

Recommended dimensions	Ready for use (mm)
Overall length – maximum	1200
Overall width – maximum	700
Overall height – maximum	1200
Turning diameter – maximum	1000
Pivot width – maximum	1300
Ground clearance - minimum	30

Product safety

Due to variant and settings the wheelchair may not meet the limits for wheelchairs transportable by train. Measures can be taken to verify whether the specific product in question meets the limits. (See table 2)

Table 2

Wheelchairs transportable by train - limits	
Overall length – maximum	1200 mm + 50 mm for feet
Overall width – maximum	700 mm + 50 mm on each side for hands
Overall height – maximum	1375 mm (including 95th percentile male occupant)
Turning diameter – maximum	1500 mm
Smallest wheels	Accommodate a gap of 75 mm horizontal and 50 mm vertical
Weight – maximum	200 kg for manual wheelchair and occupant (including baggage)
Obstacle height that can be overcome – maximum	50 mm
Ground clearance - minimum	60 mm under foot rest with upward slope angle of 10° on top for going forward
Safe slope on which the wheelchair remain stable - maximum	6° dynamic stability in all directions 9° static stability in all directions, including with brake applied

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Reuse

This product and its accessories/components are suitable for reuse.

Refurbishment for reuse

Before reuse or changing the user, follow the service checklist, maintenance and washing instructions and carry out regular inspections.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

Service checklist:

Check and adjust the following as per their intended use:

- Frame is adjustable in height
- Powered versions are able to retain power after charging the battery
- On outdoor versions, the push-brace can be folded over the frame
- If equipped with airfilled tyres, check the air pressure
- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- Wheels are moving freely and that all wheel locks/brakes are easily activated. Adjust if required
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact.

Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to:
Wipe off any residue and dirt from the product,

using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

Before washing

Avoid degreasing of the piston rods, the actuators should be retracted to minimum stroke before washing.

- Remove all accessories, unplug all cables and make sure to dismount the battery, control-box and handset. Leave the actuators with the cables mounted on the product.
- Do NOT clean the power unit with a high pressure cleaner or steam cleaner.

After washing

- Ensure correct placement of cables inserted into the control box when reinstalling the power unit system.
- Check that the handset functionalities works correct.

Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Materials

- Rubber
- Plastic
- Steel

Surface treatment

The following surface treatment processes were used to protect against corrosion:

-
- Lacquered surfaces = Polyester powder coating or ED-coating
 - Non-lacquered aluminium parts = Anodized coating
 - Non-lacquered steel surfaces = Galvanized.

Power unit

For an optimum lifetime the power unit must be connected to the mains voltages as often as possible (e.g. every night). Charge the batteries 12 hours before use.

The batteries must be charged at least every 3rd month - otherwise there might be damages due to self-discharge. It is recommended to test the battery function at least once every year.

Recharging the battery:

- Remove the plug from the power system and plug in the charging lead
- Plug the charger into the electricity supply and switch ON. A yellow light on the charger will light up
- Charge the batteries until the lamp on the charger turns green

Technical Data

Battery - BA18

- Capacity 1.3 Ah - 24V DC
- Charging current is max. 500 mA
- IPX6

The batteries have a maximum life of 4 years but this can vary depending upon the conditions of use and charging practices

Battery charger - CH01

- Mains: 100 - 240VAC/50-60 Hz switch mode
- Charging voltage: 27.6 VDC
- Charging current: Max. 500 mA
- FRIWO Gerätebau GmbH, Model FW7218M/24

Actuator - LA28/LA32

- Input voltage: 24V
- Noise level: ≤ 45 dB(A)
- IPX5
- Duty cycle: Max. 10% or 2 min. use then 18 min. without use

Handset - HB 30

- IPX6

Detachable parts

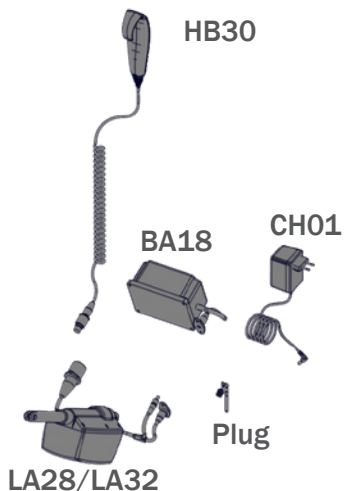
- Handset - HB 30
- Battery - BA 18

Applied part

- Handset - HB 30

Environmental conditions

- Operating temperature: +5 °C to +40 °C
- Storage temperature: -10 °C to +50 °C
- Relative humidity: 20% to 80%
- Atmospheric pressure: 700 to 1060 hPa

Ambient and surface temperature

The table below illustrates how long time it takes for the surfaces of the device to reach operating temperature from -10 °C and +50 °C.

Surface temperature			
-10 °C	to	+5 °C	108 min
+50 °C	to	+40 °C	129 min

IP-Rating table - First Digit: Solids

The first digit indicates the level of protection that the enclosure provides against access to hazardous parts (e.g., electrical conductors, moving parts) and the ingress of solid foreign objects.

IPOX	No protection against contact and ingress of objects
IP1X	Any large surface of the body, such as the back of the hand, but no protection against deliberate contact with a body part.
IP2X	Fingers or similar objects.
IP3X	Tools, thick wires, etc.
IP4X	Most wires, screws, etc.
IP5X	Ingress of dust is not entirely prevented, but it must not enter in sufficient quantity to interfere with the satisfactory operation of the equipment; complete protection against contact.
IP6X	No ingress of dust; complete protection against contact.

Second Digit: Liquids

Protection of the equipment inside the enclosure against harmful ingress of water.

IPX0	No protection
IPX1	Protection against vertically falling drops of water e.g. condensation
IPX2	Protection against direct sprays of water up to 15 ° from the vertical
IPX3	Protected against direct sprays of water up to 60 ° from the vertical
IPX4	Protection against water sprayed from all directions - limited ingress permitted
IPX5	Protected against low pressure jets of water from all directions - limited ingress
IPX6	Protected against temporary flooding of water, e.g. for use on ship decks - limited ingress permitted
IPX7	Protected against the effect of immersion between 15 cm and 1 m
IPX8	Protects against long periods of immersion under pressure

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**Etac Ltd.**
**Gelbart AG**