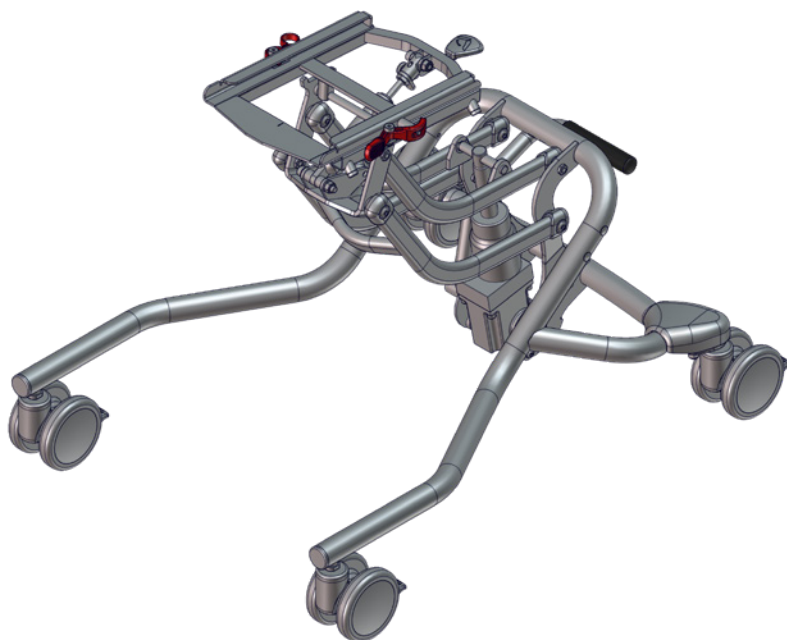
User manual

High-low:xo
with hydraulic height adjustment

R82[®]
by Etac

M1192 - 2023-12-18 - Rev. 4.0


www.etac.com


	3	ENGLISH DEUTSCH	37 – 39 40 – 44
	4	NEDERLANDS DANSK	45 – 48 49 – 51
	5	NORSK SVENSKA	52 – 54 55 – 57
	6-9	SUOMI ESPAÑOL	58 – 60 61 – 64
	10	FRANÇAIS ITALIANO	65 – 68 69 – 72
	11-12	PORTUGUÊS PORTUGUÊS DO BRASIL	73 – 76 77 – 80
	13-14	РОССИЯ POLSKI	81 – 85 86 – 89
	15	ČESKY 中國	90 – 92 93 – 95
	16-18	日本 ΕΛΛΗΝΙΚΗ	96 – 101 102 – 105
	19	ÍSLENSKA	106 – 108
	20		
	21-31		
mm (inch) kg (lb)	32-34		
	35-36		

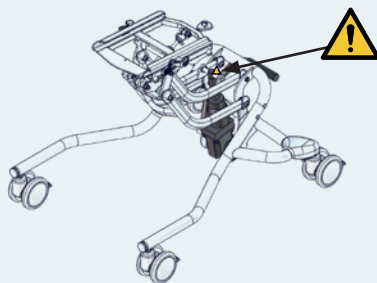
 05

Size 2

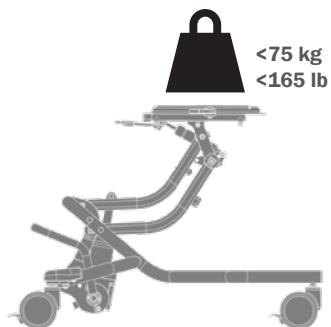
Size 3


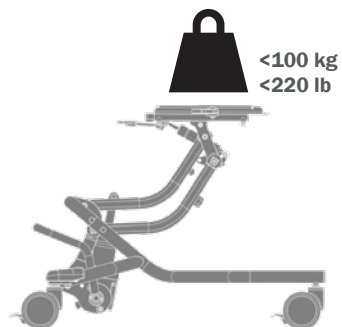
! 05

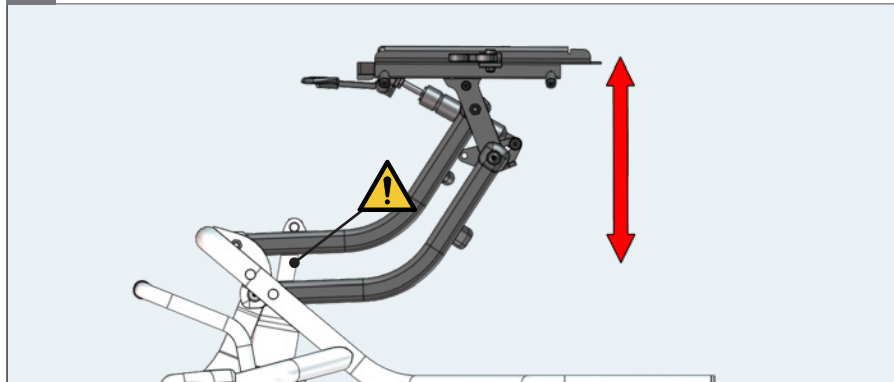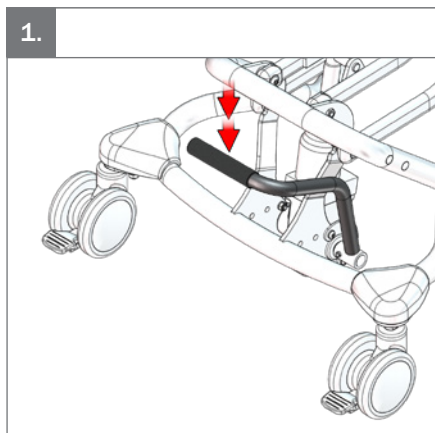1.

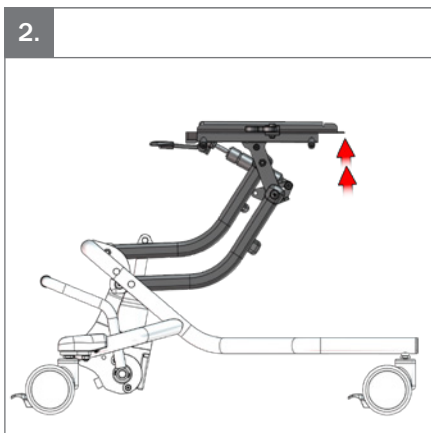
2.



⚠ 05

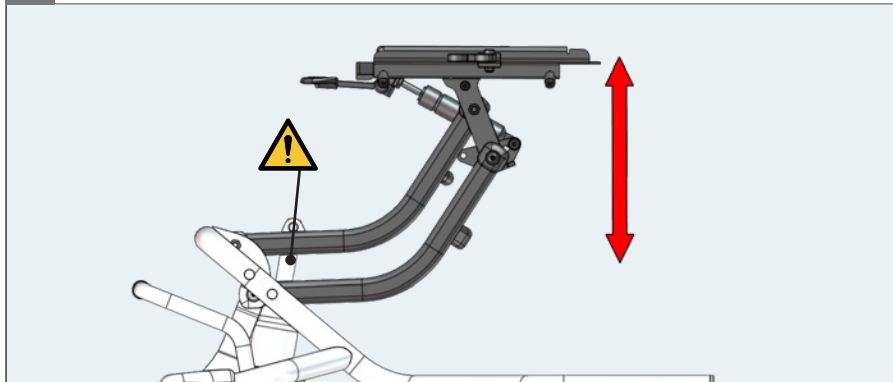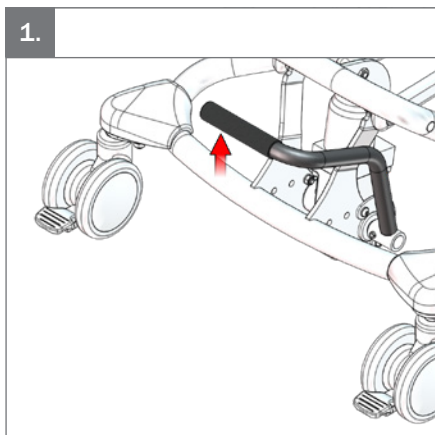1.

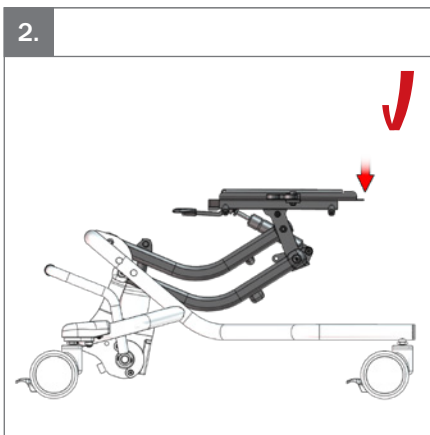
2.



Size 2

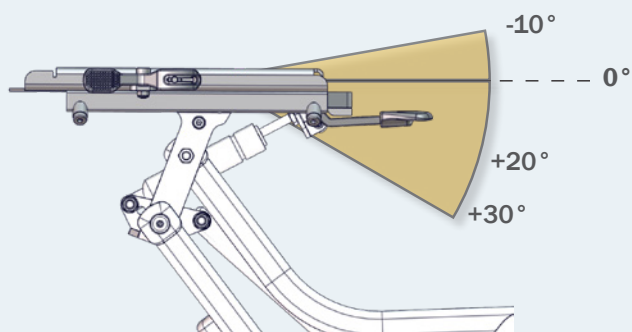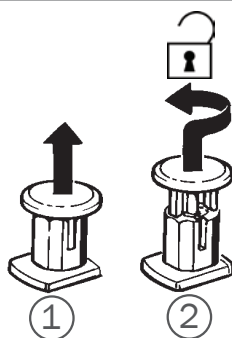
1.

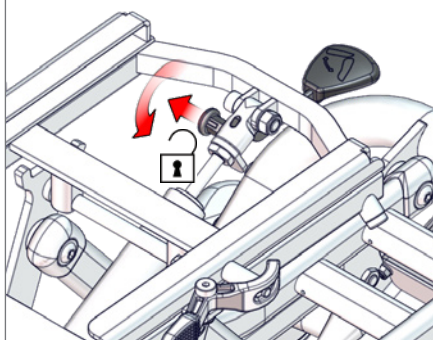
2.

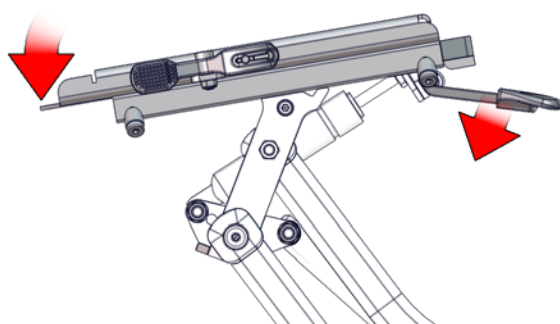
3.

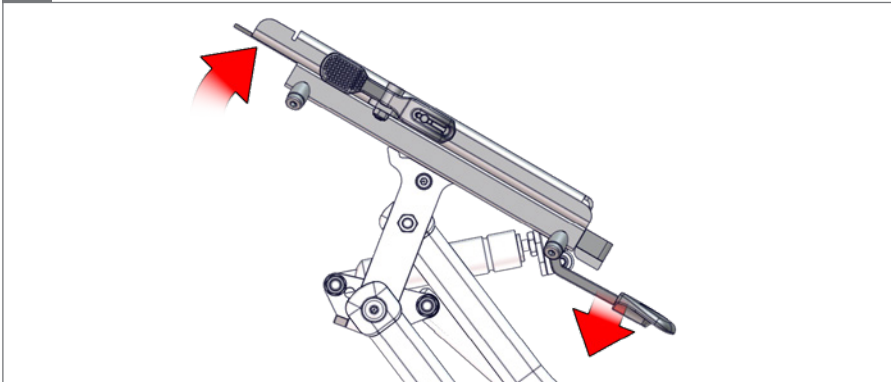
4.

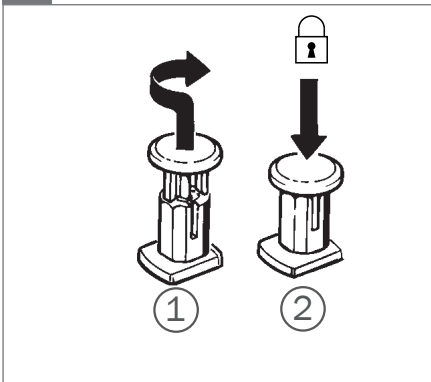5.


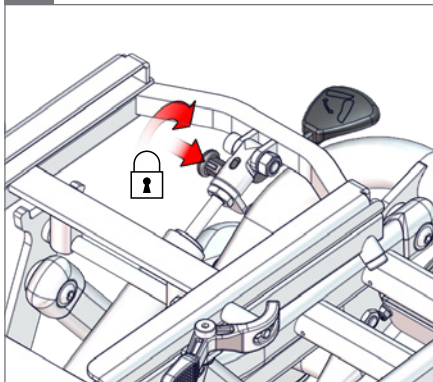
6.



Size 3

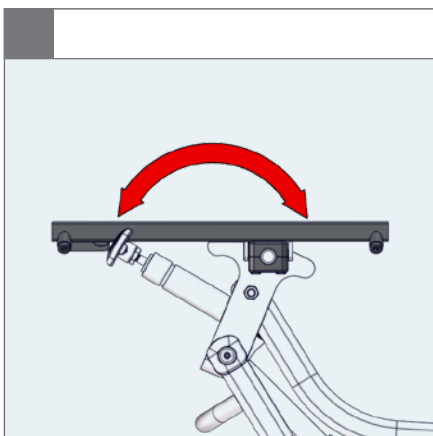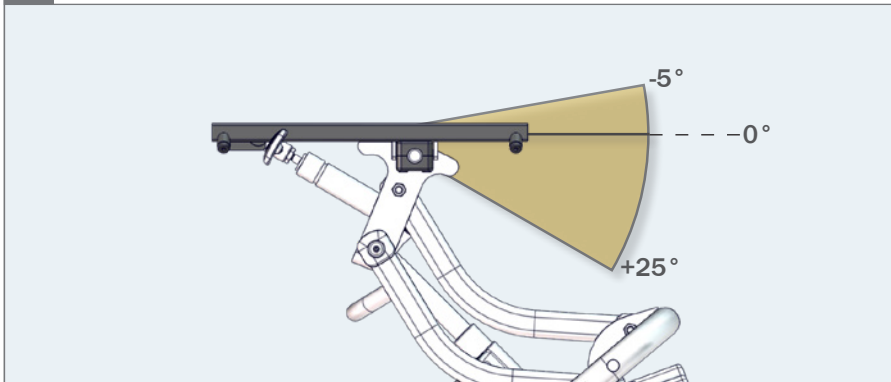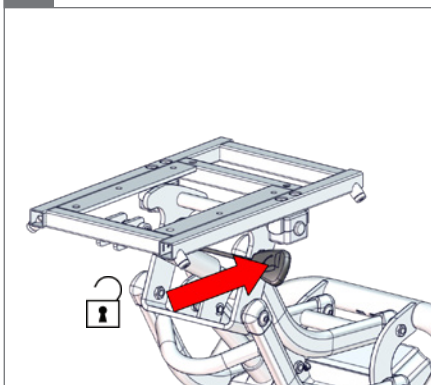
1.

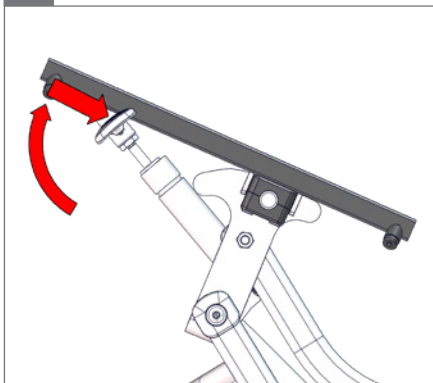2.

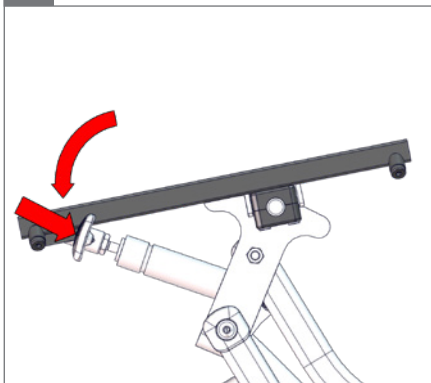
3.



Size 2 & 3

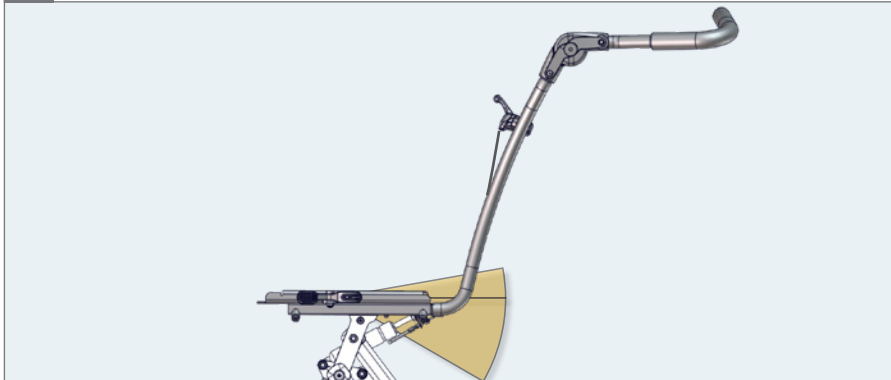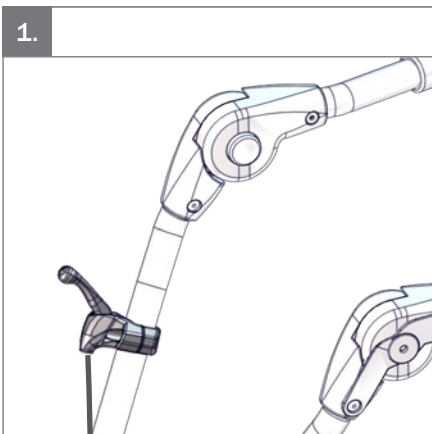
1.

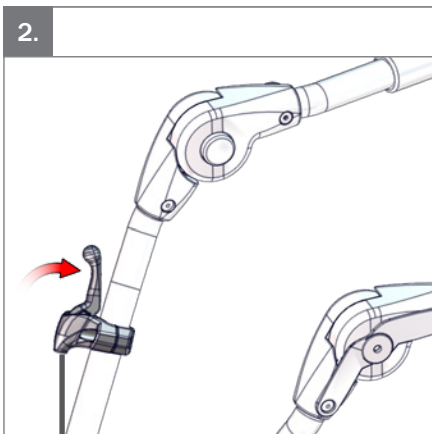
2.

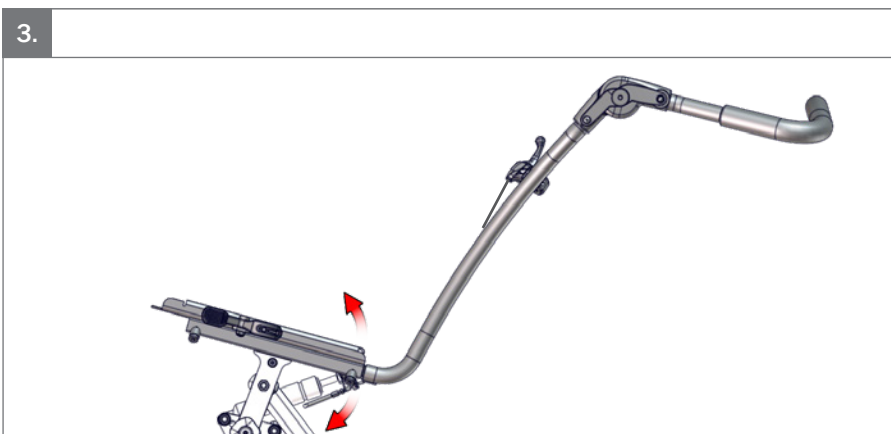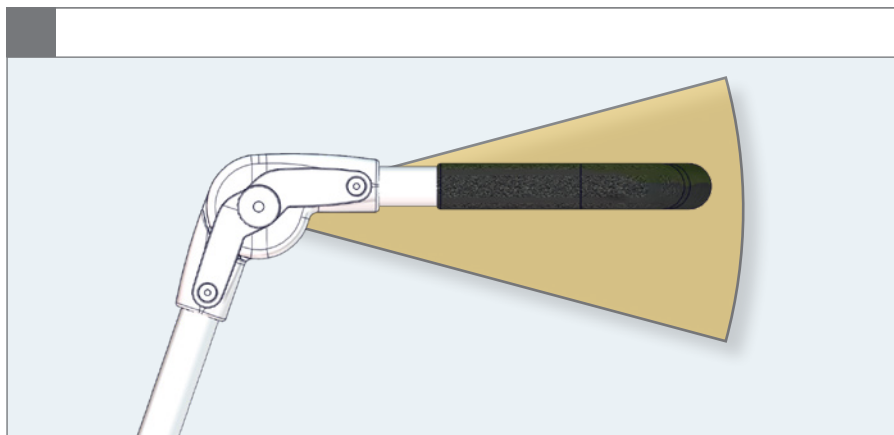
3.

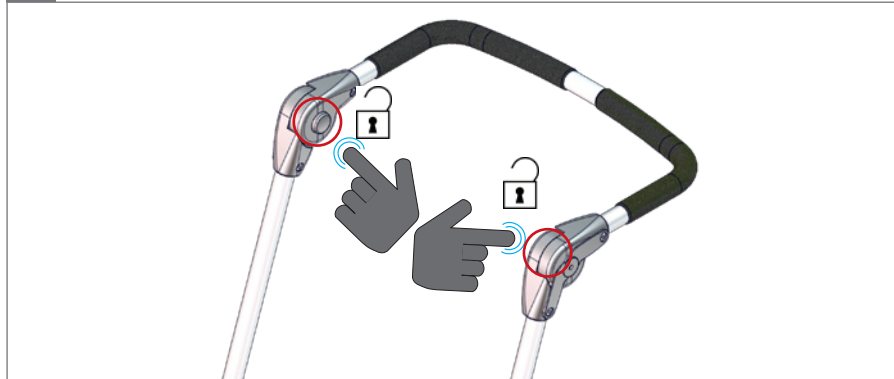1.





2.

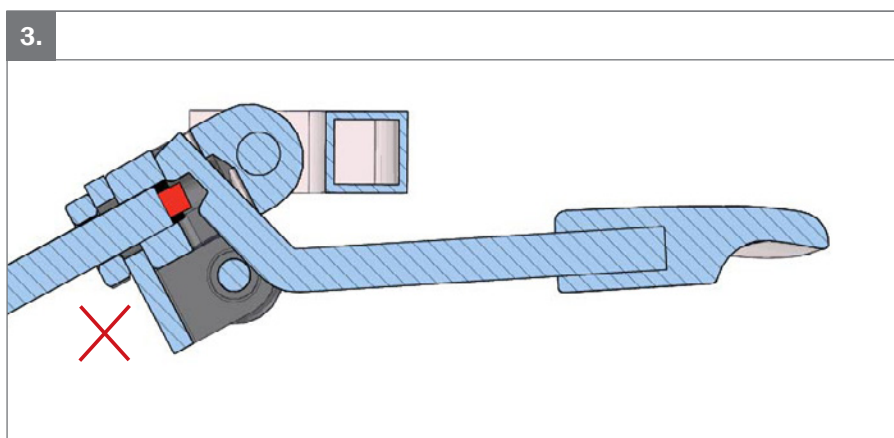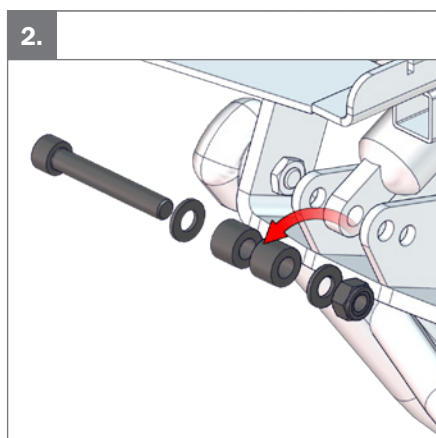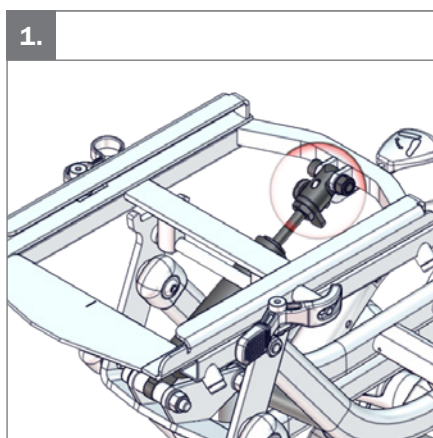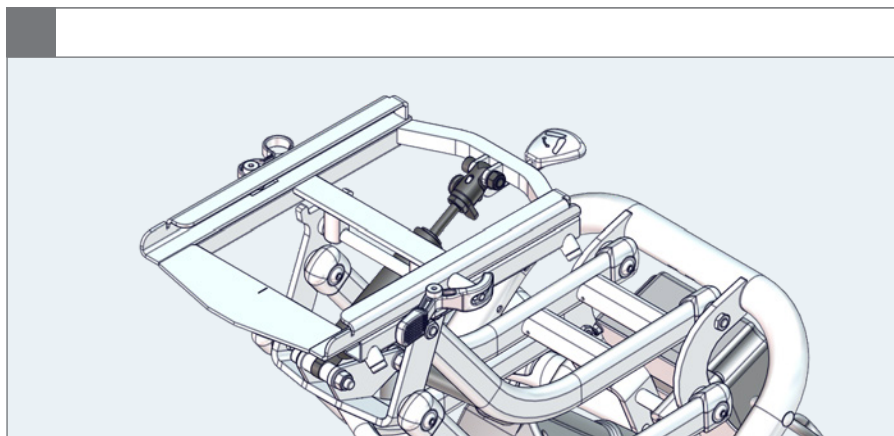
3.


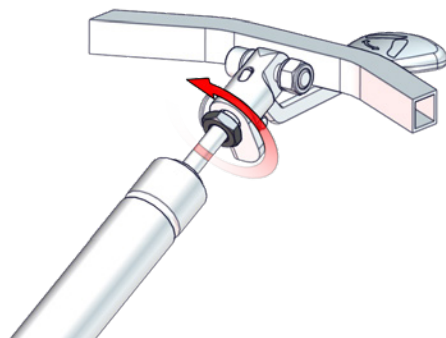
4.

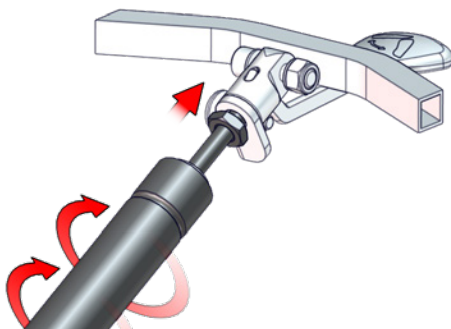
5.

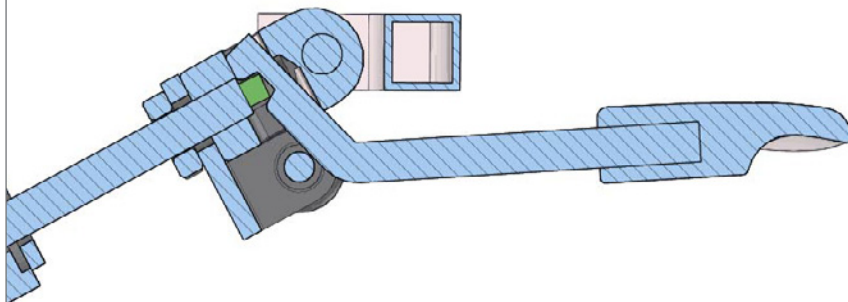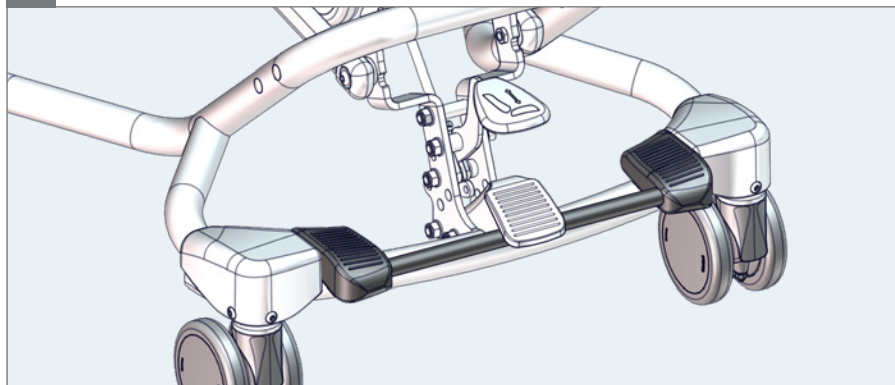
6.


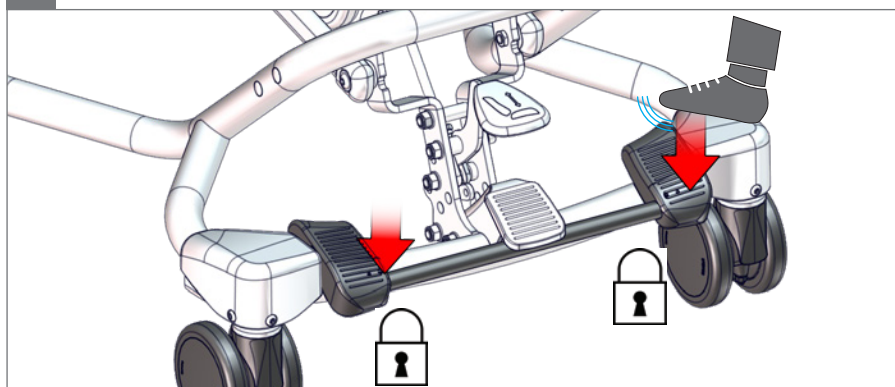
1.

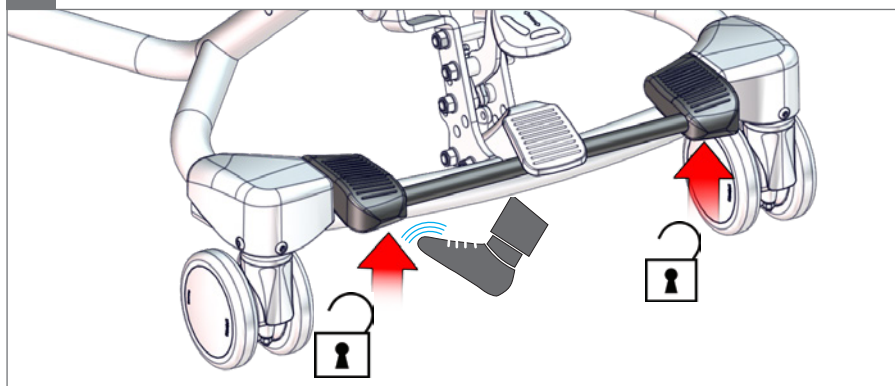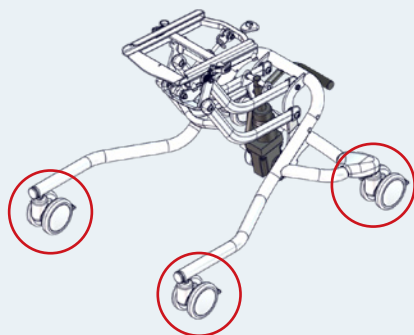
2.

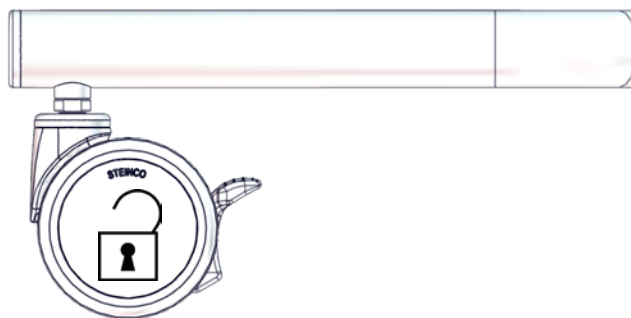
1.

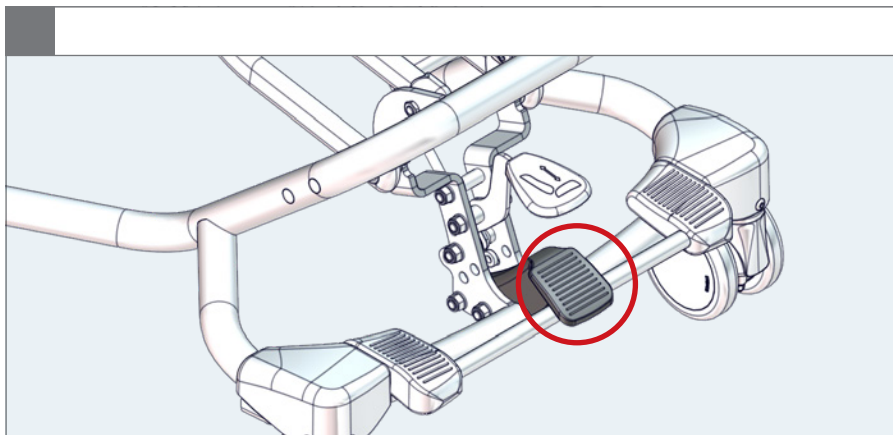
2.

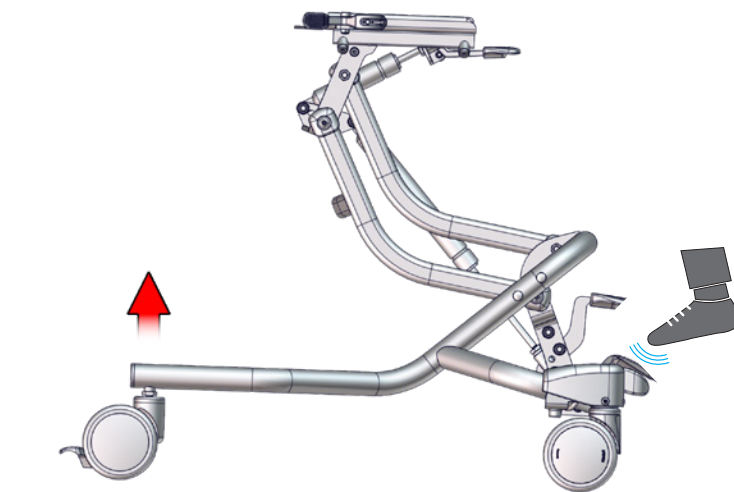
1.


1.a

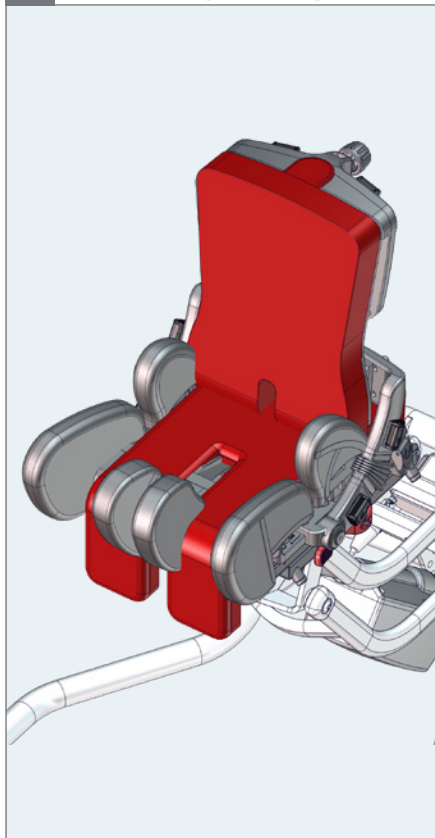Size 2 with x:panda size 3
Size 2 with x:panda shape size 2


1.b

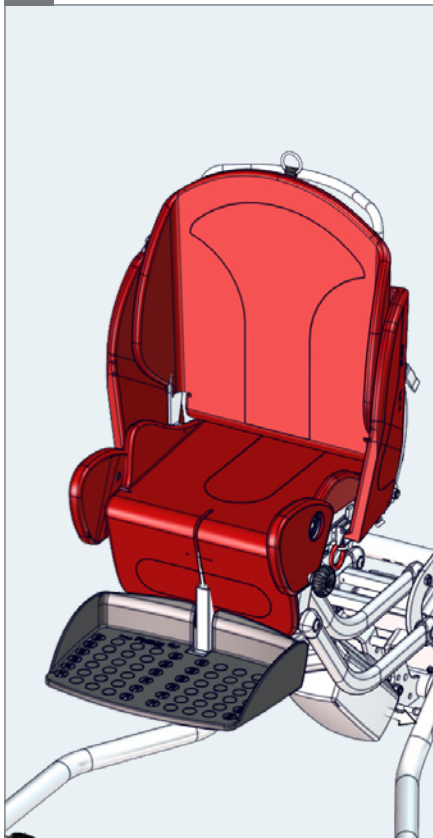
Size 2 with Stingray size 1 & 2

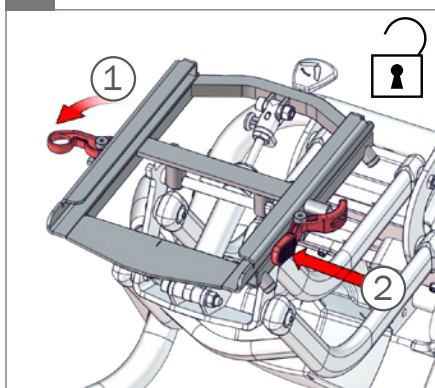2.a

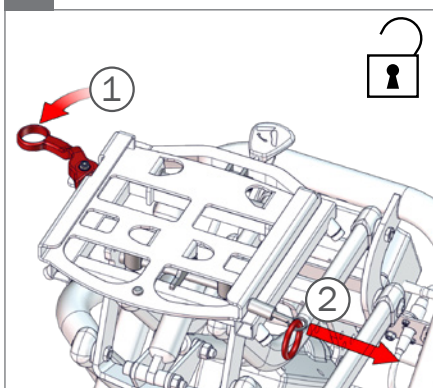
2.b



2.

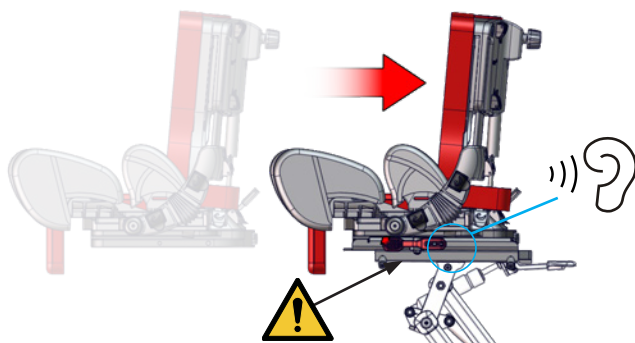⚠ 05

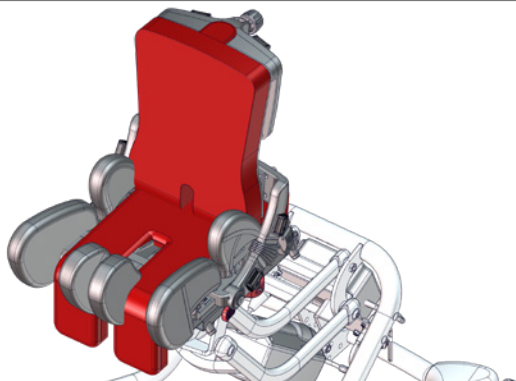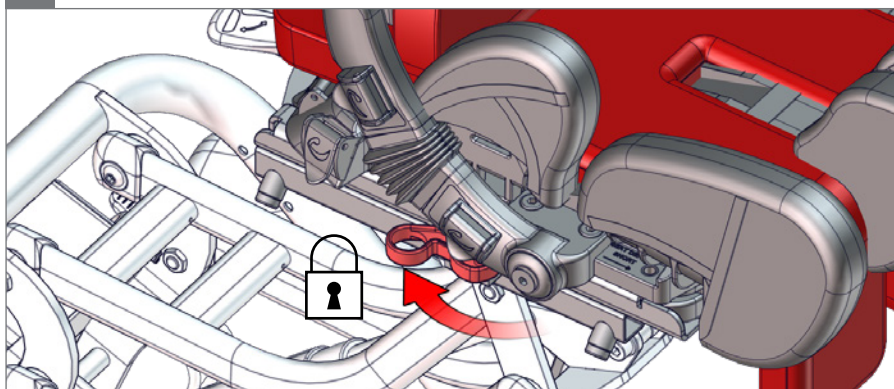
3.


Size 3 with x:panda size 4
Size 3 with x:panda shape size 3
Size 3 with Panda Futura 5

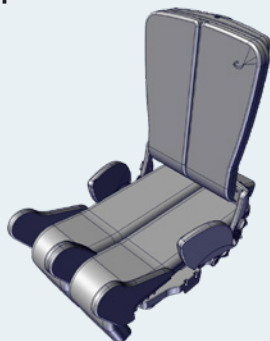x:panda size 4


+

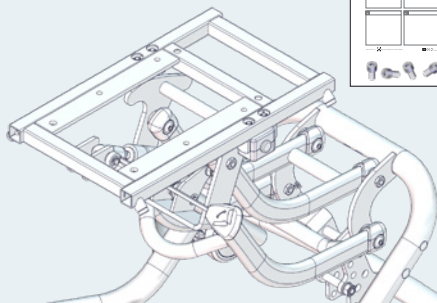x:panda shape size 3

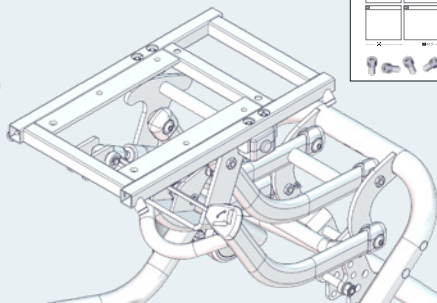
+

Panda Futura 5

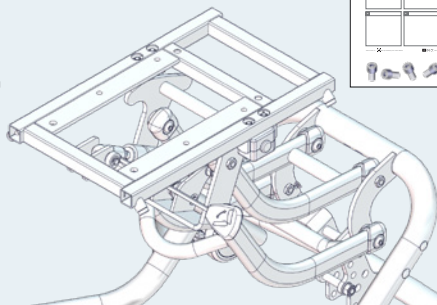
+


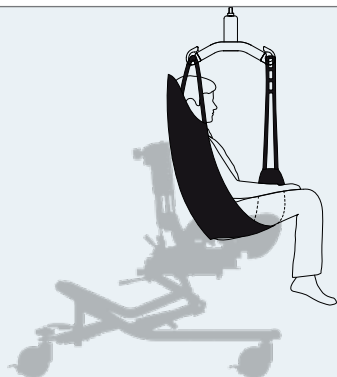
1.

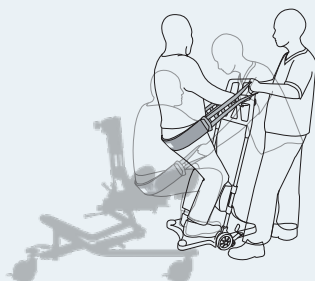
1.

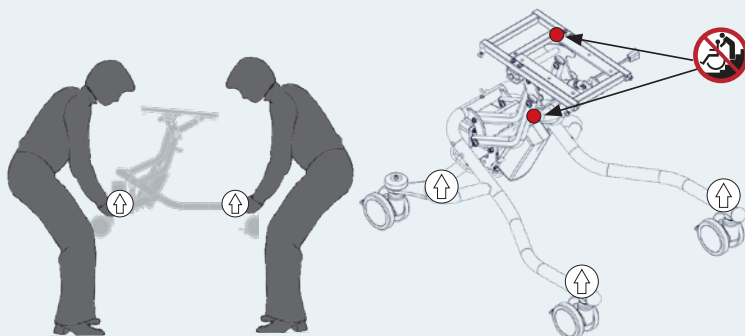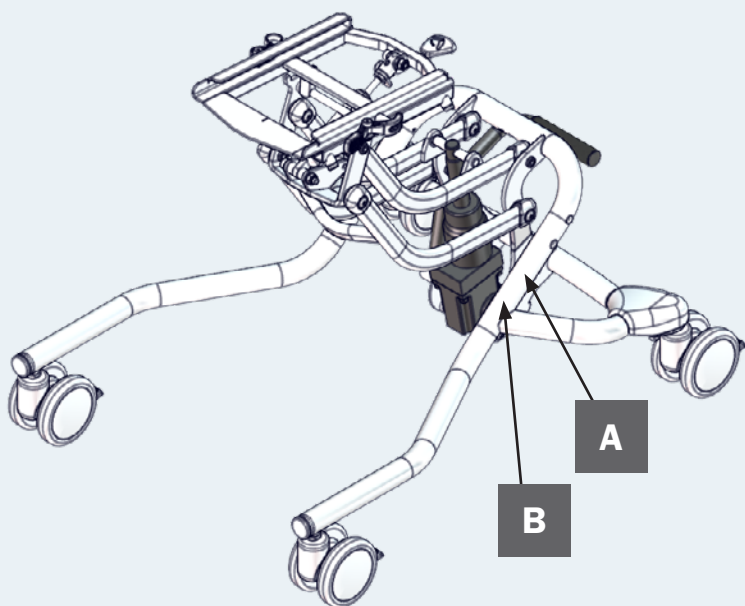1.

A

R82
by Etac

R82 A/S, Parallelvej 3
DK-8751 Gedved
etac.com
Made in xxxxxxx

MD

CE UK CA

YYYY-MM-DD

B

Posting group
Product name, size x

DMR ver.: xxxx

SN xxxxxxxxxxxx

REF xxxxxxxxxxxxxxxx

▲ Max. load: xxx kg
▲ Max. rated load: xxx kg
▲ Mass device: xxx kg

(UEN) (XX)XXXXXXXXXXXX(XX)XXXXXX(XX)XXXXXX

EN: Warning: This symbol appears in the User Guide along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger
01 = Entrapment Hazards
02 = Tilting Hazards
03 = Transportation in motor vehicles
05 = Squeezing Hazard

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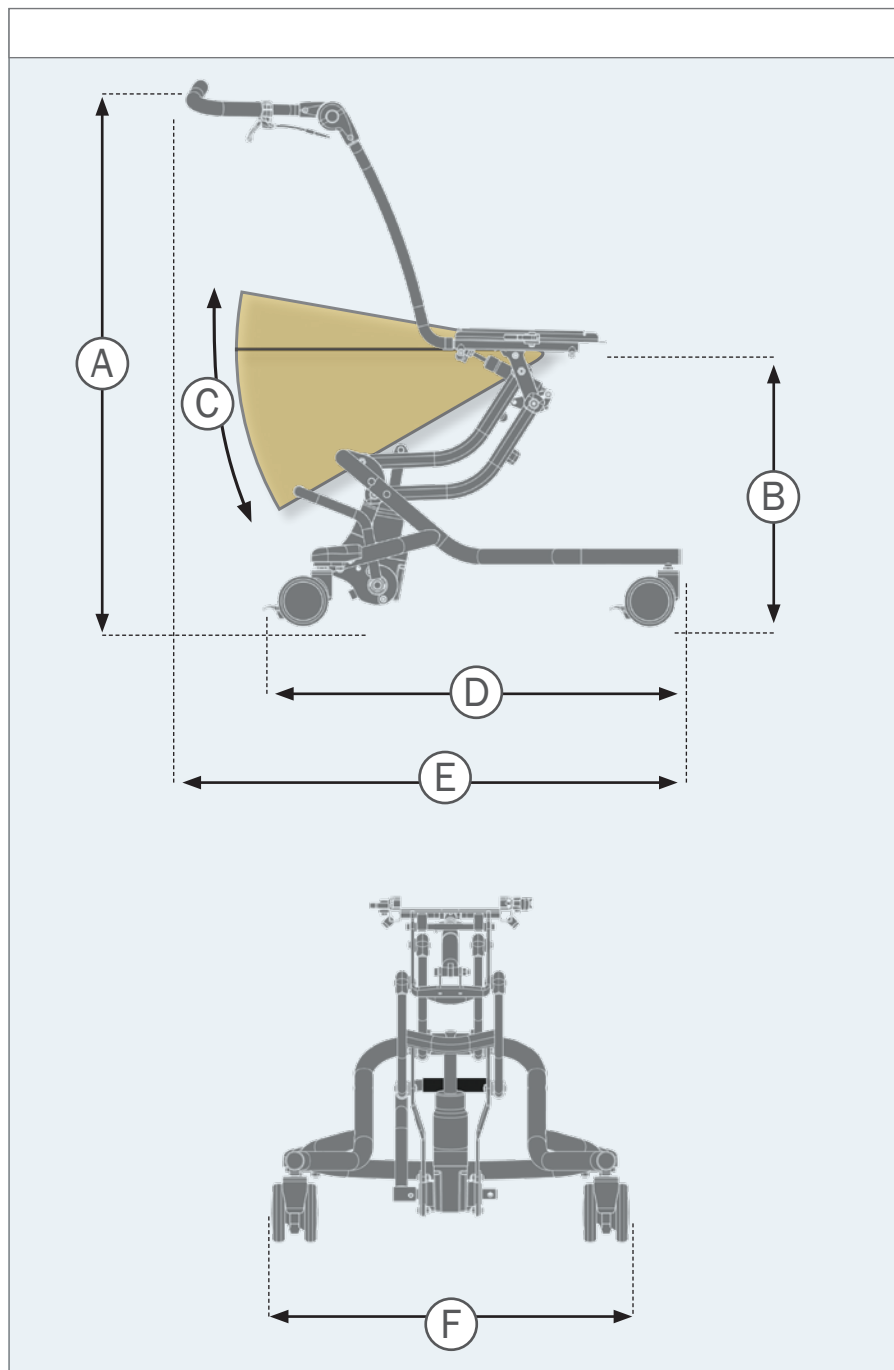
Indoor		2	3
	Overall height (A), to push brace with 100 mm castors	1130 mm (44½")	1130 mm (44½")
	Overall height (A), to push brace with 125 mm castors	1170 mm (46½")	1170 mm (46½")
	Seat surface height at front edge (B) (adapter plate) with 100 mm castors	240–600 mm (9½–24")	300–540 mm (11¾–21¾")
	Seat surface height at front edge (B) (adapter plate) with 125 mm castors	280–640 mm (11–25¼")	340–580 mm (13½–22¾")
	Seat plane angle (adapter tilt) (C)	(-10°) – (30°)	(-5°) – (25°)
	Length (D)*	795 mm (31¼")	795 mm (31¼")
	Overall length (E)	830 mm (32¾")	830 mm (32¾")
	Overall width (F)	620 mm (24¾")	620 mm (24¾")
	Castors diameter	100/125 mm (4/5")	100/125 mm (4/5")
	Minimum turning radius	475 mm (18¾")	475 mm (18¾")
	Max load	75 kg (165 lb)	100 kg (220 lb)
	Mass device*	19 kg (42 lb)	19 kg (42 lb)
	Max rated load*	96 kg (212 lb)	121 kg (267 lb)
	Static stability Sideways, downhill and uphill	10°	10°

EN: *Measurements: The High-low:xo measurements above are based on the use of 100 mm castors with the frame in the most basic configuration. Different settings may result in another set of measurements. As we supply various types of accessories, the weight is listed without.

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ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this user manual before use and save it for future reference.

Intended purpose

The High-low:xo is a frame for a seating system for children and adults with disabilities.

Indications

The device is used together with a seating system (i.e. a modular seating system like R82 x:panda or R82 Panda Futura). It can be used by children and adults with disabilities, who lack the ability to maintain a seated position and move around (within the room or between rooms) without an assistive device. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age.

The user is the person sitting in the device. The caregiver is the person operating the device. Never leave the user unattended in this device. Ensure permanent supervision by a caregiver.

The use of the device will be discontinued more times during the day where the user will be lying down, standing or sitting in another assistive device or a conventional product.

Application areas

The product is for indoor use

Contra indications

None

Precautions

None

Enclosed tools

None

CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- High-low:xo indoor: EN 12182

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements

may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Alignment of user weight/mass of the combined seat and base.
- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety.

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

For information about product safety notices and product recalls, local dealer information are available at the website etac.com

Caregiver

- Read the user manual carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user.
- Always use the correct lifting techniques and aids for this purpose.
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver.
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier.
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user guide.
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible.
- When adjusting the seat angle, keep a hand on the seat mounted on the frame to ensure slow angling.
- Ensure that moving parts does not squeeze or trap body parts when adjusting them.
- Ensure that body parts are not trapped in openings for mounting of supports/ accessories.
- Always use the product in a moderate pace. Do not run with the product.
- Information regarding frame and seat combinations including max. load in combination can be found at: etac.com.
- This product contains small parts which could present a choking hazard if removed from their intended placing.

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use.
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces.

- When using on inclines, please refer to measurements.

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered.
- Do not place more than one user in the product.
- When changing user, clean the product and disinfect it if necessary. Carry out necessary adjustments after cleaning.

Product

- Removable parts without use of tool: Foot support and seat. The removal will have adverse effect on the device.
- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children.
- Activate the brakes before placing the user into the product.
- Ensure the wheels are fully-functional and secure before use.
- Ensure the stability of the product is maintained before placing the user in it.
- When the footplate is close to the floor and the castors, the castors may be unable to swivel.
- If the frame has to be used in the lowest position, remove the footrest before adjustment.
- Inspect the product and all of its accessories and replace any worn out parts before use.
- The user must not occupy the product when it is lifted from the ground
- Unoccupied device is suitable for land and air transport.

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Reuse

This product and its accessories/components are suitable for reuse.

Refurbishment for reuse

Before reuse or changing the user, follow the service checklist, maintenance and washing instructions and carry out regular inspections.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

Service checklist:

Check and adjust the following as per their intended use:

- Frame is adjustable in height
- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment
- Wheels are moving freely and that all wheel locks/brakes are easily activated. Adjust if required
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact.

Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to:

Wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use. Carry out a visual inspection for any damaged or worn-out parts.

Washing

Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Materials

- Rubber
- Plastic
- Steel

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized.

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**Etac Ltd.**
**Gelbart AG**