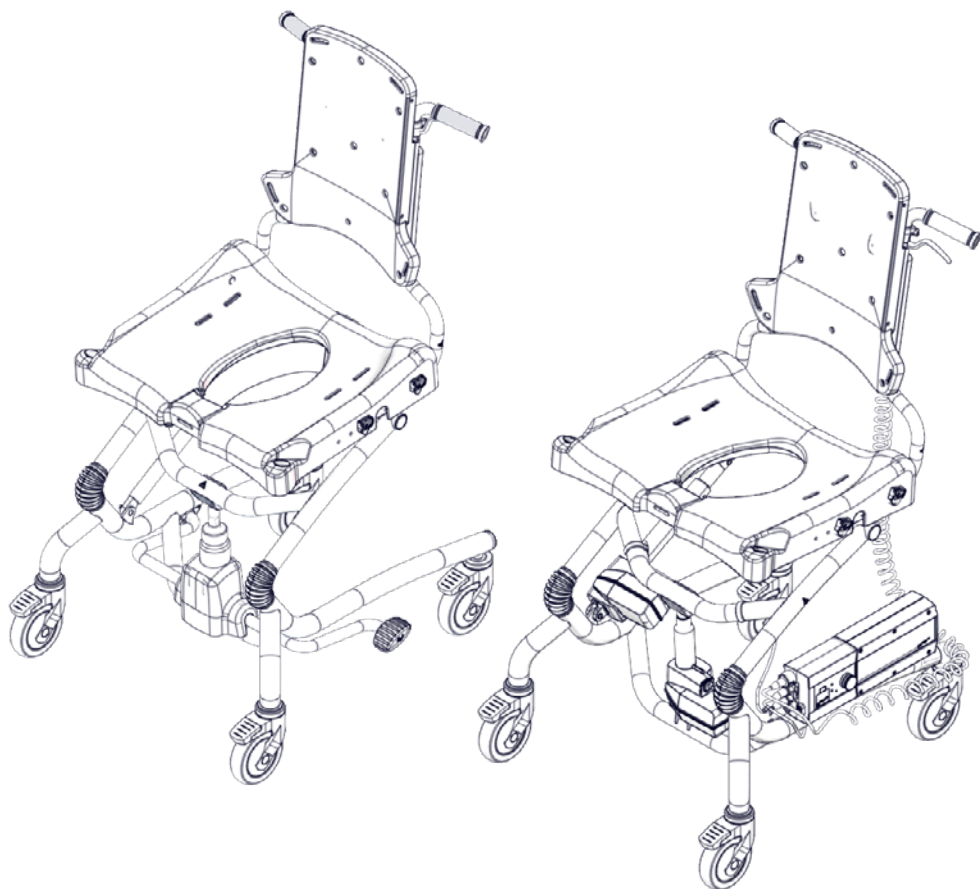
User manual

Heron

R82[®]
by Etac

M1350 - 2024-02-21 - Rev. 22

www.etac.com


	Hydraulic	Power		
	3	13	ENGLISH DEUTSCH	40 - 48 49 - 57
		14	NEDERLANDS DANSK	58 - 66 67 - 75
	4 - 5	15	NORSK SVENSKA	76 - 84 85 - 93
	6	16	SUOMI ESPAÑOL	94 - 102 103 - 111
	7	17	FRANÇAIS ITALIANO	112 - 120 121 - 129
	8	18	PORTUGUÊS PORTUGUÊS DO BRASIL	130 - 138 139 - 147
	9	19	РОССИЯ POLSKI	148 - 157 158 - 166
	10	20	ČESKY 中國	167 - 175 176 - 183
		21	日本 ΕΛΛΗΝΙΚΗ	184 - 193 194 - 202
		22 - 23	ÍSLENSKA	203 - 210
	11	24		
	12	25		
		26 - 36		
mm (inch) kg (lb)	37			
		38 - 39		


A.



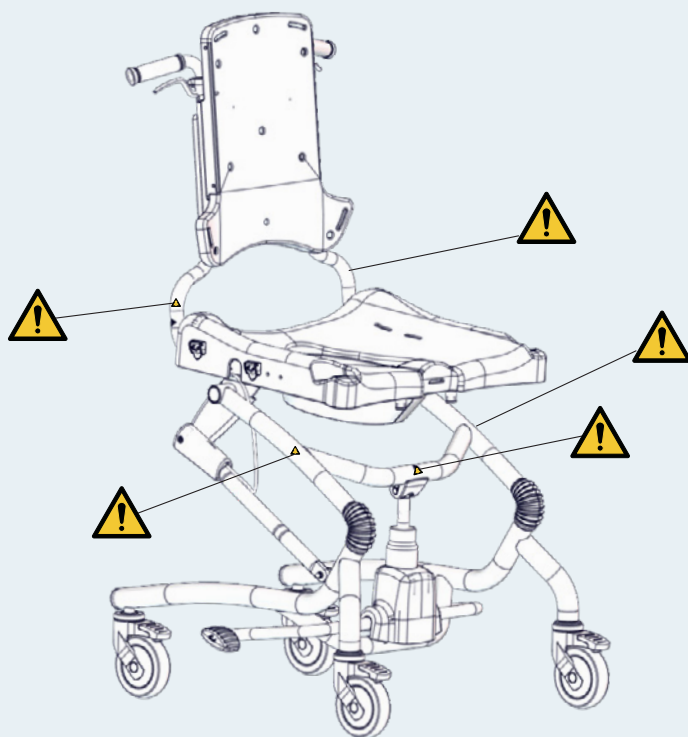
⚠ 05





B.

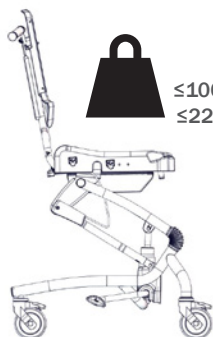
≤100 kg
≤220 lb

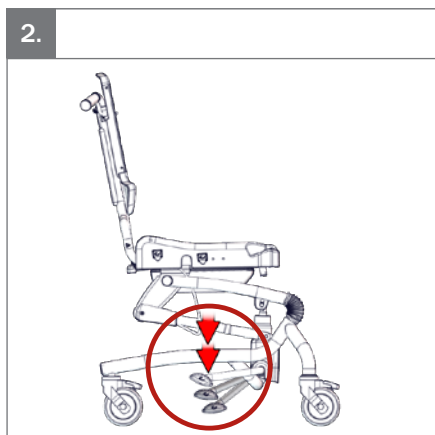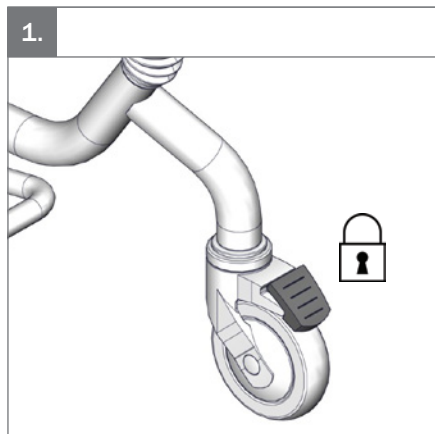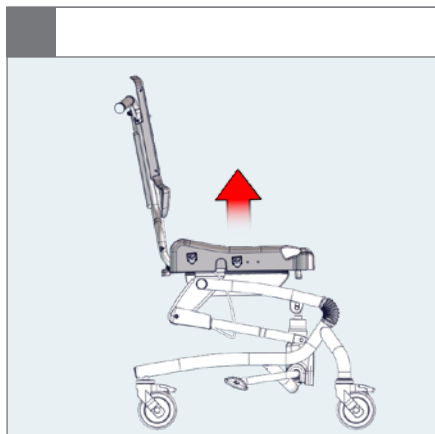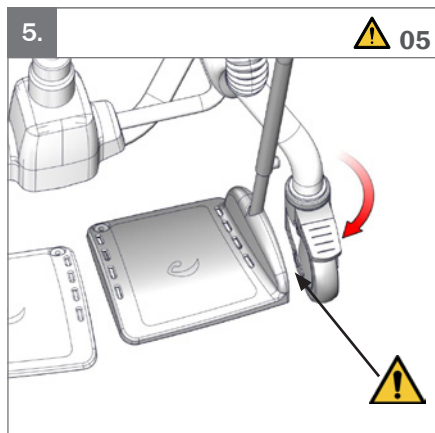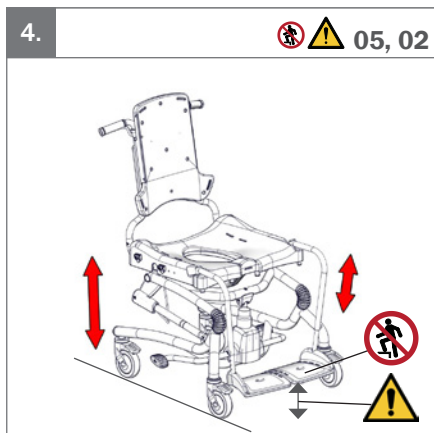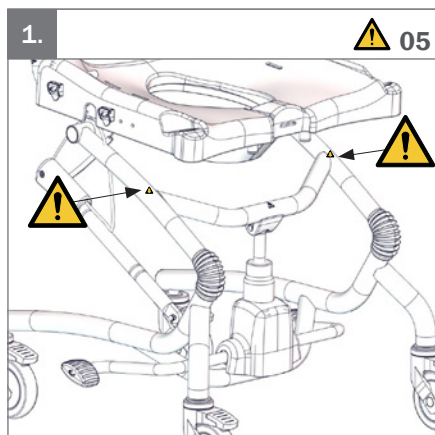
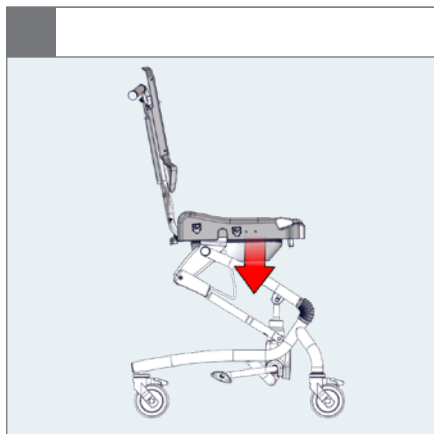
(-5°) -> (25°)

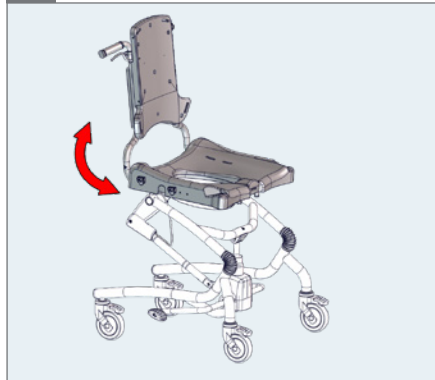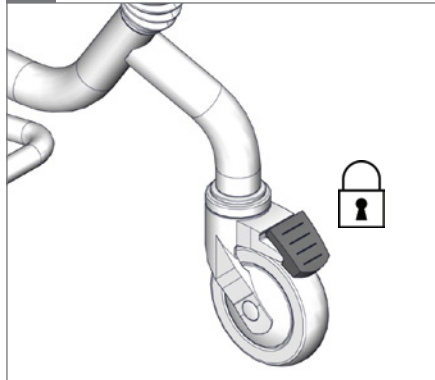
1.

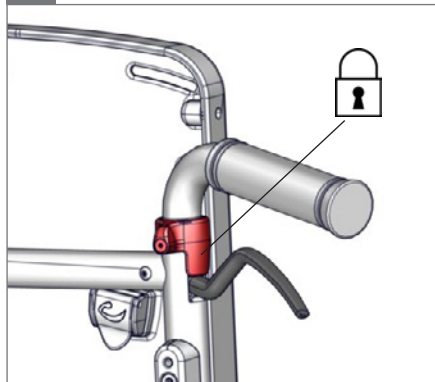
2.

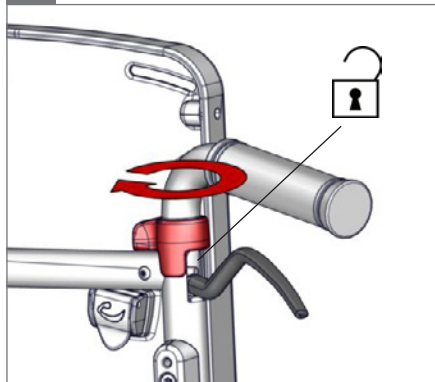
3.

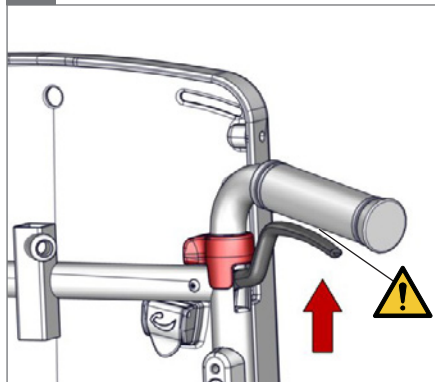4.

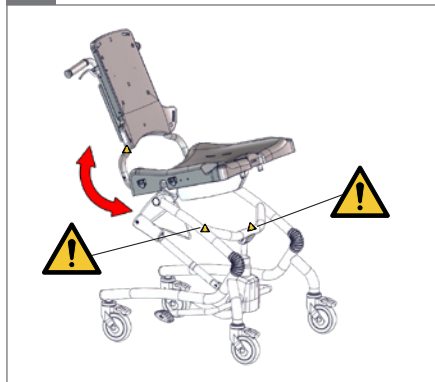
5.



(-10°) -> (30°)

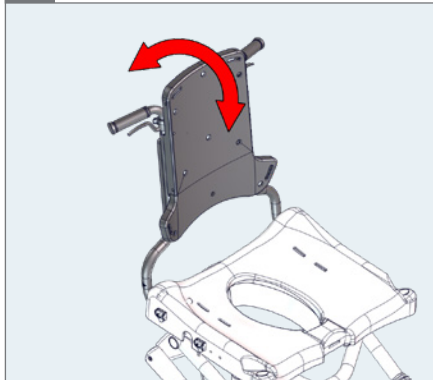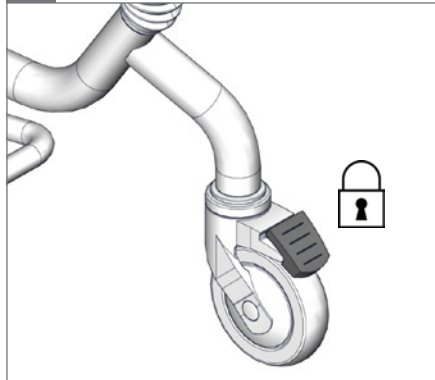
1.

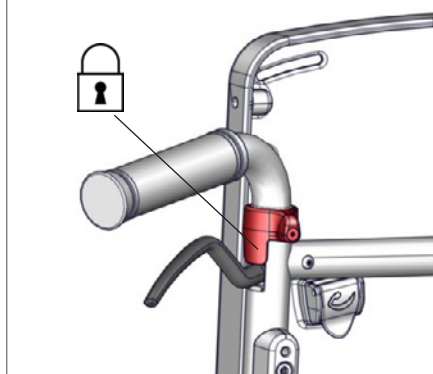2.

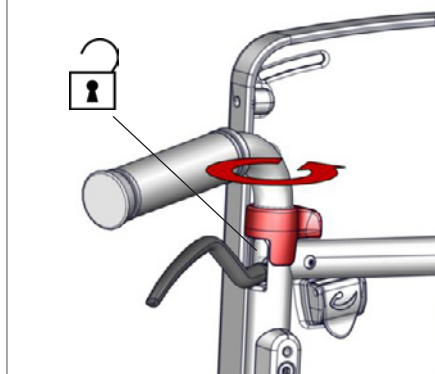
3.


4.


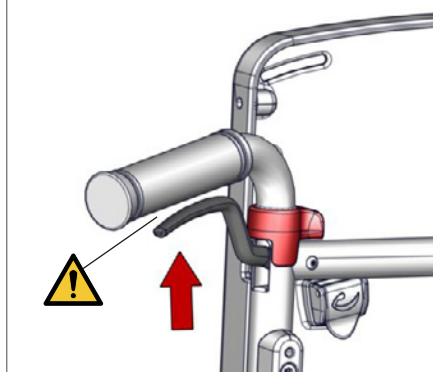
⚠ 05

5.

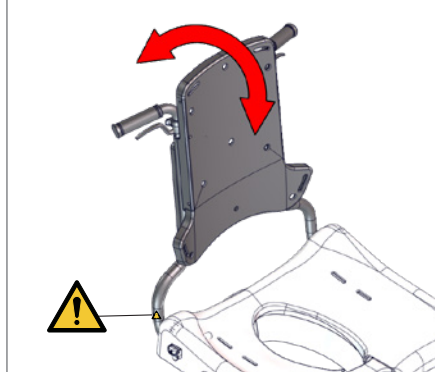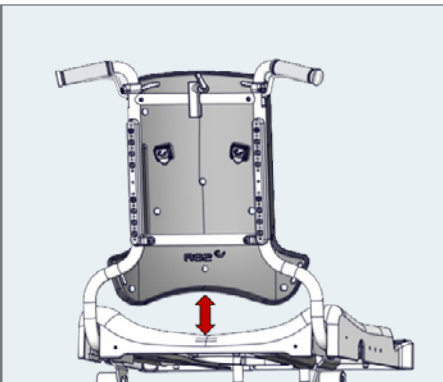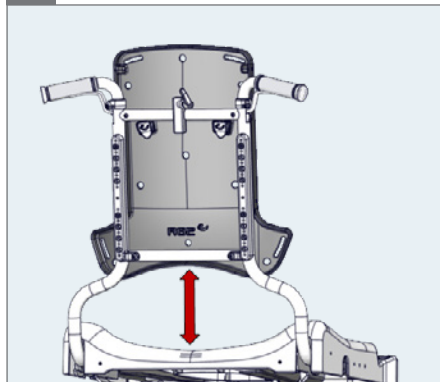
⚠ 05


1.

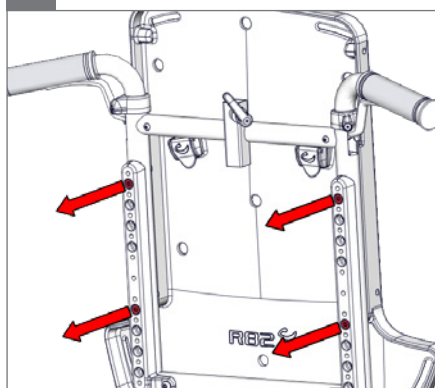○ 5 mm

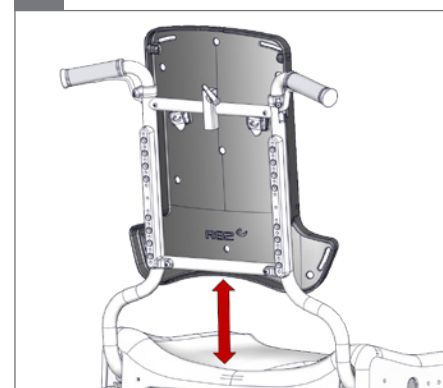
2.

3.

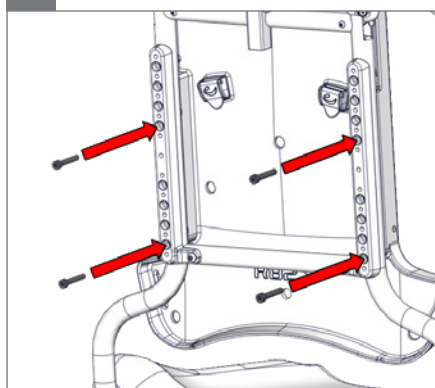○ 5 mm - 2,5 Nm



 125 mm

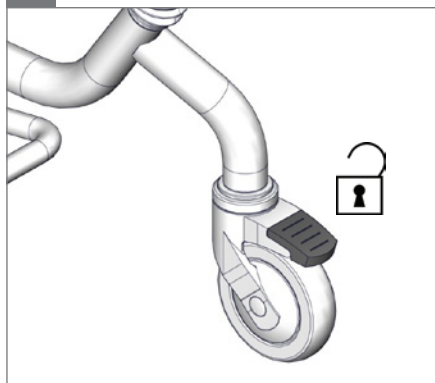
1.

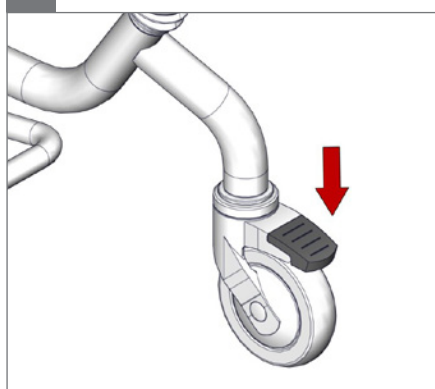
2.

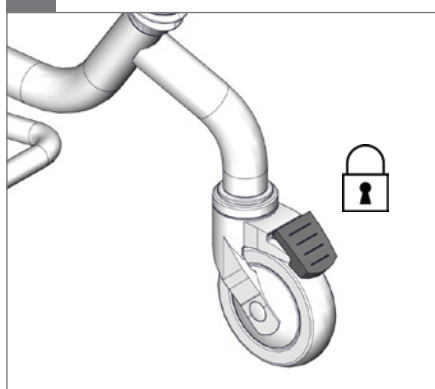
3.

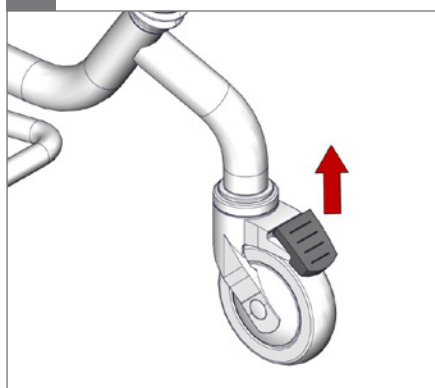
4.

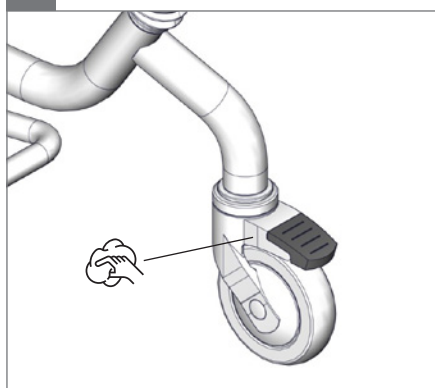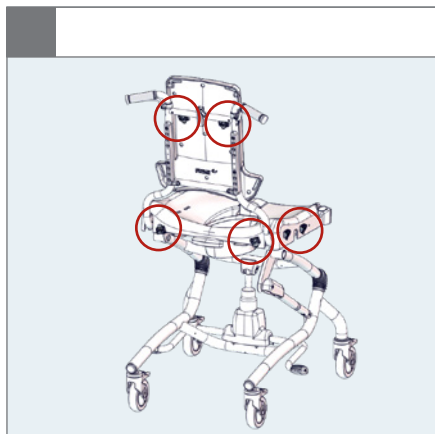
5.


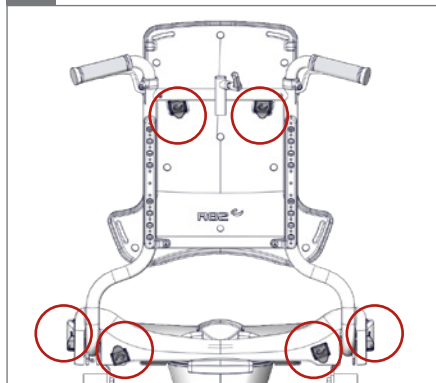
1.

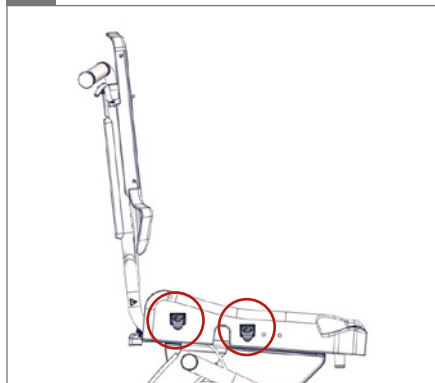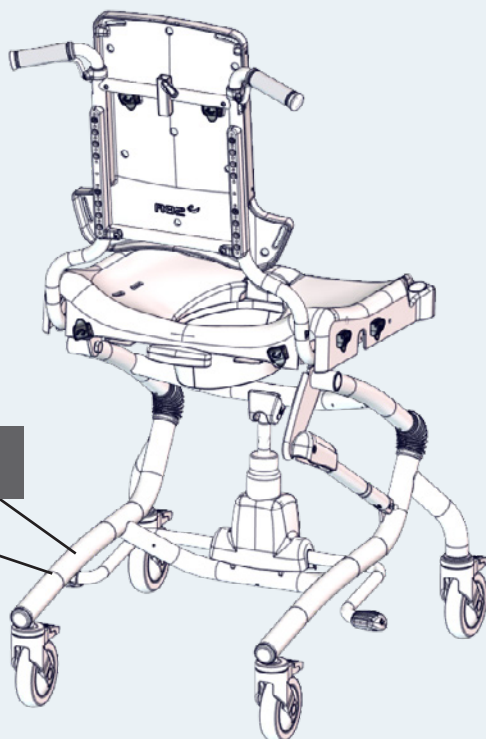2.

A

R82
by Etac

R82 A/S, Parallelvej 3
DK-8751 Gedved
etac.com
Made in xxxxxxx

MD

CE UK
CA

YYYY-MM-DD

B

Posting group
Product name, size x

DMR ver.: xxxx

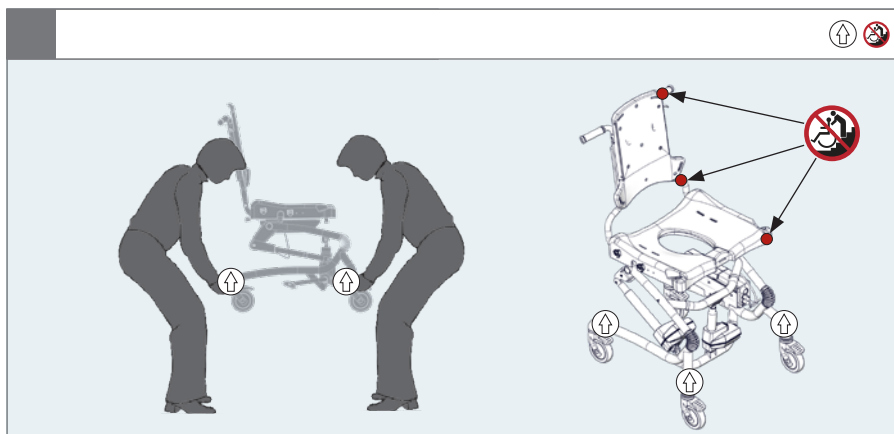
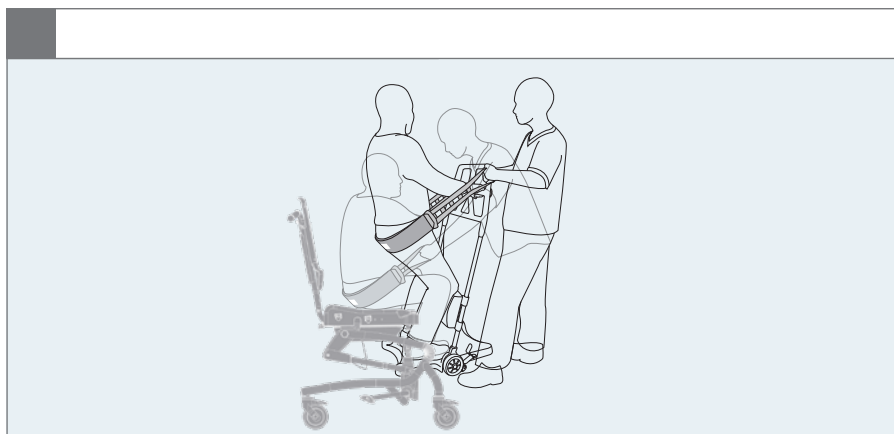
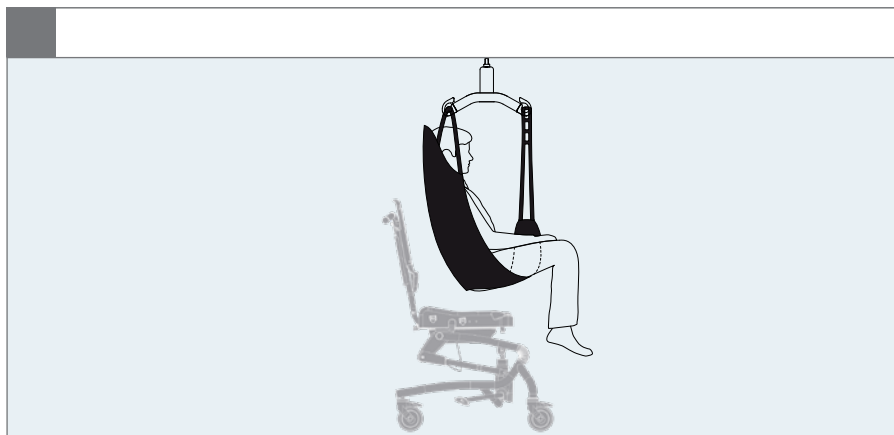
Max. user mass: xxx kg
Max. rated load: xxx kg
Mass device: xxx kg

xxxxxx

xxxxxx

xxxxxx


xxxxxx


A.

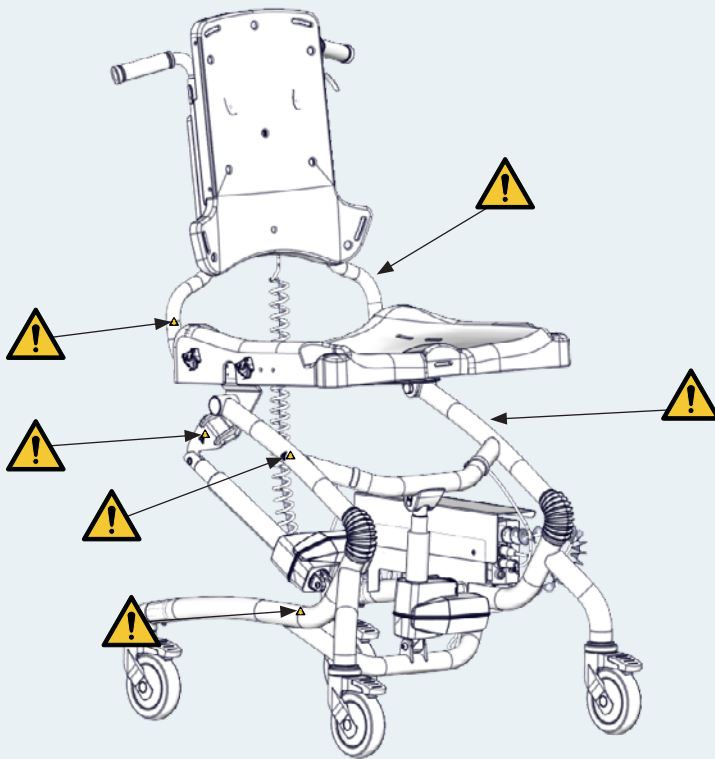
05


B.

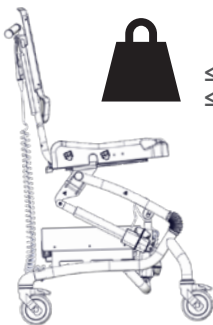≤ 130 kg
≤ 286½ lb

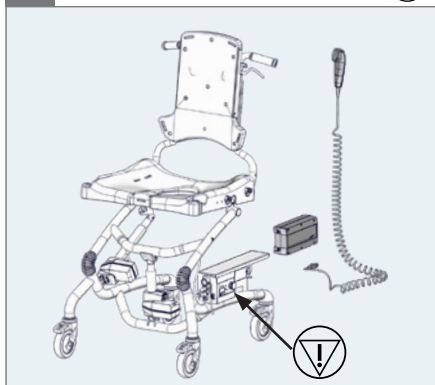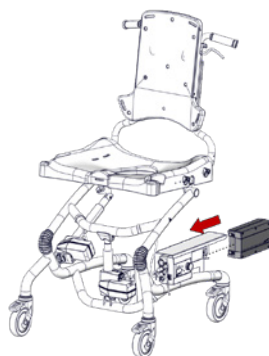
1.

2.



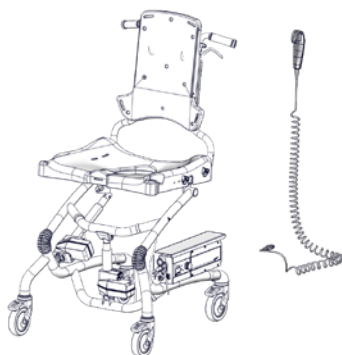
3.

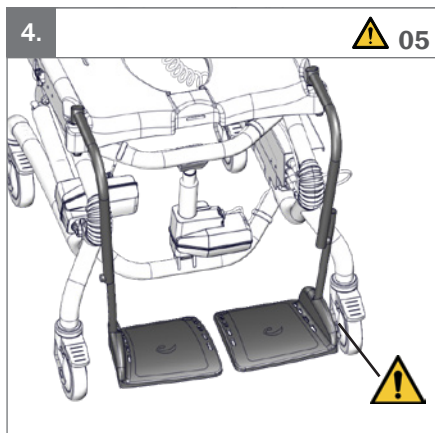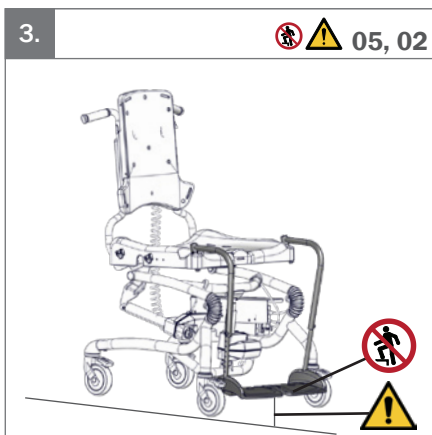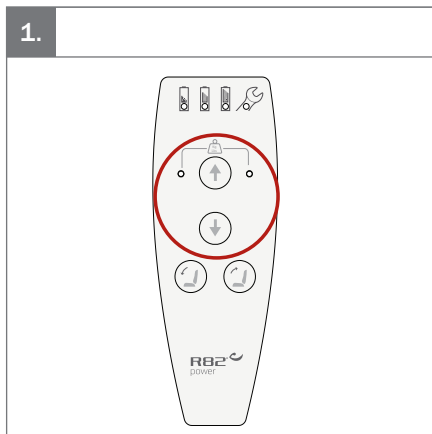4.



(-5°) -> (25°)

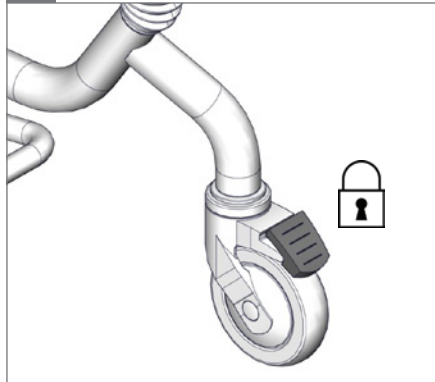1.

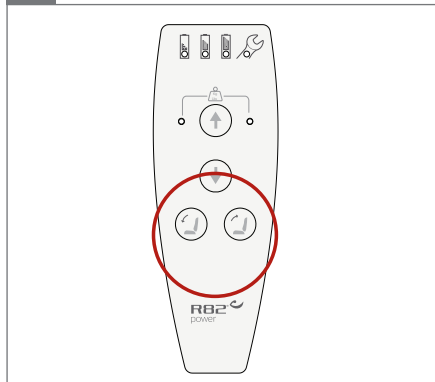2.

3.

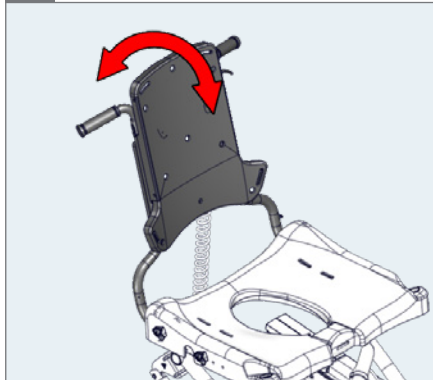
4.



(-10°) -> (30°)

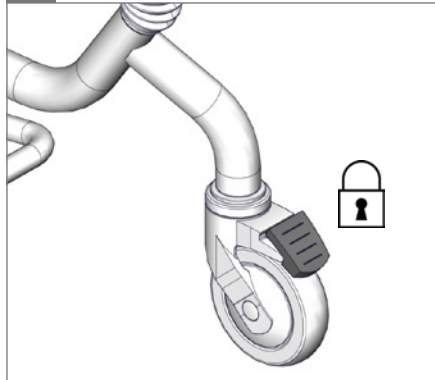
1.

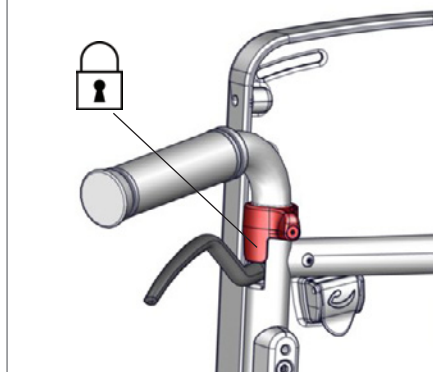
2.

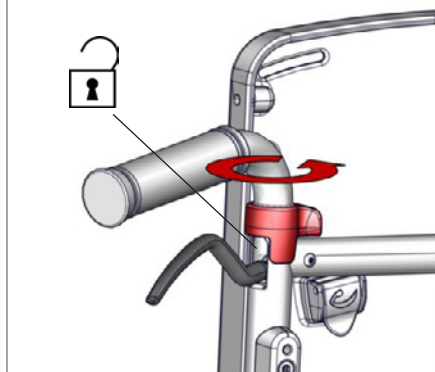
3.


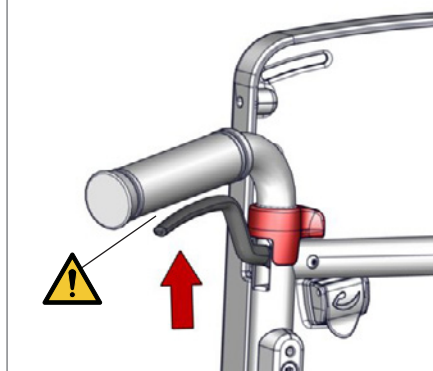
4.

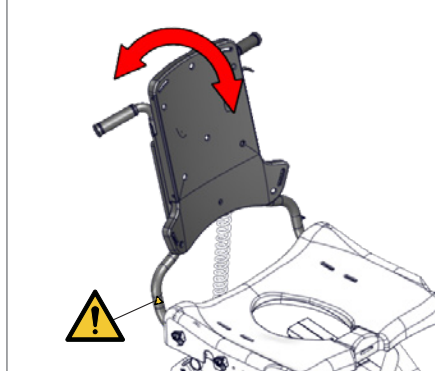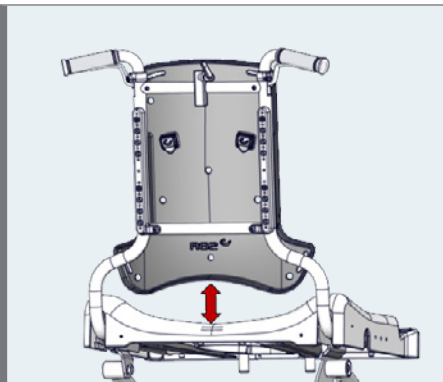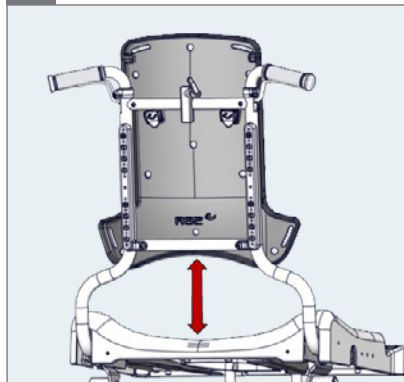
5.


1.

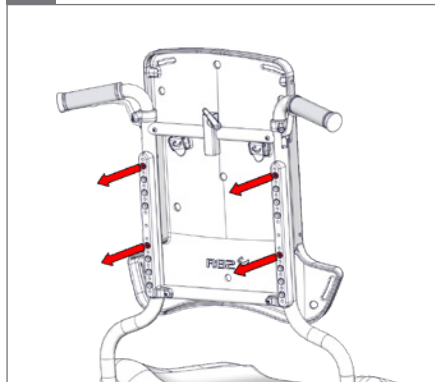○ 5 mm

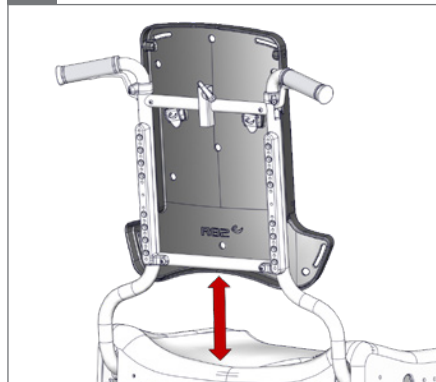
2.


3.

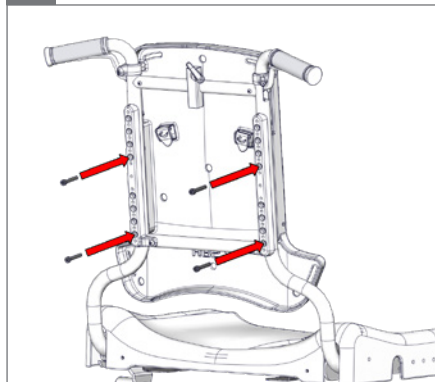○ 5 mm - 2,5 Nm



 125 mm

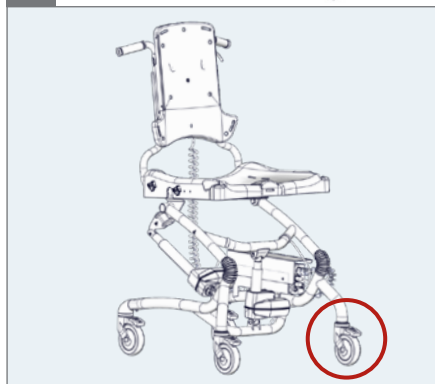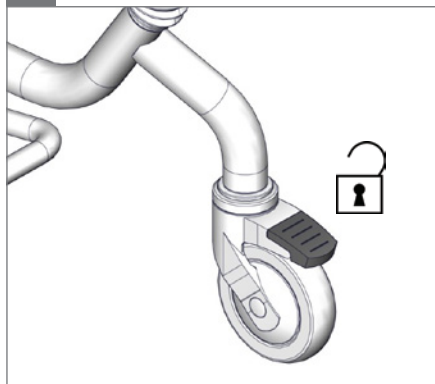
1.

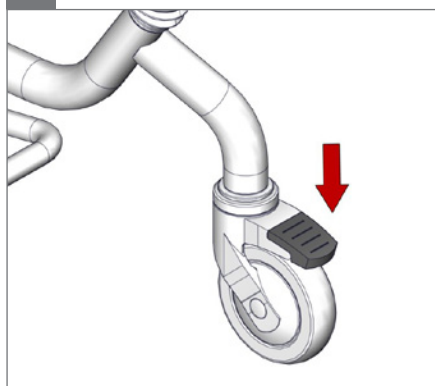
2.

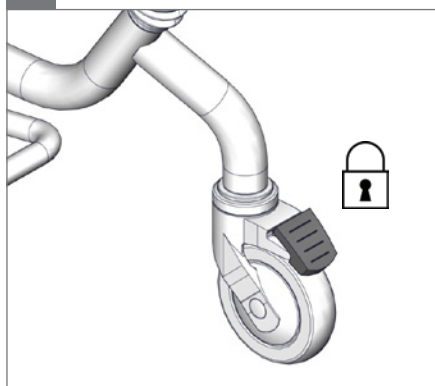
3.

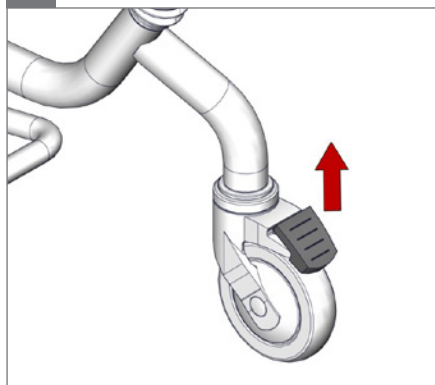
4.

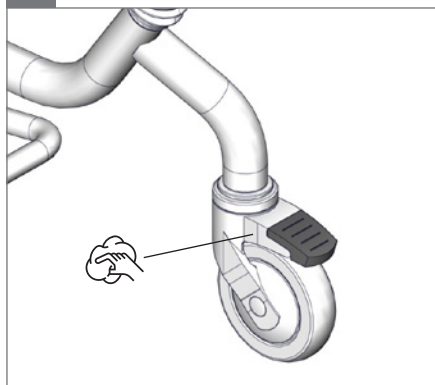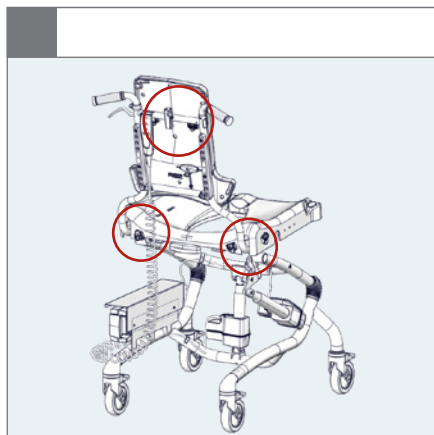
5.












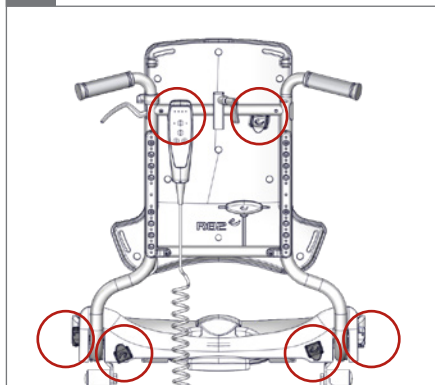
1.

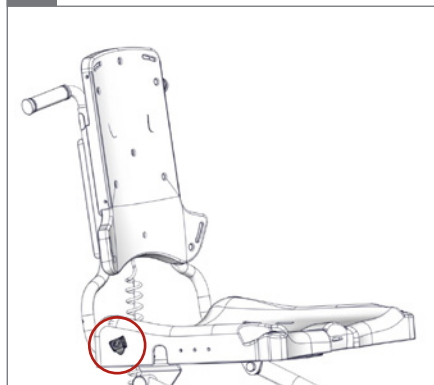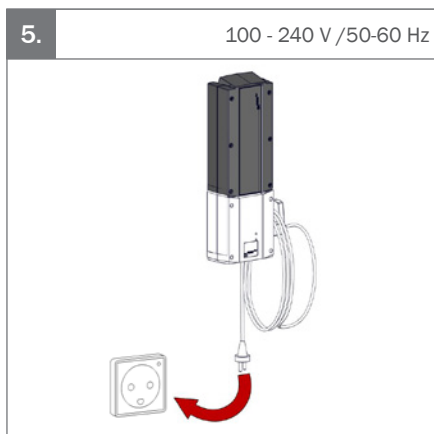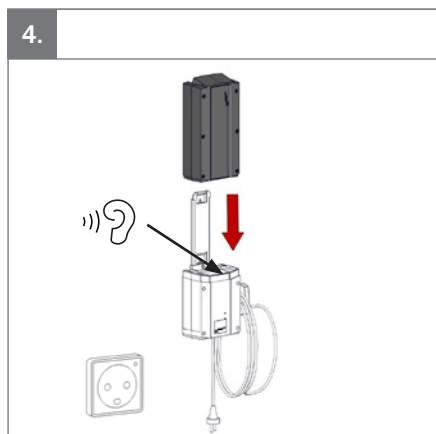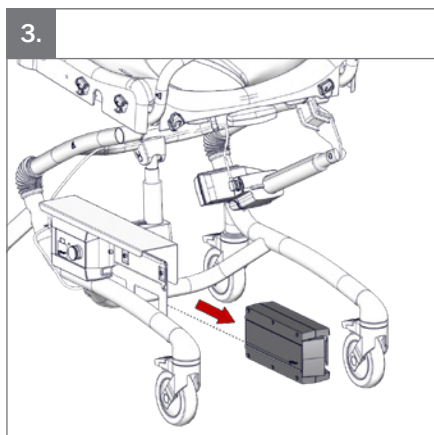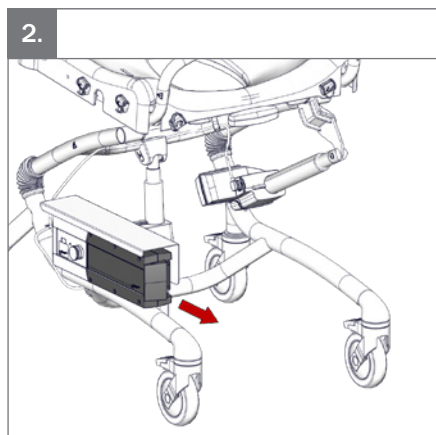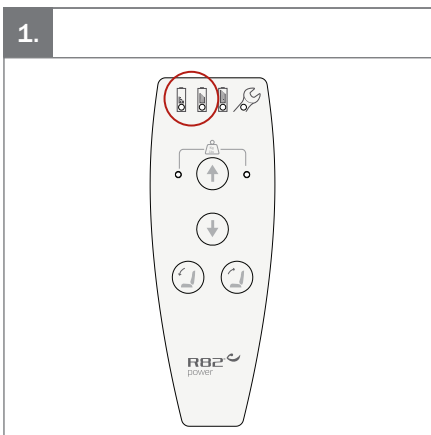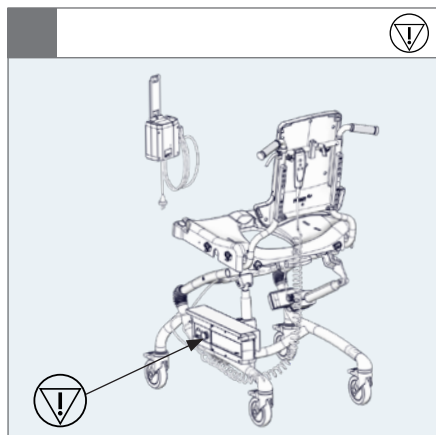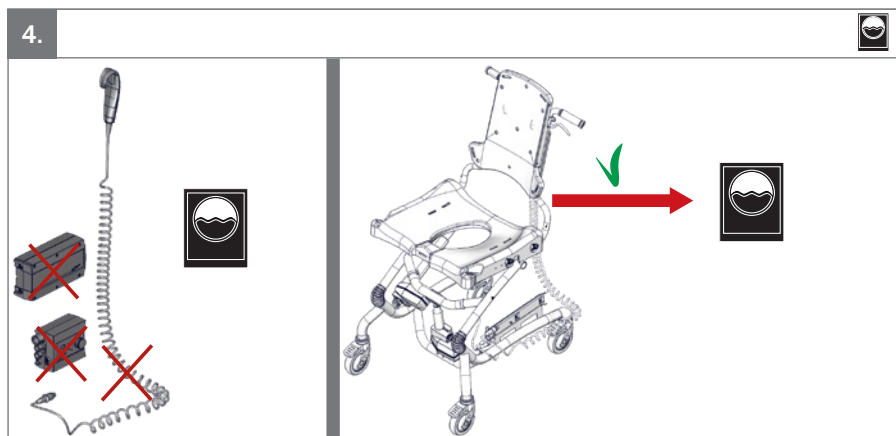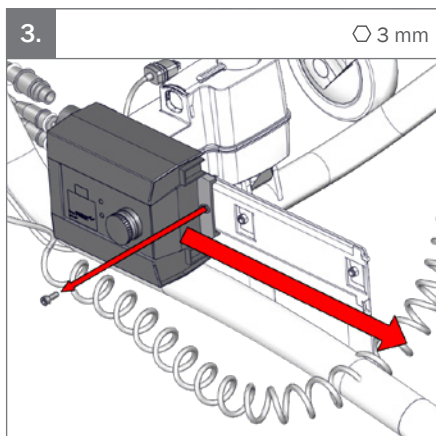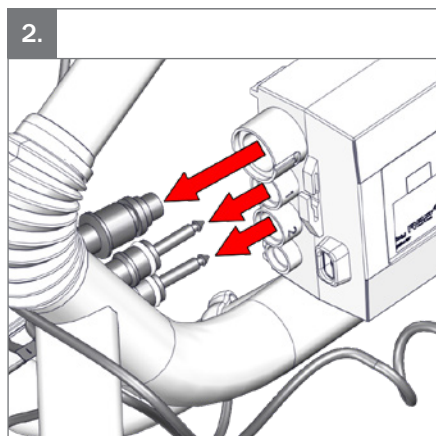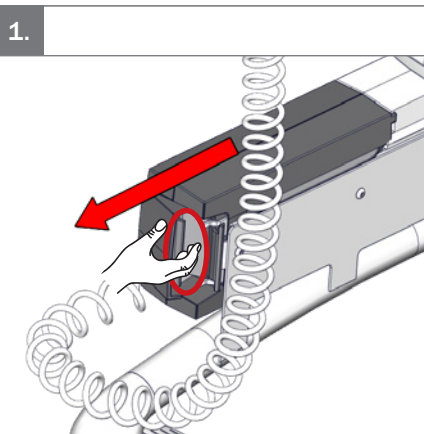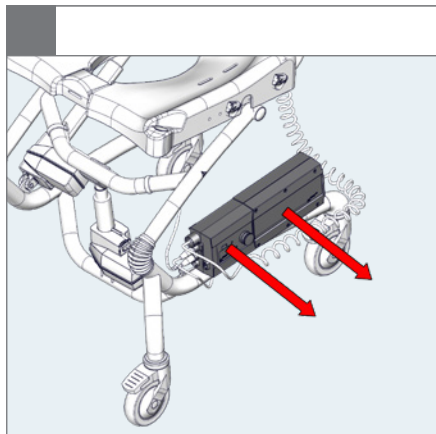2.



5.

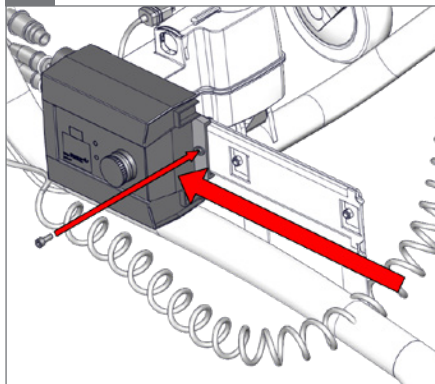
⌀ 3 mm

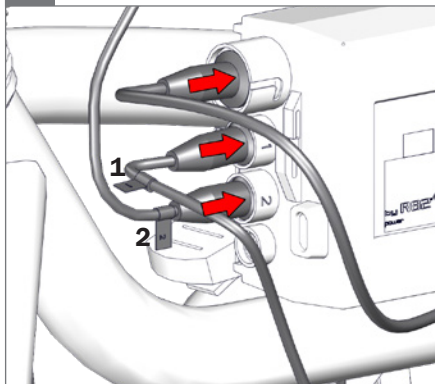
6.

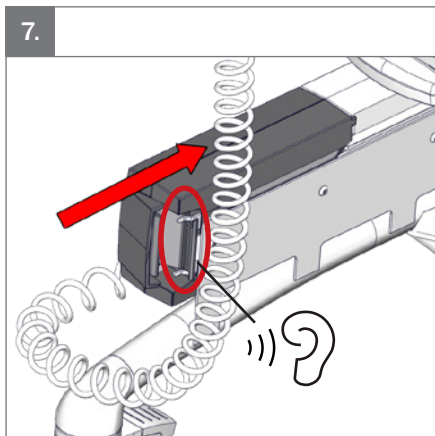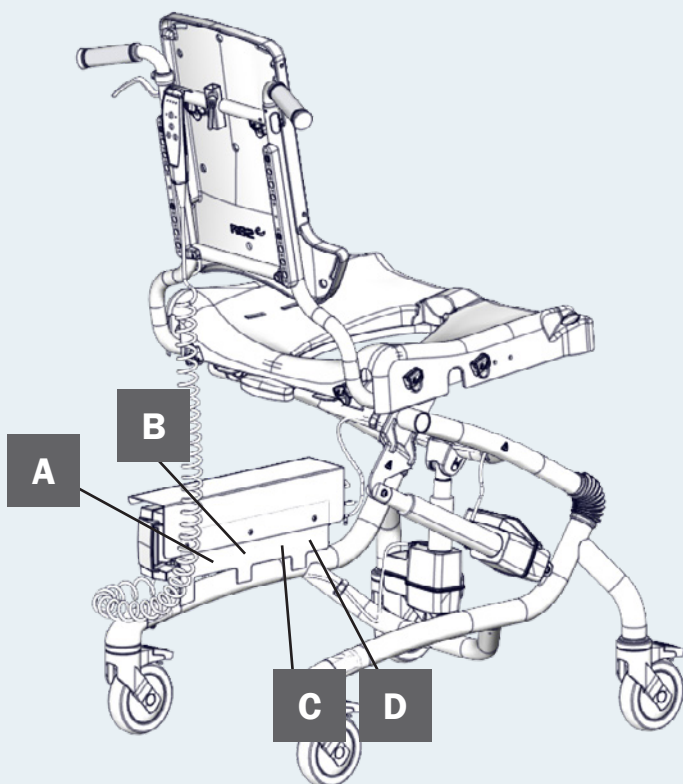7.

A

R82
by Etac

MD

R82 A/S, Parallevej 3
DK-8751 Gedved
etac.com
Made in xxxxxxx

CE UK
CA

YYYY-MM-DD

B

Posting group
Product name, size x

DMR ver.: xxxx

Max. user mass: xxx kg
Max. rated load: xxx kg
Mass device: xxx kg

SR xxxxxxxxxxxx
REP xxxxxxxxxxxxxxxxxxxx

Barcode: (XX)XXXXXXXXXXXX(XX)XXXXXX(XX)XXXXXX

C

Storage & Transport

Operating

Temperature: 50°C / 20°C




Humidity: 100% / 20%

Pressure: 1000 hPa / 800 hPa

D

IP 24

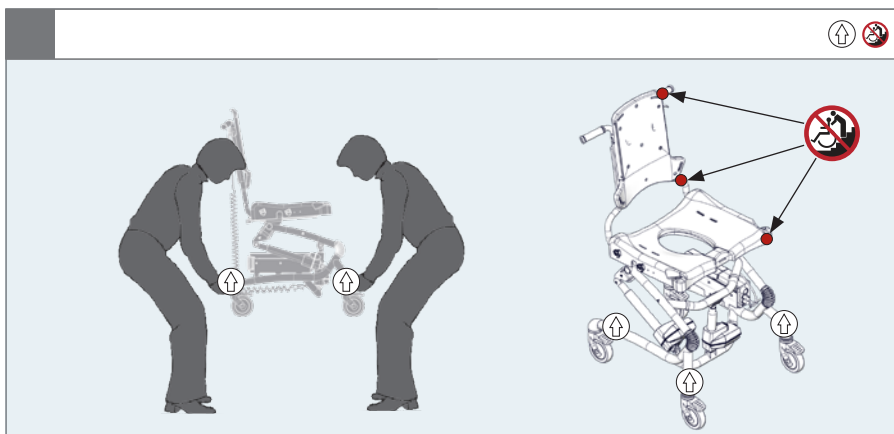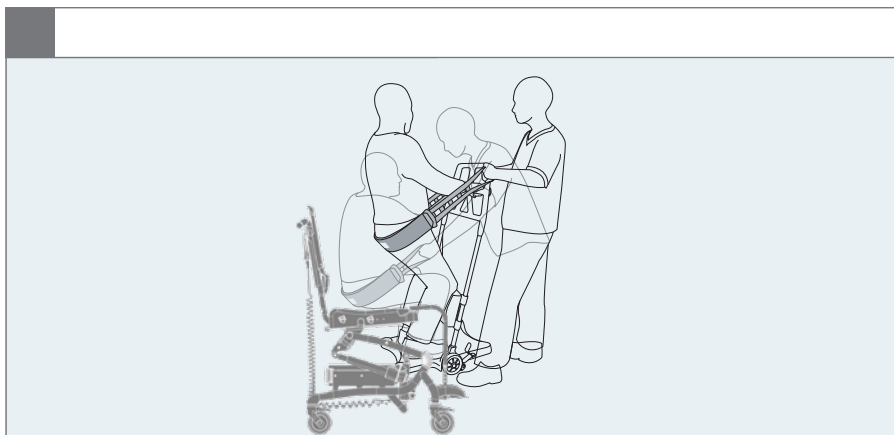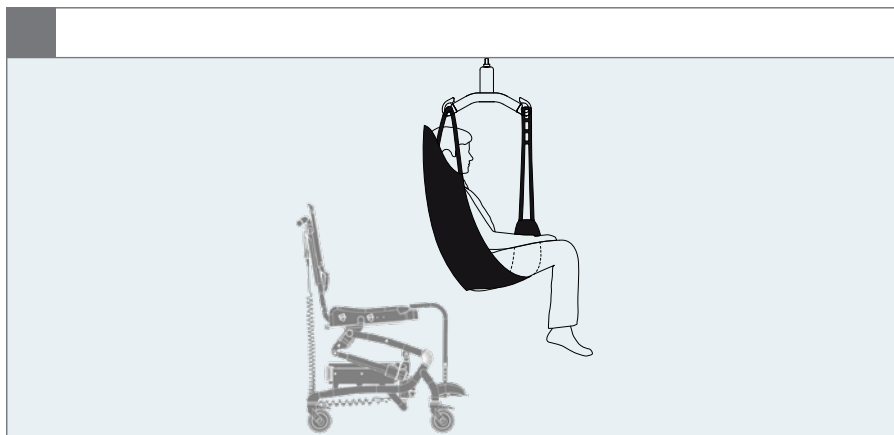Person icon

EN: Warning: This symbol appears in the User Guide along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger
01 = Entrapment Hazards
02 = Tilting Hazards
03 = Transportation in motor vehicles
05 = Squeezing Hazard

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

mm (inch) kg (lb)	Hydraulic	Power
 Overall width	670 mm (26½")	670 mm (26½")
 Length	710 mm (28")	710 mm (28")
 Overall height	1080–1350 mm (44½–53¼")	1130–1350 mm (44½–53¼")
 Effective seat width	550 mm (21¾")	550 mm (21¾")
Seat width between hip supports (accessories)	350–490 mm (13¾ - 19¼")	350–490 mm (13¾–19¼")
 Effective seat depth	450 mm (17¾")	450 mm (17¾")
 Back width, top of backrest	290 mm (11½")	290 mm (11½")
 Backrest height	555–625 mm (21¾–24½")	555–625 mm (21¾–24½")
 Seat surface height at front edge	510–780 mm (20–30¾")	560–780 mm (22–30¾")
 Seat plane angle	(-5°)–(25°)	(-5°)–(25°)
 Backrest angle	(-10°)–(30°)	(-10°)–(30°)
 Lower leg length	400–560 mm (15¾–22")	400–560 mm (15¾–22")
 Minimum turning diameter	900 mm (35½")	900 mm (35½")
 Mass device	28 kg (61.8 lb)	31 kg (68.3 lb)
 Maximum user mass	100 kg (220 lb)	130 kg (286.6 lb)
 Maximum rated load	130 kg (286.6 lb)	163 kg (359.3 lb)
 Static stability sideways	5°	5°
 Static stability uphill and downhill	10°	10°

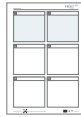
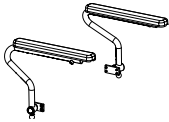
9996097103


9996097104

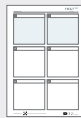
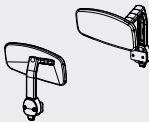
9996097105


9996097106

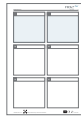
9996097107

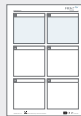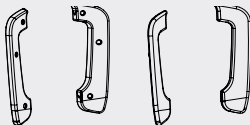9996097108


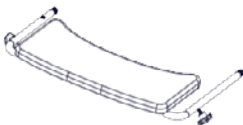
9996097109

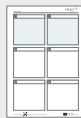
9996097110

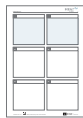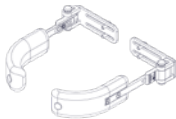9996097111

9996097440

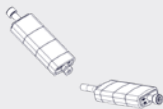9996097236

ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this user manual before use and save it for future reference.

Intended purpose

Assistive device for toileting, washing, bathing and showering for children and adults with disabilities.

Indications

The device can be used by children and adults with disabilities, who lack the ability to maintain a seated position without an assistive device or to compensate for the lack of ability to maintain a standing position while washing oneself or showering. The target group for the device is based on functional ability. It is not based on a specific health condition or age.

The user is the person sitting in the device. The caregiver is the person operating the device. Never leave the user unattended in this device. Ensure permanent supervision by a caregiver.

The device can be applied more times during the day. Between using the device, the user will be lying down, standing or sitting in another assistive device or a conventional product.

Enclosed tools:

- 3+4+5 mm Allen key

CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- EN 12182
- ISO 17966 (power unit)

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility,

flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety.

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

The symbol of the crossed-out trash can on batteries or accumulators states that they must not be disposed of in household waste at the end of their service life. If batteries or accumulators contain mercury (Hg), cadmium (Cd) or lead (Pb), you will find the respective chemical symbol below the symbol of the crossed-out trash can. You are legally obliged to return old batteries and accumulators after use. You can do this free of charge at the retail store or at another collection point in your area. Addresses of suitable collection points can be obtained from your city or local government.

Batteries can contain substances that are harmful to the environment and human health. Special care should be taken due to the special risks involved in handling lithium-containing batteries. The separate collection and recycling of old batteries and accumulators is intended to avoid negative effects on the environment and human health.

Please avoid the generation of waste from old batteries as much as possible, i.e., by preferring batteries with a longer service life or rechargeable batteries. Please avoid littering of public spaces by not carelessly leaving batteries or battery-containing electrical and electronic equipment behind. Please consider ways to reuse a battery instead of disposal, for example by reconditioning or repairing the battery.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

www.etac.com/support

Presales information and instructions

Presales information and instructions are available at www.etac.com, under your product, in the documents section.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the user manual carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Ensure the stability of the product is maintained before placing the user in it
- We recommend you notify the user before angling the seat and back
- Before moving the chair, with or without a user seated, please ensure the seat is in horizontal position, the back in vertical position. Do not move an angled seat
- Ensure the castors are fully-functional and secure before use
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- The emergency stop is activated when using the red button on the control box

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. This product contains small parts which could present a choking hazard if removed from their intended placing. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product

Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

Lifetime

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Reuse

This product and its accessories/components are suitable for reuse.

Refurbishment for reuse

Before reuse or changing the user, follow the service checklist, maintenance and washing instructions and carry out regular inspections.

Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the product is inspected yearly, every time the product is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the product.

Service checklist:

Check and adjust the following as per their intended use:

- Cushioning
 - Products equipped with a hydraulic pump for height adjustment. Frame height can be adjusted up and down
 - Powered versions are able to retain power after charging the battery
 - For powered versions, reset the power unit system using the handset by pressing the up(f) and down(g) buttons simultaneously for 5 seconds
 - Products equipped with power for height adjustment. The cables are in good condition and the handset functionalities for height adjustment works as intended.
 - Nuts and bolts on the product are tightened
 - Oil the swing-away parts
 - Fittings and buckles for signs of worn out parts
 - Product for cracks or signs for worn-out parts
 - All labels on the product are intact
- Repair or replace any damaged or worn out parts.

Maintenance

Before every use it is recommended to: Wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use carry out a visual inspection, for any damaged or worn-out parts.

Washing

Main product

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

This product can be washed at 85° using a mild detergent for 3 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Unmount all accessories and wash them separately.

Before washing

Avoid degreasing of the piston rods, the actuators should be retracted to minimum stroke and without load before washing.

- Remove all accessories, unplug all cables and make sure to dismount the battery, control-box and handset. Leave the actuators with the cables mounted on the product (see page 22)
- Do NOT clean the power unit with a high pressure cleaner or steam cleaner.

After washing

- Ensure correct placement of cables inserted into the control box when reinstalling the power unit system (see page 23)
- Check that the handset functionalities works correct

Handwash

This product and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

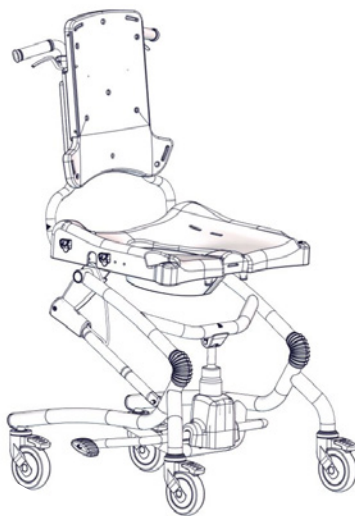
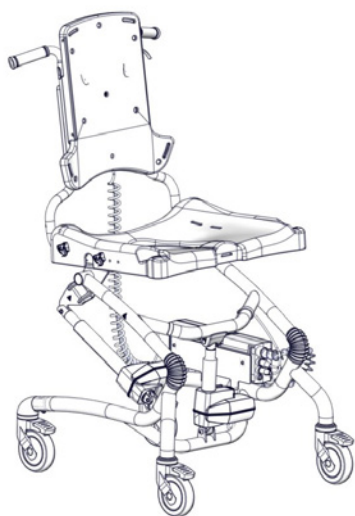
Materials

- Stainless steel
- Aluminium
- Plastic
- Rubber
- Polyurethan

Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

Power Unit system

Handset functionality

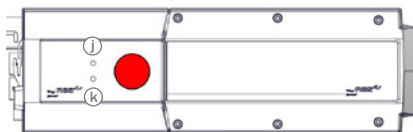
- Battery is OK, no need for charging. Green LED lights when activating functions
- Battery is soon empty, the battery needs recharging. Orange LED Lights when activating functions
- The battery needs recharging, red LED flashes when activating functions. It is still possible to lower the seat height and to tilt the seat forwards
- Service required, LED Lights. Check that all connectors are firmly fixed in the connections. If the LED continues to light up see Service interval page 43 and go through the Service check list on page 43.
- Overload, orange LED flashes when trying to heighten the seat level. Lower the seat until stop to reset the system
- Increase the seat height
- Lower the seat height
- Tilt the seat forward
- Tilt the seat backward

Troubleshooting:

Make sure that the emergency stop is not activated. In case of the handset malfunctioning the seat height can be adjusted by using the control box. Using a pen or a 3mm Allen Key, activate the up and down points in the control box.

- Increase the seat height
- Lower the seat height


Recharging the battery:

- Connect the charger to mains. The charger indicates whether the charger is connected to the mains (green LED)
- Place the battery in the charger, battery is being charged (yellow LED)
- Charging time: approximately 4-5 hours

Warning

Caregiver

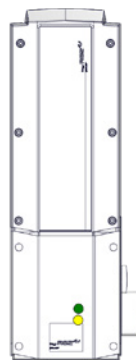- Do not operate the Power unit system unless you have received training in using it
- Do not leave children and youngsters alone with the handset
- Do not let children and youngsters play with the handset
- Do not leave the handset within reach of children and youngsters.
- It is important to remove the transport plastic bag before using the handset. Cables need to be placed in such a way that users or carers cannot trip and injure themselves.

Product

- The control box has an emergency stop
- The system must not be sprayed directly with a highpressure cleaner
- Cleaning with a steam cleaner is not permitted
- Do only use the power unit within specified working limits
- It is recommended to check the handset and cable for damage and holes caused by violent handling at least once a year
- Do not submerge the handset in water

Technical Data

Battery: BAJ1 - IPX5

- Capacity 2.9 Ah - 24V DC
- Duty cycle: Max. 10% or 2 min. use then 18 min. without use

The batteries have a maximum life of 4 years but this can vary depending upon the conditions of use and charging practices

Control box: CBJC - IPX4

- Output voltage 24V DC

Battery charger: CHJ2 - IPX5

- Mains: 100 - 240VAC/50-60 Hz switch mode
- Charging current: Max. 650 mA

Actuator: LA40 - IPX6

- Motor: 24-33 V DC, Standard motor
- Noise level: ≤ 50 db (A)

Handset: HB 30 IPX4

Detachable parts

Handset - HB 30

- Battery - BAJ1
- Control box - CBJC

Applied part

- Handset - HB 30

IP-Rating table

First Digit: Solids

The first digit indicates the level of protection that the enclosure provides against access to hazardous parts (e.g., electrical conductors, moving parts) and the ingress of solid foreign objects

IP0X	No protection against contact and ingress of objects
IP1X	Any large surface of the body, such as the back of the hand, but no protection against deliberate contact with a body part.
IP2X	Fingers or similar objects.
IP3X	Tools, thick wires, etc.
IP4X	Most wires, screws, etc.
IP5X	Ingress of dust is not entirely prevented, but it must not enter in sufficient quantity to interfere with the satisfactory operation of the equipment; complete protection against contact.
IP6X	No ingress of dust; complete protection against contact.

Ambient and surface temperature

The table below illustrates how long time it takes for the surfaces of the Heron to reach an ambient temperature of 20 °C from -10 °C and +50 °C.

Surface temperature			
-10 °C	to	+20 °C	108 min
+50 °C	to	+20 °C	129 min

Environmental conditions

- Operating temperature: +5 °C to +40 °C
- Relative humidity: 20% to 90%
- Atmospheric pressure: 800 to 1060 hPa
- Storage temperature: -10 °C to +50 °C

Second Digit: Liquids

Protection of the equipment inside the enclosure against harmful ingress of water.

IPX0	No protection
IPX1	Protection against vertically falling drops of water e.g. condensation
IPX2	Protection against direct sprays of water up to 15° from the vertical
IPX3	Protected against direct sprays of water up to 60° from the vertical
IPX4	Protection against water sprayed from all directions - limited ingress permitted
IPX5	Protected against low pressure jets of water from all directions - limited ingress
IPX6	Protected against temporary flooding of water, e.g. for use on ship decks - limited ingress permitted
IPX7	Protected against the effect of immersion between 15 cm and 1 m
IPX8	Protects against long periods of immersion under pressure

Electromagnetic compatibility (EMC) environment

This equipment is for use in home healthcare facility environment.

This equipment is not likely to cause any harmful interference in nearby equipment; however, it can be impaired by electromagnetic (EM) interference emitted by other equipment, which can lead to malfunctions. Do not use this equipment where there is a risk that other electrical and/or electronic equipment can interfere with this equipment.

This equipment complies with the following standards:

IEC 60601-1-2:2014 Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances - Requirements and tests (EN 60601-1-2:2015)

EN 12182:2012, Clause 7, Assistive products for persons with disability – General requirements and test methods

EMC - WARNINGS

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Use of accessories, transducers and cables other than those specified or provided by R82 could result in increased EM emission or decreased EM immunity of this equipment and result in improper operation. Several power plugs are supplied with this equipment. Please use the correct power plug suitable for your country. Please choose among the supplied power plug adaptor units: English, Australian, EU, US/Japanese plugs.

Portable radio frequency (RF) communication equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the heron w/power, including cables specified by R82. Otherwise, degradation of the performance of this equipment could result.

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Test modes during emission tests:

Movement mode (battery). Actuator running in/out

Charging mode, actuator not running.

EM emission standards	Classification/levels
Conducted and radiated RF emission CISPR 11	Group 1 (the product uses RF energy only for its internal function) Class B (the product is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes)
Harmonic distortion IEC 61000-3-2	Class A. No testing performed, as the rated power for this equipment is less than 75 W. No limits are specified for equipment with a rated power of 75 W or less.
Voltage fluctuations and flicker IEC 61000-3-3	Complies

Test modes during immunity tests:

Battery mode, standby armed/ready.

Charging mode, actuator not running.

EM immunity standards	IEC 60601-1-2:2014 test levels	Additional test levels
Electrostatic discharge IEC61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	± 6 kV contact
Radiated RF EM fields IEC 61000-4-3	3V/m (Professional) 10 V/m (Homecare) 80 MHz – 2.7 GHz 80 % AM at 1 kHz	20 V/m 800 MHz – 2.5 GHz 80 % AM at 1 kHz
Proximity fields from RF wireless communication equipment IEC 61000-4-3	IEC 60601-1-2:2014 Table 9	N/A
Electrical fast transients / bursts IEC 61000-4-4	± 1 kV, ± 2 kV 5 kHz repetition frequency	N/A
Surges, Line-to-line IEC 61000-4-5	± 0,5 kV, ± 1 kV, ± 2 kV	N/A
Conducted disturbance induced by RF fields IEC 61000-4-6	6 Vrms 0.15 MHz - 80 MHz 80% AM at 1 kHz 0.15 MHz - 80 MHz 80% AM at 2 kHz	N/A
Rated power frequency magnetic fields IEC 61000-4-8	30 A/m 50 Hz and 60Hz	N/A
Voltage dips, short interruptions and voltage variations IEC 61000-4-11	0 % UT for 0,5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° phase angles 0 % UT for 0,5 cycle 0 % UT for 1 cycle 70 % UT for 25 cycles (50 Hz) 70 % UT for 30 cycles (60 Hz) 0 % UT for 250 cycles	40 % UT for 5 cycles

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**Etac Ltd.**
**Gelbart AG**